



ONE-TOUCH WIRELESS HEALTH TRACKER

BODY LIFE TRACKER

User Manual

Model: VA06

Document No : AMI-PRD-UM-B-VA06

Version : 1.0

Date : 26.08.2015

Legal

Disclaimer

This publication/content contains proprietary information which is protected by copyright. No part of this publication may be reproduced, transcribed, stored in a retrieval system, translated into any language or computer language, or transmitted in any form whatsoever without the prior written consent of the publisher. American Megatrends India Private Ltd. retains the right to update, change or modify this publication at any time, without notice.

For Additional Information

Call American Megatrends India Private Ltd at 1800-419-9868.

Limitations of Liability

In no event shall American Megatrends India Private Ltd be held liable for any loss, expenses, or damage of any kind whatsoever, whether direct, indirect, incidental, or consequential, arising from the design or use of this product or the support materials provided with the product.

Limited Warranty

No warranties are made, either expressed or implied, with regard to the contents of this work, its merchantability, or fitness for a particular use. American Megatrends India Private Ltd assumes no responsibility for errors and omissions or for the uses made of the material contained herein or reader decisions based on such use.

Trademark and Copyright Acknowledgments

Copyright © 2015 American Megatrends India Private Ltd. All Rights Reserved.

AMERICAN MEGATRENDS INDIA PRIVATE LTD.,

KUMARAN NAGAR,

SEMMANCHERY VILLAGE,


CHENNAI (TN) 600119.

All product names used in this publication are for identification purposes only and are trademarks of their respective companies.

B.O.L.T VA06 User Manual

Model VA06 – NIBP Monitor (Systole Diastole Pulse Rate), Infrared Ear Thermometer (Temperature), Pulse Oximeter (SpO2 & Pulse Rate), Gluchobin (Blood Glucose, Blood Cholesterol & Blood Hemoglobin Levels)

Thank you very much for purchasing B.O.L.T the One -Touch Wireless Health Tracker. Please read this User Manual thoroughly to ensure safe and correct use of the product. Please keep this User Manual for future reference.

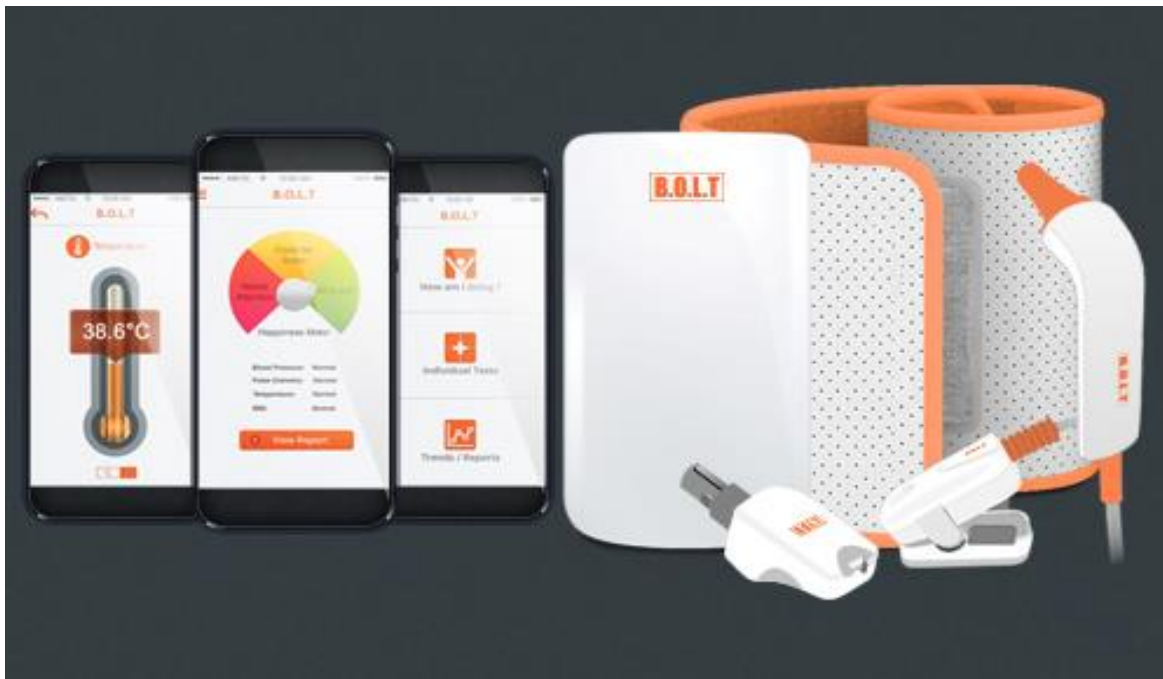

B.O.L.T is a Non Invasive medical gadget with NiBP (Non-invasive Blood Pressure), SpO2 (Pulse Oximeter) Infrared Thermometer (IRT) and Gluchobin combined into a small form factor device which gets controlled from a mobile device. B.O.L.T is a cloud enabled solution. With B.O.L.T your health vitals can be stored in your personal health cloud. For B.O.L.T to be in your hands, all you need is any leading mobile device that operates with the compatible Operating System.

Table of Contents

Safety Measures	5
Warning	5
Caution	5
Care and Maintenance	5
Know Your Unit	6
Intended use	6
Prerequisites	6
Using B.O.L.T Application	7
Blood Pressure Measurement Procedures	7
Blood Pressure measurement with B.O.L.T	8
Blood Pressure Classification for Adults	9
Blood Oxygen Level Measurement Procedures	9
Blood Oxygen Level measurement with B.O.L.T	9
Temperature Measurement Procedures	10
Temperature Classification	11
Blood Glucose Level Measurement Procedures	11
Blood Glucose Level Classification	14
Blood Hemoglobin Level Measurement Procedures	14
Blood Hemoglobin Level Classification	17
Blood Cholesterol Level Measurement Procedures	17
Blood Cholesterol Level Classification	20
Body Mass Index (BMI)	20
Replacement of Battery Pack	21
Explanation of Indicators/Symbols	22
Technical Specifications	24
FCC compliance statements	25
Warranty Information	25
Customer Care Support	26

Safety Measures

Warning

- If you encounter any problem while taking blood pressure measurement, such as inflation does not stop or air leakage in the cuff, remove the cuff or pull out the air tube from the base unit.
- Do not wrap BP cuff over the arm where an intravenous injection or transfusion is being given to the patient.
- Do not operate the gadget in places where inflammable gas, such as highly inflammable anesthetic, may be generated or in a high-pressure oxygen room or in an oxygen tent.
- Keep the gadget away from infants, small children or anyone who are incapable of using the gadget in the correct manner.
- Please do not use the same cuff for patients who are highly infectious as this practice may cause cross-infection.
- Make sure the ear canal is clean for before performing temperature measurement.
- Use dust cover for Ear Thermometer during the temperature measurement.
- Place the finger inside the SpO2 probe properly during the Blood oxygen level measurement.

Caution

- Do not use the gadget while moving in a vehicle as this may result in erroneous readings.
- Do not use any other branded cuffs as this may result in reading errors.
- Do not use unauthorized parts and accessories.
- Do not store or use this gadget in locations which are maintained above the specified temperature and humidity ranges.
- Finger nail polish or false fingernails may cause inaccurate SpO2 readings.
- SpO2 measurements may be adversely affected in the presence of high ambient light.
- Dispose the used lancet and cotton in an appropriate container.


Care and Maintenance

- Store the gadget and the components in a clean and safe location.
- Changes or modifications not approved by the manufacturer will void the user warranty.
- Keep the accessories and gadget away from rain, sweat and water.
- Do not disassemble, repair or modify the gadget and the components by yourself, hence this could void the user's authority to operate the device.

- Clean the rubber touching the finger inside the SpO2 probe gently by a soft cloth dampened with 70% isopropyl alcohol before measurement.

Know Your Unit

The base unit has an elegant look and feel with two LED indicators, an accessory connector and a power ON/OFF sliding switch as shown below. Accessories like cuff, IR Ear Thermometer with cover, SpO2 probe, Gluchobin and USB cable with power adapter are provided along with the base unit.

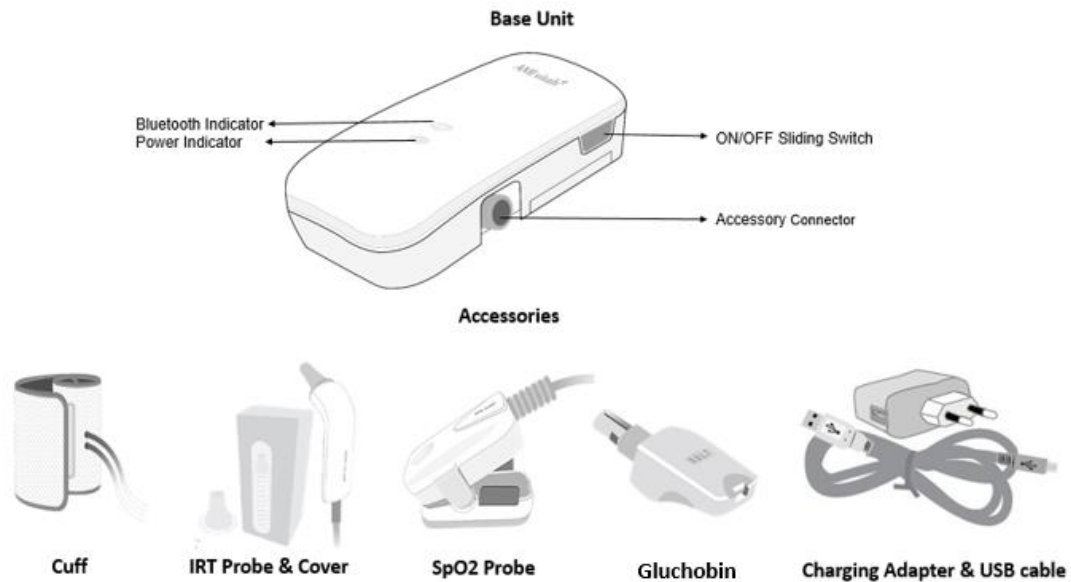

Features of the Product

- Elegant design enables simple and easy use of the product.
- Multiple Platform Support and same application for multiple users.
- Secured Cloud application to share and manage trends with health data.
- Monitor and share your health trends with doctor and loved ones.
- Safe and convenient to use.

Intended use

B.O.L.T (VA06)

It is used to measure the vital signs like Blood Pressure, Body temperature, Blood oxygen saturation level & Blood glucose, hemoglobin, cholesterol levels. The B.O.L.T device is controlled from a mobile device.

Prerequisites

Some basic requirements must be adopted as mentioned below to ensure better operation of the product.

- B.O.L.T is designed to be operated with applications like Android, iOS or Windows.
- The Android OS version of the using computer/mobile device should be 4.0 or higher.
- The iOS version of the using computer/mobile device should be 7 or higher.

- The Windows OS version of the using computer/mobile device should be 8 or higher.

Using B.O.L.T Application

Downloading and Installing B.O.L.T application in your computer/mobile device


1. Download and Install B.O.L.T application from the Play store/Application store/Windows application market.
2. On successful installation, launch the B.O.L.T application in the mobile device.
3. Create a User Name with your Email ID or use Facebook/Google+ Credentials for login.
4. On successful Login, provide permission for cloud service by enabling the **Save in cloud option** and enter near and dear details under settings panel at left top corner of the home view.

Note: You can also login as a guest user while internet unavailability which might not store the measured vital parameters in the cloud storage space on the measured parameters.


Blood Pressure Measurement Procedures

Mandatory instructions to be followed during blood pressure measurement

- Relax yourself in a comfortable position.
- Avoid eating, exercising and smoking for 30 minutes before taking a measurement.
- Avoid taking measurements during stressful times.
- Remove tight-fitting clothing from the right upper arm.
- Sit in a chair with your feet flat on the floor and place the right arm on a table as shown below.

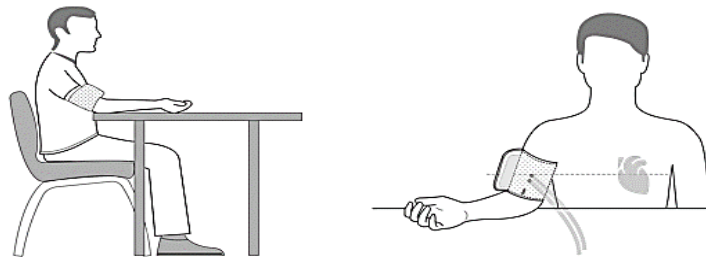

- Apply the cuff to the right upper arm so that the cuff is wrapped at the same level of your heart.
- Ensure that the arrow present in the BP cuff points towards the artery along the middle finger of the right hand as shown below.

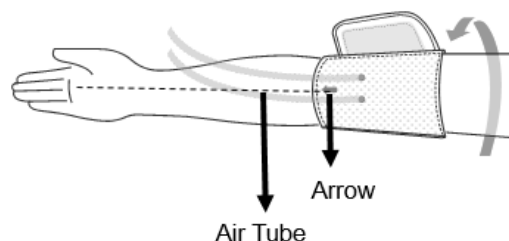


- Stay calm without shaking the hand during the measurement.

Blood Pressure measurement with B.O.L.T

B.O.L.T enables connecting an air hose through which the blood pressure is monitored.

Connecting air hose with the gadget and launching B.O.L.T application

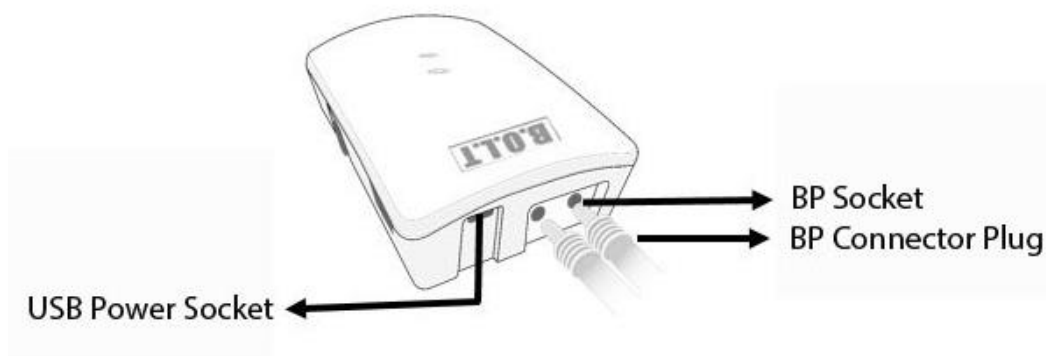

1. Insert the BP connector plug into the BP socket firmly as shown above.
2. Switch ON the gadget using the ON/OFF sliding switch.
3. Enable Bluetooth and Internet Connectivity in the computer/mobile device.
4. Launch the B.O.L.T application in the computer/mobile device.
5. Login with the User Name and Password which is used for the registration.

Measuring BP with B.O.L.T

1. On successful login of the B.O.L.T application, Select “**How is my health**” option to perform **Master Health checkup** or select “**Check Vitals**” option to perform **individual test**.
2. Confirm the Bluetooth device to turn-on.
3. Select the B.O.L.T device from the list of available Bluetooth devices and confirm.
4. Provide default password (**1234/0000**) for pairing your B.O.L.T device with mobile device on your first time usage (only for android).
5. Wrap the cuff around the upper arm of the right hand above the elbow.
6. Ensure that the arrow in the BP cuff points the artery along the middle finger of the right hand as discussed earlier in the Blood Pressure measurement procedure section.
7. Start the Blood Pressure measurement using the application screen.
8. Ensure that the person remains stable until the application displays the results.
9. On successful measurement, the Blood Pressure and Pulse rate are displayed and a copy of reading will be mailed to your Near & Dear mail IDs on saving the data.
10. Multiple readings can be taken with atleast 1 minute interval by selecting the auto-reading option.

Note: On Successful pairing of the mobile device, Bluetooth light stops blinking in the gadget. Ensure that the save option is enabled for storing the data in cloud.

Blood Pressure Classification for Adults

The World Health Organization (WHO) has created the following classification for assessing high blood pressure (Hypertension). Please note that this table can give only a generic view as other factors such as diabetes, obesity and smoking, also need to be considered for understanding the different ranges. Consult with your physician for accurate assessment.

Blood Pressure Classification	Systole	Diastole
Low	<110	<75
Normal	110-120	75-80
Pre-Hypertension (mild)	120-139	80-89
Stage 1 Hypertension (high)	140-159	90-99
Stage 2 Hypertension (severe)	>159	>99

Blood Oxygen Level Measurement Procedures

Mandatory instructions to be followed during blood oxygen level measurement

- Relax yourself in a comfortable position.
- Remain stable during the measurement.
- Place the finger properly inside the SpO2 probe.
- Clean the surface of the fingertip before the measurement.

Blood Oxygen Level measurement with B.O.L.T

B.O.L.T enables connecting a SpO2 probe through which the blood oxygen level is measured.

Connecting SpO2 probe with the gadget

Insert the SpO2 connector into the socket firmly as shown below.



Measuring Blood Oxygen Level with B.O.L.T

1. Select “**Check Vitals**” option to perform **individual test**.
2. Clip the SpO2 probe in the index finger of the person to be measured in right/left hand.
3. Select the Blood Oxygen option from the application screen.
4. On successful measurement, the SpO2 value and Pulse rate are displayed and a copy of reading will be mailed to your Near & Dear mail IDs on saving the data.

Blood Oxygen Level Classification

Please note that this table can give only a generic view. Consult with your physician for accurate assessment.

SpO2 Classification	Range
Hypoxemia (low)	< 95 %
Normal	95-100 %

Temperature Measurement Procedures

Mandatory instructions to be followed during temperature measurement


- Relax yourself in a comfortable position.
- Use new dust cover for each measurement.
- Place the Ear Thermometer firmly into the ear canal for accurate measurement.
- Remain stable during the measurement.
- Wait until the application displays the measured temperature.

Body Temperature measurement with B.O.L.T

B.O.L.T enables connecting an IR Ear Thermometer through which the temperature is measured.

Connecting IR Ear Thermometer with the gadget

Insert the IR Ear Thermometer connector into the socket firmly as shown below.

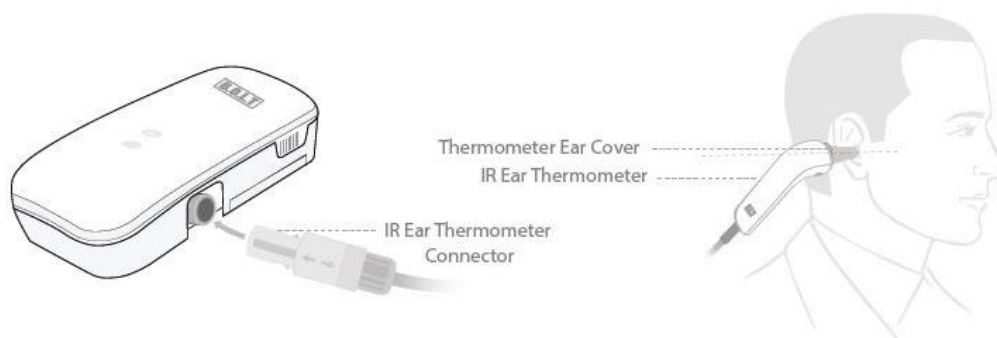

Measuring Body Temperature with B.O.L.T

1. Select “**Check Vitals**” option to perform **individual test**.
2. Use dust cover over the tip of the Ear Thermometer.
3. Place the Ear Thermometer inside the ear horizontally as such the pointer maintains horizontal position as displayed above.
4. Select the Body temperature option from the application screen.
5. On successful measurement, the temperature value is displayed and a copy of the reading will be mailed to your Near & Dear mail IDs on saving the data.

Temperature Classification

Please note that this table can give only a generic view. Consult with your physician for accurate assessment.

Temperature Classification	Range
Hypothermia	<36.3 °C (< 97.5 °F)
Normal	36.4 – 37.5 °C (97.6- 99.5 °F)
Hyperthermia	>37.5.°C (>99.5 °F)

Blood Glucose Level Measurement Procedures

Mandatory instructions to be followed during temperature measurement


- Relax yourself in a comfortable position.

Blood Glucose level measurement with B.O.L.T


B.O.L.T enables connecting a Gluchobin through which the glucose test strip are inserted for measurement.

Using Lancet pen and Lancet

1. Unscrew and remove the lancet pen `s adjustable tip as shown below.

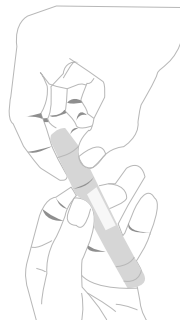

2. Insert a lancet into the carrier as shown below.

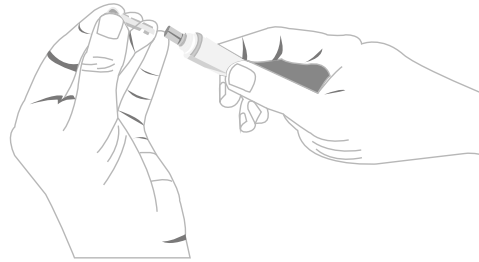

3. Twist off the protective cover.



4. Replace the adjustable tip tightly. Choose a desired skin penetration depth by rotating the top portion of the adjustable tip until the setting number lines up to the arrow.
5. Settings are based on skin type:
Depth 1-2 for soft or thin skin
Depth 3 for average skin
Depth 4-5 for thick or calloused skin
6. Hold the tip of the lancet pen with one hand and pull the sliding barrel with the other hand. When a click is felt, the trigger rises up. Release the sliding barrel, it will move back to its original position.

Connecting Gluchobin with the gadget

7. Insert the Gluchobin connector into the socket firmly as shown below.




Gluchobin connected to B.O.L.T Device

8. Select the “**Blood Glucose**” option from the application screen of the mobile device.
9. Insert the blood glucose test strip into the test strip slot of the Gluchobin as shown below.




10. Click “Start” in the application screen of the mobile device.
11. Clean your finger with alcohol swap and let it dry completely.
12. Place the lancet pen on side of your fingertip.

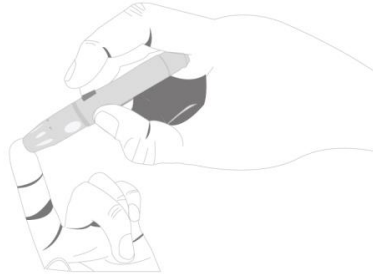

13. Press the trigger on the lancet pen as shown above.
14. Get a drop of blood and place the blood drop on the test strip target area.



15. On successful test completion, the test result will be displayed in 6 or 7 second on the application screen.
16. Wipe any excess blood on the finger with the cotton and dispose it properly.
17. Choose the options “before meal” or “just after meal” or “3 hours after meal” as per the status of the user in the application screen.
18. Pull the test strip out of the Gluchobin.
19. Unscrew the adjustable tip of the lancet pen.
20. Place the protective cover of the lancet back on the lancet as shown below.

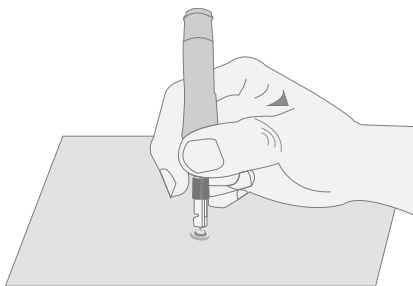

21. Discard the used lancet in an appropriate container.
22. Screw the adjustable tip back on the lancet pen.

Note: *Ensure that the test strip gets enough blood. Else try to use a new test strip to do the test again.*

To avoid accidental injury, always remove the used lancet immediately after each test.

Blood Glucose Level Classification

Blood Glucose Classification	Before Meal	3 Hours after meal
Normal	70-100	70-140
Pre-Diabetic	101-125	141-200
Diabetic	126 & above	200 & above

Blood Hemoglobin Level Measurement Procedures

Mandatory instructions to be followed during temperature measurement


- Relax yourself in a comfortable position.

Blood Hemoglobin level measurement with B.O.L.T


B.O.L.T enables connecting a Gluchobin through which the hemoglobin test strip are inserted for measurement.

Using Lancet pen and Lancet

- Unscrew and remove the lancet pen's adjustable tip as shown below.

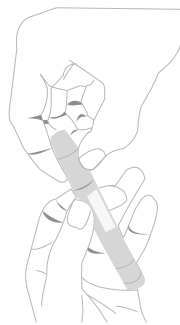

- Insert a lancet into the carrier as shown below.

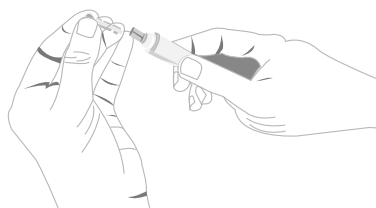

- Twist off the protective cover.



4. Replace the adjustable tip tightly. Choose a desired skin penetration depth by rotating the top portion of the adjustable tip until the setting number lines up to the arrow.
5. Settings are based on skin type:
 - Depth 1-2 for soft or thin skin*
 - Depth 3 for average skin*
 - Depth 4-5 for thick or calloused skin*
6. Hold the tip of the lancet pen with one hand and pull the sliding barrel with the other hand. When a click is felt, the trigger rises up. Release the sliding barrel, it will move back to its original position.

Connecting Gluchobin with the gadget

7. Insert the Gluchobin connector into the socket firmly as shown below.




Gluchobin connected to B.O.L.T Device

8. Select the “**Blood Hemoglobin**” option from the application screen of the mobile device.
9. Insert the hemoglobin test strip into the test strip slot of the Gluchobin as shown below.




10. Click “Start” in the application screen of the mobile device.
11. Clean your finger with alcohol swap and let it dry completely.
12. Place the lancet pen on side of your fingertip.

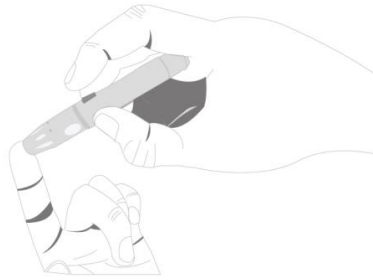

13. Press the trigger on the lancet pen as shown above.
14. Get a drop of blood and place the blood drop on the test strip target area.



15. On successful test completion, the test result will be displayed in few seconds on the application screen.
16. Wipe any excess blood on the finger with the cotton and dispose it properly.
17. Pull the test strip out of the Gluchobin.
18. Unscrew the adjustable tip of the lancet pen.
19. Place the protective cover of the lancet back on the lancet as shown below.

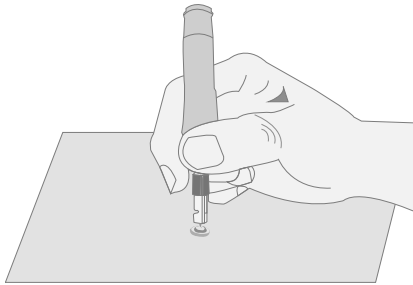

20. Discard the used lancet in an appropriate container.
21. Screw the adjustable tip back on the lancet pen.

Note: *Ensure that the test strip gets enough blood. Else try to use a new test strip to do the test again.*

To avoid accidental injury, always remove the used lancet immediately after each test.

Blood Hemoglobin Level Classification

Population	Non – Anemic g/dl	Mild g/dl	Moderate g/dl	Severe g/dl
Children 6 - 59 months of age	>11.0	10.0 – 10.9	7.0 – 9.9	< 7.0
Children 5 - 11 years of age	>11.5	11.0 – 11.4	8.0 – 10.9	< 8.0
Children 12 - 14 years of age	>12.0	11.0 – 11.9	8.0 – 10.9	< 8.0
Non-pregnant women	>12.0	11.0 – 11.9	8.0 – 10.9	< 8.0
Pregnant women	>11.0	10.0 – 10.9	7.0 – 9.9	< 7.0
Men (15 years of age and above)	>13.0	11.0 – 12.9	8.0 – 10.9	< 8.0

Blood Cholesterol Level Measurement Procedures

Mandatory instructions to be followed during temperature measurement


- Relax yourself in a comfortable position.

Blood Hemoglobin level measurement with B.O.L.T


B.O.L.T enables connecting a Gluchobin through which the cholesterol test strip are inserted for measurement.

Using Lancet pen and Lancet

1. Unscrew and remove the lancet pen `s adjustable tip as shown below.

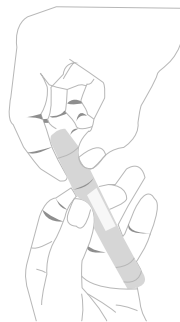

2. Insert a lancet into the carrier as shown below.

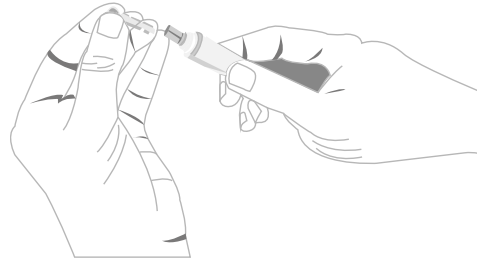

3. Twist off the protective cover.



4. Replace the adjustable tip tightly. Choose a desired skin penetration depth by rotating the top portion of the adjustable tip until the setting number lines up to the arrow.
5. Settings are based on skin type:
 - Depth 1-2 for soft or thin skin*
 - Depth 3 for average skin*
 - Depth 4-5 for thick or calloused skin*
6. Hold the tip of the lancet pen with one hand and pull the sliding barrel with the other hand. When a click is felt, the trigger rises up. Release the sliding barrel, it will move back to its original position.

Connecting Gluchobin with the gadget

7. Insert the Gluchobin connector into the socket firmly as shown below.




Gluchobin connected to B.O.L.T Device

8. Select the "**Cholesterol**" option from the application screen of the mobile device.
9. Insert the cholesterol test strip into the test strip slot of the Gluchobin as shown below.




10. Click “Start” in the application screen of the mobile device.
11. Clean your finger with alcohol swap and let it dry completely.
12. Place the lancet pen on side of your fingertip.

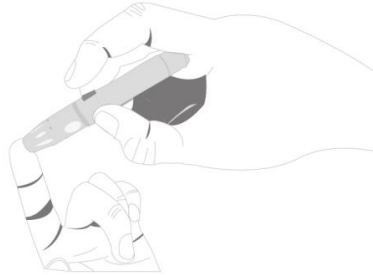

13. Press the trigger on the lancet pen as shown above.
14. Get a drop of blood and place the blood drop on the test strip target area.



15. On successful test completion, the test result will be displayed in few second on the application screen.
16. Wipe any excess blood on the finger with the cotton and dispose it properly.
17. Pull the test strip out of the Gluchobin.
18. Unscrew the adjustable tip of the lancet pen.
19. Place the protective cover of the lancet back on the lancet as shown below.

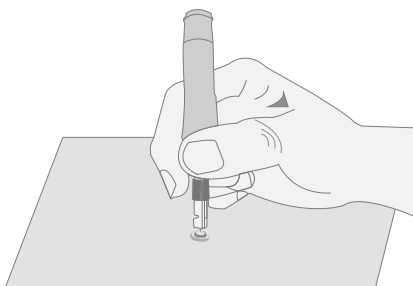

20. Discard the used lancet in an appropriate container.
21. Screw the adjustable tip back on the lancet pen.

Note: Ensure that the test strip gets enough blood. Else try to use a new test strip to do the test again.

To avoid accidental injury, always remove the used lancet immediately after each test.

Blood Cholesterol Level Classification

Blood Cholesterol Classification	
Normal	≤ 200 mg/dl
Boundary line high	200-239 mg/dl
High	≥240 mg/dl

Body Mass Index (BMI)

The **Body Mass Index (BMI)** is a measure of relative weight based on an individual's mass and height. BMI provides a simple numeric measure of a person's thickness or thinness, allowing health professionals to discuss overweight and underweight problems more objectively with their patients.

BMI Classification

Please note that this table can give only a generic view. Consult with your physician for accurate assessment.


BMI Range	Category
<20	Under Weight
20-25	Normal Weight
25-30	Over Weight
>30	Obesity

Measuring BMI with B.O.L.T

1. Choose Body Mass Index option from the application screen.
2. Enter your height (feet/cm) and weight (lbs/kg) of the person to be measured in the application.
3. On successful measurement, the BMI value is displayed which can be compared with the above BMI range classification table.

Battery Care

- Power adapter is included in the pack and the specifications are mentioned below.
AC input: 100-240V, Frequency: 50Hz to 60Hz, 0.35A Max, DC output: +5V @ 2A

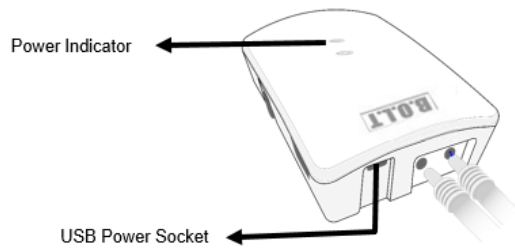


- Connect the AC power adapter to the power source and micro USB port to the base unit's USB Power socket.
 - Charge the base unit for minimum two hours during the first time usage.
 - The mobile application indicates the battery low status when the battery level goes below 25% and shows critically low status when the battery level goes below 10%.
 - Charge the base unit using the power adapter and view the charge level in the mobile application software.
 - It is recommended to unplug the power adapter from power source when the charge level reaches 100%.
 - When fully charged the base unit will automatically cut off charging internally, thus the device's design has taken safety measures into account to face various abnormal conditions like over-charging, over discharging etc.
 - When charging is needed, please connect the base unit to the power source using the power adapter.
 - The base unit can be operated by
 - Connecting to power source directly using the power adapter
 - or
 - Using the charged inbuilt battery (adapter not required)
- The device can work normally while charging.

Replacement of Battery Pack


Manufacturer highly recommended to change the battery of the base unit for every 1 year at the AMI authorized service center. Use AMI recommended battery for replacement.

Replacement procedure


1. Switch off the base unit before removing the back cover for the battery replacement.
2. Remove the single screw on the upper portion of the battery cover as shown below.

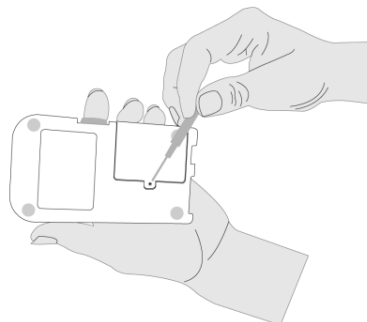


3. Press and remove the battery cover in the recommended direction as shown below.

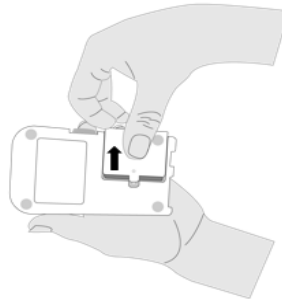

4. Disconnect the battery pack from the connector and replace with new one.

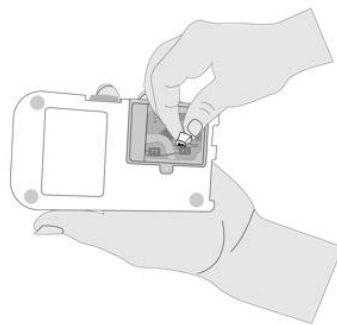

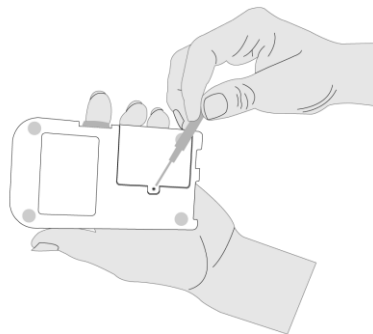


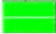











5. Install the battery cover and fasten with the screw.



6. Connect the base unit with AC adapter and then charge for more than twelve hours before use.

Explanation of Indicators/Symbols

Indicators/Symbols	Function/Description
	The indicator displays full battery power of the gadget in the computer/mobile device.
	The indicator displays low battery power of the gadget in the computer/mobile device.

SYS	Systolic blood pressure in mmHg.
DIA	Diastolic blood pressure in mmHg.
PUL./min	Pulse per minute.
	<ul style="list-style-type: none"> Flashing of the Bluetooth light indicates that the gadget is waiting for pairing with the mobile/tab. Steady Bluetooth light indicates that the gadget is paired and ready for the measurement.
	<ul style="list-style-type: none"> Steady green light indicates that the gadget is operating in battery mode. Blinking of green light indicates that the gadget has low battery level.
	Steady red light indicates that the gadget is in the charging mode.
	Symbol for "ENVIRONMENT PROTECTION"
	Symbol for "KEEP DRY"
	Symbol for "MANUFACTURER"
	Symbol for "MANUFACTURED DATE"
SN	Symbol for "SERIAL NUMBER"
	Symbol for "EUROPEAN REPRESENTATIVE"
CE	Symbol for "COMPILES WITH MDD93/42/EEC REQUIREMENTS"
	Symbol for "Federal Communication Commission"
	Symbol for "Read the documentation"
	Symbol for "Warning"
	Symbol for "Temperature limitation"
	Symbol for "Shock Protection"

Technical Specifications

Product Name	B.O.L.T
Model	VA06
Base Unit	
Base Unit Dimensions	130 (L) X 40.3 (H) X 75 (D) mm
Base Unit Weight	220 grams (Including Battery)
Power Source	DC 5V 2A
Bluetooth	Bluetooth 4.0 EDR or Lower, Bluetooth Low Energy (BLE)
Base Unit Operating Temperature	16°C to 35°C (60.8°F to 95.0°F)
Base Unit Storage and Transport Temperature	10°C to 55°C (14°F to 131°F)
Base Unit Storage Humidity	30%RH to 90%RH
Non-Invasive Blood Pressure	
Blood pressure measurement method	Oscillometric method with automatic inflation & measurement
Cuff Pressure range	0 to 280 mmHg
Pressure Accuracy	±4mmHg
BP Pulse range	30 to 180 BPM
Pulse Accuracy	± 5%
Cuff Limb Circumference	22 cm to 42 cm
Cuff Size	610 mm * 145 mm
Cuff Weight	205 Grams
IR Thermometer	
Body Temperature measurement method	Tympanic, Infrared based
Body temperature range	34°C to 42°C (93.2°F to 107.6°F)
Body temperature Accuracy	±0.2°C (from 36°C to 39.0°C)/ ± 0.4°F(from 95.0°F to 102.6°F) ±0.3°C (from 39.1°C to 44.0°C)/ ± 0.5°F(from 102.38°F to 111.2°F)
IRT Display resolution temperature range	0.1 °F
IRT Unit Dimensions	84.8 (L) X 18 (H) X 26.6 (D) mm (Highest width along the profile)
IRT Unit Weight	42 grams
Pulse Oximeter	
SpO2 measurement method	Optical method
SPO2 range	70% to 100%, below 70%: Unspecified
SpO2 Measurement Accuracy	70% to 100% - ±2%, below 70%: Unspecified
SpO2 Pulse Rate Range	30 to 180 BPM
SpO2 Unit Dimensions	58.2 (L) X 35 (H) X 35.3 (D) mm
SpO2 Unit Weight	60 grams
Gluchobin	
Gluchobin measurement method	Chronoamperometry method
Glucose range	20-600mg/dL (1.11-33.3 mmol/L)
Hemoglobin range	7 to 26 g/dL
Cholestrol range	100 - 400 mg/dL
Gluchobin Unit Dimensions	78 (L) X 27 (H) X 25(D) mm
Gluchobin Unit Weight	18 grams

NOTE: These specifications are subject to change without notice.

FCC compliance statements

This device complies with Part 15 of the FCC Rules. Its operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including Interference that may cause undesired operation

FCC Caution

- Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.

Part 15B compliance statements for digital devices:

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy. And, if not installed and used in accordance with the instructions, may Cause harmful interference to radio communications. However, there is no Guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television Reception, which can be determined by turning the equipment off and on,

The user is encouraged to try or to correct the interference by one or more of

The following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

For more information kindly visit our website: www.amibolt.com

Warranty Information

The B.O.L.T is warranted to be free from defects in materials and workmanship within one year from the date of purchase when used in accordance with the provided instructions. The warranty is extended only to the end user. The company will repair free of charge any part or parts of the product, if the defect is due to the faulty material or workmanship.

To obtain warranty service contact customer care support by calling **1800-419-9868**.

Enclose the warranty card, Proof of Purchase. Include a letter, with your name, address, phone number, and description of the specific problem. Pack the product carefully to prevent damage in transit.

Customer Care Support

AMI's flexible approach allows the customers to get Technical Support and Warranty services for B.O.L.T through our customer care support.

- Toll free number (India): 1800-419-9868
- Customer Care Support timings : 10am - 6pm Monday – Friday (Except National Holidays)

Contact us




AMERICAN MEGATRENDS INDIA PRIVATE LTD.,
KUMARAN NAGAR,
SEMMANCHERY VILLAGE,
CHENNAI (TN) 600119

Tel – (91) 44 66540922, Fax – (91) 44 66540901

Email: support@amibolt.com

Website: www.amibolt.com; www.amiindia.co.in

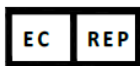

American Megatrends International GmbH

Truderingerstraße (Truderinger Strasse) 283

81825 München (Munich), Deutschland (Germany)

Tel: [49] 89 969995-10

Support: [49] 89 969995-11

Fax: [49] 89 969995-19

Email: europesales@ami.com