


Charging System

For Eterna™ Spinal Cord Stimulation System
Model 16000

USER'S GUIDE

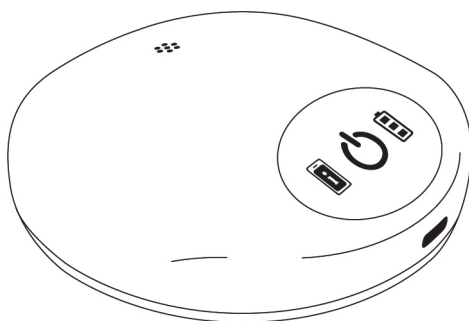

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.



WARNING: This product can expose you to chemicals including ethylene oxide, which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information, go to www.P65Warnings.ca.gov.

TM Indicates a trademark of the Abbott group of companies.

‡ Indicates a third-party trademark, which is property of its respective owner.

Bluetooth and Bluetooth logo are registered trademarks of Bluetooth SIG, Inc.

Pat. <http://www.abbott.com/patents>

© 2022 Abbott. All Rights Reserved.

Contents

About This Guide	1
Prescription and Safety Information.....	1
Intended Use.....	1
Indication for Use	1
Contraindications.....	1
Additional Prescription Information.....	1
MRI Safety Information	1
Warnings.....	2
Precautions	3
Adverse Effects.....	4
Product Description	4
Charger Indicators	7
Using Your Charger.....	8
Pairing the Charger with the Patient Controller.....	8
When to Charge Your Generator.....	9
Charging Your Charger.....	10
Setting Up the Charging Apparel.....	12
Charging Your Generator	17
Caring for and Maintaining Your Charger and Charging Apparel.....	18
Traveling with Your Charging System.....	19
Troubleshooting.....	20
Troubleshooting Charging Issues.....	20
Service and Ordering Information.....	23
Technical Support.....	23
Ordering Information	24
Appendix A: Regulatory Statements.....	25
Disposal Guidelines for Battery-Powered Devices.....	25
Statement of FCC Compliance	25
Statement of Compliance With Interference-Causing Equipment Standard (Canada).....	26
Product Classification Statement (CISPR 11, Class B).....	26
Identification Information for Product Registration.....	26
Wireless Technology Information.....	26
Quality of Service for Wireless Technology	27
Wireless Security Measures.....	28
Troubleshooting for Wireless and Coexistence Issues.....	28
Appendix B: Product Specifications.....	29
Appendix C: Technical Specifications	30
Appendix D: Electromagnetic Compatibility Guidelines	31

Appendix E: Symbols and Definitions.....37

About This Guide

This guide explains how to use the charging system (Model 16000 and charging system accessories). Follow the information in this guide to correctly use your charging system. If you have any questions about your system, contact Technical Support.

Prescription and Safety Information

Read this section to gather important prescription and safety information.

Intended Use

The charging system is intended to be used to charge the Eterna™ implantable pulse generator (IPG), also referred to as a generator.

Indication for Use

The Eterna™ Spinal Cord Stimulation (SCS) System is indicated as an aid in the management of chronic, intractable pain of the trunk and/or limbs, including unilateral or bilateral pain associated with the following: failed back surgery syndrome and intractable low back and leg pain.

Contraindications

The Eterna™ SCS System is contraindicated for patients who are unable to operate the system. Additionally, the system is contraindicated for patients who have failed to receive effective pain relief during trial stimulation.

Additional Prescription Information

Refer to the user's guide for your patient controller for instructions for use and other important information, including indication for use, contraindications, warnings, precautions, adverse effects, and the personal identification card related to your neurostimulation system.

MRI Safety Information

Do not bring your charger into the scanner magnet room. It can be affected by the magnetic resonance imaging (MRI) magnet, may present a projectile hazard, and is considered MR Unsafe.

Warnings

The following warnings apply to this charging system:

Device inspection. Before using the charging system, check the charger, charging cable, and power adapter for visible damage or indication of prior disassembly. If any damage or prior disassembly is observed, do not use the charging system and contact Technical Support.

Device modification. This charger is not serviceable by the customer. To prevent injury to yourself or damage to the charging system, do not modify the charger. If needed, return the charger to Abbott Medical for service.

Battery warning. The charger contains a lithium-ion battery and other potentially hazardous materials. Do not crush, puncture, or burn the charger because explosion or fire may result.

Battery care. Batteries should not come in contact with metal or be exposed to high temperature or fire. Batteries can explode, leak, or melt if disassembled, shorted (when battery connections contact metal), or exposed to high temperature or fire.

Power adapter and charging cable compatibility. Use only power adapters and charging cables that meet the ratings specified in “Appendix C: Technical Specifications” (page 30).

Power adapter handling. Do not position the power adapter in such a way that is difficult to disconnect from an electrical outlet.

Explosive or flammable gases. Do not use your charger around explosive or flammable gas fumes or vapors. This includes oxygen-enriched environments such as hyperbaric chambers. Operating the device near gas fumes or vapors could cause them to catch fire. If gas fumes or vapors catch fire, it could cause severe burns, injury, or death.

Strangulation. The cords in this charging system pose a strangulation risk. Keep cords out of the reach of children.

Choking. Small parts in this charging system pose a choking risk. Keep small system parts out of the reach of children.

Keep dry to avoid damage. Your charger is not waterproof. Keep it dry to avoid damage. Never use the charger in or around water. If the charger becomes wet, allow it to dry completely before use.

Precautions

The following precautions apply to this charging system.

Device components. The charging system is for use with the Eterna™ implantable pulse generator only. You may use a commercially available power adapter and charging cable as long as it meets the technical specifications listed in “Appendix C: Technical Specifications” (page 30).

Handle the device with care. The charger is a sensitive electronic device that can be damaged by rough handling, such as dropping it on the ground.

Review your user’s guide. Before using your charger, review the system information in the applicable patient controller user’s guide.

Infection. Follow proper infection control procedures. Also avoid showering and touching the bandages while your incisions are healing. If you need to charge your generator before your incisions have healed completely, take care not to disturb the bandages and follow your physician’s guidance on charging.

Home use. This product is intended for home use per clinician instruction. To avoid damage and other potential hazards, keep this product away from children and pets.

Pain-relief patches. Avoid using pain-relief patches or topical anesthetics, such as pain-relieving creams or medicated balms, over your generator site before or during charging. These pain-relief patches or topical anesthetics may reduce your ability to perceive discomfort due to charging.

Alcohol consumption. Avoid drinking alcoholic beverages before or during charging because they may reduce your ability to perceive discomfort due to charging.

Charging system immunity. Wireless communication equipment, such as wireless home network devices, mobile and cordless telephones, and walkie-talkies, can interfere with the charging system. To correct the effects of typical system interference, keep the charging system at least 0.5 m (1.6 ft) from these types of wireless communication equipment. If you continue to have problems with interference, further increase the distance between the equipment and your charging system.

Charge your charger. The charger should be fully charged after every generator charging session.

Unplug the cable carefully. To avoid damaging the charging cable, do not pull on cable when unplugging it from the charger or power adapter. Grasp the connector and pull carefully.

Patient training. Patients should use their neurostimulation system only after an authorized clinician has programmed their generator and has trained them on how to safely control stimulation and to charge the system.

Adverse Effects

The following adverse effects are associated with using this charging system. Refer to the patient controller user's guide for a complete list of adverse effects related to the use of the Eterna™ SCS system.


- **Heating or discomfort at the pocket while charging.** If you feel discomfort at the implant site while charging your generator, stop using the charger. Use your patient controller to change the charging mode or contact your physician.
- **Heating or discomfort at surgical staples while charging.** Charging should not be carried out while surgical staples are in place.
- **Interference with another implanted medical device.** This equipment generates, uses, and can radiate radiofrequency (RF) energy that may interfere with other electronic devices, including other implanted medical devices. To try to correct the effect of any interference with other devices, turn off the charging equipment or increase the distance between the charging equipment and the device being affected.

Product Description

The charging system consists of a charger, a charging cable, a power adapter, charging apparel, and a travel case for storing the charging system. The charger is used to charge the Eterna™ implantable pulse generator wirelessly. The charger interfaces with the patient controller application wirelessly through Bluetooth® wireless technology. The patient controller application allows you to modify charger settings, view the battery level of your charger and generator, and view the charging status of your charger and generator.


Charger

The charger is used to wirelessly charge your Eterna™ implantable pulse generator.

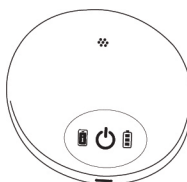


Power adapter

The power adapter connects with an electrical outlet and with the charging cable to charge the charger battery.

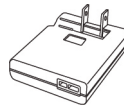


Charging cable

The charging cable connects with the power adapter and with the charger to charge the charger battery.

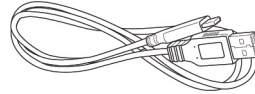


Charging apparel – abdominal

Charging apparel is used to position and stabilize the charger over your generator and charge the generator without having to manually hold the charger in place. The abdominal apparel is worn like a belt around your waist.

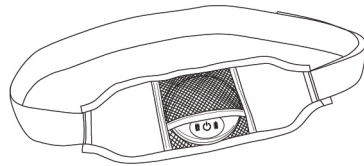


Charging apparel – thoracic

Charging apparel is used to position and stabilize the charger over your generator without having to manually hold the charger in place. The thoracic apparel can be worn in multiple configurations using the double strap or the counterweight.

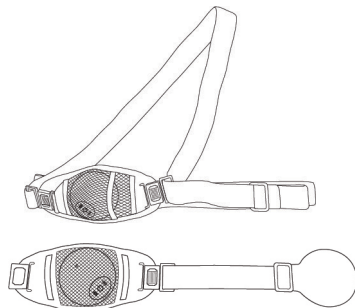


Charger travel case

The travel case is used to store the parts of the charging system.

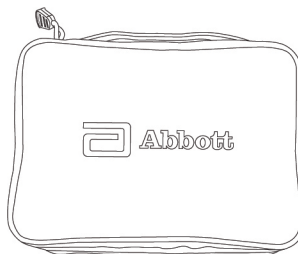

Adhesive

The single-use adhesive (available separately) can be used in place of the charging apparel to position and stabilize the charger over your generator without having to manually hold the charger in place.

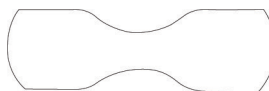

Charger Indicators

The charger has a power icon, a battery icon, and a patient controller alert icon as shown in the figure below. The charger provides indicator lights and sounds that show the charger status and the action being performed.

- PC Alert icon—The PC alert icon illuminates when the charger has encountered an error. You may view more information about the error on your patient controller.
- Power icon—The power icon illuminates when the charger is turned on. When the charger is charging the generator, the power icon blinks slowly. When the charger is in pairing mode, the power icon blinks quickly.
- Battery icon—The battery icon illuminates in segments to indicate the charger battery level. While the charger is being charged, a segment of the battery icon blinks.

NOTE: The indicator lights on the charger may be difficult to see if under direct sunlight.

Figure 1. Charger icons

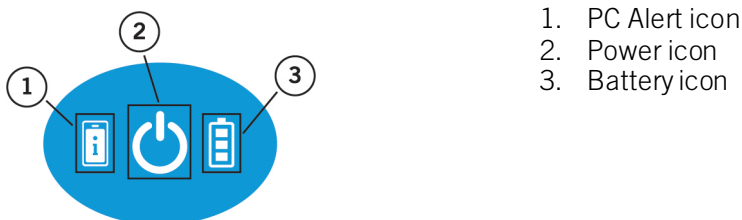


Table 1. Charger sounds

Event	Charger sound
Power on	1 beep
Volume on	1 beep
Power off	2 beeps
Error	3 beeps every 5 seconds
Low battery	3 beeps every 5 minutes
Generator found	Short, repeating beeps
Generator charging started	2 beeps low tone then high tone
Generator charging stopped (not aligned)	2 beeps high tone then low tone
Generator charging complete (fully charged)	3 beeps low tone, high tone, then low tone

Using Your Charger

The instructions in this section explain how to use your charging system to charge your generator battery and charger battery, how to use the charging apparel, and how to care for and travel with your charging system.

NOTE: Each time before using the system, inspect the device and its accessories for damage. Avoid using a damaged device or accessory. If your device is damaged, contact Technical Support.

Pairing the Charger with the Patient Controller

Before you use your charger for the first time, you may want to pair the charger with your patient controller. Pairing refers to setting up communication between the charger and the patient controller. When these devices are paired, you can use your patient controller to monitor charging sessions, adjust charger settings, such as volume control, and perform charger software updates. Refer to your patient controller user guide for more information.

NOTE: Your charger and patient controller do not need to be paired or connected to Bluetooth® wireless technology in order to charge your generator.

To prepare the charger for pairing, do the following:

1. Ensure that your patient controller and charger are charged.
2. Place the patient controller near the charger.
3. Turn on the charger.
4. Press and hold the power icon on the charger for 5 seconds, and then release the button. The power icon and the battery icon blink once. Then the power icon blinks quickly, indicating the charger is in the pairing state.
5. On the patient controller, select the charger you are pairing. Refer to your patient controller manual for more information.

NOTE: If the pairing does not occur within 5 minutes, the charger exits pairing mode and then automatically turns off after 90 seconds of inactivity.

When to Charge Your Generator

Your clinician will help you plan a charging schedule that will work best for you to maintain effective therapy. For example, you may establish a charging routine of once per week.

The following are additional considerations to help you decide when to charge your generator:

- Prior to an MRI
- Prior to an upcoming surgery
- In anticipation of travel
- When the next charging opportunity is uncertain
- When the patient controller application displays a low battery notification for your generator
- When stimulation has stopped due to generator battery depletion

Your patient controller indicates the battery level of your generator. The level ranges from 0% charge to 100% charge. The patient controller also indicates the approximate stimulation time remaining based on the generator's current battery level and therapy settings.

When your implanted generator battery is low, your patient controller displays a low battery message. When this message appears, you should charge your generator at your earliest convenience. If you do not charge your generator after the low battery message appears, your stimulation will eventually stop. When stimulation stops, your controller will display an alert indicating therapy was turned off due to a low generator battery. You should charge your generator battery so that therapy can be restored.


NOTE: To maintain therapy without interruption and improve battery longevity, always charge your generator before it is depleted completely. If your generator is repeatedly allowed to deplete completely, its performance or longevity may be reduced over time.

NOTE: If your generator is depleted completely, it will take a few minutes for the generator to turn on again and communicate with your patient controller once you start charging.

Charging Your Charger

Like your generator, your charger contains a rechargeable battery that needs to be charged on a regular basis. The battery icon on the charger displays the charger battery level. The patient controller also displays the charger battery level. If the charger is in range of the patient controller or connected to it, the battery level is indicated by a battery icon with colored lights. See the following figure.

Figure 2. Charger battery level

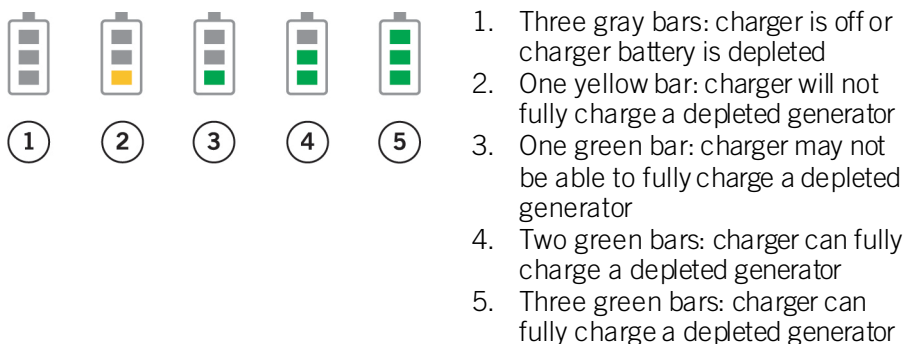

NOTE: One yellow bar indicates you have 30 minutes or less remaining before the charger turns off due to low battery.

NOTE: When the charger battery is low, the charger beeps three times every 5 minutes until the battery is depleted.

To charge your charger, follow these steps:

1. If the charger is in the charging apparel, remove the charger from the apparel before charging.


NOTE: The charging apparel is not intended to be used while charging the charger.

2. Plug the power adapter into an electrical outlet.

NOTE: While the charger is charging, it should be placed on a stable surface, such as a table, counter, or floor.

3. Connect the charging cable and power adapter.
4. Plug the charging cable into the charger. If the charger is connected successfully to the charging cable and power adapter, the charger beeps once and a segment of the battery icon blinks and then turns off. Once the charger is fully charged, all the segments of the battery icon illuminate for 60 seconds and then turn off.

Figure 3. Connecting components to charge the charger

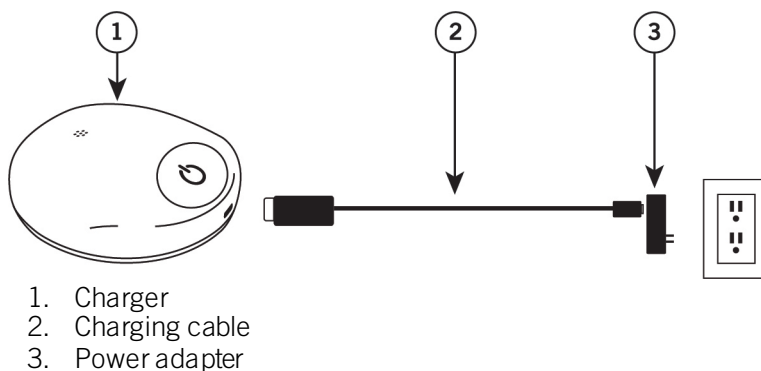

NOTE: While the charger is charging, it cannot be used to charge the generator.

NOTE: Your charger is designed to be charged using a USB Type-C to USB-A cable. If your charger is not charging, you may be using an incorrect cable or power adapter. See “Appendix C: Technical Specifications” (page 30).

5. When you are ready to use the charger, unplug the charging cable from the charger. The power adapter connected to the charging cable can remain plugged into the electrical outlet.

While the charger is being charged, the charger is able to communicate with the patient controller. The patient controller indicates the charger battery level and when the charger is being charged.

Setting Up the Charging Apparel

Your charging system comes with reusable charging apparel that you can use, if desired, to help stabilize the charger over your generator during a charging session. Depending on the implant location of your generator, you will receive either thoracic charging apparel or abdominal charging apparel. The charging apparel can be configured to be worn in different ways, depending on your preference and implant location. For example, patients with generators implanted in the lower back may choose to use the abdominal charging apparel like a belt. If you choose to use your charging apparel, you should adjust it for comfort and fit before charging your generator.

Your clinician will help you find the configuration that is most comfortable and suitable for your generator charging sessions.

NOTE: You can use the charging apparel over your clothing. However, if your clothing is very thick, the charger may not effectively charge your generator.

Placing Abdominal Charging Apparel

To set up your abdominal charging apparel, gather the charger and the charging apparel and follow these steps:

1. If the charger is connected to the charger cable, disconnect it.
NOTE: While the charger is charging, it cannot be used to charge the generator.
2. Press the power icon to turn on the charger. The three charger icons illuminate briefly and the charger beeps once if sound is enabled.
3. Insert the charger fully into the mesh pocket of the charging apparel with the icons facing the opening of the pocket.
4. Ensure the elastic edges of the pocket overlap, fully covering the charger.
5. Wrap the charging apparel around your waist so the charger is over your generator.
6. Adjust the velcro strap so the apparel is secure around your waist and the charger is on top of your generator.

NOTE: When the generator enters the charging zone, it beeps continuously until charging begins.

NOTE: If the charger does not find the generator within 90 seconds, the charger turns off automatically. You can press the power icon through the mesh pocket to turn on the charger and align the charger over your generator.

7. Once you have adjusted the charging apparel to a comfortable fit around your waist, you can trim the excess strap length, if desired.

Placing Thoracic Charging Apparel

To set up your thoracic charging apparel, gather the charger and the charging apparel and follow these steps:

1. If the charger is connected to the charging cable, disconnect it.
NOTE: While the charger is charging, it cannot be used to charge the generator.
2. Press the power icon to turn on the charger. The three charger icons illuminate briefly and the charger beeps once if sound is enabled.
3. With the charger icons facing the opening of the pocket, insert the charger into the mesh pocket of the charging apparel by lifting one side of the opening and then pulling the other side over the charger.
4. Ensure the elastic edges of the pocket overlap, fully covering the charger.



NOTE: When the generator enters the charging zone, it beeps continuously until charging begins.

NOTE: If the charger does not find the generator within 90 seconds, the charger beeps two times and then turns off automatically. You can press the power icon through the mesh pocket to turn on the charger and align the charger over your generator.

Counterweight configurations

1. If connected, release the long strap from the charger pocket. The counterweight should remain buckled to the charger pocket.
2. Position the charger over your generator.
3. Place the counterweight over your shoulder so it rests on your back or around your neck so the counterweight rests on your opposite shoulder.


NOTE: You may need to adjust the length of the strap to fit properly.

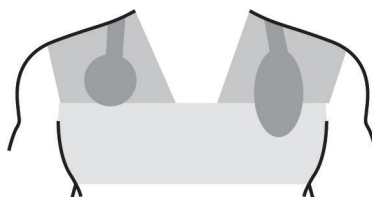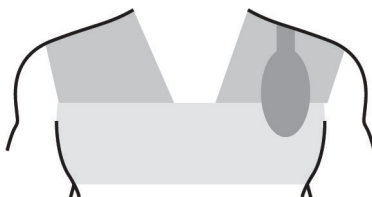


Single-strap configurations

1. Release the counterweight from the charger pocket.
2. Wrap the charging apparel around your chest or over your shoulder and fasten the buckle.

NOTE: If needed, use the excess strap near the buckle to tighten or loosen the apparel.

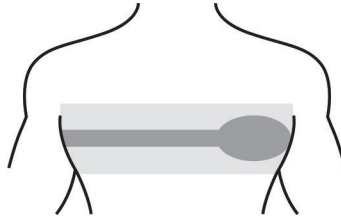

3. Adjust the strap until the charger is comfortably positioned over your generator.

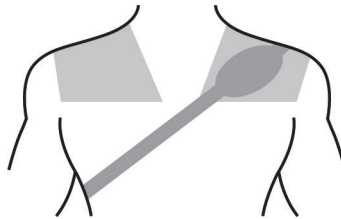


Double-strap configurations

Option 1 – Loop and charger on opposite sides

1. Release the counterweight from the charger pocket.
2. Grasp the plastic buckle so the charger pocket is hanging toward your feet.

NOTE: If your generator is located on the right side of your body, grasp the buckle with your right hand. If your generator is on your left side, grasp the buckle with your left hand.

3. Put the arm on the opposite side of your generator through the loop.

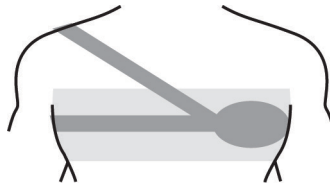


4. Adjust the charging apparel so that a strap is comfortably over your shoulder and the other strap is under your armpit. The buckle should be in front of you.
5. Ensure that the charger is positioned over your generator.
6. Fasten the buckle.
NOTE: If needed, use the excess strap near the buckle to tighten or loosen the apparel.

Option 2 – Loop and charger on same side

1. Release the counterweight from the charger pocket.
2. Grasp the charger pocket so the long strap is hanging toward your feet.

NOTE: If your generator is located on the right side of your body, hold the pocket with left hand. If your generator is on your left side, hold the pocket with your right hand.

3. Insert the opposite arm (the arm that is not holding the charger pocket) through the loop.
4. Adjust the charging apparel so that a strap is comfortably over your shoulder and the other strap is under your armpit. The buckle should be in front of you.
5. Ensure that the charger is positioned over your generator.
6. Fasten the buckle.
NOTE: If needed, use the excess strap near the buckle to tighten or loosen the apparel.

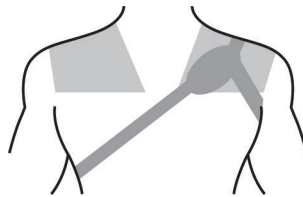

Placing the Adhesive

If you prefer to use the adhesive instead of the charging apparel to hold your charger in place while charging the generator, follow these steps:

NOTE: If the adhesive has exceeded its expiration date, do not use it and discard it.

NOTE: If you have sensitive skin, test a small part of the adhesive by applying it and removing it from your skin before applying it fully.


1. Hold the adhesive between the power icon and the speaker holes on top of the charger.
2. Peel the middle segment of the backing and adhere the adhesive to the charger.
3. Place the charger over your generator.

NOTE: If the charger is being charged, it cannot be used to charge the generator.

4. Peel the remaining backing from the adhesive and adhere to the skin, ensuring that the adhesive and charger are held securely on the skin.

NOTE: Remove the adhesive from the charger and skin as soon as you finish the charging session to avoid adhesive residue remaining on the charger or skin.

Figure 4. Adhesive placement

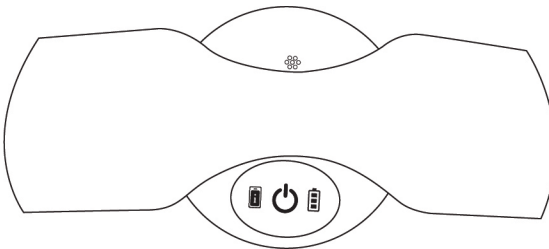

Charging Your Generator

After you have charged your charger and have set up the charging apparel (if desired), you are ready to charge your generator. It is recommended to charge your generator frequently and to charge before stimulation turns off due to a low battery.

NOTE: You can receive stimulation while charging your generator. The patient controller can still communicate with the generator while charging. However, the charger software cannot be updated while the generator is charging.

NOTE: If you have more than one Eterna™ implantable pulse generator implanted, you can use the same charger to charge your generators.

WARNING: You may perceive an increase in temperature at the generator site while you charge. Ensure that the area around the charger is well ventilated and not covered by an insulating material such as a blanket. If you feel discomfort while recharging your generator, stop using the charger and refer to “Troubleshooting” (page 20). If the discomfort persists, contact your clinician. Only charge your generator while you are awake so that you can stop the charger if charging becomes uncomfortable.

To charge your generator, follow these steps:

1. Ensure the charger is not connected to an electrical outlet.
NOTE: If the charger is connected to an electrical outlet, it will not charge the generator.
2. Turn on the charger. The three icons on the charger illuminate and the charger beeps once if the sound is enabled in the patient controller.
3. Place the charger directly over the generator, using the charging apparel to hold the charger in place if desired. Charging begins automatically when the charger is positioned correctly over the generator. If charging has not begun, try the following, one at a time, until charging begins:
 - Ensure that the charger is centered over the top half of your generator.
 - Move the charger slowly in a circular motion over the generator location, slowly increasing the size of the circle until you hear the beeps begin, indicating the charger is aligned with the generator and charging has begun.
 - Slowly swipe the charger across the generator in a single direction until you hear the beeps begin, indicating the charger is aligned with the generator and charging has begun.

4. To confirm charging is occurring, note the following. If you do not note any of the indicators listed below, charging is not occurring.
 - If the charger sound is enabled in the charger settings on the patient controller, the charger beeps when charging begins. When charging is in progress, the charger will not beep.
 - The charger power icon begins blinking slowly.
 - The patient controller displays the charging status and the generator battery level.

NOTE: If the charger loses alignment with the generator for 10 seconds, the charger will beep until alignment with the generator is restored or the charger is turned off. Try repositioning the charger so it is positioned over the generator.

5. Verify that charging is complete. When the generator is fully charged, the charger beeps 3 times every 10 seconds if the sound is enabled. You can also view charging status on the patient controller.

NOTE: The charger automatically turns off after 90 seconds once charging is complete.

NOTE: While the charger is charging the generator, the charger cannot be turned off. If you need to turn off the charger while it is charging the generator, move it away from the generator site.

6. Turn off the charger and remove the charging apparel.

NOTE: If you are using the adhesive, remove the adhesive from the charger and skin as soon as you finish the charging session to avoid adhesive residue remaining on the charger or skin.

Caring for and Maintaining Your Charger and Charging Apparel

To keep your charger running its best, follow these guidelines:

- Keep the charger dry. Do not use the charger in or around water.
- Clean the charger as needed. You can use a dry cloth or a cloth slightly dampened with water or isopropyl alcohol. Ensure the charger is completely dry before use.

NOTE: To avoid damaging the charger, do not insert anything in the speaker holes.

- Clean the charging apparel as needed.
 - Hand wash in cold water.
 - Use mild soap.
 - Lay flat to dry.

NOTE: Do not machine wash or machine dry.

- Charge your charger and store your charging system in the travel case after each use.
- Store and use your charging system within the range of conditions as shown in the following table.

Table 2. Storage, transportation, and operating conditions

Storage and transportation conditions	
Long term storage temperature	23°C (73°F)
Temperature excursions	Excursions below 15°C (59°F) or above 30°C (86°F) should be < 24 hours in duration
Temperature excursion limits	-20°C (-4°F) to 60°C (140°F)
Humidity	5% to 90% RH
Pressure	70 kPa to 106 kPa
Operating conditions	
Temperature	5°C (41°F) to 35°C (95°F)
Humidity	10% to 90% RH
Pressure	70 kPa to 106 kPa

NOTE: If the charger was stored in an environment that is outside of normal operating temperature, allow it to acclimate to the current environment for approximately 30 minutes.

- When the charger is approaching its end of service, an alert message displays on the patient controller. This message indicates that you should contact your physician for a replacement charger.

Traveling with Your Charging System

If traveling with your charging system, follow these guidelines:

- Use the travel case to store your charging system.
- Be sure to pack the charging cable and power adapter.
 Note: For short trips, if you plan to travel without the charging cable and power adapter, check the battery level of the generator and the charger to ensure continued therapy during your trip.
- If you are traveling to an international location, you may need a country-specific power adapter.

- The charger contains a lithium-ion battery. If traveling by air, the charger should be placed and kept in your carry-on baggage during flight, not in your checked baggage.
- When traveling, follow all guidelines and restrictions for use of electronic devices.

Troubleshooting

This section provides information for solving problems you may have using your charging system.

NOTE: If the PC Alert icon  illuminates, check your patient controller for more information.

NOTE: If problems occur other than those described in this section, contact Technical Support.

NOTE: Refer to the terms and conditions for repair or replacement of Abbott Medical neurostimulation system components as stated in the Limited Warranty card included in your product documentation.

Troubleshooting Charging Issues

The following table lists possible charging system problems and possible solutions.

Table 3. Troubleshooting charging issues

Problem	Possible Solution
The charger will not turn on.	Ensure the power cable is disconnected from the charger.
	Ensure the charger has sufficient charge.
The charger is not able to establish or maintain charging of the generator.	Reduce the distance between the charger and the surface of the skin as much as possible.
	Position the charger flat against the surface of the skin.
	Locate the generator under your skin and center the charger over the top of your generator.

Table 3. Troubleshooting charging issues

Problem	Possible Solution
	Enable sound on the charger to use audible tones while aligning the charger over the generator. A series of tones will indicate that charging has begun.
	Avoid sitting on or against hard surfaces that may push the charger out of position during charging.
	Ensure the charger is not plugged into an electrical outlet.
	If you do not feel the generator close to your skin, center the charger over the generator site and move the charger in small circles until the charger emits a tone and charging begins.
	If none of the charger icons are illuminated, ensure sure the power cable is disconnected and turn the charger off and on.
The temperature of the generator becomes uncomfortable during charging.	Stop charging until the discomfort subsides and then resume charging.
	If your clinician has enabled the Charging mode feature, change the Charging mode to low. See the patient controller manual for more information about this feature.
	Ensure the area around your generator is well ventilated.

Table 3. Troubleshooting charging issues








Problem	Possible Solution
	<p>Reposition the charger, aligning the bottom of the charger with the bottom edge of the generator.</p> 
	<p>Ensure the charging apparel is centered over the generator.</p>
	<p>Tighten the charging apparel to ensure the charger is positioned correctly over the generator.</p>
	<p>If you can feel the generator close to your skin, try adding a thin material on your skin over the implant site.</p>
	<p>Consider recharging more frequently for less time.</p>
	<p>If the temperature continues to be uncomfortable, contact your physician.</p>
The charger is not charging.	<p>Ensure the charging cable is a USB-IF certified Type-C to USB-A cable. To determine if a cable is USB-IF certified, look for markings such as the following:</p>  
	<p>Ensure the power adapter has one or more of the following certification markings:</p>    

Table 3. Troubleshooting charging issues

Problem	Possible Solution
The charger is not making sounds.	The charger volume may be set too low. Turn the volume up. See the user’s guide for your patient controller for instructions.
	The charger volume may be turned off. Turn the volume on. See the user’s guide for your patient controller for instructions.
The charger is not staying in the pocket of the charging apparel.	Ensure that the top flap of the charging apparel is over the charger.
The charger is not able to maintain Bluetooth® wireless technology with the patient controller.	Avoid charging your patient controller while charging your generator.
The charger will not turn off.	Contact Technical Support.

Service and Ordering Information

This section provides information for contacting Technical Support and for ordering replacement parts and accessories.

Technical Support

For technical questions and support for your product, use the following information:

- +1 855 478 5833 (toll-free within North America)
- +1 651 756 5833

For additional assistance, call your local Abbott Medical representative.

Ordering Information

To order parts, contact Technical Support. Refer to the following list for order numbers.

Table 4. Ordering information for the charging system

Description	Model
Charger kit - abdominal	36000
Charger kit - thoracic	66000
Charger	16000
Charging apparel - abdominal	16750
Charging apparel - thoracic	16760
Travel case	16730
Adhesive	14700
Power adapter/adapter cable	16740

Appendix A: Regulatory Statements

Please note the following regulatory statements related to the charging system.

Disposal Guidelines for Battery-Powered Devices

This device contains a battery and a label is affixed to the device in accordance with European Council directives 2002/96/EC and 2006/66/EC. These directives call for separate collection and disposal of electrical and electronic equipment and batteries. Sorting such waste and removing it from other forms of waste lessens the contribution of potentially toxic substances into municipal disposal systems and into the larger ecosystem. The device and its battery should be disposed of according to local laws and regulations.

Statement of FCC Compliance

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. This device has been tested and found to comply with the limits of an industrial, scientific, and medical (ISM) equipment, pursuant to part 18 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radiofrequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Operation is subject to the following two conditions:

- This device may not cause harmful interference.
- This device must accept any interference received, including interference that may cause undesired operation.

Modifications not expressly approved by the manufacturer could void the user's authority to operate the equipment under FCC rules.

Statement of Compliance With Interference-Causing Equipment Standard (Canada)

This ISM device complies with Canadian ICES-001.

Product Classification Statement (CISPR 11, Class B)

This product is class B equipment, which is intended primarily for use in the domestic environment.

Identification Information for Product Registration

This device has a label that contains, among other information, a product identifier in the following format:

Table 5. Registration identification information

Identifier Type	Registration Identifier
FCC registration number	PX2-GMCHG1
Industry Canada (IC) registration number	8454A-XXXX

Wireless Technology Information

The following table summarizes technical details of the wireless technology of the charging system.

Table 6. Wireless technology information for charging system

Charging frequency	266 kHz to 320 kHz
Charging distance	0.5 cm to 2.5 cm

NOTE: Refer to “Charging Your Generator” (page 17) for steps on how to successfully align and charge your generator.

Table 7. Bluetooth® Low Energy information

Antenna type	Chip Antenna
Antenna dimensions	7.00 mm x 2.00 mm x 1.20 mm
Modulation	GFSK
Output power (EIRP*)	1.5 mW (+2 dBm) typical
Duty cycle	14.1% or less over 6 min period
Range	Up to 2 m typical

Center frequency	2.44 GHz
Channel frequency range	2.402 GHz to 2.480 GHz
Channel	40 logical channels using AFH
Bandwidth per channel	2 MHz
Data flow	Bi-directional
Protocol	Bluetooth® Low Energy
Semi-duplex capability	Yes

* EIRP = Equivalent isotropically radiated power (not duty cycle adjusted)

The radio receiver in the device is using the same frequency and bandwidth as the transmitter.



Cables and transducers:

Cables and transducers are not used during normal use of the device.

Quality of Service for Wireless Technology

Bluetooth® Low Energy enables communication between the charger and the clinician programmer or patient controller. The quality of the wireless communication link varies depending on the use environment (operating room, recovery room, and home environment).

After the clinician programmer or patient controller is paired with a charger, the Bluetooth® wireless technology symbol is visible on the clinician programmer or patient controller in the upper right-hand corner of the screen. When Bluetooth® Low Energy is not active, the symbol appears dimmed.

The quality of service (QoS) should allow wireless data to be transferred at a net rate of 2.5 kB/sec. Each connection interval includes a semi-duplex transmission with a required acknowledge, a transmission latency in each direction (2x), and a receive-to-transmit mode (RX-to-TX) time. Data is resent if not successfully received. Each key press may transmit up to 4 data packets with up to 20 bytes per packet, depending on the number of packets that need to be transmitted (that is, if there is only one packet to transmit, only one packet will be transmitted). If the interference is high (for example, the bit error rate exceeds 0.1%), the user may experience what appears to be a slow connection, difficulty pairing devices, and a need to decrease the distance between connected devices. For information on how to improve connection issues, please refer to “Troubleshooting for Wireless and Coexistence Issues” (page 28).

Wireless Security Measures

The wireless signals are secured through device system design that includes the following:

- The charger encrypts its wireless communication with the programming device using a key that is unique to that link.
- Only one patient controller or clinician programmer may communicate with the charger at a time.
- Standard Bluetooth® Low Energy pairing methods ensure valid and legitimate pairing among devices.
- Proprietary authentication for the patient controller or clinician programmer applications in addition to the pairing procedure specified in Bluetooth® Low Energy, which includes an element of proximity.
- A proprietary algorithm that detects and prevents an unauthorized application from communicating with the charger.
- Whitelisting methods that prevent unauthorized devices from using Bluetooth® Low Energy scanning to interfere with communication from the charger to a legitimate patient controller or clinician programmer.

Troubleshooting for Wireless and Coexistence Issues

If you experience issues with the wireless communication between the charger and the patient controller, try the following:

- Decrease the distance between the devices.
- Move the devices so they share line of sight.
- Move the devices away from other devices that may be causing interference.
- Close the patient controller app; then turn the patient controller off and on
- Wait a few minutes and try connecting again.
- Do not operate other wireless devices (i.e., laptop, tablet, mobile phone, or cordless phone) at the same time.

NOTE: Wireless communication equipment, such as wireless home network devices, mobile and cordless telephones, and tablets, can affect the device.

Appendix B: Product Specifications

The charging system has the following product specifications and compatibility information.

Table 8. Product specifications

Charger input	5 V, 0.5 A
Charge connector type	USB Type C
Charger power source	Rechargeable lithium ion cell battery
Charger IP code	IP22
Charger communication protocol	Bluetooth® Low Energy
Charger expected service life	5 years
Charging accessories expected service life	5 years

NOTE: The charger is considered an applied part per the 60601-1 standard when it comes into physical contact with the user during charging.

NOTE: The USB port in the charger is restricted for charging the charger and is not intended to perform any communication per the USB protocol.

Table 9. Implantable pulse generator compatibility for the charging system

Description	Model
Eterna™ implantable pulse generator	32400

Appendix C: Technical Specifications

Use the charging cable and power adapter included with this system. You may also use a charging cable and power adapter with the following ratings.

Table 10. Technical specifications

Charging cable ratings

Voltage rating	Minimum 20 V
Current rating	Up to 3 A
Cable length	1 m
USB-IF	Certified Only use an Abbott Medical charging cable (model 16710) or a USB Type-C to USB-A cable marked with USB-IF certification such as the following:



Power adapter ratings

Power	5.25 W maximum
Input voltage	100-240 VAC
Input frequency	50-60 Hz
Output voltage	5 VDC
Output current	up to 1 A

Power adapter certification marking Power adapter should have one or more of the following certification markings:



Appendix D: Electromagnetic Compatibility Guidelines

The charging system (Model 16000), hereafter the device, is medical equipment and should be used with the following guidance:

The device requires special precautions with regard to electromagnetic compatibility (EMC) and should be used in accordance with the information provided in this manual.

The device has essential performance, as defined by IEC 60601-1:2005+AMD1:2012 (Ed 3.1). Performance of the device was maintained during electromagnetic disturbance testing.

The device is intended for use in the electromagnetic environment specified in the following tables. The user should ensure that it is used in such an environment.

CAUTION: The device complies with the limits for medical devices contained in IEC 60601-1-2:2014 (Ed 4.0) and CISPR11:2015/AMD1:2016 (BS EN 55011:2016/AMD1:2017). However, the device may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to mitigate this effect by reorienting or relocating the receiving device.

CAUTION: To avoid increasing emissions or decreasing immunity from a device or system, use only components approved by Abbott Medical with this system. Do not use Abbott Medical components with devices or systems that are not approved by Abbott Medical.

Table 11. Guidance and Manufacturer's Declaration—Electromagnetic Emissions

Emissions Test	Compliance	Electromagnetic Environment Guidance
RF emissions CISPR 11	Group 1	The device produces RF emissions as a result of its internal and system interface functions. Its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments including domestic establishments and those directly connected to the public low-voltage power supply network
Harmonic emissions	Class A*	

Table 11. Guidance and Manufacturer's Declaration—Electromagnetic Emissions

Emissions Test	Compliance	Electromagnetic Environment Guidance
IEC 61000-3-2		that supplies buildings used for domestic purposes.
Voltage fluctuations/ flicker emissions	Class 5 Full Compliance*	
IEC 61000-3-3		

* Only applicable to Power Adapter (Model 16720)

Table 12. Guidance and Manufacturer's Declaration—Electromagnetic Immunity

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±15 kV air	±8 kV contact ±15 kV air	The relative humidity should be at least 10%.
Electrical fast transient/burst IEC 61000-4-4*	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV for power supply lines ±1 kV for input/output lines	
Surge IEC 61000-4-5*	±1 kV line to line ±2 kV line to ground	±1 kV line to line ±2 kV line to ground	Mains power quality should be that of a typical commercial, hospital, or home environment.

Table 12. Guidance and Manufacturer's Declaration—Electromagnetic Immunity

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment Guidance
Voltage dips, short interruptions IEC 61000-4-11*	$<5\% U_T$ ($>95\%$ dip in U_T) for 0.5 cycle $40\% U_T$ (60% dip in U_T) for 5 cycles $70\% U_T$ (30% dip in U_T) for 25 cycles $<5\% U_T$ ($>95\%$ dip in U_T) for 5 s	$<5\% U_T$ ($>95\%$ dip in U_T) for 0.5 cycle $40\% U_T$ (60% dip in U_T) for 5 cycles $70\% U_T$ (30% dip in U_T) for 25 cycles $<5\% U_T$ ($>95\%$ dip in U_T) for 5 s	Mains power quality should be that of a typical commercial, hospital, or home environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial, hospital, or home environment.

Table 12. Guidance and Manufacturer's Declaration—Electromagnetic Immunity

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment Guidance
Conducted RF IEC 61000-4-6	<div>3 Vrms 150 kHz to 80 MHz</div> <div>6 Vrms ISM/Radio bands between 150 kHz to 80 MHz 80% AM at 1 kHz</div>	<div>3 Vrms</div> <div>6 Vrms</div>	<p>The device is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.</p> <p>Recommended minimum separation distance for higher immunity test levels</p> $E = \frac{6}{d} \sqrt{P}$ <p>where P is the maximum power in watts (W), d is the recommended separation distance in meters (m), and E is the immunity test level in volts per meter (V/m).</p>
Radiated RF IEC 61000-4-3	<div>10 V/m 80 MHz to 2.7 GHz 80% AM at 1 kHz</div>	10 V/m	

* Only applicable to Power Adapter (Model 16720)

NOTE: U_r is the AC mains voltage prior to application of the test level.

NOTE: These guidelines may not apply in all situations.
 Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

Table 13. Guidance and Manufacturer's Declaration—Proximity Fields from RF Wireless Communication Equipment

Proximity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment Guidance
IEC 61000-4-3	385 MHz: 27 V/m @ 18 Hz pulse modulation	27 V/m	Recommended minimum separation distance d = 0.3 m
	450 MHz: 28 V/m @ FM modulation	28 V/m	
	710 MHz, 745 MHz, 780 MHz: 9 V/m @ 217 Hz pulse modulation	9 V/m	
	810 MHz, 870 MHz, 930 MHz: 28 V/m @ 18 Hz pulse modulation	28 V/m	
	1720 MHz, 1845 MHz, 1970 MHz: 28 V/m @ 217 Hz pulse modulation	28 V/m	
	2450 MHz: 28 V/m @ 217 Hz pulse modulation	28 V/m	
	5240 MHz, 5500 MHz, 5785 MHz: 9 V/m @ 217 Hz pulse modulation	9 V/m	

WARNING: Do not use portable RF communications equipment (including peripherals such as antenna cables and external antennas) closer than 30 cm (12 inches) to any part of the device, including cables specified by Abbott Medical. Otherwise, performance degradation may occur.

Table 14. Guidance and Manufacturer's Declaration—Enclosure Port Immunity to RF Wireless Communication Equipment

Immunity Test	Band (MHz)	Maximum Power (W)	Immunity Test Level (V/m)
IEC 61000-4-3	380 to 390	1.8	27
	430 to 470	2	28
	704 to 787	0.2	9
	800 to 960	2	28
	1700 to 1900	2	28
	2400 to 2570	2	28
	5100 to 5800	0.2	9

Table 15. Guidance and Manufacturer's Declaration—Enclosure Port Immunity to Proximity Magnetic Fields

Immunity Test	Test Frequency	Modulation	Immunity Test Level (A/m)
IEC 61000-4-39	30.0 kHz	Continuous wave	8
	134.2 kHz	Pulse modulation 2.1 kHz	65
	13.56 MHz	Pulse modulation 50 kHz	7.5

Appendix E: Symbols and Definitions

The symbols below and harmonized symbols may be found on the product or product label. For harmonized symbols, refer to the Universal Symbols Glossary at medical.abbott/manuals.

Table 16. Symbols and definitions












Symbol	Definition
	Caution
	Refer to instruction manual/booklet
	Consult instructions for use
	Follow instructions for use on this website
	Type BF Applied Part
	MR Unsafe
	NOTE: Magnetic Resonance (MR) Unsafe, an item poses unacceptable risks to the patient, medical staff, or other persons within an MR environment
	Non-ionizing electromagnetic radiation
	Use-by date
	Class II Equipment
	Keep dry; keep away from rain
IP22	Ingress protection rating for a device that is protected from the intrusion of solid foreign objects as small as 12.5 mm in diameter and is protected from vertically dripping water when the device is tilted at an angle up to 15 degrees
	Date of manufacture

Table 16. Symbols and definitions

















Symbol	Definition
	Manufacturing facility
	Long term storage temperature
	Temperature limit
	Excursions permitted within this range
	Humidity limitation
	Atmospheric pressure limitation
	Do not use if package is damaged
	Medical Device
	Catalog number
	Manufacturer
	Packaging unit
	Accessories
	Serial number
	Batch code
	Unique Device Identification
	Prescription use only

Table 16. Symbols and definitions

Symbol	Definition
--------	------------



	<p>This product shall not be treated as household waste. The product and its battery should be disposed of according to local laws and regulations.</p> <p>By ensuring that this product is disposed of properly, you will help prevent potential negative consequences for the environment and human health, which could be caused by inappropriate waste handling of this product. The recycling of materials will help to conserve natural resources.</p>
--	--



	Authorized representative in the European Community
--	---



	<p>European conformity, affixed according to the relevant provisions of European Council Regulation 2017/745 (NB 2797) and RE directive 2014/53/EU Annex II. Hereby, Abbott Medical declares that this device complies with the relevant provisions of this regulation and directive.</p>
--	---

The full text of the European Union RE directive 2014/53/EU declaration of conformity is available at the following internet address: www.neuromodulation.abbott/euconformity.



	<p>This device is listed by the Canadian Standards Association (CSA) International as certified</p>
--	---



	<p>This device is a recognized component by Underwriters Laboratories (UL) Inc.</p>
--	---



	<p>This device complies with part 18 of the FCC rules.</p>
--	--

Abbott Medical
6901 Preston Road
Plano, Texas 75024 USA
+1 855 478 5833
+1 651 756 5833

Abbott Medical
The Corporate Village
Da Vincilaan 11 Box F1
1935 Zaventem
Belgium
+32 2 774 68 11

www.abbott.com



2022-01
ARTEN600076078-4

