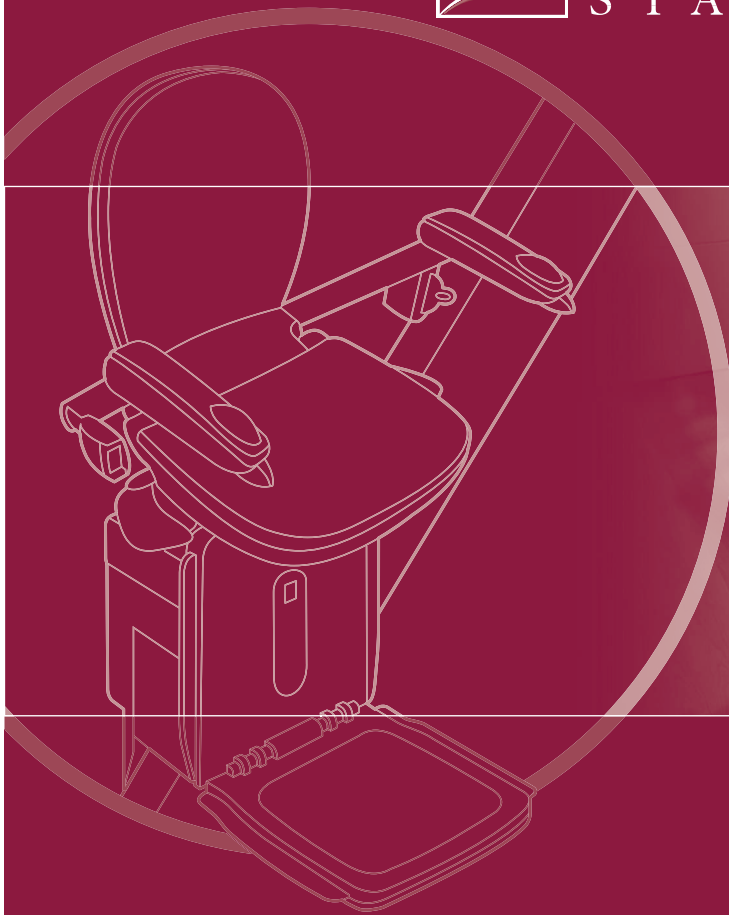
**ACORN**  
STAIRLIFTS

## User Manual

## **Acorn 180** Curved Stairlift

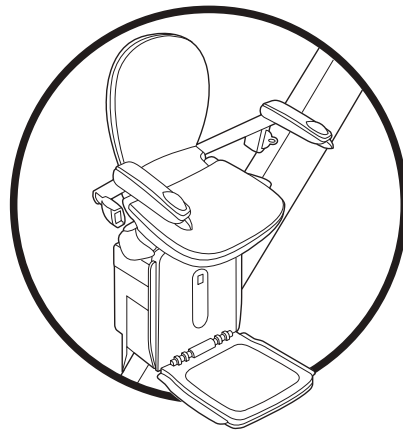
T565 Carriage

**Before using your stairlift, please read through these instructions carefully.**

ENGLISH

### **Thank you for choosing Acorn.**

You can rest assured that your stairlift will provide many years of reliable service and allow you to enjoy the full use of your home.

Your stairlift is covered by a manufacturer's warranty for 12 months that covers the cost of replacement parts.

**However, problems arising as a result of misuse of the equipment are not covered by the warranty.**

Your stairlift is expertly designed and manufactured to the very highest standards. Annual maintenance is required to ensure safe and trouble-free service.

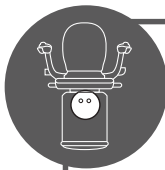
**USA / CANADA ONLY**  
**Some States require that permits are obtained prior to installation, and that a licensed engineer completes the installation. Check your local regulatory requirements.**

Acorn, in keeping with its policy of continual development, reserves the right to change specification without prior notice. All measurements are approximate. All images are shown for illustrative purposes only. Product may vary slightly.

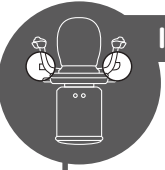
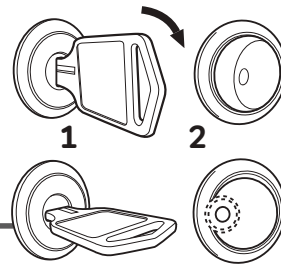
# QUICK USER GUIDE

This is a quick start reference guide to help get you started using your Acorn 180 curved stairlift. **It is important for your safety that you study this manual to completely familiarise yourself with your stairlift.**

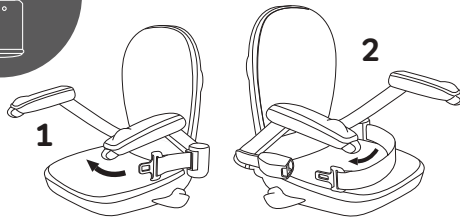
## TURNING THE LIFT ON

- 1 Insert and turn key clockwise.
- 2 Push switch until green light shows.

## IMPORTANT - SEAT BELT MUST BE WORN DURING OPERATION

### SEAT BELT

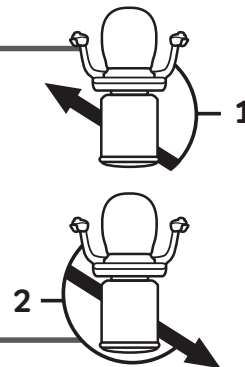
- 1 Retract the seatbelt and pull over lap.
- 2 Push seatbelt into the buckle until it clicks.

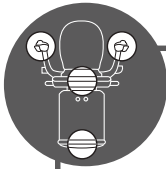
## OPERATING THE LIFT

- 1 Press the switch towards the stairs to go up.
- 2 Press the switch away from the stairs to go down.

# QUICK USER GUIDE

ENGLISH

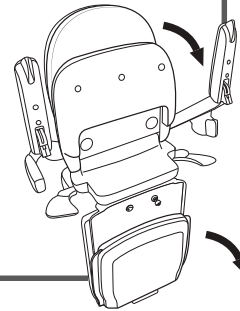
## FOLDING ARMS, SEAT & FOOTREST

Fold down the arms, seat and footrest.

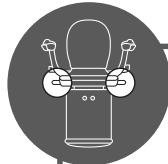
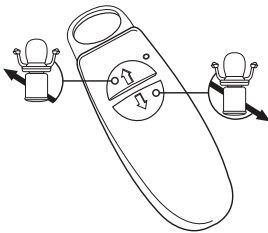
Both arms of the seat must be completely down for the stairlift to operate.

The stairlift can still be called with the remote controls with the arms in any position.

## REMOTE CONTROLS

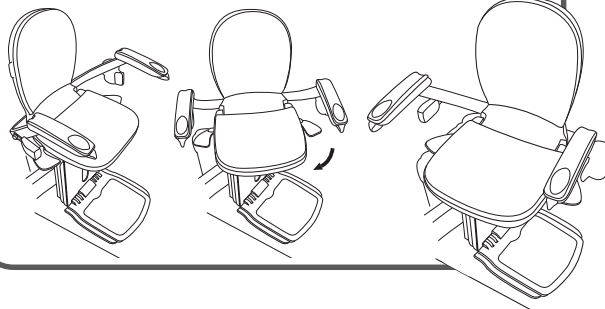
- 1 Press the up arrow for the lift to go up.
- 2 Press the down arrow for the lift to go down.

## SWIVEL SEAT

At the top of the stairs push down on the lever to swivel the seat 45° and 88° to exit the lift.

The seat must be swivelled back into position to operate.

### IMPORTANT INFORMATION FOR USERS WITH A POWERED HINGE

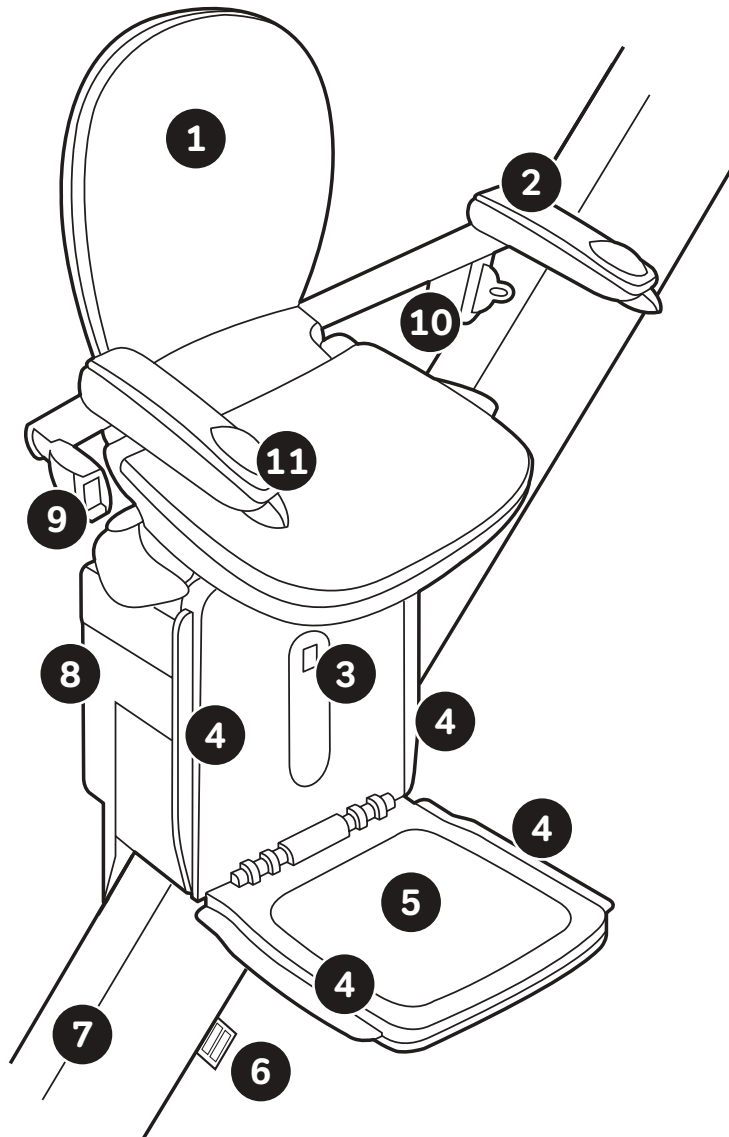
Using the arm control will start the lift in motion. When the lift reaches or leaves the programmed hinge zone the stairlift will stop and the powered hinge will be automatically raised or lowered at the bottom of the staircase.

This will allow the user to operate the hinge whilst in a line of sight. Once you are finished using the stairlift; pressing the up button on the remote handset will automatically run the lift to either, an intermediate charge point if fitted or the top charge point.

**It is important that the stairlift is moved to the nearest charge point to ensure battery drain does not occur. Images shown for illustrative purposes only and may change.**

# YOUR STAIRLIFT

ENGLISH

# YOUR STAIRLIFT

The Acorn 180 Curved Stairlift is an electrically powered stairlift designed for domestic use. If used correctly, it will provide many years of safe, reliable service. It is designed to carry one person weighing no more than 120kg (265lbs/18.9 stone).

**It is important for your safety that you study this manual to completely familiarise yourself with your stairlift.**

The pictures in this manual show a stairlift on the left hand side of the stairs. Stairlifts on the opposite side of the stairs operate in exactly the same way.

ENGLISH

- 1 Seat
- 2 Armrest
- 3 Status indicator
- 4 Safety Edges
- 5 Footrest
- 6 Charging points
- 7 Rail
- 8 Carriage
- 9 Seat swivel release paddle
- 10 Seatbelt
- 11 Direction controller

## GENERAL NOTES ON SAFETY

- To guard against injury, basic safety precautions should be observed.
- Ensure that the stairway is not being used by others and is clear of articles or obstructions before using the stairlift.
- Relax and sit well back in the seat with your arms on the armrests and your feet well back on the footrest.
- An inertia type seat belt is fitted, ensure it is fastened securely at all times during operation.
- Do not use the appliance for other than intended use.

**DANGER!** The stairlift is designed to carry one person at a time. Never attempt to carry more than one person.

**DANGER!** Do not allow children to play with the stairlift. Use by children should always be supervised. For safety, it is advisable to switch off the stairlift and remove the key if there is a risk of tampering by children.

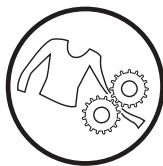
**WARNING!** Ensure other members of your household or visitors are aware that the stairlift is in use; make them aware of any potential trip hazards.

**DANGER!** Ensure that household pets are out of harms way before using the stairlift.

**DANGER!** Ensure that there are no articles of your clothing that could become trapped in the stairlift mechanism, as this could result in personal injury and/or damage to the equipment. Always check that your clothing is clear of the rail and carriage assemblies before using the stairlift.

**CAUTION!** It is possible for small items to be dropped onto the stairlift rail and then slide down the rail and into the stairlift mechanism. If you suspect that any foreign object has become trapped in the equipment, consult your Acorn representative before further use.

**WARNING!** Disconnecting the power supply will cause the stairlift batteries to lose their charge, and the stairlift will not operate (See page 9 - Charging Your Stairlift).

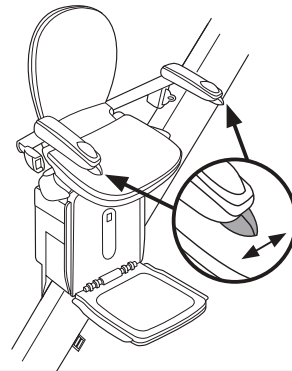
- Do not use the stairlift if you have not been given a demonstration by the installation engineer.
- The stairlift must not be used for firefighting or for evacuation during a fire.
- To avoid injury do not make contact with moving parts.
- Always consult your Acorn representative if the need for an authorised engineer should arise.
- Keep these safety instructions to hand for ease of reference.

# STAIRLIFT OPERATION

## GENERAL OPERATION

The stairlift is moved up and down the stairs by the user operating the direction controller on the armrest. **Arms must be completely in the down position for the stairlift to operate. The stairlift can still be called with the remote controls with the arms in any position.**

 **When not in use, please ensure that the seatbelt is kept clear of the stairlift rail.**

## CHARGING YOUR STAIRLIFT

The stairlift batteries are recharged from charging points when the stairlift is in its parked position at the top and the bottom of the stairs or parking on an intermediate point if a hinge is fitted. The charging points are connected to the mains electricity supply through a DC power supply.

 **IMPORTANT INFORMATION**  
**It is imperative to ensure there is a continual supply of power to the stairlift. The stairlift batteries can cover short-term interruptions to the power supply, but without a supply of power, will eventually become too low to be recharged. If this happens, the batteries will have to be replaced by an engineer, which is not covered by the manufacturer's warranty, and is a chargeable service.**

 **If there is a mains electricity supply failure, or if the supply to your stairlift is switched off, it will continue to operate for several journeys. However, the batteries will eventually lose their charge and the stairlift will not operate.**

# STAIRLIFT OPERATION

## CHARGING YOUR STAIRLIFT (continued)

If the batteries are not being recharged, you will see A1 flashing on the fault indicator, and hear a beep alarm. The stairlift will beep intermittently for 60 seconds and stop entering hibernation mode. The stairlift will beep intermittently every 20 seconds and the fault indicator will go blank..

NOT CHARGING

To ensure that the batteries are kept fully charged, the mains electricity supply should always be switched on and the stairlift parked on its charge points. The symbol will display C1 when charging or C4 when fully charged.

CHARGING

If the batteries have lost their charge because the mains electricity supply has been switched off, or interrupted, allow them to recharge before using the stairlift.

CHARGED

If the stairlift has not been charging due to lack of mains power (during a power interruption), or the lift has been used frequently enough to show a low battery sign show on the digital display E7, the stairlift will allow the user to bring the unit DOWN to the nearest charging station. The unit will not allow the user to travel in the UP direction until the batteries are sufficiently charged.

LOW BATTERY

Please note if there is a hinge option fitted, the user must stop on the intermediate charge station where they can dismount, as it does not have a charging station at ground level.

## SAFE OPERATION

**Do not allow children to play with the stairlift. Do not exceed the safe weight limit (see label under seat). Always use the seatbelt.**

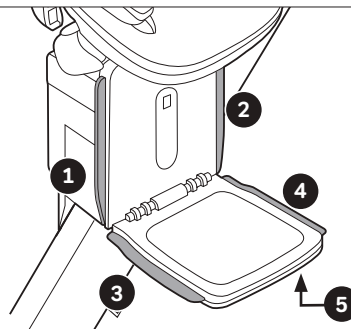
Before operating your stairlift, ensure that there are no obstructions on the staircase. The stairlift is fitted with five safety edges.

**1 & 2:** Each side of the carriage.

**3 & 4:** Each side of the footrest.

**5:** Under the footrest.

If these encounter any obstruction, the stairlift will stop immediately. Move the stairlift back in the opposite direction and remove the obstruction.

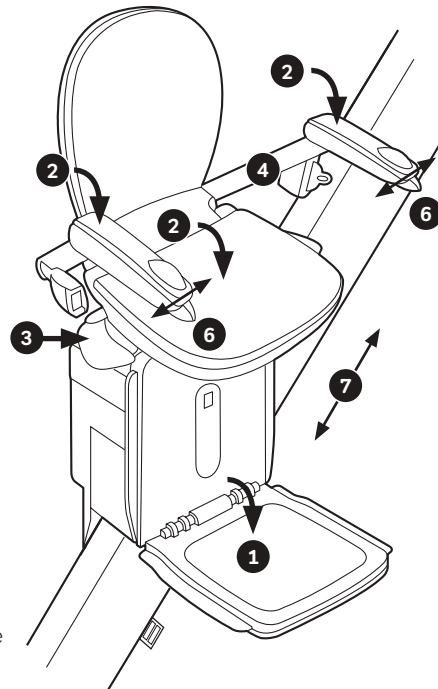
# STAIRLIFT OPERATION

## USING YOUR STAIRLIFT

ENGLISH

- 1 Lower the footrest.
- 2 Lower the seat and arms.
- 3 Hold down one of the swivel paddles and swivel the seat until it locks into its operating position as shown. (At the top of the stairs only).
- 4 Sit in the seat and fasten the seatbelt.
- 5 The arms are fitted with safety interlocks and must be completely down in position for the stairlift to work.
- 6 Press the direction controller in the direction of travel.
- 7 After a short delay, the stairlift will begin to move, slowly at first, until it reaches full speed. Keep the controller pressed down continually throughout the journey. The stairlift will slow down around certain bends for your comfort and safety.
- 8 The stairlift will automatically slow down and stop when it reaches the end of its travel.
- 9 To swivel the seat, hold down one of the seat paddles and rotate the seat. Release the paddle and keep rotating until it locks into position, partly or fully around. (At the top of the stairs only).
- 10 It is now safe to undo the seatbelt and leave the stairlift.
- 11 You may at this point swivel the seat back and fold the footrest, arms and seat up.

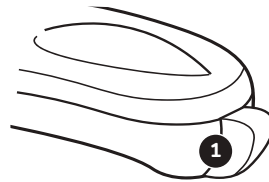
The Acorn 180 Curved Stairlift is a modular system, designed for easy assembly and maintenance utilising the rack and pinion method of travel along the rail, which is still the most reliable system available. This method is used with customer safety paramount throughout design and manufacture. You may experience some small bumps or jolts during use of the stairlift; this is normal for this type of technology

# STAIRLIFT OPERATION

## POWERED SWIVEL SEAT

If a powered seat is fitted, (at the top of the stairs only), continue to hold down the directional controller (1). After the stairlift stops moving, the seat will begin to swivel round automatically.

It is now safe to undo the seatbelt and leave the stairlift.

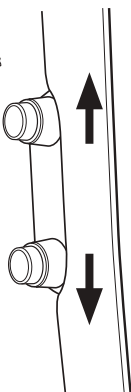
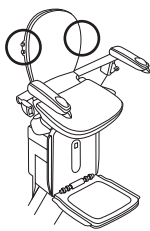
## POWERED HINGE

If a Powered hinge is fitted, the Stairlift will operate the hinge automatically. When the lift reaches or leaves the programmed hinge zone, the Stairlift will stop and the powered hinge will be automatically raised or lowered. This will allow the user to operate the hinge whilst in a line of sight.

At the bottom of the stairs, once you have finished using the stairlift, pressing the up button on the remote handset will automatically run the lift to either an intermediate charge point if fitted or the top of charge point.

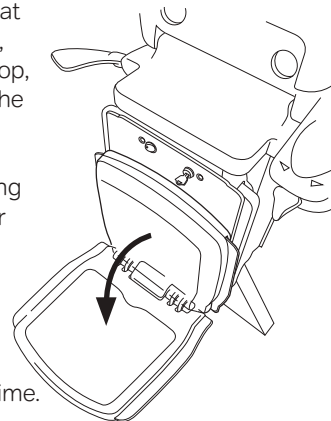
## POWERED FOOTREST

If a Powered Footrest is fitted, at the top or bottom of the stairs, after the stairlift comes to a stop, undo the seat belt and leave the stairlift.

Using the buttons located along both sides of the seat, lower or raise the footrest as required.

Alternatively, the Powered Footrest can be raised by pressing both buttons of the remote handset at the same time.

# STAIRLIFT OPERATION

## REMOTE CONTROLS

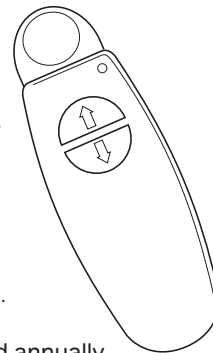
ENGLISH

The stairlift must be in its correct operating condition, with the seat facing forward and switched on before using the remote controls.

The direction controls can be used to call the stairlift. Press the up or down buttons continuously until the stairlift stops at the end of its travel. Do not leave the stairlift parked part way up the stairs unless an intermediate charge point is fitted then it can be sent to this point for re-charging.

Press the up button to send the stairlift to its autopark position.

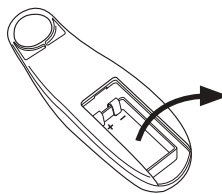
It is recommended that remote control batteries are replaced annually.

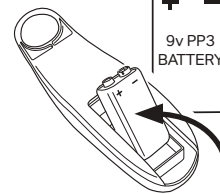
### HOW TO CHANGE THE REMOTE BATTERY

Slide the battery cover off the reverse of the remote control.

Remove the battery. Please take old batteries to a recycling centre or a collection point at your local electrical store or supermarket.

Insert the new battery base first and push down firmly to make. Slide the cover on.

Please ensure that the battery is connected correctly. If the battery is connected wrong it could damage the remote and may require replacing.

# OPERATIONAL PROBLEMS

ENGLISH

## DIAGNOSTIC DISPLAY

When the stairlift is switched on, the diagnostic display underneath the seat is illuminated. During normal operation it will display these symbols:

The stairlift is switched off or in sleep mode, <i>see page 16</i> .	
The stairlift is ready to use but the batteries are not being charged - Check mains electricity supply is switched on and the stairlift is parked on its charge points.	
Moving Up (arm switch control)	
Moving Down (arm switch control)	
Moving Up (Remote control)	
Moving Down (Remote control)	
Lift waiting for hinge to operate	
Batteries are charging	
Batteries are fully charged	
Down side foot sensitive safety edge has been activated - Check for obstructions.	
Up side foot sensitive safety edge has been activated - Check for obstructions.	
Under foot plate sensitive safety edge has been activated - Check for obstructions.	
Down side sensitive safety edge has been activated - Check for obstructions.	
Up side sensitive safety edge has been activated - Check for obstructions.	
The seat is swivelled away from its operating position.	
Battery low – Bring the unit down to the nearest charging station. The unit won't allow the user to travel in the UP direction until the batteries are sufficiently charged and the code has cleared.	
Battery flat – Audible alarm – Press the arm switch control or the remote control once to stop the alarm. Bring the unit down to the nearest charging station.	
<b>If this symbol is displayed, DO NOT use the hand winder in the down direction as this could damage the stairlift. Contact your stairlift supplier immediately.</b>	

# OPERATIONAL PROBLEMS

## LIFT WILL STILL NOT RUN

In the unlikely event that a problem should occur, contact your authorised dealer for assistance.

DEALER STAMP

ENGLISH

## HAND WINDING

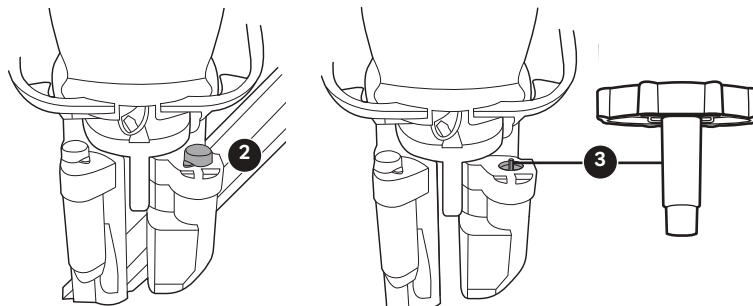
In an emergency, the stairlift may be hand wound to clear any obstruction.

If this symbol is displayed, DO NOT hand-wind the stairlift in the down direction because this could damage it.

- 1 Ensure the stairlift is switched off with the on/off holding switch.
- 2 Remove the round cover on top of the carriage (marked with an Emergency Handwind label).
- 3 Fit the hand winder onto the hexagonal spindle and follow the instructions on the label inside the round cover.

## CARE AND MAINTENANCE

### CLEANING

Wipe the seat, carriage and rail down with a dry cloth.

### REMOTE CONTROLS

Replace the remote control batteries annually. *See Page 13.*

### SERVICING

It is recommended that your Acorn 180 Stairlift be serviced every 12 months by your stairlift supplier to ensure it remains safe and reliable.

The equivalent continuous A-Weighted sound pressure level of this equipment does not exceed 70db (A).

For safety reasons, only Acorn specified parts should be used. Use of parts that are not Acorn approved could affect the safety of your stairlift.

### FCC

The equipment has been tested and complies fully with Class A of 47CFR 2011 part 15 certification. If not installed correctly by an Acorn Qualified Technician there may be some interference with other household equipment (i.e TV and Radio reception). This can be rectified by moving this equipment to a different location.

### SLEEP MODE

If the stairlift is not parked on a charge point, or in the event of a power failure, the stairlift will 'beep' continuously for approximately 60 seconds, after which the stairlift will enter SLEEP MODE. The Stairlift will beep intermittently every 20 seconds and the fault indicator will go blank. The user can 'wake' the stairlift at any time by using the paddle controls or remote control handset. Once woken, park the stairlift on a charge point as normal.

**Acorn promotes responsible recycling.**  
Do not attempt to dispose of the stairlift yourself

## SERVICE RECORD

Service Date: \_\_\_\_\_

Next Service Due: \_\_\_\_\_

Engineer Signature: \_\_\_\_\_

Customer Signature: \_\_\_\_\_

Service Date: \_\_\_\_\_

Next Service Due: \_\_\_\_\_

Engineer Signature: \_\_\_\_\_

Customer Signature: \_\_\_\_\_

Service Date: \_\_\_\_\_

Next Service Due: \_\_\_\_\_

Engineer Signature: \_\_\_\_\_

Customer Signature: \_\_\_\_\_

Service Date: \_\_\_\_\_

Next Service Due: \_\_\_\_\_

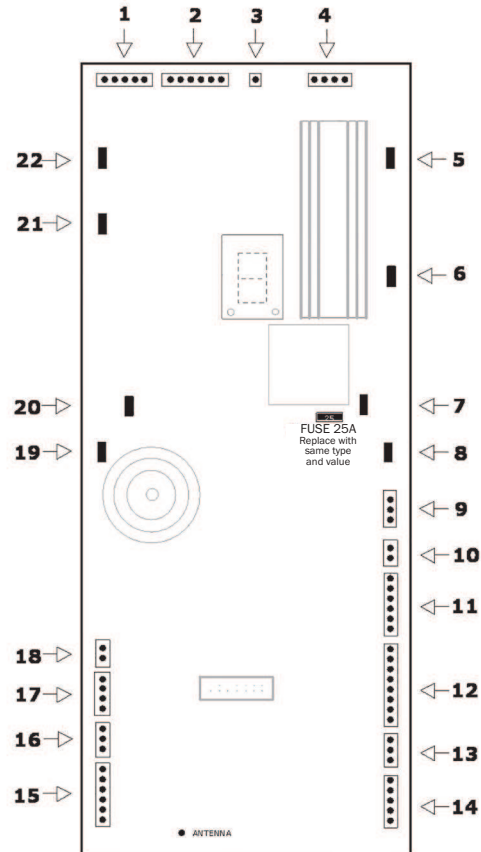
Engineer Signature: \_\_\_\_\_

Customer Signature: \_\_\_\_\_

ENGLISH

## PCB CONNECTIONS

ENGLISH

- |  |  |
|--|--|
| <b>1</b> Level Motor Encoder                     | <b>12</b> Options                                  |
| <b>2</b> Drive Motor Encoder                     | <b>13</b> Seat                                     |
| <b>3</b> Ground                                  | <b>14</b> Overspeed                                |
| <b>4</b> Main Motor Brake                        | <b>15</b> Footrest                                 |
| <b>5</b> Drive Motor Positive                    | <b>16</b> Final Limit                              |
| <b>6</b> Drive Motor Negative                    | <b>17</b> Overtravel (Levelling)                   |
| <b>7</b> Battery Positive                        | <b>18</b> Right-hand PSE (Pressure Sensitive Edge) |
| <b>8</b> Battery Negative                        | <b>19</b> Battery Positive                         |
| <b>9</b> Left-hand PSE (Pressure Sensitive Edge) | <b>20</b> Battery Negative                         |
| <b>10</b> Charger                                | <b>21</b> Level Motor Negative                     |
| <b>11</b> Arm                                    | <b>22</b> Level Motor Positive                     |

ENGLISH

### EC Declaration of Conformity

Acorn Mobility Services Ltd.  
Telecom House  
Millennium Business Park  
Station Road, Steeton  
West Yorkshire, England  
BD20 6RB

**Manufacturer:** Lothian Electric Machines Ltd.  
Hospital Road  
Haddington  
East Lothian  
Scotland  
EH41 3PD

T: +44 (0) 1535 290 000  
F: +44 (0) 1535 290 006  
www.acornstairlifts.com

T: +44 (0) 1620 828 700  
F: +44 (0) 1620 828 730  
www.lemac.com

**Product:** Stairlift  
**Model:** Acorn 180  
Curved Stairlift

**Serial No:**

The undersigned hereby declares, on behalf of Acorn Mobility Services that the above referenced product to which this declaration relates, is in conformity with the essential requirements of:

Machinery Directive 2006/42/EC; Electromagnetic Compatibility Directive 2004/108/EC; Low Voltage Directive 2006/95/EC

And in accordance with the following standards:

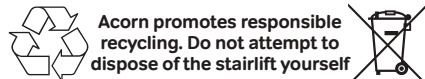
FCC:	47CFR2011 Part 15
EMC:	EN55014-1:2006; EN55014-2:1997; EN61000-3-2:2006; EN61000-3-3:2008; EN61000-4-2:1995; EN61000-4-3:2006; EN61000-4-4:2004; EN61000-4-5:2006; EN61000-4-6:2009; EN61000-4-11:2004
Machinery Directive:	ISO EN 12100:2010; EN 953; EN 349; EN81-40
LVD:	EN 61558-2-6:1997; EN 61558-1:1997+A1:1998; EN 60204-1:2006
Other Standards:	CAN/CSA B613:2002; CAN/CSA B355:2003; CSA/B44-1-04/ASME A17.5:2005; BS EN ISO 9386-2:2000 ASME A 18.1:2005; RWTUV; ISTA

The Technical Construction File, as required by Directive 2006/42/EC is maintained at the corporate headquarters of Acorn Mobility Services Ltd, Telecom House, Millennium Business Park, Station Road, Steeton, England. BD20 6RB.

Authorised Signature:

Date: **May 2015**

**William Waddell**  
**GROUP COMPLIANCE MANAGER**  
Acorn Mobility Services Ltd.

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ITALIANO

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Acorn Mobility Services Ltd.  
Telecom House  
Millennium Business Park  
Station Road, Steeton  
West Yorkshire, Inghilterra  
BD20 6RB

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# ACORN STAIRLIFTS

[www.acornstairlifts.com](http://www.acornstairlifts.com)

Acorn reserves the right to change specification without prior notice. All measurements are approximate. Images are shown for illustrative purposes only. Product may vary slightly.

All information correct at time of going to print. E&OE.

**COPYRIGHT ©2015 ACORN MOBILITY SERVICES LIMITED** All Rights Reserved

65865/ACUSM/WW0515