

American Medical Alert Corp

**Model 203
Universal Transmitter**

Installation Instructions

Part# MM203

Overview:

- The Model 203 universal transmitter operates at 433 MHz powered by an alkaline 9volt battery.
- The Model 203 operates in two different configurations, one with a touch pad and one with the hose connected to the transmitter.
- The part# for the sip & puff configuration is 203-01, the part# for the touch pad configuration is 203-02.
- Once the unit is activated, the red LED will light for three seconds, confirming the transmitter has been activated.
-

Sip & Puff Setup:

- Use hose supplied with the activator and connect to the stem at the end of the transmitter.
- Sip or puff into the hose to ensure the transmitter activates (the red LED should light for three seconds).
- Put the PERS (Personal Emergency Response System) into autolearn mode and activate the transmitter. If unsure how to use the autolearn mode, refer to the user guide for the PERS Model being used.
- Press the reset on the PERS unit to return to normal operation and activate the transmitter, the PERS unit should call in and report as pendant (code 01).

Touch Pad Setup:

- Use the touch pad activator supplied with the transmitter, connect the 3 mm plug to the jack on the end of the transmitter.
- Press on the target area on the touch pad to ensure the transmitter activates (the red LED should light for three seconds).
- Put the PERS (Personal Emergency Response System) into autolearn mode and activate the transmitter. If unsure how to use the autolearn mode, refer to the user guide for the PERS Model being used.
- Press the reset on the PERS unit to return to normal operation and activate the transmitter, the PERS unit should call in and report as pendant (code 01).

FCC Information:

FCC ID: HNB203A

Model: 203

This device complies with Part 15 of the FCC rules. Operation is subject to the two following conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

Product Information:

Model: 203

Operational Frequency: 433 MHz

Power: 9v battery

Company:

American Medical Alert Corporation
36-36 33rd Street, Suite B
Long Island City, NY 11106

Phone: 1-800-645-3244