

Angelcare® FOR YOUR PEACE OF MIND

OWNER'S MANUAL

DIGITAL VIDEO, MOVEMENT & SOUND BABY MONITOR

Model AC1100



IMPORTANT ADVICE FOR INITIAL SETUP

DON'T ORGET



Movement Alarm 🗹

Your monitor comes with the movement alarm activated. When switching on your monitor it will alarm, with or without the Sensor Pad plugged in, if no movement is detected. Deactivate the movement function to use the monitor without the Sensor Pad.



Batteries 🗹

Insert the enclosed rechargeable batteries into the Parent Unit. If inserted incorrectly the Parent Unit will not charge when placed in the charging cradle. The battery icon will flash briefly on the Parent Unit screen to indicate charging.




Cord Safety 🗹

Ensure the Nursery Unit is at least 1 metre away from your baby's cot. The Sensor Pad cord must always be kept out of reach of baby and secured using the cord cover kit provided.

PLEASE FOLLOW THE INSTRUCTIONS CAREFULLY FOR CORRECT SETUP

Please DO NOT return this product to the store. For assistance please contact our helpline on 0345 009 1789 or email helpline@babyhelpline.co.uk Mon - Fri 10am - 4pm, excluding bank holidays.



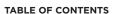

A	
WARNINGS	3
AC1100 SETUP REQUIREMENTS	
STEP 1 - Charging the Parent Unit	9
STEP 2 - Installing the Sensor Pad	10
STEP 3 - Setting up the Nursery Unit	13
STEP 4 - Turning on your Monitor	14
STEP 5 - Testing and getting to know your Monitor	15
STEP 6 - Activating and deactivating additional functions	19
Troubleshooting	25
_imited warranty	27
Technical specifications	28

OWNER'S MANUAL

KEEP YOUR OWNER'S MANUAL FOR FUTURE REFERENCE.

PLEASE READ THE WARNINGS AND INSTRUCTIONS CAREFULLY BEFORE ASSEMBLING & USING THE PRODUCT.

FAILURE TO FOLLOW THE WARNINGS AND THE INSTRUCTIONS COULD RESULT IN SERIOUS INJURY OR DEATH OR DAMAGES TO THE PRODUCT.

THE SENSOR PAD FEATURE IS NOT FOR USE WITH ADULTS, CHILDREN OR INFANTS OLDER THAN 24 MONTHS.

THE SENSOR PAD IS NOT INTENDED TO BE USED TO DETERMINE IF A CHILD IS OUT OF THE CRIB.

A WARNING

Adult assembly required. Keep small parts away from children when assembling.



This Angelcare Monitor is a Personal Care Product. Please DO NOT return this to the store where you purchased it from. If you have any questions or require assistance please call the Angelcare Helpline on 0345 009 1789 or email helpline@babyhelpline.co.uk.

The Baby Helpline is open Monday to Friday, 10am to 4pm. The helpline is closed on bank holidays.

For additional FAQs and to download instruction manuals visit; www.angelcare.co.uk. To order spare parts for your Angelcare Baby Monitor visit; www.megababy.co.uk.









▲ WARNINGS

- 1. THIS PRODUCT CANNOT REPLACE RESPONSIBLE ADULT SUPERVISION. This product is designed to aid in the monitoring of your child. When using the monitor, you must still provide the supervision necessary for the continued safety of your child. Monitoring of premature babies, or those considered to be at risk, should always be carried out under the supervision of a health professional. This monitor SHOULD NOT be used as a medical device, nor as a device to prevent Sudden Infant Death Syndrome (SIDS).
- **2.** WARNING Strangulation Hazard Children have STRANGLED in cords. Keep



adapter cords out of the reach of children (more than 1 m away). NEVER use Sensor Pad(s) without the protective cord covering. Check protective cord covering before each

use. Immediately discontinue use of Sensor Pad(s) if protective covering is damaged or broken.

- **3.** Never use extension cords with AC adapters. ONLY use the AC adapters provided.
- **4.** When an AC Adapter is plugged into an electrical outlet, DO NOT touch the exposed end.
- **5.** ALWAYS be sure that both the transmitter and receiver are working properly and are within range of each other. Test monitor before first use, periodically and when changing location of the Nursery Unit.
- **6.** ALWAYS be sure to immediately check on baby whenever an alarm sounds.
- **7.** DO NOT use monitor near water (such as bath tub, sink, etc.).



- **8.** Keep monitor AWAY from heat sources (such as stoves, radiators, etc.).
- **9.** ALWAYS place both the Nursery Unit and Parent Unit on a flat surface in an upright position and allow for proper ventilation. DO NOT place on sofas, cushions, beds, etc. which may block ventilation.
- 10. This monitor uses public airwaves to transmit signals. The Parent Unit may pick up signals or interference from other nursery monitors in your area. Your monitor may even be picked up by other households. To protect your privacy, make sure both units are turned OFF when not in use. If you experience interference, change the channel on both units.
- **11.** This product is NOT a toy. DO NOT allow children to play with it.
- **12.** This product contains small parts. Adult assembly required. Exercise care when unpacking and assembling the product and keep small parts away from children.
- **13.** When using the monitor, the room temperature should be between 50°F (10°C) and 104°F (40°C).

A BATTERY WARNING

14. Use ONLY AAA RECHARGEABLE batteries for the Parent Unit. DO NOT mix old and new batteries. DO NOT mix alkaline, standard or rechargeable batteries. Insert batteries with correct polarity. DO NOT short-circuit supply terminals. ONLY use the recommended batteries or equivalent (volts and size) and the correct adapter. Remove batteries when product is stored for long periods of time or when batteries are exhausted. Dispose exhausted batteries properly. DO NOT recharge non-rechargeable batteries. Rechargeable batteries are ONLY to be charged under adult supervision. Keep all





batteries away from children.

15. IMPORTANT SAFETY INSTRUCTIONS

- **15.1** Read these instructions All the safety instructions should be read before the product is operated.
- **15.2** Keep these instructions. The safety and operating instructions should be retained for future reference.
- **15.3** Heed all warnings All warnings on the product and in the operating instructions should be adhered to.
- **15.4** Follow all instructions All operating and use instructions should be followed.
- **15.5** Water and moisture DO NOT use this apparatus near water. For example, near a bath tub, wash bowl, kitchen sink, laundry tub, in a wet environment or near a swimming pool and the like. Care should be taken so that objects DO NOT fall onto units and that liquids are not spilled into either unit or Sensor Pad through its openings.
- **15.6** Cleaning Disconnect all units before cleaning. DO NOT immerse in water any part of the Nursery Unit, Parent Unit. Clean only with dry cloth.
- **15.7** Ventilation DO NOT block any ventilation openings. Allow for proper ventilation when units are in use. DO NOT cover the Nursery Unit or Parent Unit with any object such as a blanket. DO NOT place it in a drawer or in any location which would muffle the sound or interfere with the normal flow of air.
- **15.8** Heat sources DO NOT install near any heat sources such as radiators, heat registers, stoves, TVs or other apparatus (including amplifiers) that produce heat. Heat can damage the case or electrical parts.
- **15.9** Power cord protection Protect the Sensor Pad cord and Adapter's power cord from being walked on or pinched by furniture or other items Power supply cords should be routed so that they are not likely to be walked on or pinched by items upon or against them, paying particular attention to cords and plugs, convenience receptacle, and the point where they exit from the product.

- 15.10 Accessories Only use attachments/ accessories specified by the manufacturer. DO NOT place this product on an unstable cart, stand, tripod, bracket or table. The product may fall, causing serious injury to a child or adult and serious damage to the product. Use ONLY with a cart, stand, tripod, bracket or table recommended by the manufacturer or sold with the product. Any mounting of the product should follow the manufacturer's instructions.
- **15.11** Unplug this appliance during lightning storms or when not used for long periods of time.
- **15.12** Servicing DO NOT attempt to service this product yourself as opening or removing covers (except battery doors) may expose you to dangerous voltage or other hazards.
- **15.13** Replacement parts When replacement parts are required, be sure the replacement parts specified by the manufacturer have the same characteristic as the original part. Unauthorized substitution may result in fire, electric shock or other hazards.
- **15.14** Usage This baby monitor is intended as an aid. It is NOT a substitute for responsible and proper adult supervision and should not be used as such. Use this appliance ONLY for its intended use as described in this manual.
- **15.15** Power Sources Operate this product only from the type of power source indicated on the type plate of the adaptor. If you are unsure of the type of power supply to your home, consult your dealer or local power company.
- **15.16** Overloading DO NOT overload wall outlets, extension cords or integral convenience receptacles as this can result in a risk of fire or electric shock.
- **15.17** Object and Liquid Entry NEVER push objects of any kind into this product through casing slots as they may touch dangerous voltage points or short out parts that could result in a risk of fire or electric shock. NEVER spill liquid of any kind on the product.
- **15.18** Antenna The antenna used for this transmitter must be installed to provide



a separation distance of at least 20 cm (7.9") from all persons and must not be collocated or operating in conjunction with any other antenna or transmitter.

15.19 Power Lines - An outside antenna system should NOT be located in the vicinity of overhead power lines or other electric light or power circuit, or where it can fall into such power lines or circuits as contact with them may be fatal.

15.20 WARNING Strangulation Hazard — Children have STRANGLED in cords. Keep



adapter cords out of the reach of children (more than 1 m away). NEVER use Sensor Pad(s) without the protective cord covering. Check protective cord covering before each

use. Immediately discontinue use of Sensor Pad(s) if protective covering is damaged or broken.

15.21 Interference - This monitor uses public airwaves to transmit signals. The Parent Unit may pick up signals or interference from other nursery monitors in your area. Your monitor may even be picked up by other households. To protect your privacy, make sure both units are turned OFF when not in use. If you experience interference, change the channel on both units.

15.22 Vibration Source - When the monitor is in use, DO NOT use a crib mobile or other crib accessories that may produce vibrations. Other sources of vibrations may include fans, washing machines, loud music, etc. These sources of vibration may interfere with the performance of your Angelcare® monitor and the alarm will not sound if it continues to detect any movement other than your baby's. Be sure all vibration sources are eliminated before using the monitor.

You may experience false alarms for several reasons. The most common reason is the Nursery Unit has not been turned OFF after you have removed your baby from the crib. Another reason is the Sensor Pad has not been placed under the mattress properly.

Be sure the Sensor Pad is on a flat, firm surface with the printed side facing up. The sensitivity level may also need to be adjusted. Refer to step 5.1.

15.23 Sensor Pad - Your Sensor Pad can be used in any crib that is stationary. If the crib has a spring or slat base, place a hard board between the crib base and the Sensor Pad measuring at least 13" x 13" (33 x 33 cm) and 1/4 inch (6 mm) thick. Only use your Angelcare® monitor in a stationary sleeping environment such as a crib where the Sensor Pad can rest on a completely flat, firm surface.

15.24 The Sensor Pad is made of plastic and does not permit the air to circulate. Therefore, mildew build up could develop underneath the mattress in the sensor pad area. We suggest rotating the mattress regularly to prevent any mildew.





16. IMPORTANT SAFETY INSTRUCTIONS RELATED TO USE OF BATTERIES

- **16.1** Use ONLY the type and size of batteries specified in the owner's manual. There is a risk of explosion if batteries are replaced with an incorrect type.
- **16.2** Use ONLY AAA rechargeable batteries and/or the AC Adapters provided for the Parent Unit. You must use only rechargeable batteries in the Parent Unit. DO NOT use alkaline batteries or you will severely damage the unit.
- **16.3** Charge the batteries for 15 hours prior to first use. Be sure to have the monitor in the OFF position during this period. ONLY use the AC Adapters included with your monitor.
- **16.4** DO NOT use these adapters with any other equipments.
- **16.5** When unplugging the AC Adapter from the Nursery Unit, be sure it is switched to the OFF position. If it isn't, the unit will revert to battery mode if batteries are installed.
- **16.6** DO NOT use rechargeable batteries in the Nursery Unit. Use alkaline batteries ONLY.
- **16.7** DO NOT dispose batteries in fire. They may explode. Check with local regulations for possible special disposal instructions.
- 16.8 Exercise care in handling batteries in order not to short out the batteries with conducting materials such as rings, bracelets and keys. Overcharging, short circuiting, reverse charging, mutilation, or incineration of batteries must be avoided to prevent one or more of the following occurrences: release of toxic materials, release of hydrogen and/or oxygen gas, rise in surface temperature.

- **16.9** DO NOT attempt to recharge batteries provided with or identified for use with this product that are not intended to be charged. The batteries may leak corrosive electrolyte or explode.
- **16.10** DO NOT attempt to rejuvenate the batteries provided with or identified for use with this product by heating them. Sudden release of battery electrolyte may occur, causing burns or irritation to eyes or skin.
- **16.11** If and when necessary, the batteries must be replaced with identical new ones or equivalent (volts and size). If a battery to be replaced is connected with other batteries in series, replace the other batteries at the same time as well. DO NOT mix old and new batteries.
- **16.12** Reverse polarity installation of the batteries in the end product must be avoided. Reverse insertion of batteries can cause charging, and that may result in leakage or explosion.
- **16.13** Remove batteries from this product if you store the product over 30 days because the batteries could leak and damage the product.
- **16.14** Discard 'dead' batteries as soon as possible since 'dead' batteries are more likely to leak in a product. If a battery has leaked or vented, it should be replaced immediately using protective gloves.
- **16.15** DO NOT store this product, or the batteries provided with or identified for use with this product, in high-temperature areas.







A FC WARNING

(North America Only)

17.1 Modifications not authorized by the manufacturer may void users authority to operate this device.

17.2 NOTE: This equipment has been tested and found to comply with part 15 of the FCC Rules and the Industry Canada licence-exempt RSS standard(s). These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

17.3 However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment OFF and ON, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

17.4 This device complies with the part 15 of the FCC Rules and the Industry Canada licence - exempt RSS standards(s). Operation is subject to the following two conditions:

(1) This device may not cause harmful interference, and

(2) This device must accept any interference received, including interference that may cause undesired operation.





FCC AND IC REGULATIONS

RF radiation exposure statement

The Nursery Unit complies with FCC RF radiation exposure limits set forth for an uncontrolled environment. **Caution**: To maintain the compliance with the FCC's RF exposure guideline, install the Nursery Unit at least 20 cm from ALL persons. The transmitter of this device must not be co-located or operating in conjunction with any other antenna or transmitter.

Industry Canada

This product meets the applicable Industry Canada technical specifications. CAN ICES-3 (B)/NMB-3 (B)

This device was tested for typical body-worn operations using the supplied belt clip. To maintain compliance with RF exposure requirements, body-worn operations are restricted to the supplied belt clip.



SETUP REQUIREMENTS

VIDEO, MOVEMENT & SOUND MONITOR

Model AC1100

INCLUDED:



Rechargeable Parent Unit



Nursery Unit



Charging Cradle



Belt Clip



1 Sensor Pad (AC1100) or 2 Sensor Pads (AC1100-2SP) with pre-installed cord covers



2 AC Adapters



3 Rechargeable AAA Batteries

YOU MUST PREPARE:

One hardboard (not included) Thickness: 6 mm

Minimum dimensions:

33 cm x 33 cm

Maximum dimensions: equal to the mattress size



WHAT YOU WILL NEED (NOT INCLUDED):



3 AAA Alkaline Batteries



Miniature Philips Screwdriver

IMPORTANT


The hardboard must be larger than the Sensor Pad surface area.

BABY'S ROOM:

IMPORTANT

Place the crib near a supporting wall and far from air currents. Install the Nursery Unit on a flat, stable surface facing the crib at a distance of at least 1 meter.

Make sure to plug the adapter to the socket outlet nearest to the equipment and that it is easily accessible.

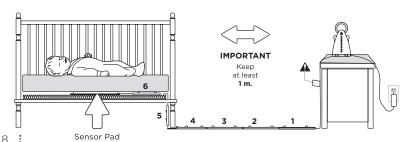









STEP 1 - CHARGING THE PARENT UNIT



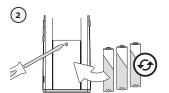

IMPORTANT

Never use regular alkaline batteries in the Parent Unit; they may explode.

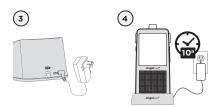


- 1) Remove the Belt Clip.
- 2) Remove the battery door and insert the Rechargeable Batteries (included).
- Connect the AC Adapter to the Charging Cradle and into an electrical outlet.
- **4)** Charge the unit for **10 continuous hours** (prior to first use).


NOTE:

To optimize your battery performance please refer to Battery Guidelines in the Trouble Shooting section.

PARENT UNIT:

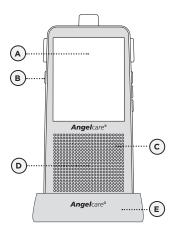



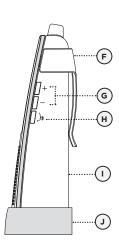

Left Side


- A) LCD Touchscreen
- B) POWER ON/OFF



- C) Microphone
- D) Speaker
- E) Charging Cradle
- F) Belt Clip
- **G)** Speaker Volume Buttons
- H) Talk Back Button
- I) Battery Door
- J) AC Adapter Outlet (not shown)



STEP 2 - INSTALLING THE SENSOR PAD

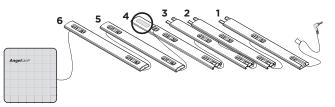

Sensor Pad with pre-installed cord covers

Cord covers 1, 2, 3, 4: Position on floor

Cord cover 4:

Contains double sided tape to securely install beneath crib leg

Cord cover 5:

Position vertically between mattress support and floor

Cord cover 6:

Place under the mattress

WARNING


STRANGULATION HAZARD.

Children have STRANGLED in cords:

- Keep adapter cords out of the reach of children (more than 1 m away).
- Because of the serious strangulation risk to infants and toddlers, parents and caregivers should never place the Nursery Unit within 1 meter of a crib.
- NEVER use Sensor Pad without the cord covers. Check cord covers before each use to ensure it is securely placed under the leg of the crib. Discontinue use of Sensor Pad if any cord covers are damaged or broken
- NEVER use extension cords with AC adapters. ONLY use the AC adapters provided. This product is not a toy. Never allow children to play with it.

2.1 - SELECT A LOCATION FOR THE NURSERY UNIT

Select a location for the Nursery Unit that is at least 1 meter away from the crib.

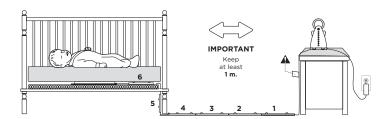







•

2.2 - SELECT THE CRIB LEG YOU WILL BE USING TO SECURE THE CORD COVERS

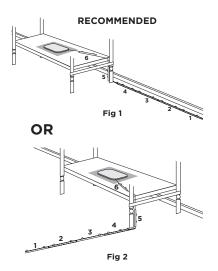

1) Use the crib leg by the wall closest to the Nursery Unit (fig 1). In cases where the Nursery Unit is across the room, use the leg closest to the Nursery Unit to position the covered wire (fig 2). It is always preferable that the covered cord run along a wall.


2.3 - INSTALL THE SENSOR PAD ON THE MATTRESS SUPPORT

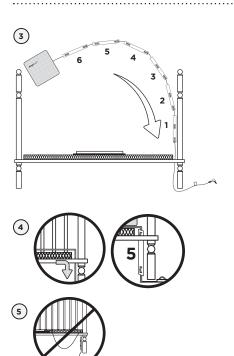



- 1) Remove the mattress from the crib.
- Place the Sensor Pad printed side up - on top of the hardboard at the center of the crib.
- 3) Run the connection pin, warning label and the first 5 cord covers down along the backside of the crib leg, between the mattress support and the inside edge of the crib.
- 4) Bring the first 4 cord covers down the floor, and keep the 5th cord cover vertically, along the backside of the crib leg, (between the mattress and the inside edge of the crib).
- **5)** Remove any cord slack to make sure no parts of the cord are loose.





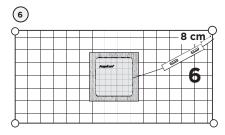

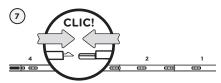

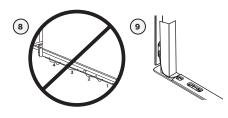



- **6)** Place the 6th cord cover approximately 8 cm from the edge of the mattress support.
- Snap together the remaining cord covers that will run along the floor.
- **8)** Remove any cord slack to make sure no parts of the cord are loose.
- 9) Peel off the double-sided tape on the 4th cord cover and place it securely under the crib leg.
- 10) Install the Nursery Unit on a flat, stable surface facing the crib at a distance of at least 1 m and plug the Sensor Pad cord into the Nursery Unit.
- 11) Check to make sure the Sensor Pad has not moved from position.
- 12) Place the mattress back inside the crib.

The Sensor Pad functions with any size and type of mattress except memory-foam mattresses and hollow frame mattresses.

The hardboard maintains a constant pressure on the Sensor Pad, enabling it to detect the baby's movements even when the baby is at the edges of the crib.





- When lowering the mattress support as your child grows, slide the vertical Cord Cover (5) upward so it remains approximately in the center between the mattress support and the floor. Slide the Sensor Pad Cord through the Floor Cover to eliminate any additional slack.
- The Sensor Pad is not designed for use with adults, children or infants older than 24 months.
- Discontinue used of the Sensor Pad when child is older than 24 months. Remove sensor pad from crib and store for future use.
- BEFORE FIRST TIME USE: To test that the unit is ready for use, refer to instruction manual on how to test that the installation has been done











STEP 3 - SETTING UP THE NURSERY UNIT

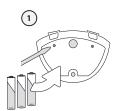

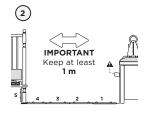




mmmm 4

- 1) Remove the battery door and insert 3 AAA Alkaline Batteries (not included).
- 2) Place the Nursery Unit at a minimum of 1 m away from the crib.
- 3) Connect the AC Adapter to the Nursery Unit and into an electrical outlet.
- 4) Direct the angle of the camera toward the crib using the rotating dial.


NOTE:

The batteries maintain power supply in case of power outage.

NURSERY UNIT:

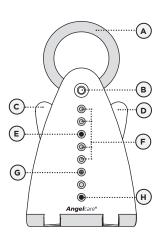



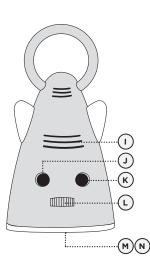

Back Side

- A) Night Light
- B) Main Button
- C) POWER ON/OFF



- D) Night Light ON/OFF (Right 'Wing')
- E) Video Camera
- F) 4 Infrared LEDs (Night Vision Camera)
- G) Power Indicator (green) Low Battery Indicator (red)
- H) Microphone
- I) Speaker
- J) Sensor Pad Outlet
- K) AC Adapter Outlet
- L) Roating Dial for Camera Angle
- M) Battery Door (not shown)
- N) Tripod Outlet* (not shown)



^{*} Nursery Unit may be installed on a tripod (not included).

•

STEP 4 - TURNING ON YOUR MONITOR









- Press the Parent Unit ON/OFF button for 2 seconds until blue screen appears.
- 2) The Parent Unit will display Video mode screen.
- **3)** Press the **Menu** icon to access main menu.
- The Parent Unit will display Menu mode.
- 5) Push and hold the Nursery Unit left 'wing' button for 2 seconds.



When Parent Unit is turned ON and Nursery Unit is OFF, Parent Unit will display Disconnected Mode and will turn itself OFF after 60 seconds.



If there is a lost connection between the 2 units (refer to troubleshooting):

- The Out-of-Range icon will appear on the screen.
- The Lost Connection icon will appear on the indicator bar.





Indicates room temperature

Temperature Alarm icon: Indicates
Temperature Alarm is active

Pendulum icon: Indicates the Motion Sensor is active

Sound icon: Indicates that Sound function is active

Alarm icon: Indicates selected alarm mode

Sound alarm

Sound alarm and vibration

Vibration only



Connection icon: Indicates the connection quality between the Parent Unit and Nursery Unit.

(Good connection

(Poor connection

2 Lost connection

Out-of-Range

Battery Level icon: Indicates battery level

Empty

1/3 Full

2/3 Full

Full







STEP 5 - TESTING AND GETTING TO KNOW YOUR MONITOR

The Angelcare® monitor can be used with the pre-program factory settings or you can also customise your settings to your preference.



All changes and adjustments made in the main menu and sub-menus will not be activated unless you confirm them by pressing the **OK** icon.

IMPORTANT

When in menu or sub-menu modes, sound transmission between Parent and Nursery Units will be temporarily deactivated. ONLY movement-detection alarm will remain active (if function is activated).

5.1 - SENSOR PAD

TO ACTIVATE THE SENSOR PAD:





- Press Menu icon to access main menu.
- Press Movement icon to activate or deactivate Movement Alarm.




ON




OFF

Press OK icon to confirm.

TO TEST THE SENSOR PAD:

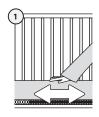

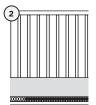

1) Move your hand lightly on the mattress.



The pendulum icon will appear on Parent Unit indicator bar. It will move as you move your hand and trigger a 'TIC' sound.

2) Remove your hand from the mattress. After 15 seconds you will hear a 'TIC' pre-alarm sound, and 5 seconds later the Movement Alarm will trigger.



MOTION ALARM

- · Alarm activates when no movement is detected by Sensor Pad for 20 seconds.
- Both the Parent Unit and Nursery Unit speakers will emit loud warning sound.
- Motion Alarm icon will show on the LCD screen.

IF THE ALARM DOES NOT SOUND


- Avoid contact with crib while Sensor Pad is in operation.
- · Place crib near a supporting wall.
- Turn OFF crib mobile, fans, loud music or anything else that may cause vibration.
- · Adjust Sensor Pad sensitivity level.





TO STOP THE ALARM



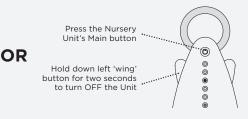

TO ADJUST THE SENSOR PAD SENSITIVITY LEVEL:







NOTE:

If you are receiving false alarms, adjust the sensitivity level.


- Press Menu icon to access main menu.
- 2) Press settings icon to access sub-menu.
- **3)** Press **movement sensitivity** icon to choose the desired sensitivity level (1 to 5).




4) Press OK icon to confirm.

TO ACTIVATE THE TIC FUNCTION:

Allows you to hear a TIC sound when Sensor Pad function is activated.

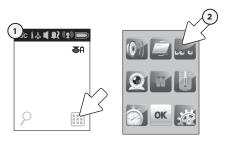

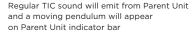

- 1) Press **Menu** icon to access the main menu.
- 2) Press TIC icon to activate or deactivate the tic function.







3) Press OK icon to confirm.

IMPORTANT

- The Sensor Pad is not designed for use with adults, children or infants older than 24 months.
- Once you are finished using the Sensor Pad, please remove it from the baby's crib and store it away.



16



