



# Wireless Resident Monitoring Solutions

## Manual Reset Device (MRD-RF) Operational Instructions

The HomeFree system was designed to monitor special needs and Dementia individuals in the care center environment. The Manual Reset Device (MRD-RF) is one of the many unique elements of this system.

The Wireless Reset Device is a small, lightweight push button unit, carried by the care center staff member, that can be used to temporary deactivate the Wireless Monitoring Door Unit alarm (for door alarm bypass) as well as reset unauthorized exit attempt alerts. **The Manual Reset Device is not for use outside the monitored area.**



## **Deactivating the Wireless Monitoring Door Unit Alert:**

Once the Wireless Monitoring Door Unit alert has been activated, move to the appropriate door / exit that activated the alarm. Escort the resident out of the range of the Wireless Monitoring Door Unit. Standing relatively close to the Wireless Monitoring Door Unit, hold the Manual Reset Device and press the Reset button. The audible alarm is deactivated and Wireless Monitoring Door Unit is reset.

## **Bypassing the Wireless Monitoring Door Alarm:**

Once you are ready to escort the resident through the monitored door / exit, ensure that the resident is out of the range of the Wireless Monitoring Door Unit. Stand relatively close to the Wireless Monitoring Door Unit and holding the Manual Reset Device, press the Reset button. You can now escort the resident through the monitored door / exit. (Bypass interval time is approximately 20 seconds).

# FCC Notice

**The Manual Reset Device Unit MRD-RF complies with Part 15 of the FCC Rules.** Operation is subject to the following conditions:

- 1) This device may not cause harmful interference and
- 2) This device must accept any interference received, including interference that may cause undesired operation.
- 3) Changes or modifications not expressly approved by Elmo Tech Ltd could void the user's authority to operate the equipment.

This equipment has been tested and found to comply with the limits of a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This device generates, uses, and can radiate radio frequency energy and, if installed and used in accordance with the instruction, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this device does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

**CAUTION:** Any changes or modifications to this equipment not expressly approved by Elmo Tech Ltd could void the user's authority to operate the equipment.

**In case of any problems please contact your local service provider:**



HomeFree Systems Inc.  
Phone: 1-800-606-0661  
Outside US call: +927-3-6488690  
E-mail: [contactusa@homefreesys.com](mailto:contactusa@homefreesys.com)  
Web Site: [www.homfreesys.com](http://www.homfreesys.com)