




TOP Page

2) English Manual (BAESLab)

| | | | |
|--|--|---|--|
|  <p>AIDER CARE Introduction</p> <p>We believe that technology enables us to improve human well-being. Because the world never stops changing resulting in a non-stop change in our way of life and society. Nowadays, Thailand has turned towards aging society with more than 10 million of senior citizens which most of them are now facing with health problems; 58 percent of elderly has a tough movement and 41% struggles with hypertension. And since they are likely to stay at home alone all by themselves, causing difficulty for their descendants to take proper care of them as they are rarely together.</p> | | <p>That's why BAESLab Company Limited to develop AIDER CARE. An innovative product to serve a family as a caregiver and assistant to reduce health risks. Lifestyle reporter via wearable device is like a personal doctor that will take care and report habitual routine of your loved ones. It can also detect even a slight change with artificial technology (AI) that learns personal routine and activity of a wearer. Area detection system will alert when spotted out of home radius which can be super helpful, especially, for a case of Alzheimer to protect a wearer from straggling, and will send alert to you at once when a wearer faces emergency to convey help just in time. We don't only design technologies but also develop innovation that enhances the quality of life</p> | <p>And provide more freedom for your loved ones.</p> |
|--|--|---|--|

| | | | |
|---|--|--|--|
| <p>Product Feature</p> <p>CLOUD ASSISTANT 24/7 access to user's backup database system which tracks and collects activity routine information such as walking, running or resting every 5 seconds and will be shown in activity graph.</p> <p>FLEXICARE With user-based design, AIDER CARE is suitable for seniors. Plus, the non complexity process makes it easy to use.</p> | <p>PUSH EMERGENCY Emergency alert button available in one click can send alert by calling or sending message to you to provide help in time.</p> <p>FALLS ALERT This innovation comes with Artificial Intelligence (AI) to learn and track personal activity routine. It knows every movement and will send alert immediately when detects any abnormal activity of its wearer such as an unlikely movement or falling. With this technology, it can be very helpful for a doctor to diagnose more accurately.</p> | <p>AREA DETECTION Protect your loved ones from getting lost via alert system which will send alert when a wearer is spotted out of the residence radius. Suitable for a senior who's home alone or has an Alzheimer disease.</p> <p>BLUETOOTH 4.0 AIDER CARE is connected via Bluetooth version 4.0 which will work even if a wearer doesn't use smartphone.</p> | <p>WATER RESISTANCE Water resistance at a level of IP65 prevents water from seeping into the product leaving a wearer no concern about taking it off from time to time as he or she is doing certain activity such as hand washing or going out on a rainy day.</p> <p>ULTRALOW POWER CONSUMPTION Equipped with Lithium Polymer Battery 100 mAh which allows an extra long life up to 7 days after single 2-3 hour charge.</p> |
|---|--|--|--|

Bottom Page

| | | | |
|--|--|---|--|
| <p>COMPATIBLE WITH</p> <div data-bbox="280 225 562 300">  </div> <div data-bbox="280 320 562 400">  </div> <p>Aider Care Designed by Baeslab Co.,Ltd., Thailand Assembled in PRC</p> <p>Email: info@baeslab.com</p> | <p>How to use</p> <p>1. Download AIDER CARE Application You can download our mobile application to sync with CLOUD ASSISTANT by following these steps:</p> <p>1.1 for iOS, go to Apple Store> Search> AIDER CARE 1.2 for Android and Google, go to Google Play> Search> AIDER CARE</p> <p>2. To charge battery 2.1 Though the product is ready to use, we recommend you charging it about 2-3 hour long prior the usage as to extend the battery life. 2.2 To charge the battery; use USB end to plug in the adapter charger and another end in the back of the product. Notice the red light indicating the product being charged will change to green when fully charged.</p> | <p>Working status</p> <p>The product will constantly flicker red light signaling it's working. If there's no flickered red light, the product is not working and it has ran out of battery and so must be charged. But in case it's not working normally after a proper charge, report to AIDER CARE department by dialing 02-739-8465</p> | <p>4. How to send emergency alert</p> <p>4.1 Automatic alert Normally our product will send a call or an alert to you immediately as any abnormal movement, activity or a fall of a wearer is detected.</p> <p>4.2 Push emergency by wearer To send an alert when facing an emergency or a fall, wearer must press a button in centre of the product by holding it for 3 seconds or longer, an emergency alert will be sent to you at once. If a wearer want to cancel the call, it can be done by pressing the button where the light is indicated. (for 3 seconds or longer.)</p> |
|--|--|---|--|

| | | | |
|--|--|---|---|
| <p>Terms and Conditions</p> <p>1. AIDER CARE must be registered before used.</p> <p>2. AIDER CARE have to be worn all the time when activating.</p> <p>3. AIDER CARE must be connected with a smart phone that is compatible with Bluetooth 4.0 or higher.</p> <p>4. AIDER CARE and a smart phone must be connected within 15 metre radius.</p> <p>5. AIDER CARE must be charged every 7 days to avoid 0 battery power and a product breakdown.</p> <p>6. AIDER CARE provides water and dust resistance at a level of IP65 (Please see more information at : https://en.wikipedia.org/wiki/IP_Code)</p> | <p>7. Smart phone that connects with AIDER CARE must be working and connecting with Bluetooth and Wifi all the time.</p> <p>8. User will need to open the application for AIDER CARE on the smart phone every time after turning on the device.</p> <p>9. AIDER CARE's terms and conditions can be changed without prior notice.</p> <p>Product Warranty</p> <p>AIDER CARE product is warranted for 1 year after the purchase from a distributor. Service provider can change warranty terms and conditions without prior notice.</p> | <p>Product Disclaimer</p> <p>AIDER CARE will not be warranted under the conditions below;</p> <p>1. The product is used more than 1 year after registration.</p> <p>2. The product is damaged or breakdown from not being used in accordance with the user guide.</p> <p>3. The product is physically and intentionally damaged.</p> <p>4. The product is damaged from liquid or humidity over limited level.</p> <p>5. The product is deconstructed.</p> <p>6. The product is modified by an unapproved service provider.</p> | <p>FCC WARNING</p> <p>This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:</p> <p>(1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.</p> <p>NOTE 1: Any changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.</p> |
|--|--|---|---|