
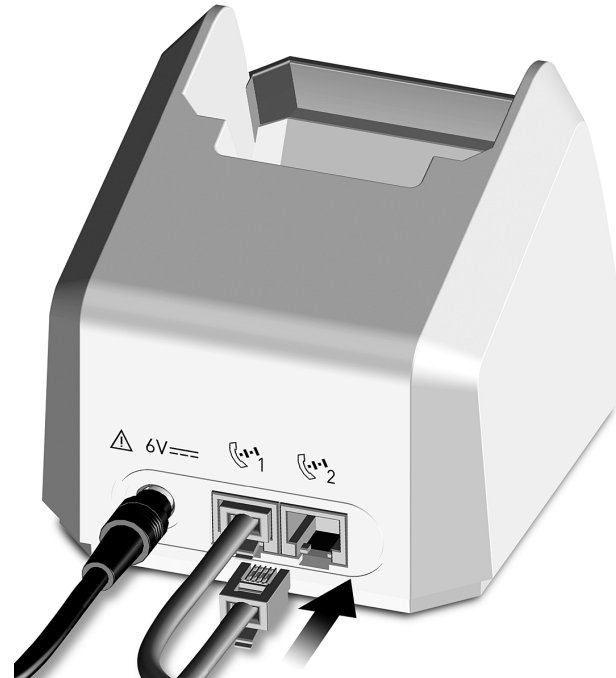
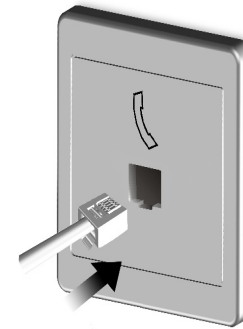
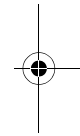
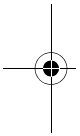
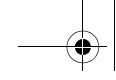
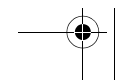
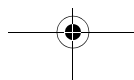


Take the telephone cable that **came with** the CardioMessenger, remove the second safety plug from the charging station and plug one end of the telephone cable into the port. 





Insert the other end of your telephone cable into the telephone socket in your wall.











Now, place the CardioMessenger into the charging station. The CardioMessenger should switch itself on immediately, unless the batteries are completely depleted. A brief tone will sound when the CardioMessenger is switched on.





If the batteries are completely depleted, then the CardioMessenger first charges itself and then switches itself on later (approximately after 30 minutes).

Additional information can be found in „Version B“ on page 19.

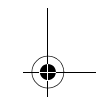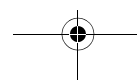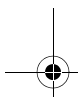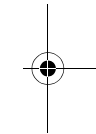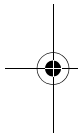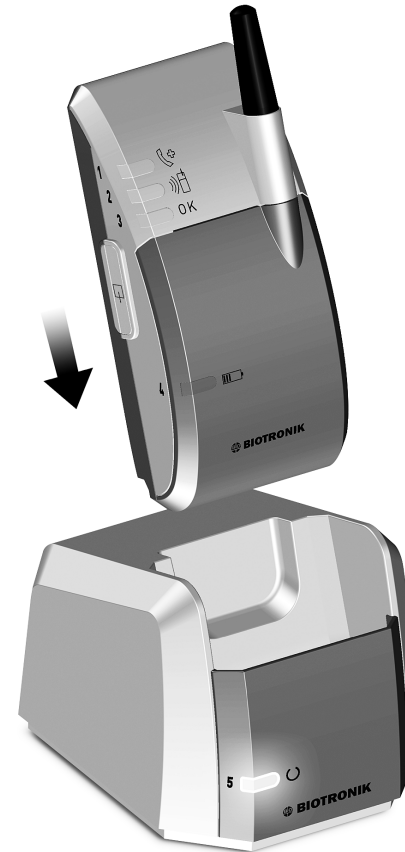




When you are finished, the connection should look like as follows:

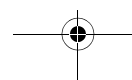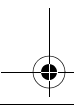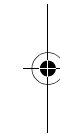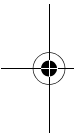

Version B






If you wish to set up the CardioMessenger close to a telephone outlet, but there is no telephone, you may connect the CardioMessenger directly to the telephone outlet. Please proceed as follows:

Take the telephone cable that came with the CardioMessenger and remove the safety plug on port 1 on the rear side of the charging station. 









Now insert one end of your telephone cable into the port of the charging station.

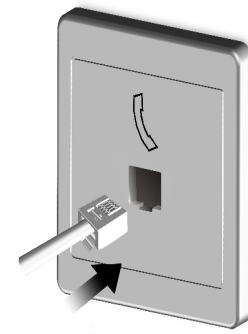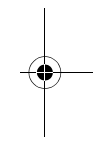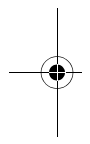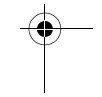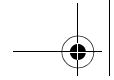

Insert the other end of your telephone cable into the telephone socket in your wall.

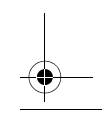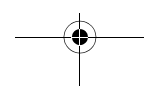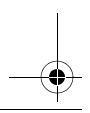

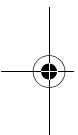

Now, place the CardioMessenger in the charging station. The CardioMessenger should switch itself on immediately, unless the batteries are completely depleted. A brief tone will sound when the CardioMessenger is switched on.

If the batteries are depleted, then the CardioMessenger first charges itself and then switches itself on later (approximately after 30 minutes).

Additional information can be found in „Version B“ on page 18.

