BébéSounds®

# NasalClear™

Battery Operated Nasal Aspirator


Model #BR140

**Instruction Manual**  
**Manual de Instrucciones**

## TABLE OF CONTENTS

General Diagram.....	page 3
Introduction.....	page 4
Key Features.....	page 5
Battery Installation.....	page 5
Instructions for Use.....	page 6
Care and Cleaning.....	page 9
Technical Specifications.....	page 11
Applied Standards and Approvals.....	page 11
Limited Product Warranty.....	page 12
Also Available.....	page 14
<b>En Español.....</b>	<b>página 15</b>

## GENERAL DIAGRAM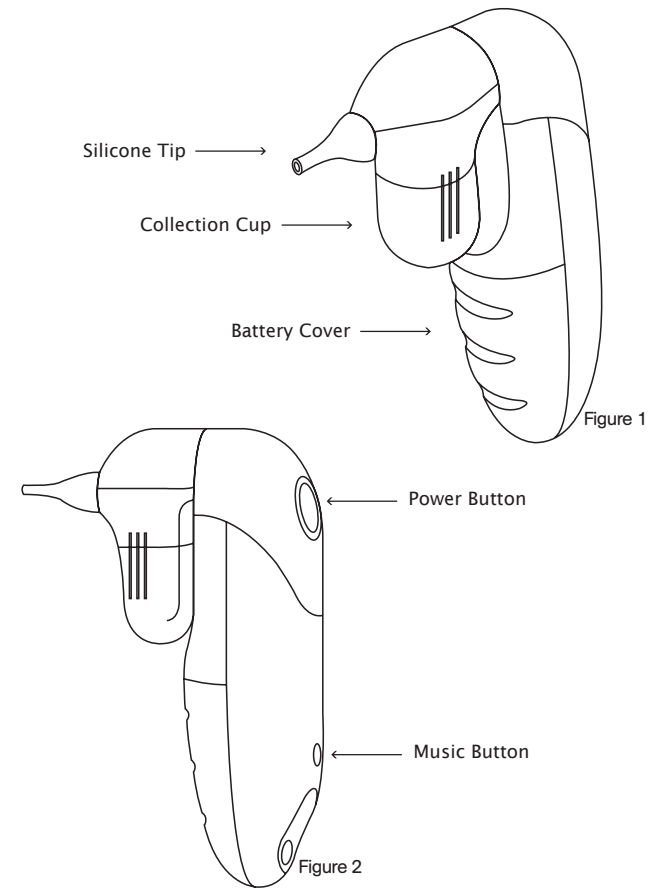

## INTRODUCTION

Thank you for purchasing the Bébésounds® NasalClear™ Nasal Aspirator. With this aspirator you can safely and quickly clear your baby's nasal congestion and help her breathe more easily. This aspirator is battery operated so it provides you with suction that is stronger than that of a traditional manual aspirator. But don't worry, the suction is still gentle enough to use safely and comfortably on your baby.

Stuffy noses are a common problem with babies and toddlers. Babies often have stuffy noses because their nasal passages are so small and because their immune system is not yet fully developed. A stuffy nose may cause difficulty in breathing and discomfort especially since it takes some time for babies to learn to breathe through their mouths. Stuffy noses may also lead to more serious problems, therefore, it is very important to keep the nasal passages clear.

Yet clearing their congestion can often be a challenge for parents. Using a traditional manual aspirator may be uncomfortable for the child making it very difficult to use. In addition, as it may not work very well, it will take longer to clean the passages. The Bébésounds® NasalClear™ Nasal Aspirator will allow you to safely clear your child's nasal passages

very quickly, comfortably, and safely to help your child breathe more freely.

And, an added feature of this aspirator is the option to play 12 different children's tunes while in use. This may help divert your child and comfort her while clearing her nasal passages.

Please read and follow these instructions carefully to ensure proper use. Keep these instructions in a safe place for future reference.

## KEY FEATURES

- Gentle suction easily clears congestion
- Battery operation makes it work much faster thereby limiting your baby's discomfort
- Small, flexible, silicone tips make it gentle and comfortable to use, even on a newborn baby
- Two silicone tips to accommodate different nostril shapes and different types of congestion
- Comes apart for easy and thorough cleaning
- Cleans simply with warm soapy water
- Clear collection cup for easy viewing
- Plays 12 different tunes to comfort and divert your baby during use
- Gentle, safe, and easy to use

## BATTERY INSTALLATION

(requires 2 AA alkaline batteries, not included)

Use only AA alkaline batteries. Always replace both batteries at the same time. Do not mix new and old

batteries. For long periods of non-use, remove batteries from the compartment.

1. Remove the battery cover by sliding it down.
2. Insert two (2) AA alkaline batteries matching positive (+) to positive (+) and negative (-) to negative (-) (See Fig. 3).
3. Replace the battery cover.
4. Replace batteries when the power or sound of the pump begins to sound weak during use.

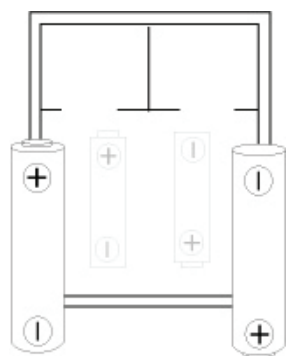
Figure 3

## INSTRUCTIONS FOR USE

1. Choose the correct tip to use based on the size and shape of your baby's nostrils. Also consider the type of congestion your baby has. The long, narrow tip is recommended for use when the mucus is loose and in a liquid form. The short tip

6

is recommended for use when the mucus is in a more solid form. However, choose the tip based on what works best for you and your baby. Be sure to place the tip securely on the aspirator.

2. Hold the baby upright and insert the tip into the baby's nose. **Be careful not to place the tip too far into the nostril or press against the inside wall of the nostril.** Be sure to keep the tip of the aspirator at a 90° angle. Do not point it up into the nostril (See Fig 4).

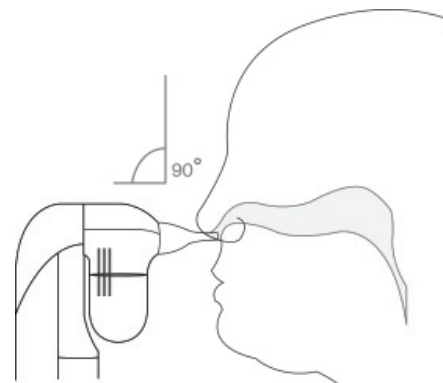
Figure 4

3. If desired, press the MUSIC button to play a tune to comfort your baby. The music will automatically turn off after the tune is finished playing. Press the MUSIC button again to play an additional tune.

7

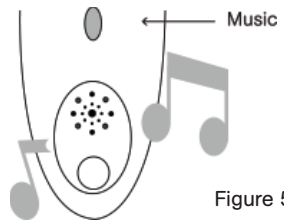
Figure 5

4. Press the POWER  button and allow the suction to clear the mucus.

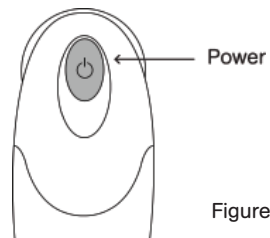
Figure 6

**NOTE:** You may not think it is working because the mucus may not immediately appear in the collection cup. However, when you take it apart you will see how well it is working.

5. Release the POWER button and remove from nostril.
6. Repeat in other nostril and as needed.
7. Clean thoroughly as instructed below.

## CARE AND CLEANING

1. Separate the suction portion of the aspirator from the base by gently pulling them apart (See Fig 7).

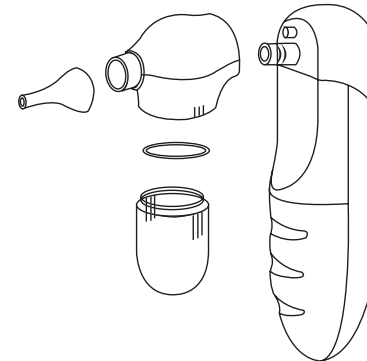
Figure 7

2. Unscrew collection cup by turning it to the left (counterclockwise).
3. Remove white rubber ring located where collection cup screws in.
4. Remove silicone tip.
5. Wash all 4 parts in warm soapy water or in the dishwasher (top shelf only) after each use.
6. Dry all parts thoroughly before using again or assembling.
7. Clean the base with a soft dry or slightly damp cloth.

**NOTE:** The base of the aspirator should never be submerged in water or any other liquid. Never put it in the dishwasher.

Figure 8

8. Be sure all parts fit tightly into each other before using.
9. Keep in provided storage bag and store in a cool dry place.
10. Keep out of the reach of children when not in use.

**CAUTION:**

Never insert the aspirator too far into the nostril. Never place the tip firmly against the inside wall of the nostril. Never use on ears, mouth, or eyes.

This aspirator is for clearing congestion in nasal passages. Do not use this device for anything other than its intended purpose.

**WARNING:**

This product is not a toy. Do not allow children to play with it. Individual parts may present a choking hazard.

If you believe your aspirator is defective or otherwise not operating properly, please check the batteries

first. If the aspirator is still not operating properly or if you are having any other technical problems, please contact the Bébésounds® Help Desk at 1-888-232-6476 Mon-Fri from 9:00AM-5:00PM EST.

## TECHNICAL SPECIFICATIONS

- For home use only
- Requires two (2) AA batteries (not included)
- Ambient storage environment:  
-4°-122°F (-20°- 50°C)
- Ambient operating environment:  
59°-95°F (15°- 35°C)

## APPLIED STANDARDS AND APPROVALS

This product conforms to the provisions of the EC directive MDD (93/42/EEC).

The following standards apply to design and/or manufacturing of the product:

- IEC / EN 980:2003: Graphical symbol for use in the labeling of medical devices
- IEC / EN 60601-1: Medical electrical equipment  
– Part 1: General requirement for safety
- IEC / EN 60601-1-2: Medical electrical equipment  
– Part 2: Collateral standard: Electromagnetic compatibility – Requirements and tests
- ISO 10993-5:1999: Biological evaluation of medical devices – Part 5: Tests for in vitro cytotoxicity
- ISO 10993-10:2002: Biological evaluation of medical devices – Part 10: Tests for irritation and delayed-typed hypersensitivity

- ISO 14971:2000: Medical devices – Application of risk management to medical devices

## LIMITED PRODUCT WARRANTY

Unisar Inc., the distributor of the Bébésounds® NasalClear™ Nasal Aspirator, warrants to the original owner of this product that this product is free from defects in materials and workmanship for a period of one (1) year from the date of the original purchase. If this aspirator fails to function properly when used as directed under normal conditions within the one-year period, Unisar Inc. will either repair or replace the product, at its option, free of charge, when it is returned in its original packaging, postage prepaid, with proof of purchase. Unisar Inc. will bear the cost of repairing or replacing the product and shipping it back to you. Repair or replacement of the defective item is subject to verification of the malfunction or defect when delivered to:

**Unisar® Service Center**  
15 West 36th Street  
New York, NY 10018

**Note:** The warrantor expressly disclaims liability for incidental, consequential, or special damages of any nature. This warranty does not cover 1) A defect that has been caused by repair or modification by any person other than a Unisar® technician; 2) A defect caused by negligence, accident, or misuse of the product; 3) A defect caused by incorrect installation of and/or defective batteries; 4) Battery life; or 5) Damage resulting from improper operation.

LEGAL RIGHTS VARY FROM STATE TO STATE.

If you have any questions, please call our technical support line toll free at 1-888-232-6476 Mon-Fri from 9:00AM–5:00PM EST.

**The NasalClear™ Nasal Aspirator is a personal care product and cannot be returned to the retailer. All exchange and repair requests should be directed to the Bébésounds® Help Desk at 1-888-232-6476 Mon–Fri from 9:00AM–5:00PM EST.**

**ALSO AVAILABLE FROM BÉBÉSOUNDS®...**

**1-Second Ear Thermometer**

Makes taking your child's temperature quick and convenient. No probe cover or "ear-tug" required. Cleans with alcohol. Lighted jumbo display. Measures in Fahrenheit and Celsius.

**AlwaysClean™ Pacifier**

Closes when dropped!™ Unique shield automatically closes when the pacifier is dropped keeping it sanitary for your baby. Silicone orthodontic nipple. 0-6 months and 6+ months.

**Deluxe Prenatal Gift Set**

Listen, talk, and play music to your unborn baby!™ Gift Set includes the Prenatal Heart Listener®, two headsets, a special recording cable, a fetal microphone, fetal speakers, a 60-minute CD of specially selected music by Mozart, and a maternity belt.

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©2006 Listro Associates