

Test model: BN-T10

Thank you for buying our bone conduction headphone.


Our mission is to use our technology to protect your ears.

Bone Conduction Wireless Hearing Aids Headphone User Manual

1. General Introduction

- This product includes two parts, one is a bone conduction headphone, and another is a transmitter, T10.
- This product will be a hearing aids when two parts are connected via Bluetooth
- T10 can connect with other electronic equipment with an audio cable, such as telephone, pad, TV etc. It lets user hear sound alone and no interruption to others
- Headphone can be used as a wireless headphone connecting with any smart phone freely. (Another user manual for reference)

2. Parts description:

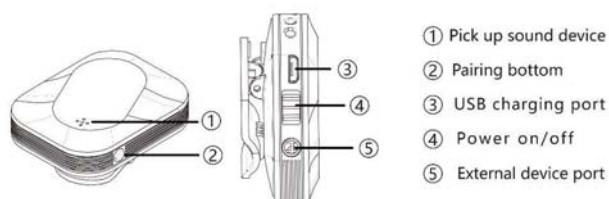

3. Accessories:

List	Details
Bone Conduction Headphone	1set
Transmitter	1set
Charge cable	1pcs
3.5mm Audio cable	1pcs
3.5mm to RCA adapter	1pcs
User manual	2pcs


4. Charging Instruction:

- When first used it, headphone and T10 both are supposed to charge fully with a standard USB cable.
- They will be fully charged around 3 hours.
- To extend battery life, do NOT use it during charging
- Turn them off after using in order to save power.

5. Bluetooth pairing

- First of all, turn on T10, the blue LED will slow flash and T10 starts waiting for pairing.
- Then turn on the headphone. Press and hold the multi-function (for 3 seconds) until the LED turns blue and flash slowly. During the very first 10 seconds, headphone will pair T10 automatically. If repairing doesn't succeed, headphone will enter the status of searching and wait for being paired.
- Press the repairing bottom on T10, the blue LED will flash quickly. T10 and headphone will be connected during 5 seconds.
- Now wear the headphone (as the sample pic showing), the sound can be received by user's cheekbones

6. How to wear

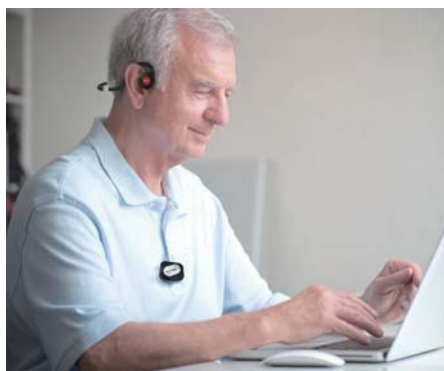

7. Connect with other electronic equipment.

- Use a 3.5mm audio cable to connect with other electronic equipment, such as telephone, pad, tablet etc., enjoy listening to sound clearly.
- Use an RCA adapter to connect with TV, sounds from TV can be received only by the headphone user, no interruption to others

Note: avoid to putting any metal obstacles between T10 and headphone.

8. Status LED

Headphone part:

LED Color	status
● Red (quick flash)	● Low power
● Red(solid)	● Charging
● Blue(solid)	● Charging complete
● Blue (quick flash)	● Pairing
● Blue (flash once every 5 seconds)	● Paired or standby
● Blue to Red (flash 1 second)	● Power off

T10 part

LED color	Status
Red (solid)	Charging
Blue (slow flash)	Waiting for pairing
Blue (solid)	Paired
Blue (quick flash)	Pairing

9. Basic operation

Headphone basic operation	
Function	Operation
Power on	Press and hold the multi-function bottom for 3 seconds
Power off	Press & hold the multi-function bottom for 5 seconds or standby for 10 minutes(automatically)
Pairing	To search T10 during the very first 10 seconds when power on. If failed, headphone will wait to be paired by hand
Re-pairing	Connect with T10 in 10 minutes automatically if headphone and T10 are separated
Adjust volume	Press the bottom “-”, “+”

T10 basic operation

Function	Operation
Power on	Turn on and wait for pairing
Power off	Turn off
Pairing bottom	Shortly press the bottom of pairing and search for headphone

10.Specification

Item	Headphone	T10
Play time	Around 11 hours	Around 12 hours
Charging time	Around 3 hours	Around 3 hours
Standby time	Around 1000 hours	Around 1000 hours
Bluetooth version	4.0	4.0
Frequency range	100Hz-18KHz	20Hz-20KHz
Wireless range	10meters	8meters

11. Trouble shooting

Issue	Possible problem	Resolution
Could not turn on	No power	Charging
Could not pair	Headphone or T10 could be connected with other equipment	Disconnect with other equipment and turn on again
Unstable connection	Headphone or T10 could be put far away or some obstacles between them	Let them be closer or remove the obstacles

12. Preservation and maintenance

- When first used, make sure the product is fully charged
- Please preserve the device at cool and dry places
- Do not use in rainy/stormy days
- When unused for a long period of time, make sure to charge before use
- Clean with dry, clean fabric, do not rinse with water

13. Warranty

- The warranty covers the repair fee of the headphones for 1 year since its purchase
- The damage due to personal or irresistible causes is not covered in the warranty
- The warranty automatically becomes invalid once the product is disassembled privately
- When repair is needed, please contact with your local agency

FCC Warning Statement

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- - Reorient or relocate the receiving antenna.
- - Increase the separation between the equipment and receiver.
- - Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- - Consult the dealer or an experienced radio/TV technician for help.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.