






**4**


Press   to select the desired Program.

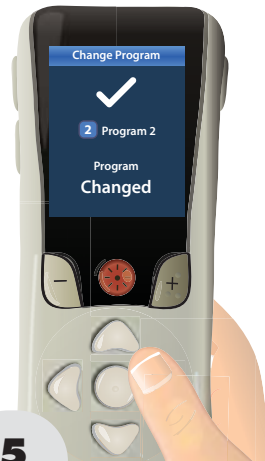

**5**

Press  to activate the selected Program.

### Note

If any changes were made to your stimulation (for example, changing stimulation strength), a Save Changes  confirmation screen will appear before proceeding with the selected operation. For more information, turn to page 48.



## Save a Program

If you have made changes to your stimulation (for example, stimulation strength), you can save the changes as the default settings for that Program. The new settings will remain until a new change is saved or the Program is restored to the settings that your healthcare provider last configured for you.



From the Main Menu  
☰, press ⏏ to  
select Programs **P**.



Press ⏏ to  
highlight Save   
Press ⏏ to  
select Save .



Select Yes ✓ to confirm or No ✗ to cancel.



The Save confirmation screen displays.

If you select Yes, the Saved screen will display to show you that you have successfully saved.

### Note

Get quick access to the Programs menu **P** by pressing the Programs button **P** on the left side of the RC.







## Restore a Program


If after saving changes to a Program, you decide that you want to change the Program back to the way it was configured by your healthcare provider, you can do so by restoring the Program.




From the Main Menu , press  to select Programs .



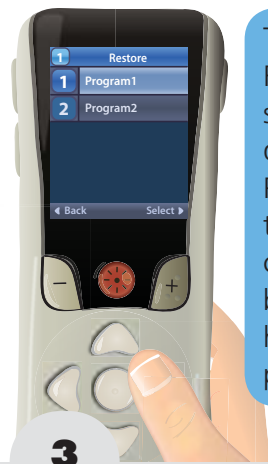

Press  twice to highlight Restore to Clinic Settings .  
Press  to select Restore to Clinic Settings .



The Restore screen will display all Programs that were configured by your healthcare provider.

Navigate to the Program you wish to restore and press .



Select Yes  to confirm or No  to cancel.



If you select Yes, the Restored screen will display to show you that you have successfully restored the program.

The Restore confirmation screen displays.




## Copy a Program

If you want to create a new Program similar to one that you already have, you can copy an existing Program to create a new one. You may want to do this, for example, if you generally like Program 1, but want to change its settings to feel something different. Copying Program 1 to a different Program will allow you to retain Program 1 as-is, but create a new Program that can be changed to better meet your therapy needs.


**1**

First, activate the Program that you want to copy. (See [Change to a Different Program](#) *page 24*)

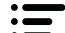**2**

From the Main Menu , select Programs , then select Copy to .



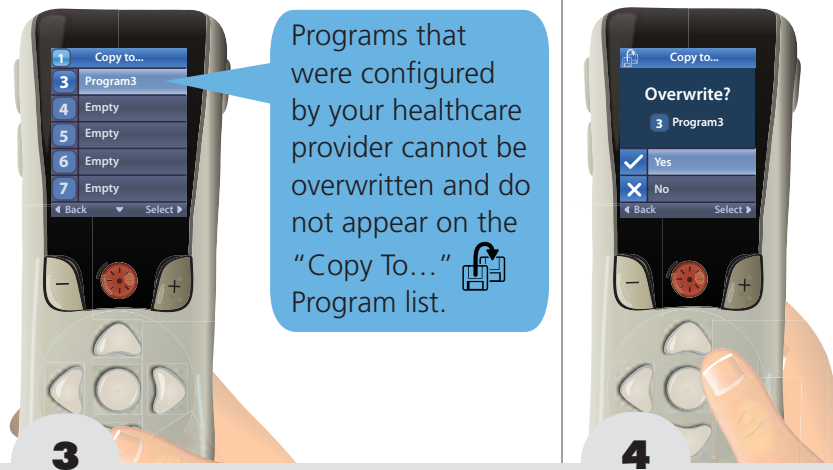

Navigate to an existing or Empty Program to which the settings are to be copied.

If you select a Program that is not Empty, the Overwrite confirmation screen displays. Select Yes to overwrite the settings for this Program or No to cancel.