

Delete a Program

Deleting a Program will completely remove the Program from the Stimulator. Once the Program is deleted, it cannot be recovered. Programs that were configured by your healthcare provider cannot be deleted and do not appear on the “Delete” Program list. Only Programs that were created using the “Copy To...” feature can be deleted.




From the Main Menu
☰, press ▶ to
select Programs **P**.






Navigate to Delete  and press .



Navigate to the Program you wish to delete and press .



Navigate to Yes  to confirm or No  to cancel, then press .





This page intentionally left blank.




Configuring Stimulation Areas

What is a Stimulation Area?	36
Change a Stimulation Area Parameter	37
Save and Restore an Area in a Program	40




What is a Stimulation Area?

A Stimulation Area  is the building block of a Program . Each Program can include settings for up to 4 Stimulation Areas. Each Stimulation Area is identified by a letter and a name, such as   ... "Area A".



These Stimulation Areas control parameters that can affect your stimulation therapy. These parameters are Amplitude , Pulse Width , and Rate . If your healthcare provider has configured these parameters to be adjustable, they can be changed and balanced for each individual Stimulation Area.



Change a Stimulation Area Parameter


 <p>1</p> <p>From the Main Menu  press  to navigate to Stimulation Areas , then press  to select it.</p>	 <p>2</p> <p>Navigate to the Area that you want to adjust and press .</p>	 <p>3</p> <p>Navigate to the parameter that you want to adjust and press .</p> <p><i>continued to next page</i> </p>
<p style="text-align: center;">AREAS</p> <p style="text-align: right;">Page 37</p>		




4 Press the  and  buttons to decrease and increase the value for the selected parameter.

5 To change another parameter, press the left Navigation button to return to the Area parameter menu. Repeat steps 3 - 4. To change a parameter in a different Area, return to the Area list in step 2.



When all desired changes have been made, press  repeatedly until you return to the Home Screen.

Note

Increasing the values for these parameters, especially Rate , may significantly increase the energy required for stimulation. This may lead to longer or more frequent recharging of the Stimulator.



Save and Restore an Area in a Program

To save any Area parameter changes so that they are the default parameters, you must save the Program **P**. This will save all changes to any Area within that Program. You cannot save the changes for each individual Area independently of changes for other Areas in the same Program. See [Save a Program](#) *page 26*.

If, after saving changes to a Stimulation Area in a Program, you decide that you want to revert the Stimulation Area back to the way it was configured by your healthcare provider, you can do so by restoring the Program **P**. Restoring the Program will restore all of the Stimulation Areas of that Program. See [Restore a Program](#) *page 28*.