


**How to Place ThermoSENSOR™ (TSS-24XX)**

Identify an area for ThermoSENSOR™ (TSS-24XX) to be pasted.

1. Place 3 fingers down from the navel.  
*Refer to diagram.*
2. Place the first layer of adhesive dressing about half the distance from the center to the side of the body.  
*Refer to diagram.*

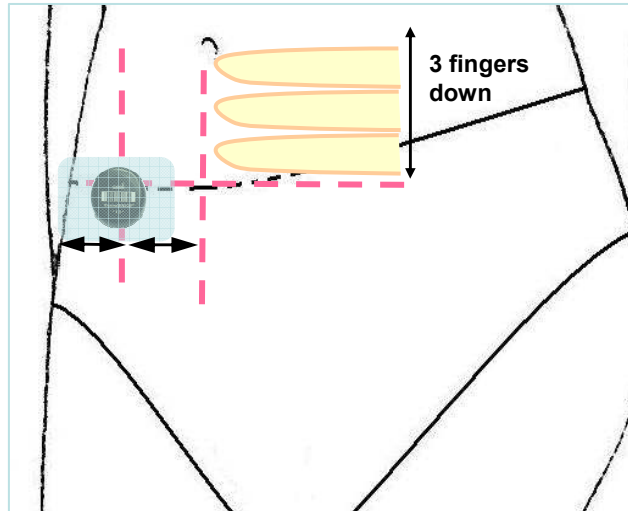


3. Place ThermoSENSOR™ on another layer of adhesive dressing\*.  
*The second layer of dressing should be pasted over the front of the sensor. Do not paste the dressing on the metal part of the sensor. You may use any suitable medical adhesive dressing e.g. 3M tegaderm*
4. Place ThermoSENSOR™ (with the second layer of adhesive dressing) over the first layer of adhesive dressing.
5. For cleaning before use, wipe the ThermoSENSOR™ with an alcohol swab.

*Please Note:*

*Cadi Scientific recommends 3M Tegederm dressing for use with the ThermoSENSOR as it has been tested to have no adverse effect on the user/ThermoSENSOR.*



CADI SCIENTIFIC PTE LTD  
1003 Bukit Merah Central, #04-40, Singapore 159836  
Tel: (65) 62762676 Fax: (65) 62766216  
Email: [info@cadi.com.sg](mailto:info@cadi.com.sg) URL: [www.cadi.com.sg](http://www.cadi.com.sg)  
Made in Singapore

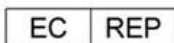

Medical Device Safety Service GmbH  
Schiffgraben 41, 30175 Hannover, Germany



This device complies with Part 15 of the FCC Rules.  
Operation is subject to the following two conditions:  
(1) this device may not cause harmful interference, and  
(2) this device must accept any interference received, including interference that may cause undesired operation.



*Please Note:*

*Changes or modifications not expressly approved by the manufacturer could void the user's authority to operate the equipment.*