

CAREMATIX 101

CPH200 HUB

Hints for connecting the HUB

1. Plug in the telephone cable into a telephone jack.
2. Before you plug in the power adapter, make sure that nobody is using the phone at that time.
3. Plug in the power adapter into an electrical outlet (120V AC).

Note: After plugging in the power adapter, you should hear a modem dialing tone. If you do not hear anything after a few seconds of having plugged in the power adapter then unplug the power adapter and plug it in again.

Care and Maintenance

1. This unit uses precision parts, care should be taken to avoid extremes in temperature variations, humidity, shock, dust and direct sunlight.
2. Do not drop or strike this unit.
3. When cleaning this unit, use only a soft, dry cloth.

Specifications

Power Supply	120V AC
Operating Temperature	+10°C to +40°C (+50°F to +104°F)
Relative humidity	85% Rh or below
Storage Temperature	-34°C to +65°C (-29°F to +149°F)
Size	106(W) x 166(D) x 53(H)mm

Specifications are subject to change without prior notice due to improvements in performance and quality.

Warranty

This equipment is guaranteed for the period of 1 year after the date of purchase against manufacturing defects when returned along with the proof of date of purchase. During this period, the unit will be repaired or replaced free of charge if the failure is attributable to faulty design or manufacture. This warranty does not cover damage or malfunctions caused by improper handling or use contrary to the instructions in this manual. Opening or taking apart the unit or its components will void the warranty. Should your unit require service, please e-mail to us at service@carematix.com

SAFETY PRECAUTIONS

The phone line should NEVER be plugged in during a thunderstorm.

CAUTION

Changes or modifications to this equipment not expressly approved by Carematix, Inc. could void the user's authority to operate the equipment.

NOTICE

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation.

Carematix, Inc.
2 NORTH LASALLE STREET, SUITE 1904
CHICAGO, IL 60602
WEB SITE: [HTTP/WWW.CAREMATIX.COM](http://WWW.CAREMATIX.COM)