



clearcount

MEDICAL SOLUTIONS

SmartSponge®
System


Quick Start Guide



For use with SmartSponge®
Surgical Sponges

100417 Rev. D

Lift

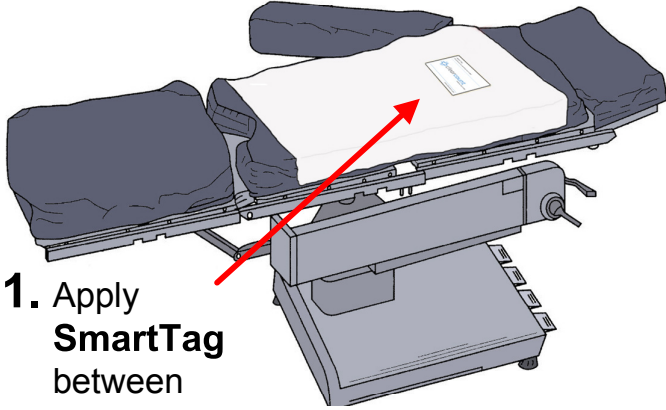


1. Apply **SmartTag** between sheets, directly below surgical site.



2. Plug in power cord, flip switch to **ON**.
(May be left in **ON** position when not in use)




3. Press **“ON”** to enter COUNT MODE.

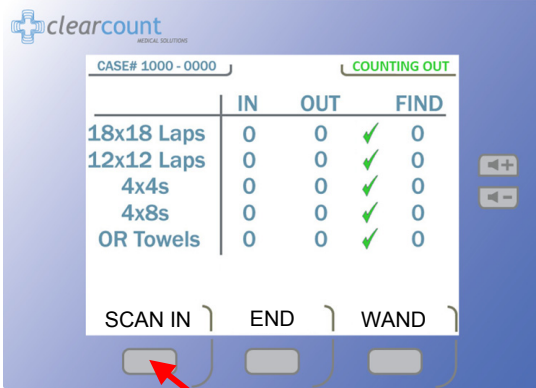



4. Place **Bucket Liner** into Count Out Bucket.

5. Press **“SCAN IN”** to scan packs in by holding them on the **In Scan Tray**.

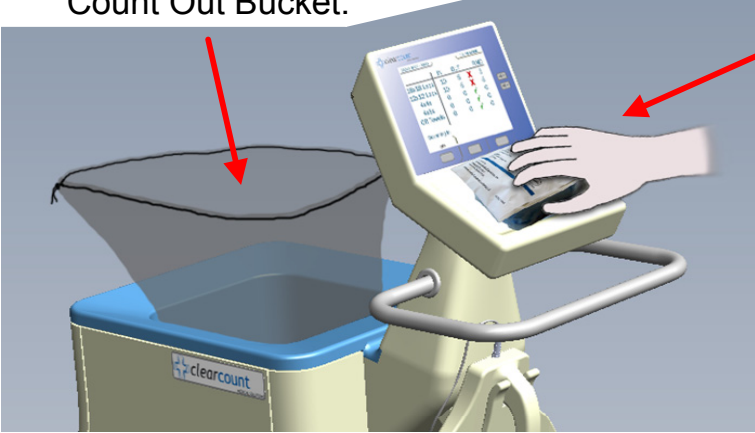



6. Open and use sponges.

1. Ensure System is in “**COUNTING OUT**” Mode; verified by display.
2. Discard all sponges directly into the **Count Out Bucket**.
3. Ensure that all sponges have been accounted for by placing them into the Count Out Bucket before ending the case.

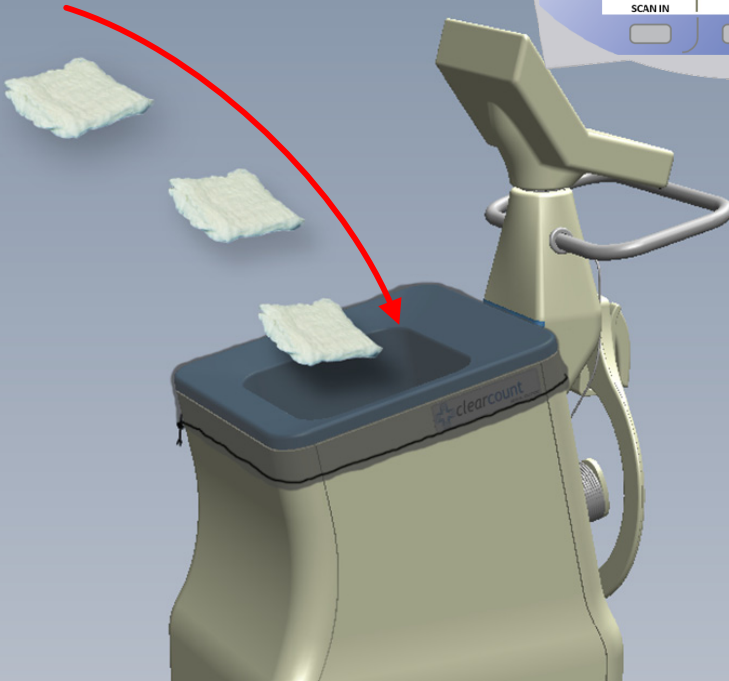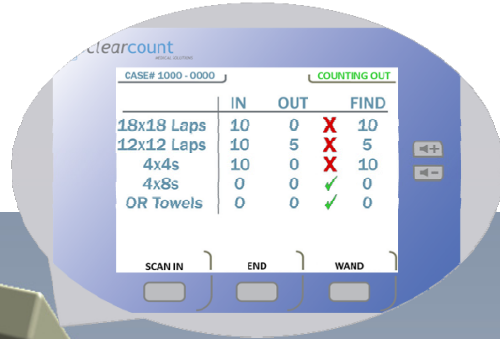

- If the “**Change Bag**” System Alert appears, remove the sponges or discard the full bag and replace with a new liner. Sponge counts will not change. Do not fill the Count Out Bucket with more than **50** sponges at a time.
- If a sponge that has not been scanned in is detected by the Count Out Bucket, the system will prompt the user on what action to take. **Accept** infers the sponge was intentionally discarded into the bucket. **Decline** infers the sponge was accidentally held too close to the Count Out Bucket prior to being scanned in.

1. Press the “**END**” button to view the final sponge counts. Make sure all columns have a green check mark.

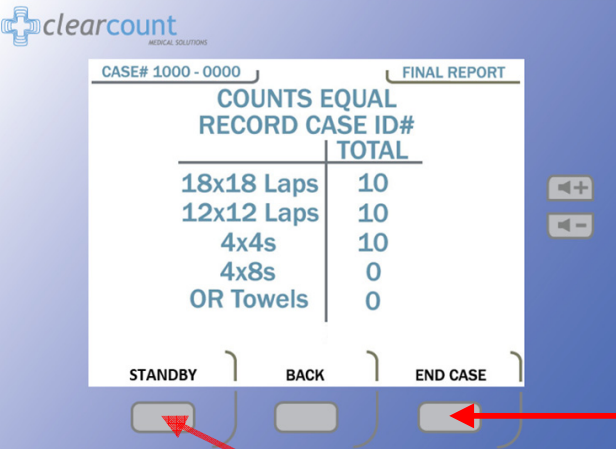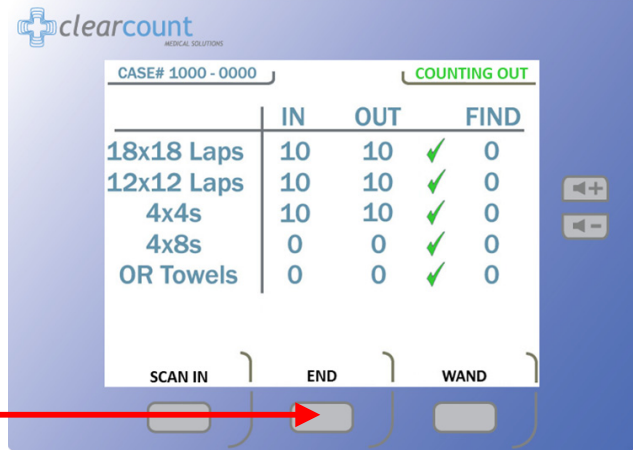


2. Press the “**END CASE**” button to save the case data and power down the device.




Note. Press the “**STANDBY**” button to retain case data and temporarily power down the device.

3. If the **Final Report** is requested and the sponge counts are not reconciled, an **Override Card** will be needed to end the case.



1. Cover the **SmartWand** with a sterile cover while passing it into the sterile field.
2. Press the **“WAND”** button to activate the wand. The green LED on the wand will illuminate when activated.

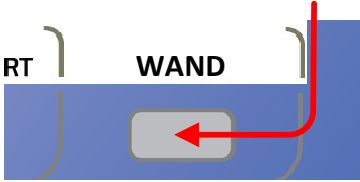

3. Hold the wand over the surgical site above the location of the **SmartTag**. When the SmartTag is detected, the on-screen figure will change to green as shown.

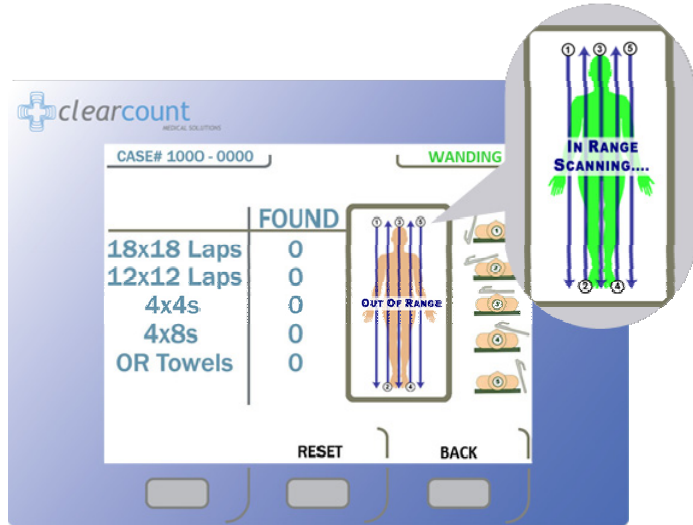

4. Slowly scan the patient holding the wand **2-3** inches above, at a rate no faster than **7** inches a second while following the onscreen directions. When a sponge is detected, the Amber LED on the wand will illuminate followed by an audible tone and the type and quantity will be displayed on the screen. Press **“RESET”** to clear the **“FOUND”** column to rescan.

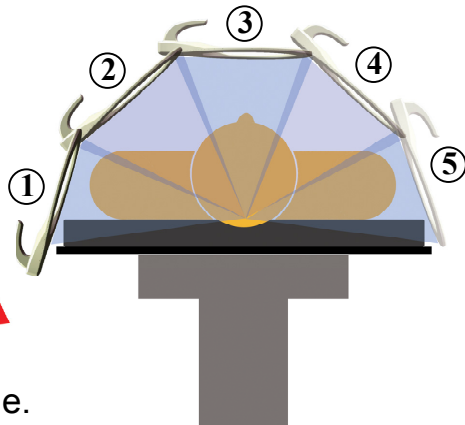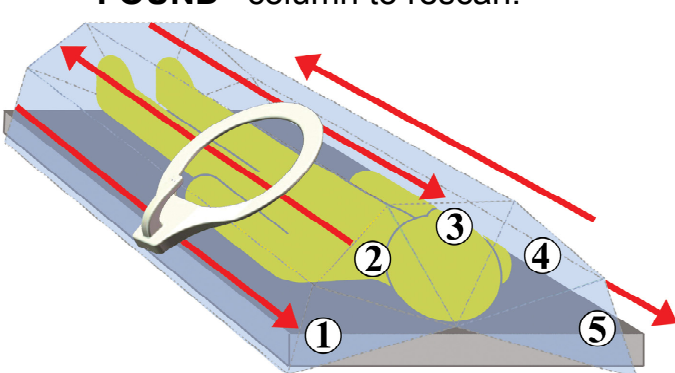

5. Press **“BACK”** to return to count mode.

System Messages are displayed on-screen as System Alerts, Warnings, or Failures.

SYSTEM MESSAGE	CAUSE	ACTION
Pack Not Verified – Retry Pack	System unable to scan pack.	Flip or Rotate pack and Rescan. If rescanning does not work, discard pack.
Discard Pack	System has detected a problem with pack.	Discard pack and resume scanning with a new pack.
Multiple Packs Detected – Remove and Scan One Pack at a Time	System unable to scan more than one pack at a time.	Ensure one pack is placed on the In-Scan Tray at a time.
Pack Already Scanned	Pack already counted.	Pack ready for use – open and use / continue.
Change Bag – Bucket Limit Has Been Exceeded – Remove Sponges to Continue	Over 50 sponges in Count Out Bucket.	Remove sponges or discard bag and replace. (Counts will not change)
Bag Overflow Warning – Bucket Limit Has Been Exceeded – Remove Sponges to Continue	Over 70 sponges in Count Out Bucket.	Remove sponges or discard bag and replace. (Counts will not change)
System Reset – Bucket Limit Has Been Exceeded – Remove Sponges and Power Cycle the System	Over 80 sponges in Count Out Bucket.	Remove sponges and power cycle system. Rescan sponges in groups of no more than 50 to assure each has been counted.

System Failure

A system failure will prevent the system from functioning properly.

In this event.

- Contact ClearCount for Service,
- Provide Numeric Error Code from screen,
- Power down the system.

Service Contact

ClearCount Medical Solutions
 101 Bellevue Road. Suite 300
 Pittsburgh, PA 15229
 (888) 931-0787
service@clearcount.com