DREEM®

BY RYTHM

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## WHAT'S INCLUDED IN THE BOX CE QUE CONTIENT VOTRE COFFRET

- Dreem Headband
  - Charger & Micro USB Cable
  - Travel Bag
  - Set of Back Sensors
  - Replacement Pads
- 
- Bandeau Dreem
  - Chargeur & Câble Micro USB
  - Étui de Voyage
  - Paire de Capteurs Arrières
  - Pads de Remplacement

We are very excited to introduce you to your new sleep companion, **Dreem**.

Rythm was founded in 2014 with a vision to enhance sleep for everyone.

Today, after three years of research and clinical tests, it's finally ready for you.

The next few pages will guide you as you become familiar with the product and join us in our journey to revolutionize sleep for millions.

Thank you for your trust and we can't wait to hear from you.

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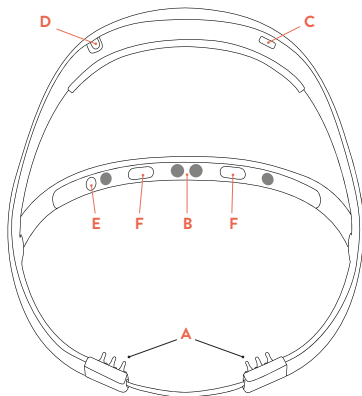
A handwritten signature in black ink, appearing to read 'H. Mercier', with a horizontal line underneath it.

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
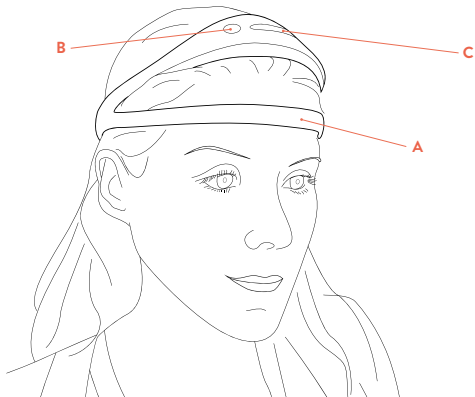
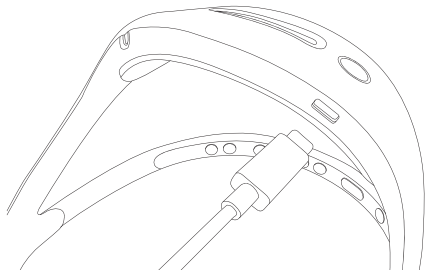
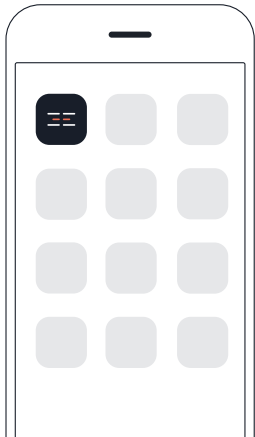
#### **DOWNLOAD**

Go to your App Store or Google Play Store and search for “Dreem: monitor, analyze and enhance your sleep”.

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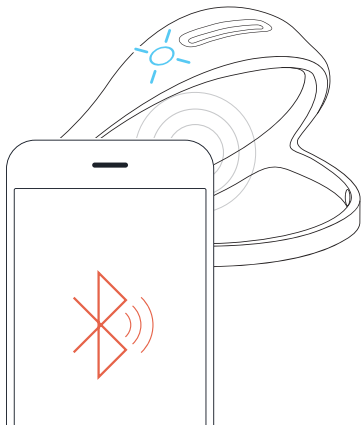**SET-UP**

Launch the Dreem App, create a Dreem account and then follow the first time setup to pair your headband with your app.

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### APP TOUR

Discover all our sleep features that help you fall asleep faster, get deeper sleep, and wake up refreshed.

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### LIVE EEG & HEART RATE

See your live EEG and heart rate with the Dreem App.

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### GET STARTED WITH A NIGHT OF GOOD SLEEP

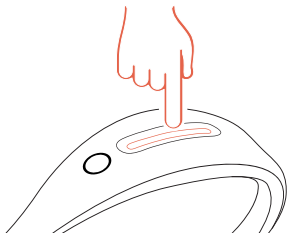
Experience your first night with the Dreem headband.

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# DREEM'S TOUCHPAD PROVIDES ADDED CONVENIENCE

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### **LONG TOUCH:**

Starts a Sleep Session with the previous settings (features, alarm, volume...)

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### **SLIDE:**

Adjusts the volume of the selected feature

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## **MORNING ROUTINE**

Once you're awake, connect your headband to the charger to stop your Sleep Session and view your metrics. They will appear in the Home section of the Dreem App.

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Information Center on your Dreem app

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Go to [support.dreem.com](https://support.dreem.com)

Aller à [support.dreem.com](https://support.dreem.com)

[facebook.com/dreemrhythm](https://facebook.com/dreemrhythm)

[@dreemrhythm](https://twitter.com/dreemrhythm)