
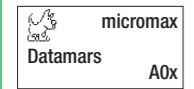
How to operate the micromax

Push the start button once.
The display indicates:


Point the top of the micromax close to the zone where the microchip of your animal has been implanted.

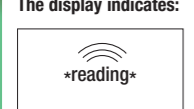Push the start button once again.
The display indicates:

The micromax reads ISO FDX-B microchips.

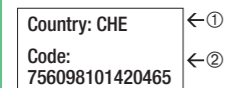If an ISO microchip is present,
the display indicates:

① Abbreviation of country code
② ID code number according to ISO 11784/5

If a microchip is not read,
the display indicates:

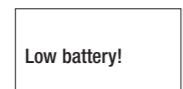
You can repeat the procedure by again pushing the start button.

The micromax switches automatically off after approx. 90 seconds.

After more than 3000 readings the batteries should be replaced.

The display indicates:

For replacement use two 1.5 Volts AAA batteries Alkaline. Another type of battery will not function properly.

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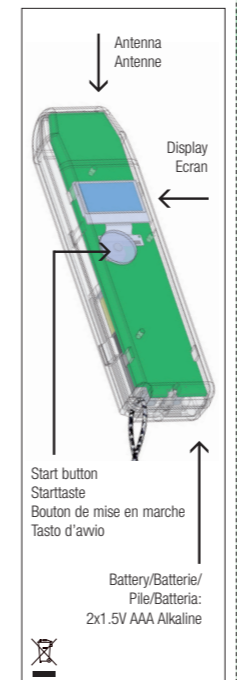
micromax

Warranty (for US and Canada)

The warranty covers damage caused by faulty material or manufacturing defects for the period of 1 year from the date of purchase. Equipment repaired by a third party, as well as damage caused by the customer are not covered by this warranty. Claims for covering of damage exceeding the limits of the present warranty are not accepted. In case of repair or replacement the warranty is neither extended nor renewed.

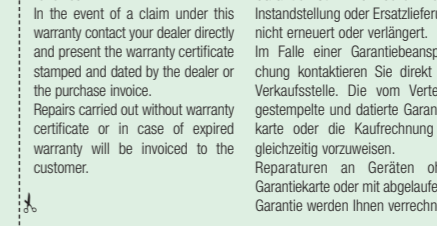
Model:
Serial number:
Date:
Distributor:

micromax

Warranty

The warranty covers damage caused by faulty material or manufacturing defects for the period of 2 years from the date of purchase. Equipment repaired by a third party, as well as damage caused by the customer are not covered by this warranty. Claims for covering of damage exceeding the limits of the present warranty are not accepted. In the event of a claim under this warranty contact your dealer directly and present the warranty certificate stamped and dated by the dealer or the purchase invoice. Repairs carried out without warranty certificate or in case of expired warranty will be invoiced to the customer.

Model:
Serial number:
Date:
Distributor: Vertretung: Distributeur: Distributore:

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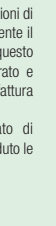
Certifications

FCC Part 15

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation. This equipment has been tested and found to comply with the limits for Class B Digital Device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the equipment and receiver
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected
- Consult the dealer or an experienced radio/TV technician for help

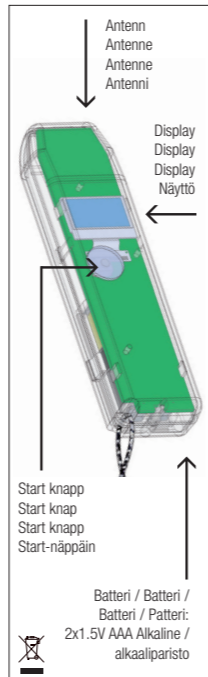
Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

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European directives

Hereby, Datamars SA via ai Prati 6930 Bedano-Lugano (CH) declares that the MICRO MAX is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC.

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