

Personal Care Ambulatory



Yes, you can.

Parts Catalog Usage Guide

The information contained in this document is subject to change without notice. This catalog contains service parts for Form No. 92-22. If you have any questions regarding this catalog, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-333-6900

For Faxed In Orders Call:

1-800-678-4682 or

1-440-366-0709

For Technical Service Call:

1-800-832-4707

Literature Can Be Found at:

www.invacare.com or

1-800-828-6272

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

© 2018 Invacare Corporation. Elyria, Ohio.

Invacare, I-Class, Ensolite, Rollite, Blue-Release, SureGlide, Court-Side Glides and Yes, you can., are trademarks of Invacare Corporation.

TABLE of CONTENTS

Description	Page Number
Accessories - Walker Tray Hardware - Model 6002	60
Accessories - Walker Tray Hardware - Model 6007	59
Accessories, Rollator (Basket/Bag)	123
Accessories, Rollator (Tray/Seat)	119
Accessories, Walker	57
Accessories, Walker (Basket and Pouch)	58
Brake Attachment	115
Canes (Bariatric - Models 8940-6, 7831-2, 7835-2)	19
Canes (Composite Quad - Model 7821/Orthopedic Grip/Folding - Model 40800-6)	21
Canes (Quad - Model 7823, 7827, 7829, 3920-2)	17
Canes (Walking - Model 8901, 8902, 8913, 40915-6, 8916, 8917, 8931, 8932, 40918-6, 8925, 8926, 8962 and 8964)	14
Crutches (Forearm - Model 6152, 6153, 6154)	25
Crutches (Forearm - Model 8153J, 8153T and 8153A)	27
Crutches (Quick Change, Quick Adjust and Bariatric Crutch)	28
Fixed Wheel Attachment - Single with Glide Tips	105
Glide Tips	125
Knee Walker - Model 65950	98
Knee Walker - Model 65960	100
Leg Extensions	107
Packaging and Instruction Sheets	12
Platform Attachment - Model 6027	103

TABLE of CONTENTS

Description	Page Number
Rollator - Fixed Wheel Attachment	101
Rollator (Four Wheeled) - Model 65350GR	61
Rollator (Four Wheeled) - Model 65420 - Spartan	70
Rollator (Four Wheeled) - Model 65450	72
Rollator (Four Wheeled) - Model 65500 - Stingray	74
Rollator (Four Wheeled) - Model 65550	76
Rollator (Four Wheeled) - Model 65650 and 65650R	78
Rollator (Four Wheeled) - Model 65700 - Frontier Lite	84
Rollator (Four Wheeled) - Model 65750	86
Rollator (Four Wheeled) - Model 65800 - Sandstorm	88
Rollator (Four Wheeled) - Model 65850	90
Rollator (Four Wheeled) - Model 65851R - 65851B	92
Rollator (Four Wheeled) - Model 65900 - Stardust	94
Rollator (Four Wheeled) - Model 67100 - FR300	96
Rollator (Rollite Four Wheeled) - Model 65100, 65100R, 65100-JR, 65100R-JR, 68100 and 68100-T	67
Rollator (Three Wheeled) - Model 65400 - Sprint	63
Rollator (Three Wheeled) - Model P429/2 & 65510 (Current) Model 65410 (Before 9/17/07) .	65
Rollator Bariatric (Four Wheeled) - Model 66500	80
Rollator Bariatric (Four Wheeled) - Model 66550	82
Sure Glide Brakes	117
Swivel Wheel Attachments - Model 6269	113

TABLE of CONTENTS

Description	Page Number
Swivel Wheel Attachments - Model 6272 & 6268	109
Swivel Wheel Attachments - Model 65100, 6267 & 6273	111
Walker (Blue-Release - Model 6241-5F and 6241-JR5F)	41
Walker (Blue-Release Walker - Model 6240-A, 6240-5F & 6240-JR5F)	43
Walker (Blue-Release Walker - Model 6241-A (Discontinued 6/07) & 6241-Jr.)	39
Walker (Dual Release - Model 6261-A, 6261-Jr., 6291-A, 6291-1 & 6291 Jr.)	45
Walker (Dual Release - Model 6291-3F, 6291-JR3F, 6291-5F & 6291-JR5F)	47
Walker (Dual Release Bariatric - Model 6441-A)	51
Walker (Dual Release, Heavy Duty - Model 6291-HDA)	49
Walker (Hemi - Model 6252)	53
Walker (Pediatric - Model 6210R, 6210B, 6210P, 6211R, 6211B & 6211P)	30
Walker (Single Release - Model 6281-3F)	37
Walker (Single Release - Model 6281-A, 6281-1 & 6281 JR)	34
Walker (Single Release - Model 6285-A, 6285-3F)	36
Walker (WalkLite - Model 6300-5F, 6300-ATA and 6300-JRA)	55
Warranty Replacement	5

Warranty Replacement

Canes

Reference Model Number	Warranty Number	Description	Warranty Package
3920-2	1055546	WARR RPLMT - Quad with Standard Grip, Medium Base	1
7821 - NLA	1064810-NLA	NLA - WARR RPLMT - Quad Composite, Standard Grip	1
7823	1064799	WARR RPLMT - Quad with Dual Cam Grip, Medium Base	1
7827	1055547	WARR RPLMT - Quad with Standard Grip, Large Base	1
7829 - NLA	1064800-NLA	NLA - WARR RPLMT - Quad with Dual Cam Grip, Large Base	1
7831-2	1141824	WARR RPLMT - Bariatric Small Base Quad Cane	1
7835-2	1141825	WARR RPLMT - Bariatric Large Base Quad Cane	1
8901 - NLA	1089064-NLA	NLA - WARR RPLMT - Custom Fit Crook with Foam Grip	1
8902 - NLA	1089065-NLA	NLA - WARR RPLMT - Custom Fit Offset with Foam Grip	1
8912 - NLA	1064789-NLA	NLA - WARR RPLMT - Standard with Foam Grip, Brown	1
8913	1064790-NLA	NLA - WARR RPLMT - Standard with Foam Grip, Black	1
8914 - NLA	1064791-NLA	NLA - WARR RPLMT - Junior Standard with Foam Grip, Silver	1
8916	1055549	WARR RPLMT - Standard Crook, Black	1
8917	1055550	WARR RPLMT - Offset, Silver	1
8919 - NLA	1055552-NLA	NLA - WARR RPLMT - Offset, Bronze	1
8925 - NLA	1055555-NLA	NLA - WARR RPLMT - Modified Crook, Silver	1
8926 - NLA	1055556-NLA	NLA - WARR RPLMT - Modified Crook, Black	1
8927 - NLA	1055557-NLA	NLA - WARR RPLMT - Modified Crook, Bronze	1
8928 - NLA	1055558-NLA	NLA - WARR RPLMT - Orthopedic, Left	1

Warranty Replacement

Canes

Reference Model Number	Warranty Number	Description	Warranty Package
8929	1055559-NLA	WARR RPLMT - Orthopedic, Right	1
8931	1064792	WARR RPLMT - Offset with Strap with Dual Cam Grip, Silver	1
8932	1064793-NLA	NLA - WARR RPLMT - Offset with Dual-Cam Grip	1
8933 - NLA	1064794-NLA	NLA - WARR RPLMT - Offset with Strap with Dual Cam Grip, Brown	1
8934 - NLA	1064795-NLA	NLA - WARR RPLMT - Offset with Strap with Dual Cam Grip, Red	1
8935	1064796-NLA	NLA - WARR RPLMT - Offset with Strap with Dual Cam Grip, Blue	1
8936	1064822-NLA	NLA - WARR RPLMT - Modified Crook with Foam Grip, Black	1
8940-6	1141826	WARR RPLMT - Bariatric Offset Handle Cane	1
40800-6	1065425	WARR RPLMT - Adjustable Folding, Black	1
40802-3	1095475-NLA	NLA - WARR RPLMT - Wood, Walnut	1
40915-6	1055548-NLA	NLA - WARR RPLMT - Standard Crook, Silver	1
40918-6	1055551	WARR RPLMT - Offset with Strap with Dual Cam Grip, Black	1

NOTE: The following part numbers will allow you to order (1) each product (i.e., 8918 Offset Cane - 6 per carton) for warranty replacement of multi-packaged item.

Warranty Replacement

Crutches

Reference Model Number	Warranty Number	Description	Warranty Package
6151 - NLA	6151 - NLA	WARR RPLMT - Eurostyle Forearm, Junior	1
6152	1055561-NLA	WARR RPLMT - HD Forearm, Junior	1
6153	1055562	WARR RPLMT - HD Forearm, Adult	1
6154	1055563	WARR RPLMT - HD Forearm, Tall Adult	1
8115-A	1038057	WARR RPLMT - Quick Change, Adult	1
8115-J	1038059	WARR RPLMT - Quick Change, Junior	1
8115-T	1038058	WARR RPLMT - Quick Change, Tall Adult	1
8120-A	1099634	WARR RPLMT - Quick Adjust, Adult	1
8120-J	1099633-NLA	WARR RPLMT - Quick Adjust, Junior	1
8120-T	1099635-NLA	WARR RPLMT - Quick Adjust, Tall Adult	1
8130-A	1106802	WARR RPLMT - Bariatric, Adult	1
8130-J	1106827-NLA	WARR RPLMT - Bariatric, Junior	1
8130-T	1106830	WARR RPLMT - Bariatric, Tall Adult	1
8153-J	1167803	WARR RPLMT - Crutch, Forearm, Junior (Pair)	1
8153-A	1167804	WARR RPLMT - Crutch, Forearm, Adult (Pair)	1
8153-T	1167805-NLA	WARR RPLMT - Crutch, Forearm, Tall Adult (Pair)	1

NOTE: The following part numbers will allow you to order (1) each product (i.e., 8918 Offset Cane - 6 per carton) for warranty replacement of multi-packaged item.

Warranty Replacement

Rollators

Reference Model Number	Warranty Number	Description	Warranty Package
65400	65400	WARR RPLMT - Three Wheeled (Sprint)	1
65100	65100	WARR RPLMT - Four Wheeled (Ultralight)	1
65410	65410	WARR RPLMT - Three Wheeled	1
65420	65420	WARR RPLMT - Four Wheeled (Spartan)	1
65500	65500	WARR RPLMT - Four Wheeled (Stingray)	1
65600	65600	WARR RPLMT - Four Wheeled (Stargazer)	1
65650R	65650R	WARR RPLMT - Four Wheeled Rollator - Red	1
65700	65700	WARR RPLMT - Four Wheeled (Frontier Lite)	1
65750	65750	WARR RPLMT - Four Wheeled, with Pushdown Brake	1
65800	65800	WARR RPLMT - Four Wheeled (Sandstorm)	1
65900	65900	WARR RPLMT - Four Wheeled (Stardust)	1
66500	66500	WARR RPLMT - Bariatric Rollator	1

NOTE: The following part numbers will allow you to order (1) each product (i.e., 8918 Offset Cane - 6 per carton) for warranty replacement of multi-packaged item.

Warranty Replacement

Walkers

Reference Model Number	Warranty Number	Description	Warranty Package
6210's	6210	WARR RPLMT - Pediatric	1
6211's	6211	WARR RPLMT - Pediatric	1
6240-A	1143738	WARR RPLMT - Blue Release, Adult	1
6240-5F	1144742-NLA	WARR RPLMT - Blue Release, Adult (5" Fixed Wheels)	1
6240-JR	1144833	WARR RPLMT - Blue Release, Junior	1
6240-JR5F	1144743	WARR RPLMT - Blue Release, Junior (5" Fixed Wheels)	1
6241-A	1105658-NLA	NLA - WARR RPLMT - Blue Release Walker, Adult	1
6241-Jr.	1109632	WARR RPLMT - Blue Release Walker, Junior	1
6241-5F	1132561	WARR RPLMT - Blue Release, Adult	1
6241-JR5F	1144743	WARR RPLMT - Blue Release, Junior	1
6252	1097113	WARR RPLMT - Hemi Adult	1
6281-A	1108105	WARR RPLMT - Single Release, Adult	1
6281-Jr.	1061876	WARR RPLMT - Single Release, Junior	1
6281-3F	1061877-NLA	NLA - WARR RPLMT - Single Release, Adult with 3" Fixed Wheel and Glide Tip	1
6282 - NLA	1061876	WARR RPLMT - Single Release, Junior	1
6283 - NLA	1061877-NLA	NLA - WARR RPLMT - Single Release, Adult with 3" Fixed Wheel and Glide Tip	1
6285-A	1143739-NLA	NLA - WARR RPLMT - Single Release, Adult	1
6285-3F	1144745-NLA	NLA - WARR RPLMT - Single release, Adult	1
6285-JR - NLA	1144744-NLA	NLA - WARR RPLMT - Single release, JUnior	1
6261-A	1064813-NLA	NLA - WARR RPLMT - Dual Release, Adult	1
6261-Jr.	1064814	WARR RPLMT - Dual Release, Junior	1

Warranty Replacement

Walkers

Reference Model Number	Warranty Number	Description	Warranty Package
6291-A	6291-1	WARR RPLMT - Dual Release, Adult	1
6291-Jr.	1061872	WARR RPLMT - Dual Release, Junior	1
6291-HDA	6291HDA-1	WARR RPLMT - Heavy Duty Dual Release, Adult	1
6291-Jr3F	1064798	WARR RPLMT - Dual Release, Junior with 3" Fixed Wheel and Glide Tip	1
6291-3F	1061873	WARR RPLMT - Dual Release, Adult with 3" Fixed Wheel and Glide Tip	1
6291-3S - NLA	1061874-NLA	WARR RPLMT - Dual Release, Adult with 3" Swivel Wheel and Glide Tip	1
6291-5F	1100858	WARR RPLMT - Dual Release, Adult with 5" Fixed Wheel and Glide Tip	1
6291-JR5F = Kit 6271 and 1061872	1061872	WARR RPLMT - Dual Release, Junior	1
6292 - NLA	1061872	WARR RPLMT - Dual Release, Junior	1
6293 - NLA	1061873	WARR RPLMT - Dual Release, Adult with 3" Fixed Wheel and Glide Tip	1
6294 - NLA	1061874-NLA	WARR RPLMT - Dual Release, Adult with 3" Swivel Wheel	1
6299 - NLA	1061878-NLA	NLA - WARR RPLMT - Adult, Rigid	1
6299-RA	1061878-NLA	NLA - WARR RPLMT - Adult, Rigid	1
6441-A	1141969	WARR RPLMT - Bariatric Dual Release Walker	1
NOTE: The following part numbers will allow you to order (1) each product (i.e., 8918 Offset Cane - 6 per carton) for warranty replacement of multi-packaged item.			

Warranty Replacement

Walker Accessories

Reference Model Number	Warranty Number	Description	Warranty Package
6013 - NLA	1055589-NLA	WARR RPLMT - Pouch (Green & Burgandy)	1
6015-4	1097324	WARR RPLMT - Pouch (Green & Blue)	1
6290	1134474	WARR RPLMT - Court-Side Glide	1

Packaging and Instruction Sheets

Model	Cartons	Inserts	Instruction Sheet/Hang Tag
Canes	N/A	N/A	N/A
7821, 7823, 3920	40131X106	N/A	1089631
7827, 7829	40131X134	N/A	1089631
8901, 8902	1086774-NLA	N/A	1089631
40915, 8916, 8917, 8918, 8925, 8926, 8928, 8929	1052255	N/A	1089631
40800	1071301-NLA	N/A	1089631
Blue Release Walker	N/A	N/A	N/A
6241-A, 6241-JR,	1058493	1109633	1130187
Dual Release Walker	N/A	N/A	N/A
6261-A, 6261-JR, 6291-1, 6291-A, 6291-JR, 6291-JR3F, 6291-3F, 6291-3S (NLA)	1040844	N/A	1130187
Hemi Walker	N/A	N/A	N/A
6252	N/A	N/A	1098312
Pediatric Walker	N/A	N/A	N/A
6210's	1101213-NLA	N/A	1098350-NLA
6211's	1100552-NLA	N/A	1098350-NLA
Rigid Walker	N/A	N/A	N/A
6299-RA	1040844	N/A	1060592-NLA
6300-5F WalkLite Walker	N/A	N/A	1122169-NLA
Rollators	N/A	N/A	N/A
6540, 6540-PB	N/A	N/A	1023669-NLA
65100, 65100R, 65100-JR, 65100R-JR, 68100, 68100-T	N/A	N/A	1125028
65400, 65420, 65500	N/A	N/A	1076166
65600	N/A	N/A	1076167
65700, 65800, 65900	N/A	N/A	1100867-NLA
Single Release Walker	N/A	N/A	N/A
6281-1, 6281-A, 6281-JR, 6281-3F	1040844	N/A	1148068
Carrying Handle for Carton	1032005	N/A	N/A
Accessories	N/A	N/A	N/A
6000-NLA	40131X044	N/A	1039646-NLA
6001-NLA, 6002-4-NLA, 6002-5-NLA	1043548	N/A	1039646-NLA
6003-NLA	40131X044	N/A	1039646-NLA
6004-NLA, 6005-NLA	1043548	N/A	1039646-NLA
6098	1089770-NLA	N/A	1089634
6007	1089755	N/A	1089635
6013	40131X011	N/A	1046726-NLA


Packaging and Instruction Sheets

Model	Cartons	Inserts	Instruction Sheet/Hang Tag
6015-1 See Note(s) A	N/A	N/A	1089636-NLA
6015-4	1063943-NLA	N/A	1089636-NLA See Note(s) A
6017-NLA	N/A	N/A	1059601-NLA
6027	40131X106	N/A	1064716
6248-NLA	40131X044	N/A	1039646-NLA
6249-NLA	1043548	N/A	1019530-NLA
6255-NLA	40131X044	N/A	1011508-NLA
6256	40131X044	N/A	1015548-NLA
6257-NLA	1043548	N/A	1089633
6259-NLA	40131X044	N/A	1016693-NLA
6263, 6264, 6265, 6266	40131X011	N/A	1089633
6267, 6268, 6269	1043548	N/A	1089633
6270, 6271	40131X011	N/A	1089633
6272, 6273	1043548	N/A	1089633
6274, 6275	40131X011	N/A	1089633
6277, 6278	1043548	N/A	1089633


NOTE: A - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.

NOTE: When adjusting or replacing parts, please refer to your Owner's Manual for assistance.


Canes (Walking)

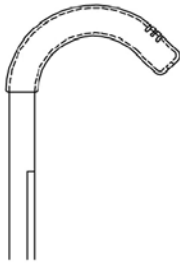


**Crook with
Foam Grip Shown**

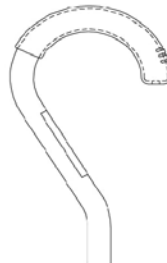

**Modified Crook with
Standard Grip shown**

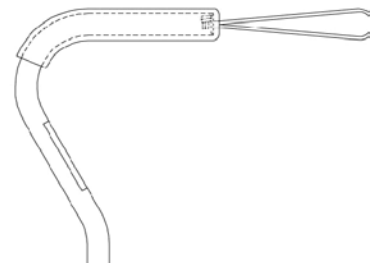

**Offset Crook with
Standard Grip and Strap Shown**

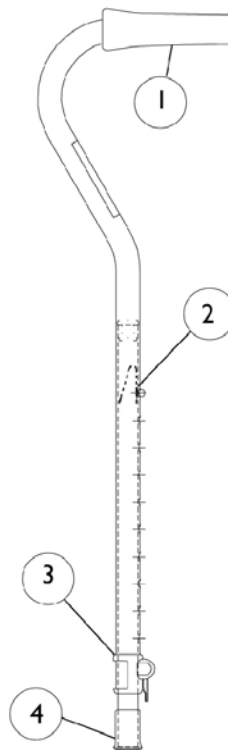

**Offset with Dual Cam Grip
Shown**

Canes (Walking)

Grip Style Chart

Model	Description	Grip	Grip Kits (Includes 10 Each)	Cane Color
8901	Custom Fit Crook - Foam	1032246 See Note(s) A	1130775-NLA	Silver
8902	Custom Fit Offset - Foam	1032246 See Note(s) A	1130775-NLA	Silver
8913	Standard Crook - Foam	1032246 See Note(s) A	1130775-NLA	Black
40915-6 (Discontinued)	Crook - Standard	1048091-NLA See Note(s) A	1130776	Silver
8916	Crook - Standard	1048091-NLA See Note(s) A	1130776	Black
8917	Offset Crook- w/Strap	1048094-NLA See Note(s) A	1130777	Silver
8931	Offset - Dual Cam	1062839-NLA See Note(s) A	1130778	Silver
8932	Offset - Dual Cam	1062839-NLA See Note(s) A	1130778	Black
40918-6 (Discontinued)	Offset - Dual Cam	1062839-NLA See Note(s) A	1130778	Black
8925	Modified Crook - Standard	1048091-NLA See Note(s) A	1130776	Silver
8926	Modified Crook - Foam	1032246 See Note(s) A	1130775-NLA	Black
8962	Crook - Variety Pack			
8964	Offset - Variety Pack			

NOTE: A - Individual items no longer available, refer to Grip Kits for replacement.


Canes (Walking)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	Grip Style Chart	1	1
2		40188X009	Button, Single Snap	1	1
3		1055948	Kit, Camlock	1	1
			Lever, Cam, Black	1	1
			Housing, Cam	1	1
			Instruction Sheet, Camlock	1	1
4	B,D	1065416	Tip, Rubber, Black (3/4" Tubing)	1	1
4	A,C	1086773-NLA	NLA - Tip, Rubber, Black (3/4" Tubing)	1	1
NOTE: A - Item no longer available, refer to Part #1128290. B - Fits all listed except 8901 and 8902. C - To be used on Models 8901 and 8902. D - After 10/1/08, appearance of the rubber tip has changed. New tips are functionally equivalent.					
NOTE: Items 2 and 3 will fit the Canes listed above EXCEPT Custom Fit Canes 8901 and 8902.					



Canes

Quad - Model 7823, 7827, 7829, 3920-2

Model 7823, Medium Base
Model 7829, Large Base

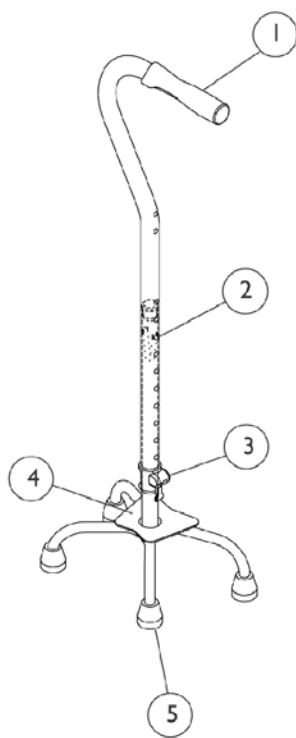

Model 3920- 2, Medium Base
Model 7827, Large Base

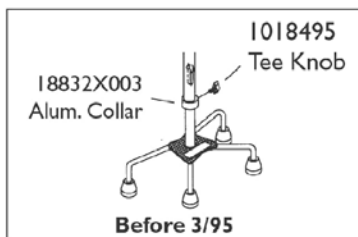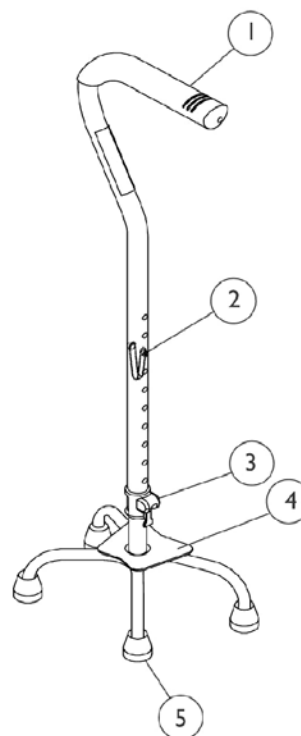

Canes

Quad - Model 7823, 7827, 7829, 3920-2

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B,H	1048091-NLA	NLA - Hand Grip, Standard	1	1
1	A,B	1130776	Kit, Standard Grips	10	1
1	C,D	1062839-NLA	NLA - Hand Grip, Dual Cam	1	1
1	C,D,I	1130778	Kit, Dual Cam Grips	10	1
2		1053041	Spring, Torsion w/ Double Buttons	1	1
3		1055948	Kit, Camlock	1	1
			Lever, Cam, Black	1	1
			Housing, Cam	1	1
			Instruction Sheet, Camlock	1	1
4	A,C,E	1052368-NLA	NLA - Base Assembly, Narrow	1	1
4	B,D,E	1052369-NLA	NLA - Base Assembly, Large	1	1
5	A,C,F,J	1049993-NLA	NLA - Tip, Cane, Rubber, Black (1/2" Tubing)	1	4
5	B,D,G	1049992	Tip, Cane, Rubber, Black (5/8" Tubing)	1	4
NOTE: A - To be used on Model 3920. B - To be used on Model 7827. C - To be used on Model 7823. D - To be used on Model 7829. E - Individual item no longer available, order complete cane for replacement. F - For 1/2" Tubing. G - For 5/8" Tubing. H - Individual item no longer available, refer to Kit Part #1130776. I - Individual item no longer available, refer to Kit Part #1130778. J - After 8/28/08 appearance of the rubber tip changed. New tips are functionally equivalent, but should be ordered in quantities of 4 if uniform appearance is required or desired.					

Bariatric Canes


8940-6, 7831-2, 7835-2



Model 8940-6



Model 7831-2

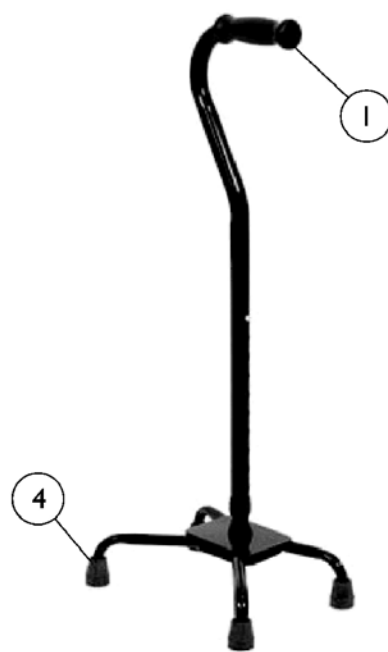

Model 7835-2

Bariatric Canes


8940-6, 7831-2, 7835-2

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1141736	Package, Grip	10	1
2	A	1141757	Package, Rubber Cane Tip, Black, Large (7/8")	10	1
3	B	1141737	Package, Rubber Cane Tip, Black, Small (1/2")	10	1
4	C	See Note	See Footnote	4	1
NOTE: A - For Model 8940-6 B - For Model 7831-2 C - For Model 7835-2, order part number 40853-4 Cane Tips (5/8") from the Personal Care Price List					

Canes

Composite Quad Cane

Model 7821 - Adult

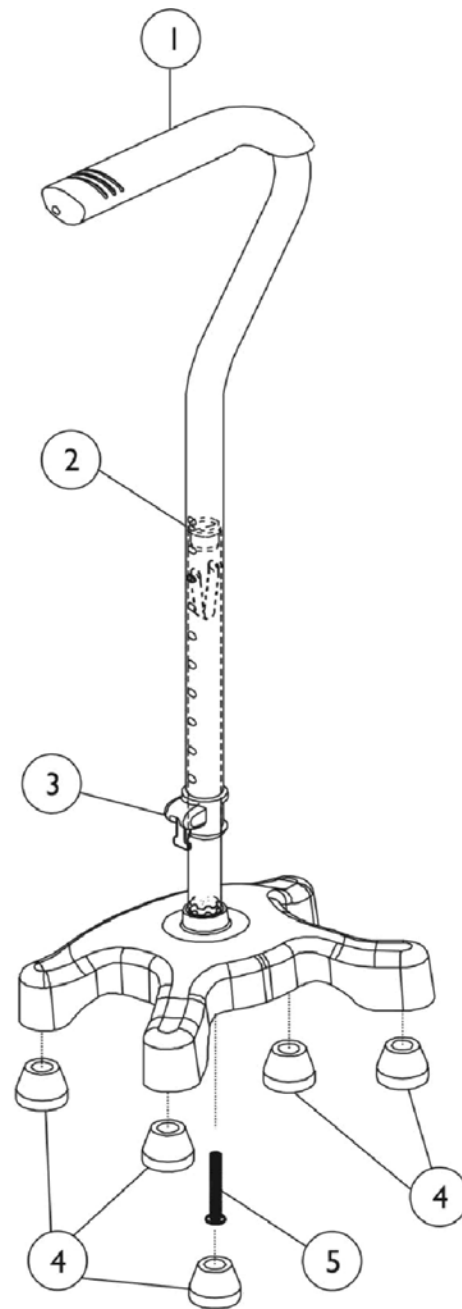

Canes

Model 7821 - Adult


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1048091-NLA	NLA - Hand Grip, Standard	1	1
1		1130776	Kit, Standard Grips	10	1
2		1053041	Spring, Torsion w/ Double Buttons	1	1
3		1055948	Kit, Camlock	1	1
			Lever, Cam, Black	1	1
			Housing, Cam	1	1
			Instruction Sheet, Camlock	1	1
4		B	1049993-NLA	NLA - Tip, Cane, Rubber, Black (1/2" Tubing)	1
5	18860X007		Screw, Phillips Pan Head (1/4-20 x 2-1/4")	1	1
NOTE: A - Individual item no longer available, refer to Kit Part #1130776. B - After 8/28/08 appearance of the rubber tip changed. New tips are functionally equivalent, but should be ordered in quantities of 5 if uniform appearance is required or desired.					

Canes


Orthopedic Grip and Folding Cane

Model 8928 or 8929

Single Side Shown

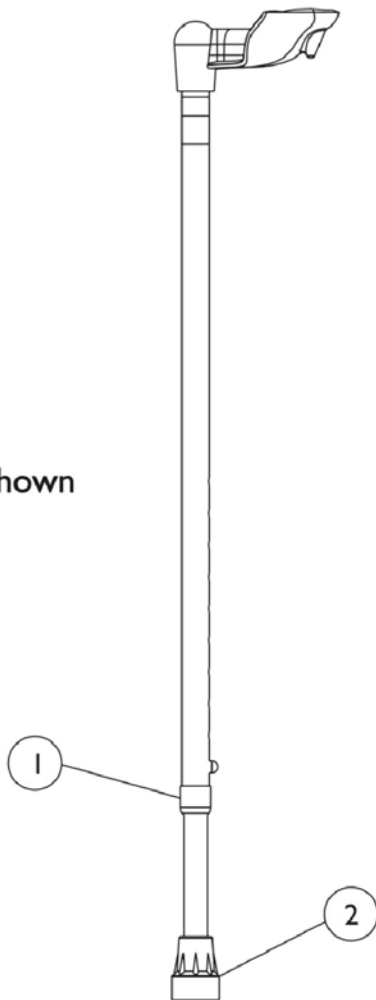

Model 40800-6

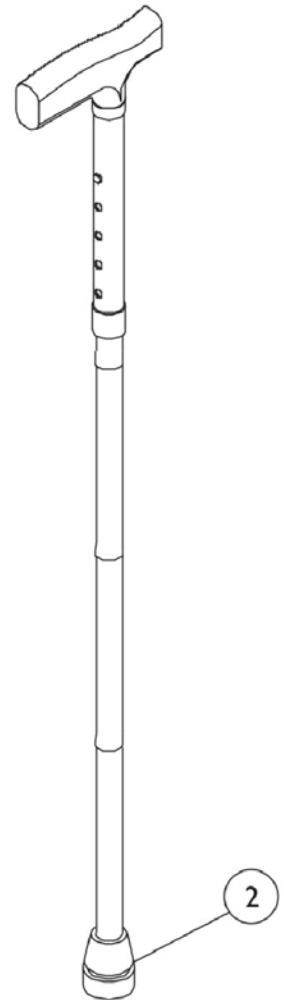

Canes

Model 8928, 8929 and Model 40800-6

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1052204-NLA	NLA - Clip, Spring	1	1
2	A,B,D	1049735-NLA	NLA - Tip, Rubber, Black (3/4" Tubing)	1	1
2	C,E	1065416	Tip, Rubber, Black (3/4" Tubing)	1	1
NOTE: A - To be used with Model 8928 and 8929. B - Due to style changes, when replacing rubber tip, measure inside diameter. C - To be used with Model 40800. D - Item no longer available, refer to part #1048113 for replacement. E - After 10/1/08, appearance of the rubber tip has changed. New tips are functionally equivalent.					


Crutches

Forearm - Model 6152, 6153, 6154

Model 6152 - Junior

Model 6153 - Adult

Model 6154 - Tall Adult

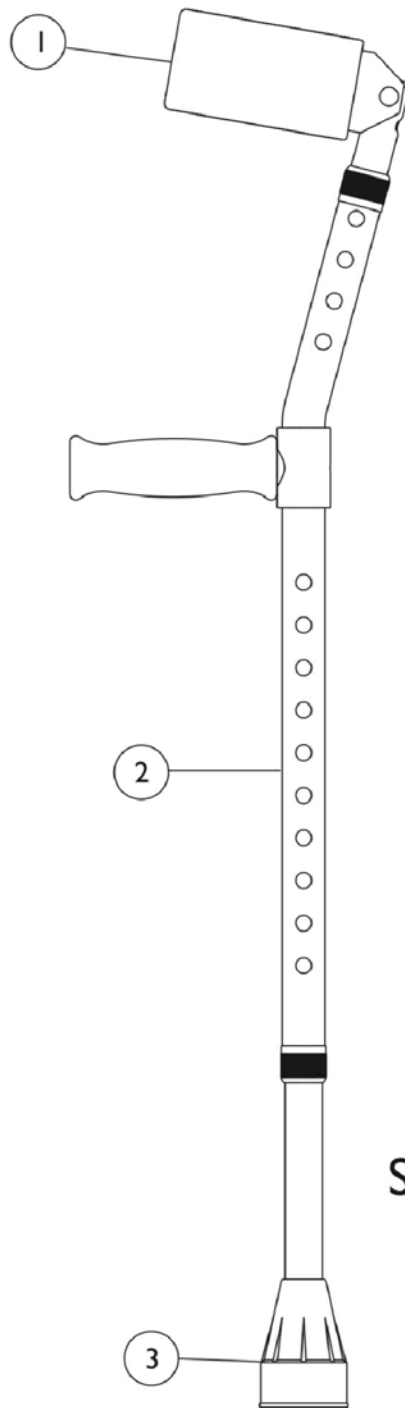

Single Side Shown

Crutches

Forearm - Model 6152, 6153, 6154

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1049537-NLA	NLA - Cuff and Support Sub-Assembly, Double Snap, Black	1	1
1	A	1095678-NLA	NLA - Cuff and Support Sub-Assembly, Single Snap, Grey	1	1
1	B	1088819-NLA	NLA - Cuff and Support Sub-Assembly, Double Snap, Grey	1	1
2		1049536-NLA	NLA - Button, Snap, Double and Spring Wire	1	1
3		1049538-NLA	NLA - Tip, Rubber, Black (7/8" Tubing)	1	1
3		6135	Kit, Tips Extra Large Rubber, Grey (2" O.D. Base)	1	1
			Tip, Crutch, Grey - Extra Large	2	1
NOTE: A - For models built before 8/1/99. B - For models built after 8/1/99.					

Forearm Crutches

Model 8153J, 8153T and 8153A




Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1168396	Cuff w/ Hardware (Pair)	2	1
2	B	1168397	Cane w/ Hardware (Pair)	2	1
3		1168398	Kit, Cuff and Cane w/ Hardware (Pair)	2	1
NOTE: A - Includes (2) Cuffs, (2) Screws, and (2) Locknuts. B - Includes (2) Canes, (2) Screws, and (2) Locknuts.					

Crutches

Quick Change, Quick Adjust and Bariatric Crutch


Quick Change

Model 8115-A - Adult
Model 8115-J - Junior
Model 8115-T - Tall Adult

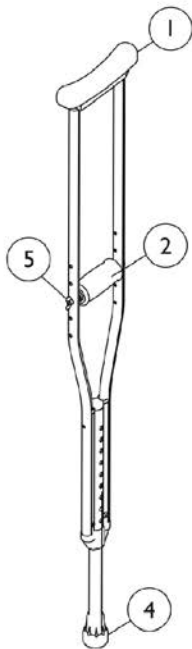


Quick Adjust

Model 8120-A - Adult
Model 8120-J - Junior
Model 8120-T - Tall Adult

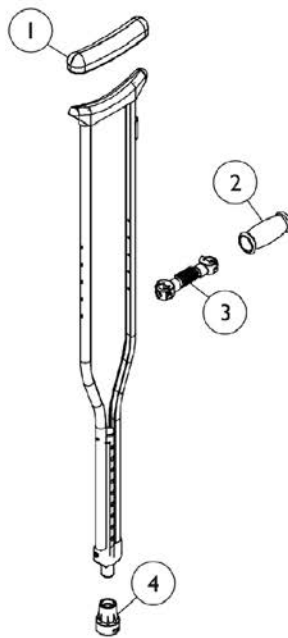

Bariatric Crutch

Model 8130-A - Adult
Model 8130-J - Junior
Model 8130-T - Tall Adult

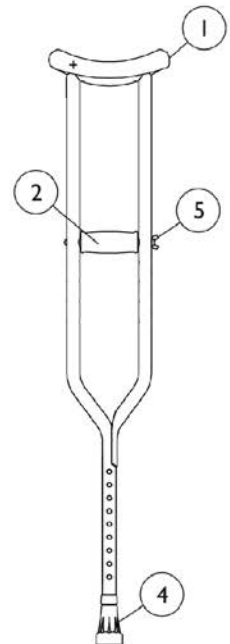

Crutches

Quick Change, Quick Adjust and Bariatric Crutch


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B,F	6131	Kit, Arm Pad, Grey	1	1
			Pad, Arm, Grey	2	1
1	C,F	7131	Kit, Arm Pad, Grey	1	1
			Pad, Arm, Grey	2	1
2	A,G	6119	Kit, Hand Grip Open, Grey	1	1
			Grip, Hand, Grey, Open	2	1
2	B,E,G	6124	Kit, Hand Grip Open, Grey	1	1
			Grip, Hand, Grey	2	1
2	B,G	6125	Kit, Hand Grip Solid, Grey	1	1
			Grip, Hand, Grey	2	1
2	A,G	6126	Kit, Hand Grip Solid, Grey	1	1
			Grip, Hand, Grey, Closed	2	1
2	C,G	7126	Kit, Hand Grip Solid, Grey	1	1
			Grip, Hand, Grey	2	1
3	B	1098970	Kit, Hand Grip, Grey	1	1
			Grip, Hand, Grey	1	1
			Handle, Crutch	1	1
4	A,B,D, H	6134	Kit, Tips Large Rubber, Grey (1-3/4" O.D. Base)	1	1
			Tip, Crutch, Grey - Large	2	1
4	C,H	6135	Kit, Tips Extra Large Rubber, Grey (2" O.D. Base)	1	1
			Tip, Crutch, Grey - Extra Large	2	1
5	I	1106801-NLA	NLA - Nut, Wing (1/4-20)	1	2
NOTE: A - To be used with Model 8115. B - To be used with Model 8120. C - To be used with Model 8130. D - Large rubber tips are standard for the Crutches. E - Has a slit for easy removal. F - Kit includes (2) Arm Pads. G - Kit includes (2) Hand Grips. H - Kit includes (2) Tips. I - Item no longer available, no replacement is available.					

Walker (Pediatric)

Sportsrunner

Model 6210R, 6210B, 6210P - Small

Model 6211R, 6211B, 6211P - Large

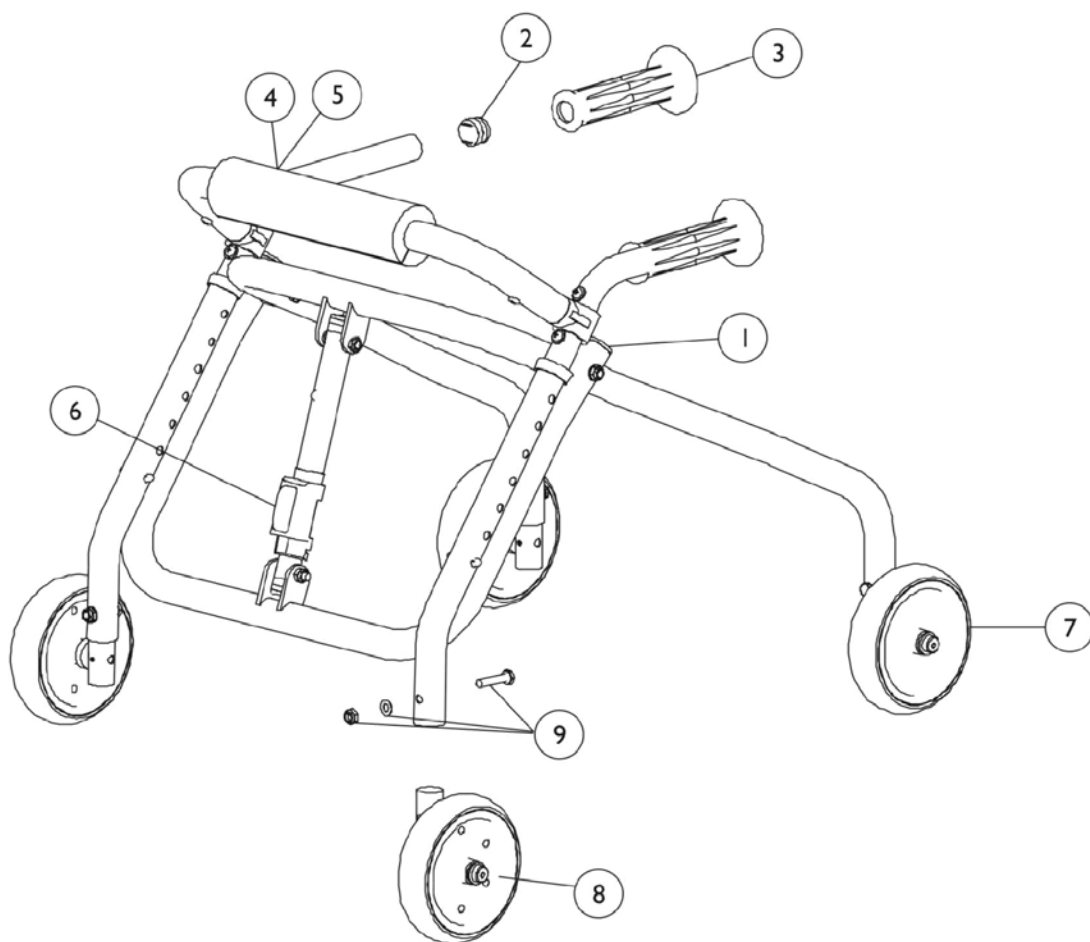

Walker (Pediatric)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1029501	Plug Button, Black (1")	1	2
2		1027095	Plug Button, Black (7/8")	1	3
3		1099924-NLA	NLA - Grip, Hand, Black	2	1
4		1098977-NLA	NLA - Pad, Foam	1	1
5		1099910-NLA	NLA - Cover, Nylon - U240	1	1
6	A	1099925	NLA - Kit, Folding Lock Assembly	1	1
			Tube, Lock Outside	1	1
			Tube, Lock Inside	1	1
			Plug Button, Black (7/8")	1	1
			Plug Button, Black (3/4")	1	1
			Housing, Button	1	1
			Button, Release	1	1
			Locknut (1/4-20)	1	2
			Washer, Nylon (1/4 x 1/2 x 1/16")	1	2
			Screw, Hex Head (1/4-20 x 1-1/2")	1	2
			Button, Snap Ramp	1	1
			Button, Snap	1	1
6	B	1099929	NLA - Kit, Folding Lock Assembly	1	1
			Tube, Lock Outside, Large	1	1
			Tube, Lock Inside, Large	1	1
			Plug Button, Black (7/8")	1	1
			Plug Button, Black (3/4")	1	1
			Housing, Button	1	1
			Button, Release	1	1
			Locknut (1/4-20)	1	2
			Washer, Nylon (1/4 x 1/2 x 1/16")	1	2
			Screw, Hex Head (1/4-20 x 1-1/2")	1	2
			Button, Snap Ramp	1	1
			Button, Snap	1	1
7		6212	Kit, Wheel One-Way	1	1
			Insert, Wheel	1	2
			NLA - Washer, Nylon (13/32 x 1 x 1/4")	1	2
			Screw, Shoulder Hex Socket (3/8 x 2", 5/16-18)	1	2
			Washer, Nylon (3/8 x 3/4 x 1/16")	1	2
			Screw, Socket Head (8-32 X 3/8")	1	2
			Wheel (5")	1	2
			Locknut (1/4-20)	1	2
			Washer (1/4 x 1/2 x 1/32")	1	2
			Screw, Hex Head (1/4-20 x 1-1/4")	1	2
7		1102915-NLA	Kit, Wheel One-Way - Right	1	1
			NLA - Washer, Nylon (13/32 x 1 x 1/4")	1	1
			Screw, Shoulder Hex Socket (3/8 x 2", 5/16-18)	1	1
			Washer, Flat (3/8 x 11/16 x 1/16")	1	1

Walker (Pediatric)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
7		1102916-NLA	Screw, Set Socket Cup Point (#8-32 x 1/4")	1	1
			Wheel (5")	1	1
			Locknut (1/4-20)	1	1
			Washer (1/4 x 1/2 x 1/32")	1	1
			Screw, Hex Head (1/4-20 x 1-1/4")	1	1
			Insert, Wheel	1	1
			Kit, Wheel One-Way - Left	1	1
			NLA - Washer, Nylon (13/32 x 1 x 1/4")	1	1
			Screw, Shoulder Hex Socket (3/8 x 2", 5/16-18)	1	1
			Washer, Flat (3/8 x 11/16 x 1/16")	1	1
			Screw, Set Socket Cup Point (#8-32 x 1/4")	1	1
			Wheel (5")	1	1
			Locknut (1/4-20)	1	1
			Washer (1/4 x 1/2 x 1/32")	1	1
8		6213	Screw, Hex Head (1/4-20 x 1-1/4")	1	1
			Insert, Wheel	1	1
			Kit, Wheel Two-Way	1	1
			Insert, Wheel	1	2
			NLA - Washer, Nylon (13/32 x 1 x 1/4")	1	2
			Screw, Shoulder Hex Socket (3/8 x 2", 5/16-18)	1	2
			Washer, Nylon (3/8 x 3/4 x 1/16")	1	2
			Screw, Socket Head (8-32 X 3/8")	1	2
			Bushing (15/32 x 3/8")	1	2
			Wheel (5" x 1")	1	2
			Locknut (1/4-20)	1	2
			Washer (1/4 x 1/2 x 1/32")	1	2
			Screw, Hex Head (1/4-20 x 1-1/4")	1	2
			NLA - Kit, Wheel Two-Way - Right/Left	1	2
9		1099926-NLA	NLA - Kit, Wheel Hardware	1	1
			Screw, Hex Head (1/4-20 x 1-1/4")	1	1
			Washer (1/4 x 1/2 x 1/32")	1	1
			Locknut (1/4-20)	1	1
		6214	Kit, Rubber Tip Assembly	1	1
			Bolt, Curved Head (1/4-20 x 1-1/4")	1	2
			Washer (1/4 x 1/2 x 1/32")	1	2
			Locknut (1/4-20)	1	2
		1099931-NLA	NLA - Tip, Rubber, Black (3/4" Tubing)	1	2
			Insert, Wheel	1	2
			NLA - Package, Rubber Tip	2	4
			NLA - Paint, Touch-Up, 17P Cobalt Blue	1	1
		1031455-NLA	NLA - Paint, Touch-Up, 33P Red	1	1
		1031456-NLA	NLA - Paint, Touch-Up, 08P Deep Purple	1	1

Walker (Pediatric)


NOTE: A - To be used with Model 6210's.
B - To be used with Model 6211's.
C - Not Shown.

Walker (Single Release)

Model 6281-A (4 Pack)

Model 6281-I (Adult Single Pack)

Model 6281-JR (4 Pack)

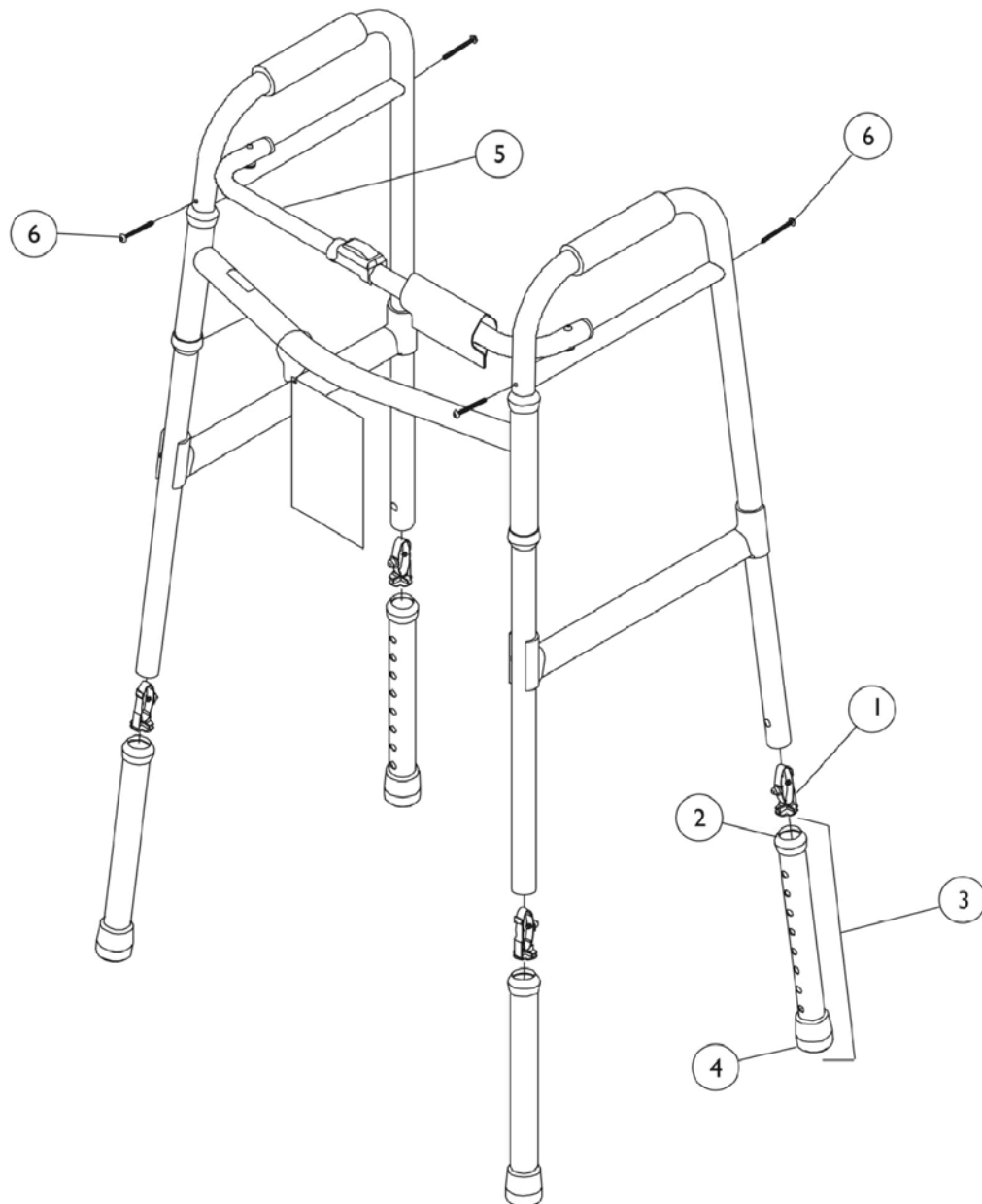

Walker (Single Release)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C	1074462-NLA	NLA - Button, Snap	1	4
1	D	1130780-NLA	NLA - Kit, Snap Buttons	10	1
1		1094945	Button, Snap	1	4
2		1040655	Anti-Rattle, Grey (1-1/8")	1	4
3	A,B	1058000	Leg Extension Assembly (10")	1	2
4		1040656	Tip, Rubber, Dark Grey (1-1/8")	1	4
5		1060558-NLA	NLA - Folding Mechanism - Adult	1	1
5		1060559-NLA	NLA - Folding Mechanism - Junior	1	1
6		17564X000-NLA	NLA - Screw, Phillips Pan Head (#10-24 x 1-3/4")	1	4
NOTE: A - Due to style changes, when replacing Leg Extensions before 6/25/98, order (4) each. Models built after 6/24/98 can order Single Assemblies. B - Includes Leg and Items 2 and 4. C - Individual item no longer available, use 1130780 as its replacement. D - Kit is No Longer Available, use 1094945 as the replacement.					

Single Release Walker

(Model 6285-A, 6285-3F)




Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1145095-NLA	NLA - Package, Tip	4	1

Walker (Single Release)

With 3" Fixed Wheel and Glide Tips
Models 6281-3F (Discontinued 6/07)

Model 6281-3F (4 Pack)

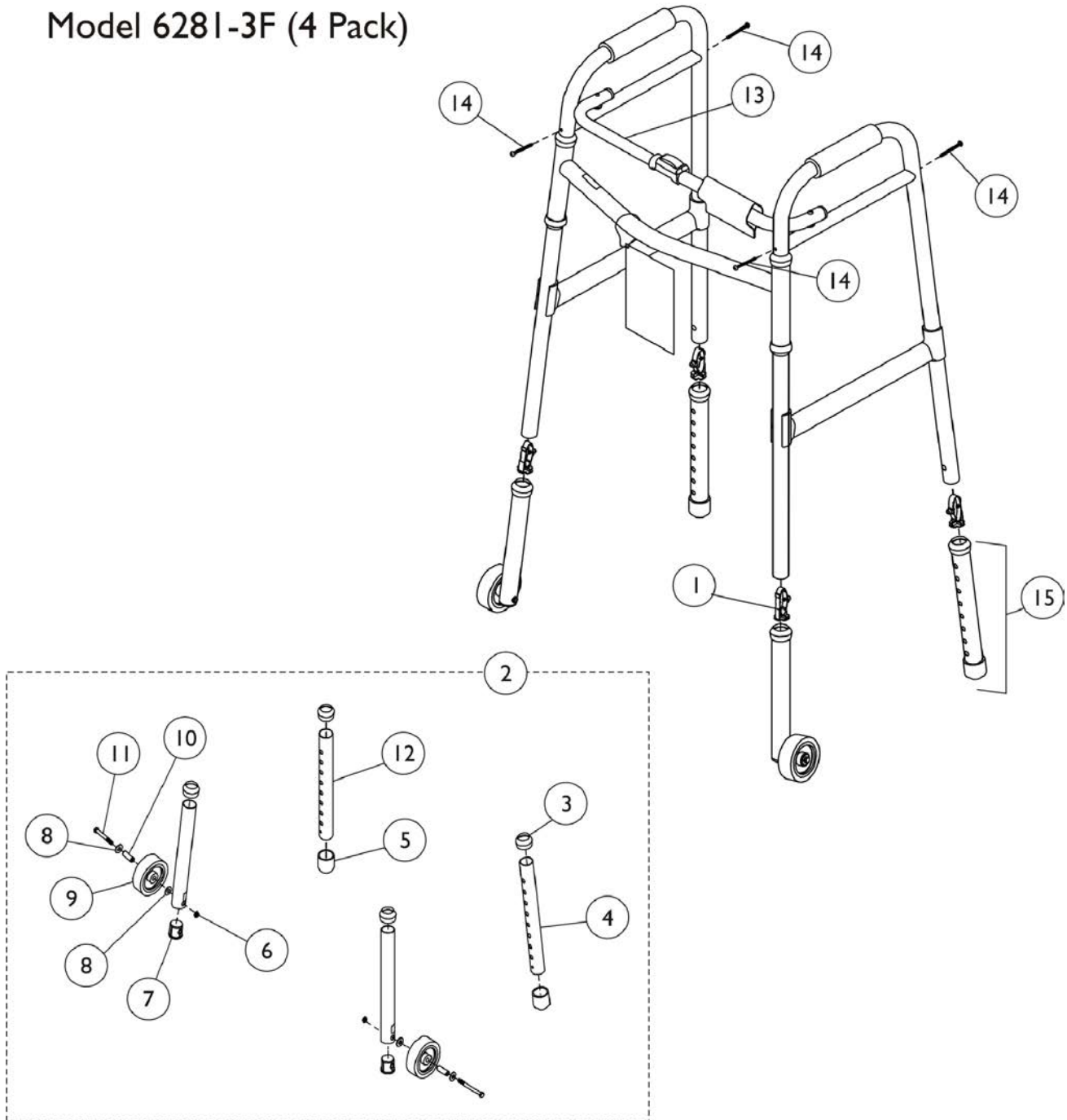

Walker (Single Release)

With 3" Fixed Wheel and Glide Tips


Models 6281-3F (Discontinued 6/07)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E	1074462-NLA	NLA - Button, Snap	1	4
1	F	1130780-NLA	NLA - Kit, Snap Buttons	10	1
1		1094945	Button, Snap	1	4
2	A,D	1060590	Kit, Leg Extension (3"), Fixed	1	1
3			Anti-Rattle, Grey (1-1/8")	1	4
4			Leg Extension Tube with Axle Hole	1	2
5			NLA - Tip, Glide, Grey (1-1/8")	1	2
6			Locknut (1/4-20)	1	2
7			Bushing (1")	1	2
8			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	4
9			NLA - Wheel (3" x 1")	1	2
10			Bushing (1/4 x 3/8 x 1/13")	1	2
11			Screw, Hex Head (1/4-20 x 2-1/2")	1	2
12			NLA - Leg, Adjustable	1	2
13		1060558-NLA	NLA - Folding Mechanism - Adult	1	1
13		1060559-NLA	NLA - Folding Mechanism - Junior	1	1
14		17564X000-NLA	NLA - Screw, Phillips Pan Head (#10-24 x 1-3/4")	1	4
15	B,C	1058000	Leg Extension Assembly (10")	1	2
NOTE: A - Includes (2) of each style. B - Due to style changes, when replacing Leg Extensions before 6/25/98, order (4) each. Models built after 6/24/98 can order Single Assemblies. C - Includes Leg, Rubber Tip and Anti-Rattle. D - Kit Includes Items 3 - 12. E - Individual item no longer available, use 1130780 as its replacement. F - Kit is no longer available, use 1094945 as its replacement.					

Walker (Blue-Release)

Model 6241 - A Adult Walker (4 Pack) - Discontinued

Model 6241 - Jr. Junior Walker (4 Pack)

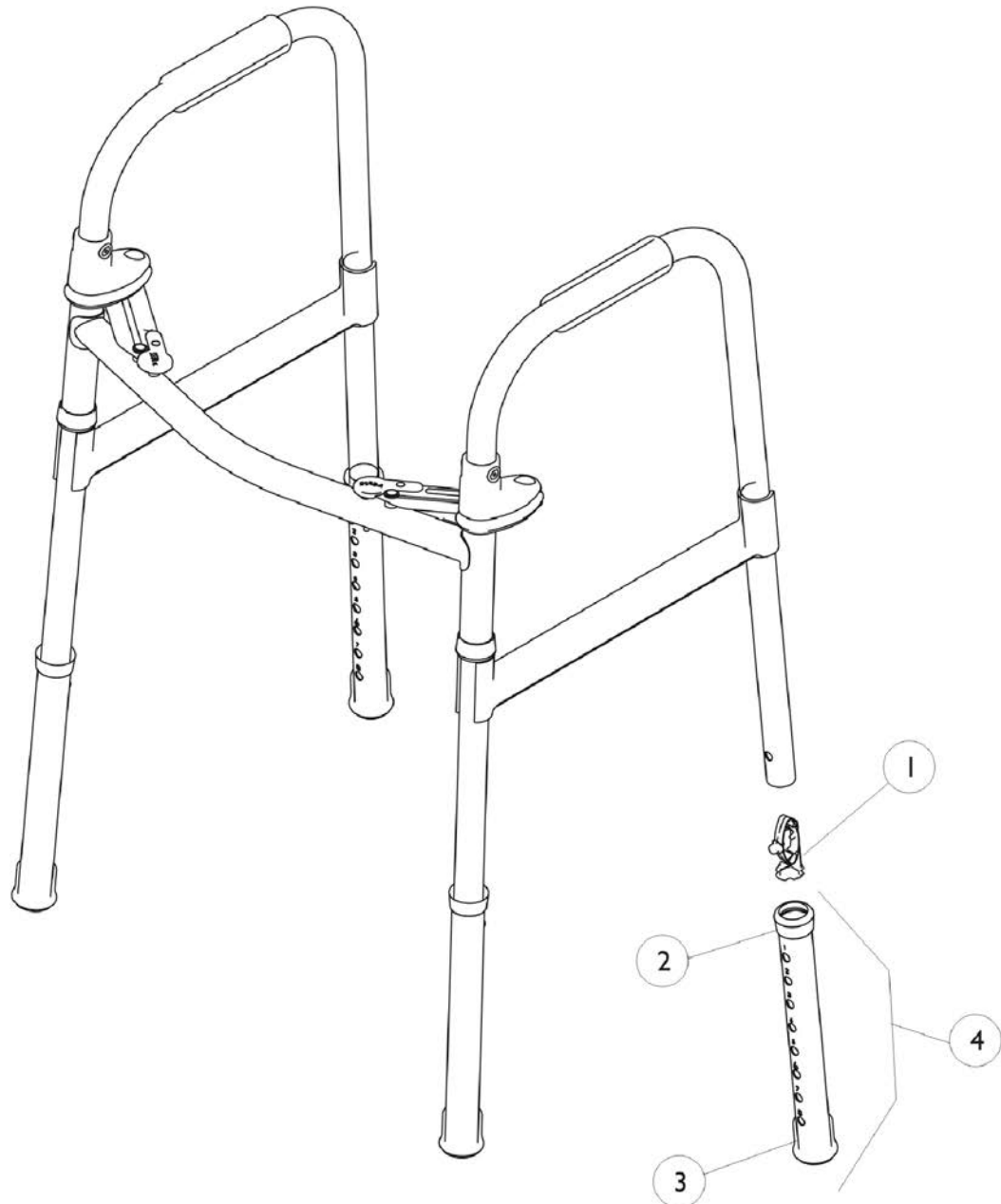

Walker (Blue-Release)


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1074462-NLA	NLA - Button, Snap	1	4
1	C	1130780-NLA	NLA - Kit, Snap Buttons	10	1
1		1094945	Button, Snap	1	4
2		1040655	Anti-Rattle, Grey (1-1/8")	1	4
3		1040656	Tip, Rubber, Dark Grey (1-1/8")	1	4
4	A	1058000	Leg Extension Assembly (10")	1	4
NOTE: A - Includes Leg and Items 2 and 3. B - Individual item no longer available, use 1130780 as its replacement. C - Kit is no longer available, use 1094945 as its replacement.					

Walker (Blue-Release)

Models (6241-5F, 6241-JR, and 6241-JR5F - Discontinued 6/07)

Model 6241-5F (4 Pack)

Model 6241-JR5F (4 Pack)

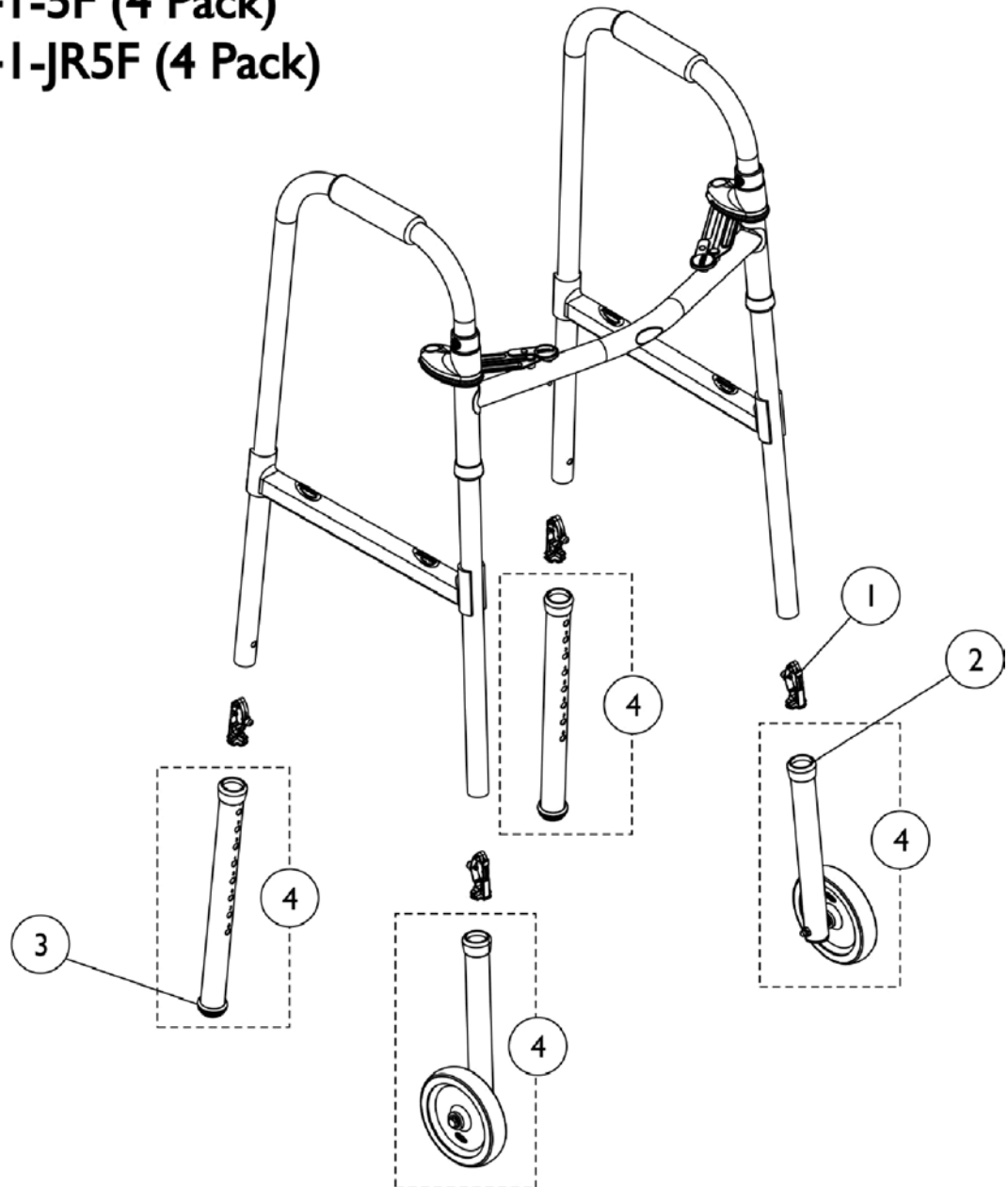

Walker (Blue-Release)


Models (6241-5F, 6241-JR, and 6241-JR5F - Discontinued 6/07)

Model 6241-5F and 6241-JR5F


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1074462-NLA	NLA - Button, Snap	1	4
1	C	1130780-NLA	NLA - Kit, Snap Buttons	10	1
1		1094945	Button, Snap	1	4
2		1040655	Anti-Rattle, Grey (1-1/8")	1	2
3	B	1041531-NLA	NLA - Tip, Glide, Grey (1-1/8")	1	2
3		1130781	Kit, Glide Tips	10	1
4		1086797	Kit, Leg Extension (5"), Fixed	1	1
			Leg Extension Tube with Axle Hole	1	2
			Bushing (1")	1	2
			Wheel (5" x 1-1/4")	1	2
			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	6
			Bushing (1/4 x 1-3/8")	1	2
			Screw, Hex Head (1/4-20 x 2-3/4")	1	2
			Anti-Rattle, Grey (1-1/8")	1	4
			Locknut (1/4-20)	1	2
			Tube, Leg Extension (13")	1	2
			NLA - Tip, Glide, Grey (1-1/8")	1	2
NOTE: A - Individual item no longer available, use 1130780 as its replacement. B - Individual item no longer available, use 1130781 as its replacement. C - Kit is no longer available, use 1094945 as its replacement.					

Blue-Release Walker

(Model 6240-A, 6240-5F & 6240-JR5F)

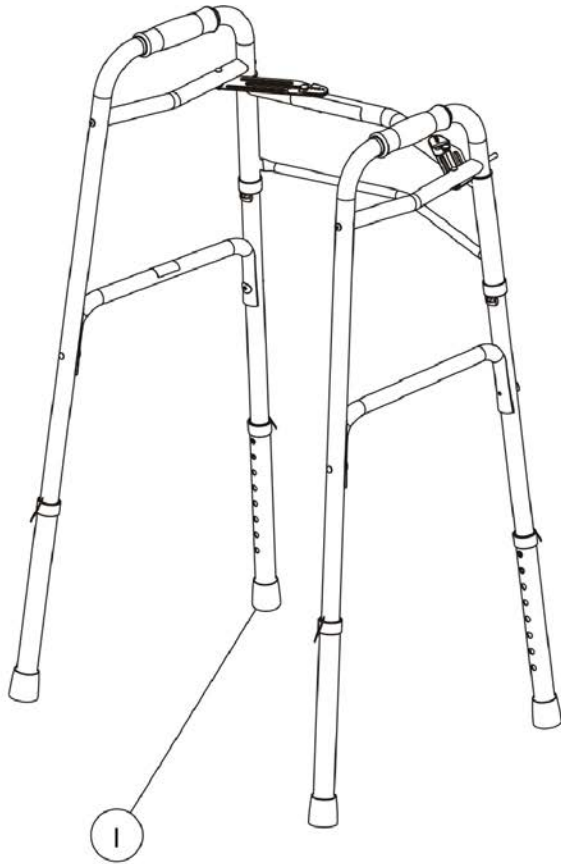

6240-A

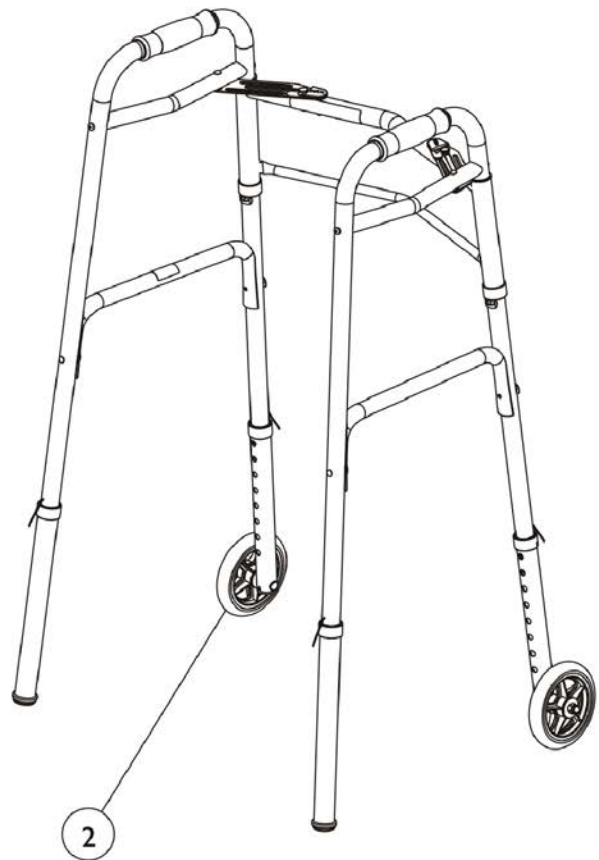

6240-5F & 6240-JR5F


Blue-Release Walker

(Model 6240-A, 6240-5F & 6240-JR5F)


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1148146	Package, Tip	10	1
2	A,B	See Note	See Footnote	--	--
NOTE: A - For replacement parts, see category titled, "Fixed Wheel Attachment - Single with Glide Tips", and reference Model 6271 replacement parts. B - See Price List for complete assembly, Model 6271, when needed.					

Walker (Dual Release)

Model 6261 - A - Adult Walker (4 Pack)
Model 6261 - JR (4 Pack)

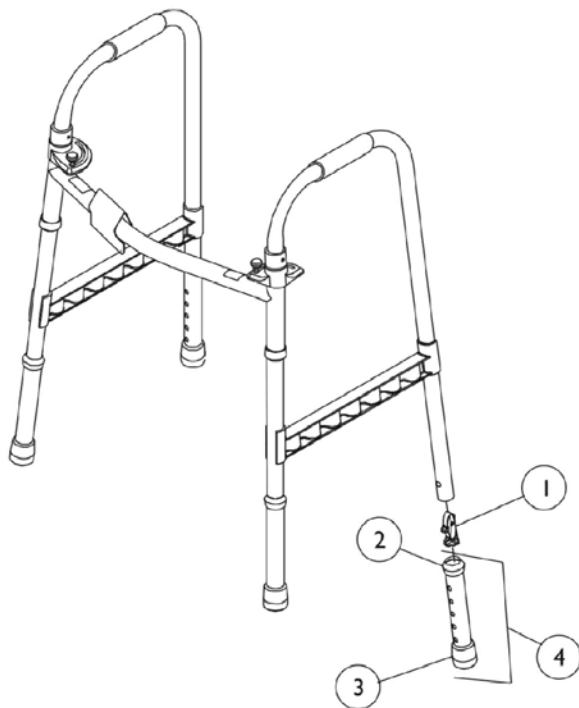

Model 6291 - A (4 Pack)
Model 6291 - I (Adult Single Pack)
Model 6291 - JR (4 Pack)



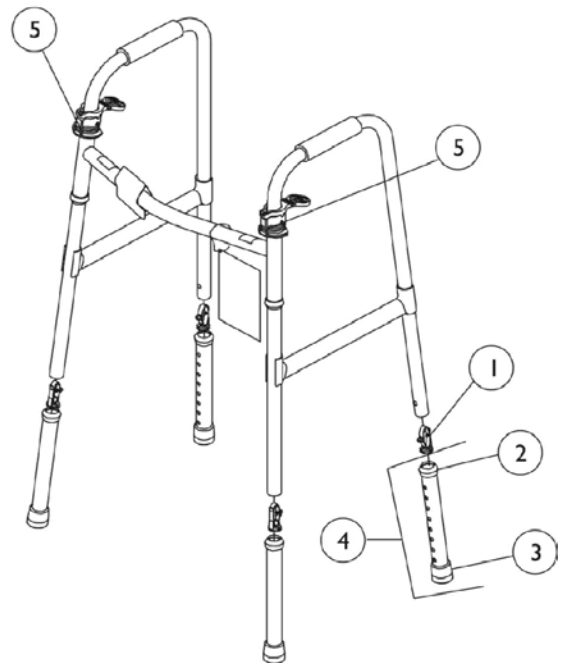


Walker (Dual Release)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D	1074462-NLA	NLA - Button, Snap	1	4
1	F	1130780-NLA	NLA - Kit, Snap Buttons	10	1
1		1094945	Button, Snap	1	4
2		1040655	Anti-Rattle, Grey (1-1/8")	1	4
3		1040656	Tip, Rubber, Dark Grey (1-1/8")	1	4
4	A,B	1058000	Leg Extension Assembly (10")	1	4
5	C,E	1043538	Nut, Push	1	4
5		1130784	Kit, Push Nut	10	1
NOTE: A - Due to style changes, when replacing Leg Extensions before 6/25/98, Order (4) each. Models built after 6/24/98 can order Single Assemblies. B - Includes Leg and Items 2 and 3. C - To be used with Models 6291-A, 6291-1 and 6291 Jr. D - Individual item no longer available, use 1130780 as its replacement. E - Individual item no longer available, use 1130784 as its replacement. F - Kit is no longer available, use 1094945 as its replacement.					

With 3" or 5" Fixed Wheel and Glide Tips

Model 629I-3F (4 Pack)
Model 629I-JR3F (4 Pack)
Model 629I-5F (4 Pack)
Model 629I-JR5F (4 Pack)

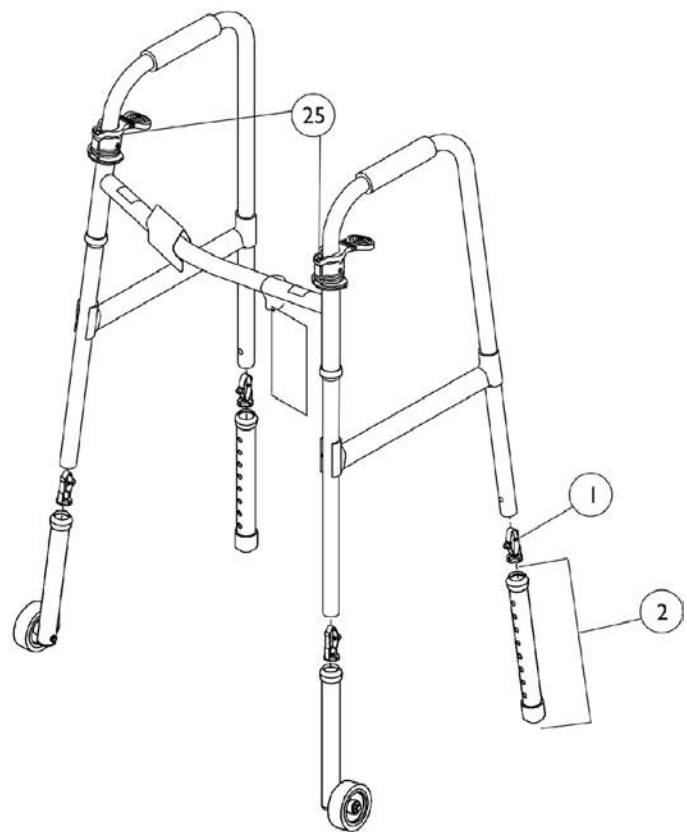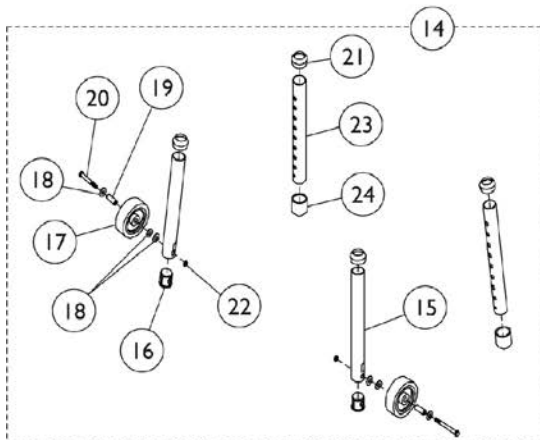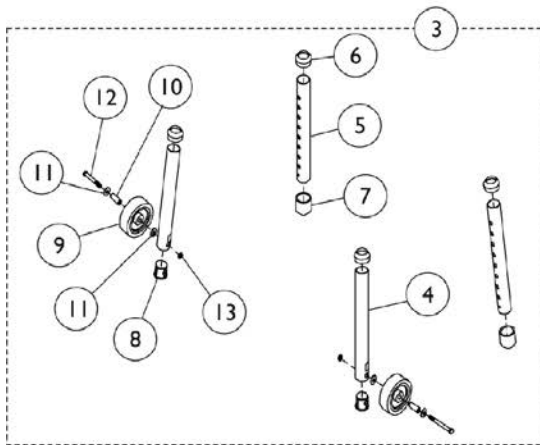

Walker (Dual Release)


With 3" or 5" Fixed Wheel and Glide Tips

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	K	1074462-NLA	NLA - Button, Snap	1	4
1	M	1130780-NLA	NLA - Kit, Snap Buttons	10	1
1		1094945	Button, Snap	1	4
2	A,B,C, D	1058000	Leg Extension Assembly (10")	1	2
2	B,E	6266	Leg Extentsion Assembly (13")	2	1
3	C,F,I	1060590	Kit, Leg Extension (3"), Fixed	1	1
4			Leg Extension Tube with Axle Hole	1	2
5			NLA - Leg, Adjustable	1	2
6			Anti-Rattle, Grey (1-1/8")	1	4
7			NLA - Tip, Glide, Grey (1-1/8")	1	2
8			Bushing (1")	1	2
9			NLA - Wheel (3" x 1")	1	2
10			Bushing (1/4 x 3/8 x 1/13")	1	2
11			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	4
12			Screw, Hex Head (1/4-20 x 2-1/2")	1	2
13			Locknut (1/4-20)	1	2
14	E,G,H, J	1086797	Kit, Leg Extension (5"), Fixed	1	1
15			Leg Extension Tube with Axle Hole	1	2
16			Bushing (1")	1	2
17			Wheel (5" x 1-1/4")	1	2
18			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	6
19			Bushing (1/4 x 1-3/8")	1	2
20			Screw, Hex Head (1/4-20 x 2-3/4")	1	2
21			Anti-Rattle, Grey (1-1/8")	1	4
22			Locknut (1/4-20)	1	2
23			Tube, Leg Extension (13")	1	2
24			NLA - Tip, Glide, Grey (1-1/8")	1	2
25	L	1043538	Nut, Push	1	4
25		1130784	Kit, Push Nut	10	1

NOTE: A - Due to style changes, when replacing Leg Extensions before 6/25/98, Order (4) each. Models built after 6/24/98 can order Single Assemblies.
 B - Includes: Leg, Anti-Rattle and Glide Tips.
 C - To be used with Model 6291-3F.
 D - To be used with Model 6291-JR3F.
 E - To be used with Model 6291-5F.
 F - To be used with 3" Wheel.
 G - To be used with 5" Wheel.
 H - Due to Wheel style change, order in pairs after 2/7/02.
 I - Kit Includes Items 4-13.
 J - Kit Includes Items 15-25.
 K - Individual item no longer available, use 1130780 as its replacement.
 L - Individual item no longer available, use 1130784 as its replacement.
 M - Kit is no longer available, use 1094945 as its replacement.

Walker (Dual Release, Heavy Duty)


Model 6291 - HDA (2 Pack)

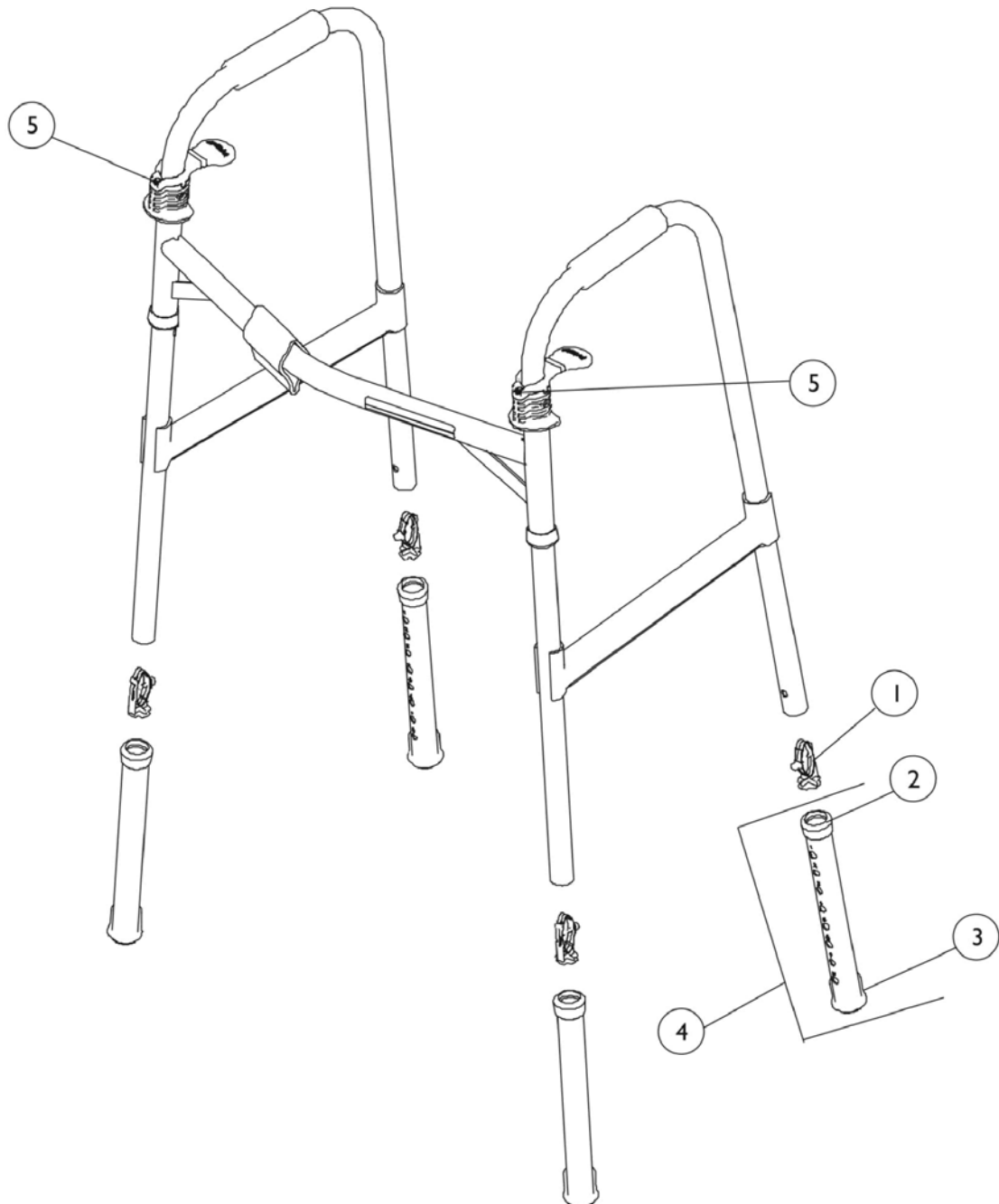

Walker (Dual Release, Heavy Duty)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1074462-NLA	NLA - Button, Snap	1	4
1	D	1130780-NLA	NLA - Kit, Snap Buttons	10	1
1		1094945	Button, Snap	1	4
2		1040655	Anti-Rattle, Grey (1-1/8")	1	4
3		1040656	Tip, Rubber, Dark Grey (1-1/8")	1	4
4	A	1058000	Leg Extension Assembly (10")	1	4
5	C	1043538	Nut, Push	1	4
5		1130784	Kit, Push Nut	10	1
NOTE: A - Includes Leg and Items 2-3. B - Individual item no longer available, use 1130780 as its replacement. C - Individual item no longer available, use 1130784 as its replacement. D - Kit is no longer available, use 1094945 as its replacement.					

Walker (Dual Release Bariatric - Model 6441-A)

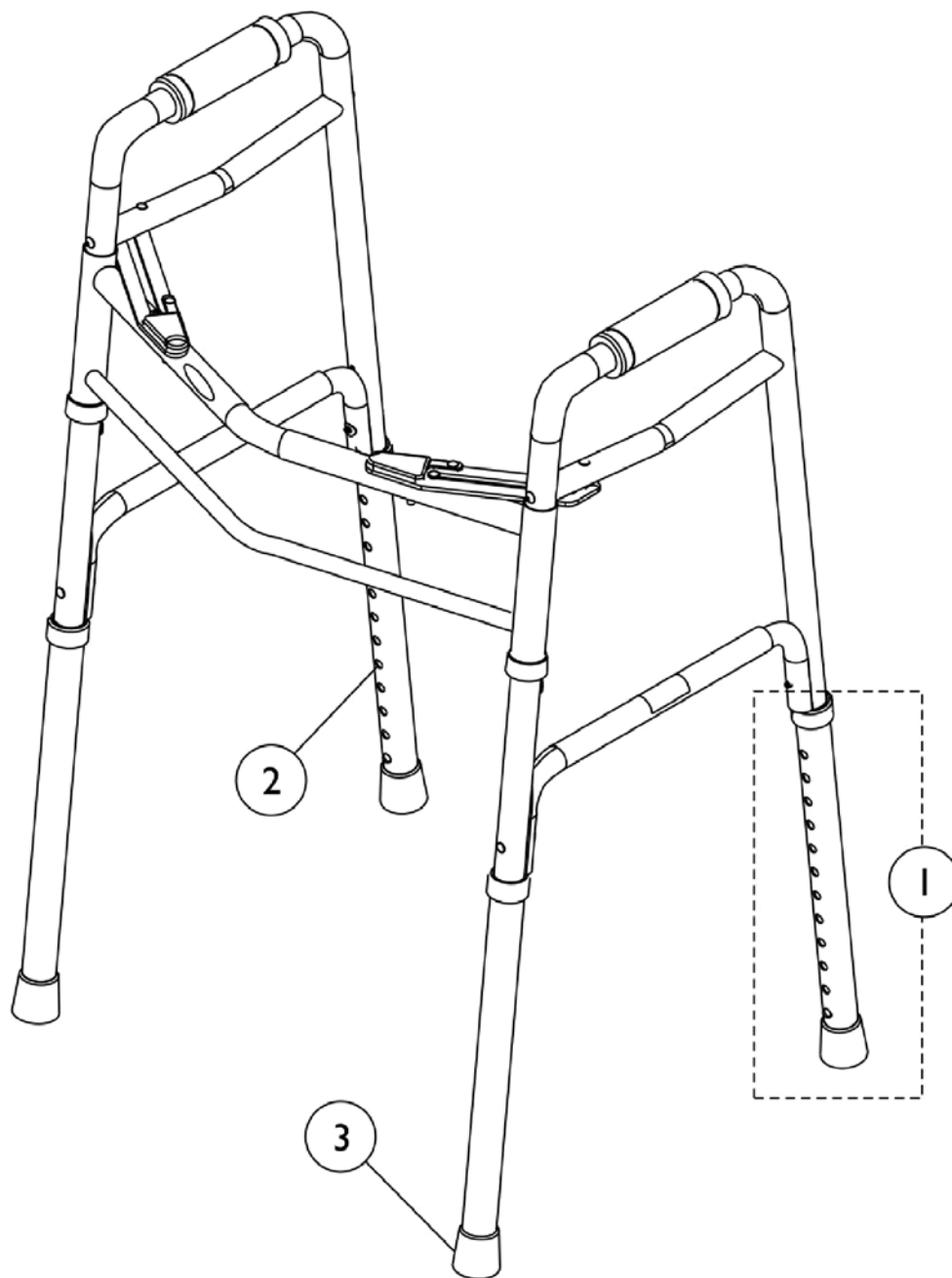


Walker (Dual Release Bariatric - Model 6441-A)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1141745	Leg Extension Assembly	1	4
2		1141746-NLA	NLA - Package, Snap Button	10	1
3		1141747-NLA	NLA - Package, Tips	10	1
3		1148146	Package, Tip	10	1
Replacement parts for the Bariatric Wheel Kit are non-serviceable and must be ordered complete.					

Walker (Hemi)

Model 6252 (Single Pack)



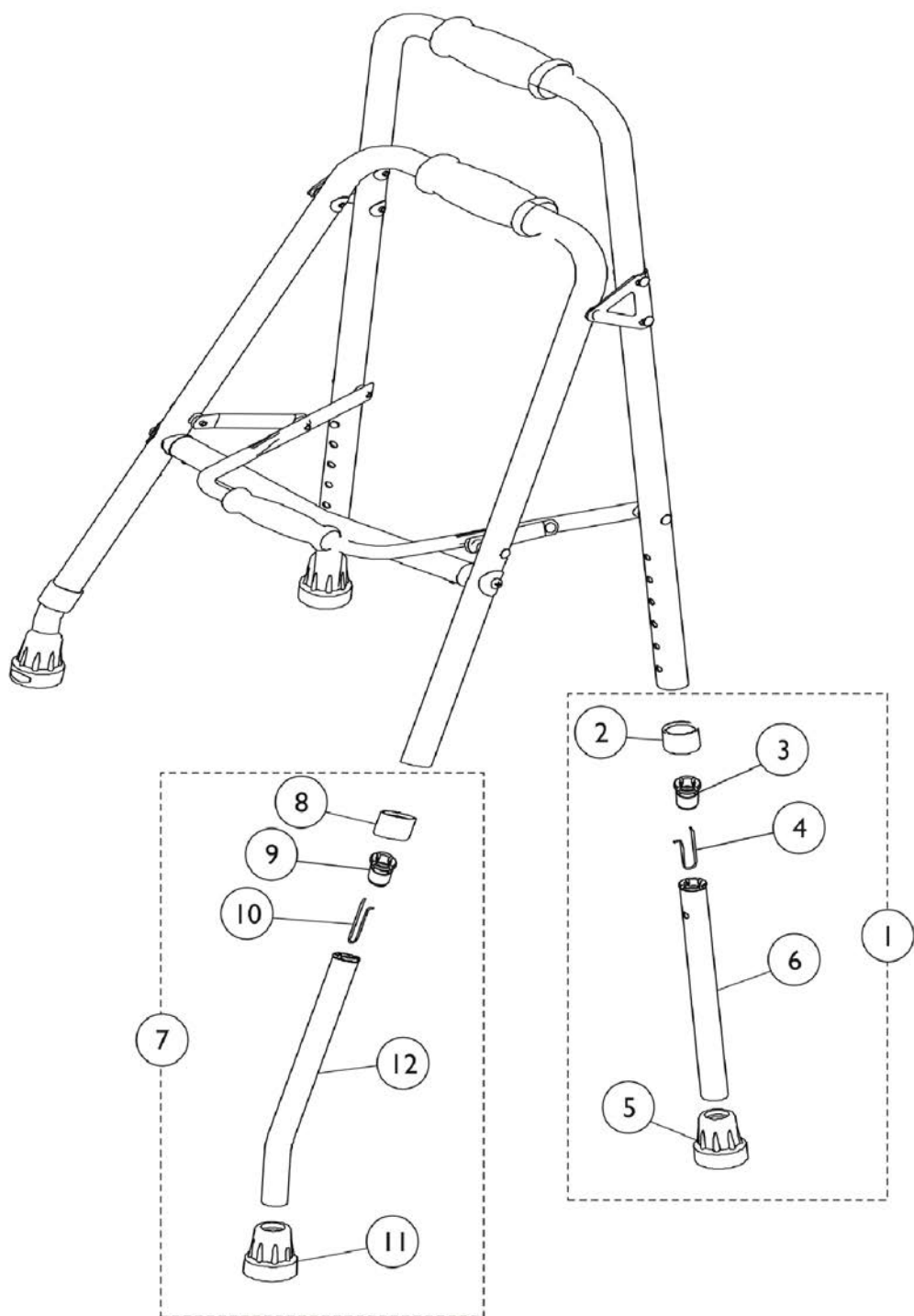


Walker (Hemi)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1099284	Kit, Straight Leg Extension, Rear	1	2
2			Anti-Rattle, External, Grey	1	1
3			Anti-Rattle, Internal	1	1
4			Button, Release	1	1
5			Tip, Rubber, Grey	1	1
6			Tube, Leg Adjustable Straight	1	1
7	B	1099286	Kit, Bent Leg Extension, Front	1	2
8			Anti-Rattle, External, Grey	1	1
9			Anti-Rattle, Internal	1	1
10			Button, Release	1	1
11			Tip, Rubber, Grey	1	1
12			Tube, Leg Adjustable Bent	1	1
NOTE:	A - Kit Includes Items 2-6. B - Kit Includes Items 8-12.				

Walker (WalkLite)

Model 6300-5F, 6300-ATA and 6300-JRA

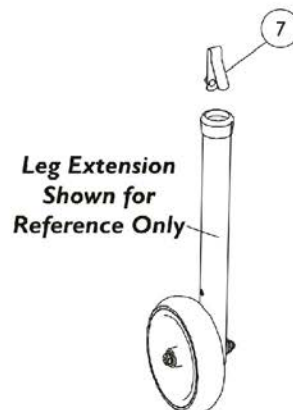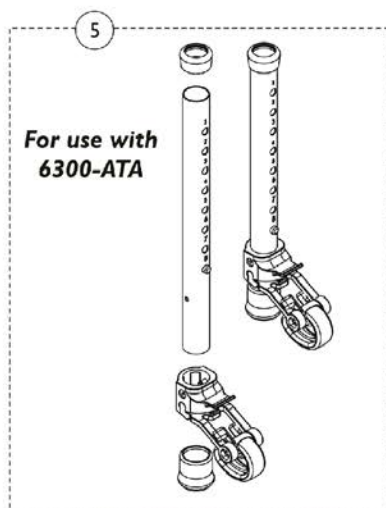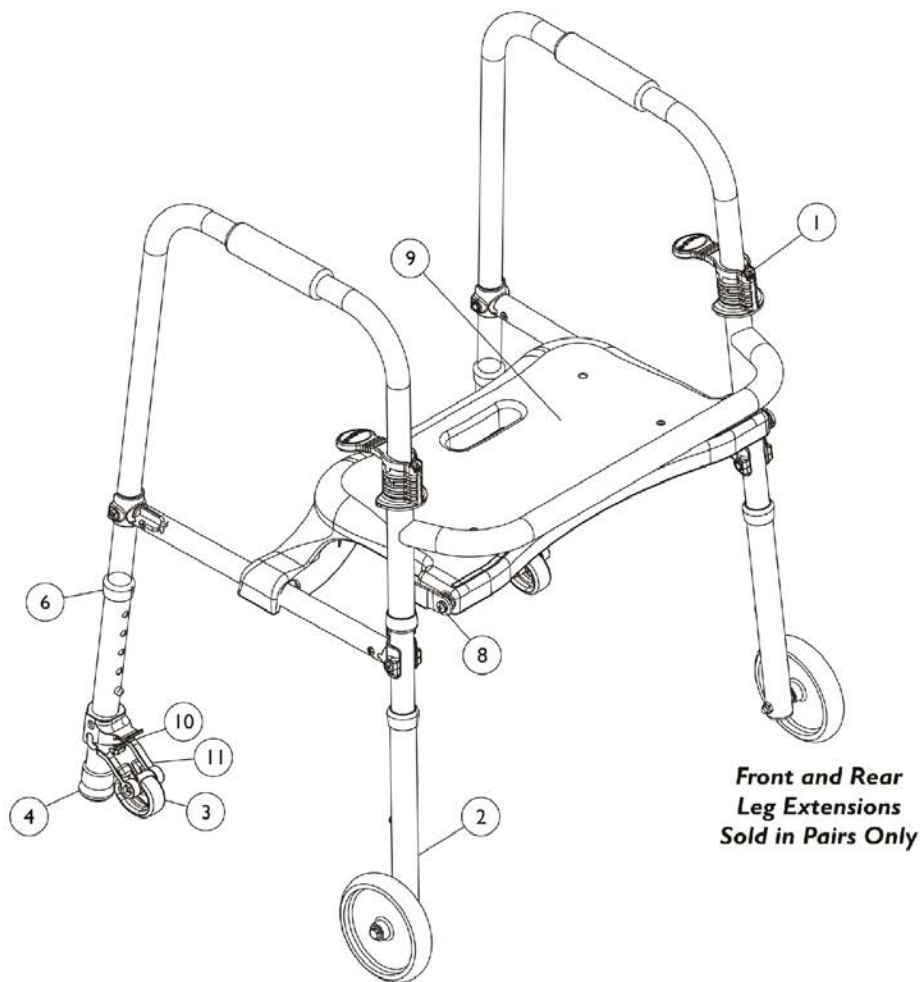

Walker (WalkLite)

Model 6300-5F, 6300-ATA and 6300-JRA


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
	A	1107793-NLA	NLA - Backrest, Nylon Upholstery	1	1
1	D	1043538	Nut, Push	1	4
1		1130784	Kit, Push Nut	10	4
2	B,H	1124254	Kit, Extension Front Leg Assembly	2	1
3		1053179	Glide	1	2
4		1040656	Tip, Rubber, Dark Grey (1-1/8")	1	2
5	C,E	1124255	Kit, Legs with Tips and Hardware (Pair)	2	1
5	C,F	1149805	Kit, Legs with Tips and Hardware (Pair)	2	1
6		1063124	Anti-Rattle, Grey (1-1/8")	1	4
7		1094945	Button, Snap	1	4
	G	1109367-NLA	NLA - Seat Hardware	1	1
8	G	1021421-NLA	NLA - Screw, Phillips Pan Head (1/4-20 x 2-1/4")	1	2
8	G	1025195	Locknut (1/4-20)	1	2
8	G	1025728	Washer (1/4 x 1/2 x 1/32")	1	2
8	G	1108477-NLA	NLA - Washer, Nylon (1/4 x 3/4 x 5/8")	1	2
9		1107737	Seat	1	1
10		1044944	Spring, Compression (31/64 x 1 x 3/64")	1	2
11		1044943	Fork	1	2
NOTE: A - Not Shown. B - Includes Leg Extension Tube, Locknut, Bushing, Washers, Wheel and Screw. C - Kit includes (2) Leg Extensions, (2) Anti-Rattles, (2) Glides, and (2) Tips. D - Individual item no longer available, refer to Kit Part #1130784. E - For use with 6300-5F and 6300-JRA. F - For use with 6300-ATA. G - Kit no longer available. Order each of item #8 in order to receive a complete kit. H - For use with 6300-5F, 6300-JRA and 6300-ATA.					

Walker Accessories

Wheel Attachments Listed Can Be Used on All Walkers Built After 12/95, excluding Models 6291-HDA and 6441-A

Models	Walker Accessories
	Assistance Attachments - (Fits all Walkers)
6098	Walker Basket (Fits all Walkers Except 6261-A, 6291-JR and 6291-HDA)
6013 - NLA	Walker Pouch (Green and Burgandy)
6015-I	Walker Pouch (Green and Deep Sea Blue)
6007	Walker Tray (Grey)
	Leg Extensions
6266	Rear Leg Extensions with Glide Tips (13")
6275	Tall Leg Extensions (Set of 4)
	Platform Attachments - (Fits all Walkers)
6027	Platform Attachment (Fits on Walkers Except 6291-HDA)
	Rear Wheel Brake Attachments
6264	Rear Wheel Brake (10")
6265	Rear Wheel Brake (13")
	Single, Fixed Wheel Attachments (3" and 5")
6270	Single, Fixed Wheel Attachments with Glide Tips (3")
6277	Single, Fixed Wheel Attachments with Glide Tips, Tall (3")
6271	Single, Fixed Wheel Attachments with Glide Tips (5")
6278	Single, Fixed Wheel Attachments with Leg Extension and Glide Tips, Tall (3")
	Sure Glide Brake Attachments
6274	Cambered Sure Glide Brake (10")
6263	Cambered Sure Glide Brake (13")
	Swivel Wheel Attachments (3" and 5")
6267	Swivel Wheels with Glide Tips (5")
6268	Swivel Wheels (5") with Sure Glide Brakes (13")
6269	Swivel Wheels (3") with Rear Leg Extension with Glide Tips (13")
6272	Swivel Wheels (3") with Sure Glide Brakes (13")
6273	Swivel Wheels (3") with Glide Tips (3")

Basket and Pouch (Walker)

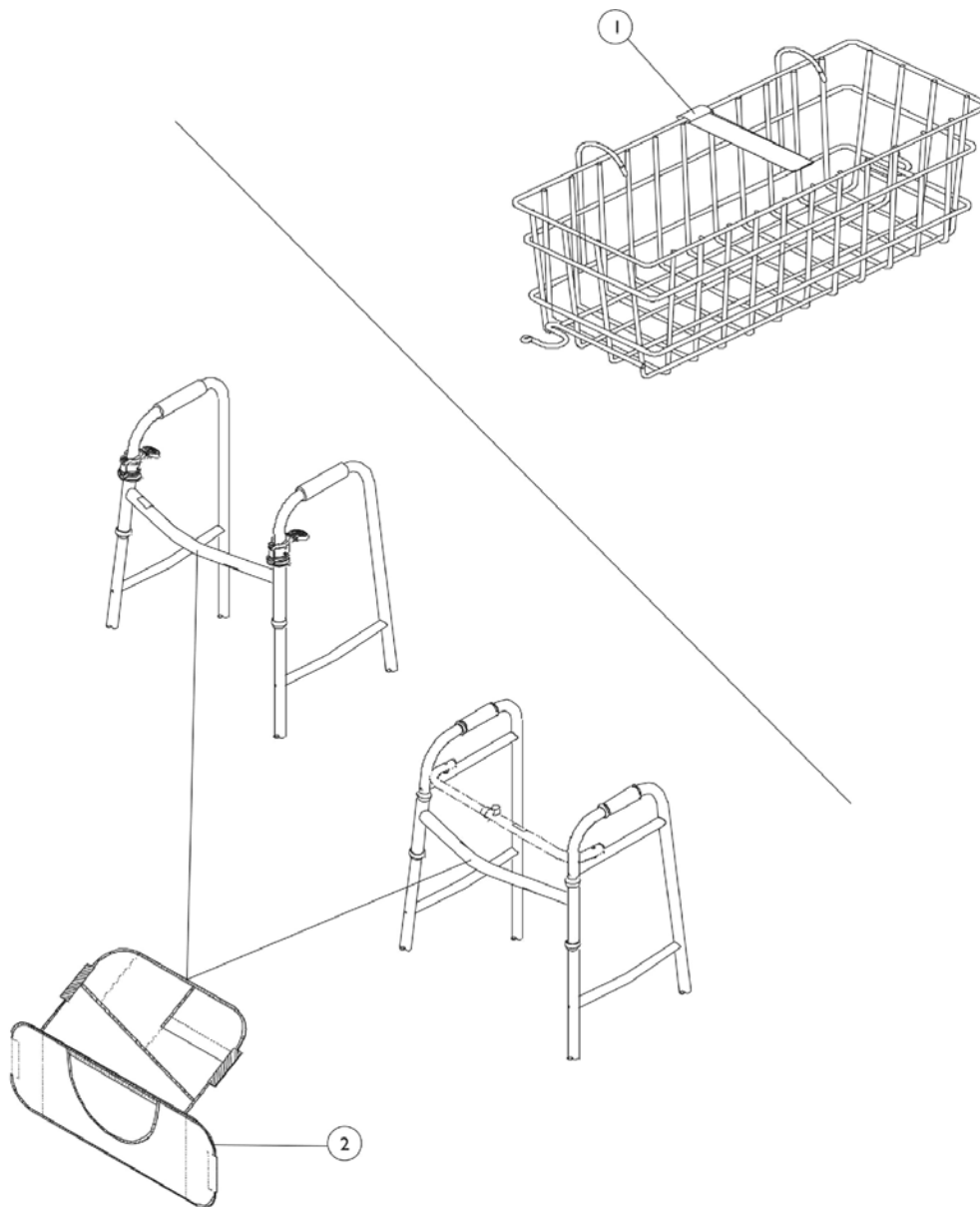


Basket and Pouch (Walker)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	6098	Basket	1	1
2		6015-1-NLA	NLA - Walker Pouch, Blue on Green	1	1
2		6013-NLA	NLA - Walker Pouch, Burgundy on Green	1	1


NOTE: A - Fits most Folding or Rigid Walkers except 6291-HD, 6252, 6300-JRA, 6300-ATA, 6240 and 6285.

Walker Tray Hardware - Model 6007

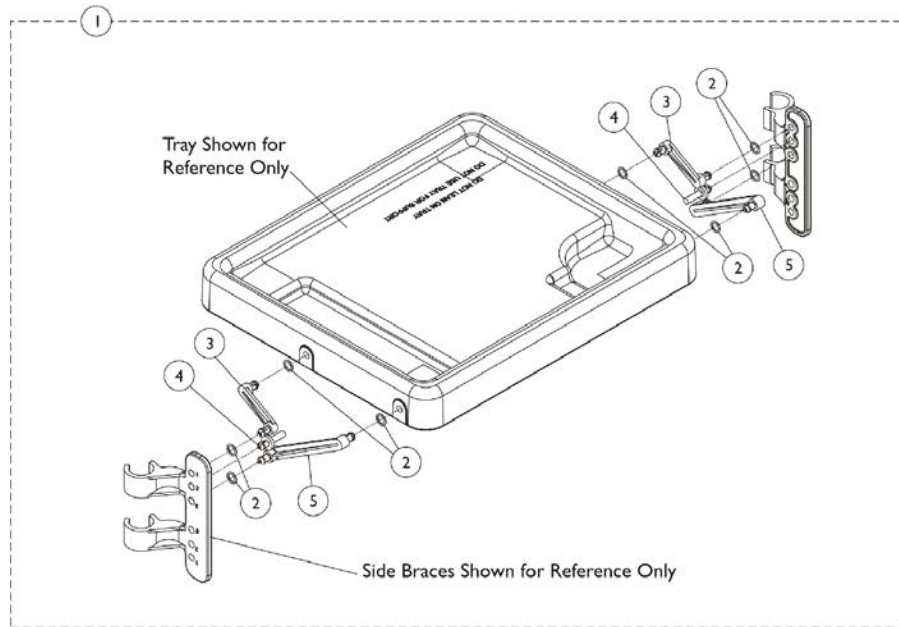

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1168407-NLA	NLA - Kit, Tray Hardware, 6007	1	1
2			See Footnote	1	8
3			Link, Upper	1	2
4			Link, Lower	1	2
5			Support, Snap	1	2
NOTE: A - Includes items 2-5. B - Item 2 is no longer serviceable in the field.					

Walker Tray Hardware - Model 6002

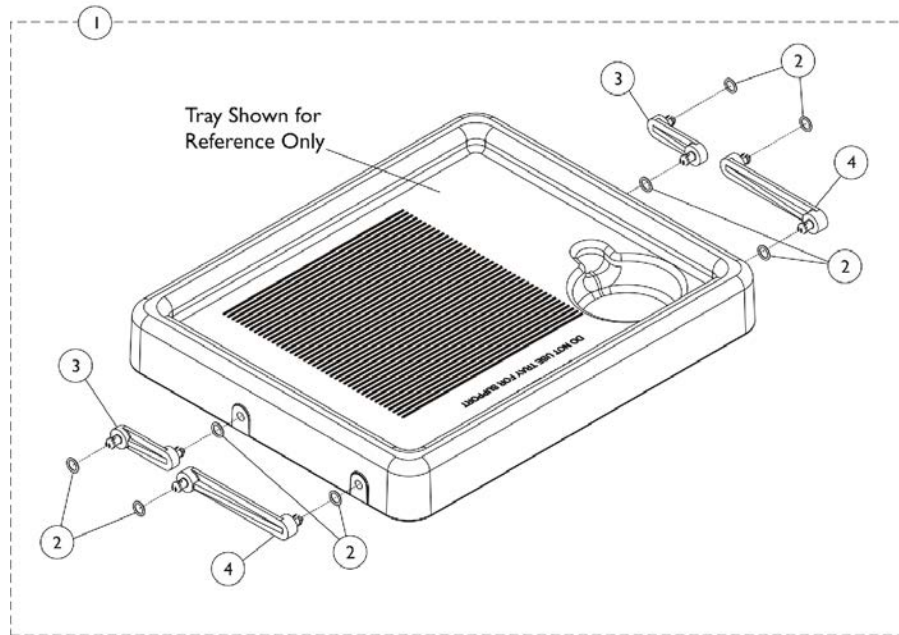

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1156512	NLA - Kit, Tray Hardware, 6002	1	1
2			See Footnote	1	8
3			Link, Upper	1	2
4			Link, Lower	1	2
NOTE: A - Includes items 2-4. B - Item 2 is no longer serviceable in the field.					

Rollator (Four Wheeled) - Model 65350GR




Rollator (Four Wheeled) - Model 65350GR

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1151023	Grip, Hand (Pair)	2	1
2	E	1153851-NLA	NLA - Brake Assembly - Left	1	1
3	E	1153850	Brake Assembly - Right	1	1
4		1151511-NLA	NLA - Backrest Assembly	1	1
5	B,G	1151021-NLA	NLA - Knob w/ Hardware (Pair)	1	1
5	H	1169007	Kit, Adjustable Knob (Pair)	2	1
6		1151015	Seat w/ Mounting Hardware	1	1
7		1151510	Basket	1	1
8		1151024	Package, Cap	6	1
9	C	1151022-NLA	NLA - Package, Springs, Brake (Pair)	2	1
10	D	1151019-NLA	NLA - Kit, Caster with Hardware - Rear (Pair)	2	1
11	F	1151018-NLA	NLA - Kit, Caster with Fork and Hardware - Front (Pair)	2	1
NOTE: A - Includes (2) Hand Grips. B - Includes (2) Knobs with Attaching Hardware. C - Includes (2) Springs. D - Includes (2) Casters and Attaching Hardware. E - Includes (1) Brake Handle, (1) Cable, (1) Brake Pad and Hardware. F - Includes (2) Casters, (2) Forks and Attaching Hardware. G - Part is no longer available as of 2/6/2012. Order Part number 1169007 as a replacement. H - Washer and Locknut not required. Replacement Knob screws directly into adjustment sleeve.					

Rollator (Three Wheeled)
Model 65400 - Sprint (Discontinued 6/07)

Model 65400 (Sprint)
Blue Finish

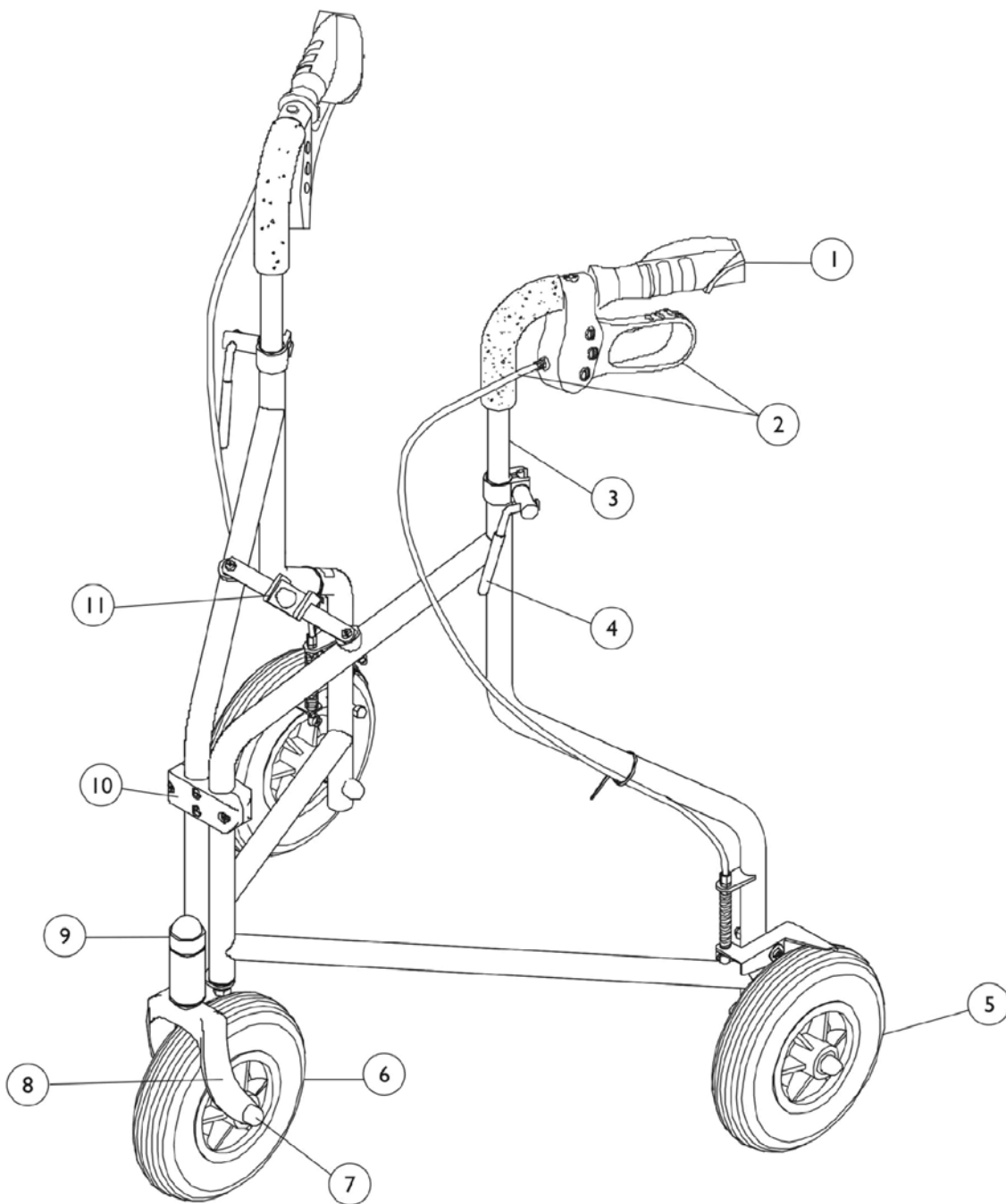


Rollator (Three Wheeled)

Model 65400 - Sprint (Discontinued 6/07)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1076186	Grips, Hand	2	1
2	A,B,M	1076194-NLA	NLA - Kit, Hand Brake - Left	1	1
2	A,B,M	1076195-NLA	NLA - Kit, Hand Brake - Right	1	1
2	C,D,E, N	1096767-NLA	NLA - Kit, Hand Brake - Right / Left	1	2
3	F	1076192-NLA	NLA - Kit, Cane Assembly - Left	1	1
3	F	1076193-NLA	NLA - Kit, Cane Assembly - Right	1	1
4	G	1076191-NLA	NLA - Lever, Hand, Adjustable	1	2
5	H	1076197-NLA	NLA - Kit, Rear Wheel	1	2
6	I	1076171-NLA	NLA - Kit, Wheel	1	1
7		1082534-NLA	NLA - Cap, Plastic, Black	8	1
8		1094533-NLA	NLA - Fork, Front	1	1
9		1094538-NLA	NLA - Cap, Plastic, Black	1	1
10	J	1094531-NLA	NLA - Brace, Front Lower	1	1
11	K	1094532-NLA	NLA - Link, Pivot	1	1
	L	65404	Bag, Tote	1	1

NOTE: A - Cable Style.
 B - Before 11/7/00.
 C - Loop Style.
 D - After 11/8/00.
 E - Due to style change, when replacing Hand Brake and Cable "Before 3/7/00" you must also replace Hand Grips.
 F - Includes Cane, Foam Sleeve and Items 1 and 2.
 G - Includes Bolt, Washer and Lever.
 H - Includes Caster, Bearing, Spacer, Screw and Caps.
 I - Includes Caster, Bearing, Spacer, Nut, Screw and Caps.
 J - Includes Front/Back Block, Screws and Nuts.
 K - Includes Top/Bottom Arm, Screw and Washer.
 L - Not Shown.
 M - Includes Hand Brake, Brake Wire, Cable Assembly and Instruction Sheet.
 N - Includes Brake Holder, Cable, Brake Wire, Handle, Hardware and Instruction Sheet.

Rollator (Three Wheeled) Model 65510 & P429/2 (Current) Model 65410 (Before 9/17/07)

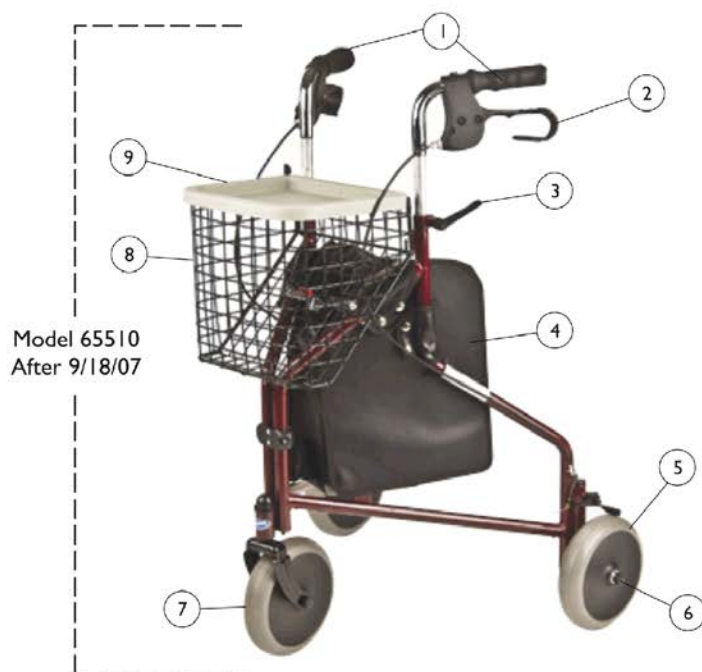


Rollator (Three Wheeled) Model 65510 & P429/2 (Current) Model 65410 (Before 9/17/07)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1142628	Grip, Hand (Pair)	2	1
2		1165507	Brake Assembly w/ Handle and Grip - Right	1	1
2		1165508	Brake Assembly w/ Handle and Grip - Left	1	1
3	A,E	1144499	Knob, Adjustable (Pair)	2	1
3	D,F	1142618	Lever, Hand (Pair)	2	1
4		1144504	Bag, Tote	1	1
5	B,E	1144500	Wheel, Rear	1	1
5	B,F	1452451	Kit, Rear Wheel w/ Hardware	1	1
6		1144505-NLA	NLA - Package, Cap, Plastic, Black	10	1
7	C,E	1144501-NLA	NLA - Wheel, Front w/ Hardware	1	1
7	C,F	1515541	Wheel, Front w/ Hardware	1	1
8		1144503	Basket	1	1
9		1144502	Tray	1	1
NOTE: A - Kit includes (2) Adjustable Knobs and Hardware. B - Kit includes (1) Caster, Attachment Hardware and Cap for Bolt. C - Kit includes (1) Wheel with Fork, Attachment Hardware and Cap for Bolt. D - For models built after 9/19/2007. Kit includes (2) Levers and (2) of all Hardware. E - For models built before 9/18/2007. F - For models built after 9/19/2007.					
Note: As of 12/09/09 components of 65510 and P429/2 are interchangeable.					


Rollite Rollator

Model 65100, 65100R, 65100-JR, 65100R-JR, 68100 and 68100-TA

68100-TA (Midnight Blue) Shown

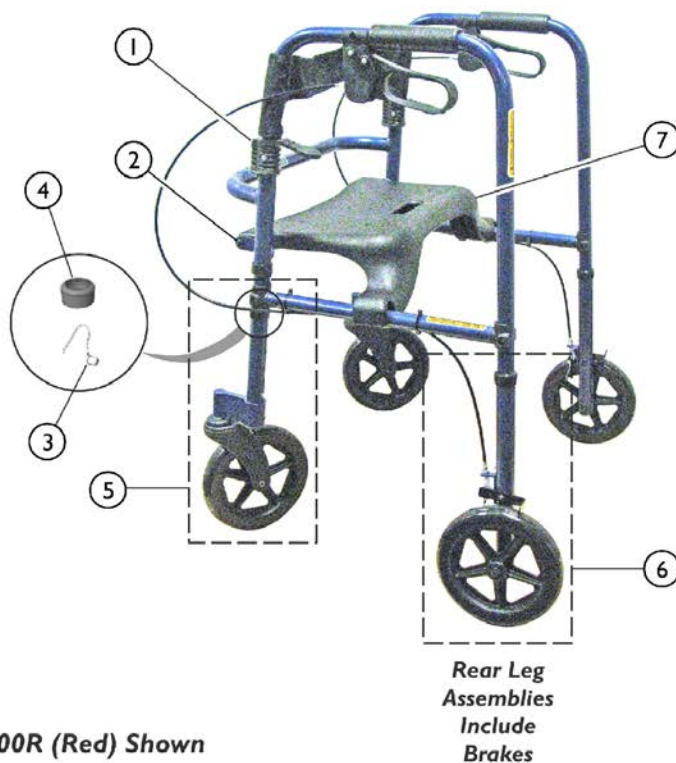

65100R (Red) Shown

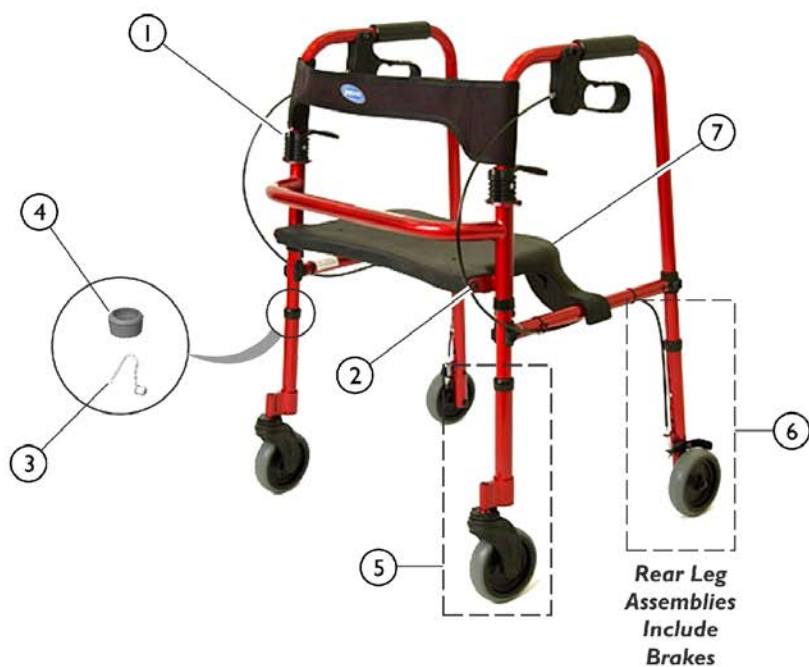

SPC_003A

Rollite Rollator

Model 65100, 65100R, 65100-JR, 65100R-JR, 68100 and 68100-TA

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	G	1043538	Nut, Push	1	4
1		1130784	Kit, Push Nut	10	1
	B	1109367-NLA	NLA - Seat Hardware	1	1
2	B	1021421-NLA	NLA - Screw, Phillips Pan Head (1/4-20 x 2-1/4")	1	2
2	B	1025195	Locknut (1/4-20)	1	2
2	B	1108477-NLA	NLA - Washer, Nylon (1/4 x 3/4 x 5/8")	1	2
2	B	1025728	Washer (1/4 x 1/2 x 1/32")	1	2
3		1094945	Button, Snap	1	4
4		1063124	Anti-Rattle, Grey (1-1/8")	1	4
5	C	1124256	Kit, Front Leg Extension Assembly, Adult - Midnight Blue, Black Wheel (65100)	2	1
5	C	1124258	Kit, Front Leg Extension Assembly, Adult - Electric Red, Black Wheel (65100R)	2	1
5	C	1124260	Kit, Front Leg Extension Assembly, Junior - Midnight Blue, Black Wheel (65100-JR)	2	1
5	C	1124262	Kit, Front Leg Extension Assembly, Junior - Electric Red, Black Wheel (65100R-JR)	2	1
5	C	1124264	Kit, Front Leg Extension Assembly, Adult - Midnight Blue, Black Wheel (68100)	2	1
5	C	1124266	Kit, Front Leg Extension Assembly, Tall Adult - Midnight Blue (68100-TA)	2	1
6	D,H	1124257	Kit, Rear Leg Extension and Brake Assembly, Adult - Midnight Blue, Black Wheel (65100)	2	1
6	D,H	1124261	Kit, Rear Leg Extension and Brake Assembly, Junior - Midnight Blue, Black Wheel (65100-JR)	2	1
6	D,H	1124259	Kit, Rear Leg Extension and Brake Assembly, Adult - Electric Red, Black Wheel (65100R)	2	1
6	D,H	1124263	Kit, Rear Leg Extension and Brake Assembly, Junior - Electric Red, Black Wheel (65100R-JR)	2	1
6	D,H	1124265	Kit, Rear Leg Extension and Brake Assembly, Adult - Midnight Blue, Black Wheel (68100)	2	1
6	D	1124267	Kit, Rear Leg Extension and Brake Assembly, Tall Adult - Midnight Blue (68100-TA)	2	1
7		1107737	Seat	1	1
	E	1046321-NLA	NLA - Paint, Touch-Up, 61P Electric Red	1	1
	E	1101326-NLA	NLA - Paint, Touch-Up, 105P Midnight Blue	1	1
	F	See Note	See Footnote	--	--
		65110	Basket, 65100, 65100R and 68100-TA	1	1
		65109	Basket, 65100-JR, 65100R-JR and 68100	1	1
		65408	Tray	1	1
	A	1107793-NLA	NLA - Backrest, Nylon Upholstery	1	1

Rollite Rollator

Model 65100, 65100R, 65100-JR, 65100R-JR, 68100 and 68100-TA



NOTE: A - Not available on Models 65100-JR, 65100R-JR and 68100.
B - Kit no longer available. Order each of item #2 in order to receive a complete kit.
C - Includes Front Caster and Leg Extension Tube.
D - Includes Hand Brake, Brake Cable, Brake Shoe, Leg Extension Tube and Hardware.
E - Not Shown.
F - For Swivel Wheel Attachment, see category titled, "Swivel Wheel Attachments."
G - Individual item no longer available, refer to Kit Part #1130784.
H - Wheels changed from Gray to Black on 8/18/2015.

NOTE: Front and Rear Leg Extensions are now sold in pairs only. Rear Leg Assembly includes the brake. Wheels (Item 6) changed from Gray to Black on 8/18/2015.

Rollator (Four Wheeled)

Model 65420 - Spartan

Model 65420 (Spartan)
Teal Finish

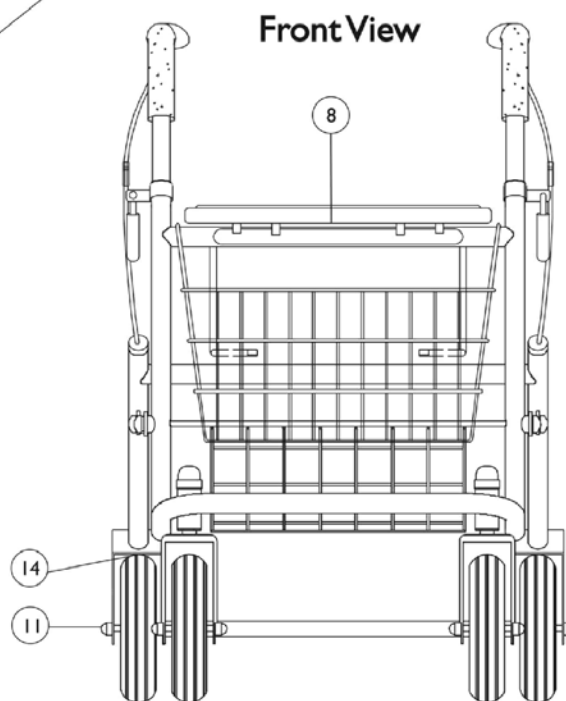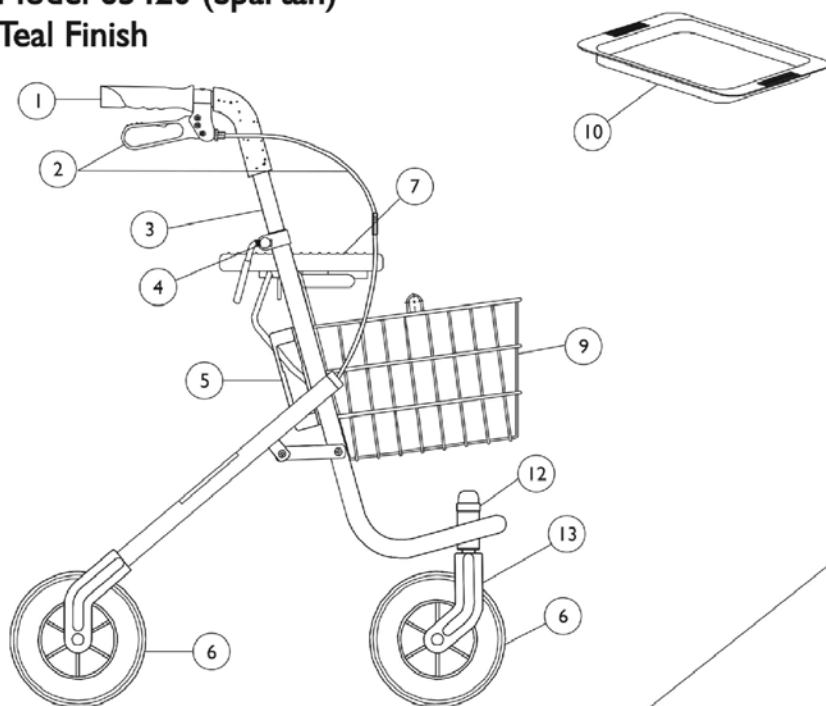


Rollator (Four Wheeled)

Model 65420 - Spartan

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1076186	Grips, Hand	2	1
2	A,B,K	1076189-NLA	NLA - Kit, Hand Brake - Left	1	1
2	A,B	1076190-NLA	NLA - Kit, Hand Brake - Right	1	1
2	C,E,L, M	1096765-NLA	NLA - Kit, Hand Brake - Right/Left	1	2
3	F	1076187-NLA	NLA - Kit, Cane Assembly - Left	1	1
3	F	1076188-NLA	NLA - Kit, Cane Assembly - Right	1	1
4	G	1076191-NLA	NLA - Lever, Hand, Adjustable	1	2
5	H	1076185-NLA	NLA - Folding Bracket Assembly	1	2
6	I	1076171-NLA	NLA - Kit, Wheel	1	4
7		1076172-NLA	NLA - Seat	1	1
8	J	1076178-NLA	NLA - Seat Hardware	1	1
9		1076168-NLA	NLA - Basket	1	1
10		1076174	Tray	1	1
11		1082534-NLA	NLA - Cap, Plastic, Black	8	1
12		1094538-NLA	NLA - Cap, Plastic, Black	1	2
13		1094534-NLA	NLA - Fork, Front	1	2
14		1102912-NLA	NLA - Pad, Brake	1	2

NOTE: A - Cable Style.
 B - Before 3/7/00.
 C - Loop Style.
 E - After 3/8/00.
 F - Includes Cane, Foam Sleeve and Items 1-2.
 G - Includes Bolt, Washer and Lever.
 H - Includes Fold Up/Down Bracket & Screw.
 I - Includes Wheel, Bearing, Hardware and Instruction Sheet.
 J - Includes Brackets, Plates and Screws.
 K - Includes Brake Handle, Cable and Instruction Sheet.
 L - Includes Brake Assembly and Instruction Sheet.
 M - No Longer Available.

Rollator (Four Wheeled) - Model 65450

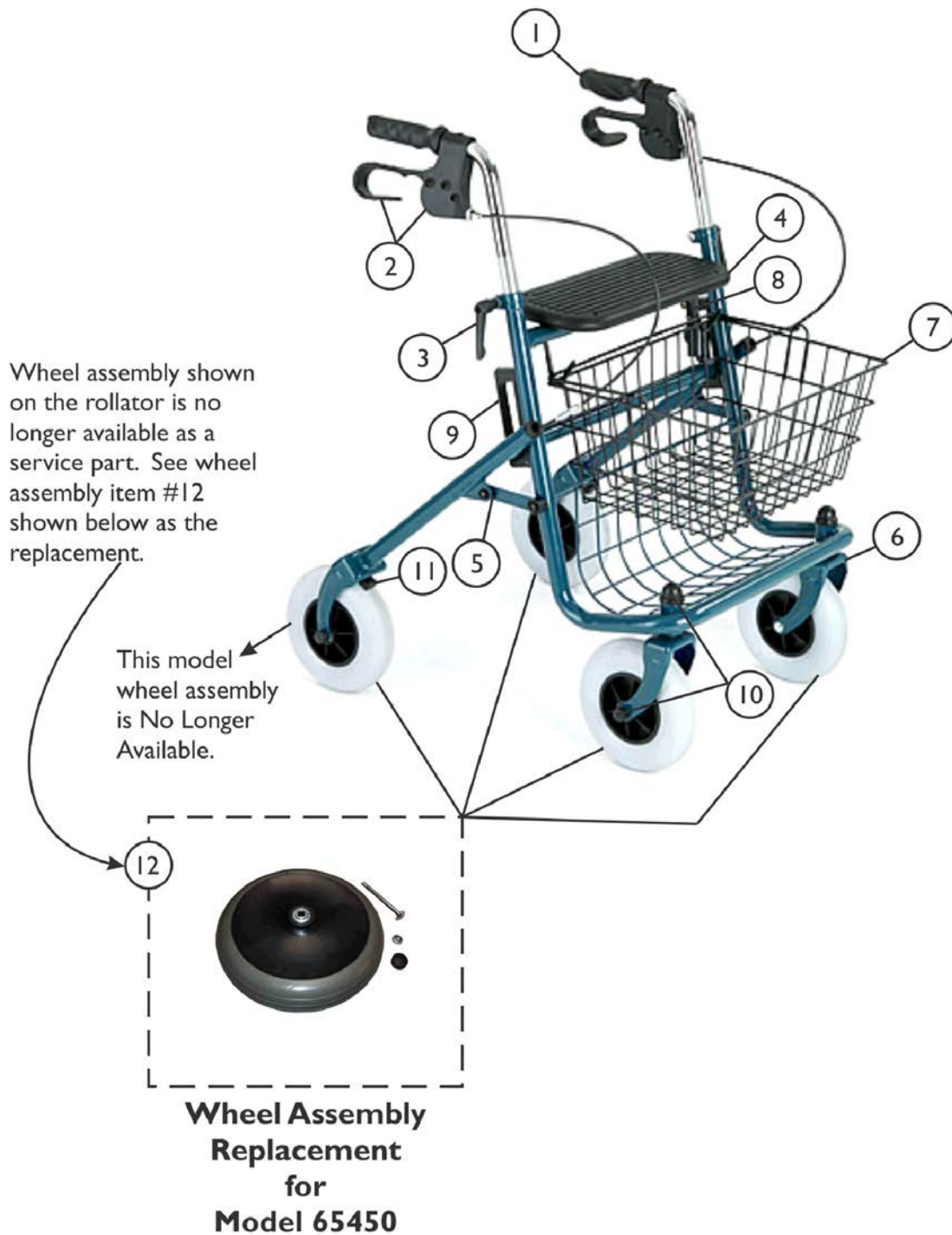


Rollator (Four Wheeled) - Model 65450

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1142615	Hand Grip (Pair)	1	1
2	A	1142616-NLA	NLA - Brake Assembly	1	1
3	B	1142618	Lever, Hand (Pair)	2	1
4		1142622-NLA	NLA - Seat w/ Hardware	1	1
5		1142626-NLA	NLA - Connection Linkage Assembly	1	2
6		1142625-NLA	NLA - Fork, Front	1	2
7		1142627-NLA	NLA - Basket	1	1
8		1142619-NLA	NLA - Folding Lock with Bracket	1	1
9		1142620-NLA	NLA - Bracket, Black	1	1
10	D	1142624-NLA	NLA - Caps, End, Black	1	1
11	E	1142617-NLA	NLA - Pad, Brake	1	2
12	C	1142621	Wheel Assembly w/ Hardware (7-1/2")	1	4
	F	1142623	Tray	1	1
NOTE: A - Includes Brake Handle with Cable, Brake Pad is not included. B - Includes (2) Levers and Hardware. C - Wheel Assembly that originally came with Rollator model 66450 is no longer available as a service part. Must verify which wheel assembly type the customer has. A quantity of (4) ea. of part #1142621 may be required to match. D - Includes (2) Fork Caps and (4) Wheel Caps. E - Includes (1) Brake Pad and (1) Screw. F - Not Shown.					

Rollator (Four Wheeled)

Model 65500 - Stingray

Model 65500 (Stingray)
Blue/Grape Finish

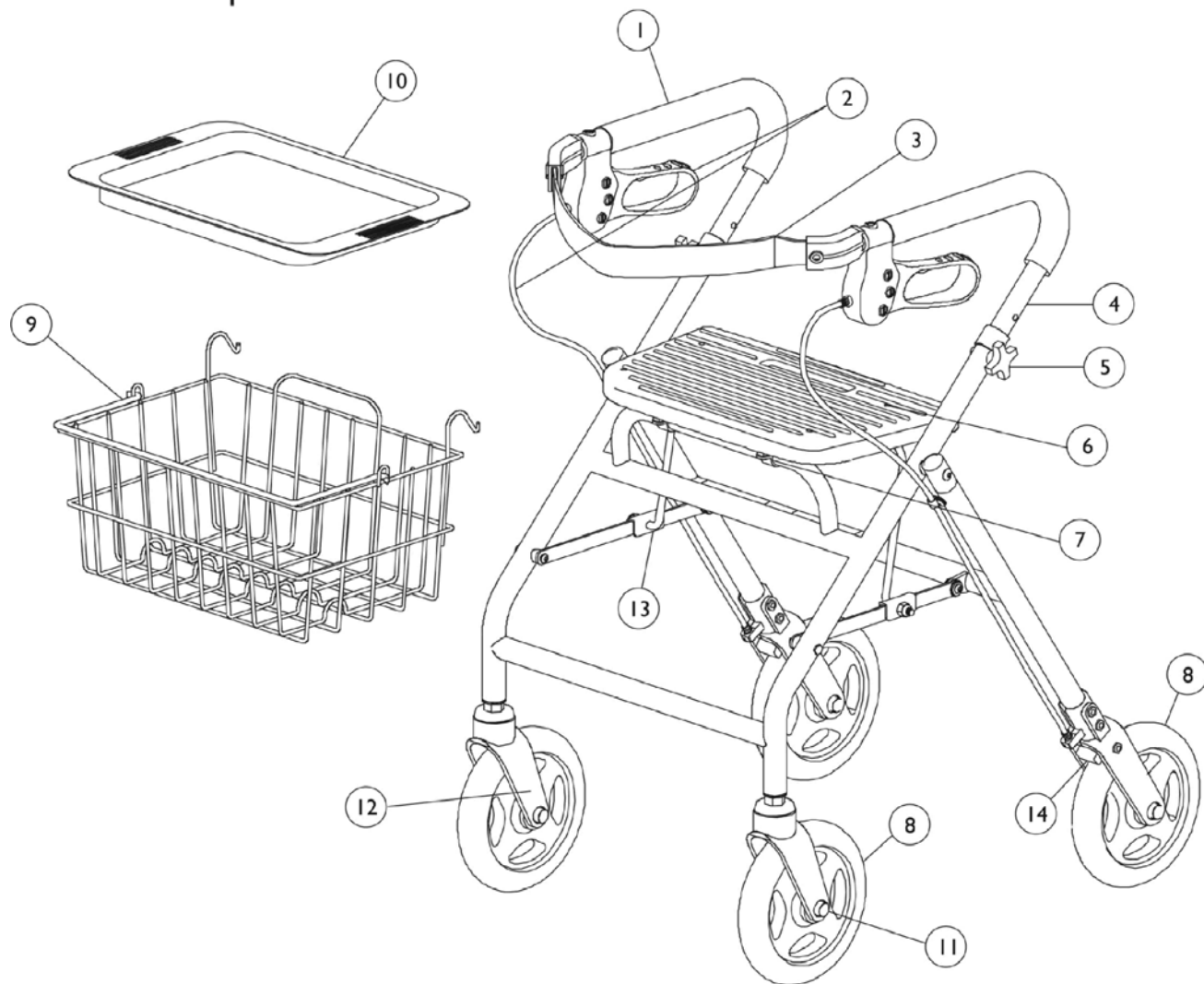

Rollator (Four Wheeled)

Model 65500 - Stingray

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1076180-NLA	NLA - Grip, Foam	2	1
2	A	1076184-NLA	NLA - Kit, Hand Brake	1	2
3	B	1076177-NLA	NLA - Kit, Back Support	1	1
4	C	1076182-NLA	NLA - Kit, Cane	1	2
5	D	1076175-NLA	NLA - Knob, Adjustable with Hardware	1	2
6		1076172-NLA	NLA - Seat	1	1
7	E	1076178-NLA	NLA - Seat Hardware	1	1
8	F	1076176-NLA	NLA - Kit, Wheel w/ Hardware	1	4
9		1076169-NLA	NLA - Basket	1	1
10		1076174	Tray	1	1
11		1082534-NLA	NLA - Cap, Plastic, Black	8	1
12		1094535-NLA	NLA - Fork, Front	1	2
13		1110750-NLA	NLA - Kit, Connecting Link w/ Hardware	1	2
14		1104754-NLA	NLA - Pad, Brake	1	2
NOTE: A - Includes Brake Handle, Brake Holder and Brake Cable Only. B - Includes Support Strap, Right and Left Anchor, Screw and Nuts. C - Includes Grip, Hand Brake, Cable and Screws. D - Includes Knob and Bolt. E - Includes Brackets, Plates and Screws. F - Includes Wheel, Bearings, Spacer, Nut, Screw and Caps.					


Rollator (Four Wheeled) - Model 65550



Rollator (Four Wheeled) - Model 65550

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1142615	Hand Grip (Pair)	2	1
2	A	1165497	Brake Assembly w/ Handle and Grip - Right (65550)	1	1
2	A	1165498	Brake Assembly w/ Handle and Grip - Left (65550)	1	1
3	D	1142618	Lever, Hand (Pair)	2	1
4		1142623	Tray	1	1
5	E	1452452	Kit, Assembly, Seat w/ Hardware	1	1
6	B	1452450	Kit, Front Wheel w/ Fork and Hardware	1	2
7	C	1452451	Kit, Rear Wheel w/ Hardware	1	2
8		1142627-NLA	NLA - Basket	1	1
NOTE: A - Includes Grip, Handle, Brake Handle, and Complete Brake Assembly. B - Includes Wheel, Fork, and Hardware. C - Includes Wheel and Hardware. D - Includes (2) Levers and Hardware. E - Includes Seat and Hardware.					

Rollator (Four Wheeled) - Model 65650 and 65650R

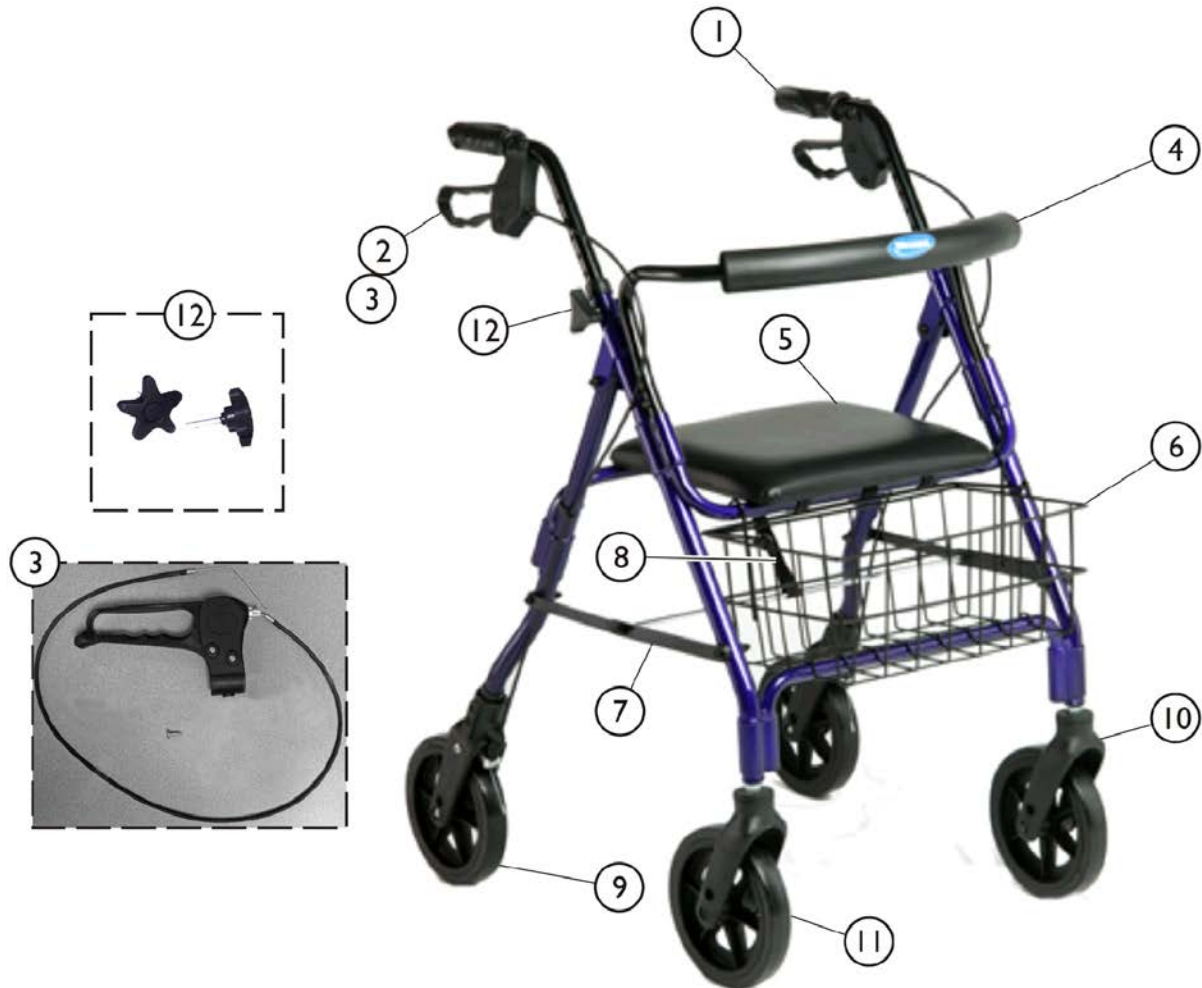

The photo represents a previous model of the 65650 Rollator, Review all Footnotes for additional information before ordering any part.

Rollator (Four Wheeled) - Model 65650 and 65650R

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1142628	Grip, Hand (Pair)	1	1
2	B,F	1142629-NLA	NLA - Brake Handle w/ Hardware Assembly	1	2
3	C,G	1142630-NLA	NLA - Brake Assembly	1	2
4		1142632	Back Support Assembly, Curved	1	1
5		1142633-NLA	NLA - Seat w/ Hardware	1	1
6		1142638-NLA	NLA - Basket	1	1
7		1142639	Connection Linkage Assembly	1	2
8		1142637	Package, Strap	10	1
9	D	1142636-NLA	NLA - Kit, Wheel w/ Hardware - Rear	1	2
10	F	1142634-NLA	NLA - Fork Assembly, Front	1	2
11		1142635-NLA	NLA - Kit, Wheel w/ Hardware - Front	1	2
12	E	1144495-NLA	NLA - Kit, Adjustable Knob (Pair)	2	1
12		1169007	Kit, Adjustable Knob (Pair)	2	1
NOTE: A - Includes (2) Grips. B - Includes Handle with attaching Hardware. C - Includes Handle with Cable, Fork, Brake Pad and Hardware. D - Includes Rear Wheel with Hardware. E - Part is no longer available. See #1169007 for replacement. F - Replacement wheel has different amount of Spokes that does not impact performance. It is recommended to replace Fork Assembly in pairs to maintain appearance. G - New Handgrips must be ordered as existing Handgrips must be cut off to install new Brake Assembly.					

Rollator Bariatric (Four Wheeled) - Model 66500



Rollator Bariatric (Four Wheeled) - Model 66500

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1141748-NLA	NLA - Kit, Hand Grip	1	1
2	B	1141749-NLA	NLA - Brake Assembly (Pair)	2	1
3		1141750-NLA	NLA - Wheel with Hardware - Rear	1	1
4	C	1141751-NLA	NLA - Front Caster Assembly	1	1
5		1141752	Back Rest Assembly	1	1
6		1141753	Basket	1	1
7		1141754	Seat w/ Hardware	1	1
	D,E	1141755-NLA	NLA - Kit, Frame Hardware	1	1
NOTE: A - Includes (2) Hand Grips. B - Includes (2) Brake Handle Assemblies, (2) Cables, (2) Break Pads and Bottom Caps. C - Includes (1) Fork and (1) Wheel with Hardware. D - Includes (2) Anti-Rattles, (2) Push Buttons, (2) Adjustable Knobs and (2) Brake Cord Holders. E - Not Shown.					

Rollator Bariatric (Four Wheeled) - Model 66550




Rollator Bariatric (Four Wheeled) - Model 66550

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1175907	Kit, Right Brake Handle and Brake Assembly	1	1
1		1175908	Kit, Left Brake Handle and Brake Assembly	1	1
2		1176369	Kit, Seat w/ Attaching Hardware	1	1
3		1175909	Kit, Rear Wheel Assembly	1	2
4		1176366	Kit, Front Wheel Assembly w/ Fork	1	2
5		1176367	Kit, Back Rest w/ Attaching Hardware	1	1
6		1176368	Bag, Underseat Storage	1	1
	A	1142617-NLA	NLA - Pad, Brake	1	1
	A	1176371	Package, Dust Cap, Black	10	1
	B	1141755-NLA	NLA - Kit, Frame Hardware	1	1
NOTE: A - Not Shown B - Not Shown. Includes (2) Anti-Rattles, (2) Push Buttons, (2) Adjustable Knobs, and (2) Brake Cord Holders.					

Rollator (Four Wheeled - Discontinued 6/07)

Model 65700 - Frontier Lite

Model 65700 (Frontier Lite)
Burgundy Finish

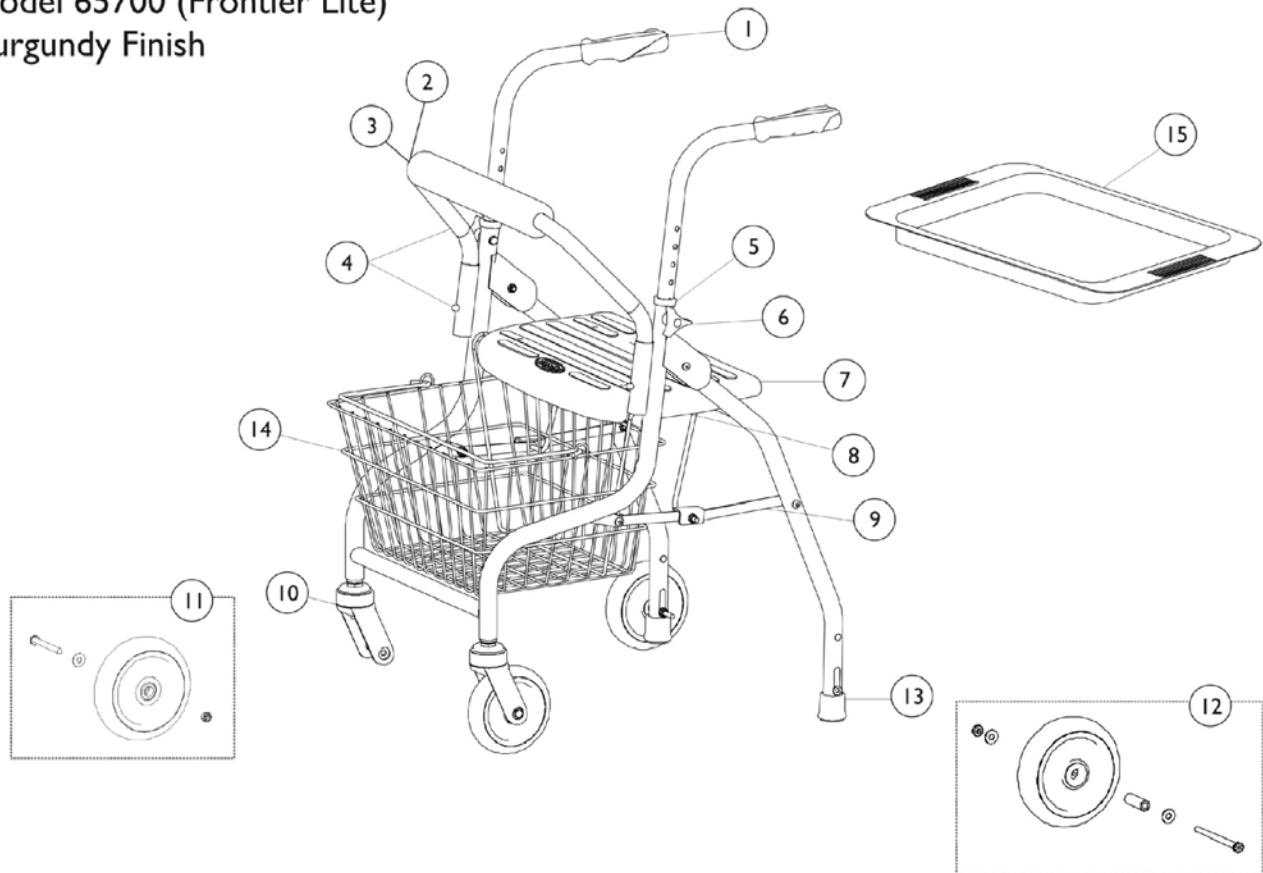

Rollator (Four Wheeled - Discontinued 6/07)

Model 65700 - Frontier Lite

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1076186	Grips, Hand	2	1
2		1099372-NLA	NLA - Cover	1	1
3		1094528-NLA	NLA - Pad, Backrest	1	1
4	A	1099632-NLA	NLA - Kit, Backrest Tube, Burgundy	1	1
5		1099367-NLA	NLA - Package, Anti-Rattle, Black (1")	2	1
6	B	1076175-NLA	NLA - Knob, Adjustable with Hardware	1	2
7		1099299-NLA	NLA - Seat	1	1
8	C	1099373-NLA	NLA - Kit, Seat Hardware	1	1
9	F	1110749-NLA	NLA - Kit, Connection Link w/ Hardware	1	2
10		1099369-NLA	NLA - Fork, Front	2	1
11	D	1099370-NLA	NLA - Kit, Wheel (5"), Front	2	1
12	E	1099371-NLA	NLA - Kit, Wheel - Rear (5")	2	1
13		1099368-NLA	NLA - Package, Rubber Tip	2	1
14		1098988-NLA	NLA - Basket	1	1
15		1076174	Tray	1	1
NOTE: A - Includes Backrest Tube & Snap Buttons. B - Includes Knob and Bolt. C - Includes Brackets and Screws. D - Includes Wheel, Bearings, Bolt, Washer and Nut. E - Includes Wheel, Bearings, Bolt, Washer, Spacer and Nut. F - Includes Front and Rear Linkage, Hardware and Instruction Sheet.					

Rollator (Four Wheeled) - Model 65750




Rollator (Four Wheeled) - Model 65750

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E	1144495-NLA	NLA - Kit, Adjustable Knob (Pair)	2	1
1		1169007	Kit, Adjustable Knob (Pair)	2	1
2	A	1144496-NLA	NLA - Wheel Assembly - Rear	1	2
3	B	1144497-NLA	NLA - Wheel Assembly - Front	1	2
4		1142638-NLA	NLA - Basket	1	1
5		1142643-NLA	NLA - Back Support Assembly	1	1
6		1142633-NLA	NLA - Seat w/ Hardware	1	1
	D,E	1144498-NLA	NLA - Package, Rubber Tip	2	1
NOTE: A - Kit includes (2) Casters with Attaching Hardware. B - Kit includes Wheel, Fork and Attaching Hardware. D - Use with the push down brake. E - Part is no longer available. See #1169007 for replacement.					

Rollator (Four Wheeled)

Model 65800 - Sandstorm

Model 65800 (Sandstorm)
Denim Blue Finish

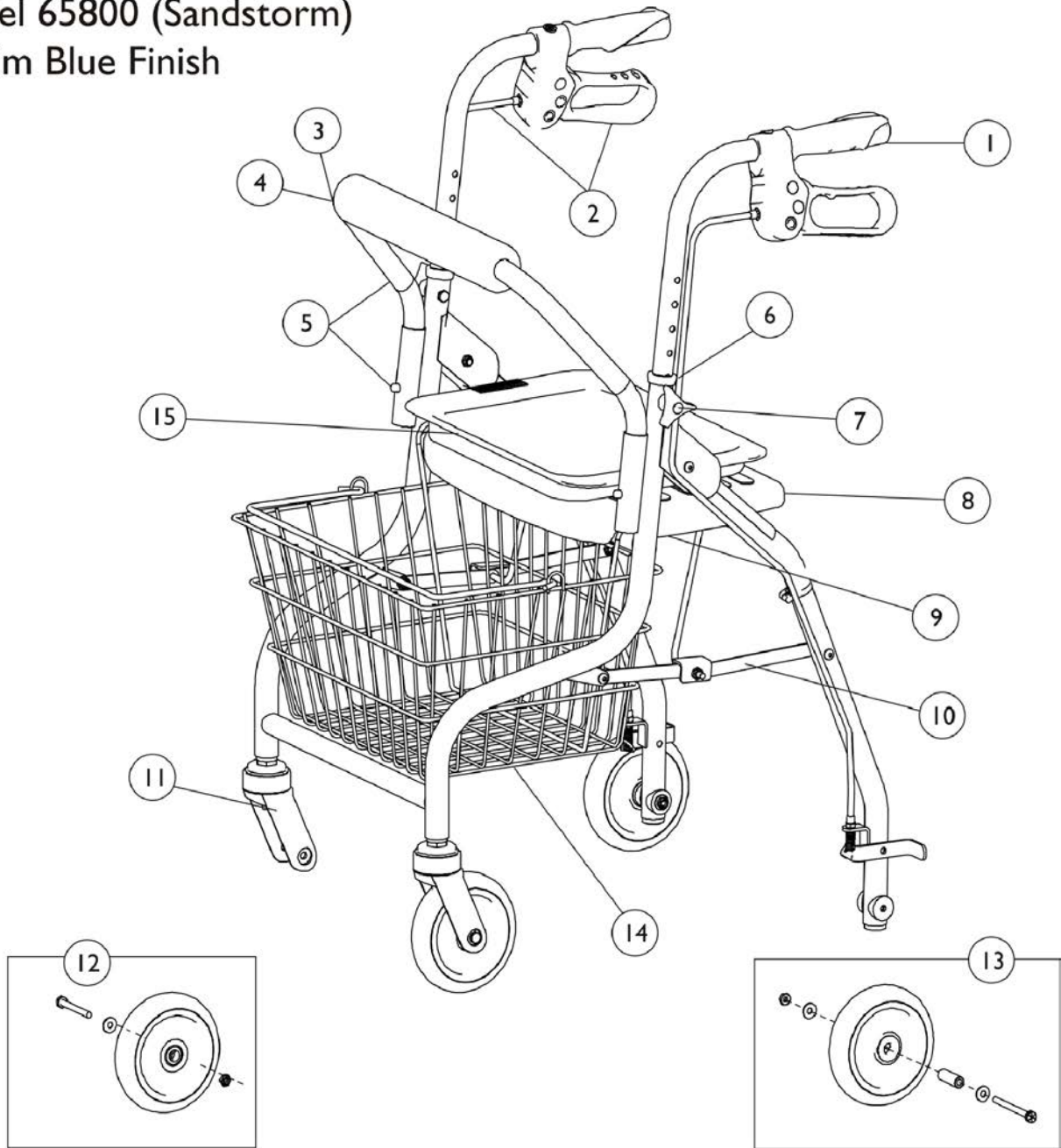

Rollator (Four Wheeled)

Model 65800 - Sandstorm

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1076186	Grips, Hand	2	1
2	A	1076184-NLA	NLA - Kit, Hand Brake	1	2
3		1099372-NLA	NLA - Cover	1	1
4		1094528-NLA	NLA - Pad, Backrest	1	1
5	B	1099631-NLA	NLA - Kit, Backrest Tube, Denim Blue	1	1
6		1099367-NLA	NLA - Package, Anti-Rattle, Black (1")	2	1
7	C	1076175-NLA	NLA - Knob, Adjustable with Hardware	1	2
8		1099299-NLA	NLA - Seat	1	1
9	D	1099373-NLA	NLA - Kit, Seat Hardware	1	1
10	G	1110749-NLA	NLA - Kit, Connection Link w/ Hardware	1	2
11		1099369-NLA	NLA - Fork, Front	2	1
12	E	1099370-NLA	NLA - Kit, Wheel (5"), Front	2	1
13	F	1099371-NLA	NLA - Kit, Wheel - Rear (5")	2	1
14		1098988-NLA	NLA - Basket	1	1
15		1076174	Tray	1	1
NOTE: A - Includes Brake Handle, Brake Holder and Brake Cable Only. B - Includes Back Tube and Snap Buttons. C - Includes Knob and Bolt. D - Includes Brackets and Screws. E - Includes Caster, Bearing, Bolt, Washer and Nut. F - Includes Caster, Bearing, Bolt, Washer, Spacer and Nut. G - Includes Front and Rear Linkage, Hardware and Instruction Sheet.					

Rollator (Four Wheeled) - Model 65850



Rollator (Four Wheeled) - Model 65850

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1142628	Grip, Hand (Pair)	1	1
2		1142640	Brake Handle with Cable	1	2
3		1142643-NLA	NLA - Back Support Assembly	1	1
4		1142644-NLA	NLA - Package, Spring Button	10	2
5		1142633-NLA	NLA - Seat w/ Hardware	1	1
6		1142638-NLA	NLA - Basket	1	1
7		1142639	Connection Linkage Assembly	1	2
8		1142641-NLA	NLA - Brake Pad Assembly - Right	1	1
9		1142642-NLA	NLA - Brake Pad Assembly - Left	1	1
10		1142646-NLA	NLA - Kit, Wheel Assembly - Front	1	2
11	D	1142634-NLA	NLA - Fork Assembly, Front	1	2
12		1142637	Package, Strap	10	1
13	C	1142631-NLA	NLA - Package, Knob	1	1
14		1141759-NLA	NLA - Kit, Wheel w/Hardware - Rear	1	2
NOTE: A - Includes (2) Hand Grips. C - Includes (2) Adjustable Knobs with Hardware, (2) Anti-Rattles and (2) Basket Guides. D - Replacement wheel has different number of spokes that does not impact performance. It is recommended to replace Fork Assembly in pairs to maintain appearance.					

Rollator (Four Wheeled) - Model 65851R - 65851B

Available in Red (65851R) and Blue (65851B)



Rollator (Four Wheeled) - Model 65851R - 65851B

Available in Red (65851R) and Blue (65851B)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1151023	Grip, Hand (Pair)	2	1
2		1165495	Brake Assembly w/ Handle and Grip - Right	1	1
3		1165496	Brake Assembly w/ Handle and Grip - Left	1	1
4		1151020	Backrest Assembly, Red	1	1
4	B,F	1165096	Backrest Assembly, Blue	1	1
5		1151021-NLA	NLA - Knob w/ Hardware (Pair)	2	1
5		1169007	Kit, Adjustable Knob (Pair)	2	1
6		1151015	Seat w/ Mounting Hardware	1	1
7	G	1151014-NLA	NLA - Basket	1	1
8		1151024	Package, Cap	6	1
9		1151022-NLA	NLA - Package, Springs, Brake (Pair)	2	1
10		1151019-NLA	NLA - Kit, Caster with Hardware - Rear (Pair)	1	1
11	E	1151018-NLA	NLA - Kit, Caster with Fork and Hardware - Front (Pair)	1	1


NOTE: A - Includes (2) Hand Grips.
 B - Includes (2) Knobs and Attaching Hardware.
 C - Includes (2) Springs.
 D - Includes (2) Rear Casters with Attaching Hardware.
 E - Includes (2) Front Casters and Fork with Attaching Hardware.
 F - Use with Rollators manufactured before 02/06/12.
 G - Use with Rollators manufactured after 02/05/12.

Specify Model Color when ordering.

Rollator (Four Wheeled)

Model 65900 - Stardust

Model 65900 (Stardust)
Jewel Green Finish

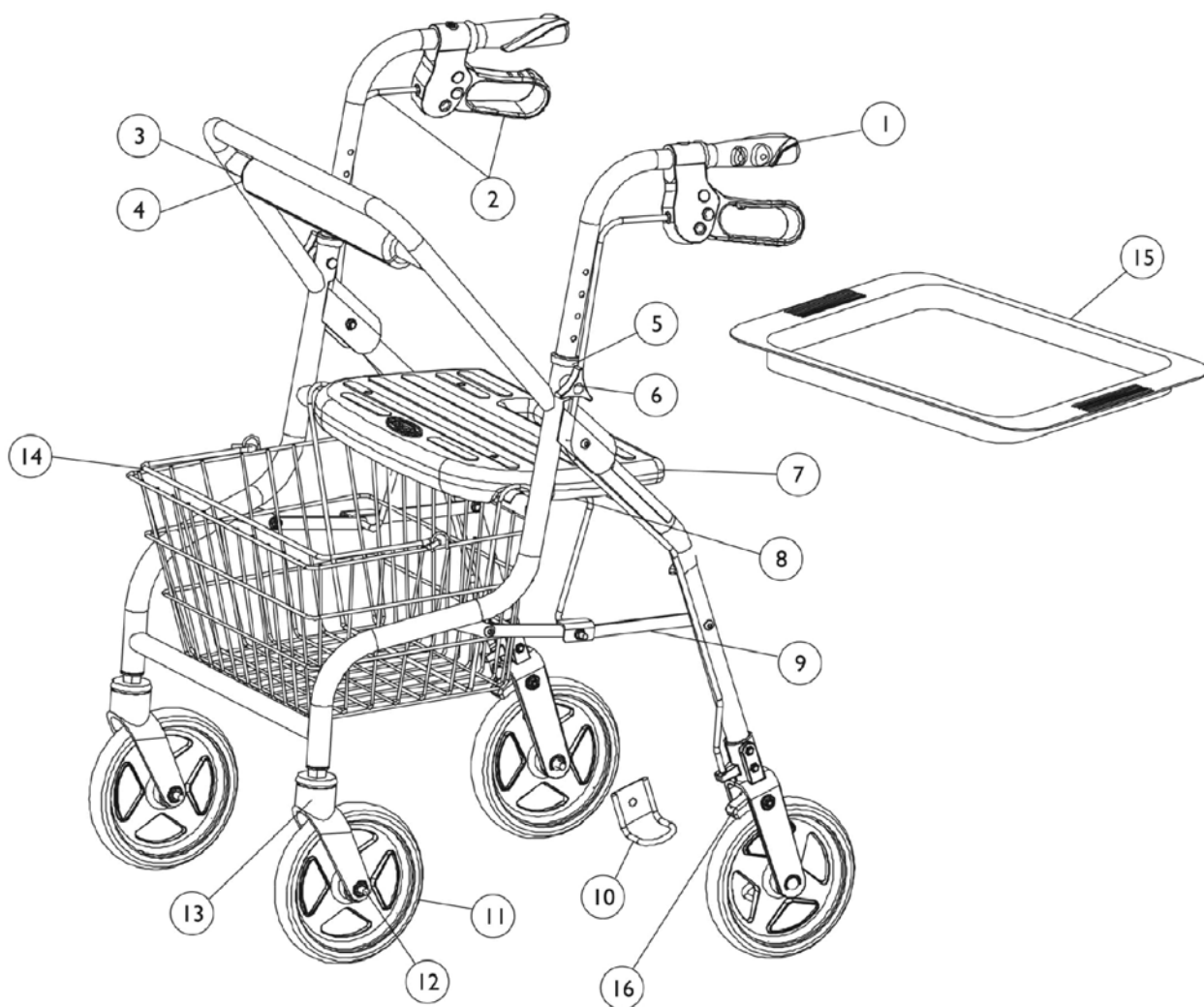

Rollator (Four Wheeled)

Model 65900 - Stardust

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1076186	Grips, Hand	2	1
2	A	1076184-NLA	NLA - Kit, Hand Brake	1	2
3		1099372-NLA	NLA - Cover	1	1
4		1094528-NLA	NLA - Pad, Backrest	1	1
5		1099367-NLA	NLA - Package, Anti-Rattle, Black (1")	2	1
6	B	1076175-NLA	NLA - Knob, Adjustable with Hardware	1	2
7		1099299-NLA	NLA - Seat	1	1
8	C	1099373-NLA	NLA - Kit, Seat Hardware	1	1
9	E	1110749-NLA	NLA - Kit, Connection Link w/ Hardware	1	2
10		65406-NLA	NLA - Kit, Curb Climber	1	2
11	D	1076176-NLA	NLA - Kit, Wheel w/ Hardware	1	4
12		1082534-NLA	NLA - Cap, Plastic, Black	8	1
13		1094535-NLA	NLA - Fork, Front	1	2
14		1098988-NLA	NLA - Basket	1	1
15		1076174	Tray	1	1
16		1104754-NLA	NLA - Pad, Brake	1	2
NOTE: A - Includes Brake Handle, Brake Holder and Brake Cable Only. B - Includes Knob and Bolt. C - Includes Brackets, Plates and Screws. D - Includes Caster, Bearings, Spacer, Nut, Screw and Caps. E - Includes Front and Rear Linkage, Hardware and Instruction Sheet.					

Rollator (Four Wheeled) - Model 67100 - FR300



Rollator (Four Wheeled) - Model 67100 - FR300

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1165881	Hand Brake Assembly w/ Cane - Right	1	1
1		1165882	Hand Brake Assembly w/ Cane - Left	1	1
2		1168395	Seat	1	1
3		1168393	Basket	1	1
4	A	1168400-NLA	NLA - Wheel Assembly w/ Hardware, Front	1	1
5	B	1165885	Wheel, Front w/ Mounting Hardware	1	1
6	A	1165883	Wheel Assembly w/ Hardware, Rear - Right	1	1
6	A	1165884	Wheel Assembly w/ Hardware, Rear - Left	1	1
7		1169824	Knob, Adjustment (Pair)	2	1
8		1142628	Grip, Hand (Pair)	2	1
	C	1168394-NLA	NLA - Tray	1	1
NOTE: A - Includes fork, wheel and attaching hardware. B - Includes wheel and attaching hardware only. C - Not shown.					

Knee Walker - Model 65950



Knee Walker - Model 65950

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1151504-NLA	NLA - Knee Pad with Hardware	1	1
2		1151505-NLA	NLA - Kit, Caster and Fork Assembly w/ Hardware and Extension Tube - Front	1	2
3		1151506-NLA	NLA - Wheel	1	4
4		1151507-NLA	NLA - Hand Brake Assembly	1	1
5		1151508-NLA	NLA - Wheel Assembly w/ Hardware - Rear	1	2
6		1151509-NLA	NLA - Basket	1	1
NOTE: A - Includes (1) Hand Brake, (1) Cable and Mounting Hardware.					

Knee Walker - Model 65960

NOTE:

**Service Parts are not available
for model 65960.**


**The Knee Walker must be returned
to Invacare Corporation.**

**Call Customer Service for assistance
at 1-800-333-6900**

Fixed Wheel Attachment (Rollator)

Model 65100

Model 65100

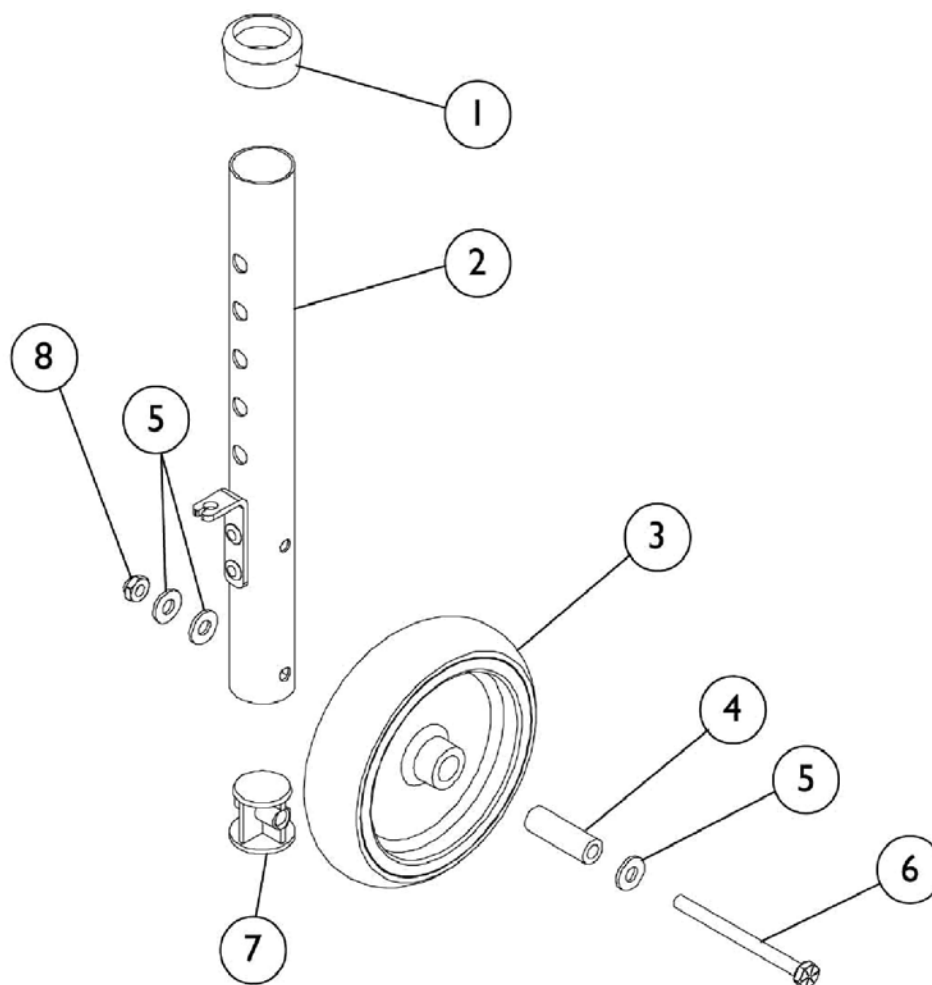

*Attachments Sold in Pairs
Single Side Shown*


Fixed Wheel Attachment (Rollator)

Model 65100

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
	A,B	1109140	NLA - Kit, Rear Wheel - Right	1	1
			Anti-Rattle, Grey (1-1/8")	1	1
			Extension, Leg	1	1
			Wheel (5" x 1-1/4")	1	1
			Bushing (1/4 x 1-3/8")	1	1
			NLA - Washer, Flat (1/4 x 5/8 x 1/16")	1	3
			NLA - Screw, Hex Head (1/4 -20 X 3")	1	1
			Plug, Black	1	1
			Locknut (1/4-20)	1	1
	A,B	1109141	Kit, Rear Wheel - Left	1	1
			Anti-Rattle, Grey (1-1/8")	1	1
			Extension, Leg	1	1
			Wheel (5" x 1-1/4")	1	1
			Bushing (1/4 x 1-3/8")	1	1
			NLA - Washer, Flat (1/4 x 5/8 x 1/16")	1	3
			NLA - Screw, Hex Head (1/4 -20 X 3")	1	1
			Plug, Black	1	1
			Locknut (1/4-20)	1	1
NOTE: A - Not Shown. B - Kit Includes Items 1-8.					

Platform Attachment

Model 6027

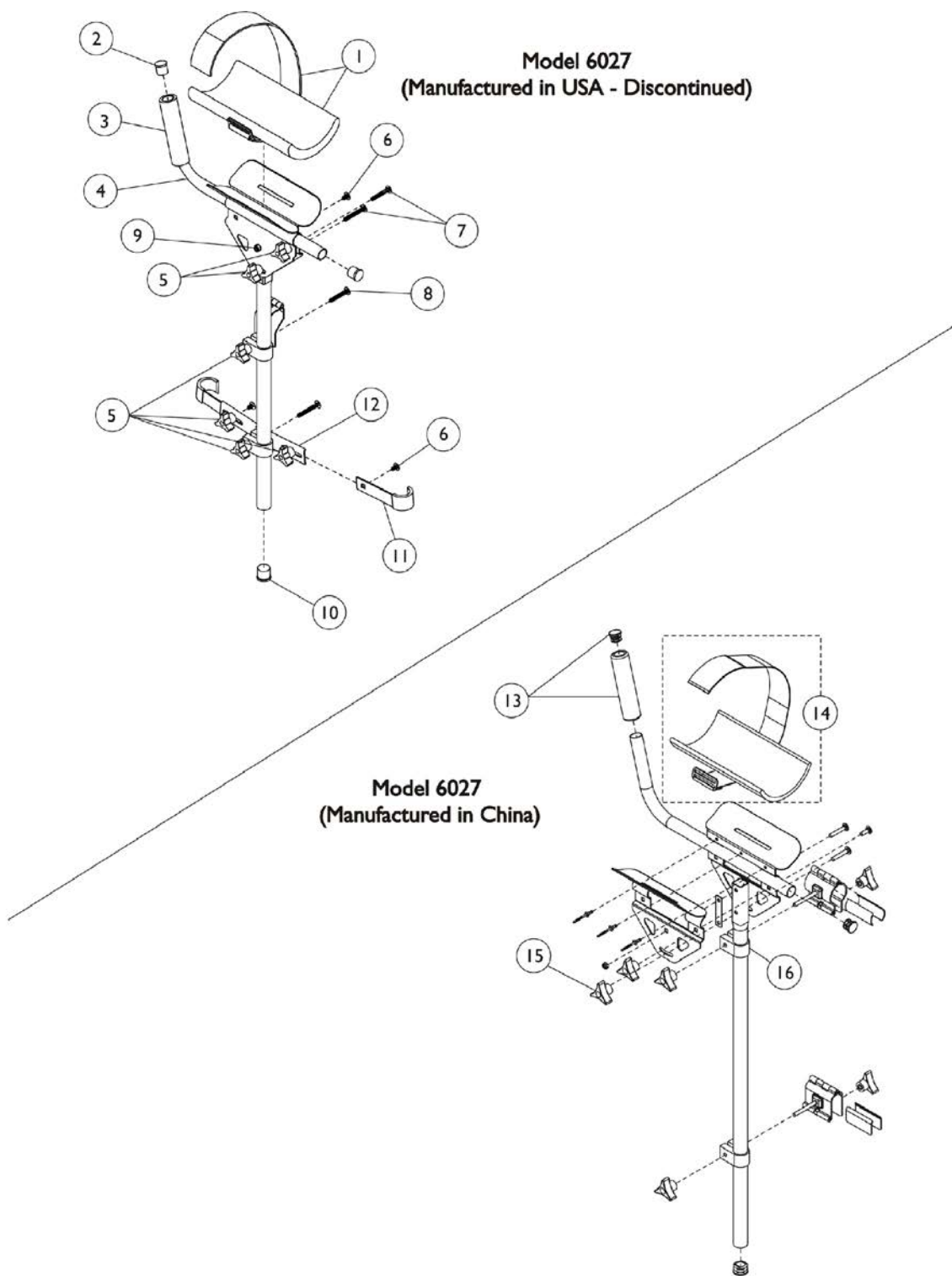

Platform Attachment Model 6027

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1071284	Arm Padding w/ Strap, Grey	1	1
2	B	1027096	Plug Button, Grey (7/8")	1	2
3	B	1016298	Grip, Push Handle	1	1
4	B	1056286	Tube, Arm	1	1
5	B	1061131	Knob with Threaded Insert, Black (1/4-20)	1	6
6	B	10123X006	Bolt, Carriage (1/4-20 x 3/4")	1	3
7	B	7930311-NLA	NLA - Package, Carriage Bolt (1/4-20 x 1-1/2")	25	1
8	B	1069015	Bolt, Carriage (1/4-20 x 2-1/4")	1	1
9	B	7930364	Package, Locknut (1/4-20)	25	1
10	B	1057287	Plug Button, Grey (1")	1	1
11	B	1068805	Slide w/ Rubber Protector	1	2
12	B	1068800	Bracket, Support, Bottom	1	1
13	A	1145517	Kit, Handgrip with Plug	1	1
14	A	1145516	Padding, Arm with Strap	1	1
15	A	1145515	Package, Knob	3	1
16		1068808	Clamp	1	1


NOTE: A - Use Only with Platform Attachment that is "Manufactured in China," with RX in the serial number.
B - Use Only with Platform Attachment that is "Manufactured in USA."

NOTE: When determining which service parts are needed, you will have to check whether it is Manufactured in China or the USA. The Platform Attachment can be used on all Walkers.


Fixed Wheel Attachment

Single with Glide Tips


Model 6270 - 3" Wheel
Model 6271 - 5" Wheel

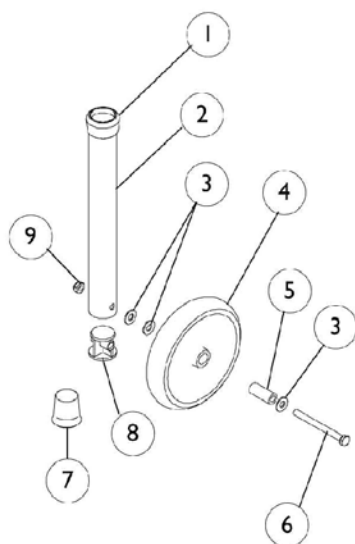

Model 6277 - Tall

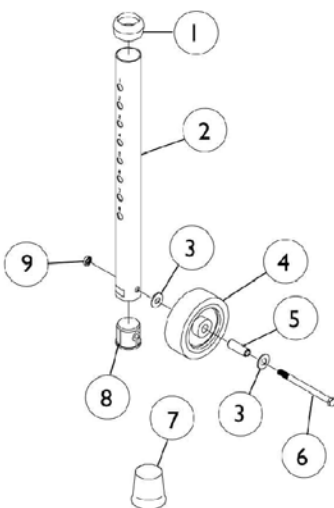

Model 6278 with
13" Rear Leg Extension

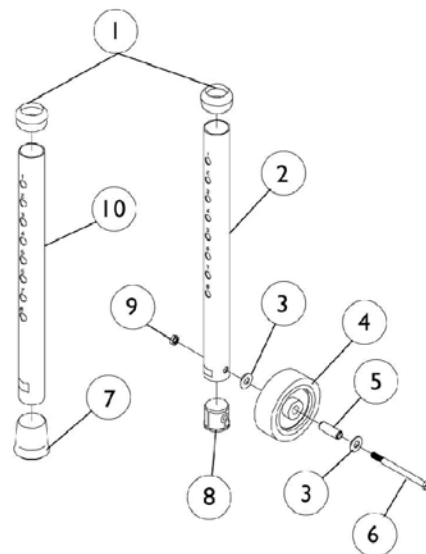

Attachments Sold in Pairs Single Side Shown

Fixed Wheel Attachment Single with Glide Tips


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B,C	1040655	Anti-Rattle, Grey (1-1/8")	1	4
2		1050684	Tube, Leg Extension (13")	1	2
3		1040292-NLA	NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	6
4	C,D,E,J	1039466-NLA	NLA - Wheel (3" x 1")	1	2
4	C,D,E	1130782-NLA	NLA - Package, Wheel (3")	10	1
4	B,G	1123534	Wheel (5" x 1-1/4")	1	2
5	A,C,E	1039465	Bushing (1/4 x 3/8 x 1/13")	1	2
5	B	1106067	Bushing (1/4 x 1-3/8")	1	2
6		1014412	Screw, Hex Head (1/4-20 x 2-1/2")	1	2
6	B	1008496	Screw, Hex Head (1/4-20 x 2-3/4")	1	2
7	F,H	6280	Package, Tip Glide, Grey (1-1/8")	24	1
7	F,I	1041531-NLA	NLA - Tip, Glide, Grey (1-1/8")	1	2
7		1130781	Kit, Glide Tips	10	1
8		1040278	Bushing (1")	1	2
9		60275X000	Locknut (1/4-20)	1	2
10	E	1040671	Tube, Leg Extension (13")	1	2
NOTE: A - To be used with Model 6270. B - To be used with Model 6271. C - To be used with Model 6277. D - To be used with Model 9270. E - To be used with Model 6278. F - Used on Rear Legs (1-1/8"). G - Due to style change, "After 2/6/02" Wheels should be ordered in pairs. H - Includes Instruction Sheet. I - Individual item no longer available, refer to Kit Part #1130781. J - Individual item no longer available, refer to Kit Part #1130782.					
NOTE: Due to 5" Wheel style change, please order in pairs after 2-6-02 for Model 6271.					

Leg Extensions

Rear 13" with Glide Tips

Model 6266 - Pair
Model 6275 - Set of Four Legs

Attachments Sold in Pairs
Single Side Shown

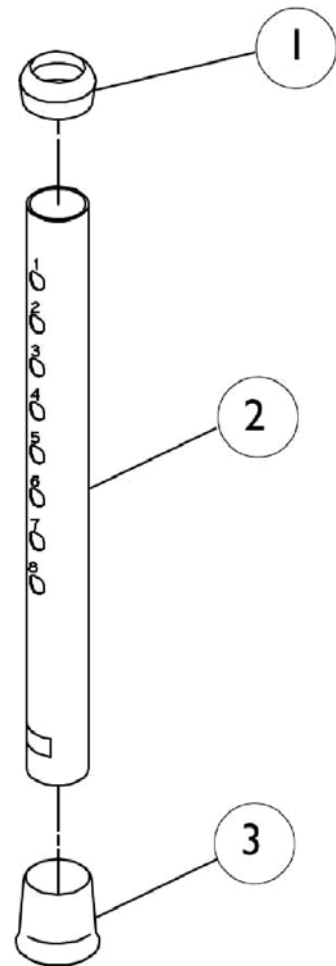

Leg Extensions



Rear 13" with Glide Tips

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1040655	Anti-Rattle, Grey (1-1/8")	1	4
2	A,B	1040671	Tube, Leg Extension (13")	1	2
3	A,C	6280	Package, Tip Glide, Grey (1-1/8")	24	2
3	A,D	1041531-NLA	NLA - Tip, Glide, Grey (1-1/8")	1	2
3		1130781	Kit, Glide Tips	10	1
3	B	1040656	Tip, Rubber, Dark Grey (1-1/8")	1	4
NOTE: A - To be used on Model 6266. B - To be used on Model 6275. C - Includes Instruction Sheet. D - Individual item no longer available, refer to Kit Part #1130781.					
NOTE: Attachments sold in pairs only.					

Swivel Wheel Attachments

With 13" Sure Glide Brakes

Model 6272 - 3" Wheel
Model 6268 - 5" Wheel

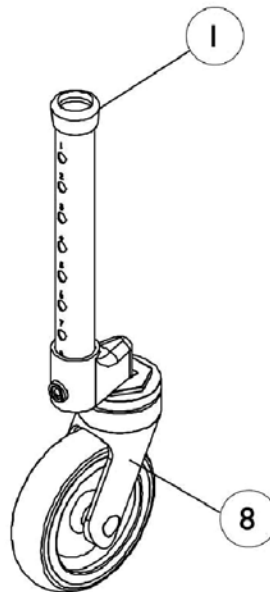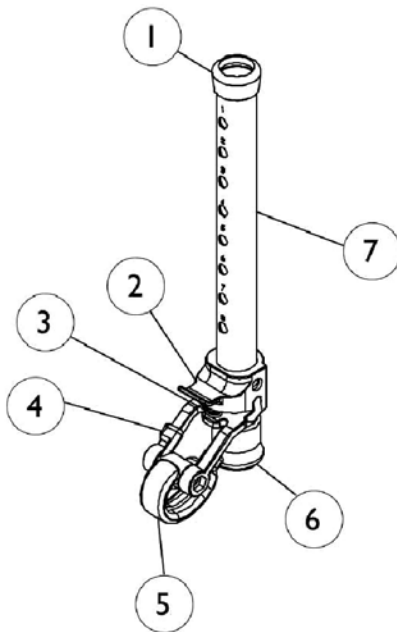


*Attachments Sold in Pairs
Single Side Shown*

Swivel Wheel Attachments With 13" Sure Glide Brakes


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1040655	Anti-Rattle, Grey (1-1/8")	1	4
2		1044942	NLA - Housing, Spring	1	2
3	A	1021734-NLA	NLA - Spring	1	2
3	B	1044944	Spring, Compression (31/64 x 1 x 3/64")	1	2
4		1044943	Fork	1	2
5		1053179	Glide	1	2
6	C	1040656	Tip, Rubber, Dark Grey (1-1/8")	1	2
7	B	1044929-NLA	NLA - Tube, Leg Extension (13")	1	2
8	E,G	1039469-NLA	NLA - Caster Assembly, Swivel (3")	1	2
8	H	1123541-NLA	NLA - Caster Assembly, Swivel (3")	1	2
8		1044509-NLA	NLA - Caster, Swivel (5")	1	2
8	I	1123542	Caster Assembly, Swivel (5")	1	2
	A,C,D	1040848-NLA	NLA - Sure Stop Sub-Assembly (13")	1	2
	A,C,D	1042876-NLA	NLA - Tube, Inside Glide	1	2
	A,C,D	1041532-NLA	NLA - Tip, Glide, Grey (3/4")	1	2
	A,C,D	1040840-NLA	NLA - Plug Button, Grey (7/8")	1	2
	C,D,F	1039493-NLA	NLA - Adapter, Bushing - Right	1	1
	C,D,F	1039494-NLA	NLA - Adapter, Bushing - Left	1	1
	C,D,F	21243X000	Screw, Phillips Pan Head (#6-20 x 3/8")	1	2
	C,D,F	1040984-NLA	NLA - Label - Right	1	1
	C,D,F	1040985-NLA	NLA - Label - Left	1	1
NOTE: A - Before 7/95. B - After 6/95. C - For 1-1/8" Tubing. D - Not Shown. E - Due to style change, when replacing 3" Wheels "Before 11/5/00" order in pairs. F - Before 8/99. G - Item no longer available, refer to Part #1123541. H - Replaces Part #1039469. I - Replaces Part #1044509.					

Swivel Wheel Attachments

Model 65100 - 5" Wheel

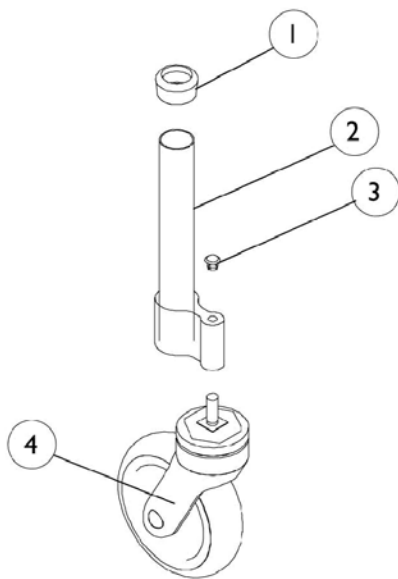

**Model 6267 - 5" Wheel
Model 6273 - 3" Wheel**

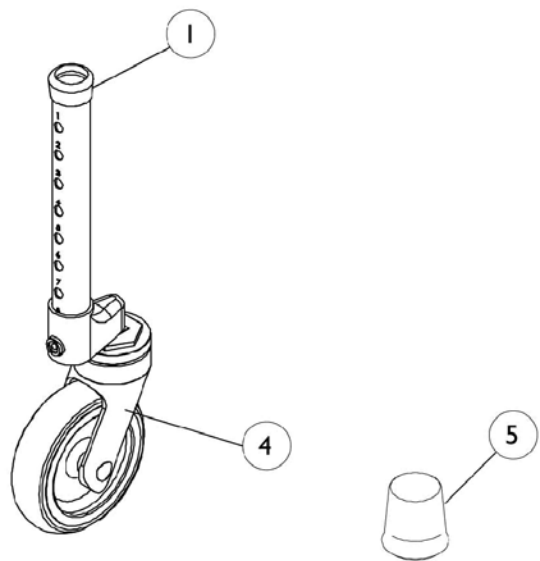

Attachments Sold in Pairs Single Side Shown



Swivel Wheel Attachments

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1040655	Anti-Rattle, Grey (1-1/8")	1	4
1	A	1063124	Anti-Rattle, Grey (1-1/8")	1	2
2		1107785-NLA	NLA - Extension, Leg	1	2
3		1109139	Plug, Black	1	2
4	B,F	1039469-NLA	NLA - Caster Assembly, Swivel (3")	1	2
4	G	1123541-NLA	NLA - Caster Assembly, Swivel (3")	1	2
4		1044509-NLA	NLA - Caster, Swivel (5")	1	2
4	H	1123542	Caster Assembly, Swivel (5")	1	2
5	E	6280	Package, Tip Glide, Grey (1-1/8")	24	2
5	I	1041531-NLA	NLA - Tip, Glide, Grey (1-1/8")	1	2
5		1130781	Kit, Glide Tips	10	1
	C,D	1039493-NLA	NLA - Adapter, Bushing - Right	1	1
	C,D	1039494-NLA	NLA - Adapter, Bushing - Left	1	1
	C,D	1040984-NLA	NLA - Label - Right	1	1
	C,D	1040985-NLA	NLA - Label - Left	1	1
	C,D	21243X000	Screw, Phillips Pan Head (#6-20 x 3/8")	1	2
NOTE: A - To be used on Model 65100. B - Due to style change, when replacing 3" Wheel "Before 11/5/00" order in pairs. C - Before 8/99. D - Not Shown. E - Includes Instruction Sheet. F - Item no longer available, refer to Part #1123541. G - Replaces Part #1039469. H - Replaces Part #1044509. I - Individual item no longer available, refer to Kit Part #1130781.					

Swivel Wheel Attachments

With 13" Leg Extension and Glide Tips

Model 6269 - 3" Wheel

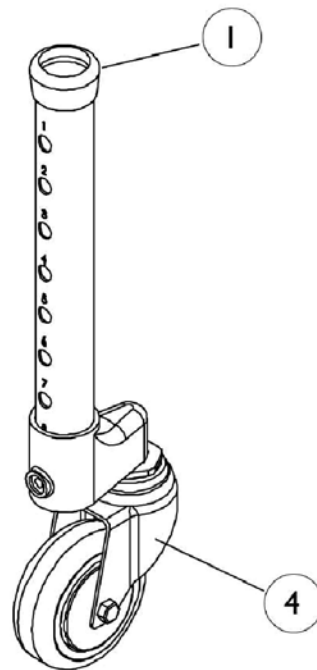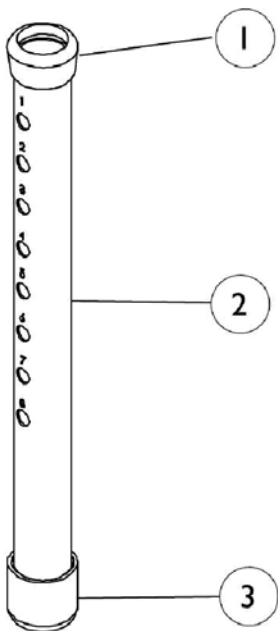

*Attachments Sold in Pairs
Single Side Shown*

Swivel Wheel Attachments


With 13" Leg Extension and Glide Tips

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1040655	Anti-Rattle, Grey (1-1/8")	1	4
2		1040671	Tube, Leg Extension (13")	1	2
3	D	6280	Package, Tip Glide, Grey (1-1/8")	24	2
3	G	1041531-NLA	NLA - Tip, Glide, Grey (1-1/8")	1	2
3		1130781	Kit, Glide Tips	10	1
4	A,E	1039469-NLA	NLA - Caster Assembly, Swivel (3")	1	2
4	F	1123541-NLA	NLA - Caster Assembly, Swivel (3")	1	2
	B,C	21243X000	Screw, Phillips Pan Head (#6-20 x 3/8")	1	2
	B,C	1040984-NLA	NLA - Label - Right	1	1
	B,C	1040985-NLA	NLA - Label - Left	1	1
	B,C	1039493-NLA	NLA - Adapter, Bushing - Right	1	1
	B,C	1039494-NLA	NLA - Adapter, Bushing - Left	1	1
NOTE: A - Due to style changes, when replacing 3" Wheels "Before 11/5/00" order in pairs. B - Not Shown. C - Before 8/99. D - Includes Instruction Sheet. E - Item no longer available, refer to Part #1123541. F - Replaces Part #1039469. G - Individual item no longer available, refer to Kit Part #1130781.					

Brake Attachment (Rear Wheel)

Model 6264 - 10" Leg Extension

Model 6265 - 13" Leg Extension

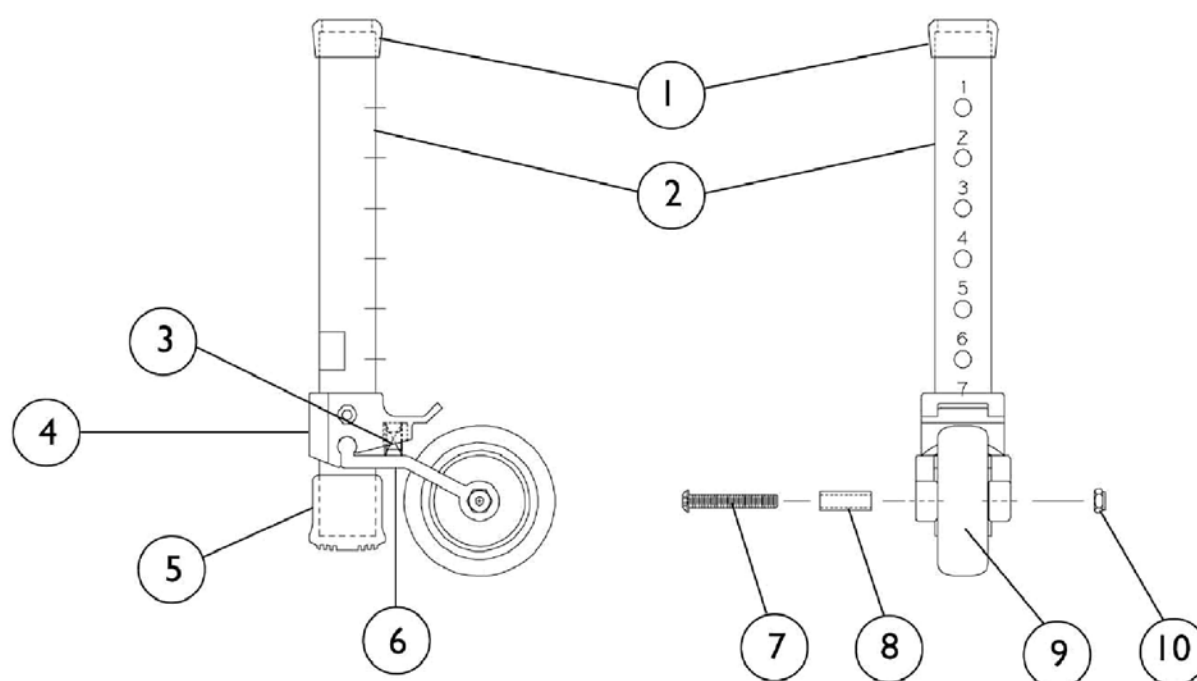

*Attachments Sold in Pairs
Single Side Shown*


Brake Attachment (Rear Wheel)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1040655	Anti-Rattle, Grey (1-1/8")	1	2
2	A	1044713-NLA	NLA - Tube, Leg Extension (10")	1	2
2	B	1044929-NLA	NLA - Tube, Leg Extension (13")	1	2
3		1044944	Spring, Compression (31/64 x 1 x 3/64")	1	2
4		1044942-NLA	NLA - Housing, Spring	1	2
5		1040656	Tip, Rubber, Dark Grey (1-1/8")	1	2
6		1044943	Fork	1	2
7		7930301	Package, Button Head Cap Screw (1/4-20 x 1-3/4")	25	2
8		1039465	Bushing (1/4 x 3/8 x 1/13")	1	2
9	C	1039466-NLA	NLA - Wheel (3" x 1")	1	2
9		1130782-NLA	NLA - Package, Wheel (3")	10	1
10		7930364	Package, Locknut (1/4-20)	25	2
NOTE: A - To be used on Model 6264. B - To be used on Model 6265. C - Individual item no longer available, refer to Kit Part #1130782.					

Sure Glide Brakes

Model 6274 - 10" Leg Extension

Model 6263 - 13" Leg Extension

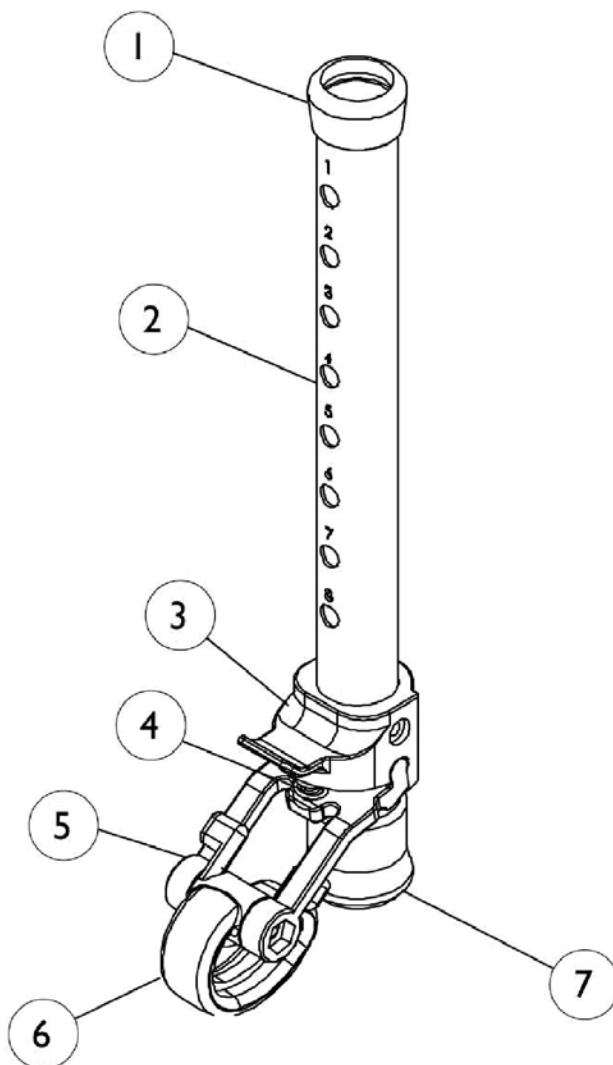


*Attachments Sold in Pairs
Single Side Shown*

Sure Glide Brakes

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1040655	Anti-Rattle, Grey (1-1/8")	1	2
2	A	1040847-NLA	NLA - Sure Stop Sub-Assembly (10")	1	2
2	B	1044713-NLA	NLA - Tube, Leg Extension (10")	1	2
2	B	1044929-NLA	NLA - Tube, Leg Extension (13")	1	2
3		1044942-NLA	NLA - Housing, Spring	1	2
4	A	1021734-NLA	NLA - Spring	1	2
4	B	1044944	Spring, Compression (31/64 x 1 x 3/64")	1	2
5		1044943	Fork	1	2
6		1053179	Glide	1	2
7		1040656	Tip, Rubber, Dark Grey (1-1/8")	1	2
	A,C	1021733-NLA	NLA - Tube, Inside Glide	1	2
	A,C	1040840-NLA	NLA - Plug Button, Grey (7/8")	1	2
	A,C	1041532-NLA	NLA - Tip, Glide, Grey (3/4")	1	2
NOTE: A - Before 7/95. B - After 6/95. C - Not Shown.					

Tray/Seat (Rollator)


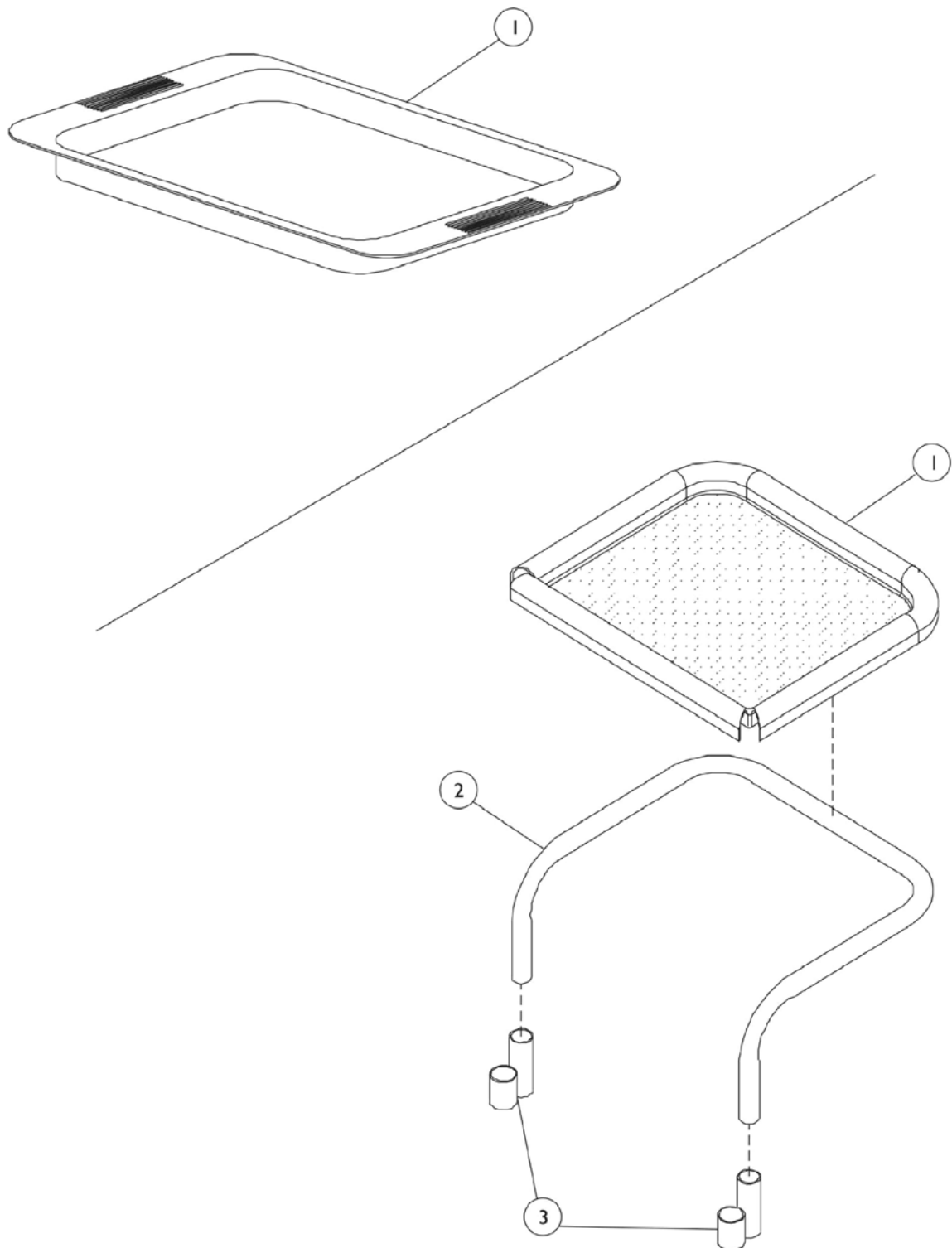


Tray/Seat (Rollator)

Tray Rollator

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1076174	Tray	1	1
1	B	65403-NLA	NLA - Tray	1	1
1	C,D	429306	Tray	1	1
	C,E,F	450014-NLA	NLA - Tray	1	1
2	D,G	429304	Support, Tray	1	1
2	C,D	450044-NLA	NLA - Tray Support	1	1
3	D,G	1075333-NLA	NLA - Brackets, Chrome	1	2
	E,H	65408	Tray	1	1
NOTE: A - To be used with Models 65500, 65600, 65700, 65800 and 65900. B - To be used with Model 65400. C - Before 3/98. D - For 3 Wheeled Rollator Tray. E - Not Shown. F - For 4 Wheeled Rollator Tray. G - Before 5/11/98. H - For use on all Rollite models.					

Tray/Seat (Rollator)

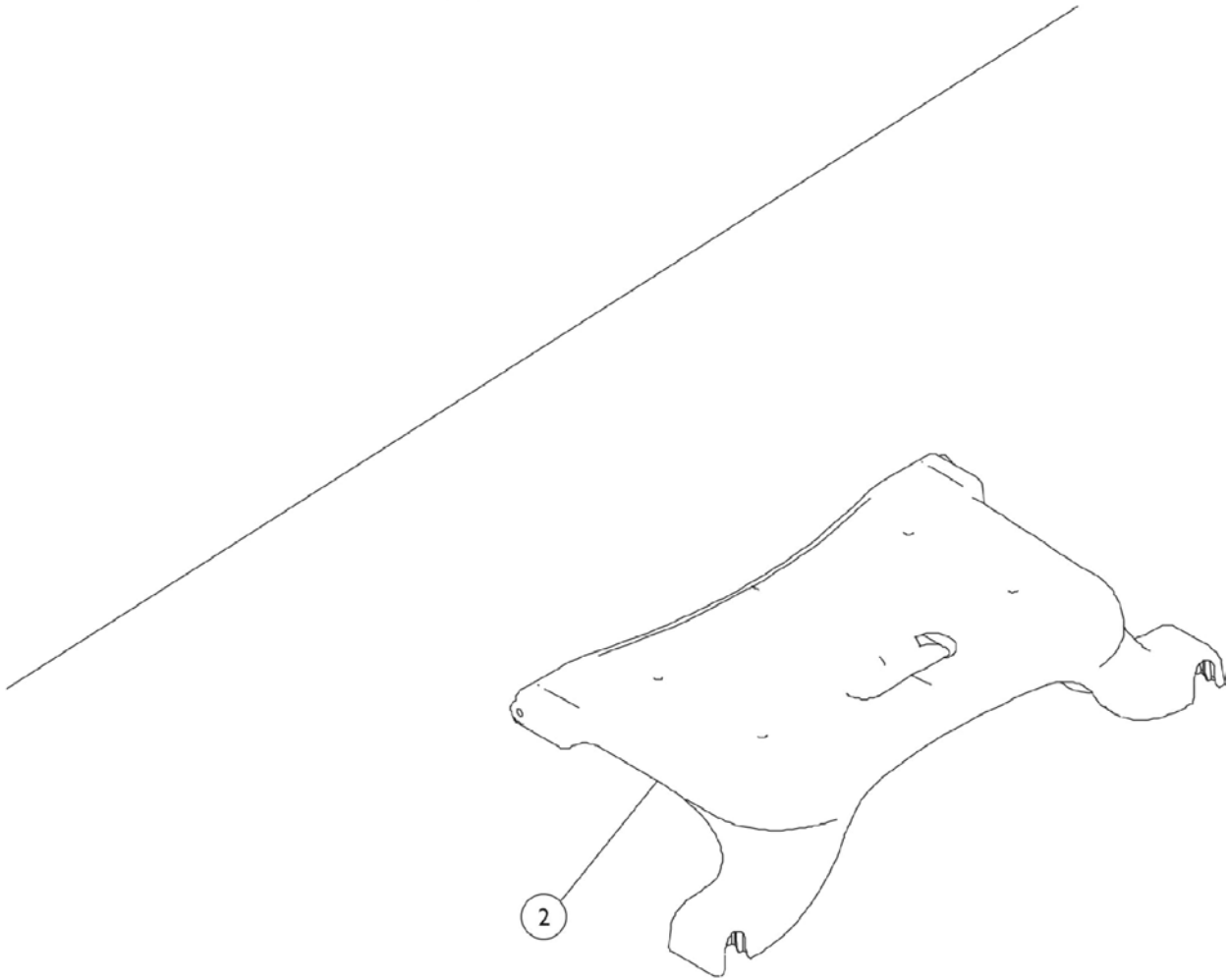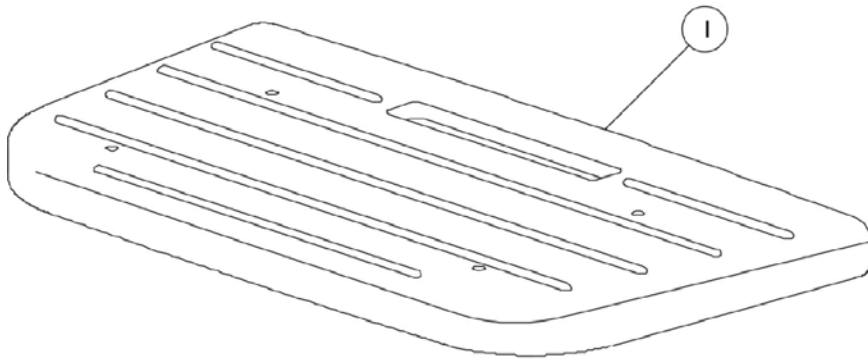


Tray/Seat (Rollator)

Seat Rollator

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1076172-NLA	NLA - Seat	1	1
1	B	1076173-NLA	NLA - Seat	1	1
2	C	1107737	Seat	1	1
2	A,D,E	1076178-NLA	NLA - Seat Hardware	1	1
2	B,E,F	1076179-NLA	NLA - Hardware, Seat	1	1
NOTE: A - To be used with Model 65420 and 65500. B - To be used with Model 65600. C - To be used with Model 65100. D - Includes (8) Screws, (2) Brackets and (2) Plates. E - Not Shown. F - Includes (16) Screws, (4) Brackets and (4) Plates.					

Basket/Bag (Rollator)


Fits Model 65400

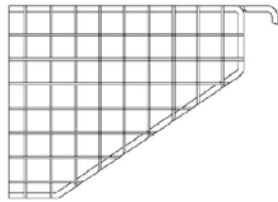

Fits Model 65400



Fits Model 65420

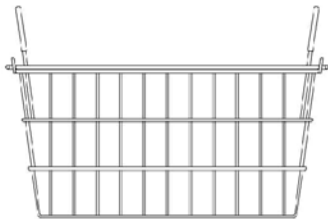

Fits Model 65500




Fits Model 65600




Fits Models 65700, 65800, 65900



Fits Model 6540, 6540-PB

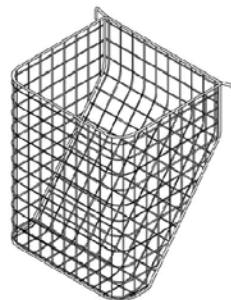

Fits Model 6542-B, 6542-BC

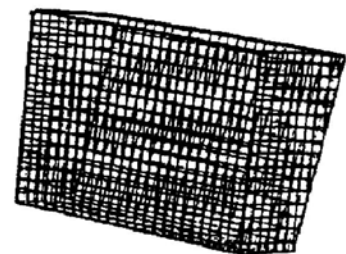


Basket/Bag (Rollator)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	65401-NLA	NLA - Basket Assembly	1	1
1	A	65404	Bag, Tote	1	1
1	B	1076168-NLA	NLA - Basket	1	1
1	C	1076169-NLA	NLA - Basket	1	1
1	D	1076170-NLA	NLA - Basket	1	1
1	E	1098988-NLA	NLA - Basket	1	1
1	F	6541-NLA	NLA - Basket	1	1
1	G	450035-NLA	NLA - Basket	1	1
1		65110	Basket, 65100, 65100R and 68100-TA	1	1
1		65109	Basket, 65100-JR, 65100R-JR and 68100	1	1
NOTE: A - To be used with Model 65400. B - To be used with Model 65420. C - To be used with Model 65500. D - To be used with Model 65600. E - To be used with Model 65700, 65800 and 65900. F - To be used with Model 6540 and 6540-PB. G - To be used with Model 6542-B and 6542-BC.					

Glide Tips

Model 6280 24 Package

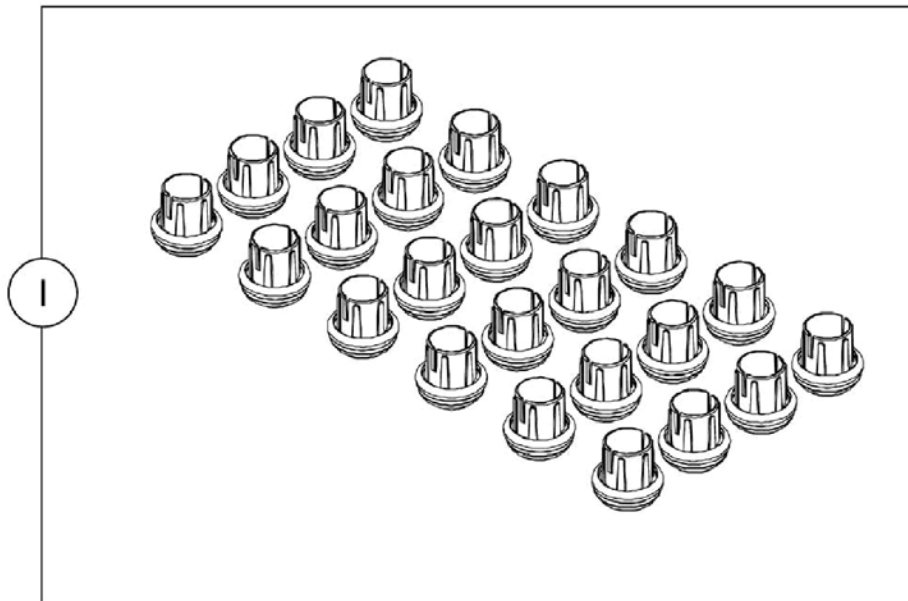

Glide Tips

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	6280	Package, Tip Glide, Grey (1-1/8")	24	2
1	A,C	1041531-NLA	NLA - Tip, Glide, Grey (1-1/8")	1	2
1		1130781	Kit, Glide Tips	10	1
NOTE: A - Single Package. B - Includes Instruction Sheet. C - Individual item no longer available, refer to Kit Part #1130781.					

Invacare Corporation

www.invacare.com



One Invacare Way
Elyria, Oh USA
44036-2125
800-333-6900

Technical Support
800-832-4707
Florida Technical Support
407-321-5630

Customer Service
800-333-6900

© 2018 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted. (Revised 1/8/18)

Form No. 92-22