
# Disposable Contour Deluxe™ Nasal Mask & Headstrap

## Instructions for Use

### Warnings

- ⚠ The Disposable Contour Deluxe Nasal Mask is designed for use with Respironics CPAP or bi-level systems that are recommended by a physician or respiratory therapist.
- ⚠ This mask must be used with a Respironics exhalation device and 22 mm flexible tubing for safe operation. **DO NOT** block or try to seal the openings in the exhalation device.
- ⚠ Apply this mask to a patient **ONLY** when the CPAP or bi-level system is turned on and operating properly. In normal CPAP system operation, new air from the system flushes out exhaled air through vent holes or exhalation ports in the mask (or connectors). If the CPAP system is off or not operating properly, the patient may rebreathe exhaled air. Rebreathing exhaled air for longer than several minutes can, in some circumstances, lead to suffocation.
- ⚠ The mask has one pressure pick-off port with a removable port cap that may be used to monitor airway pressure. Close the port cap when it is not in use.
- ⚠ Some users may experience skin redness. If this occurs, consult your doctor.

**SINGLE ADULT PATIENT  
USE ONLY**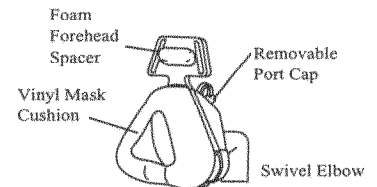


### Cautions

- ◆ Federal law restricts this device to sale by or on the order of a physician.
- ◆ When using with the Respironics disposable circuit, do not expose the tubing to temperatures above 130°F.


#### Step 1

Hold the mask lightly against the patient's face, and slide the headstrap over his/her head. Use the tabs on the headstrap to adjust strap length.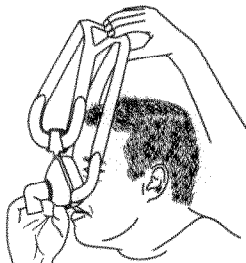

#### Step 2

Connect the disposable circuit to the swivel elbow. Turn the CPAP or bi-level system on.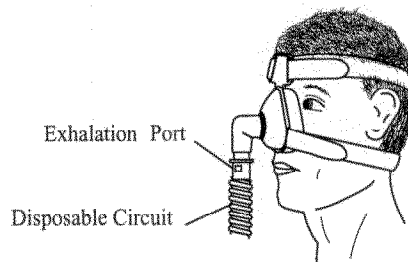

#### Step 3

Instruct the patient to recline and breathe normally through his/her nose with lips closed. Unhook and gradually tighten the straps until the mask has minimal leaks and fits comfortably. **Do NOT** overtighten the straps. Overtightening can cause or worsen leaks.


**NOTE:** The headstrap has two sets of adjustable tabs. For larger head sizes, use the tabs closest to the mask (front tabs). For smaller head sizes, use the tabs farthest from the mask (rear tabs). The extra tabs can be removed by cutting with scissors.

### To remove the headstrap and mask

Simply unfasten either bottom strap and slide the mask and headstrap over the patient's head.

### Tips for Mask Comfort

- Thoroughly clean the mask and patient's face before each use.
- Make sure the mask is the correct size for the patient's face.
- Do not overtighten the headstrap. Overtightening can irritate the patient's face, increase leaks, or cause damage to the mask.
- Replace the mask if the cushion becomes hardened or deformed.

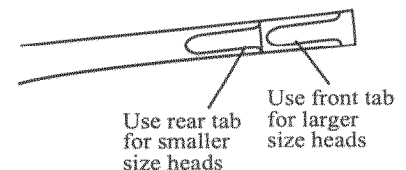### Reorder Numbers

1006194	Large (5/pack)
1006195	Medium (5/pack)
1006196	Small (5/pack)
1007365	Disposable Headstrap (5/pack)

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