



## Administrative assessment, 72 h:

Log on to the website at [tenaidentifi.sca.com](http://tenaidentifi.sca.com)


Register the caretaker on the website.

Register the logger to the caretaker.

Start the assessment on the website.

Confirm that the logger is fully charged.

## Cleaning the logger:

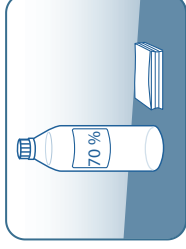


Clean the logger with alcohol solution after every change.




Do not use dishwasher or autoclave.


## Handling of incontinence protection and logger:

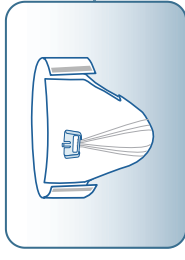


Attach the protection on the caretaker, the protection works the same way as a TENA Slip.

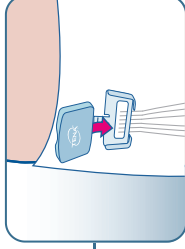


Attach the logger into the connector on the protection. A click-sound tells you when it is correctly mounted.

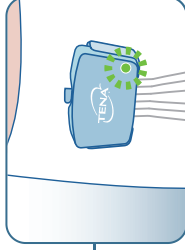

Green light that stops after 5 sec = correctly mounted.\*

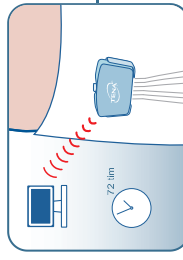

During the following 72 hours, change incontinence protection when needed and re-use the logger.

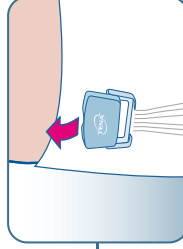


Detach the logger for re-use when product change is needed.




Do not throw away the logger.

\*If the lamp on the logger doesn't light up, test the logger in a another protection. If the lamp lights up after that, replace the protection and place the defect one in the collection box.


## Charging the logger:

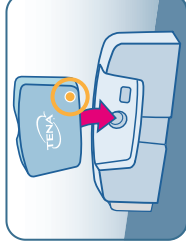

Lamp steady orange = Low battery. Place the logger into the charger.

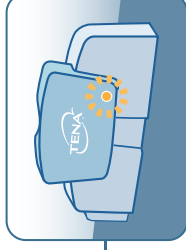

Lamp flashing orange = Charging.

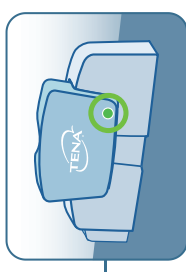

Steady green lamp = Charging complete.

## Light indication while in assessment protection:



Flashing green light =  
Correctly mounted



Steady orange light =  
Low battery

## Light indication while in charger unit:



Light flashing orange =  
Charging



Steady green light =  
Charging complete

### IMPORTANT

Read all instructions carefully and completely before initializing an assessment. If in doubt, contact SCA for support.

Save the instruction manual for future reference. Avoid placing the equipment in direct sunlight as it will decolor the plastic.

Always clean the logger unit in accordance with this guide when completing an assessment.



### CAUTIONS

**Warning!** Only use the provided power supply and only with the provided charger unit.


**Warning!** Only use a maximum of 3 charger units connected in series to a single power supply.

**Warning!** If anything is (or seem to be) damaged or broken, return to supplier for service.


**Warning!** Always charge the battery before storage exceeding 3 months. A drained battery will severely decrease the battery lifetime.

User manual


www.TENA.com

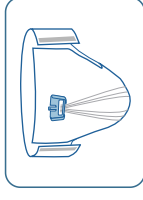

Protection  
with Connector

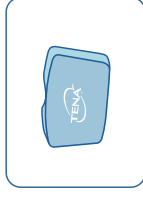

Logger

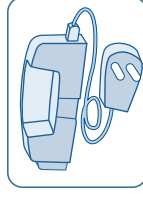

Charger  
with Cable

# TENA Identifi e-Assessment



SCA Global Hygiene Category  
SE-405 03 Gothenburg, Sweden  
Phone: +46 31 746 00 00  
Fax: +46 31 746 19 32

**c/o Life** Because our products make life easier for you and for millions of people around the world. Because our resources and the way we work are natural parts of the global lifecycle. And because we care.

