# Chapter 10

# Image Management

The Image Management screen ([Figure 10-1](#)) shows all the images stored in the selected database source. This screen is used to manage images, copy images to exams, transfer images to network hosts, or save images on USB storage device.

Topics covered include:

- [Overview](#)
- [Selecting Image Database Source](#)
  - [Loading Images from a Network Host](#)
  - [Loading Images from an USB disk](#)
- [Searching List](#)
- [Sorting by Column](#)
- [Opening Exams and Images](#)
- [Copying Exams and Images](#)
  - [Copying Exams to a Network Host](#)
  - [Copying Images to Another Exam](#)
  - [Copying Exams to a USB drive](#)
- [Deleting Exams, Series, or Images](#)
  - [Locking Exams from Deletion](#)
- [Making Patient Anonymous \(De-Identify\)](#)

## Overview

The majority of the Image Management screen is devoted to the exam list. Images are organized by exam. Each exam is a row. If multiple exams were acquired in the same session, each exam has its own row on the list. The exam expands to show the series.

Within each exam are “series” of images. A series is a collection of one or more images acquired in a session. Each protocol is a series. A new series is created when a completed exam is appended and every time an image is re-processed (in review mode).

There are two types of image series: raw and processed. Raw images are the exact images that were acquired. Processed images are the raw images with specific processing and image adjustments (such as brightness and contrast) applied. It is possible to create several processed images from one raw image.

Individual images reside within the series. Double-clicking on a series or clicking the [+] button opens the image details section of the worklist. In the Image Details section, each row is an image. Selecting a row makes a small preview image, or “thumbnail,” appear. The image may be opened for viewing or adjustment or deleted.

Selecting multiple exams on the Image Management screen is different than on the Worklist.

- The Image Management screen allows you to select exams that do not have the same Patient ID or Patient Name.
- To select a contiguous group of exams: Hold down the SHIFT key on the keyboard. Click and drag with the mouse to select the exams.
- To select non-contiguous exams: Hold down the CTRL key on the keyboard. Click on individual exams to select.

Figure 10-1 Image Management

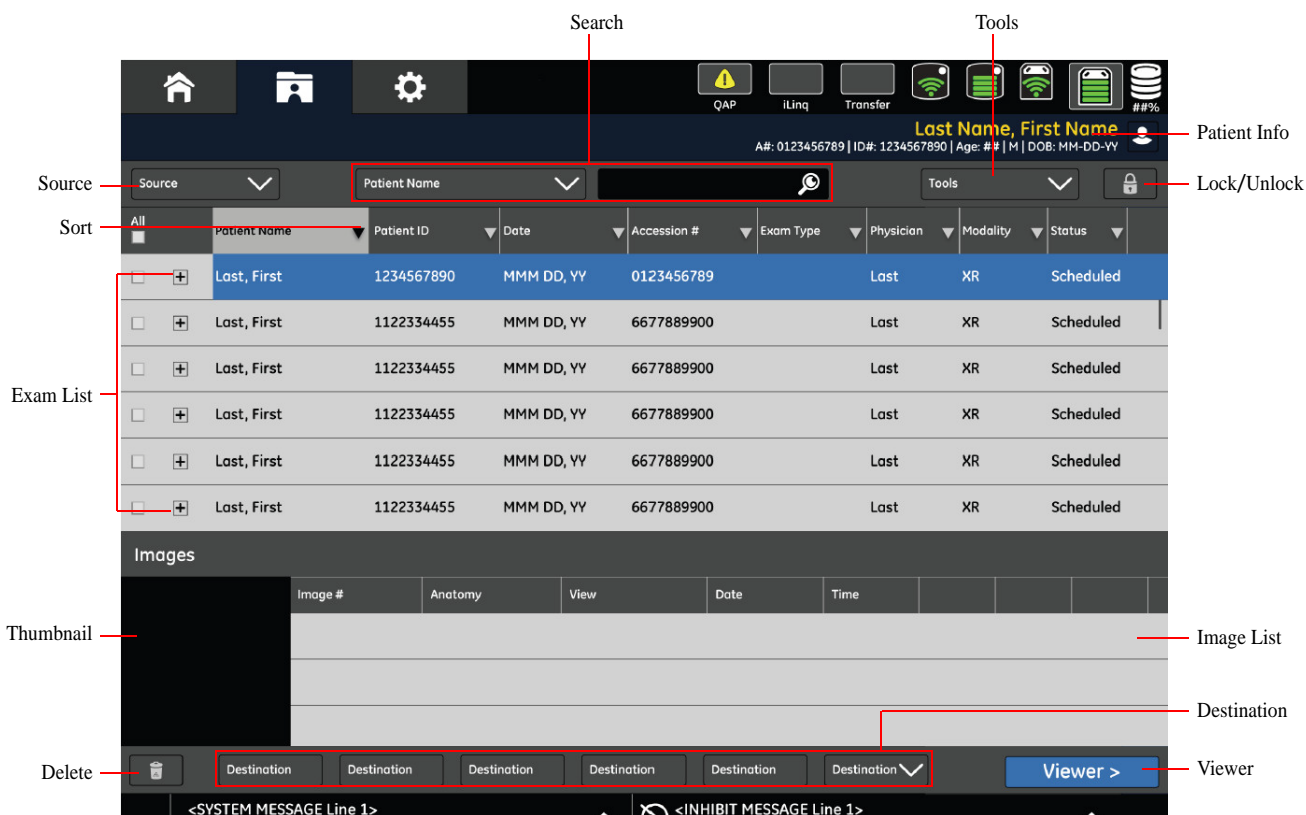
Table 10-1 Image Management functions

Function	Description
[Patient Information]	Shows the Patient Information screen for the selected procedure. <b>Note:</b> Patient Information cannot be edited once an exam has started. More information, please refer to <a href="#">Chapter 8: Worklist-Adding/Editing/Viewing Patient Information</a> .
[Source]	Selects the source of images to view (e.g., the local workstation, or a USB drive). <ul style="list-style-type: none"> <li>• 点击 [Local] 按钮显示系统中所有的检查。</li> <li>• 点击 [USB] 按钮显示插入 USB 盘中所有的检查。</li> </ul> 更多信息请参考 <a href="#">Selecting Image Database Source</a> 小节。
[Sort by Column]	For information about how to sort the column, please refer to <a href="#">Sorting by Column</a> .

Function	Description
[Search By]	Searches for procedures by the selected the column name in the drop-down list and the search criteria entered into the text box. Refer to <a href="#">Searching List</a> .
[Image Tools]	<p>Performs various functions on selected exams. Available options are:</p> <ul style="list-style-type: none"> <li>• Copy Exam: Allows all exam images for a patient to be copied to another patient, to another location, or to USB drive. More information, refer to <a href="#">Copying Exams and Images</a>.</li> <li>• De-identify: Makes the patient anonymous (removes all identifying information, including Name, ID, and accession number). Refer to <a href="#">Making Patient Anonymous (De-Identify)</a>.</li> <li>• SMPTE: Allows services personnel to access the SMPTE pattern for system calibration.</li> <li>• CBT: If available, launches Computer Based Training for the system. If Computer Based Training is not available, the option is disabled.</li> <li>• PACS Test Images: Loads a set of calibration images that are used to test the quality of images sent to PACS or printers.</li> <li>• TG18 Samples: Loads a set of test images that are used to calibrate the display monitor.</li> </ul> <p><b>Note:</b> It is not expected that you will need to access the SMPTE pattern or test images during the course of a normal day. Test images are typically used to calibrate the system or to determine the cause of quality problems.</p>
[Lock/Unlock]	Locks the selected exams from deletion. If a locked exam is selected, the button name changes to [UNLOCK]. [UNLOCK] removes the lock from the selected exams.
[Exam, Series, and Image List]	Lists the images saved in the local database categorized by exams and series. The following information is displayed: <ul style="list-style-type: none"> <li>• Exams: The exams saved in the local database.</li> <li>• Series detail: The series for the selected exam.</li> <li>• Image list: The images for the selected series.</li> <li>• Image preview: A representative thumbnail of the selected image.</li> </ul>
[Delete]	Deletes the selected exams or images from the local database.
[Destination]	Selects where images are to be copied or saved. Refer to <a href="#">Chapter 16: 设置参数</a> for more information about how to configure the different storage destination.

Function	Description
[Viewer]	Opens the Image Viewer screen and shows the images in the selected series. Images may be adjusted on the Image Viewer screen. Refer to <a href="#">Chapter 11: Image Viewer</a> for more information. <b>If Multiple Patient Print function is not activated, this icon will be unavailable when you select multiple patient images.</b>

## Selecting Image Database Source

The Source buttons allow you to load and work with images from different locations, such as a USB drive, network host, or the local workstation.

**Figure 10-2** Image sources

- Local: Shows exams and images stored on the local workstation.
- USB : 显示保存在 USB 存储盘上的检查和图像。
- Host :

### Loading Images from a Network Host

Follow this process to access images stored on a network host.

1. Select the network host from the drop-down list, if necessary.
  - ♦ The filter screen appears. Refer to [Chapter 8: Worklist](#) for more information.
2. Complete the Filter screen as appropriate.
3. Click [OK].
  - ♦ A message appears: "Retrieving requested information."
  - ♦ The Image Management screen list updates with exams from the selected source.

### Loading Images from an USB disk

Follow this process to access images stored on a USB disk.

1. 将有图像的 USB 存储器插入主机上的 USB 接口。
2. 点击 [USB].
  - ♦ The exam list updates to show the images stored on the USB disk.
3. Select the exams.
4. Copy the exams to the Local database. Refer to [Copying Exams and Images](#) for more information.
5. Open the exams from the Local database.

**Note:** If the exams are being viewed on a computer that has the DICOM viewer installed, images may be viewed directly from the USB disk.

## Searching List

The Search By feature finds procedures by column.

**Figure 10-3** Search by

1. Click the button on the Search By drop-down list to select the column you want to search.
  - ◆ If the column you want is already selected, begin at step 3.
2. Select the column. For example, Patient ID.
  - ◆ The list automatically sorts the selected column.
3. Type the search criteria into the text box. For example, you are looking for patients whose names begin with "J" , so you would type "J" into the text box.

**Note:** The text box is not case sensitive.


4. Continue typing the search criteria.
  - ◆ The list automatically selects the first procedure that matches what you have typed into the text box. This is called an incremental search.
  - ◆ If no procedures match what you have typed, the list de-selects all procedures and places the closest match at the top of the Worklist.

## Sorting by Column

Sorting allows you to organize the procedures by the column of your choice.

1. Click on the column heading you want to sort, or choose the column in the Search By drop-down list. For example, you want to see all the procedures that have a status of "Suspended", so you click on the "Scheduled Status" column heading.
  - ◆ An arrow appears in the column heading to indicate which column is currently being sorted.
2. Click the column heading again to switch between ascending and descending order.
  - ◆ An up-pointing arrow indicates that the column is sorted in ascending order. That is, sorted in alphabetical order or numerical order from smallest to largest.
  - ◆ A down-pointing arrow indicates that the column is sorted in descending order. That is, sorted in reverse alphabetical order or numerical order from largest to smallest.

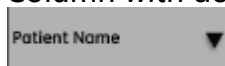**Figure 10-4** Column with descending sort

## Opening Exams and Images

Follow this process to open exams and images for viewing.

1. Double-click the exam to open it (or, select the exam and press ENTER on the keyboard).
  - ♦ The series for the exam expands below the exam.
2. Double-click the series to open it (or, select the series and click [+] image details).
  - ♦ The image detail opens.
3. Select the image.
  - ♦ A preview thumbnail appears.
4. Double-click on the image (or, select the series and click [VIEWER]).
  - ♦ The selected series opens on the Image Viewer screen with the selected image displayed.

## Copying Exams and Images

Exams (including all series and images) may be copied to a network host or to a USB drive. The images from an exam may be copied to another exam.

**Note:** Copying exams does not remove the exam from its original location.

### Copying Exams to a Network Host

Exams may be copied to a configured network location. Refer to [Chapter 16: 设置参数 - 网络连接](#) for information about configuring network hosts.

1. Select the exams to copy.
2. Click [DESTINATION].
  - ♦ A message appears: "Images will be copied to the selected exam. Images will not automatically be removed from the source exam."
3. Click [OK].
  - ♦ If there is a problem and the exams cannot be copied to the selected network host, a message appears: "The network destination is not responding. If the problem persists contact your network administrator."
  - ♦ Click OK to close the message. Try copying the exams at a later time.
  - ♦ To see the status of the exams being copied, click [TRANSFER LOG] at the top of the Image Management screen.
  - ♦ The Transfer Log screen appears.
  - ♦ Click [CLOSE] to close the screen and return to the Image Management screen.

Figure 10-5 Transfer Status

A#: 0123456789 | ID#: 1234567890 | Age: ## | M | DOB: MM-DD-YY

### Transfer Status

Source

Patient ID	Name	Exam	Submitted	Source	Destination	Status

### Completed

Patient ID	Name	Exam	Submitted	Source	Destination	Status

[Close](#)

<SYSTEM MESSAGE Line 1>      <INHIBIT MESSAGE Line 1>

## Copying Images to Another Exam

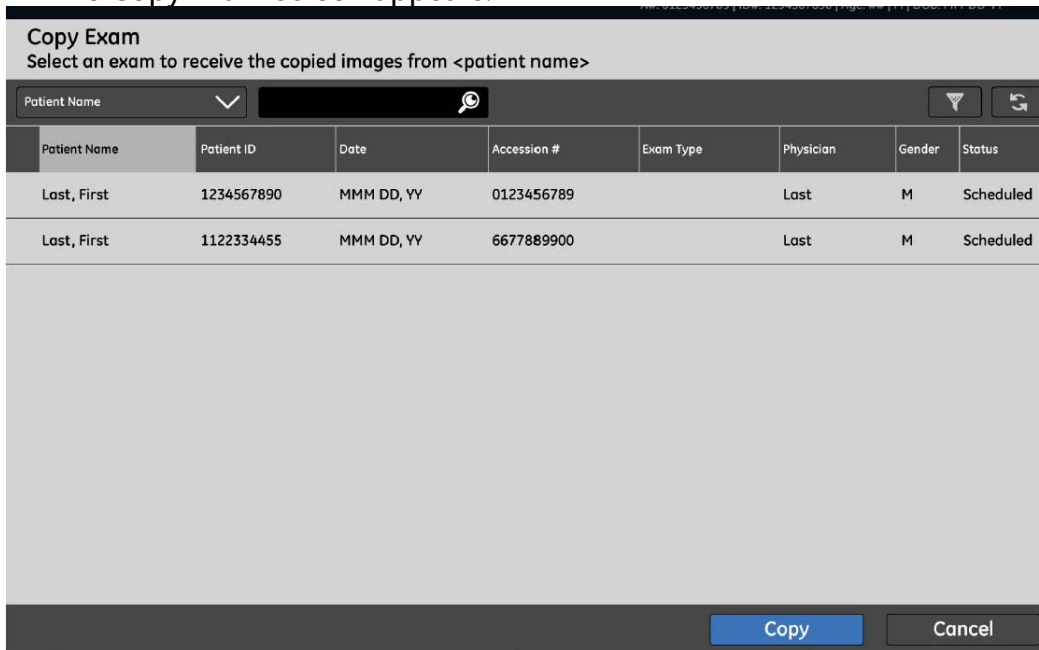
This process is used to copy images from one exam to another. Copying images is used to consolidate the images from multiple exams for a single patient or to reconcile patients to exams. For example, images taken for an emergency exam can be assigned to the patient's real name once the patient's information is known or if images were acquired for the wrong Worklist entry.

**Note:** You can only copy the images from one exam at a time.

1. If you are assigning an emergency exam to a patient, you may need to add the patient to the Worklist first.
  - a) On the Worklist screen, click [ADD PATIENT].
  - b) Enter the patient's information. Refer to [Chapter 8: Worklist-Adding/Editing/Viewing Patient Information](#).
  - c) Click [SAVE].
2. On the Image Management screen, select the exam to be copied.

3. Switch the [IMAGE TOOLS] button to COPY EXAM, if necessary.

- ◆ The Copy Exam screen appears.

Patient Name	Patient ID	Date	Accession #	Exam Type	Physician	Gender	Status
Last, First	1234567890	MMM DD, YY	0123456789		Last	M	Scheduled
Last, First	1122334455	MMM DD, YY	6677889900		Last	M	Scheduled

- ◆ The Copy Exam screen shows all available exams for the current location and all Worklist entries with the status of "Scheduled" .
4. Search or Filter the exam list to locate the destination exam.
5. Select the exam where you want the images copied to.
6. Click [OK].
- ◆ A message appears: "Images will be copied to the selected exam. Images will not automatically be removed from the source exam."
7. Click [OK].
- ◆ The message closes.
  - ◆ A message appears: "Retrieving information. Please wait."
  - ◆ [CANCEL] stops the copy process, closes the message, and returns you to the Image Management screen.
  - ◆ All series and images are copied to the exam.


## Copying Exams to a USB drive

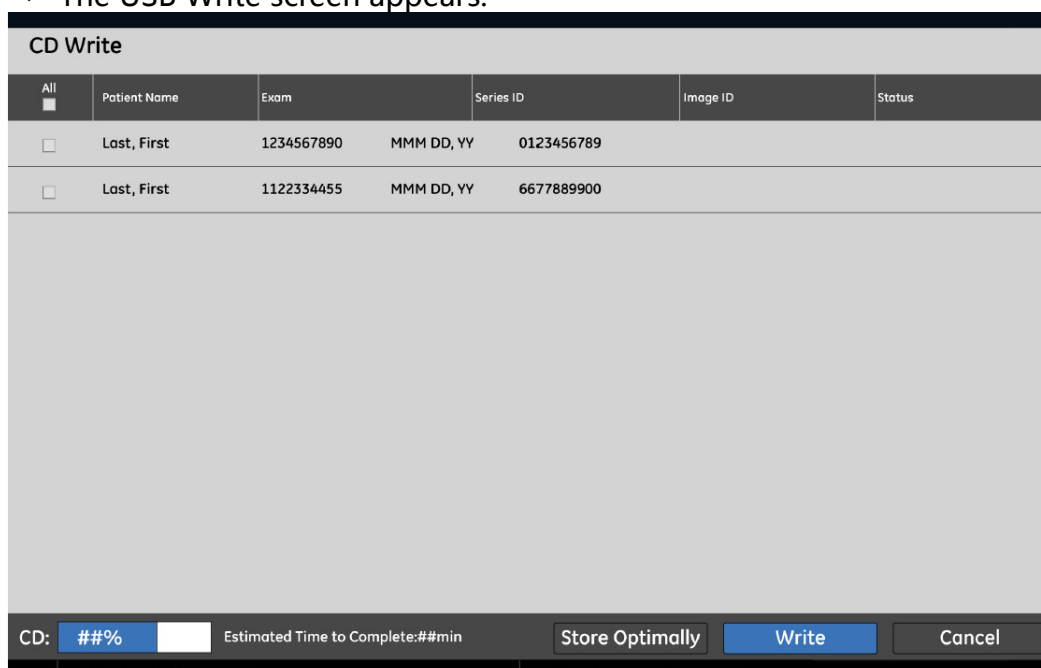Exams may be copied to a USB drive for archiving purposes, to send to a location that is not within the network, or to send with a patient's medical records.

**Note:** The disk used for copying images must be recordable. That is, the "write" button should be ON.

**Note:** 您不能将检查复制到已保存有检查或其它信息的 USB 存储器上。否则，即使有足够的空间，您在写入时也会收到“USB 存储器非空白，请插入空白 USB 存储器再继续”的错误信息。复制信息时，请总是使用空白 USB 存储器。You cannot copy exams to a CD/DVD that already has exams saved on it. You will receive a "Not enough space" error when you attempt to write to the CD, even if there is enough space. Always use a new, blank CD.

1. 将空白 USB 存储设备插入主机上的 USB 接口。
2. Select the exams to be copied.
3. Press the Destination [USB1].
  - ◆ The USB Write screen appears.

- ◆ [STORE OPTIMALLY] 按钮将决定复制哪一个检查可最大程度地利用磁盘空间。该按钮将自动取消选择任何不符合 USB 的检查。automatically checks if the selected images will fit on the CD. It will automatically un-check any items that cannot fit on the CD.
- ◆ [WRITE] begins the copying process.
- ◆ [CANCEL] closes the screen and returns you to the Image Management screen.

4. Confirm the exams to be copied. Un-check any exams that you do not want saved to the disk.
5. Click [WRITE].
  - ◆ The disk begins copying. The light on the front of the computer flashes yellow as the data is being written.
  - ◆ To see the status of the exams being copied, click [TRANSFER LOG] at the bottom of the Image Management screen.
  - ◆ The Transfer Log screen appears.
  - ◆ Press [CLOSE] to close the screen and return to the Image Management screen.

**CAUTION:** 在取出 USB 存储器前一定检查传输日志，如果在传输状态日志显示于传输日志界面下方前取出 USB 存储器，则此操作后，复制存储器中图片将失败，直到系统复位。 After the system finishes the disk, open the exams on the disk to ensure that the disk has the data written to it. Refer to Chapter 9: Image Management-Load Images from a CD/DVD for more information.

6. Remove the disk.
7. Label the disk and store in a safe place.

## Deleting Exams, Series, or Images

Exams, series, and images may be deleted from the selected database source from the Image Management screen.

Follow this process to delete exams, series, and images.

1. Select the Source, if necessary.

**Note:** 只能删除本地数据。 Items cannot be deleted from a CD.

2. Unlock exams, if necessary.
  - ◆ Refer to [Locking Exams from Deletion](#) for more information.
3. Select the items to delete.
  - ◆ The items may be exams, series with an exam, or images within a series. Multiple items may be selected and deleted at once.
4. Click [DELETE].
  - ◆ A message appears: "Are you sure that you would like to delete the selected items?"
5. Click [OK].
  - ◆ The message closes.
  - ◆ The items are deleted from the Image Management screen.
  - ◆ Clicking [CANCEL] closes the message and the items remain on the Image Management screen.

## Locking Exams from Deletion

The Image Management screen allows you to prevent, or “lock”, exams from being deleted. The exam can only be deleted if the lock is removed, or “unlocked.” The lock prevents exams from being deleted by other operators and from Auto Delete. 有关自动删除的信息，请参考 [Chapter 16: 设置参数 - 图像管理](#) 小节。

Locked exams can be copied, transferred, and viewed.

Only exams can be locked. Individual series or images cannot be locked.

Follow this process to lock and unlock exams.

1. Select the exams to lock.
2. Click [LOCK].
  - ◆ The lock icon appears in the status column of all selected exams.
  - ◆ The Lock button changes to Unlock.

## Unlocking Exams

Follow this process to unlock exams so that they may be deleted.

1. Select the locked exams.
  - ◆ The Unlock button becomes active.
2. Click [UNLOCK].
  - ◆ The lock icon is removed from the status column of the selected exams.
  - ◆ The exams may now be deleted.

## Making Patient Anonymous (De-Identify)

There may be times when you want the name of a patient to be kept confidential to maintain patient privacy. You can do this using the De-Identify feature. This feature allows you to create an anonymous set of images.

The patient examinations are copied and used to create a new patient, with the name "Anonymities patient" and a unique, randomly created Patient ID as shown in [Figure 10-7](#).

**Note:** Once an anonymous exam is created, there is no way to recover the patient's identifying information.

De-Identify is a feature available from the Image Tools button ([Figure 10-6](#)).

**Figure 10-6** Image Tools button

**Figure 10-7** Anonymities patient

Use this process to make exam images anonymous:

1. Select the patients to make anonymous.
2. Switch the Image Tools button to [DE-IDENTIFY], if necessary.
  - ♦ A message appears: "The selected exams will be copied without patient identification. The originals will not be deleted."
3. Click [OK].
  - ♦ The Image Management screen updates with copied, anonymous exams.

# Chapter 11

# Image Viewer

The Image Viewer screen (Figure 11-1) appears on the monitor once an exposure is taken in a live exam or when an image series is chosen from the Image Management screen for review. This screen is where images are adjusted and viewed. Topics covered include:

- Overview
- Image Selection Palette
- Image Acquisition Palette
- Image Tools Palette
  - Orientation
  - Annotation
  - Reprocess
- Dose Exposure Indicator (DEI)
  - DEI 不可用
  - 校正措施
  - 校正措施的例外
- Quality Check
- Quality Check
- Printing Images
  - 自动打印
  - 手动打印
- Sending Images
- Saving Changes to Images


## Overview

The right side of the screen contains all the image selection and adjustment tools. Most of the tools are categorized into 4 “palettes” that can be expanded or collapsed to reveal or hide different functions.

The majority of the screen is devoted to image display. The images on the left side of the screen update as adjustments are made. You are able to view single or multiple images at once. When viewing multiple images, as shown in [Figure 11-1](#), an aqua border identifies the currently selected image.

[Table 11-1](#) describes the functions for the Image Viewer screen.

**Figure 11-1** Image Viewer

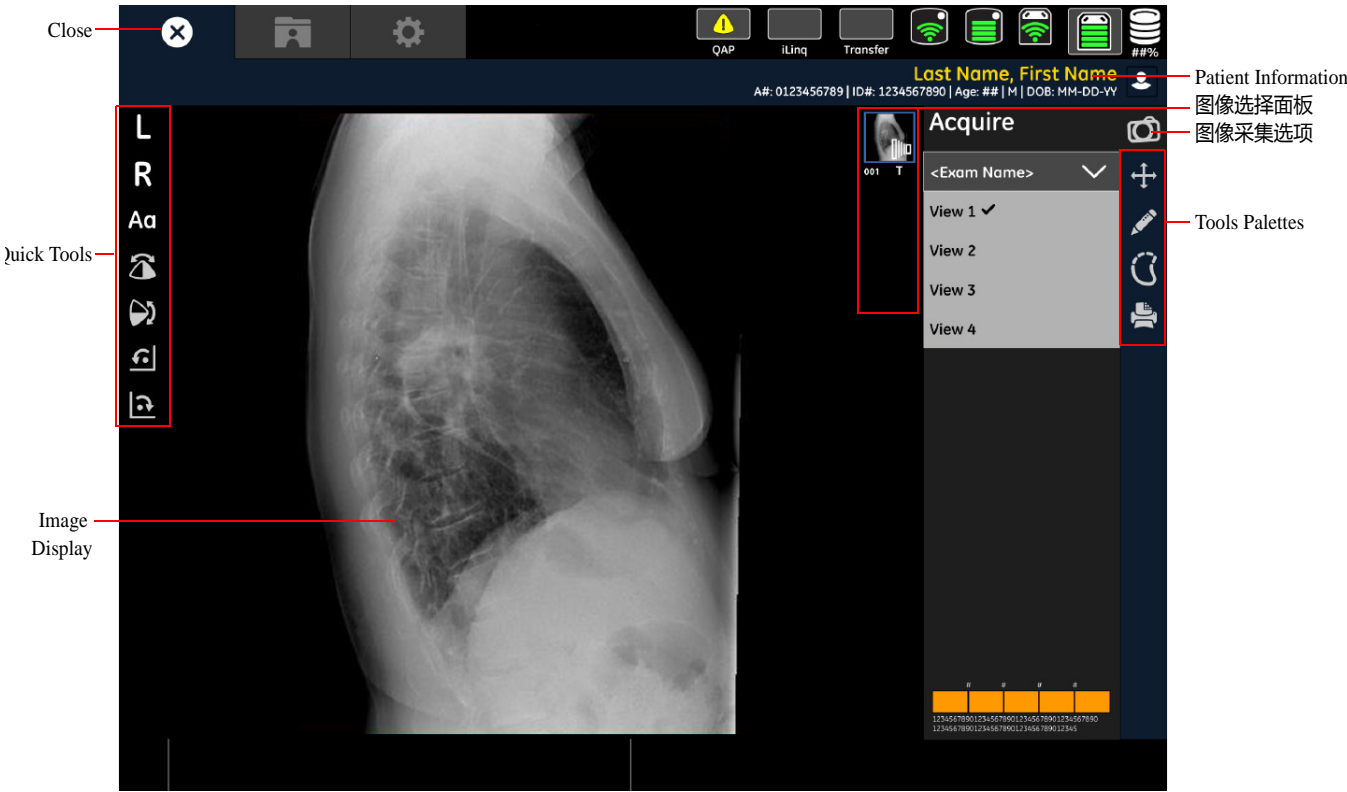**Table 11-1** Image Viewer functions

Function	Description
[Patient Identification]	Identifies the patient.
[Quick Tools]	Includes common used image processing buttons. The menus vary with different selected tool palette.
[Image Display Zone]	Shows the selected image in display zone.
[Image Selection Palette]	Image Selection Palette that shows small previews of all images in the selected series and highlights the images currently shown in the viewer. Refer to <a href="#">Image Selection Palette</a> for more information.
[Image Acquisition Palette]	If in an active exam, allows you to switch back to the Acquisition screen. If viewing images from a completed exam, the Acquisition screen tab is disabled.
	
<p><b>Warning:</b> When acquiring multiple images or if you need to retake an image, you must switch back to the Acquisition screen in order to record the exposure. Making an exposure while the Viewer screen is displayed will expose the patient to radiation, but the detector will not acquire the image.</p>	
[Tools Palettes]	Image Tools Palettes for image editing. <ul style="list-style-type: none"> <li>• Orientation, refer to <a href="#">Orientation</a>.</li> <li>• Annotation, refer to <a href="#">Annotation</a>.</li> <li>• Image reprocess, refer to <a href="#">Reprocess</a>.</li> <li>• Print, refer to <a href="#">Printing Images</a>.</li> </ul>
[Close]	Closes the Image Viewer screen and prompts you to save any changes to images. Close also initiates auto <b>print and auto push</b> , if enabled. Refer to <a href="#">Saving Changes to Images</a> for more information.

## Image Selection Palette

Image Selection Palette allows you to select which exam or series of images to view and display format.


The Images panel shows previews of all images in the selected series. The panel shows up to 8 image previews at a time. If there are more than 8 images in the series, a scrollbar appears on the right to allow you to see the rest of the images.

The image that is currently selected in the Image Viewer is shown with an orange border in the Images panel.

To view an image, click an image preview.

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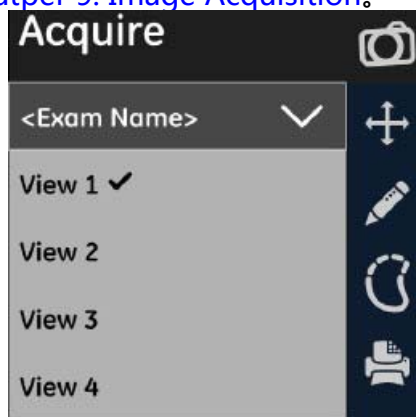
# Image Tools Palette

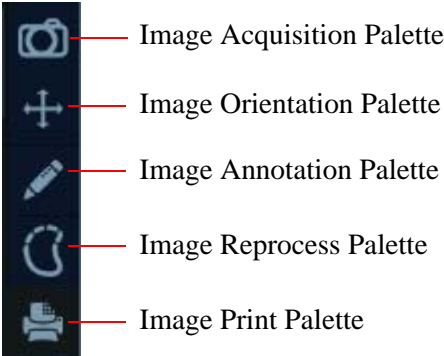Different tools and controls are accessed through the Tool Selection list.

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工具	描述
[Manual Shutter]	<p>Applies shutters manually to crop the image. In image review mode the image must be reprocessed to enable shutter icon.</p> <p><b>Note:</b> This function is only available when the image is open in a live exam. 只有当图像在活动检查中处于打开状态或用于重新处理时，此项功能才可用。</p> <p>Refer to <a href="#">添加图像注释</a> for more information.</p>
[Rotate Left]	Rotates the selected image counter-clockwise in 90 degree increments.
[Rotate Right]	Rotates the selected image clockwise in 90 degree increments.
[Vertical Flip]	Flips the selected image 180 degrees on the horizontal axis; that is, switches top for bottom.
[Horizontal Flip]	Flips the selected image 180 degrees on the vertical axis; that is, switches left for right.
[Free Rotation]	<p>Rotates the selected image both clockwise and counter-clockwise.</p> <ul style="list-style-type: none"> <li>Click the end buttons to rotate the image in 0.1 degree increments.</li> <li>Click and drag the slider to spin the image.</li> <li>Move the slider right to rotate the image clockwise.</li> <li>Move the slider left to rotate the image counter-clockwise.</li> </ul>
[ 全景图像 ]	在浏览区域内移动图像。
[ 图像放大镜 ]	放大三倍显示一小部分图像。
[ 患者方位 ]	
[1/2 Display]	Reduces the image to 50% (four pixels on the detector equals one pixel on screen).
[1:1 Display]	Shows default the image size where one pixel on the detector equals one pixel on the screen.
[Fit To Screen]	Scales the image to fit within the Image Viewer screen.
[True Size]	Shows the image in the exact size it was acquired from the detector (one centimeter on the detector equals one centimeter on screen).


## Annotation

The Annotation panel (Figure 11-3) contains the tools to annotate images. Table 11-3 describes the tools and their functions.

Image annotations are divided into two categories:

- System annotation – Information that is kept by the system, such as identifying information, exposure and acquisition information, and processing information. These annotations are displayed as text at the corners of the image. You may select which annotations appear, but you cannot control where the annotations are placed.
- User annotation – Lines, ellipses, Cobb angle, user annotation (notes), and RL markers added by the operator to measure or bring attention to a section of the image. You draw or place these annotations on the image as appropriate.

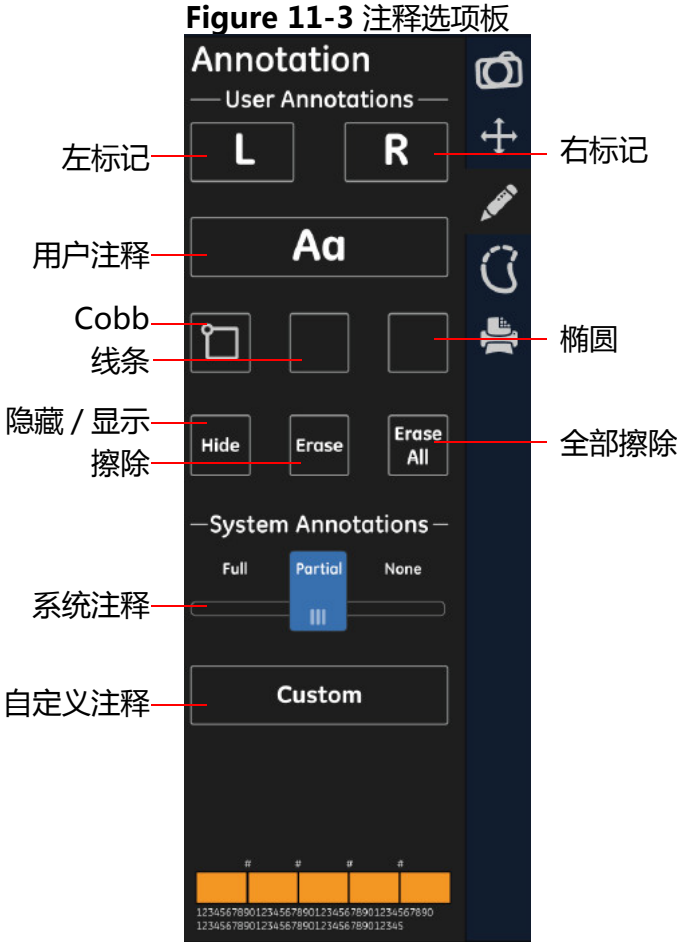**Table 11-3** Annotation Palette

Tools	Description
User Annotations	
[L/R]	<p>Select the Right or Left marker. Tap the R marker or the L marker then tap on the image where it is to be placed. Note: Only one RL marker is inserted per image. 在图像上放置一个右或左标记，以作参考。</p> <p><b>Note:</b> 只有当图像在活动检查中处于打开状态或用于重新处理时，L/R 标记器才可用。 更多信息请参考<a href="#">添加图像注释</a>小节。</p>
[Aa]	<p>Places a text box on the image that you may add notes into.</p> <p><b>Note:</b> Annotations shall be placed within the image field of view.</p>
[Cobb]	<p>Places a Cobb angle (two lines) on the image that you may re-size, move, or angle. Cobb angle specifications are shown in the User Measurements annotation at the bottom right corner of the viewer.</p>
[Line]	<p>Places a line on the image that you may re-size, move, or angle. Line specifications are shown in the User Measurements annotation at the bottom right corner of the viewer.</p>
[Ellipse]	<p>Places an ellipse (circle or oval) on the image that you may re-size, move, or re-shape. Ellipse specifications are shown in the User Measurements annotation at the bottom right corner of the viewer.</p>
[Hide/Show]	<p>Temporarily removes image annotations from the image. Click [SHOW] to see the annotations.</p>
[Erase]	<p>Deletes the selected image annotation. Deleted annotations cannot be recovered.</p>
[Erase All]	<p>Deletes all image annotations. Deleted annotations cannot be recovered.</p>
System Annotations	

Tools	Description
[Full]	<p>Places all available system annotation on the image.</p> <ul style="list-style-type: none"> <li>• Patient information – (top left corner) exam date and patient identification</li> <li>• Study information – (top left corner) exam identification</li> <li>• Series information – (top left corner) series identification</li> <li>• Image information – (top left corner) image identification</li> <li>• Acquisition information – (top right corner) dose and DAP</li> <li>• Hospital information – (top right corner) the name of the facility where the image was acquired</li> <li>• X-ray parameters – (top right corner) the mA, kVp, ms, and mAs of the exposure</li> <li>• Anatomy information – (bottom left corner) the protocol used to acquire the image</li> <li>• Processing information – (bottom left corner) the look used to process the image</li> <li>• User measurements – (bottom right corner) size and angle measurements for line, ellipse, and Cobb annotations</li> <li>• Display parameters – (bottom right corner) the size of the image and the zoom</li> </ul>
[partial]	Displays ONLY the facility name, dose information and technical factors.
[None]	Removes all system annotations from the image. System annotations can be re-applied by clicking [FULL], [PARTIAL], or [CUSTOM].
[Custom]	Brings up a screen that allows you to choose which system annotations appear. More information, please refer to <a href="#">自定义系统注释</a> .

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### Reprocess

Image re-processing allows you to extract more information from an already acquired image by changing the processing settings instead of taking additional exposures.

Re-processing can be performed on any image that has a corresponding raw data set. Images can be reprocessed both in live exams and in review mode.

When in an active exam, re-processing will create a new "PROCESSED" series. When in review mode, re-processing will create a new image in the series.

**Note:** When closing an exam or closing patient in review mode, the you must select to save changes to images, or the re-processed images will not remain in the series. More information, please refer to [Saving Changes to Images](#).

The initial image processing is determined by the default that is configured for the protocol. More information, please refer to [设置参数 - 图像管理](#).

Figure 11-5 Image Re-Processing palette

**Table 11-5** Reprocess Palette

Tool	Description
快捷工具栏	
[预览工具]	显示处理后的图像，实时查看处理结果。
[比较工具]	
[Apply Button]	Applies the changes and re-processes the image. <b>Note:</b> When in an active exam, re-processing will create a new "PROCESSED" series. When in review mode, re-processing will create a new image in the series.
[保存按钮]	保存对图像所做的处理结果。
Image Reprocess	
[Brightness]	Lightens or darkens the selected image. <ul style="list-style-type: none"> <li>• Move the slider right for a lighter image.</li> <li>• Move the slider left for a darker image.</li> </ul>
[Contrast]	Adjusts the differences between dark and light on the selected image. <ul style="list-style-type: none"> <li>• Move the slider right for more contrast (towards pure black and white).</li> <li>• Move the slider left for less contrast (towards uniform gray).</li> </ul>
[Invert]	Reverses light and dark areas of the selected image.
More Functions	
[Windowing]	Applies windowing to the selected image. Available options are: <ul style="list-style-type: none"> <li>• Normal – image as acquired</li> <li>• Hard – adjusts the image towards black and white</li> <li>• Soft – adjusts the image towards gray</li> </ul>
[Anatomy]	Changes the anatomical region.
[View]	Changes the view.
[Image Type]	Changes the image type. Available options are: <ul style="list-style-type: none"> <li>• Standard</li> </ul>
[Patient Size]	Changes the patient size.

---

Tool	Description
[Look]	<p>Changes the processing look. Looks are the way an image is processed to be viewed by a radiologist for interpretation. Looks are pre-defined combinations of brightness, contrast, edge enhancement, and tissue equalization.</p> <p>Factory Look descriptions:</p> <ul style="list-style-type: none"><li>• Factory Look 1 – similar to analog film, low edge, no TE</li><li>• Factory Look 2 – low CR look, low edge, moderate tissue contrast</li><li>• Factory Look 3 – moderate CR look, low edge, TE</li><li>• Factory Look 4 – highly digital look, high edge, high TE</li><li>• Custom looks – The system allows the user to build up to 5 custom looks in any combination of parameters. <a href="#">更多关于创建自定义外观的信息，请参考第 13 章：设置参数 - 图像处理。</a></li></ul>

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Function	Description
[# of Copies]	Defines how many copies to print.
[Printer]	Lists all available printers configured for your system.
[Orientation]	Selects vertical or horizontal orientation of the image on film or paper. Available options are: <ul style="list-style-type: none"> <li>• Landscape</li> <li>• Portrait</li> </ul>
[Print Mode]	Selects what size to print the image. Available options are: <ul style="list-style-type: none"> <li>• True Size</li> <li>• Fit to Film</li> <li>• Reduced Size</li> </ul>
[Magnification]	Only available if Reduced Size is selected for the Print Mode. 只有在 [ 打印模式 ] 中选择了 [ 缩小的尺寸 ] 时才可用。图像将按所输入的百分比值缩小。可接受的缩小范围是 40% 至 90%。如果将 [ 实际尺寸 ] 或 [ 胶片尺寸 ] 选择为 [ 打印模式 ]，此文本框将被禁用。

Function	Description
[Format]	Selects the print format. Available options are: <ul style="list-style-type: none"> <li>• Standard</li> <li>• 35 mm</li> <li>• 40 mm</li> </ul>
[Media Size]	Shows the available sizes that are configured for the selected printer.
[Media Type]	Selects the type of media to print on. Available options are: <ul style="list-style-type: none"> <li>• Paper</li> <li>• Clear film</li> <li>• Blue film</li> </ul>
[Destination]	Shows configured printers on the system.
[Magnification Type]	Selects the magnification type. Available options are: <ul style="list-style-type: none"> <li>• Cubic</li> <li>• None</li> </ul>
[Annotation]	Selects the amount of annotation to print on the image. Available options are: <ul style="list-style-type: none"> <li>• Full</li> <li>• Partial</li> <li>• Custom</li> <li>• None</li> </ul> More information, please refer to <a href="#">Annotation</a> .
[Print]	Print the selected image.
[Print All]	Print all images.
快捷按钮	
[ 添加表单 ]	在图像区域添加一个空白表单。
[ 删除表单 ]	删除当前显示的表单。
[ 删除所有表单 ]	删除所有可用的表单。
[ 打印总结 ]	将显示区域调整为 4*4 的表格，然后显示所有可用表单的缩略图。

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# Chapter 12

# Digital Detector Application

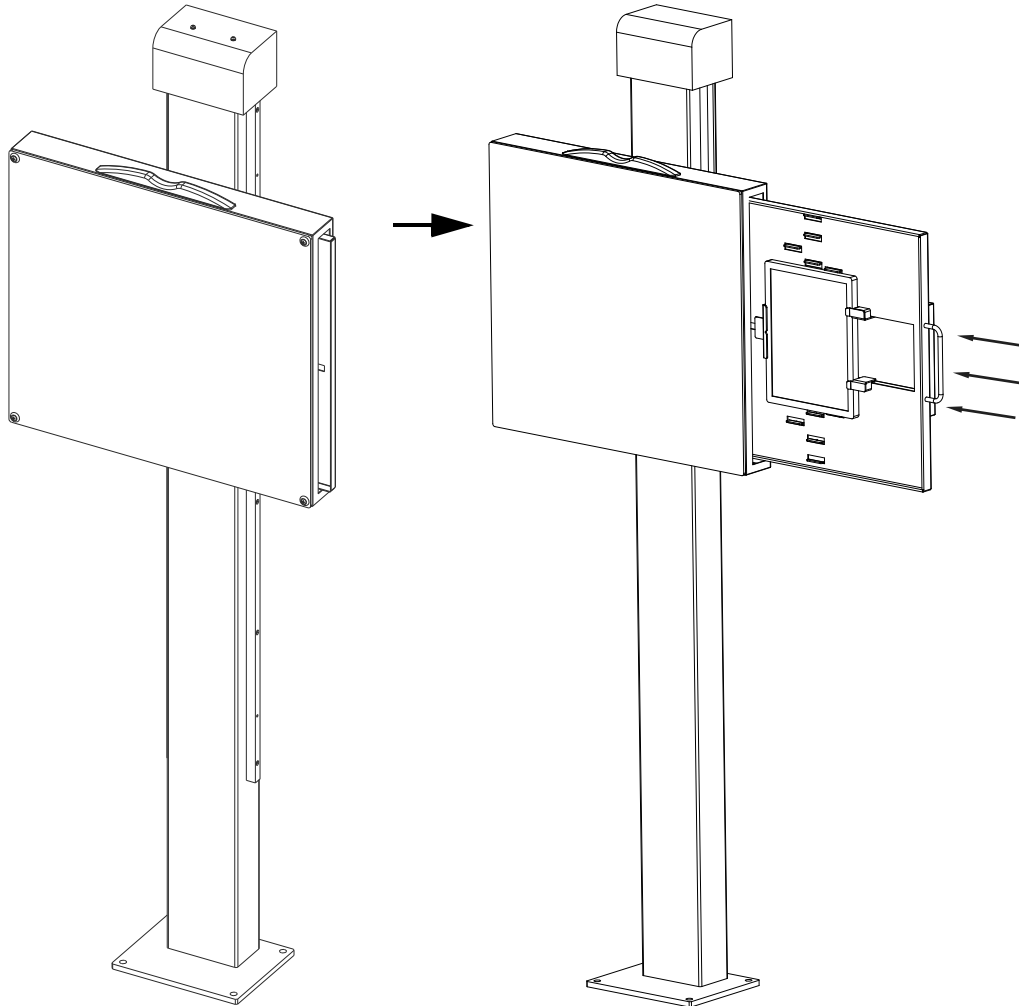
The digital detector of this system can be shift between the Digital Wall Stand and Integrated Table (Radiographic Stretcher Table / Patient Stretcher). This section is intended to outline the digital detector' s shift application.

Topics covered include:

- [Wall Stand Mode](#)
- [Integrated Table Mode](#)
- [Integrated Table Top \(Stretcher Table Top\) Mode](#)
- [Digital Cassette Mode](#)
- [Steps for cassette installation](#)

## Wall Stand Mode



Pull out the wall stand detector tray -> Insert the digital detector into the wall stand.

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## Integrated Table Mode

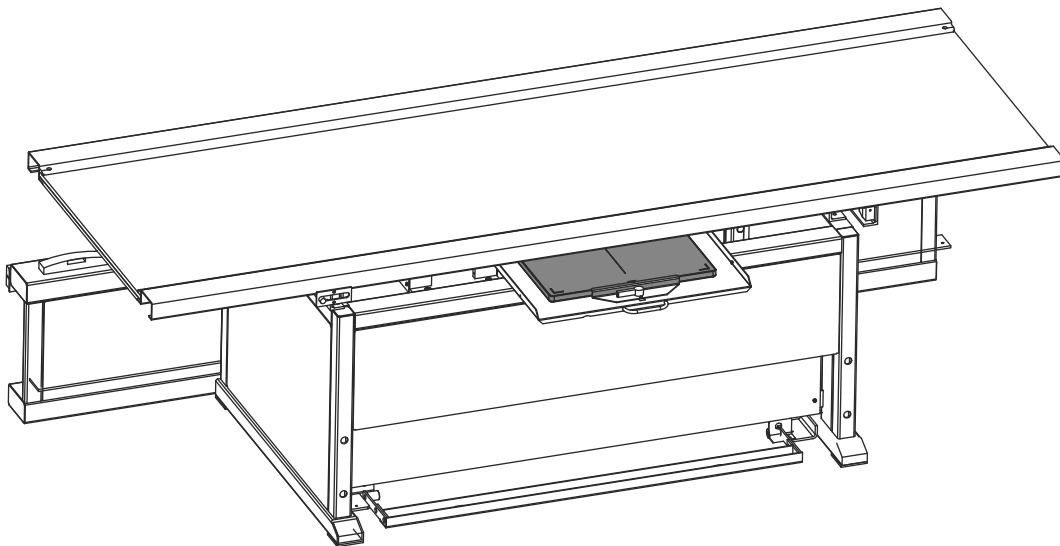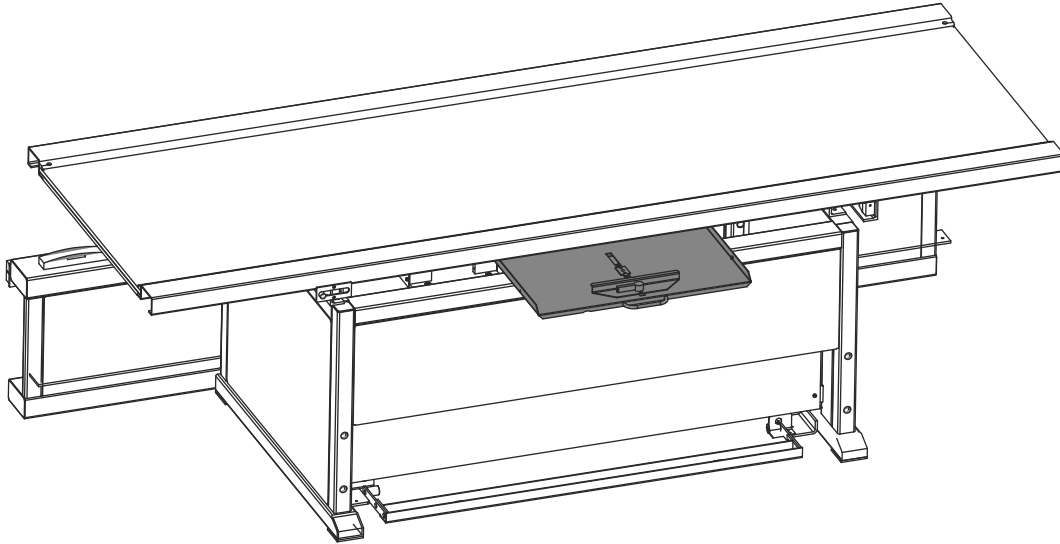
Pull out the table detector tray -> Insert the digital detector into the integrated table.

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## Integrated Table Top (Stretcher Table Top) Mode

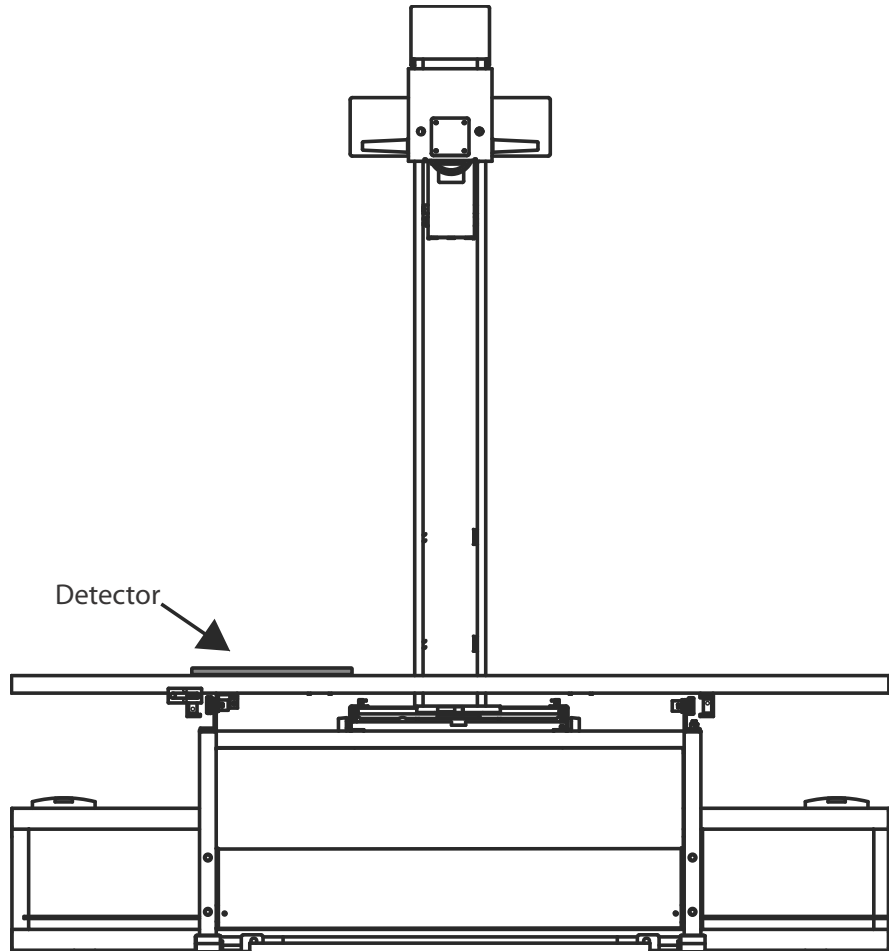
Put the digital detector to the appropriate location on the table top.

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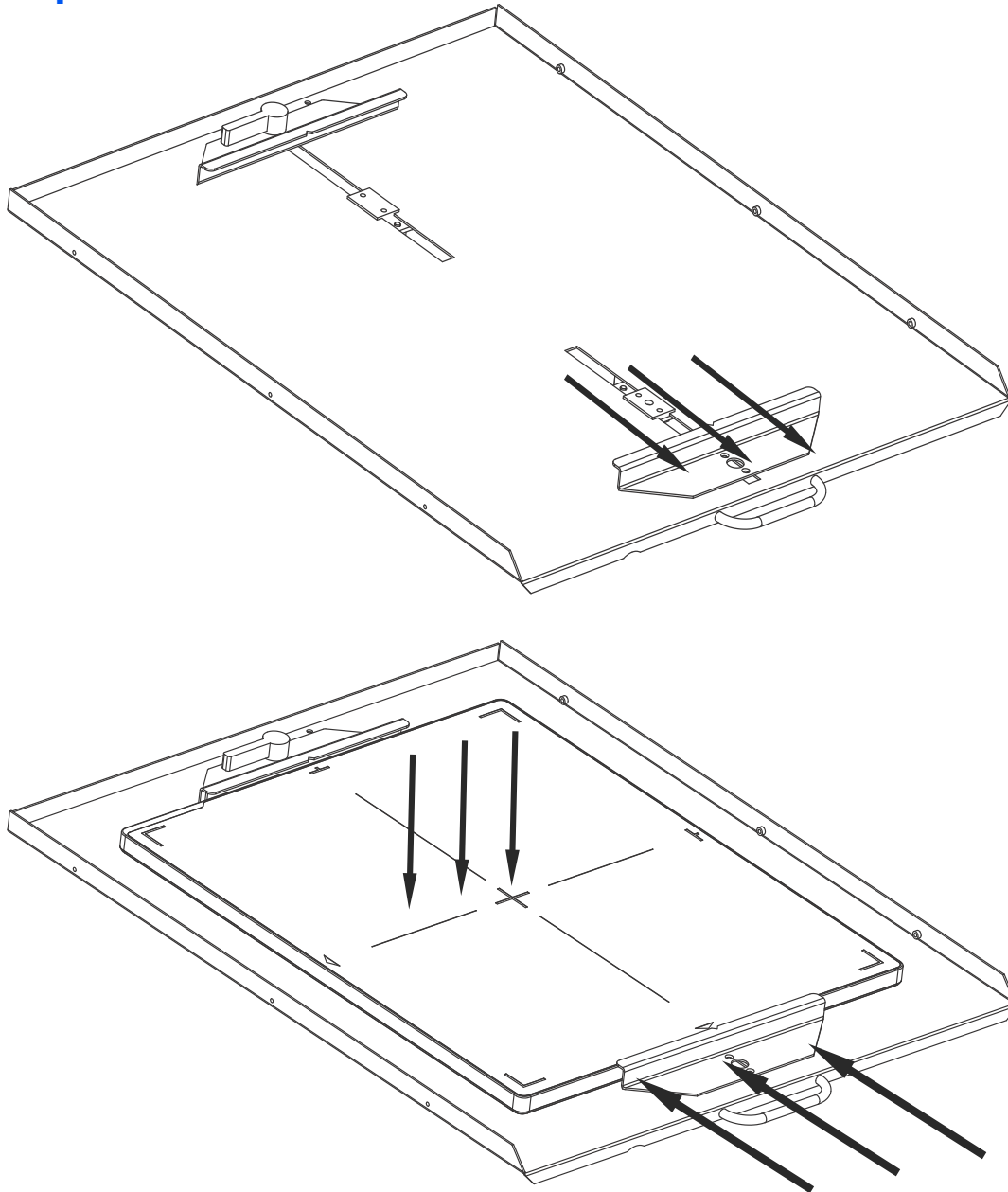IMAGE QUALITY TEST RESULTS		Fri Aug 13 14:36:19 CDT 2004		
TEST	MEASUREMENT	LSL	USL	STATUS
<b>OVERALL RESULT</b>				<b>FAIL</b>
No. Of Bad Pixels	0.00	-	30	PASS
Global Brightness Non Uniformity	24.35	-	10	FAIL
Local Brightness Non Uniformity	1.47	-	5	PASS
SNR Non Uniformity	30.65	-	60	PASS
Spatial MTF at 0.5 lp/mm	86.67	70	-	PASS
Spatial MTF at 1.0 lp/mm	69.91	53	-	PASS
Spatial MTF at 1.5 lp/mm	54.24	35	-	PASS
Spatial MTF at 2.0 lp/mm	39.25	23	-	PASS
Spatial MTF at 2.5 lp/mm	27.91	17	-	PASS
Contrast/Noise Ratio 1	10.71	3	-	PASS
Resolution Non Uniformity	30.24	-	60	PASS

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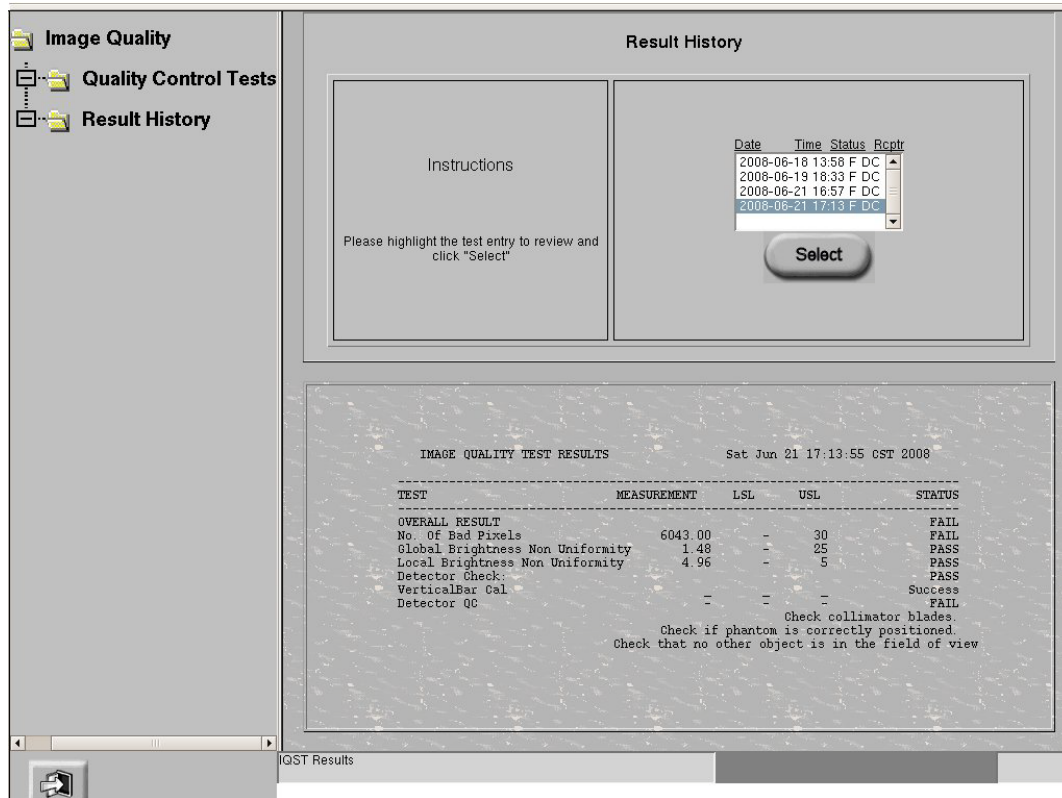
# Chapter 15

# Maintenance

It is the owners responsibility to provide regular periodic maintenance and service. Only this type of maintenance program can identify potential problems.

Topics covered include:

- [Periodic Maintenance](#)
- [Qualified Service](#)
- [Cleaning and Disinfecting](#)
- [Recycling](#)

## Periodic Maintenance

Periodic maintenance is required for continued SAFE operation. Periodic maintenance should be performed as specified in the maintenance schedule of the service manual by qualified personnel. Descriptions and scheduled frequency of the required periodic maintenance are provided in the Service manual (#  
\*\*\*\*\*) supplied with the equipment. Inspection intervals are based on average daily use of one eight hour shift.

These periodic maintenance procedures will be performed by GE service if contracted to do so. Generally, most of the parts need maintenances once a year; general cleaning and painting as required; More frequent inspection is appropriate where equipment use is above average.

**CAUTION:** Failure to perform the periodic inspection and maintenance could allow deteriorating conditions to develop without being detected. This deterioration could result in equipment failures, which could cause serious injury or equipment damage.

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## Qualified Service

Safe equipment performance requires the use of service personnel specially trained on medical X-ray apparatus. General Electric Medical Systems and its associates, maintain a world-wide organization of stations from which to furnish periodic and/or emergency service on a contract basis. A GE representative will be glad to discuss this plan. General Electric Medical Systems X-ray equipment contains operating safeguards designed to provide maximum safety. Before calling for service, be certain proper operating procedures are being used.

## Cleaning and Disinfecting

This equipment should be cleaned frequently, particularly if corroding chemicals are present. **Use a cloth moistened in warm soapy water (use mild soap) to clean the trim and nameplate of the Operator' s Controls.** Wipe with a cloth moistened in clean water. Do not use cleaners or solvents of any kind as they may dull the finish or blur the lettering. Polish with a pure liquid or paste wax. Other surfaces of the equipment can be cleaned using a clean cloth moistened slightly with a good mild cleaner and polish acceptable for use on enameled metal surfaces. Before each use, equipment surfaces that contact the patient should be cleaned with an EPA registered, low-level disinfection or sanitizing agent.

The following chemicals and products have been tested and approved by GE for cleaning the GE Image Workstation:

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**CAUTION:** In the event of equipment contacting broken skin or being used with infected or immune compromised patients, the equipment should be cleaned using EPA cleared and EPA registered high-level disinfecting agents.

**Note:** Be sure to follow the label instructions and pre-cautions for use, storage, and disposal of all disinfecting agents.

## Recycling

### **Packing Materials**

The materials used to pack our equipment are recyclable. They must be collected and processed in accordance with the regulations in force for the country where the machines or accessories are unpacked.

### **Machines or Accessories at end-of-life**

The elimination of machines and accessories must be in accordance with national regulations for waste processing. All materials and components that could pose a risk to the environment must be removed from the machines at end-of-life and accessories (examples: dry and wet cell batteries, etc.).

Please consult your local GEHC representative before discarding these products.

This page intentionally left blank.

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