

(Photo Art Under Development)

The Bravo™ pH Receiver is designed to work with the Bravo pH System™ and assist medical personnel in diagnosing and monitoring gastroesophageal reflux disease (GERD).

Disclaimer

This publication is supplied for the purpose of using the Bravo pH Receiver only. Endonetics, Inc. is responsible for the reliability and performance of the Bravo pH Receiver only if the following conditions are met:

- All repairs and modifications are carried out by Endonetics or its assigned agents.
- The equipment is used in accordance with the information presented in this user's manual.

CAUTION: Federal Law (USA) restricts these devices to sale by or on the order of a physician.

Compliance Statement

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions:

- 1 This device may not cause harmful interference, and
- 2 This device must accept any interference received, including interference that may cause undesired operation.

Warning

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Table of Contents

Chapter 1: Introduction	1
System Components	2
Bravo pH Receiver Description	3
Patient Interface	3
Controller Interface	3
Chapter 2: Operating the Bravo pH Receiver	5
Turning On the Receiver	5
Setting the Time and Date	6
Setting the Current Time	6
Setting the Current Date	7
Setup – pH Study Parameters	7
Number of pH Probes	8
pH Display	8
Audible Beep	8
Chapter 3: Calibration	9
Calibrating One pH Probe	9
Calibrating Two pH Probes	13
Calibrating the First pH Probe	13
Calibrating the Second pH Probe	16
Chapter 4: Performing a Bravo pH Study	19
Clearing Old Data from the Receiver	19
Starting a pH Study	20
Viewing the Status Parameters	21
Stopping a pH Study	22
Chapter 5: Uploading pH Data from the Receiver to the Computer	23
Chapter 6: Instructing the Patient on How to Operate the Receiver	25
Using Symptom Markers	25
Using the Patient Diary	25
Receiver Out of Range	26
Low Battery Warning	26

Chapter 7: Maintenance	27
Cleaning	27
Replacing the Main Battery	27
Chapter 8: Troubleshooting	29
Specifications	31
Receiver	31
DataLink	31

Conventions Used in This Manual

■ **NOTE:** Points out information that may be helpful.

■ **IMPORTANT:** Indicates information necessary to successfully perform the procedure.

CAUTION: Alerts you to situations that could result in instrument damage, failure in a procedure, or possible incorrect results.