D

GB

F

I

E

NL

P

GR

TR

DK

FIN

N

S

LT

PL

RUS

RO

HR

HUN

SK

BG

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PL

RUS

General Warnings and Safety Instructions

RO

You Have Chosen a Hearing System from HANSATON.

We take pleasure in your choice and congratulate you on your decision, because with a HANSATON hearing system, you own a high-quality product of the most modern technology.

HUN

For more than 50 years, we have been helping people with our devices to achieve better hearing and better speech understanding, thus a higher quality of life.

SK

Your hearing systems equipped with Bi-Com technology can be controlled by your remote control. This remote control offers you more hearing comfort in your everyday life situations and link your hearing

BG

systems to the expanding world of electronic media. Media and mini-Media will also support you when you watch television, listen to music, or talk on the phone. In this operating manual, you will find useful tips on fitting, use and maintenance. If you have further questions, please contact your hearing system professional.

Intended Use.

This remote control is intended to operate hearing systems. Read and follow the instructions of this user guide to avoid injuries or damage to remote control.

Be aware of information marked with the warning symbol **WARNING!**

Symbols.

 **WARNING!** indicates a situation which can result in serious, moderate or minor injuries.

 indicates possible property damage.

 Advice and tips on how to handle your remote control better.

• Instruction. Indicates that there is something you must do.

  Indicates an option that not every remote control has. Ask your hearing system professional to checkmark the relevant information.

Personal Safety.

 **WARNING! Risk of injury!** Do not use obviously damaged devices and return them to point of sale.

 **WARNING! Choking hazard posed by small parts.**

- Keep hearing systems, batteries, and accessories out of children's reach.
- If swallowed, immediately consult a physician or hospital.
- If infants, small children, or mentally disabled persons need to wear hearing systems, ensure adequate supervision.

 **WARNING!** Strangulation hazard when lanyard gets entangled in e.g. moving mechanical parts of machines.

- Use only the delivered breakaway lanyard. It will automatically detach

D

GB

F

I

E

NL

P

GR

TR

DK

FIN

N

S

LT

PL

RUS

RO

HR

HUN

SK

BG

D when entangled.

- Do not use lanyard for children under the age of twelve.

GB

WARNING!

Risk of affecting electronic equipment!

F

- In areas where the use of electronics or wireless devices are restricted, verify if your device has to be turned off.

I

Wireless systems may interfere with measuring devices and electronic equipment.

E

- Do not use your device in hospitals or airplanes.

NL

WARNING!

Wireless systems may interfere with measuring devices and electronic equipment.

P

- Prior to use in conjunction with an electronic implant or life supporting system, have the electromagnetic compatibility verified.

GR

If you wear a pacemaker:

TR

- Carry the remote control at a safe distance of about 12 inches (30 cm) away from the pacemaker.

DK

- Do not carry the remote control:

- in your breast pocket,
- with a lanyard around your neck,
- or directly on the skin over the active implant.

FIN

WARNING! Risk in explosive environments (e.g., mining areas)!

N

- Do not use your hearing systems and remote control in areas where there is a danger of explosions.

S

i Radiofrequency radiation exposure Information

LT

The radiated output power of the devices is far below the FCC radio frequency exposure limits. Nevertheless, the device shall be used in such a manner that the potential for human contact during normal operation is minimized.

PL

RUS

Product safety.

RO

The remote control is sensitive to extreme heat, high humidity, strong magnetic fields (> 0.1T), X-rays and mechanical stress.

HR

- Do not expose the remote control to extreme temperature or high humidity.

HUN

- Do not leave it in direct sunlight.

SK

- Do not immerse it in water.

- Do not place your remote control in a microwave oven.

BG

WARNING!

Different types of strong radiation, e. g. during X-ray or MRI head

examinations, may damage devices.

- Do not use the devices during these or similar procedures. Weaker radiation, e. g. from radio equipment or airport security, does not damage the devices.

i The performance of the i-com2 function may be affected by electromagnetic interference – e.g. from a computer monitor or fluorescent lamp system.

i When using mobile telephones, however, electromagnetic-field effects cannot be ruled out.

I Your hearing systems are designed to comply with international standards on electromagnetic compatibility but interference with

- nearby electronic devices could occur.
- In this case, move away from the source of interference.

i Your remote control is programmed to exclusively communicate with your hearing systems. If you experience problems with another person's remote control, consult your Hearing Care Professional.

i In some countries restrictions for the usage of wireless equipment exist.

- Refer to local authorities for further information.

Safety Notes for Batteries/Rechargeable Batteries.

WARNING!

- Leaking batteries/rechargeable batteries damage the system.
- Remove the battery/rechargeable batteries when the system is not in use for a prolonged period of time.
- Risk of explosion when the battery/rechargeable batteries is handled wrong.
- Do not short circuit, pierce, crush, or disassemble the battery/rechargeable batteries.
- Do not place batteries/rechargeable batteries in oven or fire.
- Do not immerse batteries/rechargeable batteries in water.
- Charge the rechargeable battery/rechargeable batteries between 0°C and 45°C.

D

GB

F

I

E

NL

P

GR

TR

DK

FIN

N

S

LT

PL

RUS

RO

HR

HUN

SK

BG

D **Technical Information.**

Operating frequencies: $F_c = 3.28$ MHz

GB Changes or modifications not expressly approved by the party responsible for compliance could void the users authority to operate the equipment.

F

I

This device complies with Part 15 of the FCC Rules.

Operation is subject to the following conditions:

E

(1) this device may not cause harmful interference, and

NL

(2) this device must accept any interference received, including interference that may cause undesired operation.

P

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

GR

TR

DK

FIN

N

- Reorient or relocate the receiving antenna.

S

- Increase the separation between the equipment and receiver.

LT

- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

PL

- Consult the dealer or an experienced radio/TV technician for help.

Disposal Information.

RUS

Within the European Union, the marked equipment is covered by „Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment“. Amended by „Directive 2003/108/EC“ (WEEE).

RO

HR

i Recycle the device and batteries according to national regulations, or return them to your hearing system professional.

HUN

SK

Batteries contain harmful substances that pollute the environment.

BG

- Do not dispose of batteries in household trash.

FDA Information (Valid for USA only)

Use Your Hearing Aids All Day Long.

The best way to ensure better hearing is to practice listening with your hearing aids until you are able to wear them comfortably all day. Generally, infrequent use of the hearing aid does not provide you with the full benefit of amplification.

Your hearing aids will not restore normal hearing. Hearing aids will not prevent or improve hearing impairment resulting from a physiological condition. Hearing aids will help you to make better use of your residual hearing.

The use of hearing aids is only part of hearing rehabilitation; auditory training and lip reading instruction may be required as well.

Possible Side Effects.

The hearing aid or earmold is a custom-made product. If you are experiencing discomfort, see your audiologist or hearing care professional immediately for modification of the hearing aid or earmold at the point of irritation. A hearing aid or earmold may cause an accelerated accumulation of cerumen (earwax). Also in rare cases, the otherwise non-allergenic material may cause a discharge from the ear, allergic reaction, or any other unusual condition. Please seek immediate consultation with a physician if these conditions occur.

Warning to Hearing Care Professionals.

A hearing care professional should advise a prospective hearing aid user to consult promptly with a licensed physician (preferably an ear specialist) before dispensing a hearing aid if the hearing care professional determines through inquiry, actual observation, or review or any other available information concerning the prospective user, that the prospective user has any of the following conditions:

- (i) Visible congenital or traumatic deformity of the ear.
- (ii) History of active drainage from the ear within the previous 90 days.
- (iii) History of sudden or rapidly progressive hearing loss within the previous 90 days.
- (iv) Acute or chronic dizziness.
- (v) Unilateral hearing loss of sudden or recent onset within the previous 90 days.
- (vi) Audiometric air-bone gap equal to or greater than 15 decibels at 500 hertz (Hz), 1000 Hz and 2000 Hz.
- (vii) Visible evidence of significant cerumen accumulation or a foreign body in the ear canal.
- (viii) Pain or discomfort in the ear.

Special care should be exercised in selecting and fitting a hearing aid whose maximum sound pressure level exceeds 132 decibels because

D

GB

F

I

E

NL

P

GR

TR

DK

FIN

N

S

LT

PL

RUS

RO

HR

HUN

SK

BG

D there may be risk of impairing the remaining hearing of the hearing aid user.

GB

Important Notice for Prospective Hearing Aid Users.

F Good health practice requires that a person with a hearing loss have a medical evaluation by a licensed physician (preferably a physician who specializes in diseases of the ear) before purchasing a hearing aid. I Licensed physicians who specialize in diseases of the ear are often referred to as otolaryngologists, otologists, or otorhinolaryngologists. E The purpose of medical evaluation is to assure that all medically treatable conditions that may affect hearing are identified and treated before the NL hearing aid is purchased.

P Following the medical evaluation, the physician will give you a written statement that states that your hearing loss has been medically evaluated and that you may be considered a candidate for a hearing aid. GR The physician will refer you to an audiologist or a hearing care professional, as appropriate, for a hearing aid evaluation.

TR The audiologist or hearing care professional will conduct a hearing aid evaluation to assess your ability to hear with and without a hearing aid. DK The hearing aid evaluation will enable the audiologist or dispenser to select and fit a hearing aid to your individual needs.

FIN If you have reservations about your ability to adapt to amplification, you should inquire about the availability of a trial-rental or purchase-option program. N Many hearing care professionals now offer programs that permit you to wear a hearing aid for a period of time for a nominal fee after which you may decide if you want to purchase the hearing aid.

S Federal law restricts the sale of hearing aids to those individuals who have obtained a medical evaluation from a licensed physician. LT Federal law permits a fully informed adult to sign a waiver statement declining the nominal fee after which you may decide if you want to purchase the hearing aid.

PL Federal law restricts the sale of hearing aids to those individuals who have obtained a medical evaluation from a licensed physician. RUS Federal law permits a fully informed adult to sign a waiver statement declining the medical evaluation for religious or personal beliefs that preclude RO consultation with a physician. The exercise of such a waiver is not in your best health interest and its use is strongly discouraged.

HR

Children with Hearing Loss.

HUN In addition to seeing a physician for a medical evaluation, a child with a hearing loss should be directed to an audiologist for evaluation and SK rehabilitation since hearing loss may cause problems in language development and the educational and social growth of a child. An audiologist is qualified by training and experience to assist in the evaluation and BG rehabilitation of a child with a hearing loss.

Identification Information.

The date of manufacture is incorporated into the serial number. The warranty is based upon the date of purchase, not date of manufacture. The serial number may look like this: AE12345. The first letter of the serial number refers to the month of manufacture of the hearing aid. For example, A to M is Jan to Dec, N to Z also denote Jan to Dec. The second letter refers to the year, where every two consecutive letters can be used in one-year increments. For example, the letters E and F denote 2013, the letters G and H denote 2014, and so on. Note that the letters I and O are not used in serial numbers, and letters J and K are not used for the year 2015.

Your hearing care professional can answer any questions you may have about the identifying code on your hearing aid.

OTHER WARNINGS!

- Do not use your hearing aid in potentially explosive environments.
- If you experience dizziness, drainage or bleeding from the ear, or sudden hearing loss, see your Physician immediately.
- A hearing aid may cause an accelerated accumulation of cerumen (earwax).
- Keep hearing aids out of the reach of children, pets, and individuals with diminished mental capacity.
- Hearing aids should be used only as directed and adjusted by your trained hearing care professional.
- A hearing aid may stop functioning, e.g. if the battery goes dead. You should be aware of this possibility, in particular if you are in traffic or otherwise depend on warning sounds.
- Do not switch hearing aids between ears or allow others to wear or use your hearing aids.
- HEARING AIDS WILL NOT RESTORE NORMAL HEARING AND WILL NOT PREVENT OR IMPROVE A HEARING IMPAIRMENT RESULTING FROM ORGANIC CONDITIONS.
- IN MOST CASES, INFREQUENT USE OF A HEARING AID DOES NOT PERMIT USER TO ATTAIN FULL BENEFIT FROM IT.
- THE USE OF THE HEARING AID IS ONLY PART OF HEARING HABILITATION AND MAY NEED TO BE SUPPLEMENTED BY AUDITORY TRAINING AND INSTRUCTION IN LIP READING.

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Hansaton Akustik GmbH · Sachsenkamp 5 · 20097 Hamburg · Germany