OWNER'S MANUAL

Flat Panel Digital X-ray Detector

Please read this manual carefully before operating your set and retain it for future reference.

14HJ701D-W

CE0123

www.lg.com

P/No.: SAC36933611 (1612-REV00)

Please note that this information is for proper use and safety of the equipment. The following symbols may indicate a hazardous situation in which, if not heeded, may result in serious injury or even death to the user or others, or damage to the equipment.

 WARNING	<p>Indicates warning and safety instructions. If not adhered to, it could result in death or serious injury to the user or others.</p>
 CAUTION	<p>Indicates a hazardous situation which, if not heeded, may result in minor or moderate injury to the user or others, or damage to the equipment.</p>

For users in the United States

- United State federal law restricts this equipment to be used by or on the order of a physician.
- Since the X-ray exposure condition can be changed depending on the age, gender and bone density of the patient, in case of Pediatric, X-ray exposure condition can be changed by expert's judge. For further information, please refer to FDA Pediatric X-ray Imaging webpage. <http://www.fda.gov/radiation-emittingproducts/radiationemittingproductsandprocedures/medicalimaging/ucm298899.htm>

For users in other countries

- This equipment is to be used by or on the order of a licensed person under the related laws for each country.

WARNING

- To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth.
- Do not modify this equipment without authorization of the manufacturer.

Intended use

- Flat Panel Digital X-ray Detector is indicated for digital imaging solution designed for general radiographic system for human anatomy. It is intended to replace film or screen based radiographic systems in all general purpose diagnostic procedures. Not to be used for mammography.

Authorized representative in the European community.

LG Electronics European Shared Service Center B.V.

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77, Sanho-daero, Gumi-si, Gyeongsangbuk-do, 39381, Republic of Korea

Tel : +82-1544-8777

WARNING: This product contains chemicals known to the State of California to cause cancer and birth defects or other reproductive harm. ***Wash hands after handling.*** (US only)

SAFETY PRECAUTION

Please read these safety precautions carefully before using the product.

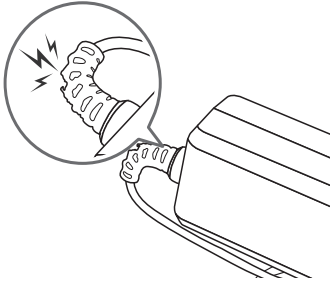
WARNING

- Do not leave any material that may cause fire in the vicinity of the detector.
- Please clean foreign material using dry cloth.
- Using any adapter or cable other than those provided with the product may result in malfunction.
- Unauthorized disassembly or reassembly of the product may result in electric shock or malfunction of the product.
- Be careful when handling the detector. Do not drop or knock the product.
- If the product emits smoke or strange odors during use, turn off the power immediately and remove the plug.
- For your safety, when cleaning the detector, turn off the detector and unplug the power cable.
- If the product is defective, do not attempt to repair it yourself. Instead, contact the Supplier.
- Always use the detector on a flat surface to prevent bending or damage.
- Make sure that the connecting parts of the detector are out of reach of patients.
- Do not apply excessive weight to the detector.
 - The detector may be damaged if the entire surface of the detector is subjected to a weight of 150 kg or more.
 - The detector may be damaged if the center of the detector is subjected to a weight of 100 kg or more.


Precautions In Using the AC adapter

Do not use the AC adapter when it is in an excessively bent position.

If the AC adapter cable is used for long periods while bent, it could cause internal wiring to become severed and cause electrical shocks or start a fire.

Integrated Adapter

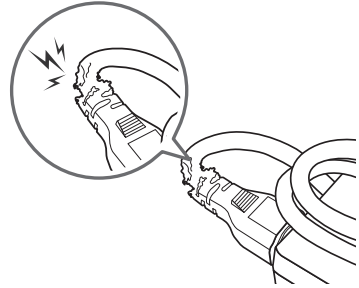Detachable Adapter

CAUTION

- Always use the detector in places that meet the following environmental requirements.

Item	Min.	Typ	Max. (%)	Units
Temperature (Storage)	-20		60	°C
Temperature (Operation)	10		35	°C
Humidity (Storage)			85	Non-condensing, Relative
Humidity (Operation)			80	Humidity
Pressure (Storage)	50		106	kPa
Pressure (Operation)	70		106	kPa

- Do not use the detector in the following places:
 - Places that are damp or in direct sunlight
 - Dusty places
 - Hot and humid places
 - Frozen places
 - Unstable places, such as those subject to severe vibrations or noise
 - Places with devices that generate a strong magnetic field
 - Any other place that may compromise the lifespan or performance of the detector
- Do not spray detergents, etc. onto the detector. Always wear gloves when cleaning the detector.

Precautions In Using the Battery

This product includes a battery. Please charge the battery fully before using the newly purchased detector for the first time.

- The battery is a consumable component. Its usable capacity reduces gradually over long-term use.
- Contact the manufacturer if any issues arise while removing or replacing the battery.
- Please refer to the manual for instructions on how to remove and replace the battery.
- Always use batteries approved and certified by manufacturer.
 - Failure to do so may cause a fire or explosion.
- Do not store the product in an enclosed space (bags, etc.) with the power on repeatedly or for long periods.
 - Overheated batteries may cause fire.
- Do not expose the product to flames.
 - This may result in explosion or fire.
- Keep the product away from metallic objects, such as car keys and paper clips, when storing or moving.
 - Excess current may cause a rapid temperature increase, resulting in fire or burns.
- Contact the manufacturer if the battery leaks or emits an odor.
 - Otherwise, it may cause explosion or fire.
- Do not recharge the product using a power supply or circuit other than the one supplied with the product.
 - This may damage the battery or cause a fire

SAFETY INFORMATION

Safety Standard

Medical Device Classification

Classification by protection type against Electric Shock	Class I equipment and Internally Powered Equipment
Classification according to the degree of protection against ingress of water or particulate matter	IP41
Mode of operation	Continuous Operation
Environment of Use	This equipment is not suitable for use in the presence of flammable anesthetic or oxygen.
Applied Part	Type : BF Type, Location : Front of Detector (only for effective area)

Regulations

Safety and Electromagnetic Compatibility Information

This equipment has been tested and found to comply with the limits for medical devices in IEC 60601-1-2. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation.

This equipment generates, uses and can radiate radio frequency energy. If not installed and used in accordance with the instructions, it may cause harmful interference to other devices in the vicinity.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures.

- Reorient or relocate the equipment.
- Increase the separation between the equipment.
- Connect the equipment into an outlet on a circuit different from that to which the other devices are connected.
- Contact manufacturer or authorized agent for help.

Radio Frequency compliance

FCC (For USA)

- 5.15- 5.25 GHz band is restricted to indoor operations only.
- Compliance with FCC requirement 15.407(c)

Data transmission is always initiated by software, which is then passed down through the MAC, through the digital and analog baseband, and finally to the RF chip. Several special packets are initiated by the MAC. These are the only ways the digital baseband portion will turn on the RF transmitter, which it then turns off at the end of the packet. Therefore, the transmitter will be on only while one of the aforementioned packets is being transmitted. In other words, this device automatically discontinues transmission in case of either absence of information to transmit or operational failure.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

FCC CAUTION

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This transmitter must not be co-located or operated in conjunction with any other antenna or transmitter.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment and meets the FCC radio frequency (RF) Exposure Guidelines in Supplement C to OET65.

When you use the detector with wire mode, the wireless function is automatically off.

- This device contains certified module. Contains FCC ID: BEJLGSWFAC73 / Contains IC: 2703H-LGSWFAC73

FCC WARNING

This equipment may generate or use radio frequency energy changes or modifications to this equipment may cause harmful interference unless the modifications are expressly approved in the instruction manual. The user could lose the authority to operate this equipment if an unauthorized change or modification is made

Note : This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment.

This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance of this equipment in a residential area is likely to cause harmful interference in which cause the user will be required to correct the interference at his own expense.

For Europe**DECLARATION OF CONFORMITY****[For Wi-Fi/Bluetooth built-in model or Wi-Fi built-in model]****CE 0197** 

Hereby, LG Electronics European Shared Service Center B.V., declares that this Flat Panel Digital X-ray Detector is compliant with the essential requirements and other relevant provisions of Directive 1999/5/EC. The complete Declaration of Conformity may be requested through the following postal address:

LG Electronics European Shared Service Center B.V., Krijgsman 1, 1186 DM Amstelveen, The Netherlands

or can be requested at our dedicated DoC website: <http://www.lg.com/global/support/cedoc/cedoc#>

This device is a 2.4 GHz and 5 GHz wideband transmission system, intended for use in all EU member states and EFTA countries, with indoor use restriction for 5 GHz band. (5150 – 5250 MHz)

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For Canada

This device complies with industry Canada's licence-exempt RSSs. Operation is subject to the following two conditions:

- (1) This device may not cause interference; and
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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5150~5250 MHz band is restricted to indoor operations only.

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Electro-Magnetic Compatibility Information

Electro-Magnetic Emissions

This EUT is intended for use in the electromagnetic environment specified below.

The customer or the user of the EUT should assure that it is used in such an environment.

Immunity Test	Compliance	Electromagnetic Environment – Guidance
RF Emissions CISPR 11	Group 1	The EUT uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions CISPR 11	Class A	The EUT is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	A	
Voltage fluctuations/ Flicker emissions IEC 61000-3-3	Complies	

Electro-Magnetic Immunity

This EUT is intended for use in the electromagnetic environment specified below.

The customer or the user of the EUT should assure that it is used in such an environment.

Immunity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic Environment – Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6kV Contact ±8kV Air	±6kV Contact ±8kV AIR	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Electrical fast transient/ burst IEC 61000-4-4	±2kV for power supply lines ± 1 kV for input/output lines	±2kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2kV common mode	±1 kV differential mode ±2kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines) IEC 61000-4-11	<5% UT (>95% dip in UT for 0.5 cycle). 40% UT (60% dip in UT for 5 cycle) 70% UT (30% dip in UT for 25cycle) <5% UT (<95% dip in UT) for 5s	<5% UT (>95% dip in UT for 0.5 cycle). 40% UT (60% dip in UT for 5 cycle) 70% UT (30% dip in UT for 25cycle) <5% UT (<95% dip in UT) for 5s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the EUT image intensifier requires continued operation during power mains interruptions, it is recommended that the EUT image intensifier be powered from an uninterruptible power supply or a battery.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

! NOTE

- UT is THE A.C mains voltage prior to application of the test level.

Immunity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic Environment – Guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms 150 kHz to 80 MHz	<p>Portable and mobile RF communications equipment should be used no closer to any part of the EUT, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance :</p> $d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$ $d = \left[\frac{3,5}{E_1} \right] \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left[\frac{7}{E_1} \right] \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts(W) according to the transmitter manufacturer and d is the recommended separation distance in meters(M).</p> <p>Field strengths from fixed RF transmitters as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol :</p> 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	E1=3 V/m	

! NOTE

- At 80 MHz and 800 MHz, the higher frequency range applies.
- These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- 1 Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the EUT is used exceeds the applicable RF compliance level above, the EUT should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the EUT.
- 2 Over the frequency range 150 KHz to 80 MHz, field strengths should be less than [V1] V/m.

Recommended separation distance between portable and mobile RF communications equipment and the EUT

There is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The user of the EUT can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the EUT as recommended below, according to the maximum output power of the communications equipment.

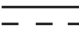
Separation distance according to frequency of transmitter [m] IEC 60601-1-2			
Frequency of Transmitter	150 kHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2.5 GHz
Equation	$d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$	$d = \left[\frac{3,5}{E_1} \right] \sqrt{P}$	$d = \left[\frac{7}{E_1} \right] \sqrt{P}$
	$V_1 = 3 \text{ Vrms}$	$E_1 = 3 \text{ V/m}$	$E_1 = 3 \text{ V/m}$
Rated maximum output power of transmitter [W]	Separation Distance (meters)	Separation Distance (meters)	Separation Distance (meters)
0.01	0.116	0.1166	0.2333
0.1	0.368	0.3687	0.7378
1	1.166	1.1660	2.3333
10	3.687	3.6872	7.3785
100	11.660	11.6600	23.333

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

! NOTE

- At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.
 - These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.
- 1 Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the EUT is used exceeds the applicable RF compliance level above, the EUT should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the EUT.
 - 2 Over the frequency range 150kHz to 80MHz, field strengths should be less than $[V_1] \text{ V/m}$.

Symbols

Symbols	Descriptions
	Refer to instruction manual / booklet
	Alternate current
	Direct current
	Protective earth (Ground)
	Stand-by Symbol.
	Off (power : disconnect from the main switch)
	On (power : connect from the main switch)
	Warning
	Caution
	UL classified mark of medical equipment with respect to electrical shock, fire and mechanical hazards only in accordance with ANSI/AAMI ES60601-1 (2005) + AMD 1 (2012), CAN/CSA-C22.2 No. 60601-1 (2014)
	Manufacturer
	Date of manufacture
	Type BF applied part
	Serial number

	Non-ionizing radiation
	WEEE : Waste Electrical and Electronic Equipment
	Authorized representative in the European community.
	Batteries Marking
CE0123	The Official Mark Of Europe Certificate
CE0197 ⓘ	CE Marking For Wireless
	Temperature limit
	Humidity limitation
	Pressure limitation
Rx ONLY	For the customers in the U.S.A. Caution Federal law (United States of America) restricts this device to sale by or on the order of a licensed healthcare practitioner.
IPN_iN₂	Ingress of protection

Warning

WARNING: To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth.

Connection

Do not connect the detector with anything other than specified. Otherwise, it may result in fire or electric shock.

To avoid the risk of electric shock, this detector must only be connected to supply mains with protective earth.

Handling

Always be sure to keep checking the condition of the system and the patient to ensure they are normal during the use of the detector. If any problem is found, take appropriate measures, such as stopping the operation of the detector, as required.

Never disassemble or modify the detector as it may result in fire or electric shock.

Also, since the detector incorporates parts that may cause electric shocks and other hazardous parts, touching them may cause death or serious injury.

Do not hit or drop the detector. The detector may be damaged if it receives a strong jolt, which may result in fire or electric shock if the detector is used without being repaired.

The operator must not touch connectors of the monitor and the patient simultaneously.

The product has lower breaking capacity type. So do not install at the building power system prospective short-circuit current exceeding 35 A.

Caution

Environment of Use and Storage

Do not install the detector in a location with the conditions listed below. Otherwise, it may result in failure or malfunction, cause fire or injury.

- Close to facilities where water is used.
- Where it will be exposed to direct sunlight.
- Close to air-conditioner or ventilation equipment.
- Close to heat source such as a heater.
- Prone to vibration
- Insecure place.
- Dusty environment.
- Saline or sulfurous environment.
- High temperature or humidity.
- Freezing or condensation.

Do not place the storage case in a location with the conditions listed below.

- Where the cable of the detector unit will be strongly pulled when the detector is put into the case, otherwise, the cable may be damaged, resulting in fire or electric shock.
- Where someone might get their foot caught in the cable of the detector.

Handling

For safety reasons, be sure to turn off the power of each equipment when detector is not used.

Location of Cables

Make sure all cables are located so that they cannot be stepped on, tripped over, or otherwise subjected to damage or stress.

Maintenance and Inspection

If the detector is defective, do not disassemble the detector randomly. Maintenance of the detector should be done by a manufacturer.

Battery

This product includes a battery. Please charge the battery fully before using the newly purchased detector for the first time.

- The battery is a consumable component. Its usable capacity reduces gradually over long-term use.
- Contact the manufacturer if any issues arise while removing or replacing the battery.
- How to replace the batteries, please refer to the user manual.
- Always use batteries approved and certified by manufacturer.
 - Failure to do so may cause a fire or explosion.
- Do not store the product in an enclosed space (bags, etc.) with the power on repeatedly or for long periods.
 - Overheated batteries may cause fire.
- Do not expose the product to flames.
 - This may result in explosion or fire.
- Keep the product away from metallic objects, such as car keys and paper clips, when storing or moving.
 - Excess current may cause a rapid temperature increase, resulting in fire or burns.
- Contact the manufacturer if the battery leaks or emits an odor.
 - Otherwise, it may cause explosion or fire.
- Do not recharge the product using a power supply or circuit other than the one supplied with the product.
 - This may damage the battery or cause a fire

Do not install the ME Equipment in a location without easy disconnect accessibility.

Changing the IT-Network at your own discretion may cause various problems. If you experience issues with your IT-Network, please contact the manufacturer to take the appropriate measures.

MANUFACTURER will make available on request circuit diagrams, component part lists, descriptions, calibration instructions, or other information that will assist SERVICE PERSONNEL to repair those parts of ME EQUIPMENT

Indications

The device is intended to capture for display radiographic images of human anatomy including both pediatric and adult patients. This device is used for generating diagnostic images by converting x-rays into electronic signals. Excluded from the indications for use are mammography.

**Contraindications**

The device is designed to be integrated into a complete X-ray system by qualified system integrator.

The device is not intended to be used as a primary barrier to X-rays.

Before using the X-ray system please refer to the regulation in force in your area concerning paediatric patients, pregnant women and anyone with health issues that contraindicate the use of X-rays. Investigate and make sure of this condition before starting the exposure.

**Clinical Risks and Benefits**

There is always a slight chance of cancer from excessive exposure to radiation. However, the benefit of an accurate diagnosis far outweighs the risk.

INFORMATION FOR ENVIRONMENTAL PRESERVATION

ENGLISH

Information for Environmental Preservation

LGE. announced the 'LG Declaration for a Cleaner Environment' in 1994, and this ideal has served as a guiding managerial principle ever since. The Declaration is a foundation that has allowed us to undertake environmentally friendly activities in careful consideration of economic, environmental, and social aspects.

We promote activities for environmental preservation, and we specifically develop our products to embrace the concept of environment-friendly. We minimize the hazardous materials contained in our products. For example, there is no cadmium to be found in our product.

Information for recycling

This product may contain parts which could be hazardous to the environment. It is important that this product be recycled after use.

LGE. handles all waste product through an environmentally acceptable recycling method. There are several take-back and recycling systems currently in operation worldwide. Many parts will be reused and recycled, while harmful substances and heavy metals are treated by an environmentally friendly method.

If you want to find out more information about our recycling program, please contact your local LG vendor or a corporate representative of LG.

We set our vision and policies on a cleaner world by selecting the issue of the global environment as a task for corporate improvement. Please visit our website for more information about our 'green' policies. <http://www.lg.com/global/sustainability/environment/take-back-recycling.jsp>

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NOM MARK (Mexico only)

(MARCA NOM (Solo para México))

CU Conformity Notice (Eurasian Customs Union only)

WEEE

ENGLISH

Disposal of your old appliance

- 1 This crossed-out wheeled bin symbol indicates that waste electrical and electronic products (WEEE) should be disposed of separately from the municipal waste stream.
- 2 Old electrical products can contain hazardous substances so correct disposal of your old appliance will help prevent potential negative consequences for the environment and human health. Your old appliance may contain reusable parts that could be used to repair other products, and other valuable materials that can be recycled to conserve limited resources.
- 3 You can take your appliance either to the shop where you purchased the product, or contact your local government waste office for details of your nearest authorised WEEE collection point. For the most up to date information for your country please see www.lg.com/global/recycling

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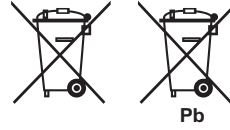
Battery

ENGLISH

Disposal of waste batteries/accumulators

This symbol may be combined with chemical symbols for mercury(Hg), cadmium(Cd) or lead(Pb) if the battery Contains more that 0.0005% of mercury, 0.002% of cadmium or 0.004% of lead.All batteries/accumulators should be disposed separately from the municipal waste stream via designated collection facilities appointed by the government or the local authorities.

The correct disposal of Your old batteries/accumulators will help to prevent potential negative consequences for the environment, animal and human health.

For more detailed information about disposal of Your old batteries/accumulators, please contact Your city office, waste disposal service or the shop where You purchased the product. (<http://www.lg.com/global/sustainability/environment/take-back-recycling/global-network-europe>)

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CE 0197

English

[For Wi-Fi/Bluetooth built-in model or Wi-Fi built-in model]

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LG Electronics European Shared Service Center B.V., Krijgsman 1, 1186 DM Amstelveen, The Netherlands or can be requested at our dedicated DoC website: <http://www.lg.com/global/support/cedoc/cedoc#>

This device is a 2.4 GHz and 5 GHz wideband transmission system, intended for use in all EU member states and EFTA countries, with indoor use restriction for 5GHz band. (5150 – 5250MHz)

WARNING : This is a class A product. In a domestic environment this product may cause radio interference in which case the user may be required to take adequate measures.

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