

# **User Manual**

## **WMTS Access Point**

### **AP70**

### **Mindray**

Shenzhen Mindray BIO-Medical electronics Co.,LTD.  
Mindray Building, Keji 12th Road South, Hi-tech Ind, Shenzhen, China



© Copyright 2019 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights reserved.

- Release time: 2019-10
- Revision: 4.0



## **WARNING**

---


- **Federal Law (USA) restricts this device to sale by or on the order of a physician or other practitioner licensed by U.S. state law to use or order the use of this device.**
- 
-

# Intellectual Property Statement

SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD. (hereinafter called Mindray) owns the intellectual property rights to this Mindray product and this manual. This manual may refer to information protected by copyrights or patents and does not convey any license under the patent rights of Mindray, nor the rights of others.

Mindray intends to maintain the contents of this manual as confidential information. Disclosure of the information in this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.

Release, amendment, reproduction, distribution, rental, adaption and translation of this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.

 **mindray**, **MINDRAY**, and **MINDRAY** are the registered trademarks or trademarks owned by Mindray in China and other countries. All other trademarks that appear in this manual are used only for editorial purposes without the intention of improperly using them. They are the property of their respective owners.

This posting serves as notice under 35 U.S.C. §287(a) for Mindray patents: <http://www.mindrayna.com/patents>.

## Manufacturer's Responsibility

Contents of this manual are subject to changes without prior notice.

All information contained in this manual is believed to be correct. Mindray shall not be liable for errors contained herein nor for incidental or consequential damages in connection with the furnishing, performance, or use of this manual.

Mindray is responsible for the effects on safety, reliability and performance of this product, only if:

- all installation operations, expansions, changes, modifications and repairs of this product are conducted by Mindray authorized personnel;
- the electrical installation of the relevant room complies with the applicable national and local requirements;
- the product is used in accordance with the instructions for use.

**WARNING**

---

- **Only skilled/trained clinical professionals should operate this equipment.**
  - **It is important for the hospital or organization that employs this equipment to carry out a reasonable service/maintenance plan. Neglect of this may result in machine breakdown or personal injury.**
- 

## Warranty

Mindray warrants that components within its products will be free from defects in workmanship and materials for a period of three years from the date of purchase except that disposable or one-time use products are warranted to be free from defects in workmanship and materials up to a date one year from the date of purchase or the date of first use, whichever is sooner.

This warranty does not cover consumable items such as, but not limited to, batteries, external cables, and sensors.

Mindray shall not be liable for any incidental, special, or consequential loss, damage, or expense directly or indirectly arising from the use of its products. Liability under this warranty and the buyer's exclusive remedy under this warranty is limited to servicing or replacing the affected products, at Mindray option, at the factory or at an authorized distributor, for any product which shall under normal use and service appear to Mindray to have been defective in material or workmanship. Recommended preventative maintenance, as prescribed in the service manual, is the responsibility of the user and is not covered by this warranty.

No agent, employee, or representative of Mindray has any authority to bind Mindray to any affirmation, representation, or warranty concerning its products, and any affirmation, representation or warranty made by any agent, employee, or representative shall not be enforceable by buyer or user.

THIS WARRANTY IS EXPRESSLY IN LIEU OF, AND MINDRAY EXPRESSLY DISCLAIMS, ANY OTHER EXPRESS OR IMPLIED WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY, NON-INFRINGEMENT, OR FITNESS FOR A PARTICULAR PURPOSE, AND OF ANY OTHER OBLIGATION ON THE PART OF MINDRAY.

Damage to any product or parts through misuse, neglect, accident, or by affixing any non-standard accessory attachments, or by any customer modification voids this warranty. Mindray makes no warranty whatsoever in regard to trade accessories, such being subject to the warranty of their respective manufacturers.

A condition of this warranty is that the equipment or accessories which are claimed to be defective be returned when authorized, freight prepaid to Mindray DS USA, Inc., Mahwah, New Jersey 07430 or its authorized representative. Mindray shall not have any responsibility in the event of loss or damage in transit.

## Exemptions

Mindray's obligation or liability under this warranty does not include any transportation or other charges or liability for direct, indirect or consequential damages or delay resulting from the improper use or application of the product or the use of parts or accessories not approved by Mindray or repairs by people other than Mindray authorized personnel.

This warranty does not extend to:

- Malfunction or damage caused by improper use or man-made failure.
- Malfunction or damage caused by unstable or out-of-range power input.
- Malfunction or damage caused by force majeure events, such as (i) flood, fire and earthquake or other similar elements of nature or acts of God; (ii) riots, war, civil disorders, rebellions, or revolutions in any country; or (iii) any other cause beyond the reasonable control of Mindray.
- Malfunction or damage caused by improper operation or repair by unqualified or unauthorized service people.
- Malfunction of the instrument or part whose serial number is not legible.
- Others not caused by instrument or part itself.

## Company Contact

Manufacturer: Shenzhen Mindray Bio-Medical Electronics Co., Ltd.  
Address: Mindray Building, Keji 12th Road South, Hi-tech industrial park, Nanshan, Shenzhen 518057, P.R.China  
Website: [www.mindray.com](http://www.mindray.com)  
E-mail Address: [service@mindray.com](mailto:service@mindray.com)  
Tel: +86 755 81888998  
Fax: +86 755 26582680

Distributor: Mindray DS USA, Inc.  
Address: 800 MacArthur Boulevard, Mahwah, New Jersey 07430 USA  
Tel: 1.800.288.2121, 1.201.995.8000  
Website: [www.mindray.com](http://www.mindray.com)

EC-Representative: Shanghai International Holding Corp. GmbH (Europe)  
Address: Eiffestraße 80, 20537 Hamburg, Germany  
Tel: 0049-40-2513175  
Fax: 0049-40-255726

# Preface

## Manual Purpose

This manual contains the instructions necessary to operate the product safely and in accordance with its function and intended use. Observance of this manual is a prerequisite for proper product performance and correct operation and ensures patient and operator safety.

This manual is based on the maximum configuration and therefore some contents may not apply to your product. If you have any questions, please contact Mindray.

This manual is an integral part of the product. It should always be kept close to the equipment so that it can be obtained conveniently when needed.

## Intended Audience

This manual is geared for clinical professionals who are expected to have a working knowledge of medical procedures, practices and terminology as required for monitoring of critically ill patients.

## Illustrations

All illustrations in this manual serve as examples only. They may not necessarily reflect the setup or data displayed on your telemetry monitor.

## Conventions

- *Italic text* is used in this manual to quote the referenced chapters or sections.
- [ ] is used to enclose screen texts.
- → is used to indicate operational procedures.

**This page intentionally left blank.**



# Contents

<b>1 Safety .....</b>	<b>1 - 1</b>
1.1 Safety Information .....	1 - 2
1.1.1 Warnings .....	1 - 3
1.1.2 Cautions .....	1 - 4
1.1.3 Notes .....	1 - 5
1.2 Equipment Symbols .....	1 - 6
<b>2 General Product Description .....</b>	<b>2 - 1</b>
2.1 Indications for Use .....	2 - 2
2.2 Applied Parts .....	2 - 2
2.3 Network Components of TM80/TM70 .....	2 - 3
2.4 Key Features .....	2 - 4
2.5 System Components .....	2 - 5
2.6 Physical View .....	2 - 6
2.7 Touch Screen Display .....	2 - 7
2.7.1 Display Screen .....	2 - 8
2.7.2 On-Screen Keyboard .....	2 - 10
<b>3 Getting Started .....</b>	<b>3 - 1</b>
3.1 Unpacking and Checking .....	3 - 2
3.2 Environmental Requirements .....	3 - 3
3.3 Connecting the ECG Leadwire .....	3 - 4
3.4 Installing the Batteries .....	3 - 5
3.4.1 Installing the Lithium-ion Rechargeable Battery .....	3 - 6
3.4.2 Installing the AA Batteries .....	3 - 7
3.5 Powering On the Unit .....	3 - 8
3.6 Understanding Touch Gestures .....	3 - 9
3.7 Basic Operations .....	3 - 10
3.7.1 Understanding the Screen Display Orientation .....	3 - 10
3.7.2 Browsing the Screen Display .....	3 - 10
3.7.3 Switching the Screen Display Orientation .....	3 - 11
3.7.4 Displaying the Quick Keys Area .....	3 - 11
3.7.5 Entering the Main Menu .....	3 - 12
3.7.6 Turning the Display Off .....	3 - 13
3.7.7 Turning the Display On .....	3 - 13
3.7.8 Unlocking the Screen in Locked Mode .....	3 - 13
3.7.9 Unlocking the Screen in View Only Mode .....	3 - 14
3.7.10 Acknowledging the Nurse Call .....	3 - 14
3.8 Using the Pouch .....	3 - 14
3.8.1 Securing the Pouch .....	3 - 14

<b>4 User Configurations .....</b>	<b>4 - 1</b>
4.1 Introduction .....	4 - 2
4.2 Configuring the Display .....	4 - 2
4.2.1 Entering the Display Setup Menu .....	4 - 2
4.2.2 Configuring the Default Display Orientation .....	4 - 2
4.2.3 Understanding Portrait Orientation Display Rules .....	4 - 2
4.2.4 Configuring the Portrait Display .....	4 - 2
4.2.5 Understanding Landscape Orientation Display Rules .....	4 - 3
4.2.6 Configuring the Landscape Display .....	4 - 3
4.2.7 Configuring the Display Brightness .....	4 - 4
4.3 Configuring the Audio Volume .....	4 - 4
<b>5 Patient Management .....</b>	<b>5 - 1</b>
5.1 Introduction .....	5 - 2
5.2 Admitting a Patient .....	5 - 2
5.3 Changing Patient Information .....	5 - 2
5.3.1 Changing the Patient Category .....	5 - 2
5.3.2 Changing Paced Status .....	5 - 3
5.3.3 Changing Department Name, Room Number, and Bed Number .....	5 - 3
5.4 Placing a Device in Standby .....	5 - 3
5.5 Exiting the Standby Mode .....	5 - 4
5.6 Discharging the Patient .....	5 - 4
5.6.1 Selecting the [Discharge Patient] menu .....	5 - 4
5.6.2 Restarting the TM80/TM70 .....	5 - 5
<b>6 Alarms .....</b>	<b>6 - 1</b>
6.1 Introduction .....	6 - 2
6.2 Alarm Safety Information .....	6 - 2
6.3 Alarm Categories .....	6 - 3
6.4 Alarm Levels .....	6 - 3
6.5 Alarm Indicators .....	6 - 4
6.5.1 Alarm Light .....	6 - 4
6.5.2 Alarm Tones .....	6 - 4
6.5.3 Flashing Numeric .....	6 - 5
6.5.4 Alarm Messages .....	6 - 5
6.5.5 Alarm Status Symbols .....	6 - 6
6.6 Viewing Alarm List .....	6 - 6
6.7 Configuring the Alarms .....	6 - 6
6.7.1 Changing Alarm Properties .....	6 - 6
6.7.2 Initiating Auto Alarm Limits .....	6 - 7
6.7.3 Restoring the Default Alarm Settings .....	6 - 8
6.8 Pausing Alarms .....	6 - 8
6.9 Resetting Alarms .....	6 - 9
6.9.1 Resetting Physiological Alarms .....	6 - 9
6.9.2 Resetting Technical Alarms .....	6 - 10

6.10 Latching Alarms .....	6 - 10
6.11 Actions When an Alarm Occurs .....	6 - 10
<b>7 Monitoring ECG .....</b>	<b>7 - 1</b>
7.1 Introduction .....	7 - 2
7.2 Safety .....	7 - 2
7.3 Preparation for Monitoring ECG .....	7 - 3
7.3.1 Preparing the Patient's Skin .....	7 - 3
7.3.2 Positioning the Electrodes .....	7 - 3
7.3.3 Setting ECG Lead Labeling .....	7 - 5
7.3.4 Placing the Electrodes .....	7 - 5
7.3.5 Checking the Lead Placement .....	7 - 10
7.3.6 Checking the Paced Status .....	7 - 11
7.4 Changing the ECG Settings .....	7 - 12
7.4.1 Configuring the ECG Setup .....	7 - 13
7.4.2 ECG Leadwire Types .....	7 - 14
7.4.3 Configuring the ECG Waveforms .....	7 - 14
7.4.4 Configuring the Pacer .....	7 - 15
7.4.5 Configuring the ECG Waveform Size .....	7 - 15
7.4.6 Configuring ECG Alarm Settings .....	7 - 16
7.4.7 Setting the Notch Filter .....	7 - 16
7.5 Configuring the HR Alarm Source .....	7 - 16
7.6 Understanding the ECG Display .....	7 - 17
7.6.1 HR Digital Area .....	7 - 17
7.6.2 About the HR Digital Area .....	7 - 17
7.6.3 ECG Waveform Area .....	7 - 18
7.6.4 About the ECG Waveform Area .....	7 - 18
7.7 ST Monitoring .....	7 - 18
7.7.1 ST Safety Information .....	7 - 19
7.7.2 Switching ST Monitoring On and Off .....	7 - 19
7.7.3 Displaying ST Numerics .....	7 - 20
7.7.4 Setting ST Alarms .....	7 - 20
7.8 QT/QTc Interval Monitoring .....	7 - 21
7.8.1 QT/QTc Monitoring Limitations .....	7 - 22
7.8.2 Enabling QT/QTc Monitoring .....	7 - 22
7.8.3 Displaying QT/QTc Numerics and Segments .....	7 - 23
7.8.4 Setting QT Alarms .....	7 - 23
7.9 Arrhythmia Monitoring .....	7 - 24
7.9.1 Arrhythmia Safety Information .....	7 - 24
7.9.2 Arrhythmia Events .....	7 - 25
7.9.3 Changing Arrhythmia Settings .....	7 - 27
7.9.4 Understanding the Arrhythmia Display .....	7 - 34
7.10 Relearning .....	7 - 34
7.10.1 Automatically Initiating an ECG Relearning .....	7 - 34
7.10.2 Manually Initiating an ECG Relearning .....	7 - 35

<b>8 Monitoring SpO2 (Optional)</b>	<b>8 - 1</b>
8.1 Introduction	8 - 2
8.2 Measurement Limitations	8 - 2
8.3 Safety	8 - 3
8.4 Connecting the SpO2 Module	8 - 4
8.5 Changing the SpO2 Settings	8 - 6
8.5.1 Configuring the SpO2 Setup	8 - 6
8.5.2 Configuring the SpO2 Waveform	8 - 9
8.5.3 Configuring SpO2 Alarm Settings	8 - 9
8.6 SpO2 Measurement	8 - 11
8.6.1 Identifying SpO2 Modules	8 - 11
8.6.2 Applying the Sensor	8 - 11
8.7 Understanding the SpO2 Display	8 - 12
8.7.1 SpO2 Digital Area	8 - 12
8.7.2 About the SpO2 Digital Area	8 - 13
8.7.3 SpO2 Waveform Area	8 - 13
8.7.4 About the SpO2 Waveform Area	8 - 14
8.8 Masimo Information	8 - 14
8.9 Nellcor Information	8 - 15
<b>9 Monitoring Resp (Optional)</b>	<b>9 - 1</b>
9.1 Introduction	9 - 2
9.2 Resp Safety Information	9 - 2
9.3 Preparing for Resp Monitoring	9 - 3
9.3.1 Preparing the Patient	9 - 3
9.3.2 Placing the Electrodes	9 - 3
9.4 Changing Resp Settings	9 - 4
9.4.1 Configuring the Resp Setup	9 - 4
9.4.2 Configuring the Resp Waveform	9 - 5
9.4.3 Configuring the Resp Alarm Settings	9 - 5
9.5 Understanding the Resp Display	9 - 6
9.5.1 Resp Digital Area	9 - 6
9.5.2 Resp Waveform Area	9 - 7
<b>10 Monitoring NIBP (Optional)</b>	<b>10 - 1</b>
10.1 Introduction	10 - 2
10.2 Pairing the TM80/TM70 with the BP10	10 - 2
10.2.1 Pairing Procedure	10 - 2
10.2.2 System Responses after Successful Pairing	10 - 3
10.3 Unpairing the TM80/TM70 with the BP10	10 - 4
10.3.1 Unpairing via the TM80/TM70	10 - 4
10.3.2 Unpairing via BP10	10 - 5
10.3.3 System Responses after Successful Unpairing	10 - 5
10.4 Screen Display after Pairing a TM80/TM70 with a BP10	10 - 6

10.5 Interactions after Pairing a TM80/TM70 with a BP10 .....	10 - 7
10.5.1 Overview of Interactions .....	10 - 7
10.5.2 Operating the BP10 via the TM80/TM70 .....	10 - 7
10.5.3 NIBP Operations at the TM80/TM70 .....	10 - 8
10.5.4 Adjusting NIBP Limits .....	10 - 8
<b>11 Review .....</b>	<b>11 - 1</b>
11.1 Introduction .....	11 - 2
11.2 Entering the List Trends Menu .....	11 - 2
11.3 Example List Trends .....	11 - 2
11.4 Changing the Resolution of Trend Data .....	11 - 4
11.5 Searching Trend Data .....	11 - 5
<b>12 Monitoring with the TM80/TM70 at the CMS .....</b>	<b>12 - 1</b>
12.1 Introduction .....	12 - 2
12.2 Safety .....	12 - 2
12.3 Physiological Alarms .....	12 - 3
12.4 Factory Default Parameter Alarm Settings .....	12 - 3
12.5 Factory Default Arrhythmia Alarm Settings .....	12 - 6
12.6 Arrhythmia Threshold Settings .....	12 - 8
12.7 ECG Monitoring .....	12 - 8
12.7.1 Changing ECG Alarm Properties .....	12 - 9
12.7.2 Changing ECG Wave Settings .....	12 - 9
12.7.3 Changing Pacer Settings .....	12 - 10
12.7.4 Adjusting the Minimum QRS Detection Threshold .....	12 - 10
12.7.5 ECG Display .....	12 - 11
12.8 QT Monitoring .....	12 - 12
12.8.1 Measurement Limitations .....	12 - 12
12.8.2 Entering the QT Tab .....	12 - 13
12.8.3 Changing QT Alarm Properties .....	12 - 13
12.8.4 Changing QT Settings .....	12 - 13
12.8.5 QT View Menu .....	12 - 13
12.8.6 QT Display .....	12 - 15
12.9 ST Monitoring .....	12 - 15
12.9.1 Measurement Limitations .....	12 - 16
12.9.2 Entering the ST Tab .....	12 - 17
12.9.3 Changing ST Alarm Properties .....	12 - 17
12.9.4 Changing ST Settings .....	12 - 17
12.9.5 Adjusting ST Measurement Points .....	12 - 18
12.9.6 Entering the ST Graphic Window .....	12 - 18
12.9.7 Entering the ST View .....	12 - 19
12.9.8 ST Display .....	12 - 20
12.9.9 ST Segment Display .....	12 - 20
12.10 Arrhythmia Monitoring .....	12 - 21
12.10.1 Entering the Arrhythmia Tab .....	12 - 21
12.10.2 Measurement Limitations .....	12 - 21

12.10.3 Understanding the Arrhythmia Events .....	12 - 22
12.10.4 Setting Arrhythmia Alarm Properties .....	12 - 22
12.10.5 Changing Arrhythmia Alarm Threshold Settings .....	12 - 22
12.10.6 Setting PVC-Related Alarms Threshold .....	12 - 22
12.10.7 Configuring the Arrhythmia Shield Time .....	12 - 23
12.10.8 Arrhythmia Display .....	12 - 24
12.10.9 Relearning .....	12 - 24
12.11 Resp Monitoring .....	12 - 24
12.11.1 Changing Resp Alarm Properties .....	12 - 25
12.11.2 Changing Resp Settings .....	12 - 25
12.11.3 Resp Display .....	12 - 25
12.12 SpO2 Monitoring .....	12 - 26
12.12.1 Measurement Limitations .....	12 - 26
12.12.2 Changing SpO2 Alarm Properties .....	12 - 27
12.12.3 Changing SpO2 Settings .....	12 - 27
12.12.4 SpO2 Display .....	12 - 27
12.12.5 SpO2 Waveform Area .....	12 - 28
12.13 NIBP Monitoring .....	12 - 29
12.13.1 Measurement Limitations .....	12 - 29
12.13.2 Changing NIBP Alarm Properties .....	12 - 29
12.13.3 Changing NIBP Settings .....	12 - 29
12.13.4 NIBP Display .....	12 - 29
12.13.5 NIBP List .....	12 - 30
12.14 Locating the TM80/TM70 .....	12 - 30
<b>13 Configuring the TM80/TM70 .....</b>	<b>13 - 1</b>
13.1 Introduction .....	13 - 2
13.2 Maintenance Menu .....	13 - 2
13.2.1 Entering the Maintenance menu .....	13 - 2
13.2.2 Configuring the General Menu .....	13 - 2
13.2.3 Configuring the Alarms Menu .....	13 - 4
13.2.4 Quick Keys Menu .....	13 - 6
13.2.5 Configuring the Network Menu .....	13 - 7
13.2.6 Configuring the Defaults Menu .....	13 - 12
13.2.7 Transferring a Configuration .....	13 - 13
13.2.8 Screen Lock Menu .....	13 - 14
13.2.9 Changing the Passwords .....	13 - 15
13.2.10 Changing the Device Name .....	13 - 15
13.3 Setting CMS Disconnect Alarm .....	13 - 16
13.3.1 Demo Mode .....	13 - 16
13.3.2 Service Menu .....	13 - 16
13.4 Configuring WMTS Setting .....	13 - 16
<b>14 Battery .....</b>	<b>14 - 1</b>
14.1 Introduction .....	14 - 2
14.2 Safety .....	14 - 2

14.3 Installing the Battery .....	14 - 3
14.4 Checking the Battery Charge Status .....	14 - 4
14.5 Removing the Battery .....	14 - 4
14.6 Charging the Rechargeable Lithium-ion Battery .....	14 - 5
14.7 Storing the Batteries .....	14 - 6
14.7.1 Storing Rechargeable Lithium-ion Battery .....	14 - 6
14.7.2 Storing AA Batteries .....	14 - 6
14.8 Maintaining the Rechargeable Lithium-ion Battery .....	14 - 7
14.9 Disposing of the Batteries .....	14 - 8
14.9.1 Disposing of the Rechargeable Lithium-ion Battery .....	14 - 8
14.9.2 Disposing of the AA Batteries .....	14 - 8
<b>15 Troubleshooting .....</b>	<b>15 - 1</b>
15.1 General Problems .....	15 - 2
15.2 Physiological Alarm Messages at the TM80/TM70 .....	15 - 2
15.3 Technical Alarm Messages at the TM80/TM70 .....	15 - 5
<b>16 Cleaning and Disinfecting .....</b>	<b>16 - 1</b>
16.1 Introduction .....	16 - 2
16.2 Safety Information .....	16 - 2
16.3 Cleaning the TM80/TM70 .....	16 - 3
16.4 Cleaning the Reusable ECG Leadwires, SpO2 Modules and Sensors .....	16 - 4
16.5 Cleaning the Battery and Battery Compartment .....	16 - 5
16.6 Disinfection .....	16 - 6
16.7 Sterilization .....	16 - 8
<b>17 Maintenance .....</b>	<b>17 - 1</b>
17.1 Introduction .....	17 - 2
17.2 Safety .....	17 - 2
17.3 Regular Check .....	17 - 3
17.3.1 Power-on Test .....	17 - 3
17.3.2 Battery Check .....	17 - 3
17.4 Maintenance and Testing Schedule .....	17 - 4
17.5 Checking the System Information .....	17 - 4
17.6 Verifying the ECG .....	17 - 5
17.6.1 NIBP Accuracy Test .....	17 - 5
17.6.2 NIBP Leakage Test .....	17 - 5
17.6.3 Nurse Call Test .....	17 - 5
<b>18 Accessories .....</b>	<b>18 - 1</b>
18.1 ECG Accessories .....	18 - 2
18.1.1 ECG Electrodes .....	18 - 2
18.1.2 ECG Leadsets .....	18 - 3
18.2 SpO2 Accessories .....	18 - 4

18.2.1 Masimo SpO2 Module .....	18 - 4
18.2.2 Masimo SpO2 Sensor .....	18 - 4
18.2.3 Nonin SpO2 Module .....	18 - 5
18.2.4 Nonin SpO2 Sensor .....	18 - 5
18.2.5 Nellcor SpO2 Module .....	18 - 5
18.2.6 Nellcor SpO2 Sensor .....	18 - 5
18.3 Miscellaneous .....	18 - 6
18.4 NIBP Accessories .....	18 - 6

## **A Product Specifications ..... A - 1**

A.1 Classifications .....	A - 2
A.2 Environmental Specifications .....	A - 2
A.3 Power Supply Specifications .....	A - 3
A.3.1 TM80/TM70 .....	A - 3
A.3.2 Central Charger .....	A - 4
A.4 Physical Specifications .....	A - 4
A.5 Hardware Specifications.....	A - 4
A.5.1 TM80/TM70 .....	A - 4
A.5.2 AP70 .....	A - 5
A.5.3 AC70 .....	A - 6
A.5.4 SYNC70 .....	A - 6
A.5.5 Central Charger .....	A - 7
A.6 Wireless Specification .....	A - 8
A.6.1 TM80 wifi specification .....	A - 8
A.6.2 TM70 and AP70 WMTS Specifications .....	A - 10
A.7 Bluetooth Specification .....	A - 12
A.7.1 Technical Specification .....	A - 12
A.7.2 Implemented Function .....	A - 13
A.7.3 Function Specification .....	A - 14
A.8 Measurement Specifications .....	A - 15
A.8.1 ECG .....	A - 15
A.8.2 SpO2 .....	A - 18
A.8.3 Resp .....	A - 25

## **B EMC and Radio Regulatory Compliance ..... B - 1**

B.1 EMC .....	B - 2
B.2 Radio Regulatory Compliance .....	B - 7

## **C Symbols and Abbreviations ..... C - 1**

C.1 Units .....	C - 2
C.2 Symbols .....	C - 3
C.3 Abbreviations .....	C - 3



---

# 1      **Safety**

---

Safety Information.....	1-2
Equipment Symbols.....	1-6

---

## 1.1 Safety Information

---



### **WARNING**

---

- **Indicates a potential hazard or unsafe practice that, if not avoided, could result in death or serious injury.**
- 



### **CAUTION**

---

- **Indicates a potential hazard or unsafe practice that, if not avoided, could result in minor personal injury or product/property damage.**
- 

### **NOTE**

---

- **Provides application tips or other useful information to ensure that you get the most from your product.**
-

---

## 1.1.1 Warnings

---



### WARNING

---

- The TM80/TM70 is intended to be used for a single patient at a time.
  - The TM80/TM70 must be operated by medical personnel in hospitals.
  - To avoid explosion hazard, do not use the equipment in the presence of oxygen-rich atmospheres, flammable anesthetics, or other flammable agents.
  - Do not use TM80/TM70 in conjunction with Electro Surgical Unit (ESU).
  - Do not expose the TM80/TM70 to a Magnetic Resonance (MR) environment.
    - ◆ Thermal injury and burns may occur due to the metal components of the equipment which can heat during MR scanning.
    - ◆ The equipment may present a risk of projectile injury due to the presence of ferromagnetic materials which can be attracted by the MR magnet core.
    - ◆ The leadwires and electrodes will generate artifacts in the MR image.
    - ◆ The equipment will not function properly due to the strong magnetic and radio frequency fields generated by the MR scanner.
  - Before putting the system into operation, the operator must verify that the equipment, connecting cables and accessories are in correct working order and operating condition.
  - Do not come into contact with the patient during defibrillation. Otherwise serious injury or death could result.
  - Do not touch the patient and live parts simultaneously.
  - Do not open the equipment housings. All servicing and future upgrades must be carried out by trained and authorized personnel.
  - Do not rely exclusively on the audible alarm system for monitoring. Adjustment of alarm volume to a low level may result in a hazard to the patient. Always keep the patient under close surveillance.
  - The physiological data and alarm messages displayed on the TM80/TM70 are for reference only and cannot be directly used for diagnostic interpretation.
  - Do not operate the touch screen with water on the hand.
-



## **WARNING**

---

- Only use parts and accessories specified in this manual.
  - Route, wrap and secure the cables to avoid inadvertent disconnection, stumbling and entanglement
- 

### **1.1.2 Cautions**

---



## **CAUTION**

---

- Do not let the display of the TM80/TM70 directly touch the patient when the display is on.
  - When the Central Monitoring System presents the alarm “Offline” for TM80/TM70, the setting being performed on the TM80/TM70 may not be transferred to the Central Monitoring System. Check the patient condition and the settings on the Central Monitoring System.
  - When disposing of the packaging material, be sure to observe the applicable waste control regulations and keep it out of children’s reach.
  - The patient should be required to move in a specified area. If the patient is at the edge of or outside the network coverage range, unstable network connection may compromise the monitoring performance. The patient’s location is of vital importance for the TM80/TM70. If a life-threatening situation occurs for a patient, this patient must be located and found by medical staff immediately.
  - Magnetic and electrical fields are capable of interfering with the proper performance of the equipment. For this reason make sure that all external equipment operated in the vicinity of the equipment comply with the relevant EMC requirements. Mobile phone, X-ray equipment or MRI equipment are a possible source of interference as they may emit higher levels of electromagnetic radiation.
  - Always install or carry the TM80/TM70 properly to avoid damage caused by drop, impact, strong vibration or other mechanical force.
  - Dry the TM80/TM70 immediately in case of rain or water spray.
-



## CAUTION















- 
- **The TM80/TM70 generates and uses the Radio Frequency (RF) energy. If it is not installed correctly or not used as per the manual, RF interference to other equipment could result.**
  - **Signal quality can be impacted on an ambulatory patient by the construction materials used within the hospital.**
  - **At the end of its service life, the equipment, and its accessories, must be disposed of in compliance with the guidelines regulating the disposal of such products. If you have any questions concerning disposal of the equipment, please contact Mindray.**
- 

### 1.1.3 Notes

#### NOTE

- 
- **Put the equipment in a location where you can easily view and operate the equipment.**
  - **The equipment uses a mains plug as isolation means to the mains power supply. Please do not locate the equipment in a place difficult to operate the mains plug.**
  - **In normal use, the operator shall stand in front of the equipment.**
  - **The software was developed in compliance with IEC60601-1-4. The possibility of hazards arising from software errors is minimized.**
  - **This manual describes all features and options. Your equipment may not have all of them.**
  - **Keep this manual in the vicinity of the equipment so that it can be obtained conveniently when needed.**
-

## 1.2 Equipment Symbols

Symbol	Description	Symbol	Description
	Power On/Off key		Main menu key
	Nurse call key		Symbol for aligning the SpO <sub>2</sub> connector
	Charge indicator		Refer to instruction manual/booklet
	Defibrillation-proof Type CF applied part		Serial number
	Date of Manufacture		Symbol for "MANUFACTURER"
	MR Unsafe – do not subject to magnetic resonance imaging (MRI)	IPX7	Protection against fluid ingress
	Interference may occur in the vicinity of equipment marked with this symbol		General warning sign
	<p>The presence of this label indicates the machine was certified by ETL with the statement:</p> <p>Conforms to AAMI Std ES 60601-1, IEC 60601-1-6, IEC Std 60601-1-8, IEC Std 60601-2-27, IEC Std 60601-2-49, ISO Std 80601-2-61, IEC Std. 80601-2-30,</p> <p>Certified to CSA Std C22.2 NO. 60601-1, NO. 60601-1-6, NO. 60601-1-8, NO. 60601-2-27, NO. 60601-2-49, NO. 80601-2-61, NO. 80601-2-30</p>		

### NOTE

- Some symbols may not appear on your equipment.

---

# 2

## General Product Description

---

Indications for Use .....	2-2
Applied Parts .....	2-2
Network Components of TM80/TM70.....	2-3
System Components .....	2-5
Physical View .....	2-6
Touch Screen Display.....	2-7

## 2.1 Indications for Use

The TM80/TM70 telemetry monitor is intended for use on Adult and Pediatric patients over three years old to monitor ECG, SpO<sub>2</sub>, NIBP and Resp physiological data. The physiological data can be analyzed, alarmed, stored, reviewed locally on the display of the monitor, and the CentralStation can config and display the physiological parameters from the TM80/TM70.

The TM80/TM70 telemetry monitor is intended for use in professional healthcare facilities under the direct supervision of a licensed healthcare practitioner.



### WARNING

- 
- **Only skilled/trained clinical professionals should operate the TM80/TM70.**
  - **The TM80/TM70 telemetry monitor is not designed for monitoring critically ill patients.**
  - **If the accuracy of any value displayed at the Central Monitoring System (hereinafter abbreviated as CMS) or TM80/TM70 is questionable, determine the patient's vital signs by alternative means and verify that the telemetry device is working correctly.**
  - **The TM80/TM70 telemetry monitor transmits the data through the wireless connection. There might be a risk of data loss.**
- 
- 

## 2.2 Applied Parts

The TM80/TM70 has the following applied parts:

- ECG leadwires
- SpO<sub>2</sub> cables
- SpO<sub>2</sub> sensors
- NIBP cuffs



## 2.3 Network Components of TM80/TM70

TM80/TM70 network consists of the following components:

- Central Monitoring System (CMS):

The BeneVision Central Monitoring System ( CMS) is a networked patient monitoring system intended for use in a fixed location, installed in professional healthcare facilities to provide clinicians remote patient monitoring.

- AC70:

The AC70 is the access controller for AP70 and TM70 management and event review.

- SYNC70:

SYNC70 is used to synchronize time between AP70s and TM70s to reduce wireless interference.

- AP70/AP:

The AP70 is used to bridge TM70 to the wired network while AP is used to bridge TM80 to the wired network.


- TM80/TM70

The TM80/TM70 is used to monitor the patient's ECG, SpO2, NIBP, and Resp physiological data.

- NIBP module

NIBP module is used to measure, display, review, store the NIBP parameter for ambulating adult and pediatric patients, and transfer the information to other devices in this system.

The AC70, CMS, and SYNC70 are connected with the AP70/AP through the wired network. The TM80/TM70 is connected with the AP70/AP through the wireless network.

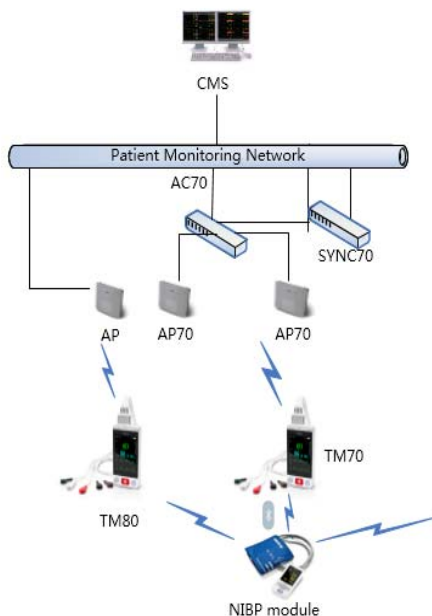

TM80/TM70 Network Components

## 2.4 Key Features

- 3.5" color PTC touch screen display is easy for clinicians to use.
- Small, portable, and lightweight for patients to wear.
- The TM80/TM70 supports 3/5/6-lead ECG.
- Supports Masimo SpO<sub>2</sub> module, Nonin SpO<sub>2</sub> module, and Nellcor SpO<sub>2</sub> module.
- Communication to the CMS utilizes WLAN for the TM80/TM70.
- Displays the battery status and supports the multiple levels of battery alarms.
- Displays Heart Rate (HR) , Resp, SpO<sub>2</sub>, and NIBP parameter numerics and ECG, SpO<sub>2</sub>, and Resp waveforms.
- Battery options of three AA or lithium-ion battery pack are available.
- Supports parameter auto-sizing.

---


## 2.5 System Components

The TM80/TM70 telemetry monitor includes the following components:

- TM80/TM70 Telemetry Monitor main unit
- NIBP module (BP10)
- Central charger
- Accessories

This manual only describes the TM80/TM70 Telemetry Monitor main unit (hereinafter called the TM80/TM70). For information about the BP10, the central charger, and the central monitoring system, refer to ***BP10 NIBP Module Operator's Manual (P/N 046-008269-00)***, ***BeneVison Central Charger Operator's Manual (P/N 046-007059-00)***, and ***BeneVision Central Monitoring System Operator's Manual (P/N 046-007960-00)*** respectively.

## 2.6 Physical View

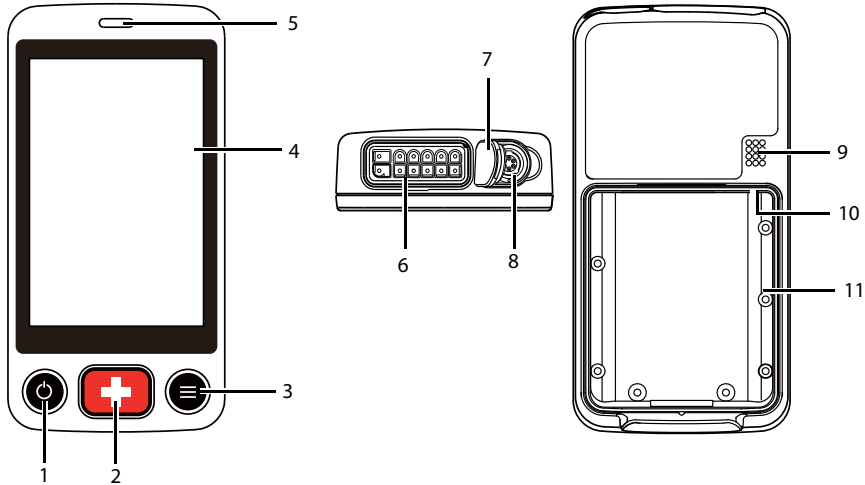

### 1. Display Activation (Power On/Off) key

When the device is powered off

- ◆ Pressing this key will turn the device on.

When the device is powered on

- ◆ If the screen display is on, pressing this key will turn the display off.
- ◆ If the screen display is off, pressing this key will turn the display on.
- ◆ Press and hold this key for two seconds to display the power off confirmation menu.

### 2. Nurse Call key

Pressing this key will send a nurse call request to the CMS. The alarm light/indicator will illuminate cyan, and a "Nurse Call Initiated" message will display in the message area if the display is on.

### 3. Main Menu key

- ◆ Pressing this key when on the main screen will open the main menu.
- ◆ Pressing this key when a menu is open will return to the main screen.
- ◆ Pressing this key when the display is off will turn the display on.
- ◆ Pressing this key when the screen lock mode is configured for View Only will display the **[Screen Locked]** menu.

### 4. Display

The touch screen is the primary user interface for operating the device and viewing information (or patient data).

5. Alarm light/indicator  
Flashes in different color and frequency corresponding to the alarm level.
6. ECG connector  
Connects the ECG leadset.
7. SpO<sub>2</sub> cap  
Covers SpO<sub>2</sub> connector when SpO<sub>2</sub> is not in use.
8. SpO<sub>2</sub> connector  
Connects the SpO<sub>2</sub> module.
9. Speaker
10. USB connector  
It is only available for authorized service personnel.
11. Battery compartment  
Contains the lithium-ion battery pack or AA battery frame.

## 2.7 Touch Screen Display



### WARNING

- 
- **Do not operate the touch screen with water on the hand.**
- 
- 


Move your finger on the touch screen display to operate the TM80/TM70. For details about the supported touch gestures, refer to ***"Understanding the Screen Display Orientation" on page 3 - 10.***

2.7.1 Display Screen

The main screen displays patient parameters and waveforms. A typical display screen of TM80/TM70 is shown below.



TM80

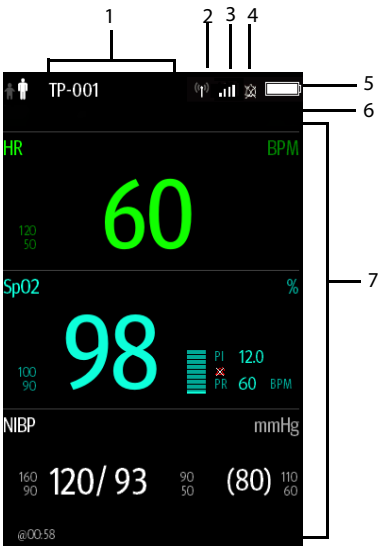



TM70











1. Patient information area

This area shows the patient information such as patient size, device name, department name, room number, and bed number. Tapping this area displays the **[Patient Info]** menu.




2. Mindray Patient Area Network (herein abbreviated as MPAN) indicator

- ◆  indicates that the TM80/TM70 is not paired with the BP10 or one TM80/TM70 is not connected to another TM80/TM70 for configuration transfer.
- ◆  indicates that the TM80/TM70 is paired with the BP10 successfully or configuration transfer of the TM80/TM70 is in progress.

3. Wireless symbol

- ◆  /  indicates that the TM80/TM70 has been connected to the wireless access point and the signal coverage is very good. Stable data transmission via wireless network can be guaranteed.
- ◆  /  indicates that the TM80/TM70 has been connected to the wireless access point and the signal coverage is good. Stable data transmission via wireless network can be guaranteed.
- ◆  /  indicates that the TM80/TM70 has been connected to the wireless access point and the signal coverage can basically meet the requirements. Stable data transmission via wireless network can be guaranteed.
- ◆  /  indicates that the TM80/TM70 has been connected to the wireless access point and the signal coverage is weak. wireless offline may occur.
- ◆  /  indicates that the TM80/TM70 is not connected to the wireless access point.

4. Alarm symbols

- ◆  indicates that all the alarms are paused.
- ◆  indicates that the alarm system is reset.
- ◆  indicates that the alarm audio is turned off.

5. Battery symbol

This symbol indicates the battery charge status. For details about the battery status symbols, refer to **"Checking the Battery Charge Status" on page 14 - 4**. Tapping the battery symbol opens the **[System Info]** menu to the battery section.

6. Message area

This area shows physiological alarm messages, technical alarm messages, and informational messages, where there are multiple messages, the messages scroll.

Tapping this area displays the **[Alarm Summary]** menu, where you can view alarm messages.

7. Patient data area

This user configurable area can display parameter/waveform data. The parameter/waveform is labeled in the upper left corner. You may also tap this area to display the Setup menu for the corresponding parameter/waveform.

For details about the touch screen operations, refer to **"Basic Operations" on page 3 - 10**.

2.7.2 On-Screen Keyboard


The TM80/TM70 uses an on-screen keyboard to enter alphanumeric information, such as the device name and passwords.

2.7.2.1 Alphabetic Keyboard



- 1. Alphabetic buttons: tap to input the desired alphabetic text.
- 2. Delete button: tap to erase the text to the left of the cursor.
- 3. Accept button: tap to save the settings and exit the keyboard.
- 4. Space button: tap to input a space.
- 5. Numeric switch button: tap to switch to the numeric layout.
- 6. Case shift button: tap to switch the case of the letter.  
This switch is active for one character entry.

2.7.2.2 Numeric Keyboard

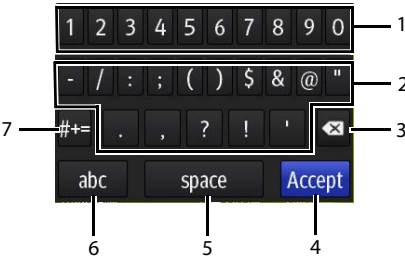



1. Numeric buttons: tap to input the desired numbers.
2. Punctuation buttons: tap to input the desired punctuation mark or symbol.
3. Delete button: tap to erase the text to the left of the cursor.
4. Accept button: tap to save the settings and exit the keyboard.
5. Space button: tap to input a space.
6. Alphabetic switch button: tap to switch to the alphabetic layout.
7. More punctuation buttons: tap to display the punctuation keyboard, as shown below.





---

# 3

## Getting Started

---

Unpacking and Checking.....	3-2
Environmental Requirements .....	3-3
Connecting the ECG Leadwire .....	3-4
Installing the Batteries .....	3-5
Powering On the Unit.....	3-8
Understanding Touch Gestures .....	3-9
Basic Operations.....	3-10
Using the Pouch .....	3-14



## WARNING

---

- **The network shall be installed by Mindray authorized personnel.**
  - **The software equipment copyright is solely owned by Mindray. No organization or individual shall resort to altering, copying, or exchanging it or to any other infringement on it in any form or by any means without due permission.**
  - **Connect only approved devices to this system. Devices connected to the equipment must meet the requirements of the applicable IEC standards (e.g. IEC 60950 safety standards for information technology equipment and IEC 60601-1 safety standards for medical electrical equipment). The system configuration must meet the requirements of the IEC 60601-1 medical electrical systems standard. Any personnel who connect devices to the equipment's signal input/output port is responsible for providing evidence that the safety certification of the devices has been performed in accordance to the IEC 60601-1. If you have any questions, please contact Mindray.**
  - **If it is not evident from the equipment specifications whether a particular combination with other devices is hazardous, for example, due to summation of leakage currents, please consult the manufacturers or else an expert in the field, to ensure the necessary safety of patients and all devices concerned will not be impaired by the proposed combination.**
  - **Contact Mindray to relocate the TM80/TM70.**
  - **Only Mindray authorized personnel can update the TM80/TM70.**
- 

## 3.1 Unpacking and Checking

Before unpacking, examine the packing case carefully for signs of damage. If any damage is detected, contact the carrier or Mindray.

If the packing case is intact, open the package and remove the device and accessories carefully. Check all materials against the packing list and check for any mechanical damage. Contact Mindray in case of any problem.

**WARNING**

- **Before use, please verify whether the packages are intact, especially the packages of single use accessories. In case of any damage, do not apply it to patients.**
- 
- 

**NOTE**

- **Save the packing case and packaging material as they can be used if the device must be reshipped.**
- 

## 3.2 Environmental Requirements

The operating environment of the system must meet the requirements specified in this manual.

The equipment operating environment should be reasonably free from noises, vibration, dust, corrosive, flammable and explosive substances.

When the device is moved from one place to another, condensation may occur as a result of temperature or humidity difference. In this case, never start the system before the condensation disappears.

**WARNING**

- **Make sure that the device operating environment meets the specifications. Otherwise unexpected consequences, e.g. damage to the device, could result.**
- 
-

---


**NOTE**

---


- **The system transmits data through a wireless connection. External radio frequency interference may result in occasionally data dropout. Contact Mindray for any questions regarding the electromagnetic environment.**
- 

### 3.3 Connecting the ECG Leadwire

1. Align the ECG leadwire plug with the ECG connector as indicated by the arrow in the following figure.

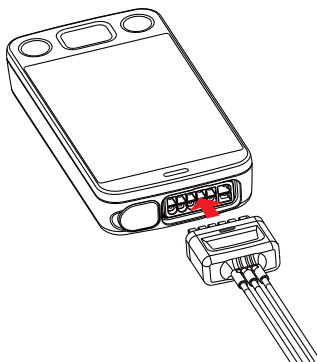

2. Insert the ECG leadwire plug into the ECG connector as shown in the enlarged figure below.

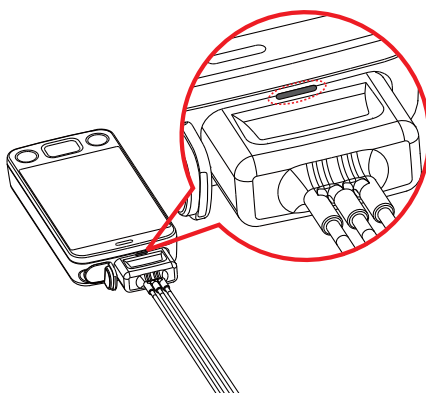



## WARNING

---

- **Insert the ECG lead set into the ECG connector. The following performance may be affected by a weak connection:**
    - ◆ ECG signal quality
    - ◆ Wireless signal strength
    - ◆ Water resistance
  - **Do not use the ECG leadwire to move or lift the TM80/TM70. This may cause the device to fall, which may damage the equipment or injure the patient.**
- 
- 

## NOTE

---

- **Insert the SpO<sub>2</sub> cap in the SpO<sub>2</sub> connector when SpO<sub>2</sub> is not in use.**
- 

## 3.4 Installing the Batteries

You can use three AA batteries or a lithium-ion rechargeable battery pack to run the TM80/TM70.

The runtime is dependent on the battery solution your chose. A lithium-ion battery pack will provide the longest runtime. For details about the recommended AA batteries, refer to *"Miscellaneous" on page 18 - 6*.

## NOTE

---

- **Always keep the battery compartment dry.**
  - **Never use brute force to install the lithium-ion battery pack or AA battery frame. Otherwise the waterproof ring surrounding the battery frame edge may be broken to affect the waterproof performance.**
-

---

### 3.4.1 Installing the Lithium-ion Rechargeable Battery

---




#### WARNING

- **Only use specified lithium-ion rechargeable batteries. Use of other lithium-ion batteries will adversely affect the batteries:**
    - ◆ **Level reporting**
    - ◆ **Low battery alarms**
    - ◆ **Life performance**
- 

#### NOTE


- **The lithium-ion rechargeable battery should be fully charged prior to first use.**
- 

1. Make sure the battery compartment is empty.
2. Align the hook on the upper part of the lithium-ion battery pack with the slot on the battery compartment, as indicated by the enlarged figure below.

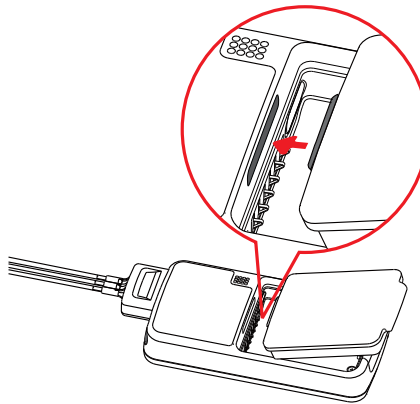

3. Press down the battery pack until it is installed firmly, as indicated by the arrow in the following figure.



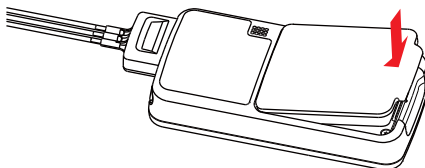


The TM80/TM70 is automatically powered on after the battery is installed.

### 3.4.2 Installing the AA Batteries


To install the AA batteries:

1. Make sure the battery compartment is empty.
2. Insert three 1.5V alkaline AA batteries according to the diagram in the bottom of the battery frame as shown in the images below.

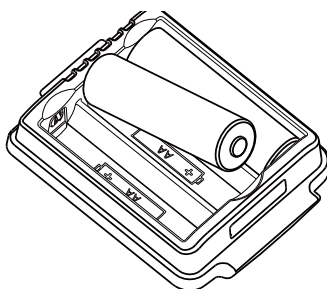


Installing three AA batteries

3. Align the hook on the upper part of the battery frame with the slot on the battery compartment, as indicated by the enlarged part in the following figure.

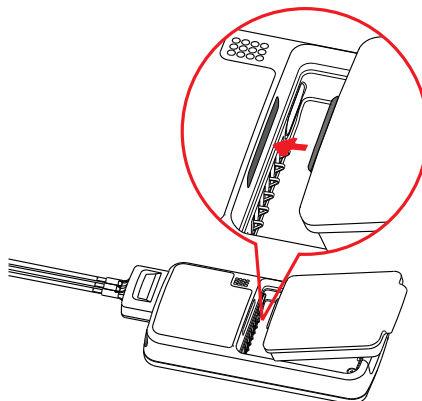


4. Press down the battery frame until it closes firmly, as indicated by the arrow in the following figure.

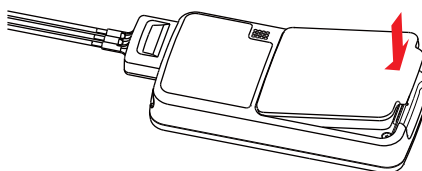

The TM80/TM70 is automatically powered on after the batteries are installed.

## 3.5 Powering On the Unit

Press the  key to turn on the TM80/TM70. The cyan alarm light will momentarily turn on to indicate that the device is starting. The TM80/TM70 performs a self-test during startup. The alarm light flashes red, yellow, and cyan in turn, and then off. This indicates that the alarm system functions correctly.

Upon powering up, there are two situations:

- If the TM80/TM70 is turned on at first time, the device will request you to configure first time startup.
- If the TM80/TM70 is turned on next time, the device will prompt whether it is a new patient. Select **[Yes]** or **[No]** as desired. If the device is a lock mode, a passcode is required.







**WARNING**

- **Check that visual and auditory alarm signals are presented correctly when the equipment is powered on. Do not use the equipment for any monitoring procedure on a patient if you suspect the equipment is not working properly or if the equipment is mechanically damaged. Contact your service personnel or Mindray.**

**3.6 Understanding Touch Gestures**

Before using the TM80/TM70, understand the supported touch screen gestures:

Gesture	Description
 Tap	Briefly touch the surface with your fingertip to select a target.
 Press and hold	Touch the surface for extended period of time.
 Drag	Move your fingertip over the surface without losing contact.
 Flick	Quickly brush the surface with your fingertip.

## 3.7 Basic Operations

This section describes the basic operations for the TM80/TM70.



### WARNING

- **Patients should be instructed not to interact with the display of the device and to not open the battery compartment while the TM80/TM70 is in use.**

### 3.7.1 Understanding the Screen Display Orientation

The TM80/TM70 supports both the portrait and landscape display orientations.



Example of portrait display







Example of landscape display

- Portrait: both digital and waveform tiles take up the entire width of the screen.
- Landscape: the digital tile takes up one half of the width of the screen; the waveform tile takes up the entire width of the screen.

### 3.7.2 Browsing the Screen Display

- To scroll the screen display, drag your finger up or down on the screen.
- To quickly move through the parameters, flick your finger up or down on the screen.

### 3.7.3 Switching the Screen Display Orientation

1. Flick your finger down from the top of the main screen to display the screen orientation switch buttons.
  - ◆ : switches portrait display to landscape display counterclockwise.
  - ◆ : switches portrait display to landscape display clockwise.
  - ◆ : switches landscape display horizontally.
  - ◆ : switches landscape display to portrait display.
2. Tap the desired button.

### 3.7.4 Displaying the Quick Keys Area


Flick your finger up from the bottom of the main screen to display the quick keys area.

The following table lists the six default quick keys:


Quick keys	Description
Discharge Patient	Tap the button to enter the <b>[Discharge Patient]</b> menu. Refer to <b>"Discharging the Patient" on page 5 - 4</b> for details.
Change Lead	Tap the button to change the current first ECG lead waveform to the next ECG lead waveform that is available in sequential order. For example, if the current first ECG lead waveform is I lead, tap the button, the I lead waveform is changed to II lead waveform.
Alarm Pause	Tap the button to pause the alarm system. Refer to <b>"Pausing Alarms" on page 6 - 8</b> for details.
Print	Tap the button to notify the Central Monitoring System (CMS) to start real-time print. The "Print Initiated" message displays on the screen.
Mark Event	Tap the button to notify the CMS to save the event to the event database. The "Event Marked" message displays on the screen.
Alarm Reset	Tap the button to reset the alarm system. Refer to <b>"Resetting Alarms" on page 6 - 9</b> for details.

You can customize the most frequently used functions to the quick keys. For details about setting the quick keys, refer to **"Quick Keys Menu" on page 13 - 6**.

### 3.7.5 Entering the Main Menu

Press the  key to enter the main menu.

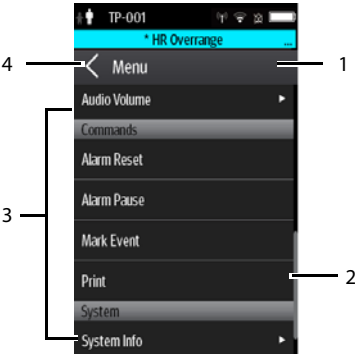


The main menu allows access to most of the system functions and settings.




Example of the Main Menu

All menus contain the following parts:

- 1. Heading: displays the current menu title.
- 2. Scroll bar: drag the bar to scroll the menu.
- 3. Main body: contains menus, buttons, and other controls to configure and operate the device.


Controls	Description
	Accesses a submenu to reveal more options or information.
	Indicates that a password is required for access.
Submenus	Contains more operations or information related to the corresponding menu.
Buttons	Provides an option to operate a function.
Switch 	Drag to right to enable the switch; drag to left to disable the switch.

- 4.  exits the current menu and return to the previous menu or the main screen, and saves any modified settings.

---

### 3.7.6 Turning the Display Off

You can manually turn the display off, or let the display automatically turn off based on the configured timeout.

Press the  key to manually turn the display off.

If the touch screen is not touched for the configured Display Auto Off time, then the screen will turn off after the configured Display Auto Off time.

For details about configuring the time for Display Auto Off, refer to **"Configuring the General Menu" on page 13 - 2**.

#### NOTE

---

- **While the display is off, the TM80/TM70 enters the power saving mode, and does not provide audio and visual alarms.**
- 

### 3.7.7 Turning the Display On

If the screen is off, press the  or  key to turn the display on.





#### CAUTION


---

- **Before use, please verify whether the packages are intact, especially the packages of single use accessories. In case of any damage, do not apply it to patients.**
- 




### 3.7.8 Unlocking the Screen in Locked Mode


If you set the screen lock, you need to input the correct passcode to unlock the screen after the display turns off.

1. If the screen is off, press the  or  key to turn the display on and access the **[Screen Locked]** menu.
2. Input the passcode to unlock the screen.

Once the passcode is entered the screen is temporarily unlocked. If the  is pressed or the device times out, the screen will lock again and a passcode must be entered.

### 3.7.9 Unlocking the Screen in View Only Mode

1. If the screen is off, press the  or  key to turn the display on.
2. Press the  key to display the **[Screen Locked]** menu.
3. Input the passcode to unlock the screen.

Once the passcode is entered the screen is temporarily unlocked. If the  is pressed or the device times out, the screen will lock again and a passcode must be entered.

For details about setting the screen lock, refer to **"Screen Lock Menu" on page 13 - 14.**

### 3.7.10 Acknowledging the Nurse Call

To acknowledge the triggered nurse call, tap **[Attendant Present]** in the main menu. The "Nurse Call Cancelled" message will display in the message area.

For details about how to trigger a nurse call, refer to **"Physical View" on page 2 - 6.**



#### WARNING

- **Do not only rely on the nurse call function, the medical personnel should also pay close attention to the patient's condition.**
- 

## 3.8 Using the Pouch


The TM80/TM70 is not intended for direct contact with the patient's skin. During normal use, the TM80/TM70 should be worn over clothing, in a pocket, or in a pouch. The water-proof pouch with clear front is an appropriate means for holding the TM80/TM70. Both disposable and reusable pouches specified in this manual can be used for the TM80/TM70. For details about the pouch, refer to **"Miscellaneous" on page 18 - 6.**

### 3.8.1 Securing the Pouch

To secure the pouch:

1. Place the TM80/TM70 into the pouch with the ECG leadwires and the SpO<sub>2</sub> sensor cable, if used, exiting from the pouch opening, as shown in the following figures.



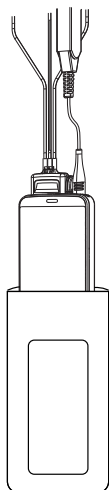


For disposable pouch




For reusable pouch

2. Pinch the snap-fastener to close the pouch.
3. Secure the pouch on the patient with ties around the patient's shoulder and under the arm, as shown in the following figure.

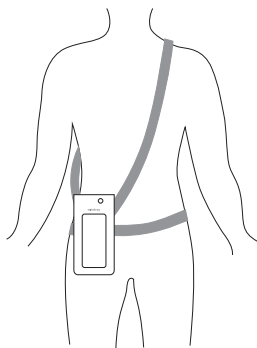

Wearing the disposable pouch

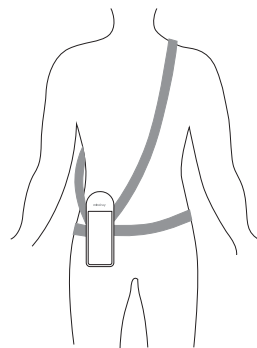

Wearing the reusable pouch

For more information on using the pouch, refer to ***Pouch Instructions for Use (P/N 046-007058-00)***.



## **WARNING**

- 
- **While using a pouch with the TM80/TM70 on the patient, consider the patient's condition. Be careful about the placement of the straps, as the straps could present a strangulation hazard.**
- 
- 

## **NOTE**

- 
- **The pouch is used only for the TM80/TM70. The pouch cannot be used for carrying other personal devices, such as a mobile phone.**
-

---

# 4

## User Configurations

---

Introduction.....	4-2
Configuring the Display.....	4-2
Configuring the Audio Volume.....	4-4

## 4.1 Introduction


This chapter describes the configurations available for users to do, such as configuring the Display Setup, and Audio Volume.

## 4.2 Configuring the Display

You can configure the display layout, display orientation, and screen brightness in the **[Display Setup]** menu.

### 4.2.1 Entering the Display Setup Menu

To enter the **[Display Setup]** menu, follow this procedure:

1. Press  to enter the main menu.
2. Tap **[Display Setup]**.

### 4.2.2 Configuring the Default Display Orientation

For details about the display orientation, refer to *"Understanding the Screen Display Orientation" on page 3 - 10*.

To configure the default display orientation, follow this procedure:



1. In the **[Setup]** section of the **[Display Setup]** menu, tap **[Default Orientation]**.  
Two buttons display: **[Portrait]** and **[Landscape]**.
2. Tap a button to set the default orientation.  
The selected orientation displays to the right of **[Default Orientation]**.
3. Restart the equipment to apply the setting.

### 4.2.3 Understanding Portrait Orientation Display Rules

In portrait orientation, both digital and waveform areas take up the entire width of the screen. Therefore, these parameters will be displayed in the exact order of the **[Display Setup]** menu, provided the sensor is attached and monitoring data.

### 4.2.4 Configuring the Portrait Display

To configure the portrait display, follow this procedure:

1. In the **[Portrait]** section of the **[Display Setup]** menu, tap **[Rows]**.  
Three options display: **[2]**, **[3]**, and **[4]**.
2. Tap an option to set the row numbers.  
The selected option displays to the right of **[Rows]**.
3. Tap **[Portrait Order]** to enter the **[Portrait Order]** menu.
4. Tap a parameter or waveform to select it.  
The  icon displays to the right of the selected parameter or waveform.
5. Drag the selected parameter or waveform to the desired position, and then release it.
6. Repeat steps 4 and 5 until the desired order is configured.
7. Tap the  icon to exit the **[Portrait Order]** menu.

#### 4.2.5 Understanding Landscape Orientation Display Rules

In landscape orientation, waveform areas take up the entire width of the screen. Digital areas only take up one half of the width of the screen. The following rules define how the tiles will be laid out:


1. The areas shall be displayed in the order of the **[Display Setup]** menu except the digital area locations shall be optimized to reduce blank tiles.
2. A waveform area always takes up the entire width of the screen.
3. A digital area always takes up one half of the width of the screen. Therefore, a row with a digital tile in it shall be split into two half tiles.
4. A digital area shall not be the only parameter in a row unless an odd number of digital areas exist. In this case, the last digital parameter area shall have one tile on the left side and the right half will be blank.
5. Digital areas shall be paired with the next available digital area to satisfy rule 4. This means that a digital area may be moved ahead of a waveform area if a half of a row needs to be filled.


#### 4.2.6 Configuring the Landscape Display

To configure the landscape display, follow this procedure:

1. In the **[Landscape]** section of the **[Display Setup]** menu, tap **[Rows]**.  
Three options display: **[2]**, **[3]**, and **[4]**.
2. Tap an option to set the row numbers.  
The selected option displays to the right of **[Rows]**.
3. Tap **[Landscape Order]** to enter the **[Landscape Order]** menu.


4. Tap a parameter or waveform option to select it.

The  icon displays to the right side of the selected parameter or waveform.

5. Drag the selected parameter or waveform to the desired position, and then release it.
6. Repeat steps 4 and 5 until the desired order is configured.
7. Tap the  icon to exit the **[Landscape Order]** menu.

### 4.2.7 Configuring the Display Brightness


To configure the display brightness, follow this procedure:

1. In the **[Setup]** section of the **[Display Setup]** menu, tap **[Display Brightness]**.  
The **[Display Brightness]** menu displays.
2. Drag the slider to left or right to adjust the brightness.
3. Tap the  icon to exit the **[Display Brightness]** menu.


## 4.3 Configuring the Audio Volume

You can independently set the alarm volume, touch screen click, and systole beep volume. The method for setting the three volumes are the same.

To change the volume settings, follow this procedure:

1. In the main menu, tap **[Audio Volume]**.
2. In the **[Sounds]**, **[Touch Screen Click]**, or **[Systole Beep]** section, drag the slider to the left or right to adjust the volume.
3. Tap the  icon to exit the **[Audio Volume]** menu.

### NOTE

- The  icon indicates that the audio volume is turned off.
- The minimum value for the alarm volume depends on the minimum alarm volume, refer to *"Configuring the Alarms Menu" on page 13 - 4* for details.

---

# 5

## Patient Management


---

Introduction.....	5-2
Admitting a Patient .....	5-2
Changing Patient Information .....	5-2
Changing Paced Status.....	5-3
Exiting the Standby Mode .....	5-4
Discharging the Patient .....	5-4

## 5.1 Introduction

The chapter describes how to admit a patient, change the patient category, enter and exit the Standby mode, and discharge the patient.

## 5.2 Admitting a Patient

After discharging the current patient, you can admit a new patient by pressing the  button. For how to discharge a patient, refer to "*Discharging the Patient*" on page 5 - 4. After admitting a patient, please check the patient category and make changes appropriately.

## 5.3 Changing Patient Information


You can change patient category, paced status, department name, room number, and bed number.

### 5.3.1 Changing the Patient Category

#### NOTE

---

- **Match the patient category selection to the actual patient before monitoring begins.**
- 

1. In the main menu, tap **[Patient Info]**.
2. In the **[Patient Info]** menu, tap **[Patient Category]** to select the desired patient category.  
The screen displays the "Are you sure you want to change the patient category?" message.
3. Select **[Yes]** to confirm the changing.  
The selected patient category displays to the right of **[Patient Category]**.
4. Tap  to save the settings and exit the **[Patient Info]** menu.

#### NOTE

---

- **When the device is connected to the CMS, the patient category at the CMS is updated if the patient category is changed at the TM80/TM70, and vice versa. Refer to the *BeneVision Central Monitoring System Operator's Manual (P/N 046-007960-00)* for details.**
-



### 5.3.2 Changing Paced Status

It is important to correctly set the patient's paced status before patient monitoring.

You can also set the patient's paced status from the [ECG] menu. For information regarding how to set paced status, refer to **"Checking the Paced Status" on page 7 - 11.**

### 5.3.3 Changing Department Name, Room Number, and Bed Number

When modifying department name, room number, and bed number is allowed in the [Maintenance] menu, you can change a department name, room number, and bed number.

For more information on related settings in the [Maintenance] menu, refer to **"Configuring Device Location" on page 13 - 4.**

To change department name, room number, and bed number:

1. In the main menu, tap [Patient Info].
2. In the [Patient Info] menu, tap [Department], [Room No.], or [Bed No.].
3. Enter the desired contents.

## 5.4 Placing a Device in Standby

### NOTE

---

- **When the device is connected to the CMS and enters or exits Standby mode, the CMS is also notified to enter or exit Standby mode. Refer to the *BeneVision Central Monitoring System Operator's Manual (P/N 046-007960-00)* for details.**
- 


To enter the Standby mode:

1. In the main menu, tap [Standby].
2. In the [Standby] confirmation menu, tap [Yes].

Placing a device into Standby mode does the following:

- Suspends patient monitoring
- Alarm system is suspended
- Removes all patient data from the screen and displays [Standby] on the screen.
- The screen display automatically turns off after the device enters the Standby mode for 30 seconds.
- CMS is notified.

## 5.5 Exiting the Standby Mode

Press the  key to exit Standby mode.

After you exit the Standby mode, the system responds as follows:

- Restores parameter measurement as previously configured.
- Alarm system is activated.
- The TM80/TM70 notifies the CMS of returning to the Monitoring mode.

## 5.6 Discharging the Patient

When the TM80/TM70 is connected to the CMS, discharging the patient will stop monitoring at the TM80/TM70 and CMS.


After discharging the patient, the patient's configuration is cleared at the TM80/TM70. The TM80/TM70 can admit next new patient by applying the user configuration. If the TM80/TM70 does not save the user configuration, it will load the departmental settings sent by the CMS for the new patient.

You can discharge the patient by selecting the **[Discharge Patient]** menu or restarting the TM80/TM70.


### NOTE

- **When the TM80/TM70 is connected to a CMS, the patient being discharged from the TM80/TM70 is also discharged from the CMS. Refer to the *BeneVision Central Monitoring System Operator's Manual (P/N 046-007960-00)* for details.**

### 5.6.1 Selecting the [Discharge Patient] menu

1. In the main menu, tap **[Discharge Patient]**.
2. In the **[Discharge Patient]** confirmation menu, select **[Yes]**.
  - ◆ The patient is discharged from both the TM80/TM70 and the CMS.
  - ◆ The patient's configuration is cleared and the default departmental settings are restored.
  - ◆ The **[Discharged Pat.]** window of the CMS adds a record of the patient's history data.
3. Press the  key to start monitoring by applying the default departmental settings.

## 5.6.2 Restarting the TM80/TM70

1. If the TM80/TM70 is powered off, press the  key to turn on the device.  
The device prompts whether this is a new patient or not.
2. Select **[Yes]** to enter the **[Discharge Patient]** confirmation menu.
3. Select **[Yes]** to clear the current patient's configuration and start monitoring by applying the user configuration.

If the TM80/TM70 does not save the user configuration, it will apply the default departmental settings sent by the CMS for the new patient.

**This page intentionally left blank.**

---

# 6

## Alarms

---

Introduction.....	6-2
Alarm Categories.....	6-3
Alarm Levels.....	6-3
Alarm Indicators .....	6-4
Viewing Alarm List.....	6-6
Resetting Alarms.....	6-9

## 6.1 Introduction

Alarms, triggered by abnormal vital signs or technical problems, are visually and audibly indicated to the user when the display is on.

For the full list of alarm messages displayed at the TM80/TM70, refer to **"Troubleshooting" on page 15 - 1**.

## 6.2 Alarm Safety Information



### WARNING

- **A potential hazard can exist if different alarm presets and default configuration settings are used for the same or similar equipment in the same care area.**
  - **If the TM80/TM70 is connected to the CMS, alarms can be presented and controlled at the CMS. Remote suspension or reset of alarms at the CMS may cause a potential hazard. For more information, refer to *BeneVision Central Monitoring System Operator's Manual*.**
  - **The TM80s/TM70s in your care area may each have different alarm settings to suit different patients. Always check that the alarm settings are appropriate for your patient before starting monitoring. Always make sure that necessary alarm limits are active and set according to the patient's clinical condition.**
  - **Setting alarm limits to extreme values may cause the alarm system to become ineffective.**
  - **When the alarm sound is switched off, the TM80/TM70 gives no alarm tones even if a new alarm occurs. Be careful about whether to switch off the alarm sound or not. When the alarms are off or while alarm audio is paused either temporarily or indefinitely, observe the patient frequently.**
  - **When monitoring patients that are not continuously attended by a clinical operator, properly configure the alarm system and adjust alarm settings as per the patient's condition.**
  - **Do not rely exclusively on the audible alarm system for monitoring. Adjustment of alarm volume to a low level may result in a hazard to the patient.**
-



**WARNING**

- **Always make sure that the audio alarm volume level is adequate in your care environment. Always keep the patient under close surveillance.**
- **The reception failure of alarm signals may occur in the distributed alarm system.**

### 6.3 Alarm Categories

The alarms can be classified into two categories: physiological alarms and technical alarms. Alarm messages are displayed in the message area on the main screen.

- Physiological alarms, also called patient status alarms, are triggered by a monitored parameter value that violates set alarm limits or an abnormal patient condition.
- Technical alarms, also called system status alarms, are triggered by a device malfunction or a patient data distortion due to improper operation or mechanical problems.

Apart from the physiological and technical alarm messages, the TM80/TM70 will show some messages telling the system status.

### 6.4 Alarm Levels

The alarms can be classified into three severity levels: high level, medium level and low level.

Alarm Levels	Physiological Alarms	Technical Alarms
High level	Indicates that the patient is in a life threatening situation, such as Asystole V-Fib/ V-Tac and so forth, and an emergency treatment is required.	Indicates a severe device malfunction or an improper operation, which could make it possible that the monitor cannot detect critical patient status and thus threaten the patient's life, such as low battery.
Medium level	Indicates that the patient's vital signs appear abnormal and immediate treatment is required.	Indicates a device malfunction or an improper operation, which may not threaten the patient's life but may compromise the monitoring of vital physiological parameters.
Low level	Indicates that the patient's vital signs appear abnormal and immediate treatment may be required.	Indicates a device malfunction or an improper operation, which may compromise a certain monitoring function but will not threaten the patient's life.

Alarm Levels	Physiological Alarms	Technical Alarms
Message	Provides additional information on the patient status.	Provides additional information on the system status.

## 6.5 Alarm Indicators

When an alarm occurs, the TM80/TM70 notifies the user through visual or audible alarm indications.

- Alarm light
- Audible alarm tones
- Flashing numerics
- Alarm message

### NOTE

- **When the display is off, the user must activate the screen to view any local alarms.**

### 6.5.1 Alarm Light

If an alarm occurs, the alarm light on the TM80/TM70 flashes. The color and flashing frequency correspond to the alarm level as follows:

- High level alarms: the lamp quickly flashes red.
- Medium level alarms: the lamp slowly flashes yellow.
- Low level alarms: the lamp lights cyan without flashing.

### 6.5.2 Alarm Tones

The TM80/TM70 has three alarm tone configurations: ISO, Mode 1 and Mode 2. For each configuration, the alarm tones enunciate the alarm levels as follows:

- ISO pattern:
  - ◆ High level alarms: triple+double+triple+double beep
  - ◆ Medium level alarms: triple beep
  - ◆ Low level alarms: single beep
- Mode 1:



- ◆ High level alarms: high-pitched single beep
- ◆ Medium level alarms: double beep
- ◆ Low level alarms: low-pitched single beep
- Mode 2:
  - ◆ High level alarms: high-pitched triple beep
  - ◆ Medium level alarms: double beep
  - ◆ Low level alarms: low-pitched single beep

NOTE

- **When multiple alarms of different levels occur simultaneously, the TM80/TM70 selects the alarm of the highest level to light the alarm light and sound alarms accordingly, while all the alarm messages scroll in the message area on the top of the screen.**
- **Some physiological alarms, such as Asystole, are exclusive. They have identical alarm tones and alarm lights with normal high level physiological alarms, but their alarm messages are displayed exclusively. That is, when an exclusive physiological alarm and a normal high level physiological alarm are triggered simultaneously, only exclusive physiological alarm message will be displayed.**

6.5.3 Flashing Numeric

If an alarm triggered by an alarm limit violation occurs, the numeric of the measurement in alarm will flash every second, and the corresponding alarm limit will also flash at the same frequency indicating the high or low alarm limit is violated.

6.5.4 Alarm Messages




When an alarm occurs on the screen, the alarm message appears in the message area. The background color of the alarm message and the asterisk symbols (\*) before the alarm message are designed to indicate the alarm level.

Alarms	Background color	Asterisk symbols (*)
High level alarms	red	***
Medium level alarms	yellow	**
Low level alarms	cyan	*

You can view the alarm messages by selecting the message area. Refer to ***"Display Screen" on page 2 - 8*** for details.

## 6.5.5 Alarm Status Symbols

The TM80/TM70 still uses the following symbols indicating the alarm status:

-  : indicates that all the alarms are paused.
-  : indicates the technical alarm audio is turned off.
-  : indicates the alarm system is reset.

## 6.6 Viewing Alarm List

1. Tap the message area on the main screen. The **[Alarm List]** menu is displayed.
2. In the **[Physiological Alarms]** section, you can view the number of physiological alarms and alarm messages.
3. In the **[Technical Alarms]** section, you can view the number of technical alarms and alarm messages.



## 6.7 Configuring the Alarms

You can set alarm-related items such as alarm properties, alarm volume, and alarm setup menu. For how to set alarm volume and alarm setup menus, refer to **"Configuring the Audio Volume" on page 4 - 4** and **"Configuring the Alarms Menu" on page 13 - 4** respectively.

### 6.7.1 Changing Alarm Properties

You can change alarm properties for parameters collectively or individually.

To change alarm properties, follow this procedure:

1. Enter the **[Alarm Limit]** menu in either of the following ways:
  - ◆ Press the  key to enter the main menu → tap **[Alarms]** → tap **[Alarm Limits]**.
  - ◆ Tap the desired digital area or waveform area to enter corresponding parameter setup menu → from the **[Alarms]** section tap alarm-related option.
2. In the **[Alarm Limits]** section, switch on or off a parameter alarm.
3. Tap the desired parameter to enter its alarm setup menu.
4. In the **[Limits]** section, tap the alarm limit which you want to change.
5. Enter the new alarm limit.
6. Tap  to return to the parameter alarm setup menu.
7. In the **[Responses]** section, set the desired alarm priority.


## NOTE

- **When the TM80/TM70 is connected to the CMS, any changes to the alarm properties at the TM80/TM70 or the CMS will be communicated to the other side to make sure that the alarm settings are consistent.**

### 6.7.2 Initiating Auto Alarm Limits

The TM80/TM70 provides the auto alarm limits function to automatically adjust alarm limits according to the patient's vital signs used. When auto limits are selected, the TM80/TM70 calculates safe auto limits based on the latest measured values. To get accurate auto alarm limits, you need to collect a set of measured vital signs as a baseline.

To initiate auto alarm limits, follow this procedure:

1. Press the  key to enter the main menu.
2. Tap **[Alarms]**.
3. Tap **[Alarm Limits]**. The **[Alarm Limits]** menu is displayed.
4. Tap **[Auto Limits]**.
5. In the dialog that pops up, tap **[OK]**.


Then the TM80/TM70 will automatically calculate alarm limits based on the latest measured values. Before applying these automatically created alarm limits, confirm if they are appropriate for your patient from the Limits menu. If not, you can adjust them manually. These alarm limits will remain unchanged until you select auto limits again or adjust them manually.

The TM80/TM70 calculates auto limits basing on the following rules:

Module	Parameter	Lower Limit	Upper Limit	Auto Limit Range
		Adult/ Pediatric	Adult/ Pediatric	
ECG	HR/PR (bpm)	(HR/PR×0.8) or 40 (whichever is greater)	(HR/PR×1.25) or 240 (whichever is smaller)	Adult/pediatric: 35~240
Resp	RR (rpm)	(RR×0.5) or 6 (whichever is greater)	(RR×1.5) or 30(whichever is smaller)	Adult/pediatric: 6~55
SpO <sub>2</sub>	SpO <sub>2</sub> (%)	Same as the default alarm limit	Same as the default alarm limit	Same as the measurement range


Module	Parameter	Lower Limit	Upper Limit	Auto Limit Range
		Adult/ Pediatric	Adult/ Pediatric	
NIBP	NIBP-S (mmHg)	(SYS×0.68 + 10)	SYS×0.86 + 38	Adult: 45~270 Pediatric: 45~185
	NIBP-D (mmHg)	(Dia×0.68 + 6)	(Dia×0.86 + 32)	Adult: 25~225 Pediatric: 25~150
	NIBP-M (mmHg)	(Mean×0.68 + 8)	Mean×0.86 + 35	Adult: 30~245 Pediatric: 30~165

6.7.3 Restoring the Default Alarm Settings

- 1. Press the  key to enter the main menu.
- 2. Tap **[Alarms]**.
- 3. Tap **[Alarm Limits]**. The **[Alarm Limits]** menu is displayed.
- 4. Tap **[Defaults]**.
- 5. In the dialog that pops up, tap **[OK]**.

6.8 Pausing Alarms


When an alarm occurs, follow this procedure to pause the alarm system.

- Press the  key to enter the main menu, and then tap **[Alarm Pause]** from the **[Commands]** section.

OR

- 1. Flick your finger up at the bottom of the main screen to display the quick keys area.
- 2. Tap the **[Alarm Pause]** quick key.

When alarms are paused, the following indicators are given:

- No physiological alarm will be presented within the alarm pause time.
- For technical alarms, alarm sounds are paused, but the alarm lamp and alarm messages remain presented.
- The remaining alarm pause time is displayed in the message area.
- The alarm pause symbol  is displayed in the upper right corner of the main screen.