

OMRON 5 series Blood Pressure Monitor Model BP742N Instruction Manual

Introduction

Thank you for purchasing the OMRON® BP742N Blood Pressure Monitor. Your new blood pressure monitor uses the oscillometric method of blood pressure measurement. This means the monitor detects your blood movement through your blood artery and converts the movement into a digital reading. An oscillometric monitor does not need a stethoscope so the monitor is simple to use.

Intended Use
This device is a digital monitor intended for use in measuring blood pressure and pulse rate in adult patients who can understand this instruction manual with the arm circumference range printed on the arm cuff. The device detects the appearance of irregular heartbeats during measurement and gives a warning signal with the measurement start.

Please read this instruction manual thoroughly before using the device. Please keep for future reference. For specific information about various blood pressure, CONSULT YOUR PHYSICIAN.

Important Safety Information

Warning: Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

General Usage
Do NOT adjust medication based on measurement results from this blood pressure monitor. Take medication as prescribed by your physician. Only a physician is qualified to diagnose and treat High Blood Pressure.

This device is not intended to be a diagnostic device.

Consult your physician before using the device if pregnancy including pre-eclampsia, or if diagnosed with arrhythmia or atherosclerosis.

Do not use the device on the injured arm or the arm under medical treatment.

Do not apply the arm cuff to the arm while being on an intravenous drip or blood transfusion.

Consult your physician before using the device on the arm with an arterio-venous (A-V) shunt.

Do not use the device with other medical electrical (ME) equipment simultaneously.

Do not use the AC adapter if the device or the power cord is damaged. Turn off the power and unplug the power cord immediately.

Plug the AC adapter into the appropriate voltage outlet. Do not use in a multi-outlet plug.

A new plug is required if taping the power cord from the electric outlet with wet hands.

Caution: Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

General Usage
Always consult your physician. Self-diagnosis of measurement results and self-treatment are dangerous.

People with severe blood flow problems, or blood disorders, should consult a physician before using the device, as the arm cuff inflation can cause bruising.

Remove the arm cuff if it does not start deflating during the measurement.

Do not use the device on limbs or persons who can not express the limb numbness.

Do not use the device for any purpose other than measuring blood pressure.

Use only the approved arm cuff for this device. Use of other arm cuffs may result in incorrect measurement results.

Do not use a mobile phone or other devices that emit electromagnetic fields near the device. This may result in incorrect operation of the device.

Do not disassemble the monitor or arm cuff. This may cause an inaccurate reading.

Do not use in a location with moisture, or a location where water may splash on the device. This may damage the device.

Do not use the device in a moving vehicle (car, airplane).

Do not take measurements more than necessary. It may cause bruising due to blood flow interference.

Consult your physician before using the device if you had a mastectomy.

Read "If your systolic pressure is more than 210 mmHg" of this instruction manual. If your systolic pressure is known to be more than 210 mmHg, in addition to a higher pressure than necessary may result in bruising when the arm cuff is applied.

(AC Adapter (optional) Usage)
Fully insert the power plug into the outlet.

When disconnecting the power plug from the outlet, do not pull the power cord. Be sure to pull from the power plug safely.

When handling the power cord, take care not to do the following:
Do not damage. Do not bend it.
Do not tamper with it. Do not touch the end or plug.
Do not twist. Do not bend during use.
Do not pinch. Do not place under heavy objects.

Unplug the device from the power plug.

Use only an OMRON AC adapter designed for this device. Use of unsupported adapters may damage and/or may be hazardous to the device.

(Battery Usage)
Do not use if the batteries with their polarities are incorrectly aligned.

Use only 4 "AA" alkaline or manganese batteries with the device. Do not use other types of batteries. Do not use new and used batteries together.

Remove the batteries if the device will not be used for three months or more.

General Precautions

Do not forcibly cease the arm cuff or the air tube excessively.

Do not press the air tube while taking a measurement.

To replace the air plug, pull the air plug at the connector with the monitor, not the tube itself.

Do not drop the monitor or subject of device to strong shocks or vibrations.

Do not inflate the arm cuff when it is not wrapped around your arm.




Do not use the device outside the specified environment. It may cause an inaccurate reading.

Dispose of the device, components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.

1. Know Your Device

Contents: Monitor, arm cuff, battery set, instruction manual, quick start guide

Monitor:

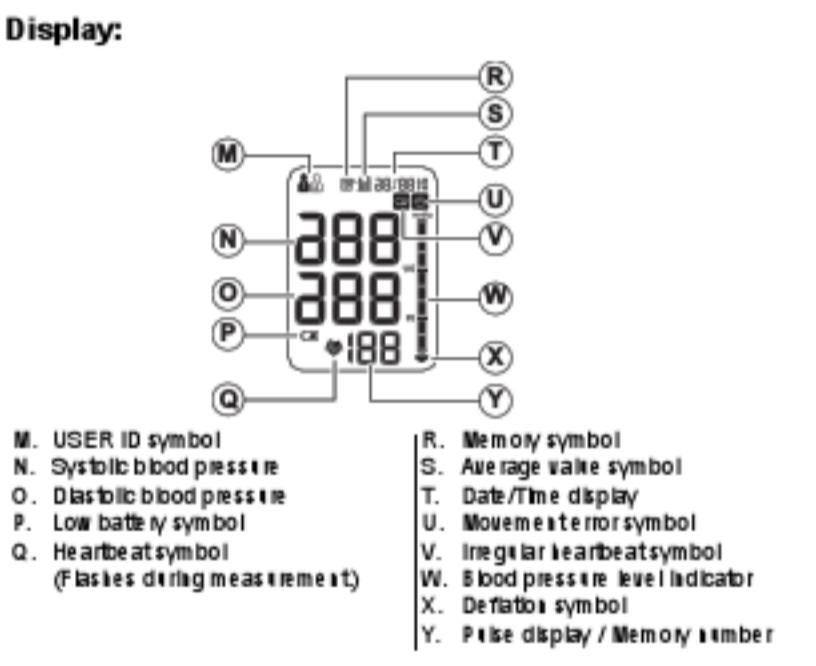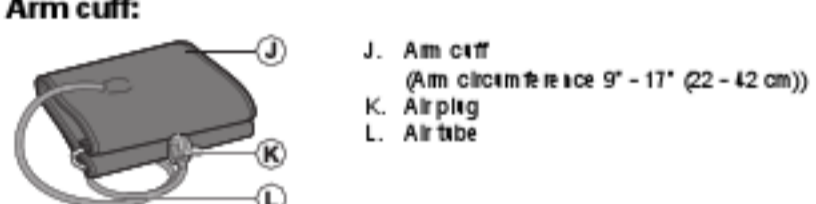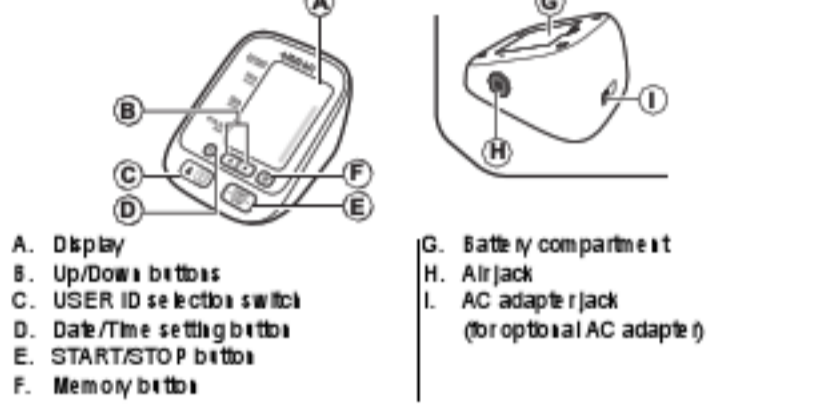


1.1 Display symbols

Irregular Heartbeat Symbol (H)
When the monitor detects an irregular rhythm two or more times during the measurement, the irregular heartbeat symbol (H) will appear on the display with the measurement start. An irregular heartbeat rhythm is defined as a rhythm that is 25% less or 25% more than the average rhythm detected while the monitor is measuring the systolic and diastolic blood pressure. If the irregular heartbeat symbol (H) displays with your measurement results, we recommend you consult your physician. Follow the directions of your physician.

Movement Error Symbol (M)
The movement error symbol is displayed if you move your body during the measurement. Please remove the arm cuff, and wait 2-3 minutes. Take another measurement, remain still during measurement.

Average Value Symbol (AV)
The average value symbol is displayed when you press and hold the memory button for more than 3 seconds. The most recent average value appears on the display screen.

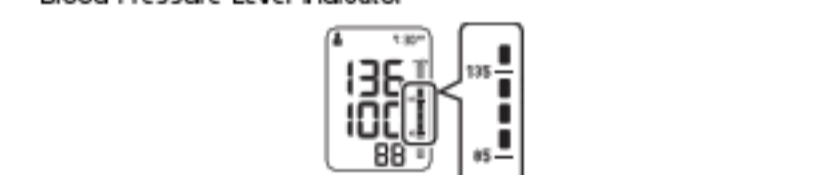

2018 ESH/ESC Guidelines for the management of arterial hypertension. Definition of type 1 and 2 hypertension and some blood pressure levels. These are from statistical values for blood pressure.

	Office	Home
Systolic Blood Pressure	≥ 140 mmHg	≥ 135 mmHg
Diastolic Blood Pressure	≥ 90 mmHg	≥ 85 mmHg

1.2 Before Taking a Measurement


To be precise an accurate reading, follow these directions:

- Avoid bathing, drinking alcohol or caffeine, smoking, exercising and eating for 30 minutes before taking a measurement. Rest for at least 15 minutes before taking the measurement.
- Stress raises blood pressure. Avoid taking measurements during stressful times.
- Measurements should be taken in a quiet place.
- Remove tight-fitting clothing from your arm.
- Sit on a chair with your feet flat on the floor. Rest your arm on a table so that the arm cuff is at the same level as your heart.
- Remain still and do not talk during the measurement.
- Keep a record of your blood pressure and pulse readings for your physician. A single measurement does not provide an accurate indication of your true blood pressure. You need to take and record several readings over a period of time. Try to measure your blood pressure at the same time each day for consistency.

2. Preparation

2.1 Battery Installation

- Remove the battery cover.

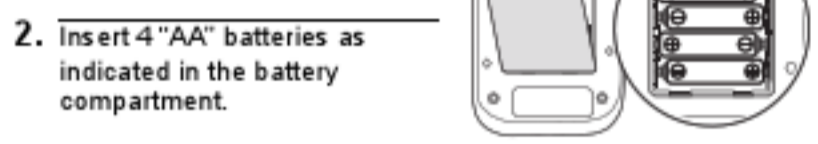

- Insert 4 "AA" batteries as indicated in the battery compartment.


- Replace the battery cover.
Notes:
 - When the low battery symbol (L) appears on the display, turn the monitor off, then replace all batteries at the same time. Long life alkaline batteries are recommended.
 - The measurement values continue to be stored in memory even after the batteries are replaced.
 - The supplied batteries may have a shorter life.

Dispose of the device, components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.

2.2 Setting the Date and Time

Set the monitor to the correct date and the before taking a measurement for the first time.

- Press the (M) button.
- Push (L) or (R) to change the year.

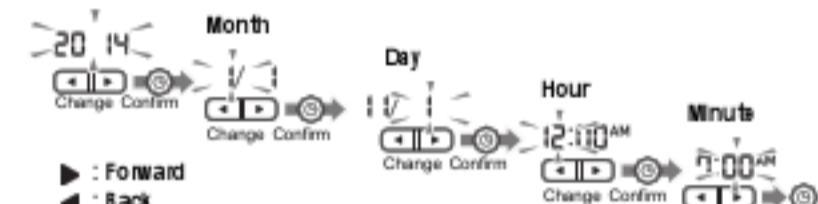

- Press the START/STOP button to turn the monitor off.
Notes:
 - If the batteries have been replaced, the date and time setting will need to be reset.
 - If the date and time are not set, "--" appears during or after measurement.

3. Using the Device

3.1 Applying the Arm Cuff

Remove tight-fitting clothing or tight rolled up sleeve from your left upper arm. Do not place the arm cuff over thick clothes.

- Insert the air plug into the air jack securely.



- Wrap the arm cuff firmly in place around your left upper arm.



The bottom edge of the arm cuff should be 1/2 inch (1 to 2 cm) above the elbow. Air tube is on the back of your arm and aligned with your bicep ridge.

- Secure closed with the fabric fastener.



Notes:

- When you take a measurement on the right arm, the air tube will be at the side of your elbow. Be careful not to rest your arm on the air tube.
- The blood pressure can differ between the right arm and the left arm, and the measured blood pressure values can be different. OMRON recommends to always use the same arm for measurement. If the values between both arms differ substantially, please check with your physician which arm to use for your measurement.

3.2 How to Sit Correctly

To take a measurement, you need to be relaxed and comfortably seated, under comfortable room temperature. Avoid bathing, drinking alcohol or caffeine, smoking, exercising or eating 30 minutes before taking a measurement.

- Sit on a chair with your feet flat on the floor.
- Sit upright with your back straight.
- Stretch your back and arm being supported.
- The arm cuff should be placed on your arm at the same level as your heart.



3.3 Taking a Measurement


- To stop a measurement, press the START/STOP button once to deflate the arm cuff.
- Remain still while taking a measurement.

The monitor is designed to take measurements and store the measurement values in the memory for 2 people using USER ID 1 and USER ID 2.

- Select your User ID (1 or 2).



- Press the START/STOP button. The arm cuff will start to inflate automatically.

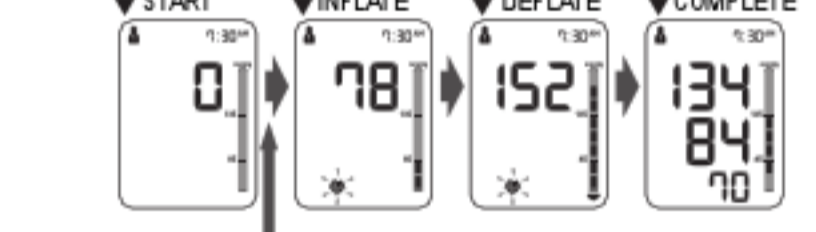

If your systolic pressure is more than 210 mmHg After the arm cuff starts to inflate, press and hold the START/STOP button until the monitor inflates 30 to 40 mmHg higher than your expected systolic pressure.

- Remove the arm cuff.


- Press the START/STOP button to turn the monitor off. The monitor automatically stores the measurement result in its memory. It will automatically turn off after 2 minutes.

Note: Wait 2-3 minutes before taking another measurement. Waiting between measurements allows the arteries to return to the condition prior to taking a measurement.

Using the Guest Mode

The monitor stores measurement values for 2 users in the memory. The guest mode can be used to take a single measurement for another user. No measurement values are stored in the memory when the guest mode is selected.

- Press and hold the START/STOP button for more than 3 seconds. The USER ID symbol and the Date/Time display will disappear.

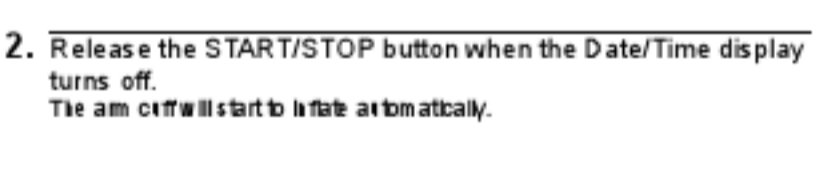

- Release the START/STOP button when the Date/Time display turns off. The arm cuff will start to inflate automatically.

Do NOT adjust medication based on measurement results from this blood pressure monitor. Take medication as prescribed by your physician. Only a physician is qualified to diagnose and treat High Blood Pressure.

This device is not intended to be a diagnostic device.

Always consult your physician. Self-diagnosis of measurement results and self-treatment are dangerous.

Inflation to a higher pressure than necessary may result in bruising when the arm cuff is applied.

3.4 Using the Memory Function

The monitor automatically stores the results up to 50 sets for each user (1 and 2). It can also calculate an average value based on the last 3 measurement values taken within 10 minutes.

- If there are only 2 measurement values in the memory for that period, the average value will be based on these 2 values.
- If there is 1 measurement value in the memory for that period, this is displayed as the average.
- If the memory is full, the monitor will delete the oldest value.
- When viewing the measurement value taken without setting the date and time, "--" is displayed instead of the date and time.

To View the Measurement Values Stored in Memory

- Select your User ID (1 or 2).
- Press the (M) button. The Memory symbol appears for a second before the pulse rate is displayed. The newest set is numbered "1".



- Press the (L) or (R) button to view the values stored in memory.
 - (L): To view the older values
 - (R): To view the more recent values

To View the Average Value

- Select your User ID (1 or 2).
- Press and hold the (M) button for more than 3 seconds.



- If the previous measurement was taken without setting the date and time, the average value is not calculated.
- If there are no measurement results stored in the memory, the screen to the right is displayed.

To Delete All the Values Stored in Memory

- Select your User ID (1 or 2).
- Press the (M) button, while the memory symbol appears.

- While holding the (M) button down, press the START/STOP button for more than 3 seconds.



Note: You can not partially delete the values stored in the memory. All values for the user you selected will be deleted.

4. Error Messages and Troubleshooting

4.1 Error Messages

Error Display	Cause	Solution
	Irregular heartbeats are detected.	Remove the arm cuff. Wait 2-3 minutes and take another measurement. Repeat the steps in section 3.3. If the error continues to appear, contact your physician.
	Movement during measurement.	Carefully read and repeat the steps in section 3.3.
	The batteries are low.	You should replace the batteries with new ones ahead of time. Refer to section 2.1.
	The batteries are exhausted.	You should replace the batteries with new ones ahead of time. Refer to section 2.1.
	Air plug disconnected.	Insert the plug securely. Refer to section 3.1.
E1	Arm cuff is applied too loosely.	Apply the arm cuff tightly. Refer to section 3.1.
	Air is leaking from the arm cuff.	Replace the arm cuff with a new one. Refer to section 3.3.
E2	Movement during measurement and the arm cuff has not been inflated sufficiently.	Repeat measurement. Remain still and do not talk during measurement. Refer to section 3.3.
	The arm cuff was inflated exceeding the maximum allowable pressure, and the cuff deflated automatically while inflating the arm cuff manually.	Do not touch the arm cuff or the air tube while taking a measurement. Do not inflate the arm cuff more than necessary. Refer to section 3.3.
E3	Movement during measurement.	Repeat measurement. Remain still and do not talk during measurement. Refer to section 3.3.
E4	Clothing is interfering with the arm cuff.	Remove any clothing interfering with the arm cuff. Refer to section 3.1.
E5	Device error.	Contact Customer Service.

4.2 Troubleshooting

Problem	Cause and Solution
No power. No display appears on the monitor.	Replace all batteries with new ones. Check the battery installation for proper placement of the battery polarities. Refer to section 2.1.
Measurement values appear too high or too low.	Blood pressure varies constantly. Many factors including stress, the day of the week, and how you wrap the cuff, may affect your blood pressure. Review the section 1.2 and section 3.3.

5. Maintenance and Storage

5.1 Maintenance

To protect your device from damage, please observe the following:


- Store the device and the components in a clean, safe location.
- Do not use any abrasive or volatile cleaners.
- Do not wash the device and any components or immerses them in water.
- Do not use gasoline, thinners or similar solvents to clean the device.



- Use a soft and dry cloth, or a soft and moistened cloth and neutral soap to clean on the monitor and the arm cuff.
- Changes or modification not approved by the manufacturer will void the user warranty. Do not disassemble or attempt to repair the device or components. Consult Customer Service.


5.2 Storage

1. Unplug the air plug from the air jack.
2. Gently fold the air tube into the arm cuff.

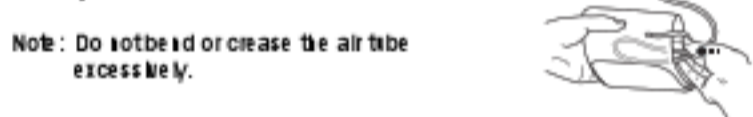

- Note: Do not store or use the air tube excessively.
- Do not store the device in the following situations:
 - If the device is wet.
 - Locations exposed to extreme temperatures, humidity, direct sunlight, dust or corrosive vapors such as bleach.
 - Locations exposed to vibrations, shocks or where it will be at an angle.

5.3 Optional Medical Accessories

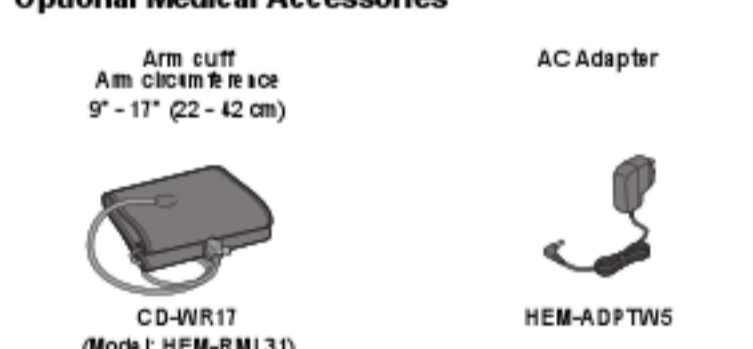

Using the Optional AC Adapter

1. Insert the AC adapter plug into the AC adapter jack on the rear side of the monitor.
2. Plug the AC adapter into an electrical outlet.



To disconnect the AC adapter, unplug the AC adapter from the electrical outlet first, and then remove the AC adapter plug from the monitor.

6. Specifications

Model	BP742N (HEM-7131-2)
Display	LC D digital display
Measurement range	Pressure: 0 to 250 mmHg Pulse: 40 to 180 beats/min.
Accuracy	Pressure: ±3 mmHg or 2% of reading Pulse: ±5% of display reading
Inflation	Fuzzy logic controlled by electric pump
Deflation	Automatic pressure release valve
Measurement method	Oscillometric method
IP classification	IP 20
Power source	4 "AA" batteries 1.5V or optional AC adapter (BP742N AC 100-240V S06002-D-12A)
Battery life	Approximately 1000 measurements (using new alkaline batteries)
Operating temperature / humidity	50°F to 104°F (10°C to 40°C) / 10 to 90% RH
Storage temperature / humidity / air pressure	-4°F to 140°F (-20°C to 60°C) / 10 to 95% RH / 700 to 1050 hPa
Weight	Monitor: Approximately 10.98 oz. (309g) not including batteries Arm cuff: Approximately 6 oz. (170g)
Dimensions	Monitor: Approximately 4 1/4" (W) x 3 1/8" (H) x 5 1/2" (D) (107mm x 76mm x 141mm) Arm cuff: Approximately 5 3/4" x 23 1/2" (air tube: 29 1/2") (145mm x 594mm (air tube: 750mm))
Cut-off dimension	5" to 11" (20 to 430 mm)
Memory	Up to 50 per user
Contents	Monitor, arm cuff, battery set, instruction manual, quick start guide

- Note:
- These specifications are subject to change without notice.
 - In the clinical validation study, the 5th phase was used on 85 subjects for one month of data for blood pressure.
 - This device has not been validated for use on pregnant patients.

7. FCC Statement

FCC CAUTION
Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note:
This equipment has been tested and found to comply with the limits for Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment uses radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

8. Limited Warranty

Your BP742N Automatic Blood Pressure Monitor, excluding the arm cuff and batteries, is warranted to be free from defects in materials and workmanship appearing within 2 years from the date of purchase, when used in accordance with the instructions provided with the monitor. The arm cuff is warranted to be free from defects in materials and workmanship appearing within 1 year from the date of purchase when the monitor is used in accordance with the instructions provided with the monitor. The above warranty extends only to the original retail purchaser. We will, at our option, replace or repair any monitor arm covered under the above warranty. Replacement is at our responsibility and you may remedy under the above warranty.

To obtain warranty service contact Customer Service by calling 1-800-634-4350 for the address of the respective center and the return shipping and handling fee. Enclose the original printed receipt, include a letter, with your name, address, phone number, and description of the specific problem. Pack the product carefully to prevent damage in transit. Because of possible loss in transit, we recommend insuring the product in transit receipt requested.

THE FOREGOING IS THE SOLE WARRANTY PROVIDED BY OMRON IN CONNECTION WITH THIS PRODUCT, AND OMRON HEREBY DISCLAIMS ANY OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IMPLIED WARRANTIES AND OTHER TERMS THAT MAY BE IMPOSED BY LAW, IF ANY, ARE LIMITED IN DURATION TO THE PERIOD OF THE ABOVE EXPRESS WARRANTY.

OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER SPECIAL, INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES.

This warranty provides you with specific legal rights, and you may have other rights that vary by jurisdiction. Because of special local requirements, some of the above limitations and exclusions may not apply to you.

FOR CUSTOMER SERVICE
Visit our website at: www.omronhealthcare.com
Call toll free: 1-800-634-4350

9. Guidance and Manufacturer's Declaration

OMRON Automatic Blood Pressure Monitor
In compliance with applicable regulatory requirements, the scope of IEC60601-1-2:2007, Model BP742N including AC-adapter.

Important information regarding Electro-Magnetic Compatibility (EMC)
With the increased number of electronic devices such as PCs and mobile (cellular) telephones, medical devices in use may be susceptible to electromagnetic interference from other devices. Electromagnetic interference may result in incorrect operation of the medical device and create potentially unsafe situations. Medical devices should also not interfere with other devices.
In order to regulate the requirements for EMC (Electro-Magnetic Compatibility) with the aim to prevent unsafe product situations, the IEC 60601-1-2 standard has been implemented. This standard defines the levels of immunity to electromagnetic interferences as well as maximum levels of electromagnetic emissions for medical devices.
Medical devices manufactured by OMRON Healthcare conform to this IEC 60601-1-2:2007 standard for both immunity and emissions.
Nevertheless, special precautions need to be observed:
• The use of accessories and cables other than those specified by OMRON II, with the exception of cables sold by OMRON as replacement parts for internal components, may result in increased emission or decreased immunity of the device.
• The medical devices should not be used adjacent to or stacked with other equipment.
• In case adjacent or stacked use is necessary, the medical device should be observed to prevent normal operation in the configuration in which it will be used.
• Refer to further guidance below regarding the EMC environment in which the device should be used.
• The MEDICAL ELECTRICAL EQUIPMENT BP742N including AC-adapter needs special precautions regarding EMC and needs to be installed and pulled into service according to the EMC information provided in this documentation.
• The Essential Performance of the BP742N including AC-adapter is to measure a blood pressure and pulse rate and using the memory function.
The BP742N including AC-adapter may be interfered with by other equipment, even if that other equipment complies with IEC 60601-1 requirements.

Immunity test	Compliance level	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The OMRON II BP742N including AC-adapter uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The OMRON II BP742N including AC-adapter is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions IEC 61000-3-2	Class A	
Voltage fluctuations Flicker emissions IEC 61000-3-3	Complies	

Guidance and manufacturer's declaration - also electromagnetic immunity

OMRON II BP742N including AC-adapter is intended for use in the electromagnetic environment specified below. The customer or the user of this OMRON II BP742N including AC-adapter should ensure that it is used in such environment.

Immunity to test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±4 kV air	±6 kV contact ±4 kV air	Floor should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transients (burst) IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV for power supply lines ±1 kV for input/output lines	Main power quality should be that of a typical commercial and/or hospital environment.
Surge IEC 61000-4-5	±1 kV (line-to-line), ±2 kV (line-to-earth)	±1 kV (line-to-line), ±2 kV (line-to-earth)	Main power quality should be that of a typical commercial and/or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply lines IEC 61000-4-11	5% U _n (0.95 U _n) for 0.5 cycle 40% U _n (0.60 U _n) for 5 cycles 70% U _n (0.30 U _n) for 25 cycles 5% U _n (0.95 U _n) for 5 sec.	5% U _n (0.95 U _n) for 0.5 cycle 40% U _n (0.60 U _n) for 5 cycles 70% U _n (0.30 U _n) for 25 cycles 5% U _n (0.95 U _n) for 5 sec.	Main power quality should be that of a typical commercial and/or hospital environment, if the user of the OMRON II BP742N including AC-adapter requires continued operation during power mains interruption, it is recommended that the OMRON II BP742N including AC-adapter be powered from an uninterrupted power supply.
Power frequency magnetic field IEC 61000-4-6	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Note: U_n is the AC mains voltage prior to application of the test level.

Guidance and manufacturer's declaration - also electromagnetic immunity

OMRON II BP742N including AC-adapter is intended for use in the electromagnetic environment specified below. The customer or the user of this OMRON II BP742N including AC-adapter should ensure that it is used in such environment.

Immunity to test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Recommended separation distance d = 1.2 √P where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strength from fixed RF transmitters as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol:
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	Recommended separation distance d = 1.2 √P 80 MHz to 800 MHz d = 2.3 √P 800 MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strength from fixed RF transmitters as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol:

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.
Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

Field strength from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radio, AM and FM radio broadcast, and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the OMRON II BP742N including AC-adapter is used exceeds the applicable RF compliance level above, the OMRON II BP742N including AC-adapter should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the OMRON II BP742N including AC-adapter.
Over the frequency range 150 kHz to 80 MHz, field strength should be less than 3 V/m.

Recommended separation distance between portable and mobile RF communications equipment and the OMRON II BP742N including AC-adapter

OMRON II BP742N including AC-adapter is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of this OMRON II BP742N including AC-adapter can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the OMRON II BP742N including AC-adapter as recommended below, according to the maximum output power of the communications equipment.

Output Power of Transmitter in Watt	Separation distance according to frequency of transmitter in meter			
	150 kHz to 80 MHz d = 1.2 √P	80 MHz to 800 MHz d = 1.2 √P	800 MHz to 2.5 GHz d = 2.3 √P	
0.01	0.12	0.12	0.23	
0.1	0.38	0.38	0.73	
1	1.2	1.2	2.3	
10	3.8	3.8	7.3	
100	12	12	23	

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.
Note: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.
Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

Manufactured by: OMRON HEALTHCARE CO., LTD.
53, Hurotsubo, Tsuru-cho, Muko, Kyoto, 617-0002 JAPAN
Distributed by: OMRON HEALTHCARE, INC.
1925 West Field Court Lake Forest, IL 60045 U.S.A.
www.omronhealthcare.com
© 2013 OMRON HEALTHCARE, INC. Made in Vietnam