

CAUTIONS

DO NOT continue use until further instruction by a physician if your skin is irritated or inflamed around the Patch.

DO NOT place in locations where your skin is scraped, cracked, inflamed, or irritated.

DO NOT place in a location that overlaps the area of the most recently removed Patch.

DO NOT use if you are allergic to adhesive tape.

DO NOT wear the same Patch for longer than one week.

DO NOT drop or bump with excessive force.

DO NOT use to diagnose heart-related conditions.

DO NOT wear during airplane travel.

DO NOT wear during magnetic resonance imaging (MRI), cautery, and external defibrillation procedures.

DO check the expiration date before applying.

DO NOT store in extremely hot, cold, or humid conditions.

SPECIFICATIONS

Battery	Coin cell
Data recording time	5 - 7 days
Storage conditions	Original package under normal room temperature and humidity
Water Resistant	1 Meter

ELECTROMAGNETIC & OTHER INTERFERENCES

The Patch has been tested and deemed in conformance with the relevant requirements in EN60601-1-2:2007 Class B for Electromagnetic Compatibility (EMC).

EQUIPMENT SYMBOLS

LOT Batch code


SN Serial number


REF Catalogue number

ECI REP European Authorized Representative

 Manufacturer


 Read instructions before use

 Type BF applied part

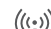
 Emits radio waves


 Single use only

 Temperature range




 Humidity range

 Atmospheric Pressure range

 Use by yyyy-mm

 Do not dispose with household waste

IP27 Ingress Protection rating of IP27

 →  →  Remember: Use mobile device to start patch change


THE PATCH

PATCH BUTTON

Press and hold the patch button for 3 seconds to initiate communication to Mobile Computing Device.

INDICATOR LIGHT

A blinking green light indicates that communication to the Mobile Computing Device has begun.

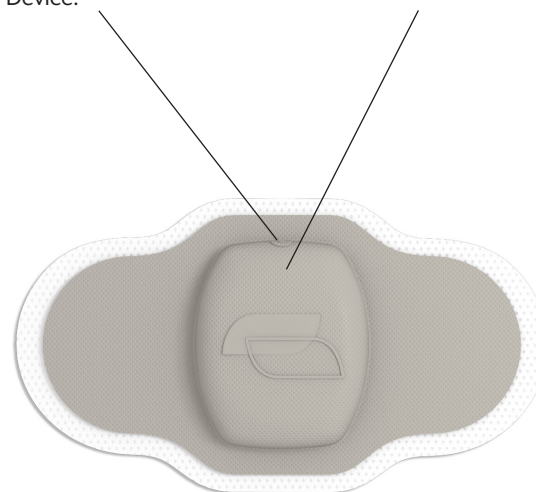

Manufacturer

Proteus Digital Health
2600 Bridge Pkwy, Suite 101
Redwood City, CA 94065
Ph: +1-650-632-4031

ECI REP Proteus Digital Health UK Ltd.
6th Floor, 41 - 44 Great Queen St.
London WC2B 5AD

US Patent No. 8114021
Made in USA
LBL0093, Rev 2 17MAY2013

proteus
DIGITAL HEALTH

INSTRUCTIONS FOR USE

Proteus Patch

CAUTION: Investigational device.
Limited by United States law to
investigational use.

The Proteus Patch is a body-worn sensor that collects physiological and behavioral metrics (*heart rate, activity, body angle, and time-stamped patient-logged events*). Installation involves connecting the Patch to a Mobile Computing Device and affixing the Patch to your skin. The Patch may be worn during most activities, including exercising and bathing.

REPLACING A PATCH

To Start...

Turn on the Mobile Computing Device and launch application. Open the Patch screen and tap “New Patch.”

1. CLEAR

Before putting on a new Patch, follow the instructions on your Mobile Computing Device for clearing your used Patch.



To Remove a Used Patch...

Gently pull the Patch away from your skin.

Dispose of the Patch as you would any household batteries or electronics (*please check your local electronic waste disposal standards*).


2. CONNECT

After the data is cleared from your used Patch, your Mobile Computing Device will instruct you to connect a new Patch.



3. PLACE

When you are connected:

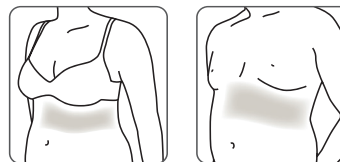


1. **Select a location** on the left side of your body just above the lower edge of your rib cage (*see shaded areas in figure*).

Avoid locations:

- On your abdomen below your rib cage.
- Where undergarments may press against the Patch.
- Where your previous Patch was just removed.


To facilitate later Patch removal, use clippers to trim any hair from your selected location.

2. **Prepare your skin** by gently patting the location you have selected with the provided alcohol wipe. Wait for your skin to dry.

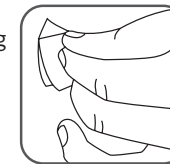


3. Apply the Patch


a. Remove a Patch from its pouch and peel off the **first backing**. Take care not to wrinkle the adhesive.

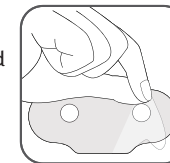

b. Stand in a comfortable, upright position. With the Patch button facing upwards, apply the Patch to your selected area.

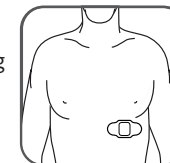

c. Make sure to peel off the **second backing**. Then smooth the edges around the Patch with your fingers.

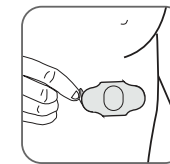

PATCH MAINTENANCE

High levels of activity and water exposure (*e.g. showering, bathing, or swimming*) may cause the Patch to peel off. To help the Patch remain attached, wait at least one hour after applying the Patch before exercising or exposing it to water. When you do exercise or get the Patch wet, you should pat the Patch dry and gently run your fingers along the adhesive to prevent peeling.

ADDITIONAL INFORMATION

For further instruction on troubleshooting and technical information, please visit: www.proteusdigitalhealth.com/suppinfo/RP4_patch.pdf