

CAUTIONS

DO NOT continue use until further instructed by a physician if your skin is irritated or inflamed around the replaceable adhesive strip.

DO NOT place in locations where your skin is scraped, cracked, inflamed, or irritated.

DO NOT place in a location that overlaps the area of the most recently removed replaceable adhesive strip.

DO NOT use if you are allergic to adhesive tape.

DO NOT wear the same replaceable adhesive strip for longer than one week.

DO NOT drop or bump with excessive force.

DO NOT use to diagnose heart-related conditions.

DO NOT wear during airplane travel.

DO NOT wear during magnetic resonance imaging (MRI), cautery, and external defibrillation procedures.

DO check the expiration date before applying.

DO NOT store in extremely hot, cold, or humid conditions.

CAUTION: Patch pod is a choking hazard.

SPECIFICATIONS

Battery _____ Coin cell

Data recording time _____ 30 days

Storage conditions _____ Original package under normal room temperature and humidity

Water Resistant _____ 1 Meter

ELECTROMAGNETIC & OTHER INTERFERENCES

The Patch has been tested and deemed in conformance with the relevant requirements of EN60601-1-2:2007 Class B for Electromagnetic Compatibility (EMC).

EQUIPMENT SYMBOLS

LOT Batch code

SN Serial number

REF Catalogue number

EC REP European Authorized Representative

 Manufacturer

 Read instructions before use

 Emits radio waves

 Single use only

 Temperature range

 Humidity range

 Atmospheric Pressure range

 Use by yyyy-mm

 Do not dispose with household waste

IP27 Ingress Protection rating of IP27

 General warning



Remember: Use mobile device to start Patch pod change


THE PATCH

PATCH BUTTON

Press and hold the patch button for 3 seconds to initiate communication to Mobile Computing Device.

INDICATOR LIGHT

A blinking green light indicates that communication to the Mobile Computing Device has begun.

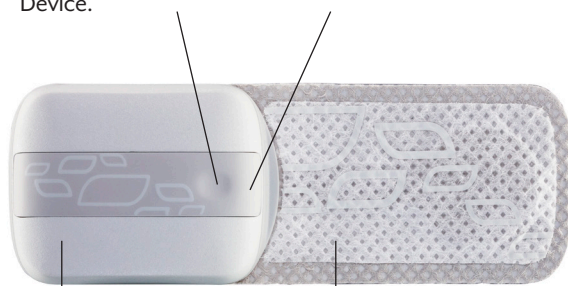

PATCH POD

Contains electronics.

REPLACEABLE ADHESIVE STRIP

Contains skin electrodes. Must be used with Patch pod.

 Proteus Digital Health
2600 Bridge Pkwy, Suite 101
Redwood City, CA 94065
Ph: +1-650-632-4031

EC REP Proteus Digital Health UK Ltd.
6th Floor, 41 - 44 Great Queen St.
London WC2B 5AD

CE 0086

US Patent No. 8114021
Made in USA
LBL-0140, Rev 2 | 19SEP2013

proteus
DIGITAL HEALTH

INSTRUCTIONS FOR USE

Proteus Patch — Extended Use

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

COMPATIBLE WITH:

Replaceable Adhesive Strip (SPC-0656)


The Proteus Patch is a body-worn sensor that collects physiological and behavioral metrics (*heart rate, activity, body angle, and time-stamped patient logged events*).

Installation involves connecting the Patch pod to a Mobile Computing Device and affixing it to your skin using a replaceable adhesive strip (strip). The Patch may be worn during most activities, including exercising and bathing.

REPLACING A PATCH POD *monthly*

1. REMOVE

Gently press in the middle of the strip to release the used Patch pod. Do not discard your used Patch pod yet.

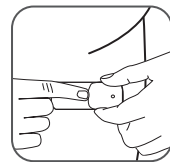


2. CONNECT

Turn on the Mobile Computing Device and launch application. Open the Patch screen and tap “New Pod.” Follow instructions on your Mobile Computing Device to connect a new Patch pod.

3. ATTACH

Press the Patch pod into the frame of the strip until you hear it click.

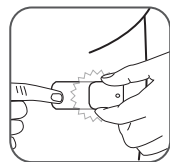


4. DISPOSE

Check status of the used Patch pod. When the data transfer is complete, dispose of the used Patch pod as you would any household battery or electronics (*please check your local electronic waste disposal standards*).

REPLACING A STRIP *weekly*

1. REMOVE

Gently press in the middle of the strip to release the Patch pod. Set it aside for the final step. Pull the used strip away from your skin and discard.


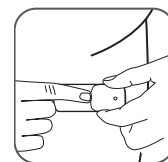


2. CHOOSE

Select a location on the left side of your body just above the lower edge of your rib cage (see shaded areas to right).

Avoid locations


- Where undergarments may press against the Patch
- Where your previous Patch was just removed

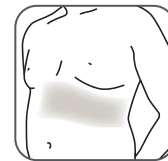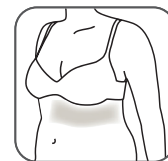


To facilitate later Patch removal, use clippers to trim any hair from your selected location.

3. PLACE


Gently pat your skin at the selected location with the provided alcohol wipe. Wait for your skin to dry.

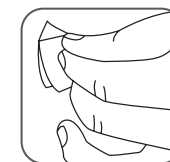

Peel off the first backing from a strip. Take care not to wrinkle the adhesive.

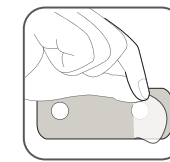


Stand in a comfortable, upright position. Orient the strip horizontally and apply to your selected location. Remove the second backing, and smooth the strip with your fingers.

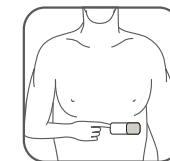

4. ATTACH

Press your Patch pod into the frame on the strip until you hear it click.

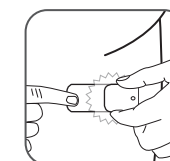

PATCH MAINTENANCE

High levels of activity and water exposure (*e.g. showering, bathing, or swimming*) may cause the Patch to peel off. To help the Patch remain attached, wait at least one hour after applying the Patch before exercising or exposing it to water. When you do exercise or get the Patch wet, you should pat the Patch dry and gently run your fingers along the adhesive to prevent peeling.

ADDITIONAL INFORMATION

For further instruction on troubleshooting and technical information, please visit: www.proteus.com/supinfo/ProteusPatch_ExtendedUse.pdf