Otto Bock®




QUALITY FOR LIFE

FC
CE 0681

D	60X3	BionicLink
GB	60X3	BionicLink
I	60X3	BionicLink

Version 1.0

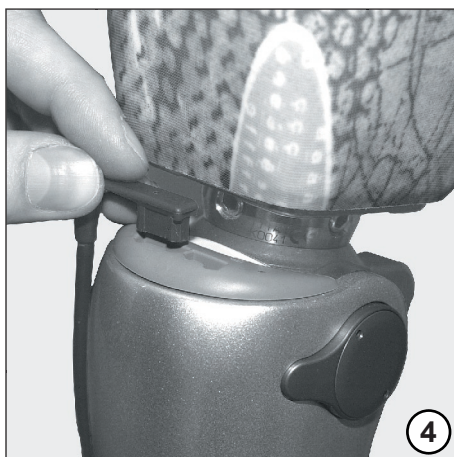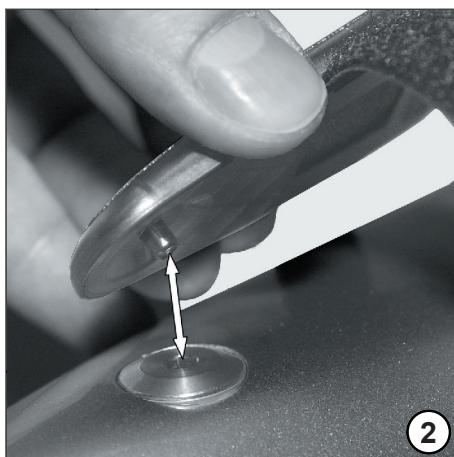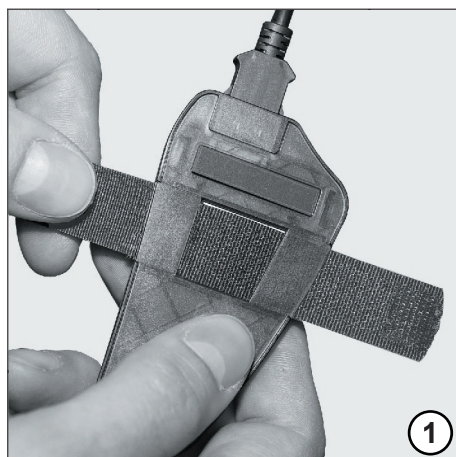Deutsch

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English

Please read this Manual carefully. Take special notice of the Safety Information!

1 Application

The 60X3 BionicLink is intended **exclusively** for data communication when adjusting the C-Leg® Knee Joint.

2 Safety Information

- The BionicLink may only be used by orthopaedic technicians and CPOs certified for C-Leg® fittings.
- Do not allow foreign particles or liquids to enter the device. Do not subject the device to intensive smoke or dust nor to mechanical vibrations or shocks. Avoid staying near sources of high magnetic or electric interference (e.g. transmitters, transformers).
- The BionicLink may only be used within the confines of an orthopaedic workshop or during a stay in a rehabilitation clinic.
- After making the adjustments with the BT BionicLink disconnect it from the prosthesis of the patient.
- The BionicLink cannot automatically identify defects of prostheses or prosthetic components. Therefore it is necessary that the CPO possesses a solid knowledge of its functions and carefully monitors the prosthesis (components).
- The BionicLink is powered by the C-Leg® battery. We therefore recommend to make

adjustments only with a fully charged C-Leg® battery.

- Do not take the BionicLink into operation if there is visible damage to the C-Leg® Radio Adapter Cable or to the housing. If you see any damage, please return the device to an Otto Bock Service Facility.
- Avoid working temperatures under 0°C and over 60°C.
- Do not bend or crush the BionicLink cable.

3 Scope of Delivery and Accessories

3.1 Scope of Delivery

- 1 pce. 60X3 BionicLink
- 1 pce. C-Leg® Radio Adapter Cable
- 1 pce. Velcro® strap
- 1 pce. Operating Manual

3.2 Accessories

To operate the 60X3 BionicLink you need a PC with Bluetooth® technology. If your PC does not have a built-in Bluetooth® function then we recommend the 60X4 BionicLink PC from Otto Bock.

Should you experience problems when using Bluetooth® components from other manufacturers, please contact your Otto Bock Service Facility.

4 Description

The 60X3 BionicLink is for the wireless communication between the C-Leg® Knee Joint and a PC with Bluetooth® functionality.

Patient specific adjustments can be carried out without a cable, thus allowing the patient more room to move while making adjustments. Fixation of the BionicLink under the foam cover while making adjustments is possible and recommended since this offers the possibility to take the influence of the foam cover on the movement cycle into consideration. After making the adjustments with the BT BionicLink disconnect it from the prosthesis of the patient.

The BionicLink is suitable for fitting on either the right or left side.

Attention: The BionicLink does not need batteries since power is supplied by the C-Leg® battery.

5 Applying the 60X3 BionicLink (Fig. 1 to Fig. 4)

1. Plug C-Leg® Radio Adapter Cable into BionicLink (you do not have to remove the plug anymore).
2. Place Velcro® into slit on backside of BionicLink (velcro facing outwards) (Fig.1)
3. Snap BionicLink onto C-Leg® Knee Joint (Fig. 2)

Attention: Position BionicLink in a left sided fitting medially, in a right sided fitting laterally on the C-Leg® Knee Joint.

4. Loop free end of Velcro® around the C-Leg® Knee Joint, pull it through the plastic ring,

pull tight and fasten shut (Fig. 3).

5. Plug C-Leg® Radio Adapter Cable into C-Leg® jack (Fig. 4).
6. The lit up LED signalizes that the BionicLink is ready for operation.

6 Start-up

1. Set-up connection between BionicLink and PC. For details regarding set-up of communication and of a serial communication with Bluetooth®, please refer to your Bluetooth® documentation.
2. Enter PIN.

PIN code needed for communication set-up: 2534

3. Start C-Leg® adjustment software on PC (Version from C-Soft 1.0 upwards).
4. Make the necessary adjustments (Remind the amputee to stand safe and still during the adjustments).
5. Data transmission to BionicLink will function properly with visual contact within a 10 meter perimeter of the PC.
During data transfer the LED will blink.
6. After making adjustments remove BionicLink and C-Leg® Radio Adapter Cable and replace the protective plug on the C-Leg®.

Attention!

If the C-Leg® is out of range or if the transmission path is shaded (e.g. by steel or reinforced concrete) then the connection can be interrupted.
The message "Transmission interrupted" will appear on your screen.

7 Technical Data

Radio Link Range	approx. 10 meter
Ambient Temperature	0 °C - 60 °C
Power Supply	by C-Leg® battery
Radio Transmission	Bluetooth® Class II
Recommended Adjustment Software	from C-Soft Version 1.0 upwards

8 Warranty

Warranty only applies if the 60X3 BionicLink for C-Leg® Knee Joint is used according to the specified conditions and for the intended purposes.

Otto Bock warranty applies only to tested component combinations.

9 Declaration of Conformity

The CE-marking has been carried out in accordance with the requirements of the 93/42/EEC and 1999/5/EG guidelines.

Bluetooth® is a registered trademark.

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This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

(1) This device may not cause harmful interference, and
(2) this device must accept any interference received, including interference that may cause undesired operation.

This device and its antenna must not be co-located or operating in conjunction with any other antenna or transmitter.

Modifications not expressly approved by Otto Bock could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment.

This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications.

Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

Otto Bock Healthcare Products GmbH
Kaiserstraße 39 · 1070 Wien · Austria
Tel. (+43-1) 526 95 48 · Telefax (+43-1) 526 79 85
vertrieb.austria@ottobock.com · www.ottobock.at

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