Introduction

Final format will be

- 1 language per booklet
- Each product delivery will be done with the correct user guides (e.g. GB and FR for Canada).
- All standard languages to be supported US EN, DE, FR, IT, ES, PT, NL, DN, FIN, SW, NW
- Special languages, which are supported by the phone:
 - Turkish
 - Polish

NOTE:

The following status is still containing many German parts, which are now in translation process.

The German part however only affects the end-user content. **All regulatory aspects are in English already.**

See Chapter 3: Important safety information

Chapter 9: Information on product safety

Chapter 10: Service Warranty

Chapter 11: Compliance information

Chapter 12: Information and explanation of symbols

[[ArtNr. number to be updated]]

Phonak DECT CP1 Cordless phone

User Guide

PHONAK
life is on

Manufacturer:
Phonak AG
Laubisrütistrasse 28
CH-8712 Stäfa
Switzerland

EU Representative:
Phonak GmbH
D-70736 Fellbach-Oeffingen
Germany

www.phonak.com

029-0235-02/M1.01/2012-09/misynt Printed in Switzerland © Phonak AG All rights reserved

Of course the content will be formatted later.

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1. Welcome

The cordless phone DECT CP1 is a quality product developed by Phonak, one of the world's leading companies in the field of hearing technology.

Please read the user guide thoroughly in order to be able to benefit from all the possibilities this product can offer.

If you have any questions, please consult your hearing care professional or go to www.phonak.com/dect

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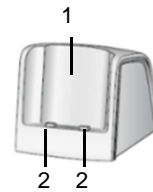
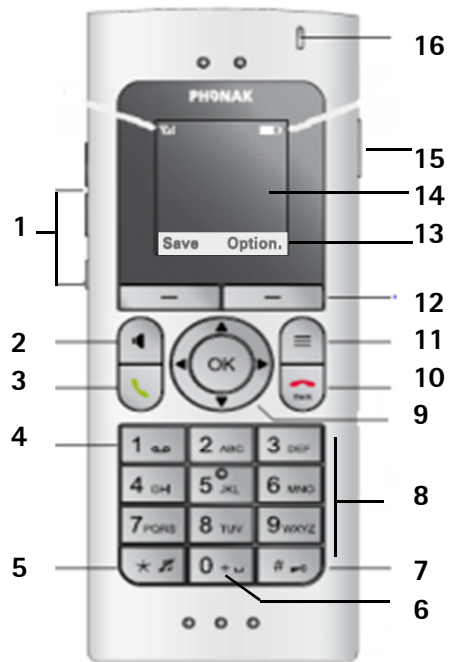
3. Power supply input (miniUSB, 5VDC, 1000mAh)
4. Page Handset: short press will force handsets to ring.
Press 2 sec.= Manual pairing of the handset to the base.
5. Analog phone line connector.

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Mobilteil

8

1. Volume +/- and volume boost (→ p.xx)
Press 2 sec.=Hearing aid test (→ p.xx).
 2. Hands-free (phone speaker) on/off (→ p.xx).
 3. Talk/repeat call (→ p.xx).
 4. Press **[*]** 2 sec.= Call voicemail number (→ p.xx).
 5. Press **[*]** 2 sec.= All ring tones off/on (→ p.xx).
In writing mode: switch upper and lower case.
 6. Press **[*]** 2 sec.= Dial international prefix "+".
 7. Press **[*]** 2 sec.= Keypad lock on/off (→ p.xx).
 8. Number keypad. In writing mode: alphabetic character entry.
 9. Navigation keys (→ p.xx).
 10. End call / Back (one menu level up) (→ p.xx).
Press 2 sec. = back to home screen.
Press 2 sec. in home screen = Phone on/off (→ p.xx).
 11. Function key[Funktionstaste] (internal call, direct call, etc.) (→ p.xx).
 12. Display keys[Bildschirmtasten] (→ p.xx)
 13. Context menu (use the display keys to select functions depending on the display)
 14. Display[Bildschirm]
 15. 3.5mm headset plug (→ p.xx).
 16. Indicator light (→ p.xx).
- ① Um die Anzeigesprache zu ändern gehen Sie vor wie in 4.5 beschrieben.

9

Display symbols and navigation

Idle mode display and fixed first level menu items:

The diagram shows a mobile phone display in idle mode. At the top, there are signal strength bars (9, 8, 7, 6, 5, 4) and icons for signal strength, call, lock, and battery. Below these, it displays 'DECT 1', the time '15:45', and the date '7. Februar'. At the bottom, there are three navigation buttons labeled 1, 2, and 3. Below the display, there are icons for 'Kontakte', 'Anrufe', 'Sprachbox', 'Wecker', 'Audio Einstellungen', 'Hörgerät', and 'Einstellungen'. A horizontal navigation bar contains icons for home, call, end call, alarm, volume, mute, and settings, with corresponding page numbers below them: p. 5.7, p. 5.1, p. 5.12, p. 6.5, p. 6.2, p. 5.3, and p. 6.4.

1. Time
2. Date
3. Alarm clock on
4. Battery and charging state
5. Ringtone off
6. Keypad locked
7. Handset name (visible if more than one handset is used with the same base)
8. Eco-mode switched on
9. Signal strength from base

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p. 5.7 p. 5.1 p. 5.12 p. 6.5 p. 6.2 p. 5.3 p. 6.4

10

Example display if call is in progress

The diagram shows a mobile phone display during an active call. At the top, there are signal strength bars (8, 7, 6, 5, 4, 3) and icons for call, mute, and HD. Below these, it displays the phone number '0800800800' and the elapsed time '0:05'. At the bottom, there are two navigation buttons labeled 1 and 2. Below the display, there are icons for 'Mikrofon' and 'Option'.

1. Context dependent functions
2. Elapsed time of call
3. HD (high definition) voice active (e.g. when doing internal call)
4. Phone number
5. Active call indicator
6. Booster mode on
7. Phone microphone muted
8. Hands free on

Mikrofon Option

① The content of the display varies depending on the settings.

2.4. Hearing aid compatibility

This product uses a dedicated wireless link to the Phonak hearing aids. Please ask your hearing care professional about the compatibility to your hearing aids.

The compatibility can easily be checked using the demo mode (→ p. 4.8).

11

3. Important safety information

Please read the information on the following pages before using your Phonak DECT CP1.

- ⚠ These devices may only be repaired by an authorized service center. Changes or modifications to the device that were not explicitly approved by Phonak are not permitted. Such changes may damage these devices, your ear or the hearing aid.
- ⚠ Opening any of the devices might damage them. If problems occur which cannot be resolved by following the instructions of this user guide, consult your hearing care professional.
- ⚠ Do not use the devices in explosive areas (mines or industrial areas with danger of explosions).
- ⚠ Do not use the devices in areas where electronic equipment is prohibited.

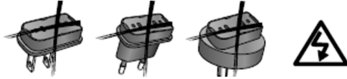
12

-
- ⚠ The handset contains small parts that can cause choking, if swallowed by children. Keep out of reach of children under 3 years, mentally challenged people and pets. If swallowed, consult a physician or hospital immediately.
 - ⚠ The following is only applicable for persons with active implantable medical devices (i.e. pacemakers, defibrillators, etc.):

If you experience any interference, do not use the device and contact the manufacturer of the active implant (please, note that interference can also be caused by power lines, electrostatic discharge, airport metal detectors etc.).
 - ⚠ Please contact the local recycling facility for removal of the battery and dispose of electrical components in accordance with your local regulations.
 - ⚠ External devices may only be connected if they have been tested in accordance with corresponding IECXXXXX standards.

13

-
- ⚠ Using the cables in any way contradictory to their intended purpose (e.g., wearing the phone cable around the neck) can cause injury.
 - ⚠ The mini-USB port is to be used for the described purpose only.
 - ⚠ Caution: electric shock. Do not insert plug alone into electrical outlets. [BILD GA COMPILOT S.75]

- ⚠ For safety reasons, only use chargers supplied by Phonak or USB certified chargers with a rating of 5VDC, min. 1000 mA.
- ⚠ Never supply the base from a PC or laptop USB port. This may damage your equipment.
- ⚠ Do not connect an USB or phone cable exceeding 3 meters (9 ft) in length to the base.
- ⚠ Only insert a rechargeable battery that meets the specification in the Phonak DECT CP1 data sheet as inserting wrong batteries can damage the device or cause serious injuries.

-
- ⚠ Do not hold the device onto the ear during ringtone or in hands free mode. This may damage the ear.
 - ⚠ The handset may cause interference with analog hearing aids. Please contact the hearing care professional in case of problems.
 - ⚠ Always include the user guide when handing over your Phonak DECT CP1 to other people.
 - ⚠ A defective base must be removed from the network as it could disturb other radio services.

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
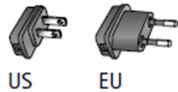
4. Getting started

4.1. Setting up the power supply

1. Select the adapter that corresponds to your country.

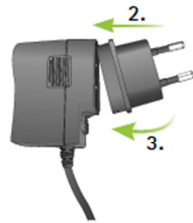
2. Insert the round corner of the adapter to the round corner of the universal power supply until it is completely inserted.

3. Click the tail of the adapter in to lock it into place. Make sure the adapter is securely locked.

To remove the power adapter:

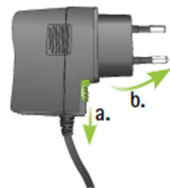- a. Pull back the tab on the universal power supply and hold.
- b. Gently pull the adapter upwards to remove it from receptacle.
- c. Then start over with step 1 (above).

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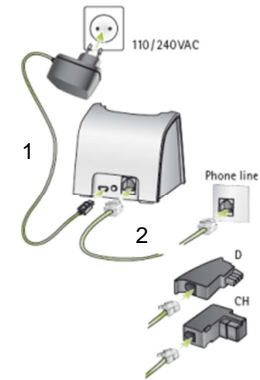
1. Plug the mini-USB plug into the base and the power supply into a power outlet.
2. Connect the line input of the base with the analog phone socket using the phone cable.

Use an appropriate phone line adapter if required (German and Swiss adapter is part of the package).

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1. Make sure the battery contacts are located as shown.
2. Insert the contacts side first.
3. Insert the battery completely. You will hear the startup melody as the phone will switch on automatically.
4. Align the battery cover flat on top of the battery.
5. Then slide it gently to click it into place.

- ① If you open the battery cover, please lift the small latch at the bottom first, then slide the cover open.

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69

9. Information on product safety

- ① Protect the device connectors, plugs, charging slot and power supply from dirt and debris.
- ① Never immerse the devices in water! Protect them from excessive moisture. Never use these devices in the shower, bath, or swimming area, as the device contains sensitive electronic parts.
- ① Protect the devices from heat (never leave it near a window.) Never use a microwave or other heating devices to dry the devices.
- ① Do not drop these devices! Dropping onto a hard surface can damage these devices.
- ① Clean devices using a damp cloth. Never use household cleaning products (washing powder, soap, etc.) or alcohol to clean them.

70

-
- ① Special medical or dental examination including radiation described below, may adversely affect the correct functioning of your device. Remove and keep it outside the examination room/area before undergoing:
 - Medical or dental examination with X-ray (also CT scan).
 - Medical examination with MRI/NMRI scans, generating magnetic fields.
 - ① When operating the system near computer equipment, larger electronic installations, strong electromagnetic fields, or metallic structures, the operating range may significantly be reduced. it may be necessary to be at least 60 cm (24") away from the interfering device to ensure proper operation.
 - ① If the hearing aids do not respond to the Phone because of an unusual field disturbance, move away from the disturbing field.

71

10. Service warranty

10.1. Local warranty

Please ask the hearing care professional, where you purchased your Phonak DECT CP1, about the terms of the local warranty.

10.2. International warranty

Phonak offers a one-year limited international warranty, valid starting from the date of purchase. This limited warranty covers manufacturing and material defects. The warranty only comes into force if proof of purchase is shown.

The international warranty does not affect any legal rights that you might have under applicable national legislation governing sale of consumer goods.

10.3. Warranty limitation

This warranty does not cover damage from improper handling or care, exposure to chemicals, immersion in water or undue stress. Damage caused by third parties or non-authorized service centers renders the warranty null and void. This warranty does not include any services performed by a hearing care professional in their office.

Serial number:

Authorized hearing
care professional
(stamp/signature):

Date of purchase:

11. Compliance information

Europe:

Declaration of Conformity

Hereby Phonak AG declares that this Phonak product is in compliance with the essential requirements of the Radio and Telecommunications Terminal Equipment Directive 1999/5/EC. The full text of the Declaration of Conformity can be obtained from the manufacturer or the local Phonak representative whose address can be taken from the list on <http://www.phonak.com> (Phonak worldwide locations).

US and Canada:

1. This equipment complies with Part 68 of the FCC rules and the requirements adopted by the ACTA. On the bottom of this equipment is a label that contains, among other information, a product identifier in the format US:KWCTE00BX8170. If requested, this number must be provided to the telephone company.

74

2. A plug and jack used to connect this equipment to the premises wiring and telephone network must comply with the applicable FCC Part 68 rules and requirements adopted by the ACTA. A compliant telephone cord and modular plug is provided with this product. It is designed to be connected to a compatible modular jack that is also compliant. See installation instructions for details.

3. If this equipment Phonak DECT CP1 causes harm to the telephone network, the telephone company will notify you in advance that temporary discontinuance of service may be required. But if advance notice isn't practical, the telephone company will notify the customer as soon as possible. Also, you will be advised of your right to file a complaint with the FCC if you believe it is necessary.

4. The telephone company may make changes in its facilities, equipment, operations or procedures that could affect the operation of the equipment. If this happens the telephone company will provide advance notice in order for you to make necessary modifications to maintain uninterrupted service.

75

5. If trouble is experienced with this equipment Phonak DECT CP1, for repair or warranty information, Service can be facilitated through our office at:

Phonak Inc.
4520 Weaver Parkway
Warrenville, IL 60555
Tel: 800-777-7333
Fax: 630-393-7400

If the equipment is causing harm to the telephone network, the telephone company may request that you disconnect the equipment until the problem is resolved.

6. Do not alternate or repair any parts of device except specified. For repair procedures, follow the instructions outlined under the limited warranty.

7. Connection to party line service is subject to state tariffs. Contact the state public utility commission, public service commission or corporation commission for information.

8. If your home has specially wired alarm equipment connected to the telephone line, ensure the installation of this 1.9GHz DECT Product for PSTN does not disable your alarm equipment. If you have questions about what will disable alarm equipment, consult your network operator or a qualified installer.

9. If the telephone company requests information on what equipment is connected to their lines, inform them of:

- a) The ringer equivalence number [00]
- b) The USOC jack required [RJ11C]
- c) Facility Interface Codes ("FIC") [02LS2]
- d) Service Order Codes ("SOC") [9.0Y]
- e) The FCC Registration Number [KWCTE00BX8170]

10. The REN is used to determine the number of devices that may be connected to a telephone line. Excessive RENs on a telephone line may result in the devices not ringing in response to an incoming call. In most but not all areas, the sum of RENs should not exceed five (5.0). To be certain of the number of devices that may be connected to a line, as determined by the total RENs, contact the local network provider. The REN for this product is part of the product identifier that has the format US:AAAEQ##TXXXX. The digits represented by ## are the REN without a decimal point. For this product the FCC Registration number is [US: KWCTE00BX8170] indicates the REN would be 00.

11. This product is hearing aid compatible.

FCC Caution:

Changes or modifications made to this device not expressly approved by Phonak may void the FCC authorization to operate this device.

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions:

- 1) this device may not cause harmful interference, and
- 2) this device must accept any interference received, including interference that may cause undesired operation.

IMPORTANT NOTE:

FCC Radiation Exposure Statement:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance 20cm between the base & your body.

This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Note:

This device has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. This device generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this device does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the device and receiver.
- Connect the device into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

80

Specific Absorption Rate (SAR) information

SAR tests are conducted using standard operating positions accepted by the FCC with the phone transmitting at its highest certified power level in all tested frequency bands, although the SAR is determined at the highest certified power level, the actual SAR level of the phone while operating can be well below the maximum value, in general, the closer you are to a wireless base station antenna, the lower the power output.

Before a new model phone is available for sale to the public, it must be tested and certified to the FCC that it does not exceed the exposure limit established by the FCC, Tests for each phone are performed in positions and locations (e.g. at the ear and worn on the body) as required by the FCC.

For body worn operation, this handset has also been tested and meets the FCC RF exposure guideline. Use of other accessories may not ensure compliance with FCC RF exposure guidelines..

Non-compliance with the above restrictions may result in violation of RF exposure guidelines.

81

Privacy of communications may not be ensured when using this telephone!

IC Warning

CS03:

This product meets the applicable Industry Canada technical specifications.

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RSS(Category I Equipment):

This device complies with Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions:

- (1) this device may not cause interference, and
- (2) this device must accept any interference, including interference that may cause undesired operation of the device.

Under Industry Canada regulations, this radio transmitter may only operate using an antenna of a type and maximum (or lesser) gain approved for the transmitter by Industry Canada. To reduce potential radio interference to other users, the antenna type and its gain should be so chosen that the equivalent isotropically radiated power (e.i.r.p.) is not more than that necessary for successful communication.

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83

12. Information and explanation of symbols

[[all copied from actual user guide]]

With the CE symbol, Phonak AG confirms that this Phonak product – including accessories – meets the requirements of the Medical Devices Directive 93/42/EEC as well as the R&TTE Directive 1999/5/EC on radio and telecommunications equipment. The numbers after the CE symbol correspond to the code of certified institutions that were consulted under the above-mentioned directives.

84

This symbol indicates that it is important for the user to read and take into account the relevant information in these user guides.

This symbol indicates that it is important for the user to pay attention to the relevant warning notices in these user guides.

Important information for handling and product safety.

Operating conditions

The product is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in these user guides.

85

**Transport
and storage
conditions**

Temperature: -20° to +60° Celsius
(-4° to +140° Fahrenheit)
Humidity Transportation: Up to 90%
(non condensing)
Humidity Storage: 0% to 70%, if not
in use. See instruction in chapter 2.2
regarding drying the hearing aid after
use.
Atmospheric pressure: 200 hPa to
1500 hPa

The symbol with the crossed-out
garbage bin is to make you aware
that this hearing aid may not be
thrown away as normal household
waste. Please dispose of old or unused
hearing aids, at waste disposal sites
intended for electronic waste, or
give your hearing aid to your hearing
care professional for disposal. Proper
disposal protects the environment and
health.