


# ULTRAMATIC

MF#600  
REV A

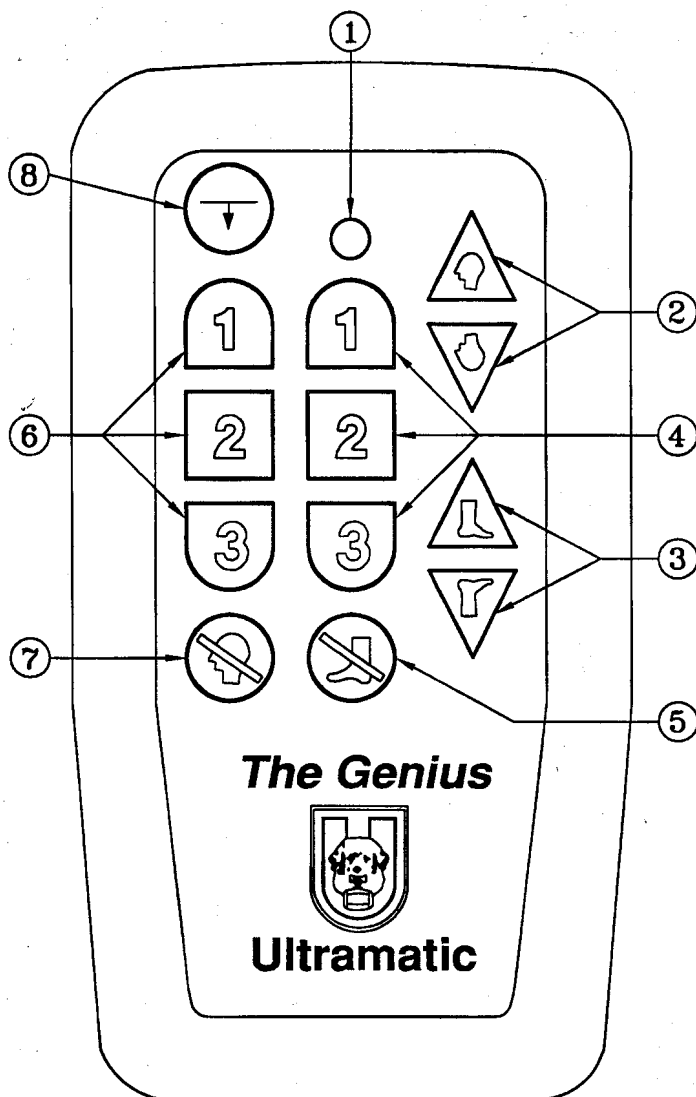

- ① **TRANSMISSION INDICATOR LIGHT:**  
Verifies a button is being pressed.
- ② **HEAD UP/DOWN BUTTONS:**  
Adjusts the head section of the bed to the desired position.
- ③ **FOOT UP/DOWN BUTTONS:**  
Adjusts the foot section of the bed to the desired position.
- ④ **FOOT MASSAGE ON/SPEED CONTROL BUTTONS:**  
1 LOW, 2 MEDIUM, 3 HIGH  
When turned on, massage motor will shut off in 30 minutes.
- ⑤ **FOOT MASSAGE OFF BUTTON:**  
Turns off foot massage.
- ⑥ **HEAD MASSAGE ON/SPEED CONTROL BUTTONS:**  
1 LOW, 2 MEDIUM, 3 HIGH  
When turned on, massage motor will shut off in 30 minutes.
- ⑦ **HEAD MASSAGE OFF BUTTON:**  
Turns off head massage.
- ⑧ **FLAT BUTTON:**  
Levels the head and foot sections to the flat position and turns off massage functions when button is pressed and released.
- ⑨ **AUTO LEARN FUNCTION:**  
Simultaneously press and hold the flat and head massage off buttons. While holding these buttons the LED transmission indicator light (at the top of the wand) will turn off and then it will begin flashing rapidly. Continue to hold for 2 seconds then release. The new wand code has been learned. Please note that when using the auto learn feature when more than one wireless equipped bed is in transmission range (approx. 50 ft. radius) you must remove power from all beds not intended to learn the new hand control address.

## OTHER FEATURES:

- \* Both head and and foot functions may be operated simultaneously.
- \* Radio Frequency is 310 MHz.
- \* To conserve battery life, the transmitter turns off all functions if any button is pressed for more than 50 consecutive seconds.
- \* **WARNING:** The transmitter and the receiver of the wireless control system have been tested and comply with FCC and Industry Canada rules. Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.