## Instruction for Use


### ReliOn® Digital Temple Thermometer

Thank you for purchasing the ReliOn® Digital Temple Thermometer. The thermometer enables accurate temperature measurements in approximately 6-10 seconds, using simple steps.

#### Table of Contents:

#### QUICK START

- Section 1 About Body Temperature Measurement
- Section 2 Installing and Replacing the Batteries
- Section 3 Guidelines for Use
- Section 4 Care and Cleaning
- Section 5 Troubleshooting and Special Indications
- Section 6 Technical Information
- Section 7 Warranty and Service Information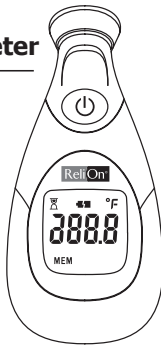

**IMPORTANT! Read user manual before using the thermometer.**

#### QUICK START

1. Install batteries into the thermometer. Make sure the polarity is correct (see section 2).
2. Press and release the blue ON/OFF button. Unit will beep once. Wait until it beeps again twice and °F appears in the display (see section 3).
3. Place and hold the thermometer probe firmly to the skin at the temple and wait several seconds for the device to beep once more (see section 3).
4. Read the temperature on the display (see section 3).

#### General Warnings:


1. Measurement body location is the temple, in proximity to the temporal artery, between the eye socket and the hairline.
2. Do not place the thermometer on scarred tissue compromised by skin disorder, open sores or abrasions and patients in trauma.
3. Consult your physician when using drug therapies that may raise the local skin temperature in the forehead; this may lead to incorrect reading.
4. Do not place the thermometer on skin exposed to direct sunlight, fireplace heat, air conditioner flow and cold/warm compress therapies.
5. Keep the thermometer out of reach of unsupervised children.
6. Do not drop the thermometer or expose to heavy shock. The unit is not shock proof.

7. Do not immerse the thermometer in water or any other liquid. The unit is not water proof.
8. In order to ensure accurate measurement, please wait at least two minutes between consecutive measurements for the unit to readjust to room temperature.
9. The device is not suitable for use in presence of flammable anesthetic mixture with air or with Oxygen or Nitrous Oxide.
10. Do not store the unit in direct sunlight, at a high temperature, in high humidity or dust. Performance of the unit may be degraded.
11. Stop using the thermometer if it operates erratically or if display malfunctions.
12. Clean the thermometer probe before storing. (See section 4, Care & Cleaning)
13. Do not attempt to disassemble the unit except to replace the batteries.
14. If the thermometer has been stored at below-freezing temperatures, allow the unit to warm naturally to room temperature before using.
15. Performance of the device may be degraded if operated or stored outside stated temperatures and humidity ranges or if the patient's temperature is below the ambient (room) temperature.

#### 1. About Body Temperature Measurement:

How does the thermometer work?

Your ReliOn® Digital Temple Thermometer accurately measures body heat using Rapid, Accurate, State of the Art Technology. Sensor **A** rapidly tracks the heat flow generated from blood vessels **B** to the skins surface **C** and converts measurement to body temperature.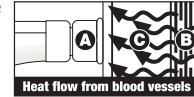

#### Information about Body Temperature

##### What is "normal" body temperature?

"Normal" body temperature varies from person to person and changes according to time of day and body location. Hot or cold baths, physical exercise, smoking and drinking, hot or cold drinks may change body temperature by one degree or more.

The following table shows the Temperature ranges:

	What is fever?	Average normal temperature ranges
<b>Forehead</b>	Over 100.0°F / 37.8°C	97.0°F - 100.0°F / 36.1°C - 37.8°C
<b>Rectal*</b>	Over 100.4°F / 38°C	97.3°F - 100.4°F / 36.3°C - 38°C
<b>Oral (mouth)*</b>	Over 100.0°F / 37.8°C	97.0°F - 100.0°F / 36.1°C - 37.8°C
<b>Axilla (armpit)*</b>	Over 99.0°F / 37.2°C	95.4°F - 99.0°F / 35.2°C - 37.2°C

\* Fever definitions according to the American Medical Association

1

2

The thermometer is designed to measure with precision the temperature in the forehead, in proximity to the temporal artery, between the eye socket and the hairline. It is important to know that the thermal characteristics of each measurement body location – temporal, axillary, oral or rectal - are unique. The variance in temperature measurement in different body locations is due to the human body and not the thermometer. The thermometer readings are equivalent to oral measurements.

#### IMPORTANT:

- " It is recommended that you measure the body temperature of each individual when healthy to establish a "normal" temperature.
- " Due to the unique body physiology of each person, it is recommended that you take a persons temperature at the same temple location in comparative measurements.

#### 2. Installing and Replacing the Batteries:

The thermometer uses 2 standard AAA 1.5v batteries. To install/replace :

1. Remove the battery cover by sliding it out of its housing.
2. Firmly attach the batteries to their place, make sure polarity is correct.
3. Slide the battery cover back into place.

**IMPORTANT:** It is recommended to remove the batteries from the thermometer when it is not used for a long period.

**WARNING:** Do not dispose of batteries in fire, recharge, put in backwards, disassemble, mix with used or other battery types – they may explode or leak and cause injury. Waste batteries should be deposited in batteries recycling trash only.

#### 3. Guidelines for Use:


- a. Use this thermometer on clean dry skin. Wipe away any perspiration prior to use.
- b. Make sure that the thermometer and the patient do not move while measurement is in progress.
- c. Correct placing and good contact with the body ensure reliable readings.
- d. Do not try to take a measurement until the two beeps are heard.

#### Setting the Operating Scale °F or °C

The device setting is by default Fahrenheit (°F), however at any time, the scale may be changed. To change the scale to Celsius (°C), or back to °F, by pressing the Power button within the first 3 seconds of turning on the unit.

#### Operating the Thermometer

1. Press and release the blue Power button. You will hear one beep. All segments will be displayed.
2. The result of the last temperature measurement will be displayed for 2 seconds\*.
3. Wait for the hourglass icon to disappear, then two beeps will be heard to indicate that the thermometer is now "ready to measure".
4. Take a measurement according to the following guidelines:

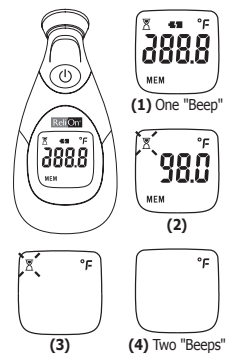#### Taking Temperature:

- a. Make sure temporal area is dry and clean.
- b. Remove hair from the temporal area.
- c. Place the thermometer probe in the center of the temporal area between the eye socket and the hairline. Gently press it against the temple to ensure good contact with the skin is maintained.
- d. Hold the device still and keep the patient still for the duration of the measurement.
- e. As soon as the thermometer is in place, the measurement starts automatically. A running circle will appear indicating that measurement is progressing successfully.
- f. A temperature reading will be displayed within 6 – 10 seconds. One long beep will be heard to indicate that measurement is complete.
- g. Wait for the hourglass icon to disappear, then two beeps will be heard to indicate that the thermometer is now "ready for measurement again".
- h. Press and release the blue Power button to turn the thermometer off.
- i. If you want to get the maximum accuracy, please wait two minutes after turning the thermometer off and then begin again at Step 1 of "Operating the Thermometer".

(f) One long "Beep"

(g) Two "Beeps"

(h)


3

4

### Memory Mode


#### Recalling Memory:

- If the power is on, turn off the power first.
- Press and hold the Power button for about 3 seconds until the last recorded Temperature is displayed.
- Pressing the Power button repeatedly will take you to each recorded temperature in the memory.
- After you release the Power button the unit will return to all segments display.
- Wait for the hourglass icon to disappear, then two beeps will be heard to indicate that the thermometer is now "ready to measure".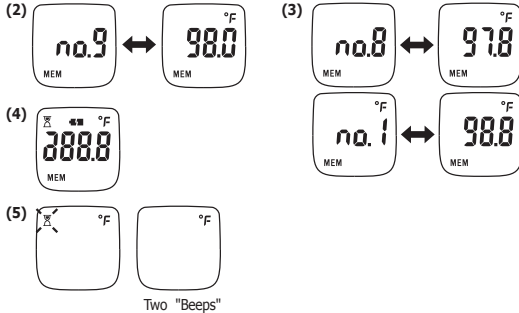

#### Deleting Memories:

- Repeat the first two steps of "Recalling Memories".
- Press and hold the Power button for three seconds until "dEL" is displayed.
- (a) To delete:  
Press and release the Power button to erase all of the records in the unit. A beep is heard.
- (b) To cancel:  
DO NOT press the Power button for 10 seconds, the unit will return to all segments display. Wait for the hourglass icon to disappear, then two beeps will be heard to indicate that the thermometer is now "ready to measure".

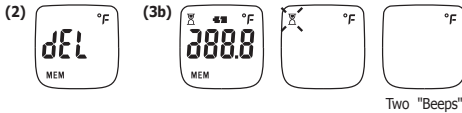### IMPORTANT:

- Clean the thermometer after each use (see section 4).
- In order to ensure accurate measurement, please wait at least two minutes between consecutive measurements for the unit to readjust to room temperature.**
- If the thermometer is not turned off, it will shut off automatically within 1 minute.
- Once the thermometer is "ready to measure", it will shut off automatically after 1 minute if it has not made good contact or maintained good contact with the patient's skin.
- Because the thermometer is a very sensitive device, the patient and thermometer should be in the same room temperature (approx. 60.8~104.0° F) for at least 20 minutes before using to avoid any incorrect measurements.

### 4. Care and Cleaning:

- Wipe the tip and probe with an alcohol swab or cotton swab moistened with 70% isopropyl alcohol before & after each use. \* After cleaning, **wait at least 2 minutes** before taking a measurement for the unit to readjust to room temperature.
- Use only a soft dry cloth to clean the body of the thermometer (if necessary).
- Never use an abrasive cleanser or submerge the thermometer in water or other liquids.**
- Do not expose the thermometer to extreme temperatures, humidity, direct sunlight or shock. Store at room temperature.

### 5. Troubleshooting and Special Indications:

The thermometer has been designed to display feedback/warnings to ensure proper usage and accurate temperature measurement. When an error or indication appears, one "beep" signal will be heard once every second for 5 seconds and then the thermometer will turn off automatically. In the event of a malfunction or incorrect temperature measurement, the following warnings will be displayed:

**Indication:** The measured temperature is below 95°F (35°C) Display will read LO.

**Instruction:** Turn the unit OFF to wait at least 2 minutes and then ON again. Retake temperature using proper technique (see section 3).

**Indication:** The measured temperature is above 107.6°F (42°C). Display will read HI.

**Instruction:** Turn the unit OFF to wait at least 2 minutes and then ON again. To verify the reading, retake temperature using proper technique (see section 3).

**\*\* WARNING - A high fever condition requires seeking immediate medical attention \*\***  
**HIGH FEVERS ARE DANGEROUS**

5

6

**Indication 1:** The thermometer is too cold or too hot to operate.

**Instruction:** Verify room temperature is within product range 60.8°F to 104.0°F (16°C to 40°C). Retake temperature using proper technique (see section 3).

**Indication 2:** Temperature did not register because of poor contact, patient movement or poor placement.

**Instruction:** Turn the unit OFF, wait two minutes, turn it ON again and retake temperature using proper technique (see section 3).

**Indication 3:** Unit malfunctioned.

**Instruction:** Do not use the thermometer – contact distributor for instructions (see section 7).

**Low Batteries Indication:** Battery icon is on continuously.

**Instruction:** The batteries have enough remaining life for approximately 30 more measurements.

**Drained Batteries Indication:** Displays OFF with battery icon flashing when thermometer is turned on and the device automatically turns itself OFF after 15 seconds.

**Instruction:** The batteries are drained - replace batteries (see section 2).

### 6. Technical Specifications:

Feature	Performance / Data
Measurement time	6–10 seconds
Measurement locations	Temporal artery area of the forehead
Measurement range	95.5°F to 107.6°F (35°C to 42°C)
Operation environment	60.8°F to 104.0°F (16°C to 40°C) Humidity: 15-95% (non condensing)
Storage environment	-4°F to 122°F (-20° C to 50° C), Humidity: 15-95% (non condensing)
Measurement readout	Fahrenheit or Celsius
Accuracy	Complies with ASTM E 1112-00 standard requirements
Display	TN LCD 1" x 1" (27.0mm x 27.0mm)
Batteries	2 AAA 1.5volt batteries
Battery life	Minimum 1000 full duty cycles over 24 month period
Weight	2.24oz (56.2gr) including batteries
Dimensions	3.86" x 1.79" x 1.17" (9.9 cm x 4.6 cm x 3.0 cm)
Memory	Nine sets
Auto shut off	After 60 seconds
Case material	Food grade ABS plastic
Probe material	Stainless steel and food grade ABS plastic
Warranty	1 year

### 7. Warranty and Service Information:

This thermometer has a One Year Limited Warranty against manufacturers defect under normal use. Following the directions closely will ensure years of dependable operation.

If the thermometer does not function properly, first check the batteries (see Battery Replacement instructions – section 2). Replace if necessary.

If the thermometer does not function properly, please write BESTMED, LLC at 331 Corporate Circle, Ste E, Golden, CO 80401 or you may call customer service at 1-877-299-6700. Our representative will provide further instruction on how to correct the trouble or will ask you to return it for repair or replacement.

\* Internally powered device \* Type - BF Applied part.

\* Degree of protection against ingress of water: Ordinary

⚠ Read the Instructions For use 🧑‍🔧 Applied part type BF equipment

Distributed by: BESTMED, LLC" Golden, CO 80401

Manufactured by: K-jump Health Co., Ltd.

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Distribuido por: BESTMED, LLC Golden, CO 80401
Fabricado por: K-jump Health Co., Ltd.