

# **cobas h 232 POC system** *Operator's Manual*



<b>Manual version</b>	<b>Revision date</b>	<b>Changes</b>
Version 1.0	2006-10	New document
Version 2.0	2009-09	Minor revisions from internal review
Version 3.0	2011-09	Update to SW 03; enhanced error messages
Version 4.0	2014-05	Added description of new OTS functionality, miscellaneous minor revisions
Version 5.0	2015-xx	Transition to new meter hardware: 2D barcode scanner; wireless LAN option; new battery pack. Update to SW 04.00. Update of cleaning and disinfection section; added safety information on battery packs; deleted appendix for addresses; misc. editorial revisions.

# **cobas h 232 POC system**

## **Operator's Manual**

**Version 5.0**

© 2006-2015, Roche Diagnostics GmbH. All rights reserved

The contents of this document, including all graphics, are the property of Roche Diagnostics. Information in this document is subject to change without notice. Roche Diagnostics shall not be liable for technical or editorial errors or omissions contained herein. No part of this document may be reproduced or transmitted in any form or by any means, electronic or mechanical, for any purpose, without the express written permission of Roche Diagnostics.

Please send questions or comments about this manual to your local Roche representative.

ROCHE CARDIAC, COBAS, COBAS H and IQC are trademarks of Roche.



The Wi-Fi CERTIFIED Logo is a certification mark of the Wi-Fi Alliance.

On the packaging and on the identification plate of the meter you may encounter the following symbols, shown here with their meaning:



Caution, consult accompanying documents. Refer to safety-related notes in the instructions for use accompanying this instrument.



Temperature limitation (Store at)



Manufacturer



Catalogue number



In vitro diagnostic medical device



This product fulfills the requirements of the European Directive 98/79/EC on in vitro diagnostic medical devices



The system fulfills the Canadian and U.S. safety requirements (UL LISTED, in accordance with UL 61010A-1:02 and CAN/CSA-C22.2 No.61010-1-04).

On meters with WLAN capability:



This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada

For other WLAN certifications, see label on bottom of battery compartment and addendum for information on WLAN registration.

The user is fully responsible for the installation, use and upkeep of the **cobas h 232** meter.

This page intentionally left blank.

<b>1</b>	<b>Introduction</b>	11
	The <b>cobas h</b> 232 meter .....	11
	Test principle .....	13
	Contents of the Pack .....	13
1.1	Important safety instructions and additional information.....	14
	Safety information.....	15
	Disposal of the system .....	17
	Battery pack.....	17
	General care.....	19
	Electrical safety .....	19
	Electromagnetic interference .....	20
	Touchscreen.....	20
	Local Area Network: protection from unauthorized access.....	20
	Wired network connection .....	21
	Wireless connectivity .....	22
	Radiofrequency radiation exposure information.....	22
1.2	Overview of the meter and its accessories.....	24
	Meter .....	24
	Power supply.....	27
	Test strip.....	29
	Handheld Base Unit.....	30
1.3	Overview of the Buttons and Icons used on Screen.....	31
<b>2</b>	<b>Putting the Meter into Operation</b>	35
2.1	Installing or replacing the battery pack.....	36
	Removing the battery pack .....	37
	Installing the battery pack .....	38
	Powering the meter on and off .....	41
<b>3</b>	<b>Meter Setup</b>	43
	Settings summary.....	44
3.1	<i>Basics</i> setup .....	49
	Contrast .....	49
	Language .....	51
	Setting the date.....	53
	Setting the time.....	55
	Setting the display options for date and time .....	57
	Sound.....	59
	Auto off .....	62
3.2	<i>Data Handling</i> setup .....	64
	Connection .....	64
	QR Code .....	65
	Computer.....	68
	Printer .....	69
	Result memory .....	71
	Result units.....	75
	Result display mode .....	78
	Diagnostics.....	80
3.3	<i>ID Setup</i> setting.....	82
	Administrator ID .....	83
	Operator ID.....	89
	Patient ID.....	92

## Table of Contents

3.4	Lockout setup .....	94
	Operator lockout.....	95
	Quality control (QC) settings .....	98
	Quality control (QC) lockout.....	100
	Instrument quality control (IQC) lockout.....	101
	QC result format.....	102
	Custom Range Troponin T.....	103
	Reset test parameters.....	105
	Cleaning lockout.....	106
	STAT test configuration .....	107
3.5	Optional Screens setup .....	109
<b>4</b>	<b>Performing a Test</b> .....	<b>113</b>
	Sample material .....	115
4.1	Preparing to test .....	115
	Code chip .....	116
	Inserting the code chip .....	117
	Test steps (overview).....	118
	Powering on the meter.....	119
	Logging in .....	120
4.2	Performing a test.....	123
	Inserting a test strip .....	127
	Displaying, confirming or adding comments to results.....	132
	Displaying the test result as QR code .....	135
	STAT tests .....	136
<b>5</b>	<b>Control Testing and Quality Control</b> .....	<b>137</b>
5.1	Preparing to run a quality control test .....	137
5.2	Performing a quality control test.....	139
	Quality control (QC) .....	139
	Displaying the QC result as QR code .....	146
	Instrument quality control (IQC).....	147
<b>6</b>	<b>Review Results</b> .....	<b>151</b>
	Reviewing test results.....	151
	Patient history.....	153
	All results.....	154
	Quality control (QC) results.....	155
	Instrument quality control (IQC) results .....	156
	Maintenance history .....	157
<b>7</b>	<b>Extended Functionalities</b> .....	<b>159</b>
	Data handling .....	159
	Computer (Setup option).....	160
	Operator lists.....	161
	Patient lists.....	162
	Barcode scanner.....	163
	Stored test results and comments .....	163



<b>8</b>	<b>Maintenance and Care</b>	165
8.1	Conditions for storage and shipping .....	165
	Storage .....	165
	Shipping .....	166
8.2	Cleaning and disinfecting the meter .....	167
	Difference between cleaning and disinfecting .....	167
	When should the meter be cleaned and disinfected? .....	167
	What to clean and disinfect? .....	167
	Recommended cleaning/disinfecting agents .....	169
	Cleaning/disinfecting the exterior (meter housing) .....	169
	Cleaning after contamination due to mispiping .....	170
	Cleaning the test strip guide cover .....	171
	Cleaning the visible area of the test strip guide .....	172
<b>9</b>	<b>Troubleshooting</b>	173
	Errors and unusual behavior without error messages .....	174
	Meter reset .....	176
<b>10</b>	<b>General Product Specifications</b>	177
10.1	Operating conditions and technical data .....	177
	Technical data .....	177
	Sample material .....	178
	Storage and transport conditions .....	178
10.2	Further information .....	178
	Ordering information .....	178
	Product limitations .....	180
	Information about software licenses .....	180
	Repairs .....	180
<b>11</b>	<b>Warranty</b>	180
<b>A</b>	<b>Appendix</b>	181
A.1	Working with barcodes .....	181
A.2	Operator and patient ID barcode masks .....	182
A.3	Example of barcode symbologies .....	183
A.4	Supported characters in 2D barcodes .....	185
<b>B</b>	<b>Appendix</b>	187
B.1	Option: Wireless network (WLAN) .....	187
	Preliminary note .....	187
	Background .....	187
	Technical implementation .....	188
	RF specific functionalities and effective performance claims .....	190
<b>C</b>	<b>Supplement for Observed Test Sequence</b>	193
	Observed Test Sequence (OTS) .....	193
	Using the OTS function .....	194
<b>D</b>	<b>Contact Roche</b>	197
	<b>Index</b>	199

This page intentionally left blank.

# 1 Introduction

## The **cobas h 232** meter

The **cobas h 232** meter is an instrument for the quantitative evaluation of immunoassays using the gold-labeling technique. The rapid diagnostic tests in strip format available for this meter support efficient diagnosis and assessment of cardiovascular diseases. The evaluation of these tests with the **cobas h 232** meter combines the advantages of a rapid diagnosis with enhanced clinical interpretation of quantitative values (in comparison with qualitative tests). In addition, automated evaluation provides more reliable results by eliminating the potential sources of error associated with visual reading. Refer to the package inserts accompanying the test strips for detailed information on specific tests.

Readings may be carried out directly where the blood samples are taken. Therefore, the **cobas h 232** meter is ideal for use at the point of care in emergency rooms, intensive care units and ambulances, as well as by cardiologists and general practitioners. The **cobas h 232** meter is rapid and easy to operate: Insert an unused strip in the meter and apply the sample. After the reaction period, the meter provides a quantitative result; in addition, a qualitative result is provided prior to the end of some tests.

The **cobas h 232** meter has the ability to connect to a data management system (DMS) by means of wireless communication (if the meter is equipped with WLAN functionality) or through the Handheld Base Unit from Roche Diagnostics (available separately). The **cobas h 232** meter supports data exchange via the POCT1A standard or directly via WLAN. Data management systems may have the ability to expand the security features of the meter, such as enabling operator lockouts. Data management systems may also enable data transfer to an LIS or HIS. Refer to the manuals of the Handheld Base Unit and of your DMS for technical details.

Read this operator's manual, as well as the package inserts for all relevant consumables, before using the system for the first time. You must configure the **cobas h 232** meter according to your needs before initial use. Refer to Chapter 3, "Meter Setup". Be sure to read the "Important safety instructions and additional information" section in this chapter before operating the system.

For all questions about the **cobas h 232** system that are not answered in this manual, contact your Roche representative (see "Contact Roche" on page 197). In order to expedite troubleshooting, please have ready your **cobas h 232** meter, its serial number, this manual, and all related consumables when you call.

If you connect your **cobas h 232** meter to a **cobas IT 1000** data management system or another PC/DMS, you will not be able to print directly from the meter to a printer. In order to print out meter data, use printers connected to the respective PC/DMS.

## Test principle

Two lines (signal and control line) in the detection zone of the test strip indicate whether the analyte to be determined is present in the sample material. These lines are detected by the **cobas h 232** meter with the help of an LED (lighting the detection zone) and a camera sensor (imaging the detection zone). The test signal (signal line) increases in intensity in proportion to the concentration of the respective analyte. Integrated system software converts the signal intensity to a quantitative result, which is then displayed on the screen at the end of the measurement.

The accuracy of the measurement is ensured through a simple principle: Every test strip box includes a code chip that contains all test and lot-specific information in electronic format. The test strips are labelled with a barcode on their underside and are hereby assigned to a specific code chip. When you insert a test strip from a new strip lot for the first time, the meter prompts you to plug in the corresponding code chip. The information is now read from the code chip and stored for future tests.

## Contents of the Pack

- **cobas h 232** meter
- Power adapter
- Universal Battery Pack
- Operator's manual in English
- CD-ROM with operator's manual in other languages

**Optionally available** (not included in the scope of delivery):

- Handheld Base Unit (docking station) for data transfer within a network or via USB (Universal Serial Bus)

For a personal printout of the operator's manual in your language, contact your local Roche organization (see Chapter A).

## 1.1 Important safety instructions and additional information

This section explains how safety-related messages and information related to the proper handling of the system are presented in the **cobas h 232** Operator's Manual. Please read these passages carefully.



---

The safety alert symbol alone (without a signal word) promotes awareness to hazards which are generic or directs the reader to related safety information.

---



---

Indicates a hazardous situation which, if not avoided, could result in death or serious injury.

---



---

Indicates a hazardous situation which, if not avoided, could result in minor or moderate injury.

---

**NOTICE**


---

Indicates a hazardous situation which, if not avoided, may result in damage to the system.


---

Important information that is not safety relevant is presented against a colored background (without a symbol). Here you will find additional information on correct use of the meter or useful tips.

Illustrations in this manual show two different kinds of hands:

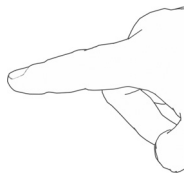

Hand without glove

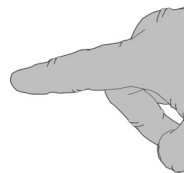

Hand with glove

## Safety information



---

### Operator qualification

Only trained healthcare professionals may operate the **cobas h 232** meter. Operators must also have received comprehensive instruction in the operation, quality control, and care of the **cobas h 232** meter.

---



---

### Protection against infection and blood-borne pathogens

Healthcare professionals using the **cobas h 232** meter to perform tests must be aware that any object coming into contact with human blood is a potential source of infection. Operators need to adhere to Standard Precautions when handling or using the **cobas h 232** meter. All parts of this system should be considered potentially infectious and are capable of transmitting blood-borne pathogens between patients and between patients and healthcare professionals.

- Use gloves. Wear a new pair of clean gloves for testing each patient.
  - Wash hands thoroughly with soap and water before putting on a new pair of gloves and performing the next patient test.
  - Dispose of used syringes, tubes, pipettes, test strips, and all other materials coming into contact with blood according to your institution's infection control policy.
  - Follow all health and safety regulations in force locally.
-



---

### **Allergy or injury caused by reagents and other working solutions**

Direct contact with reagents, detergents, cleaning/disinfection solutions, or other working solutions may cause skin irritation or inflammation.

- Always use protective gloves.
  - Observe the cautions given in the package inserts of the reagents and cleaning/disinfection solutions.
  - If a reagent, control, or cleaning/disinfection solution comes into contact with your skin, wash it off immediately with water.
  - Follow all health and safety regulations in force locally.
- 



---

### **Avoidance of electrical shock, fire, and explosions**

- Only use Roche Diagnostics original accessories (cables, power supply units, battery packs, and spare parts). Third-party cables, power supply units, and battery packs can cause the battery pack to explode or the meter to become damaged.
  - Do not use loose power sockets or damaged power supply units, cables, plugs, or battery packs.
  - Do not short circuit the power supply unit, the handheld base unit contacts, or the battery pack.
  - Do not drop the **cobas h 232** meter, the power supply unit, or the battery pack and protect these against shaking and vibrations.
-



## Disposal of the system



---

### Infection by a potentially biohazardous instrument

The **cobas h 232** meter or its components must be treated as potentially biohazardous waste. Decontamination (i.e., a combination of processes including cleaning, disinfection and/or sterilization) is required before reuse, recycling, or disposal.

Dispose of the system or its components according to the appropriate local regulations. Always remove the battery pack before thermal disinfection.

---

## Battery pack

The meter contains a rechargeable battery pack that begins charging as soon as the power adapter is connected or the meter is placed on an active Handheld Base Unit (i.e., one connected to a power adapter).

### NOTICE

---

Use only the specially designed battery pack provided by Roche Diagnostics. Using any other type of battery may damage the system.

---



---

### Possible hazards posed by the battery pack

Damaged or swollen battery packs can overheat, catch fire, or leak. Immediately cease use of **cobas h** 232 meters with damaged or swollen battery packs and under no circumstances recharge them (do not place in the Handheld Base Unit).

Overheating can cause the battery pack to catch fire or explode.

- Never throw the battery pack or the meters onto a fire. Do not dismantle, compress, or pierce the battery pack as this could cause an internal short circuit that leads to overheating.
- Do not place either the battery pack or the **cobas h** 232 meter on or in heating appliances, such as a microwave, conventional oven, or radiator.
- Avoid prolonged exposure to direct sunlight, e.g., when the meter is docked in the Handheld Base Unit. Keep this in mind when positioning the Handheld Base Unit.

Battery fluid or materials leaking from damaged battery packs can irritate your skin or cause burns due to high temperatures.

- Avoid contact with leaking battery fluid. In the event of accidental contact with the skin, rinse with water. If you get battery fluid in your eye(s), you should also seek medical attention.

Handle and dispose of battery packs with care.

---

Extreme temperatures reduce the charging capacity and usage period of the meter and the battery pack.

Observe the following general safety instructions for handling the battery pack:




---

### Disposal of used battery packs

Do not dispose of the battery pack with normal domestic waste. Dispose of used battery packs in accordance with applicable local regulations and directives and your facility's guidelines on the disposal of electronic waste equipment.

---

- When storing or disposing of the battery pack, use the manufacturer's original packaging.

Save or download data from the meter prior to replacing the battery pack to prevent loss of data (see Chapter 7).

- Always power the meter **off** before removing the battery pack.
- When the *Battery Low* warning is displayed, the meter must be returned **as soon as possible** to the Handheld Base Unit or connected to the power adapter for recharging.
- When the battery capacity is too low for further tests, the meter must be returned **immediately** to the Handheld Base Unit or connected to the power adapter for recharging.

### General care

#### NOTICE

---

Clean the meter only with the solutions recommended (see page 165). Using other solutions may result in incorrect operation and possible failure of the system. Make sure that the meter is thoroughly dried after cleaning and disinfecting.

---

### Electrical safety

#### NOTICE

---

Never run the meter if the electrical power adapter or the attached cable is visibly damaged. If there is any visible damage contact your local Roche service for inspections.

---

## Electromagnetic interference

The meter fulfills the IEC 61326-2-6 requirements for emitted interference and interference immunity.



---

Do not use the meter near strong electromagnetic fields, which could interfere with the proper operation of the meter.

Electrostatic discharges may cause malfunction of the meter.

---

## Touchscreen

### NOTICE

- 
- Use only your finger (even when wearing gloves) or special pens designed for use with handheld devices to touch the screen elements. Using pointed or sharp-edged objects can damage the touchscreen.
  - Do not use the system in direct sunlight. Direct sunlight may reduce the life expectancy and functionality of the screen.
- 

## Local Area Network: protection from unauthorized access

If this product is connected to a local area network, this network must be protected against unauthorized access. In particular, it must not be linked directly to any other network or the Internet. Customers are responsible for the security of their local area network, especially in protecting it against malicious software and attacks. This protection might include measures, such as a firewall, to separate the device from uncontrolled networks as well as measures that ensure that the connected network is free of malicious code.

## Wired network connection

If connected to a local area network, the **cobas h** 232 Handheld Base Unit must be protected against unauthorized access by means of a **strong password management**. Observe your own facility guidelines on password management where available, or apply the following rules:

### Characteristics of strong passwords

- Passwords must not contain the user's account name or parts of the user's full name that exceed two consecutive characters.
- Passwords must be at least eight characters in length.
- Passwords must contain characters from at least three of the following four categories:
  - English **uppercase alphabetic** characters (A through Z)
  - English **lowercase alphabetic** characters (a through z)
  - **Numeric** characters (0 through 9)
  - **Non-alphabetic** characters (for example, !, \$, #, %)

### Examples of weak passwords

- **uhxwze11** contains no upper case letter.
- **UHXW13SF** contains no lower case letter.
- **uxxxx7F** contains the same character more than four times.
- **x12useridF** contains a substring of the user ID longer than four characters.

To ensure that your **cobas h** 232 meter functions properly, observe the operating and storage conditions as given in the chapter "General Product Specifications", starting on page 177.

## Wireless connectivity

### If the meter is equipped with WLAN functionality:

Wireless connectivity allows the meter to send data (test results, patient IDs, operator IDs, etc.) to the data management system without the need to return the meter to the Handheld Base Unit. This feature must be configured by the system administrator. Observe the guidelines of your facility for using wireless local area network connections. For a description of the **cobas h 232** meter's ability to connect to Wireless Local Area Networks (WLAN, Wi-Fi), see appendix B.

## Radiofrequency radiation exposure information

### Glossary:

- "FCC" stands for "Federal Communications Commission" (USA).
- "RF" stands for "radio frequency"
- "RSS" stands for "Radio Standards Specification" (Canada).
- "WLAN" stands for "Wireless Local Area Network"

The Industrial, Scientific and Medical (ISM) radio frequencies may contain emissions from microwave ovens, heaters, and other noncommunication devices. While these types of devices usually pose no threat of interference as they are low-powered devices, the possibility exists that some industrial high power systems may wipe out any attempted communication use of a WLAN. Therefore, perform a site survey and interference analysis with a spectrum analyzer to view the entire spectrum, looking for signals that might not only be within the frequency range of the intended WLAN but also could be near or at the same frequency and cause interference.



Roche Diagnostics supports industry wireless standards and recommends using products that have Wi-Fi certification. This certification tests products to the 802.11 industry standards for basic connectivity, security, authentication, Quality of Service (QoS), interoperability and reliability. The Wi-Fi CERTIFIED logo is an assurance that the Wi-Fi Alliance has tested a product in numerous configurations and with a diverse sampling of other devices to ensure compatibility with other Wi-Fi CERTIFIED equipment that operates in the same frequency band. The Wi-Fi Alliance network of independent test labs conducts interoperability testing programs to ensure that wireless devices work together and support secure connections.

The **cobas h** 232 system complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance of 20 cm (8 inches) between the radiator and your body.

This transmitter must not be co-located or operated in conjunction with any other antenna or transmitter.

Changes or modifications made to this equipment not expressly approved by Roche Diagnostics may void the FCC authorization to operate this equipment.

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions:

(1) this device may not cause harmful interference,

and

(2) this device must accept any interference received, including interference that may cause undesired operation.

The **cobas h** 232 system complies with the emission and immunity requirements described in EN 61326-2-6. It has been designed and tested to CISPR 11 Class B.


This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by powering the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.


This Class B digital apparatus complies with Canadian ICES-003.

## 1.2 Overview of the meter and its accessories

### Meter

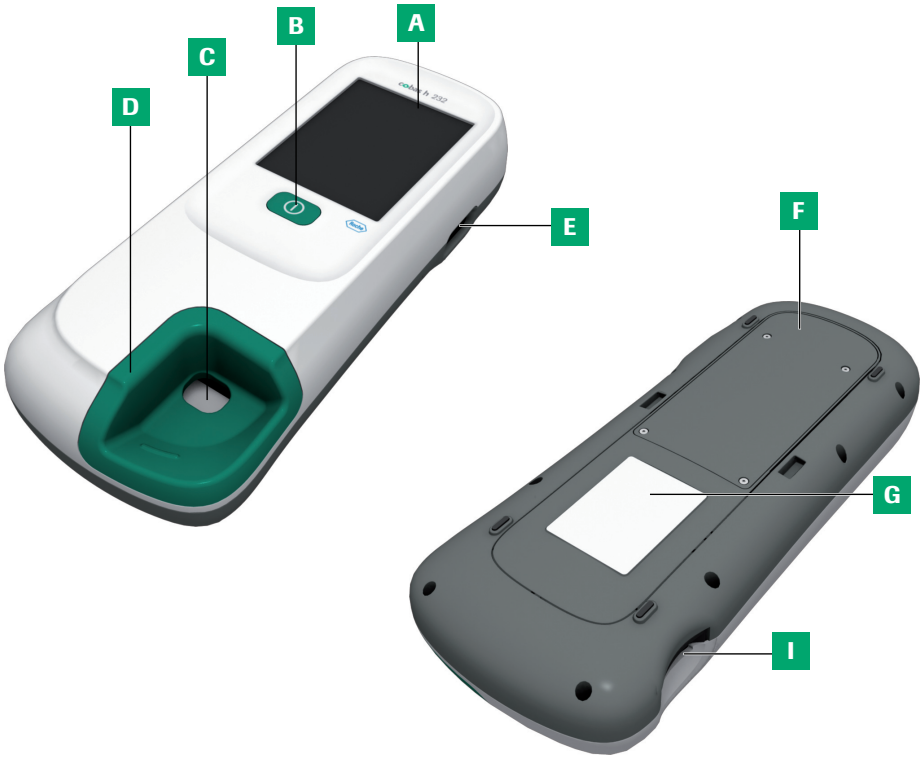

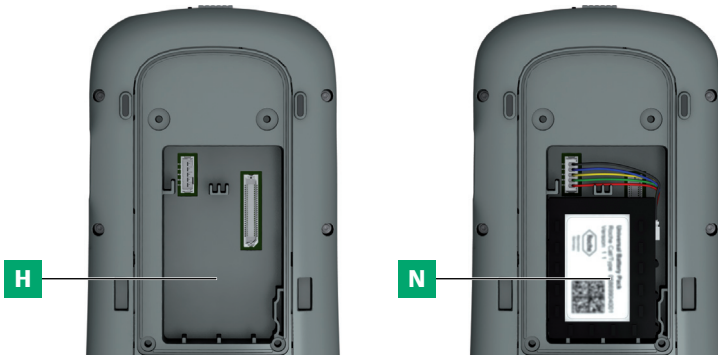

**A Touchscreen**

Shows results, information, icons and results saved in the memory. To select an option, simply touch the button lightly.

**B On/Off button**

Press this button to power the meter on or off.

**C Opening for sample application**

Opening in the test strip guide cover that enables you to apply blood once the test strip is inserted.

**D Test strip guide cover**

Remove this cover to clean the area underneath (if it has become soiled, e.g., with blood).

**E Barcode scanner (LED)**

Operator and patient IDs can be read into the meter using the integrated barcode scanner (only meter versions with a serial number greater than 200 000).

**F Battery compartment cover**


Remove to insert the battery pack.

**G Meter identification plate**

See page 5 for symbol explanation.

**H Wireless LAN label**

If the meter supports wireless connectivity: This label displays registration numbers that are specific to the meter RF hardware.

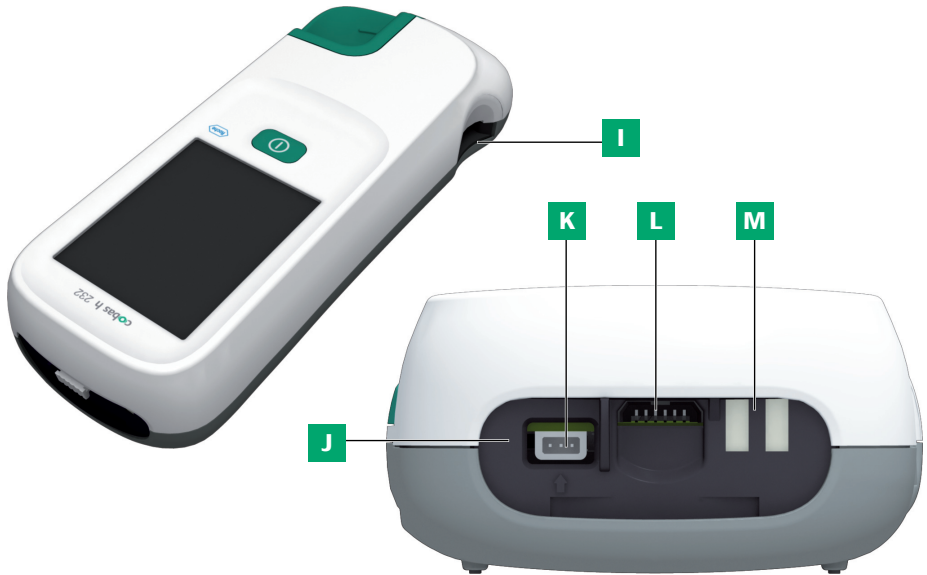

**I Test strip guide**

Insert the test strip here.

**J Infrared (IR) window**

Supports data communication with the (optional) Handheld Base Unit. Covered by the semi-transparent rear panel.

**K Connection socket for power adapter**

Plug in the power adapter here.


**L Code chip slot**

Insert the code chip here.

**M Charging terminals**

Used for power supply and/or charging the battery pack when the meter is docked in the (optional) Handheld Base Unit.

## Power supply

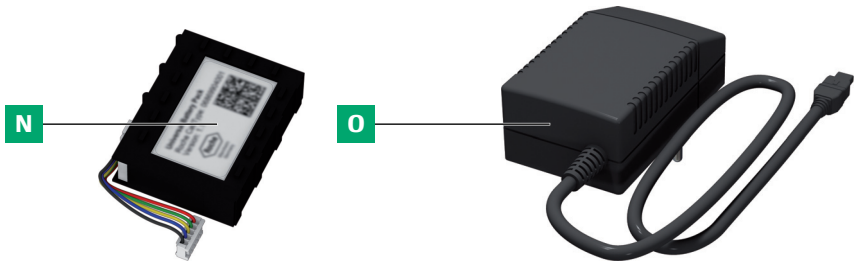

### **N Battery Pack**


Powers the device.

### **O Power adapter**

Powers the device and charges the battery pack.

The meter can be operated with the rechargeable battery pack only or together with the power adapter or the (optional) Handheld Base Unit, which both charge the battery pack when inserted. Insert the battery pack even when always using the power adapter or the Handheld Base Unit. This ensures that you will not lose the date and time settings if the power goes out. Results are retained in the memory together with the corresponding date and time, as well as all other settings, even when no battery pack is inserted.

To save power, the meter can automatically power itself off after a programmable period of time, if no buttons are pressed or new test strips are inserted. When the meter powers itself off, all test results obtained up to that point remain in memory and the settings are retained (see “Auto off” in the chapter entitled “Data Handling setup” on page 62).

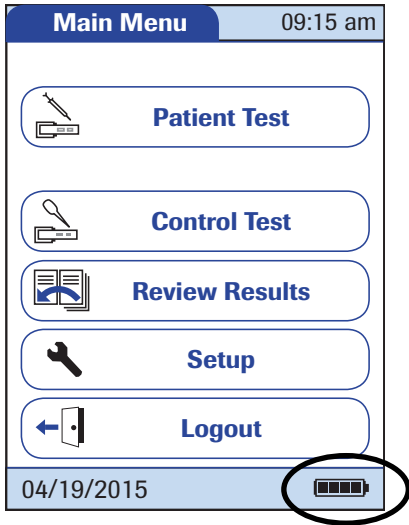

During battery operation, the meter always displays the power level of the battery pack. The battery icon is divided into four segments which correspond to the battery power level.

When **replacing the battery pack**, insert the new battery pack within 24 hours of removing the old one. Otherwise you may need to re-enter date and time.




---

Dispose of used battery packs in an environmentally responsible manner in accordance with applicable local regulations and directives. See "Infection by a potentially biohazardous instrument" on page 17.

---

## Test strip

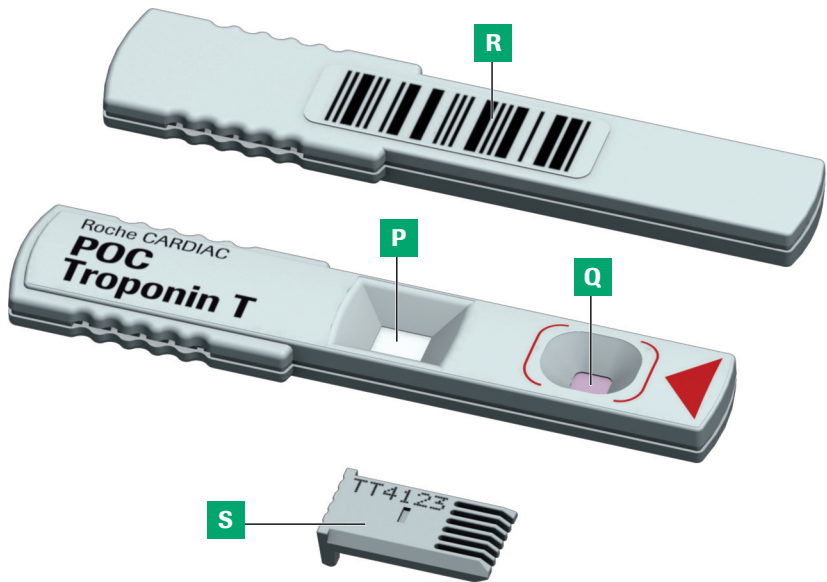

**P Test area**

This area is evaluated by the meter via the camera.

**Q Sample application area**

The sample is applied to this area after inserting the test strip in the meter.


**R Barcode**

Assigns the strip to the corresponding code chip. The barcode is automatically read by the meter when the strip is inserted into the test strip guide.

**S Code chip**

Contains strip lot specific data.

## Handheld Base Unit

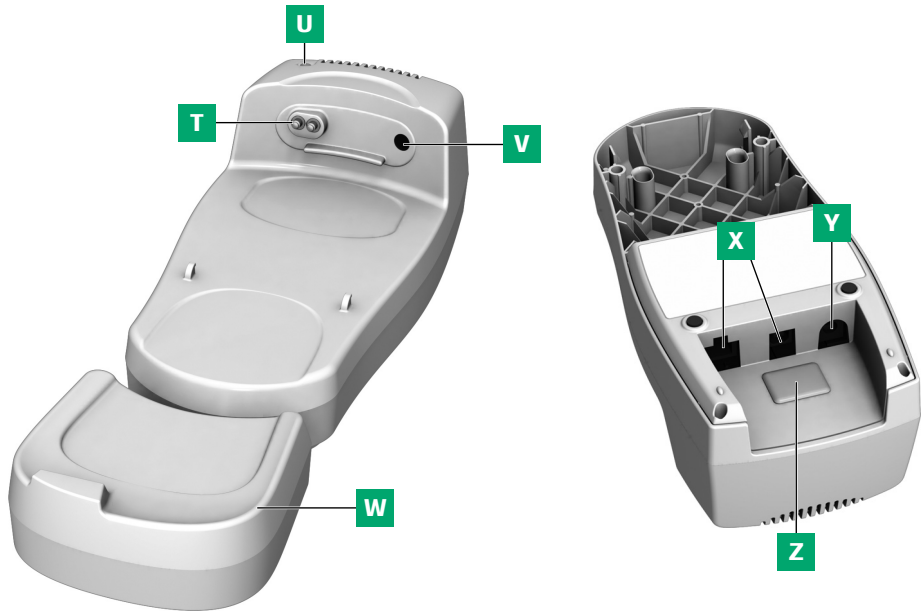

**T Charging contacts**

Used for power supply and/or charging the battery pack.

**U Status indicator**

Lights up when power is connected, charge indicator.

**V Infrared (IR) window**

For communication with the meter.

**W Extension piece**

For **cobas h 232** meter.

**X Data ports (Ethernet/RJ45 and USB)**

For connecting the device to a Data Management System (DMS).

**Y Connection socket for the power adapter**

Here you can plug in the power adapter of the Handheld Base Unit.



















**Z Removable cover for configuration switch**







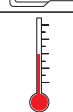


The switch sets the mode of operation for the Handheld Base Unit.

The Handheld Base Unit can be ordered separately. For detailed information on usage and configuration please consult the operator's manual of the Handheld Base Unit and the Technical Note stored on the Handheld Base Unit itself.

### 1.3 Overview of the Buttons and Icons used on Screen





The buttons and icons that appear during normal operation are shown here, along with a general explanation. Error messages and the description of the icons linked to them are provided in a separate chapter. See “Troubleshooting” starting on page 173.

Button/Icon	Meaning
	OK, save setting
	Cancel, discard setting
	Return (to previous menu)
	Decrease/increase a numeric value or Scroll through lists that are too long to be displayed at once
	Inactive button; value cannot be further decreased/increased or End of list in this direction is reached
	Return to the <i>Main Menu</i> screen
	List of tests of an individual patient
	Scroll through stored results
	Print displayed result (via infrared interface to corresponding printer)
	Display test result as QR code
	Add a comment
	Operator logout
	Operator login
	Operator must wait until the meter has completed an action.
	Insert test strip
	Remove test strip
	Test strip warming up
	Apply sample (the time left to apply sample is counted down in the screen alongside the required sample amount)

Button/Icon	Meaning
	Apply QC sample (the time left to apply sample is counted down in the screen alongside the required sample amount)
	Insert code chip
	Open test strip guide cover for cleaning
	Battery status: <ul style="list-style-type: none"> <li>▪ If the battery pack still has its full charge, all segments are lit</li> <li>▪ Individual segments disappear one by one as the battery pack becomes weaker</li> <li>▪ When there is no segment remaining, you can no longer perform a test. You can, however, still access the meter's memory</li> </ul>
	Operation with power adapter
am	Time between midnight and noon (in 12-hour time format)
pm	Time between noon and midnight (in 12-hour time format)
	Marks an information message
	Marks an error message or a warning (see Chapter 9 "Troubleshooting")
	Room or meter temperature is outside the acceptable range.
	The test strip guide cover is open.
<b>QC!</b>	In a QC lockout condition, this icon indicates that one or more parameters are locked, while others still can be measured.



The following icons may appear when using the meter in conjunction with a data management system (DMS).

	Infrared interface is enabled (for communication with the computer and/or printer)
	Communication is taking place via WLAN
	An OTS request is pending
	Cleaning/Disinfection necessary

This page intentionally left blank.

## 2 Putting the Meter into Operation

Before using the meter for the first time, perform the following steps:


- 1 Install the battery pack (see page 38)
- 2 Connect the power adapter to charge the battery pack
- 3 Set the current date and time as well as the appropriate display format (see Chapter “Meter Setup” starting on page 43)
- 4 Enter the settings of choice (language, quality controls – where necessary, user administration, etc.)

If the meter has **no date/time settings** (either because you are powering on the meter for the first time or because the battery pack was removed from the meter for more than 24 hours), you cannot perform a test. In that case powering on the meter takes you immediately to the *Setup* mode, where you must set the date and time.

## 2.1 Installing or replacing the battery pack


When shipped, the battery pack is not installed in the meter.

Unused battery packs lose their charge over time and have to be recharged before they can be used. After installing a new battery pack, the meter should be charged overnight before testing. Please note that the battery pack will only reach full capacity once it has been fully emptied and charged several times.

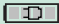Whenever the meter is placed on an active Handheld Base Unit or powered by the power adapter, the  icon is displayed. This icon shows that power is available and the meter can be charged if necessary.

Make sure that the permitted temperature range for charging the battery pack (12-32 °C or 54-90 °F) is maintained during installation and initial setup.

## Removing the battery pack

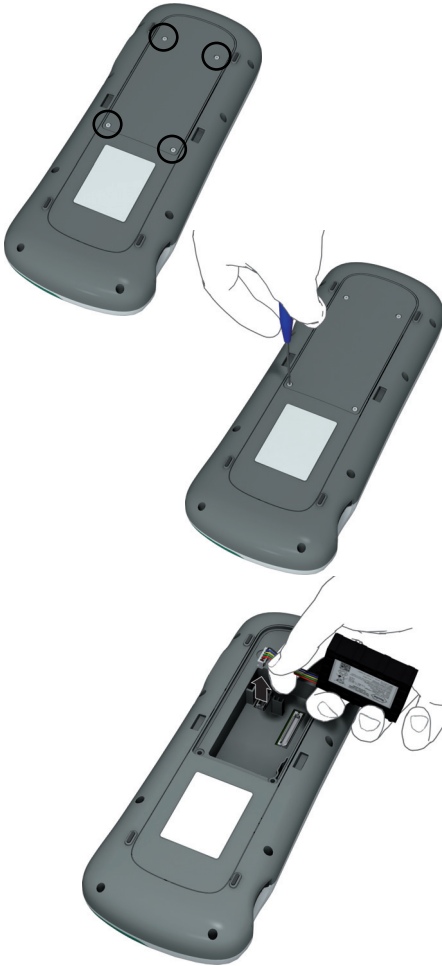

- 1 If a battery pack is already installed, make sure that the meter is powered off.
- 2 Place the meter face down on a level surface.
- 3 Using an appropriate starshaped screwdriver, e.g. Torx<sup>®</sup> size T5, remove the four screws holding the battery compartment cover in place.
- 4 Remove the battery compartment cover from the meter. The battery pack now visible is connected to the meter by a plug.
- 5 Carefully lift the battery pack and remove the plug connector.





---

### Disposal of used battery packs

Do not dispose of the battery pack with normal domestic waste. Dispose of used battery packs in accordance with applicable local regulations and directives and your facility's guidelines on the disposal of electronic waste equipment.

---

## Installing the battery pack

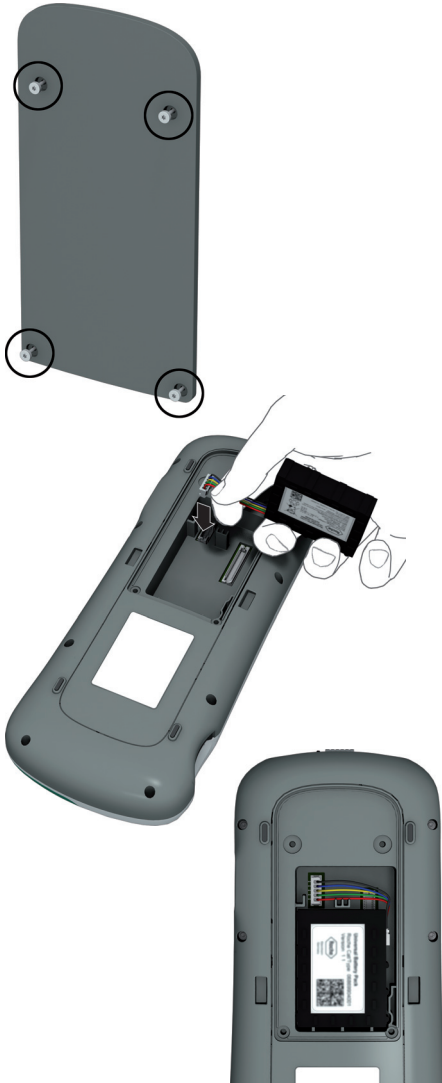

- 1** Loosen the screws on the battery compartment cover until they are protruding about 4-5 mm (2/10 in).
- 2** Hold the battery pack in your hand, with the wires and the plug pinched between your thumb and index finger.
- 3** Plug the connector plug into the socket and make sure it is fully inserted.
- 4** Place the battery pack inside the battery compartment as shown.

To position the battery pack correctly, always align the ridges on the side of the battery pack with the ridges on the inside of the battery compartment.





**5** Place the cover on the battery compartment. Make sure that the plug connector wires do not get pinched between meter and cover.

**6** Tighten all four screws until snug (do not overtighten).



After inserting a new battery pack, the meter powers on automatically.

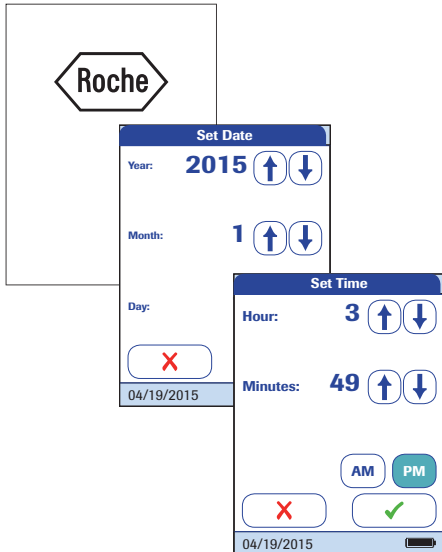

- The Roche logo is displayed. If the meter does not power on automatically, the battery pack may be empty. Connect the power adapter for a minimum of 30 minutes, then remove the plug and try to power the meter on. If it powers on, the battery pack is charging properly.
  - Within a short period of time, the start screen should appear.
  - If the meter has been without power for too long, it will show the date and time settings dialog and you have to re-enter the date and time settings.
  - The screens for entering the date and the time appear.
- 7** Enter the date and time. After you have entered the correct information, confirm each screen with .

After installing a new battery pack, the meter should be charged overnight before testing.



## Powering the meter on and off



- 1** Power the meter on by pressing the **ⓘ** button for approximately 1 second.

You can also power on the meter directly by connecting the power adapter or by placing the meter on the Handheld Base Unit.

- 2** To power the meter off after use, press the **ⓘ** button for approximately 1 second.

This page intentionally left blank.


### 3 Meter Setup

Buttons are screen prompts that cause something to happen when touched. In this manual the names of all buttons are either shown as **bold** text or as the icon used on the button (e.g., ✓ for **OK** ).

When text refers to other screen elements (e.g., Menu titles), these are written in *italics> . These screen elements are not active.*



You can open any displayed function by touching (or tapping) the button for it with your finger (or a special pen for this purpose). “Tap” means: Touch the button, then remove your finger from the touchscreen. The next screen appears once you remove your finger.

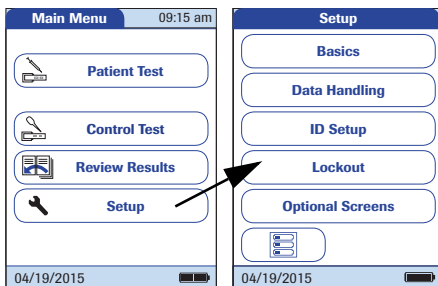


- 1 Touch **Setup** to call up the *Setup* menu.
- 2 Select the group of settings of choice (see the Settings summary following this section).

For a description of the buttons and icons used on screen see page 31.

### Settings summary

The diagram below gives an overview of the setup areas that can be accessed on the meter.

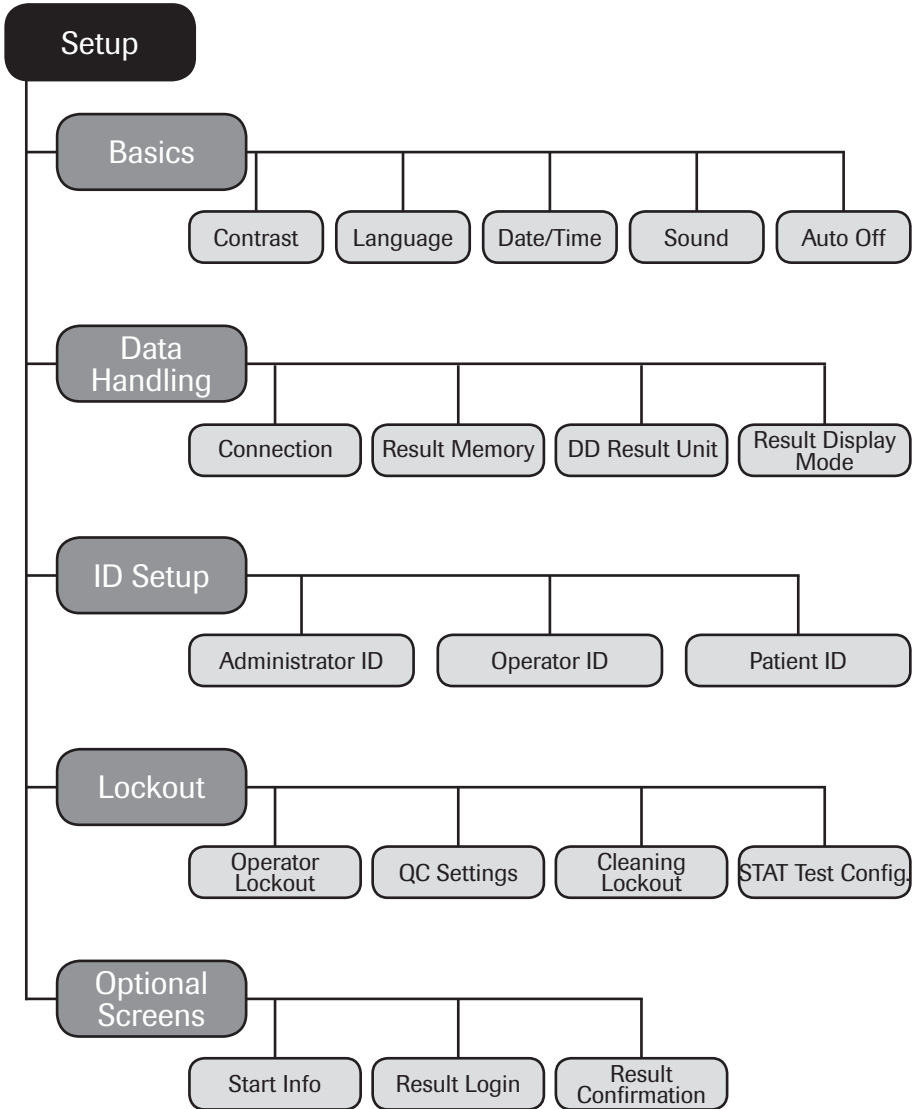

Group	Subgroup	Setting	Values *
Basics	Contrast		0 - 10 (5*)
	Language		Dansk
			Deutsch
			English *
			Español
			Français
			Italiano
			Nederlands
			Norsk
			Português
			Svenska
		An installable language	
Date/Time	Date	Date	01/01/2011 *
		Time	12:00 am *
	Date formats		Day.Month.Year (31.12.2011)
			Month/Day/Year (12/31/2011) *
			Year-Month-Day (2011-12-31)
	Time formats		24-hour time format (24H)
		12-hour time format (12H), with am/pm *	
Sound (Beeper)	Volume		Off
			Low
			Medium *
			High
	Key Click		Enable
			Disable *
Auto Off	[minutes]		Off
			1 ... 10 (default: 5 min *)
			15, 20, 25, 30
			40, 50, 60

\* Default settings are labelled with an asterisk (\*).

Group	Subgroup	Setting	Values *	
<b>Data Handling</b>	Connection		QR Code	
			Off *	
			Computer	
			Printer	
	Result Memory	Result Display Filter		All results *
				Current Op. Res.
		Result Storage Mode		No results deletion *
				Delete oldest result
	Result Unit	Select DD Res. Unit		µg/mL *
				ng/mL
				mg/L
				µg/L
		Select TT Res. Unit		ng/L *
			pg/mL	
	ng/mL			
		µg/L		
Result Display Mode			Static *	
			Flashing	
<b>ID Setup</b>	Administrator ID		Blank (off) *	
	Operator ID	(with DMS and list available on meter)		None *
				List
				Hidden List
		(No list available on meter)		None *
				Optional
				Required
	Patient ID			Scan Only
				None
				Optional *
			Required	
			Hidden List	

\* Default settings are labelled with an asterisk (\*).

Group	Subgroup	Setting	Values *	
Lockout	Operator Lockout (only if "Operator ID" option is enabled)		No *	
			Daily	
			Weekly	
			Monthly	
			Every 3 months	
			Every 6 months	
			Yearly	
		QC Settings	QC Lockout	New Lot: Yes/No *
				No *
				Daily
				Weekly
				Monthly
			IQC Lockout	No *
				Daily
				Weekly
				Monthly
			QC Result Format	Pass/Fail *
				Value
				Value & Pass/Fail
			Custom Range Trop T	Default Range*
				Custom Range
	Reset Test Param.	Do Reset		
		No Reset *		
	Cleaning Lockout		No *	
			Daily	
			Weekly	
			Monthly	
	STAT Test Config.		Enable	
			Disable *	

\* Default settings are labelled with an asterisk (\*).

Group	Subgroup	Setting	Values *
Optional screens	Start Info		Enable
			Disable *
	Result Login		Enable
			Disable *
	Result Confirmation		Enable
			Disable *

\* Default settings are labelled with an asterisk (\*).




### 3.1 *Basics* setup


The *Basics* setup area contains the basic options for changing the user interface.



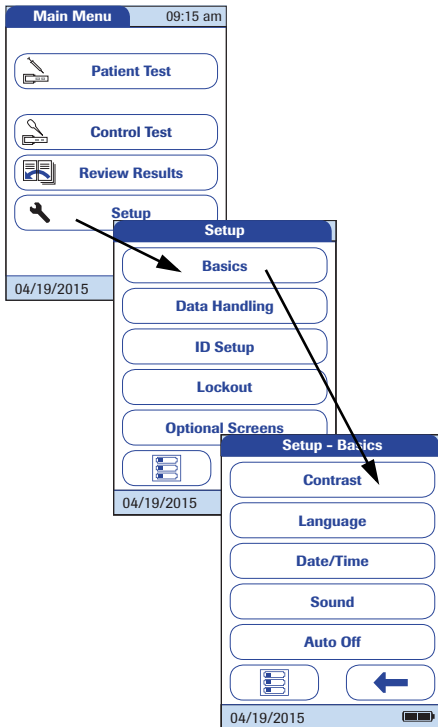

#### Contrast

Use the *Contrast* menu to adjust the display to your ambient light conditions and make it easier to read.

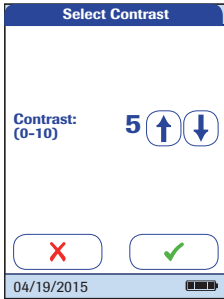


- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Basics**.
- 3 From the *Setup-Basics* menu, touch **Contrast**.



- 4 Touch  or  to change the contrast in a range from 0 to 10.
  - Contrast “0” makes the screen very dark.
  - Contrast “10” makes the screen very light.
- 5 Touch  to save this setting, or touch  to exit this menu without saving any changes. The display automatically returns to the previous screen.

## Language

Use this setting to select the language for all displays (that contain text).

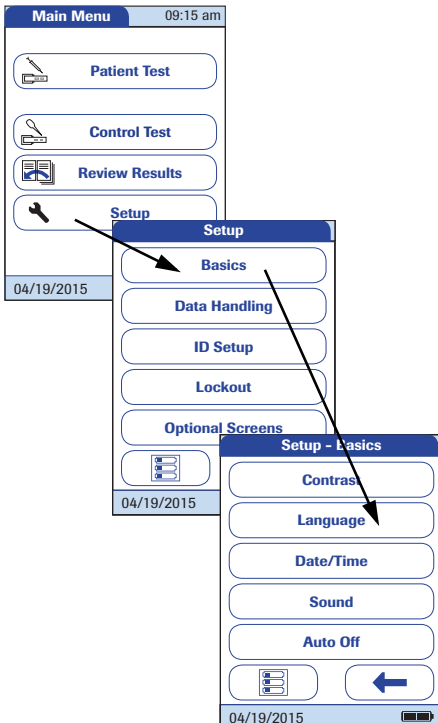




- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Basics**.
- 3 From the *Setup-Basics* menu, touch **Language**.



The current language setting is highlighted (white type on a blue background). You can select either:

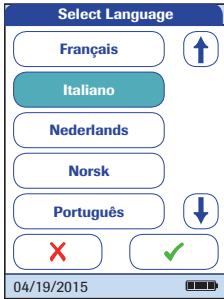

- Dansk
- Deutsch
- English
- Español
- Français
- Italiano
- Nederlands
- Norsk
- Português
- Svenska

An optional language can be installed upon request (either directly at the plant or later by an authorised Roche Diagnostics service technician).




- 4 Touch  or  to display the language of choice on the screen.

If the arrow is just an outline  , you have reached the end of the list in the respective direction.


- 5 Touch the button to select the language of choice. Your selection is now highlighted.
- 6 Touch  to save this setting, or touch  to exit this menu without saving any changes. The display automatically returns to the previous screen.





## Setting the date

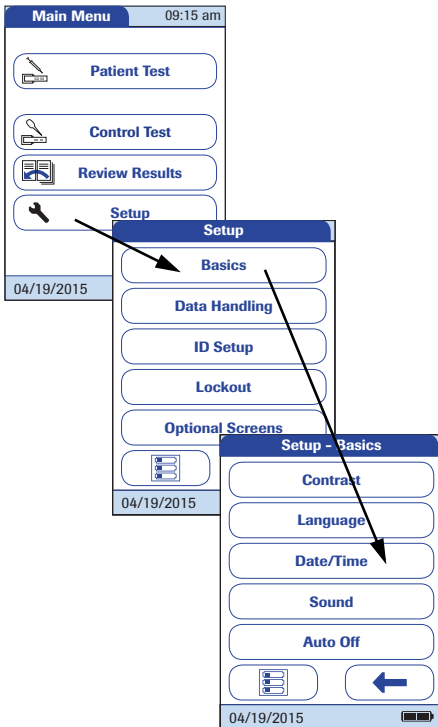
Use this menu to set the date of the meter.

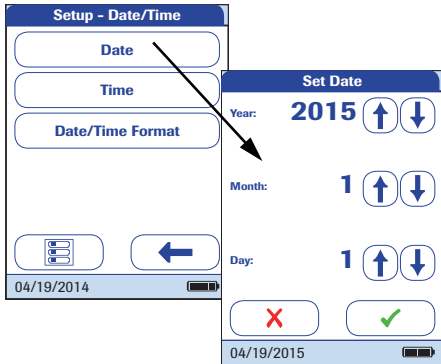


- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Basics**.
- 3 From the *Setup-Basics* menu, touch **Date/Time**.

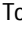


- 4 From the *Setup-Date/Time* menu, touch **Date** to set the date.
- 5 Touch  or  to set the year, then the month, then the day.
- 6 Touch  to save this setting, or touch  to exit this menu without saving any changes. The display automatically returns to the previous screen.

If the *Set Date* menu appeared automatically after powering the meter on, you **must** touch  to complete the date setting. The *Set Time* menu will then be displayed next.


- 7 In the *Setup-Date/Time* menu, touch  to return to the *Setup-Basics* menu or touch **Time** to move to the time setting.



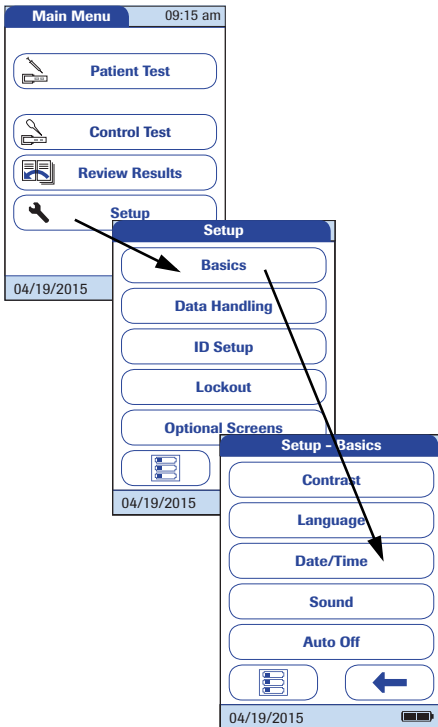

## Setting the time


Use this menu to set the time of the meter.

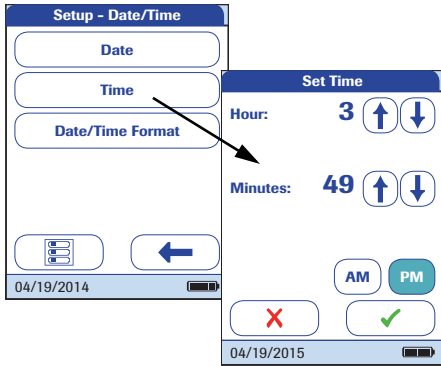


- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Basics**.
- 3 From the *Setup-Basics* menu, touch **Date/Time**.



- 4 From the *Date/Time* menu, touch **Time** to set the time.
- 5 Touch  or  to set the hours, then the minutes.
- 6 Touch  to save this setting, or touch  to exit this menu without saving any changes. The display automatically returns to the previous screen.


If the *Set Time* menu appeared automatically after leaving the *Set Date* menu, you **must** touch  to complete the time setting. The *Main Menu* will then be displayed next.

- 7 Touch  to return to the *Setup-Basics* menu or touch **Date/Time Format** to move to the display options.




## Setting the display options for date and time

Select your preferred format for the date and time display.

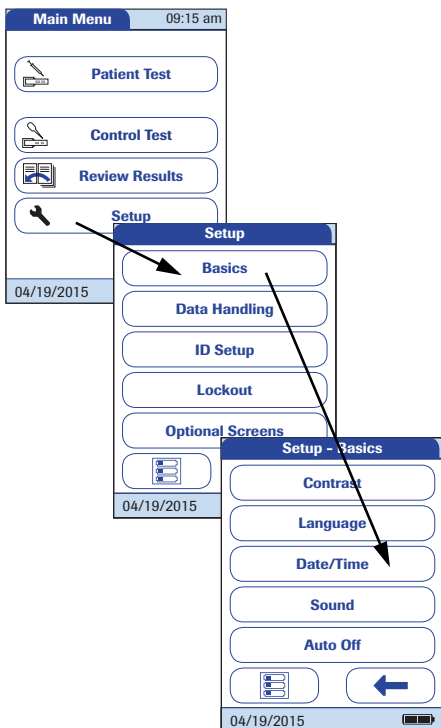

- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Basics**.
- 3 From the *Setup-Basics* menu, touch **Date/Time**.

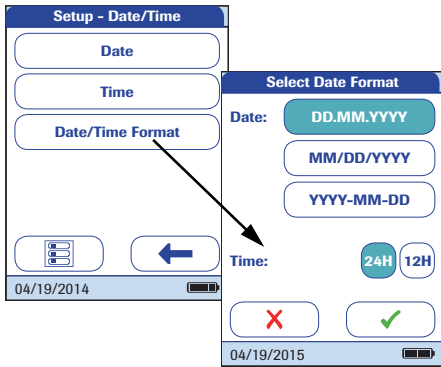




- 4 From the *Setup-Date/Time* menu screen, touch **Date/Time Format** to set the display format.

The current settings are highlighted (white type on a blue background). You can select one of the following display formats for the date:

- Day.Month.Year, e.g., 31.12.2011
- Month/Day/Year, e.g., 12/31/2011
- Year-Month-Day, e.g., 2011-12-31

You can also select one of the following display formats for the time:

- 24H (24-hour time format), e.g., 14:52
- 12H (12-hour time format, supplemented by am/pm), e.g., 2:52 pm

- 5 Touch the button with the display format of choice for the date and time. Your selection is now highlighted.
- 6 Touch  to save this setting, or touch  to exit this menu without saving any changes. The display automatically returns to the previous screen.
- 7 Touch  to return to the *Setup-Basics* menu.


## Sound

The **cobas h 232** meter can display information visually and alert you to special circumstances with a beep sound. When the *Sound* is activated, the meter beeps when:


- it is powered on
- it detects a test strip
- pre-heating of the test strip is complete and you need to apply a sample
- the test is completed and the results are displayed (a long beep)
- an error occurs (three short beeps)
- the power adapter is connected when the meter is on
- a barcode is scanned successfully (short beep)
- the barcode scanner is disabled (two short beeps)
- a positive test result can be expected, while the measurement still is in progress (a long beep)

We recommend that you keep the *Sound* (beeper) activated at all times.

You can also activate a *Key Click*. When the *Key Click* is activated, the meter clicks briefly every time a button is touched, facilitating the input of information.

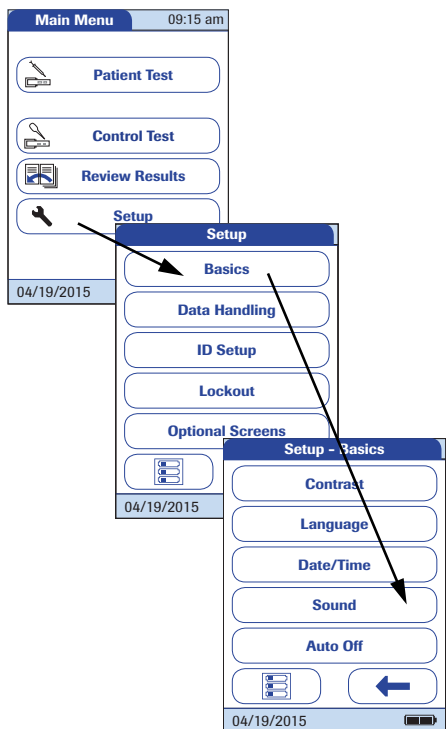

- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Basics**.
- 3 From the *Setup-Basics* menu, touch **Sound**.

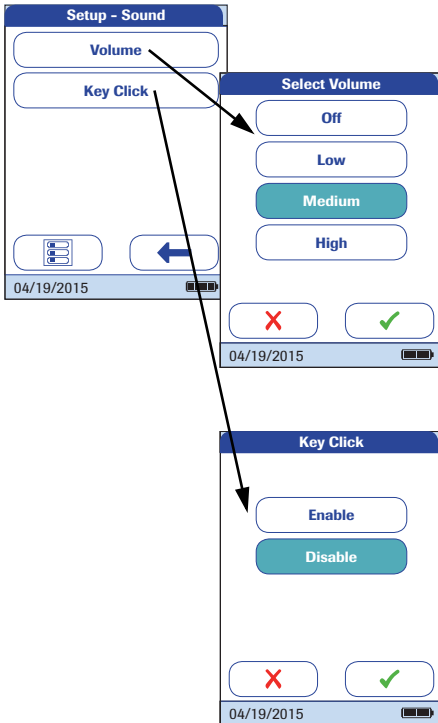

- From the *Setup-Sound* menu, touch **Volume** to set the volume level of the beeper.

The current setting is highlighted (white type on a blue background). You may select from the following options:

- Off
- Low
- Medium
- High

- Touch **Key Click** in the *Setup-Sound* menu to turn the key click on or off.

You may select from the following options:


- Enable
- Disable

- Touch **checkmark** to save this setting, or touch **X** to exit this menu without saving any changes. The display automatically returns to the previous screen.

- Touch **blue arrow** to return to the *Setup-Basics* menu.

## Auto off

You can set up your meter so that it powers itself off automatically if it has not been used (no buttons touched or tests run) for a preselected time period. Use this feature to save power and extend the life of the batteries.

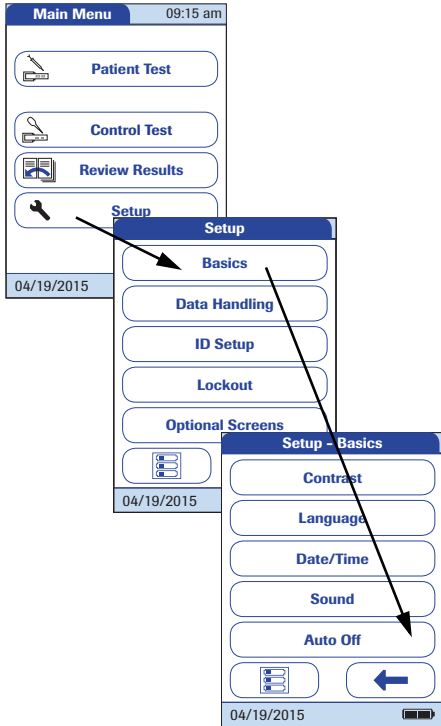


- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Basics**.
- 3 In the *Setup-Basics* menu, touch **Auto Off**.






If the meter is connected to the power adapter or the Handheld Base Unit, the *Auto Off* function has a different effect:

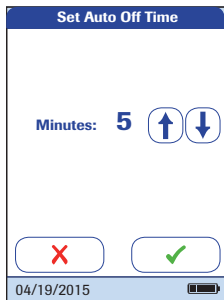- If you do not work with operator IDs, the meter switches to the *Main Menu*, once *Auto Off* is triggered.
- If you work with operator IDs, the meter switches to *Operator Login*, once *Auto Off* is triggered.

For information on *Operator ID*, see “Operator ID” on page 89.




You may select from the following options:

- Off (meter never powers itself off)
  - Time until meter powers itself off:  
1...10, 15, 20, 25, 30, 40, 50, 60 minutes
- 4 Touch  or  to select the time of choice in minutes or to deactivate the feature.
  - 5 Touch  to save this setting, or touch  to exit this menu without saving any changes.  
The display automatically returns to the previous screen.
  - 6 Touch  to return to the *Setup* menu.

## 3.2 Data Handling setup

### Connection


In the *Connection* menu you can configure the data exchange with external devices. The meter can be connected either to a computer or a printer. In addition to this direct data output, test results may also be encoded as QR codes, which can be scanned to be used with other applications.

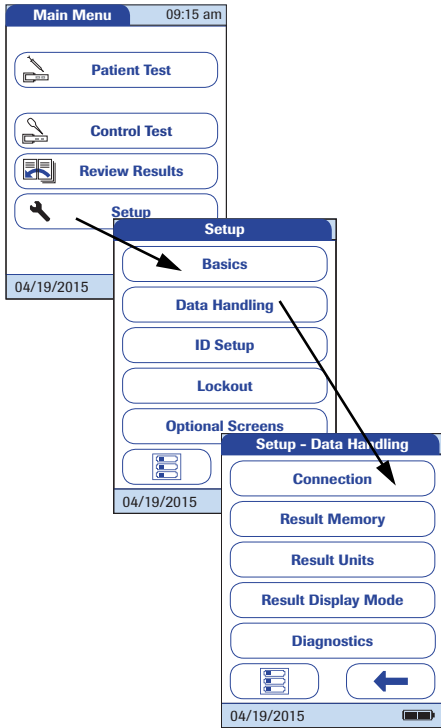

- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Data Handling**.
- 3 From the *Setup-Data Handling* menu, touch **Connection**.




## QR Code

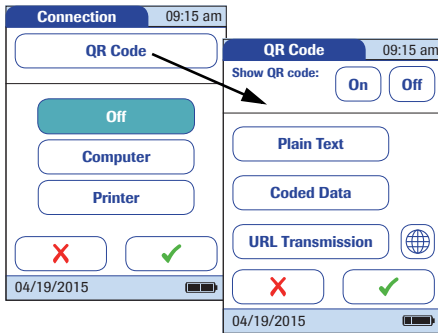

- 4 From the *Connection* menu, touch **QR Code**.
- 5 Touch **On** to enable, or touch **Off** to disable QR code display. Your selection is now highlighted.
  - If you have enabled this feature, select the style of the QR code to be displayed:
    - *Plain Text*  
Use this option to scan the result into a PC hosted application (e.g. electronic medical record). In order to do so, you need an external 2D barcode scanner attached to your PC. Once scanned, the test result and related meta-information (e.g. date, time, comments) will appear as plain text in your PC application.
    - *Coded Data*  
Use this option to scan the result into a smartphone or tablet app, or a PC hosted application.
    - *URL Transmission*  
Use this option to upload the result to a web based service (e.g. such as an electronic health record, etc.). For this option further parameters (URL, encryption key) must be set.

Ensure that your environment for reading QR codes (QR code reader, operating system, text processing application) is appropriate for your language. Disregard may lead to unpredictable behavior of your receiving component (PC, mobile device).


For the options *Coded Data* and *URL Transmission* dedicated software programs or apps are required in order to use this feature. If you are a customer or a 3rd party IT provider, and you are interested to use this feature, contact your local Roche Diagnostics representative for additional information.

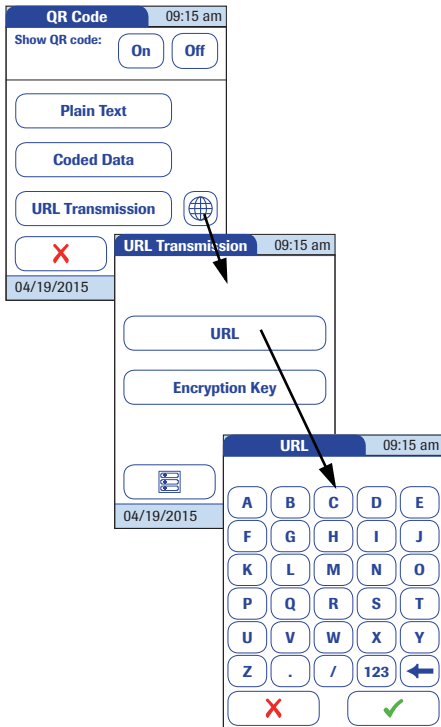

- 6 Touch **Plain Text**, **Coded Data** or **URL Transmission**, depending on the intended use. Your selection is now highlighted.

If you have selected *URL Transmission*, the globe button will become active. Use this button to set the additional parameters.

- 7 Touch globe to open the *URL Transmission* menu.
- 8 Touch **URL** to enter the URL to which the test result data will be uploaded (note: the URL will be provided by your specialized service provider).
  - Use 123 to switch to input of numbers.
  - Use ABC to switch back to input of text.
  - Use ← to backspace and correct a mistake.
- 9 Touch ✓ to save this setting, or touch ✗ to exit this menu without saving any changes. The display automatically returns to the previous screen.

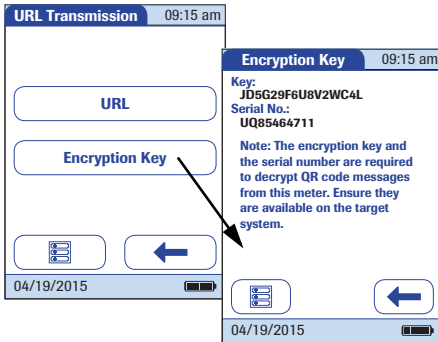




- 10** Touch **Encryption Key** to display the *Encryption Key* menu.

The *Encryption Key* menu displays two types of information which are required in order to identify and decode the transmitted test result. This information is required by the IT system, to which the test result is transmitted via the defined URL.

- *Key*  
This key is auto-generated by the meter and is required to decrypt the QR code information.
- *Serial No.*  
The meter's serial number is required to map the test result to the encryption key.

Touch the **New** button, if you want the meter to generate a new encryption key.

- 11** Touch  twice to return to the *QR Code* menu.
- 12** Touch  to save this setting, or touch  to exit this menu without saving any changes. The display automatically returns to the previous screen.

The *Encryption Key* information needs to be exchanged once with your specialized service provider prior to using the URL transmission method. Contact your specialized service provider in order to exchange this information.

## Computer


The **cobas h 232** meter can connect with a computer or host system running appropriate software (that is, a DMS must be installed). To use this connectivity feature, however, you need the optional Handheld Base Unit or the meter must be configured for wireless communication. If no wireless communication is configured, the connection is established in two steps.



- The meter connects to the Handheld Base Unit via infrared.
- The Handheld Base Unit is either connected to a single computer (via USB) or to a network/host system (via Ethernet).

The option *Computer* (when activated) can be used together with a DMS to set up:

- operator lists, or
- patient lists (lists of patients to be tested)

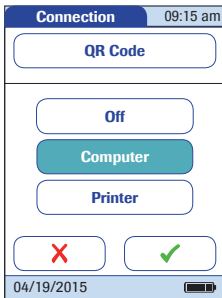This eliminates the need for manual entry of these data. In addition, you can transfer test results stored in the meter to other systems for archiving or further evaluation. The option *Computer* controls the meter's ability to communicate with a computer or a network.



- 13 From the *Connection* menu, touch **Computer**. Your selection is now highlighted.
- 14 Touch  to save this setting, or touch  to exit this menu without saving any changes. The display automatically returns to the previous screen.

Extended data handling functionality is dependent on the capabilities of the particular Data Management System (DMS) being used and may vary.


## Printer

The meter can connect directly with three different infrared printers. The Handheld Base Unit cannot be used for this purpose.


The option to print is displayed in a test result as well as directly after a test and when calling up stored results. Using the settings you enter here, you can activate or deactivate the meter's ability to print.

Connection to a printer can only be established via infrared.


Enabling the connection to a printer disables the connection to a computer (and vice versa).

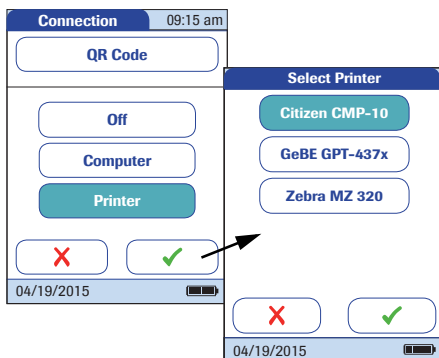

- 15 From the *Connection* menu, touch **Printer**. Your selection is now highlighted.
- 16 Touch ✓ to confirm the selection. The next screen allows you to choose the type of printer you are using.
- 17 Touch ✓ to save this setting, or touch ✗ to exit this menu without saving any changes. The display automatically returns to the previous screen.

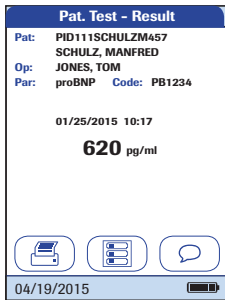

To print :

- Align the meter with the IR printer
- At any test or memory screen, touch .


The printer icon only appears if the printer function is activated. Otherwise it is not displayed.

Note: (Only applies if you are working with the meter in a language other than English.) With the exception of information you have entered (such as patient ID and name, operator ID, comments), the printout will be in English.


## Result memory

Result memory settings allow to apply a *Result Display Filter* and to set the *Result Storage Mode* (see page 73).

All results recorded by the meter (patient results and quality controls) are stored automatically. Use the *Sel. Res. Display Filter* menu to select whether to display results (when calling up the *Memory* function) for all existing tests or only those from the current operator.

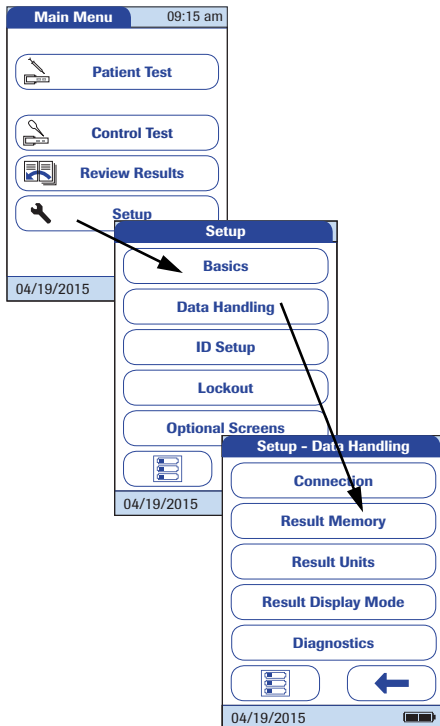

- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Data Handling**.
- 3 From the *Setup-Data Handling* menu, touch **Result Memory**.

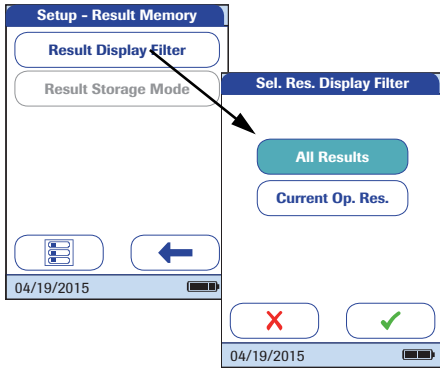

- 4 From the *Setup-Result Memory* menu, touch **Result Display Filter** to select the results to display.

The current setting is highlighted (white type on a blue background). You may select from the following options:

- **All Results**
- **Current Op(erator) Res(ults)**

- 5 Touch **✓** to save this setting, or touch **X** to exit this menu without saving any changes. The display automatically returns to the previous screen.
- 6 Touch **←** to return to the *Setup-Data Handling* menu.



The *Sel. Res. Storage Mode* menu tells the meter what to do when the memory is full.


The meter memory can store a maximum of 2000 patient tests, 500 liquid controls, and 200 instrument quality controls.

In case of a full memory, you can choose between:

- *No Result Deletion*. This prevents data from being deleted inadvertently, but may (in case of a full memory) lead to the situation that you cannot perform a new measurement. Further measurements will only be possible if stored data are transferred to the data management system (DMS) or you allow automatic deletion.
- *Del. Oldest Result*. Based on the 1st in/1st out principle, the oldest result will be automatically deleted when necessary. This is the only option if the meter is used without a DMS (i.e. no data will be transferred and archived).

The option *No Result Deletion* is available only when you are using a DMS. For more details see “Extended Functionalities”, starting on page 159. If you are not working with a DMS, this option is set to *Del. Oldest Result*.

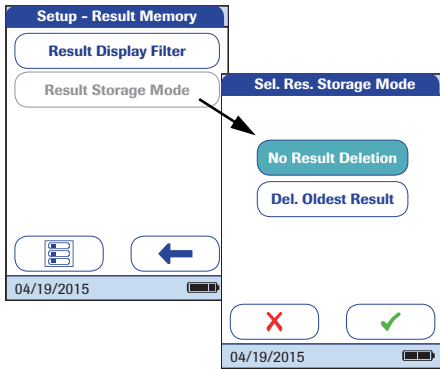


However, you only can select between these two options, if the meter is used together with a DMS and the *Computer* connection is enabled. In this case data stored in the meter are flagged, as soon as they have been transferred to the computer. Now the meter is allowed to delete those flagged data when necessary, even if you have selected the *No Result Deletion* option.



- 7 From the *Setup-Result Memory* menu, touch **Result Storage Mode** to select your storage mode.


You may select from the following options:

- **No Result Deletion**
- **Del(ete) Oldest Result**


- 8 Touch  to save this setting, or touch  to exit this menu without saving any changes. The display automatically returns to the previous screen.
- 9 Touch  to return to the *Setup-Data Handling* menu.

## Result units

The result unit setting applies to the result display of **D-Dimer** and **Troponin T** tests only.

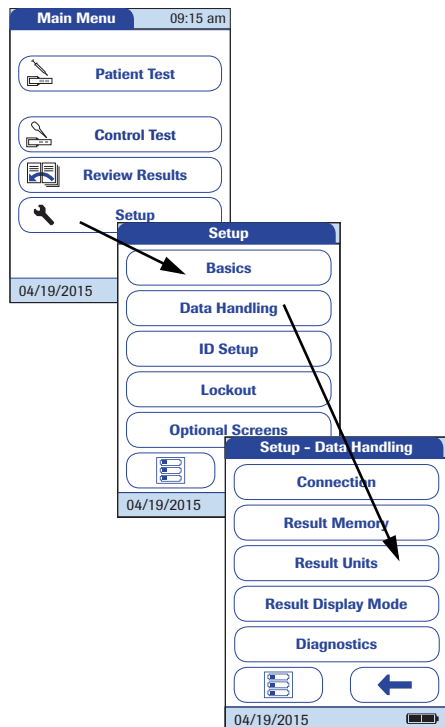


- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Data Handling**.
- 3 From the *Setup-Data Handling* menu, touch **Result Units**.

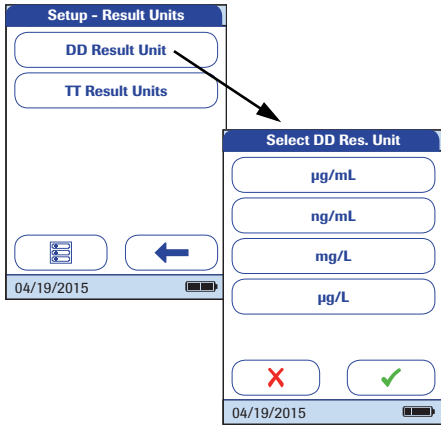

- 4 From the *Setup-Result Units* menu, touch **DD Result Unit**.
- 5 From the *Select DD Res. Unit* menu, select the unit to be used for the D-Dimer result display.

The current setting is highlighted (white type on a blue background). You may select from the following options:

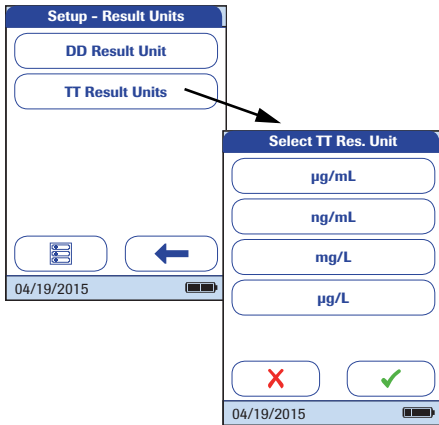


- **µg/mL**
  - **ng/mL**
  - **mg/L**
  - **µg/L**
- 6 Touch **✓** to save this setting, or touch **✗** to exit this menu without saving any changes. The display automatically returns to the previous screen.



- 7 From the *Setup-Result Units* menu, touch **TT Result Unit**.
- 8 From the *Select TT Res. Unit* menu, select the unit to be used for the Troponin T result display.


The current setting is highlighted (white type on a blue background). You may select from the following options:

- ng/L
- pg/mL
- ng/mL
- µg/L


- 9 Touch  to save this setting, or touch  to exit this menu without saving any changes. The display automatically returns to the previous screen.
- 10 Touch  to return to the *Setup-Data Handling* menu.

## Result display mode

Test results may be displayed either with a static or a flashing display, the latter visually indicating that the measurement process has been finished. Once a result is available, the display starts flashing. Depending on the settings, this may be either a quantitative or qualitative result screen or an operator login screen.

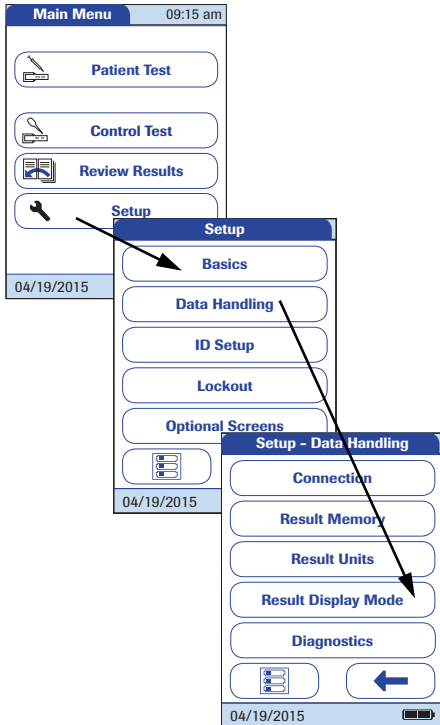

- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Data Handling**.
- 3 From the *Setup-Data Handling* menu, touch **Result Display Mode**.

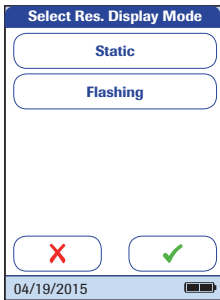

- 4 From the *Select Res. Display Mode* menu, select your display mode.

The current setting is highlighted (white type on a blue background). You may select from the following options:


- **Static**
- **Flashing**

- 5 Touch ✓ to save this setting, or touch ✗ to exit this menu without saving any changes. The display automatically returns to the previous screen.
- 6 Touch ← to return to the *Setup-Data Handling* menu.

## Diagnostics


Under *Diagnostics*, you will find information about the system, such as software version, number of data records stored, and configuration details. The *Diagnostics* menu contains functions relevant for installation, maintenance and troubleshooting purposes. In case of technical problems the firmware version (Build) is an important information for the Roche Diagnostics customer support.

The diagnostics screens shown here are for illustration purposes only. The Information shown on your meter may differ.

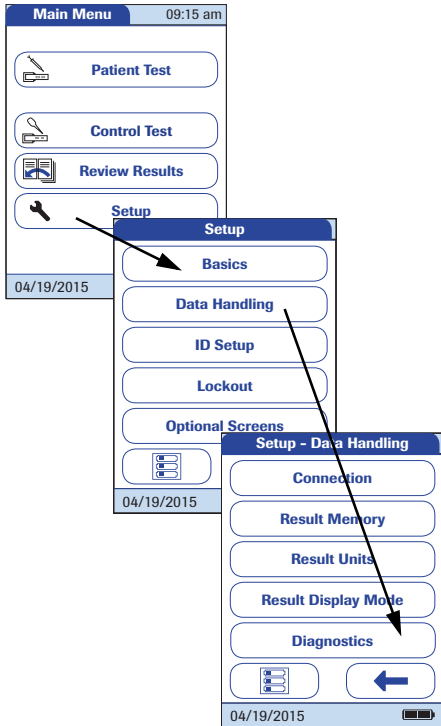

- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Data Handling**.
- 3 From the *Setup-Data Handling* menu, touch **Diagnostics**.



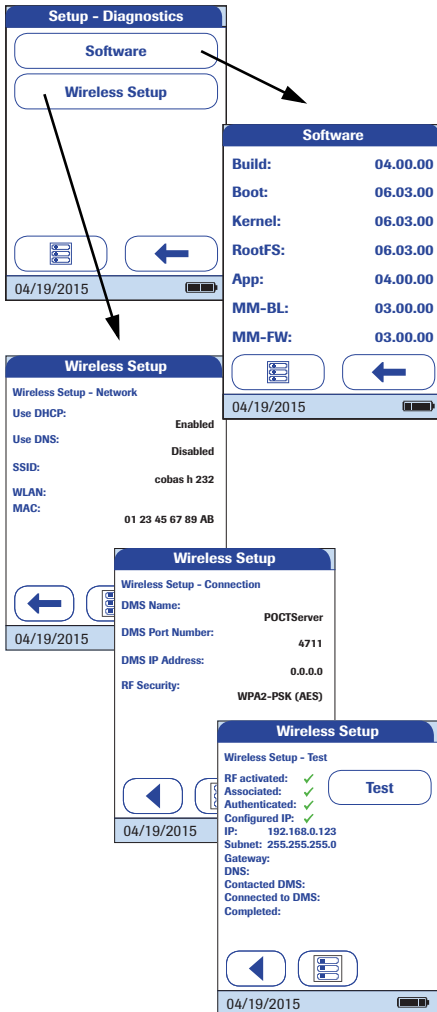

- 4 From the *Setup-Diagnostics* menu, touch the button of the diagnostics screen you want to display.
- 5 Use ◀ and ▶ to toggle between the *Wireless Setup* screens.
- 6 In either diagnostics screen, touch ◀ to return to the *Diagnostics* menu.

Note: The **Wireless Setup** button is only available if the the WLAN function is enabled on this meter.

### 3.3 ID Setup setting

Use the *ID Setup* menu to enter settings for user management and patient management. These settings are optional and set to *Off/Inactive* by default; the meter can be operated without these settings.

There are three types of identification used with the meter:

- **System Administrator:** The administrator has special rights to enter certain meter settings and is the only one who can enter and change these settings. It is not necessary for administrator identification to be activated to use the **cobas h 232** meter. However, it might be desired or necessary, depending on the regulatory environment and the site of use.
- **Operator:** The *Operator ID* is assigned to persons who use the meter to run tests. If you want to use *Operator IDs*, you have several options:
  - You may use *Operator IDs* to restrict the use of the meter to qualified personnel or a predefined group of users. In this case an operator list created on a DMS must be transferred to the meter, enabling you to select an *Operator ID* when logging in. For more details see “Data handling”, starting on page 159.
  - You may use *Operator IDs* for informational purposes only, in order to assign stored measurement results to the users who performed the test. In this case *Operator IDs* may directly be entered on the meter (by keypad or scanner), with or without an operator list being available.
- **Patient:** The *Patient ID* is assigned to the person, whose test results are recorded. You can either:
  - block input of a unique *Patient ID* (in this case, every test is simply numbered in consecutive order)
  - allow a unique *Patient ID* as optional, or
  - require a unique *Patient ID* for every test. Patient lists created on a DMS can also be transferred to the meter, enabling you to select *Patient IDs* for a test from these lists. For more details see “Data handling”, starting on page 159.

*Operator IDs* can be selected from a list (if available) or read by the barcode scanner on the front of the meter. If passwords were created, they **must** be entered via the onscreen keypad.

*Patient IDs* can be entered by using the onscreen keypad or the barcode scanner on the front of the meter.

## Administrator ID

In the default setting, the meter is not protected with an *Administrator ID*, and all setup options are accessible to every user. If you set up an *Administrator ID*, the following setup areas are automatically reserved only for the administrator (i.e., the person who knows the password):

- Screen: Date/Time
- Data Handling
- ID Setup
- Quality control and the associated lockouts
- Optional screens


The setup of an *Administrator ID* does not limit or alter the usage of the meter for measurements in any way. Furthermore setting up the *Administrator ID* has no impact on the usage of *Operator IDs*. Only the setup options as listed above are tied to administrator access.



When you enter an *Administrator ID*, this ID must be entered from this point forward before any future IDs can be set up (anywhere in the *ID Setup* menu). The *Administrator ID* must also be entered before you can delete or change the *Administrator ID*.

If you forget the *Administrator ID*, the meter setup may be unlocked via the external data management system (e.g., cobas IT 1000). If you don't use such a system and need to reset the *Administrator ID*, contact your Roche representative (see Chapter 10).

**If an *Administrator ID* has not been set up yet:**

- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **ID Setup**.
- 3 From the *Setup-ID Setup* menu, touch **Administrator ID**.

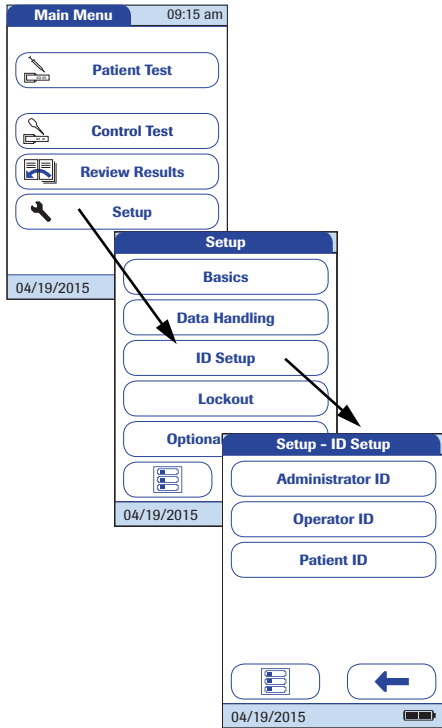

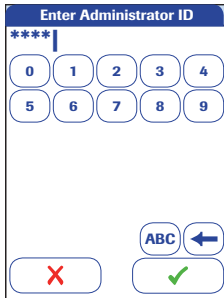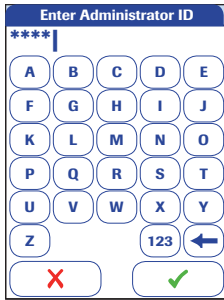


- 4 Using the onscreen keypad, enter the *Administrator ID* of choice (or the password provided by Roche Diagnostics). The ID can consist of up to 20 alphanumeric characters.


Alphanumeric characters are any combination of A - Z and 0 - 9. Additionally "." (period), or "-" (hyphen) may be used.


Pay close attention to the buttons you press, because the characters are not displayed on the screen. Asterisks are displayed instead (as if entering a password on a computer).

- 5 Use to switch to the input of numbers.
- 6 Use to switch back to the input of text.
- 7 Use to backspace and correct a mistake.
- 8 Touch to save this *Administrator ID*, or touch to exit this menu without saving any changes.

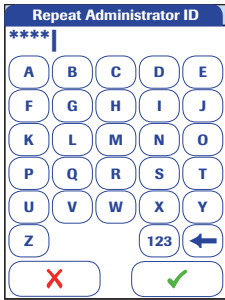


- 9 Enter the *Administrator ID* again (the onscreen keypad is automatically displayed again) to confirm the first entry.

Touch  to save this entry, (the *Administrator ID* is now set), or

Touch  to exit this menu, the *Administrator ID* is not set and is therefore still inactive.

The display automatically returns to the *Setup-ID Setup* menu. After you exit the *Setup* menu, only an authenticated administrator may set up any further IDs.

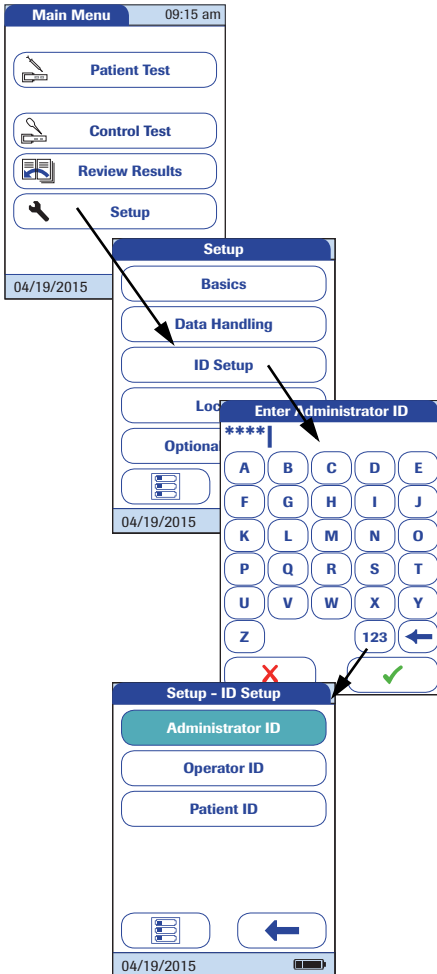


### Changing an existing *Administrator ID*:

- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **ID Setup**.
- 3 Using the onscreen keypad, enter the valid *Administrator ID*.

The *ID Setup* is displayed. The **Administrator ID** button is highlighted, which means an *Administrator ID* is active.

- 4 Touch **Administrator ID** and use the onscreen keypad to enter a new ID of choice (enter it twice).

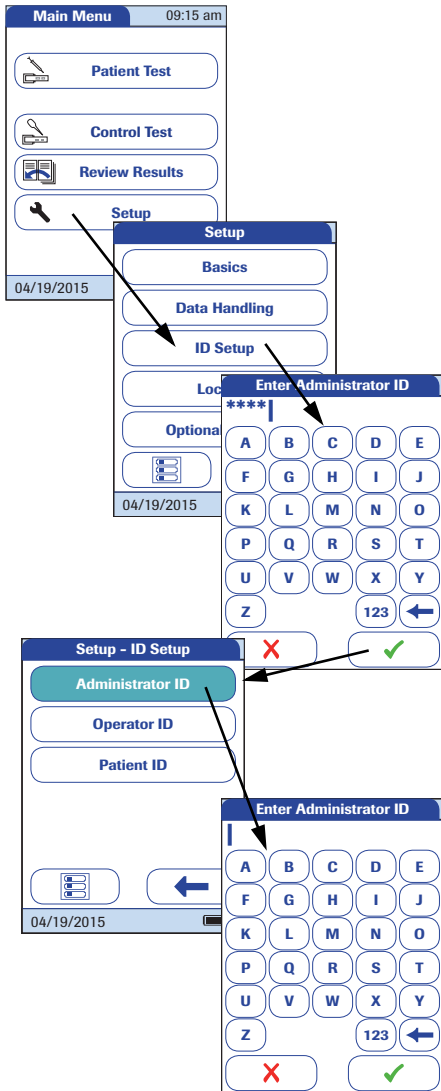

### Disabling an existing *Administrator ID*:

- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **ID Setup**.
- 3 Using the onscreen keypad, enter the valid *Administrator ID*.

The *ID Setup* is displayed. The **Administrator ID** button is highlighted, which means an *Administrator ID* is active.


- 4 Touch **Administrator ID**.
- 5 Touch **✓** to close the onscreen keypad **without** entering an ID number.

The *Administrator ID* has been deleted and therefore deactivated. The **Administrator ID** button is no longer highlighted.





## Operator ID

In the default setting, the *Operator* login is not activated. You can either activate or deactivate *Operator* login on the meter. If activated, an *Operator* has to log in before the *Main Menu* will be displayed and measurements can be performed. Without a list, operators can log in directly by entering their operator name. This setup menu allows you to select whether or not an operator login is possible or required.

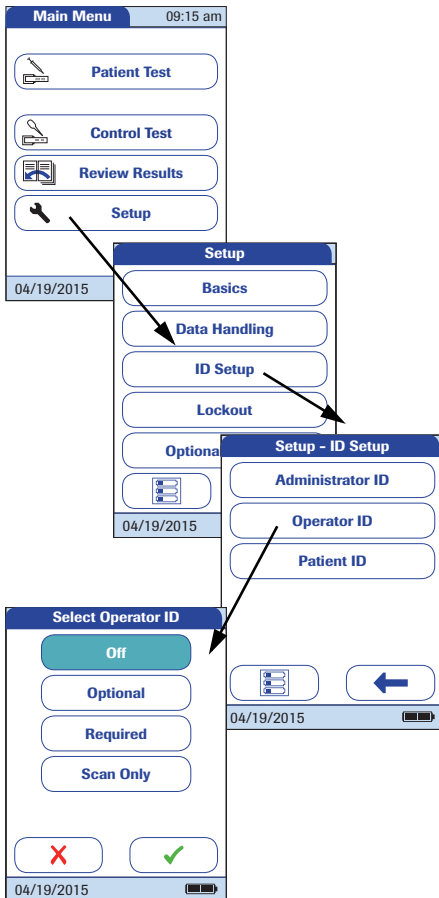


- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **ID Setup**.
- 3 From the *Setup-ID Setup* menu, touch **Operator ID**.

You may select from the following options:

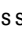- **Off** (*Operator ID* cannot be entered)
  - **Optional** (*Operator ID* can be entered, but is not required)
  - **Required** (An *Operator ID* must be entered)
  - **Scan Only** (An *Operator ID* must be scanned)
- 4 Touch the button with the setting of choice. Your selection is now highlighted.
  - 5 Touch  to save this setting, or touch  to exit this menu without saving any changes. The display automatically returns to the previous screen or (depending on the option you selected) proceeds to the next screen.

The entries for the option **Off** are now completed. For the options **Optional** and **Required**, continue by selecting the input format.

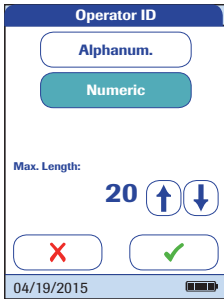





6 Select the form for input of the *Operator ID*.

You may select from the following options:

- **Alphanum.** (alphanumeric)  
Enter any combination of A - Z and 0 - 9. Additionally "." (period), or "-" (hyphen) may be used.
- **Numeric**  
Enter numbers only, e.g., "3387"
- **Max. Length**  
Enter the maximum number of characters (1 ... 20) the *Operator ID* can have.

7 Touch the button with the form of choice for setting up the *Operator ID*. Your selection is now highlighted.


8 Touch  or  to set the number of characters (length) of choice.

9 Touch  to save this setting, or touch  to exit this menu without saving any changes.

The meter automatically returns to the *Setup-ID Setup* menu.

If you want to create a list of *Operator IDs* from which you can select an operator, additional software (a data management system) and a WLAN connection or the Handheld Base Unit are required (for more details see “Data handling”, starting on page 159).

With an *Operator* list being transferred to the meter, the *Operator ID* options are different from the options in “standalone” mode.

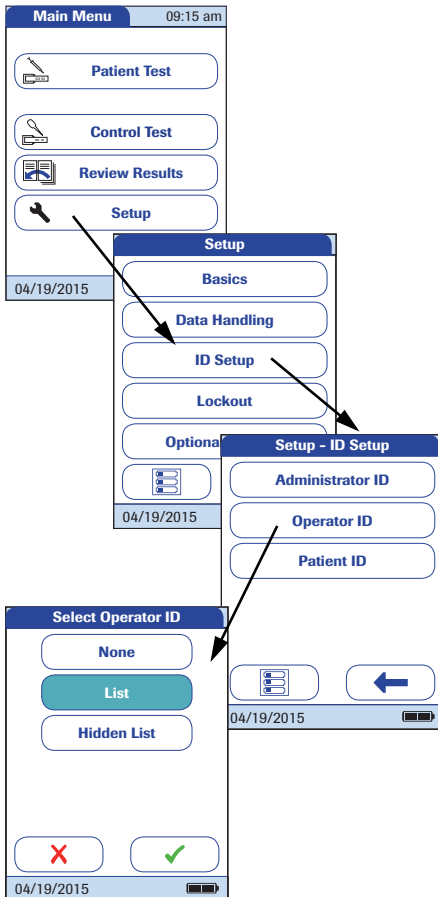

- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **ID Setup**.
- 3 From the *Setup-ID Setup* menu, touch **Operator ID**.


You may select from the following options:

- **List** (*Operator ID* must be selected from the list or scanned using the barcode scanner)
  - **None** (*Operator ID* cannot be entered or selected)
  - **Hidden List** (*Operator ID* must be entered using the barcode scanner or the onscreen keypad)
- 4 Touch the button with the setting of choice. Your selection is now highlighted.
  - 5 Touch ✓ to save this setting, or touch ✗ to exit this menu without saving any changes. The display automatically returns to the previous screen.

## Patient ID

*Patient IDs* help you to assign stored results to individual patients. In the default setting, input of *Patient IDs* is set to *No*. This means that a consecutive number is assigned to each test. However, you can require that a *Patient ID* must be entered or make it optional.

If you want to create a list of *Patient IDs* from which you can select a patient for testing, additional software (a data management system) and a WLAN connection or the Handheld Base Unit are required (see page 159).

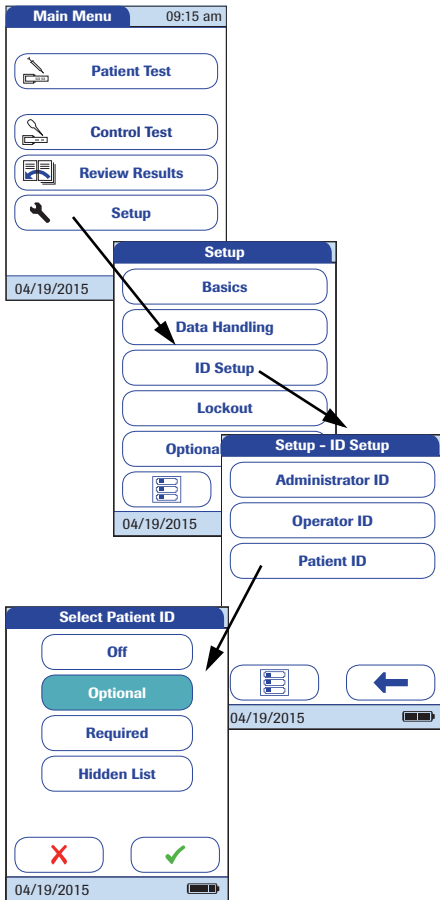

- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **ID Setup**.
- 3 From the *Setup-ID Setup* menu, touch **Patient ID**.

You may select from the following options:

- **Off** (*Patient ID* cannot be entered)
  - **Optional** (*Patient ID* can be entered, but is not required)
  - **Required** (The operator must enter the *Patient ID*)
  - **Hidden List** (Only available when working with a DMS. The list will not be shown. The operator needs to enter the *Patient ID* manually or via barcode scan.)
- 4 Touch the button with the setting of choice. Your selection is now highlighted.
  - 5 Touch to save this setting, or touch to exit this menu without saving any changes. The display automatically returns to the previous screen or (depending on the option you selected) proceeds to the next screen.



The entries for the option **Off** are now completed. For the options **Optional** and **Required**, continue by selecting the input format.



- 6 Select the form for input of the *Patient ID* before each test.

You may select from the following options:

- **Alphanum.** (alphanumeric)  
Enter any combination of A - Z and 0 - 9. Additionally "." (period), or "-" (hyphen) may be used.
- **Numeric**  
Enter numbers only, e.g., "3387"
- **Max. Length**  
Enter the maximum number of characters (1 ... 20) the *Patient ID* can have.

- 7 Touch the button with the form of choice for the *Patient ID*. Your selection is now highlighted.

- 8 Touch  or  to set the number of characters (length) of choice.

- 9 Touch  to save this setting, or touch  to exit this menu without saving any changes.

The display automatically returns to the *Setup-ID Setup* menu.

### 3.4 Lockout setup

The *Lockout* menu contains the options for quality control tests that the operator is required to perform at specified intervals or based on specific triggers. This is a list of the available lockout options:

- *Operator Lockout*
- *QC Settings*, containing
  - *New Lot Lockout*
  - *QC Lockout*
  - *IQC Lockout*
  - *Custom Range Trop T*
- *Cleaning Lockout*
- *STAT Test Config.* (overriding a lockout in an emergency situation)

If the quality control test is not performed as required or if the result is outside the target value range, the meter, parameter or test strip lot is locked from further use. Lockouts can also be set up on an operator-specific basis, i.e., each operator must perform the corresponding quality control tests at specified intervals.

A quality control test must be completed successfully before the meter is made available again (either to the operator or in general) for testing.


The option of setting up lockouts for operators is available only when the meter is set up for operator logins (see page 89) and operator lists are stored in the meter.

In case of an emergency, there may not be the time to first complete the required quality control test if a lockout has been triggered and the meter is "locked". For such cases you may set up so-called *STAT Tests* (STAT = **S**hort **T**urn**A**round **T**ime). This option allows to perform a limited number of tests when an operator lockout or QC lockout is in force. However, once the defined number of STAT tests has been performed, further use of the meter is no longer possible (for the affected test parameter or operator), unless the required quality control test is successfully completed.

STAT tests cannot be performed if IQC lockout is in place or a new lot has not (yet) passed a quality control test (New Lot Lockout).

## Operator lockout

To perform a test with the meter, the individual steps must be performed properly. Quality control tests can be performed on a regular basis by every operator to ensure these steps are performed properly. By activating the operator lockout, operators are required to perform regular quality control tests.

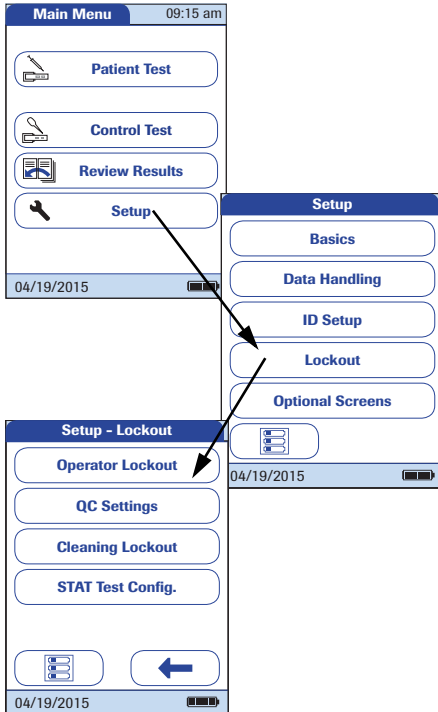





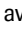


- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Lockout**.
- 3 From the *Setup-Lockout* menu, touch **Operator Lockout**.  
If this button appears in gray (inactive), the option *Operator ID* is deactivated.

- 4 Select the time interval of choice in which the required quality control tests must be performed.

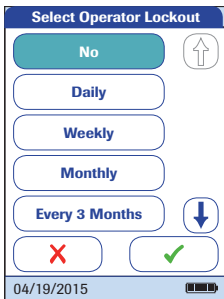You may select from the following options:


- **No** (deactivated)
- **Daily**
- **Weekly**
- **Monthly**
- **Every 3 or 6 months**
- **Yearly**



- 5 Touch  or  to display the option of choice on the screen. If the arrow is just an outline  , this means no other options are available in the direction indicated.
- 6 Touch the button to select the time interval of choice. Your selection is now highlighted.
- 7 Touch  to save this setting, or touch  to exit this menu without saving any changes.

If you selected **No**, *Operator Lockout* is deactivated.



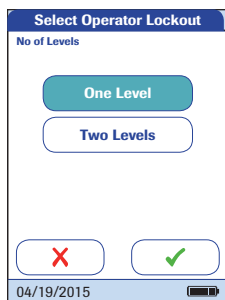


- 8 Quality control tests may include one or two levels. Select the number of levels which must be performed (not applicable, if you selected **No**).
- 9 Touch ✓ to save this setting, or touch ✗ to exit this menu without saving any changes. The display automatically returns to the *Setup-Lockout* menu.

## Quality control (QC) settings

*QC Settings* define that operator-independent quality control tests must be run on a regular basis. Lockouts may be triggered by selectable time periods, and lockouts may be triggered by certain events. The *QC Settings* menu contains the following options:

- *QC Lockout* (time-based), with the additional option of a *New Lot Lockout* (applies every time a new test strip lot is used)
- *IQC Lockout* (time-based)
- *Custom Range Trop. T* (customize the QC Range in accordance with applicable local guidelines)
- *QC Result Format* (select the format for QC result displays)
- *Reset Test Param.* (reset all test parameters, including code chip data and QC lockouts)

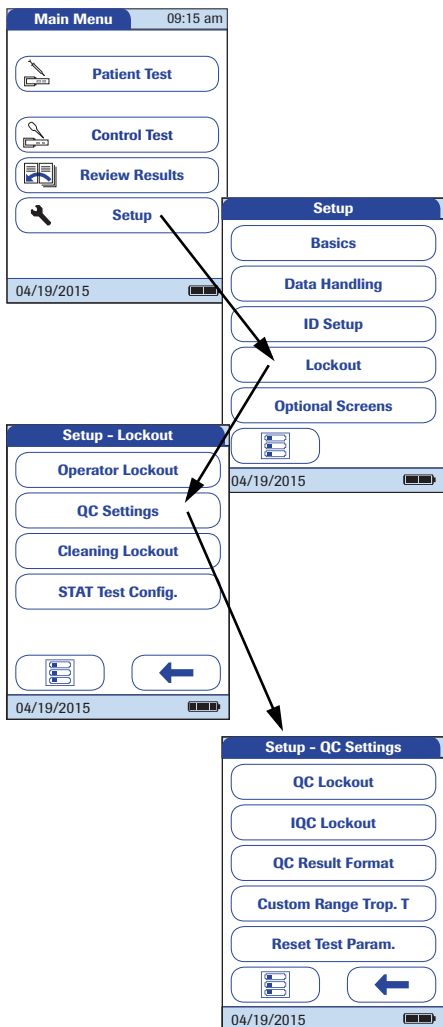

- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Lockout**.
- 3 From the *Setup-Lockout* menu, touch **QC Settings**.

## Quality control (QC) lockout

A *QC Lockout* requires a quality control test to be performed. You may also set the number of levels for this test. You may select from the following options:


- **New Lot: Yes/No** (applies every time a new test strip lot is used)

The general intervals are:

- **No**
- **Daily**
- **Weekly**
- **Monthly**

All enabled QC Lockouts, if the time period has elapsed, are activated at 8:00 a.m. (default setting, time can be changed with a DMS).

See page 99 on how to access the *Setup-QC Settings* menu.

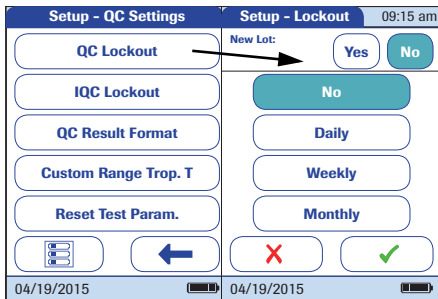

- 1 Touch **QC Lockout** to set regularly recurring quality control tests for available test parameters.
- 2 New Lot: Touch **Yes** if you want to force a control test every time a new test strip lot is used. Otherwise touch **No**.
- 3 Time intervals: Touch the button to select the time interval of choice (e.g., **Daily**). Your selection is now highlighted.
- 4 Touch **checkmark** to save this setting, or touch **X** to exit this menu without saving any changes.
- 5 Quality control tests may include one or two levels. Select the number of levels which must be performed (not applicable, if you selected **No**).
- 6 Touch **checkmark** to save this setting, or touch **X** to exit this menu without saving any changes. The display automatically returns to the previous screen.


## Instrument quality control (IQC) lockout



An *IQC Lockout* also requires a quality control test with dedicated IQC test strips to be performed. You may select from the following options:

- **No**
- **Daily**
- **Weekly**
- **Monthly**

All enabled IQC Lockouts, if the time period has elapsed, are activated at 8:00 a.m. (default setting, time can be changed with a DMS).

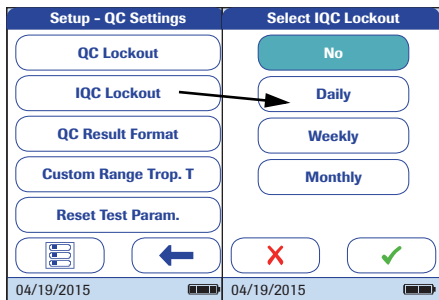See page 99 on how to access the *Setup-QC Settings* menu.




- 1 Touch **IQC Lockout** to set regularly recurring instrument quality control tests (IQC = **I**nstrument **Q**uality **C**ontrol).
- 2 Touch the button to select the time interval of choice. Your selection is now highlighted.
- 3 Touch  to save this setting, or touch  to exit this menu without saving any changes. The display automatically returns to the previous screen.



## QC result format

The QC Result format defines the kind of information that is displayed in a QC result screen. You may select from the following options:

- Display without a value, but with text “Pass” or “Fail”
- Display a value
- Display as value and with text “Pass” or “Fail”

See page 99 on how to access the *Setup-QC Settings* menu.

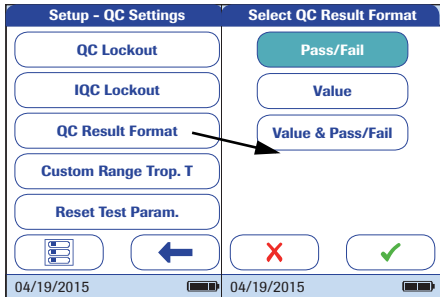

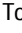
- 1 Touch **QC Result Format** to set the display format.
- 2 Touch the button to select the result format of choice. Your selection is now highlighted.
- 3 Touch  to save this setting, or touch  to exit this menu without saving any changes. The display automatically returns to the previous screen.

## Custom Range Troponin T

The quality control range for Troponin T can be customized to comply with local guidelines. The *Custom Range Trop. T* function enables you to narrow the default range.



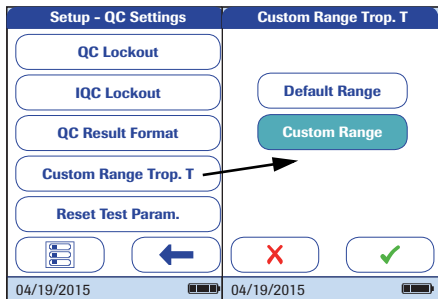

- **Default Range:** The meter displays the quality control range provided by Roche in the code chip.
- **Custom Range:** The option *Custom Range* lets the user define their own quality control range within the default range.


See page 99 on how to access the *Setup-QC Settings* menu.



- 1 Touch **Custom Range Trop. T** to set the quality control range.

You may select from the following options:

- **Default Range** (Range provided in the code chip is selected and displayed.)
  - **Custom Range** (The allowed percentage deviation from target value can now be customized).
- 2 Touch the button with the setting of choice. Your selection is now highlighted.
    - If you selected **Default Range**, touch  to save this setting, or touch  to exit this menu without saving any changes.
    - If you selected **Custom Range**, touch  to proceed with corresponding settings, or touch  to exit this menu without saving any changes. The display automatically returns to the previous screen.

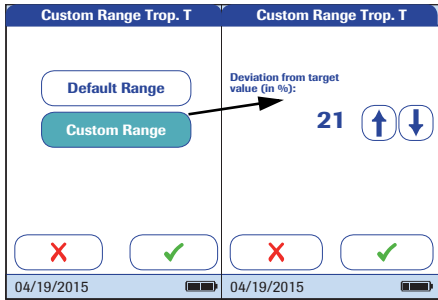

If you selected **Custom Range**, the *Custom Range* screen opens and offers you the option of customizing the deviation from target value (percentage value).

For the control solutions, the target value always comes from the information stored in the code chip. If you have chosen **Custom Range**, you can now select an allowed deviation from target value in the range of 0 to 50% (in the illustration, 21%).


- 3 Use the arrows to set the allowed percentage deviation from the target value.
  - Touch **✓** to save this setting, or touch **X** to exit this menu without saving any changes. The display automatically returns to the previous screen.





## Reset test parameters

Under certain circumstances, it may be necessary to delete the test parameters stored in the meter (for example, if a lockout exists for a test strip lot that is no longer available). However, do not use this method for overriding lockouts in general because, after resetting the test parameters, the meter may go into a QC lockout (if configured). If the meter or a parameter is locked by a QC lockout or an operator lockout, use STAT tests for emergency tests. STAT tests cannot be performed if IQC lockout is in place or a new lot has not (yet) passed a quality control test (New Lot Lockout).

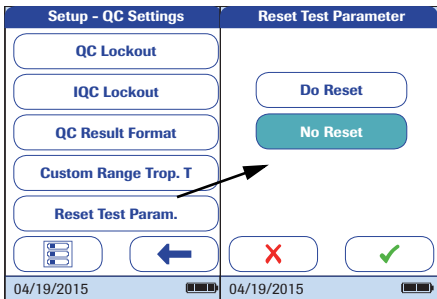
See page 99 on how to access the *Setup-QC Settings* menu.



- 4 Touch **Reset Test Param.** to reset all stored test parameters and QC lockouts.
- 5 Select **Do Reset** and confirm by touching  to reset the test parameters, or touch  to exit without resetting the test parameters (this leads to the same result as selecting and confirming **No Reset**). The display automatically returns to the previous screen.

## Cleaning lockout

Under normal handling conditions, your meter will not be exposed to significant dirt or contamination. However, circumstances might require regular checks of the test strip guide and (when necessary) cleaning of the meter. The *Cleaning Lockout* allows operators to specify time intervals for cleaning.

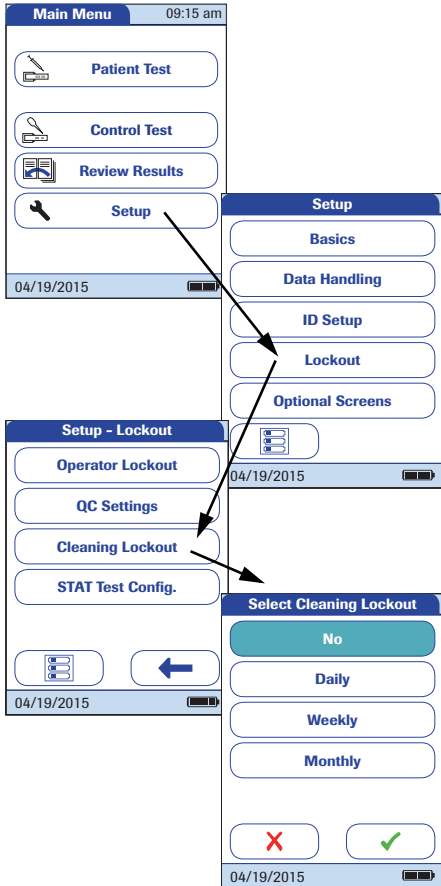

- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Lockout**.
- 3 From the *Setup-Lockout* menu, touch **Cleaning Lockout**.

You may select from the following options:

- **No**
  - **Daily**
  - **Weekly**
  - **Monthly**
- 4 Touch the button to select the time interval of choice. Your selection is now highlighted.
  - 5 Touch **✓** to save this setting, or touch **✗** to exit this menu without saving any changes. The display automatically returns to the previous screen.


## STAT test configuration





Quality control tests ensure consistent quality and accurate measurements. In emergency situations, however, it may be necessary to perform a test without delay. To override an active lockout for a test parameter, lot or operator, you can allow for *STAT Tests* (STAT = **S**hort **T**urn**A**round **T**ime). STAT tests cannot be performed if IQC lockout is in place or a new lot has not (yet) passed a quality control test (New Lot Lockout).

You can set a number of tests permitted beyond the lockout. Once the number of allowed *STAT Tests* has been reached (for a single parameter), additional tests for this parameter are locked until you successfully perform a quality control test.

*STAT Tests* will be counted separately for each test parameter. Thus the actual availability of *STAT Tests* may differ for each parameter.

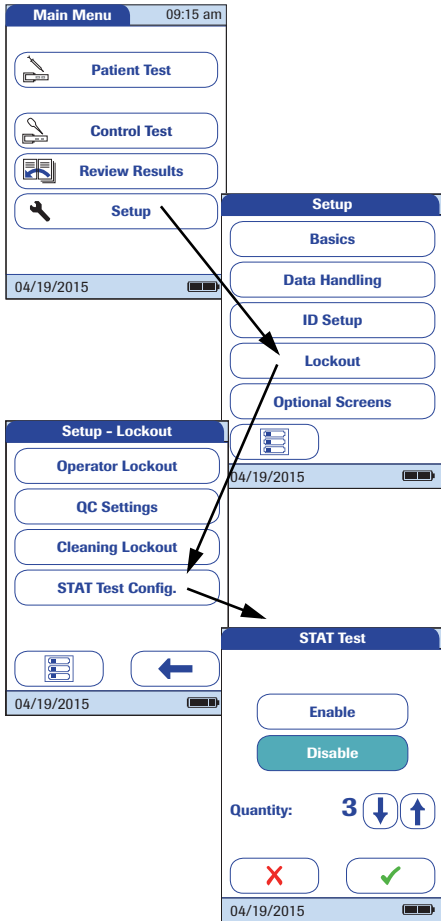
STAT tests are labelled accordingly in the result window.

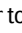

- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Lockout**.
- 3 From the *Setup-Lockout* menu, touch **STAT Test Config.**
- 4 Touch **Enable** to allow *STAT Tests* or: Touch **Disable** to prohibit *STAT Tests*. Your selection is now highlighted.
- 5 If you have enabled the option, touch  or  to set the number of additional tests.
- 6 Touch  to save this setting, or touch  to exit this menu without saving any changes. The display automatically returns to the previous screen.

### 3.5 *Optional Screens* setup

In this screen, you can customise display formats and test procedures. You can choose to display additional information and select from options for managing the result display.

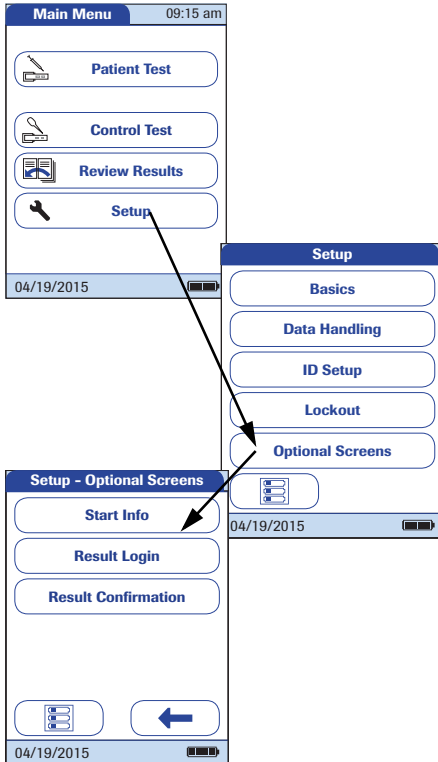




- 1 From the *Main Menu*, touch **Setup** to open the meter settings.
- 2 From the *Setup* menu, touch **Optional Screens**.

In this menu screen, you may select from the following options:


- Display additional information on meter status
- Select to display results only after new login by operator
- Enable operators to confirm or reject test results

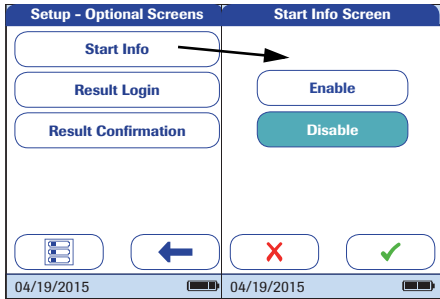

The additional status information is displayed after powering on the meter or after logging off as an operator. This includes information about current lockouts, the number of results not transferred (to the DMS) and the time since the last data transfer.




- 3 Touch **Start Info**.
- 4 Touch the button to select the state of the status display. Your selection is now highlighted.
- 5 Touch  to save this setting, or touch  to exit this menu without saving any changes.

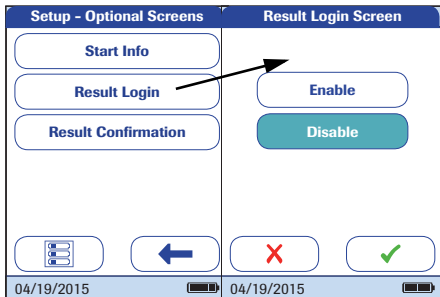In environments with multiple operators, it can be useful to display a test only if the operator carrying out the test is present. By requiring operators to login again before displaying results, you ensure that only authorised persons view the results.





- 6 Touch **Result Login**.
- 7 Touch the button to select the state of the option. Your selection is now highlighted.
- 8 Touch  to save this setting, or touch  to exit this menu without saving any changes.

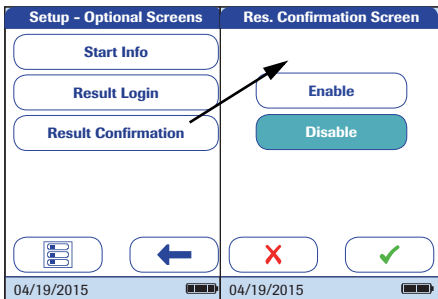In some circumstances, it may be useful for operators to confirm the validity of their results. To do so, an option can be enabled that prompts operators to confirm the results of every test.



**9** Touch **Result Confirmation**.

**10** Touch the button to select the state of the option. Your selection is now highlighted.

**11** Touch  to save this setting, or touch  to exit this menu without saving any changes.

This page intentionally left blank.



## 4 Performing a Test

What you need:

- Suitable blood sampling system to collect venous whole blood

Only heparinized blood collection tubes are allowed. Refer to test strip package insert.

- **cobas h** 232 meter
- Roche CARDIAC test strips for the desired test, with the supplied code chip
- Roche CARDIAC pipettes (or other suitable pipettes with pipetting volume of 150 µL) for applying the sample to the test strip

The Roche CARDIAC family includes several strips. Always ensure that the disposables you are using (e.g., test strips) are intended for use with the **cobas h** 232 meter. Some tests might require a minimum software version. Ensure that the software version on the **cobas h** 232 meter is compatible with the test you are running (refer to the corresponding package insert for details). Contact your Roche representative if the **cobas h** 232 meter requires a software update.

**Always ...**

- ... operate the meter according to the specified operating conditions (see page 177).
- ... follow the information on correct handling of test strips in the package insert.
- ... place the meter on a level, vibration-free surface when applying the sample to the sample application area of the test strip. Once the sample has been completely absorbed by the test strip, you can move the meter.
- ... keep the meter clean. See “Maintenance and Care” on page 165.

**Never ...**

- ... use Roche CARDIAC pipettes for collecting blood from the patient.
- ... touch or remove the test strip during a test.
- ... add more blood after the test has begun.
- ... remove or insert the code chip while the meter is performing a test.
- ... move or pick up the meter while applying the sample - wait until the necessary sample has been absorbed completely by the test strip.
- ... store the meter at extreme temperatures.
- ... store the meter in damp or humid conditions without protection.
- ... store the meter in a dusty environment without protection.



---

**Accuracy/precision of measured results:**

Failure to comply with the above may lead to inaccurate results. An incorrect result may lead to an error in diagnosis, therefore posing danger to the patient.

---

## Sample material

**Heparinized venous whole blood** is used as sample material. For each test use exactly 150 µL of sample.

Use a suitable sampling system to collect the blood sample. Use only heparinized blood collection tubes. Do not use blood collection tubes containing EDTA, citrate, sodium fluoride or other additives. For details refer to the test strip package insert.

**IMPORTANT NOTE:** Never use Roche CARDIAC pipettes for blood sampling directly from the vein nor for administering injections. They are not sterile-packaged and do not contain anti-coagulants. Roche CARDIAC pipettes are intended only for drawing the sample from the blood collection tube and applying it to the test strip. See page 129.



- Observe the applicable regulations and directives for hygiene and safety when collecting blood samples.
- Observe the applicable regulations and directives for disposing of potentially infectious samples and materials.
- Dispose of used consumables in line with the disposal policy of your hospital, institute or medical practice.
- See safety message “Protection against infection and blood-borne pathogens” on page 15.


## 4.1 Preparing to test



- 1 Prepare the required test strips (shown here: Roche CARDIAC POC for testing Troponin T).
- 2 Make sure that the code chip supplied with these test strips is also at hand.

## Code chip

The code chip provides the meter with important information on the manufacturer-specific parameters of the respective test strip lots. It contains information about the test method, the lot number and the expiration date.

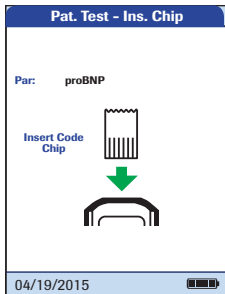

- Each test strip box contains its own code chip. Make sure you have the code chip at hand before performing the first test with a new test strip lot.
- When you insert a test strip from a new lot for the first time, the meter prompts you to insert the corresponding code chip. At this point, compare the code number you see on the display with the number that is printed on the test strip package you are using. If the two code numbers are identical, insert the new code chip into the slot in the meter.
- Once inserted, the code chip information is read and stored on the meter. The meter can store up to 200 code chip datasets (100 test strip lots and 100 quality control lots). Further tests using the same strip lot can be performed without inserting the code chip again.
- The code chip is no longer required, once the data is stored. You may leave it in the meter, or you can use it with other meters using the same test strip lot.

Roche recommends keeping the code chip in the meter to protect the contacts from becoming dirty. Protect the code chip from moisture and equipment which produces magnetic fields and electrostatic discharges. Once the test strip lot is used up, dispose of the old code chip promptly to prevent mix-ups.

### Inserting the code chip




- 1** Remove the old code chip, if one is inserted in the meter.
- 2** Always make sure that the number on the code chip matches the number on the label of the test strip package.  
  
By scanning the barcode on the strip, the meter gathers information about the number of the required code chip.
- 3** Slide the new code chip into the slot on the top of the meter as shown until you feel it snaps into place.

If the code chip does not match the test strip lot, is missing or incorrectly inserted, an error message appears in the display (see “Troubleshooting” starting on page 173).

**Test steps (overview)**

The following illustration shows the steps performed during testing. The steps you actively perform are displayed with a dark background. Depending on the configuration, the individual steps shown here may include additional actions (e.g., entering a password). Some steps may be different, working with or without Operator IDs and Patient IDs. On the following pages these steps are explained separately for each configuration.

- 1 Without operator and patient lists (or both operator and patient ID disabled)
- 2 With operator list (or operator ID enabled)
- 3 With operator and patient list (or operator and patient ID enabled)

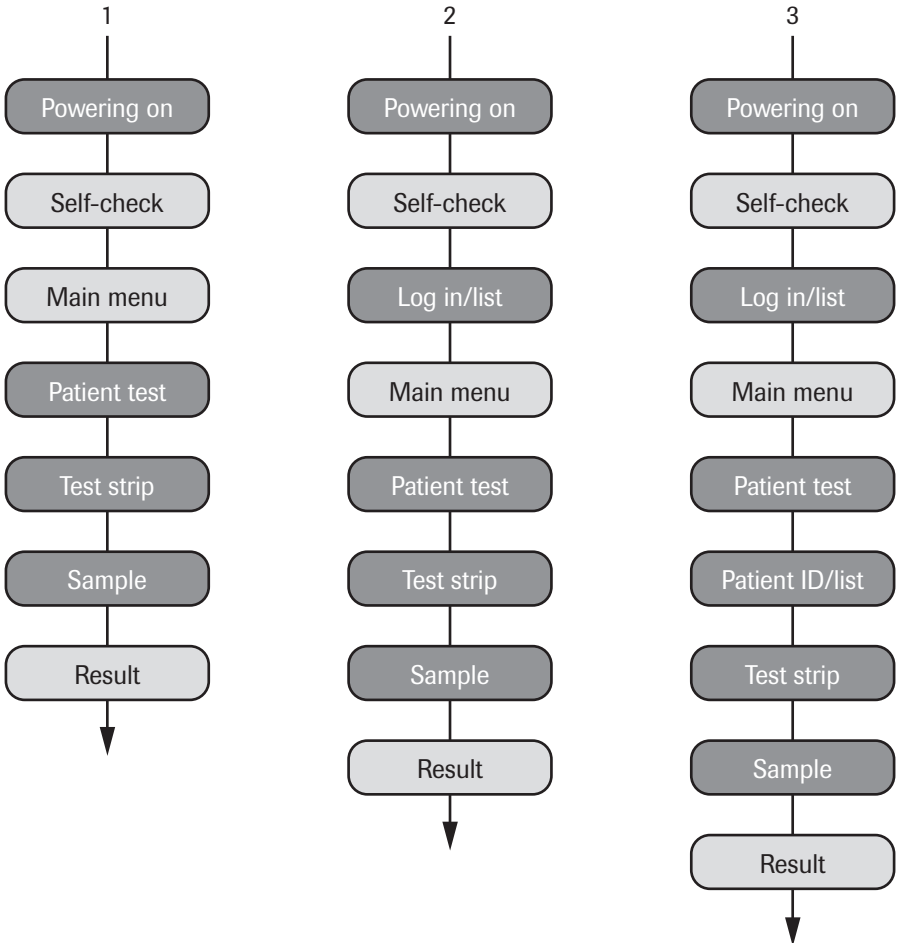


## Powering on the meter



- 1 Power the meter on by pressing the **ⓘ** button for approximately 1 second.

You can also power on the meter directly by connecting the power adapter or by placing the meter on the Handheld Base Unit.

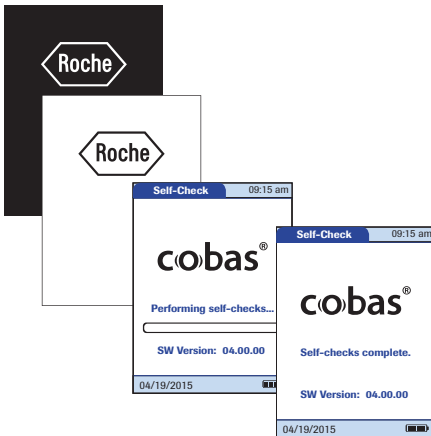


The meter performs a self-check at startup.

## Logging in


The initial steps until the *Main Menu* is displayed depend on the configuration and are performed as follows:

### Without operator login required

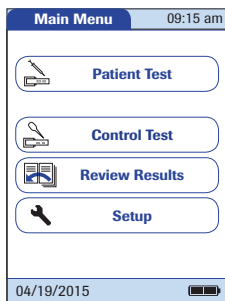


- 2 Wait until the main menu is displayed.

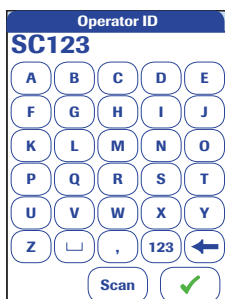

### With operator login (no operator list)



- 2 Enter or scan your *Operator ID*.
- 3 Touch ✓ to confirm your entry.
- 4 Wait until the main menu is displayed.

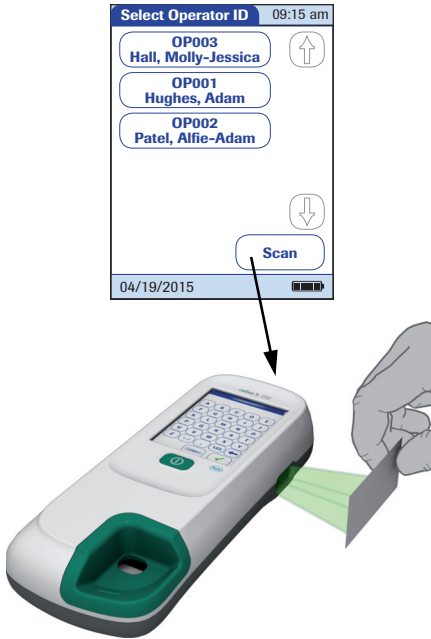




**With operator login (list available)**


- 2 If an operator list exists, wait until this list is displayed.
- 3 Touch  or  to scroll through the list. Select the operator of choice by touching the corresponding button.

As an alternative (meters with a scanner), the operator login can be performed via the built-in barcode scanner. Touch **Scan** and hold the card with the barcode approx. 10-20 cm (4-8 in) from the scanner.

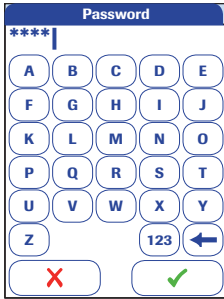It is not possible to connect an external barcode scanner to the meter.




- 4 Enter the password.
- 5 Touch  to confirm your entry, or touch  to display the operator list again.
- 6 Wait until the main menu is displayed.

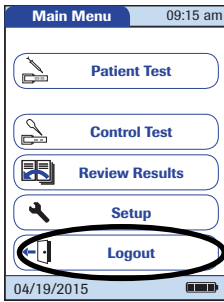


When the tests are completed or another operator wants to perform additional tests, touch **Logout** to log out. The meter returns to the display of the operator list.

## 4.2 Performing a test

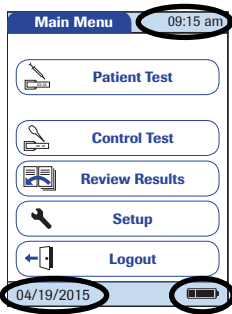


- 1 Check the charge level of the battery pack. If there are no bars left in the battery icon, you cannot perform any more tests.
- 2 Check that the date and time are correct. Correct any incorrect settings as described in the chapter entitled “Meter Setup/Setting the date”.

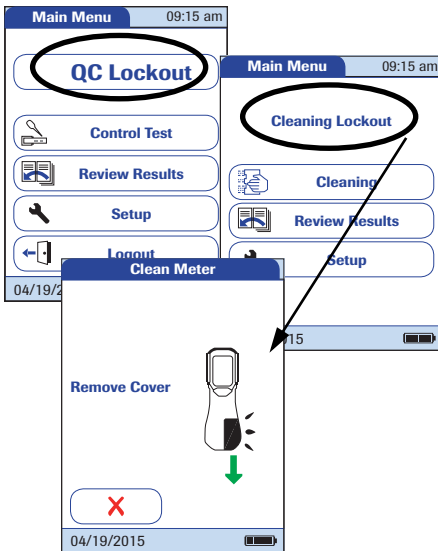

If a *QC lockout* is displayed instead of **Patient Test**, you must run a quality control test before you can perform a test (see “Control Testing and Quality Control” starting on page 137). If the meter is locked, a test can only be performed as a STAT test (if this function is enabled and if there are still STAT tests available).

If a *Cleaning Lockout* is displayed instead of **Patient Test**, you must touch **Cleaning** and follow the instructions on the screen and clean the meter. See “Maintenance and Care” beginning on page 165. Once you have re-attached the test strip guide cover and powered the meter on again, the meter is no longer locked.

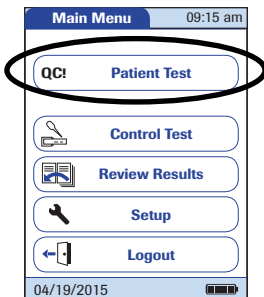




If the **Patient Test** button is available, but a lockout is displayed, you must first perform a quality control test for certain test parameters. Other test parameters are not affected.

**Without** patient list

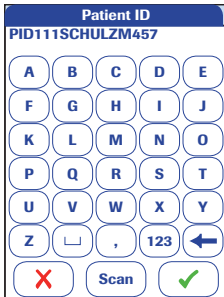

**3** Touch **Patient Test**.



- 4** If you selected the *Patient ID* as *optional*, you **can** enter or scan a *Patient ID*.  
If you selected the *Patient ID* as *required*, you **must** enter or scan the *Patient ID*.  
If you selected no *Patient ID* (*None*), the meter automatically proceeds to the next step.
- 5** Touch  to confirm the ID, or touch  to cancel the ID entry.


Continue reading on page 127.



**With patient list**




**3 Touch Patient Test.**

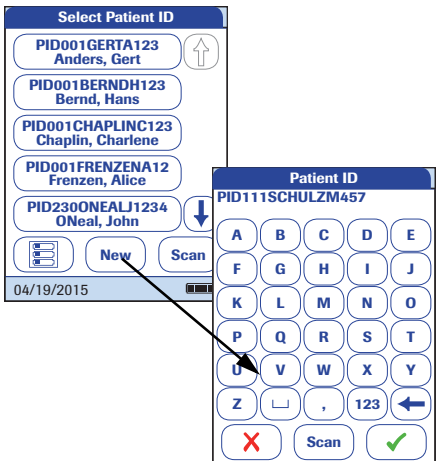Patient lists can only be created on a DMS and must be transferred to the meter. Lists cannot be created on the meter.



**4** Touch  or  to scroll through the list. Select the patient to be tested by touching the corresponding button.

**5** If the patient is not in the list, touch **New** to create a new entry. You must now enter a *Patient ID* manually or scan it using the barcode scanner.

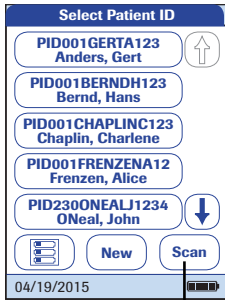


The *Patient ID* can also be entered via barcode (meters with a barcode scanner only). Touch **Scan** and hold either

- a card with the barcode approx. 10 cm from the scanner or
- the meter over a patient bracelet.

The scanner is also active, once you open the *Enter Patient ID* screen using the **New** button (for approx. 10 seconds).

## Inserting a test strip

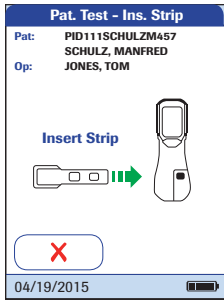


- 1 The test strip icon prompts you to insert a test strip. Remove the test strip from its foil pouch.
- 2 Hold the test strip so that the application area and test area are facing up.
- 3 Hold the test strip straight and flat. Insert it in the test strip guide of the meter. Be careful that the strip is not shifted, angled or tilt during insertion. It is also important that you insert the test strip using one smooth movement.



Inserting the test strip too slowly or too fast can cause a barcode reading error.

Slide the test strip in as far as it will go. A beep tone indicates that the meter has detected the test strip (provided the beeper is enabled).

Exposure to external influences (e.g., humidity) may deteriorate the test strips and may lead to error messages! Remove the test strip from its foil pouch only when you are ready to perform a test.

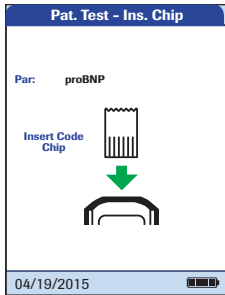


If you use a new test strip lot, you must insert the corresponding code chip once. See "Code chip" on page 116.


Depending on the meter setting, you may also be required to run a quality control test at this point.

The meter identifies the required code chip based on the test strip barcode and displays the code number.

For each test parameter and each new test strip lot a different code key number is displayed.

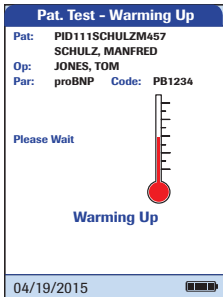

The thermometer icon shows that the test strip is warming up. A beep tone indicates when the warming-up process is complete (provided the beeper is enabled).

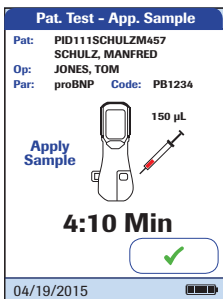


The pipette icon indicates that the meter is ready to perform the test and is waiting for blood to be applied.

Simultaneously, a 5-minute countdown begins. You must apply the sample within this time. Otherwise you will receive an error message (after the sample detection has timed out). The test strip is now unusable, repeat the test with a new test strip.

In case of errors, see "Troubleshooting" on page 173.



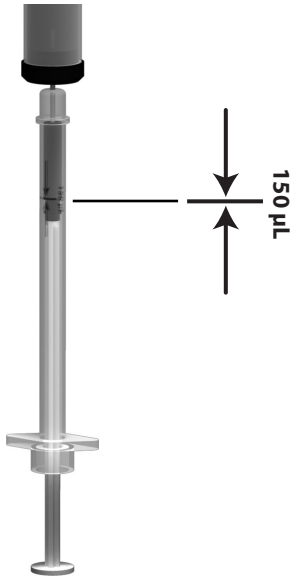


- 4 Using the Roche CARDIAC pipette, draw **exactly 150 µL** (to the blue mark on the pipette) heparinized blood from the blood collection tube. Make sure the sample contains no air bubbles.




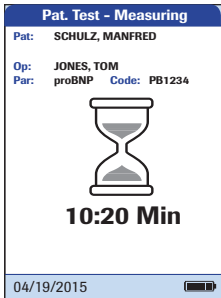
- 5 Apply the entire sample to the sample application area of the test strip.

Always place the meter on a level, vibration-free surface when applying the sample to the sample application area of the test strip. Once the sample has been completely absorbed by the test strip, you can move the meter.

- 6 Touch  to confirm that the sample has been applied.

The hourglass icon appears as the meter begins to process the sample. For better control over the measurement process, you should always confirm this yourself, instead of waiting for the meter to detect the sample.

After you confirm that the sample has been applied, or if the meter automatically recognises that the sample has been applied, the  button disappears.




The hourglass icon spins until the sample has been detected and the test begins. The time remaining for the test is displayed.

**Measurement time:**

The time needed to run a test depends on the test parameter (refer to package insert for details). Typically, **8 to 12 minutes** are required.


Do not add more blood. Do not touch the test strip until the result is displayed.

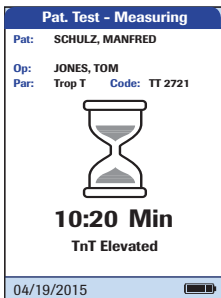

**For Troponin T only:**

As soon as the meter detects a signal (this may take a few minutes), a qualitative result (*TnT Elevated*) appears under the hourglass icon (see illustration).

The quantitative result is displayed when the test is completed. Refer to the Roche CARDIAC T Quantitative or Roche CARDIAC POC package insert for more details.

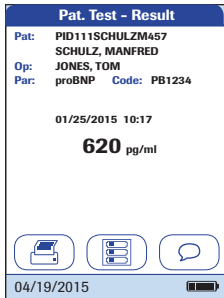

The test result is shown and stored automatically.

When interpreting your result, please read the test strip package insert carefully.



*Print* button:  
Print results (infrared interface)



*QR Code* button:  
Display result as QR code



*Menu* button:  
Return to main menu




*Comment* button:  
Add predefined or custom comments to the test result

- 7** Remove the test strip from the meter.
- 8** Power the meter off by pressing the **ⓘ** button for approximately 1 second if you do not wish to perform any further tests.
- 9** Discard the used disposable item and test strip in compliance with the disposal policy of your hospital or medical practice.
- 10** Clean the meter if necessary. See “Maintenance and Care” on page 165).

## Displaying, confirming or adding comments to results

As described in the chapter “Optional Screens setup” starting on page 109, there are many options for displaying results that can be activated:

If a new *Operator Login* is required to display the result:

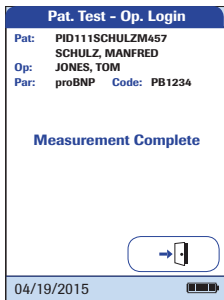

- 1 Touch the button for *Operator Login*.
- 2 Log in as described on page 121.

The result is now displayed.




If the test result requires a confirmation:

- When the result is displayed, touch the button to select the confirmation of choice.

You can choose to either

- **Reject**
- **Accept**

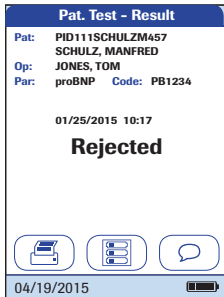


If you reject the result, the result value is no longer displayed. However, the test entry is stored.


## Performing a Test

You can add up to three comments to a test result. Comments can provide, for example, additional information about the test conditions or the patient. A comment may be up to 20 characters in length. You can open the function for adding comments directly in the results screen.

If you want to add a comment, do not remove the test strip. Once the strip is removed, the meter automatically returns to *Main Menu* and a comment can no longer be added.

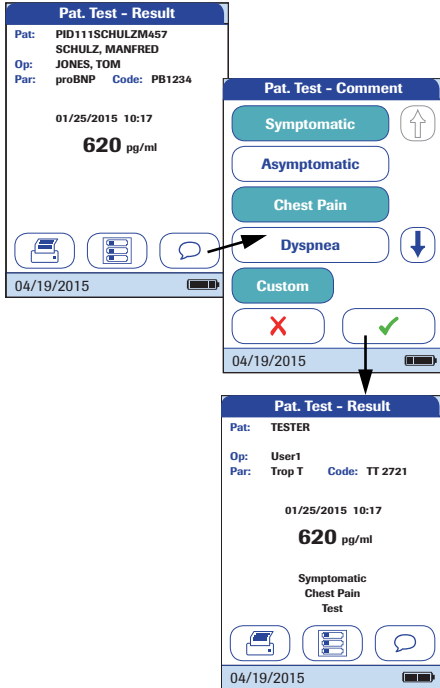

If you want to add a comment to a test result:



- 1 In the result screen, touch .
- 2 Select a predefined comment from the list and/or
- 3 Touch **Custom** to enter your own comment using the onscreen keypad.

The comments are stored with the test result.

## Displaying the test result as QR code




The QR code button only appears if the QR code display function is activated (see page 65 and following). Otherwise it is not displayed.


- 1 Touch  to display the result as QR code.
- 2 Touch  to return to the standard result display.


The size of the QR code displayed on the screen varies according to the amount of data encoded.

## STAT tests

STAT tests are a limited number of tests that can be performed in emergency situations, see page 107. If the meter is configured to perform STAT tests, you have the option of performing a test by choosing STAT even though a QC Lockout is in place.

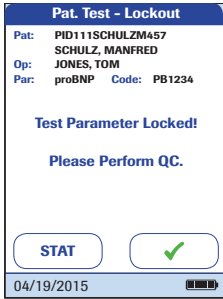

- 1 To perform the measurement **without** performing a quality control test, touch **STAT**, or:
- 2 Touch  to perform the quality control test before the measurement.

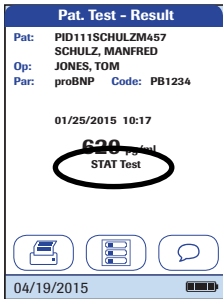

When a STAT test is performed, this information is stored with the result. The number of permissible STAT tests is reduced by 1. After all pending quality control tests are performed, the specified number of STAT tests is available again in case of a new lockout.



## 5 Control Testing and Quality Control

The **cobas h** 232 POC system employs several methods to check that the meter and test strips are working properly and that the operator is performing the test correctly.

The meter has a number of built-in quality control functions:

- A self-check of the electronic components and functions every time the meter is powered on.
- A check of test temperature when warming up and during the test.
- A check of the expiration date and lot information on the test strip using the information from the code chip.

In addition, the **cobas h** 232 POC system provides:

- Configurable, prescribed quality control tests using Roche CARDIAC IQC test strips for internal functionality tests.
- Definable, prescribed control tests using Roche CARDIAC Control materials for the available test parameters.

### 5.1 Preparing to run a quality control test

Prepare for a quality control test in the same way you would prepare to perform a test with a patient sample. The only difference is the use of control solution instead of blood.

To perform a quality control test using control solutions, you need:

- **cobas h** 232 meter
- Test strips for the desired tests, with the supplied code chip
- Roche CARDIAC pipettes (or other suitable pipettes with pipetting volume of 150 µL)
- Control set containing two vials of Roche CARDIAC Control material and code chip



---

Always refer to the package insert provided with the control material for specific handling and testing instructions.

---

Reconstituted control material taken from the refrigerator must be allowed to reach room temperature before use.

You can specify the frequency of quality control tests according to your own requirements (see “Meter Setup/Lockout setup” starting on page 94).

If you encounter a lockout for a test strip lot that no longer exists, see “Reset test parameters” on page 105. Please note that, when you use this method, **all** saved test parameters and QC lockouts will be deleted.

## 5.2 Performing a quality control test


### Quality control (QC)





- 1 Power the meter on by pressing the **ⓘ** button for approximately 1 second.


You can also power on the meter directly by connecting the power adapter or by placing the meter on the Handheld Base Unit.

Depending on the system settings the operator may have to log in and/or enter a password. Follow the procedures as described on page 121.

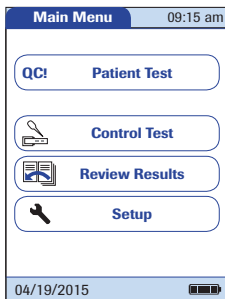

- 2 Wait until the main menu is displayed.

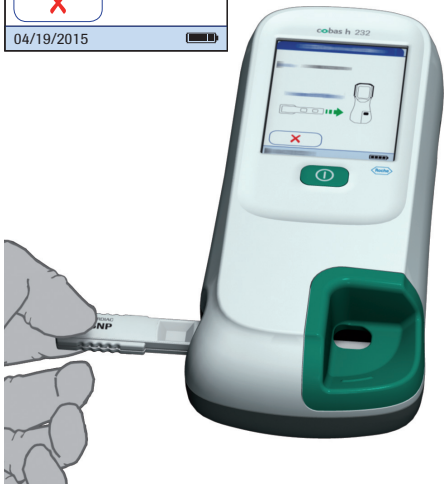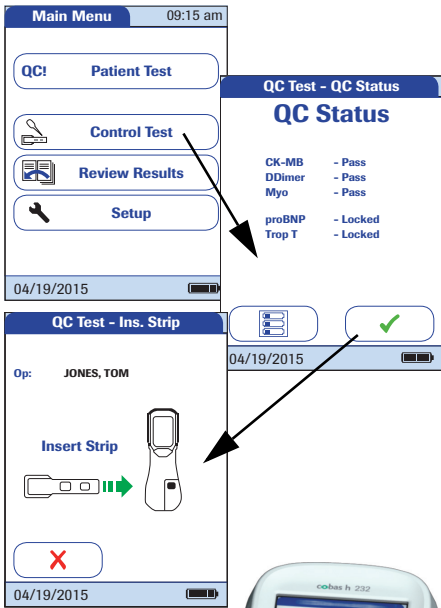


- 3 Touch **Control Test**.
- 4 In the *QC Status* screen, touch  to continue with the quality control test.
- 5 The test strip icon now prompts you to insert a test strip. Remove the test strip from its foil pouch.
- 6 Hold the test strip so that the application area and test area are facing up.
- 7 Hold the test strip straight and flat. Insert it in the test strip guide of the meter. Be careful that the strip is not shifted, angled or tilt during insertion. It is also important that you insert the test strip using one smooth movement.

Inserting the test strip too slowly or too fast can cause a barcode reading error.

Slide the test strip in as far as it will go. A beep tone indicates that the meter has detected the test strip (provided the beeper is enabled).


Exposure to external influences (e.g., humidity) may deteriorate the test strips and may lead to error messages! Remove the test strip from its foil pouch only when you are ready to perform a test.

If you use a new test strip lot and have not inserted the code chip yet, you must do so now. Otherwise you cannot perform a quality control test.

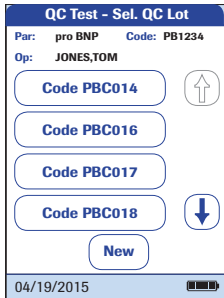


As with the test strips, a code chip is also provided with the control materials. The information on the code chip is stored in the memory so you can use the same control materials again at any time.


- 8 Select the code stored for your current control material, or touch **New** to use a new control material.

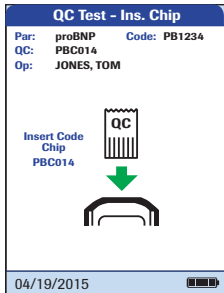


If you are using a new control material, remove the code chip from the meter and insert the code chip that came with the control material instead.

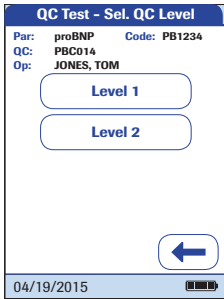

- 9 If the control material box contains more than one level, select the level of choice for this measurement.

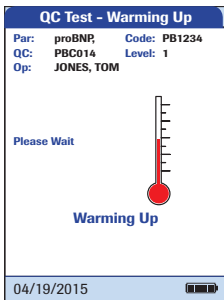

The thermometer icon shows that the test strip is warming up. A beep tone indicates when the warming-up process is complete (provided the beeper is enabled).

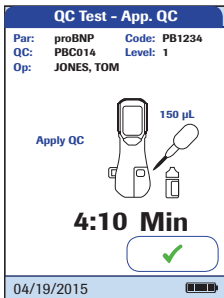

The pipette icon indicates that the meter is ready to perform the test and is waiting for the sample to be applied.

Simultaneously, a 5-minute countdown begins. You must apply the sample within this time, otherwise you will receive an error message (after the sample detection has timed out).


In case of errors, see "Troubleshooting" on page 173.




**10** Using the pipette, draw up the dissolved contents of the vial.

**11** Apply the sample (exactly 150  $\mu\text{L}$ ) to the application area of the test strip.

Always place the meter on a level, vibration-free surface when applying the sample to the sample application area of the test strip. Once the sample has been completely absorbed by the test strip, you can move the meter.


**12** Touch  to confirm that the sample has been applied.


The hourglass icon appears as the meter begins to process the sample. For better control over the measurement process, you should always confirm this yourself, instead of waiting for the meter to detect the sample.

**13** After you confirm that the sample has been applied, or if the meter automatically recognises that the sample has been applied, the  button disappears.

The hourglass icon spins until the sample has been detected and the test begins.

Unlike regular tests, this procedure can be very short and therefore might not be displayed (especially in case of a lack of or delay in confirmation, see step **12**).

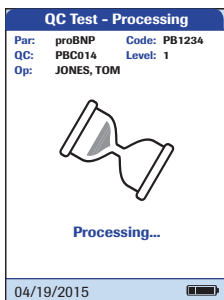


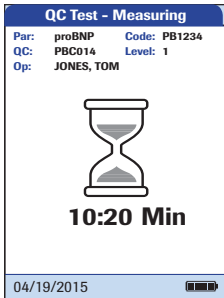

The time remaining for the test is displayed.

**Measurement time:**

The time needed to run a test depends on the test parameter (refer to package insert for details). Typically, **8 to 12 minutes** are required.

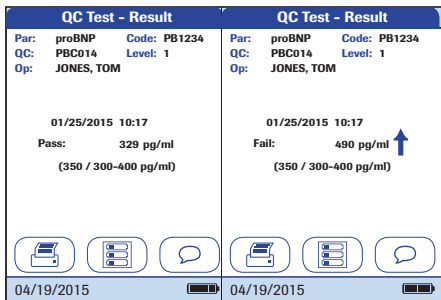

The result of this quality control test is displayed and is automatically saved to memory.

The target value and the range in which the results for this control material should be located is displayed below the current result. If a quality control test fails, an up arrow (too high) or down arrow (too low) is displayed.

There are three options for displaying a QC test result:

- Pass/Fail
- Value
- Value & Pass/Fail

For details refer to “QC result format” on page 102 and “Settings summary” on page 47.





*Print* button:  
Print results (infrared interface)



*QR Code* button:  
Display result as QR code



*Menu* button:  
Return to main menu





*Comment* button:  
Add predefined or custom comments to the test result

- 14** Remove the test strip from the meter.
- 15** Power the meter off by pressing the **ⓘ** button for approximately 1 second if you do not wish to perform any further tests.
- 16** Discard the used disposable item and test strip in compliance with the disposal policy of your hospital or medical practice.
- 17** Clean the meter if necessary (see “Maintenance and Care” on page 165).

## Displaying the QC result as QR code



The QR code button only appears if the QR code display function is activated (see page 65 and following). Otherwise it is not displayed.

- 1 Touch  to display the result as QR code.
- 2 Touch  to return to the standard result display.

The size of the QR code displayed on the screen varies according to the amount of data encoded.

## Instrument quality control (IQC)


The Roche CARDIAC IQC test serves as a performance check for the optical system of the **cobas h 232** POC system. The instrument determines the reflectance value of the signal and displays “Pass”, “Fail” or the percentage of remission depending on your settings.


“Pass” means that the optical system of the **cobas h 232** POC system is functioning properly. If “Fail” appears, the measured reflectance value is outside the confidence interval.




- 1 Prepare the required IQC test strips (shown here: Roche CARDIAC IQC high).
- 2 Make sure that the code chip supplied with these test strips is also at hand.


The first steps in this quality control test are identical to the procedure described before, see page 139. The following description therefore starts at the main menu.

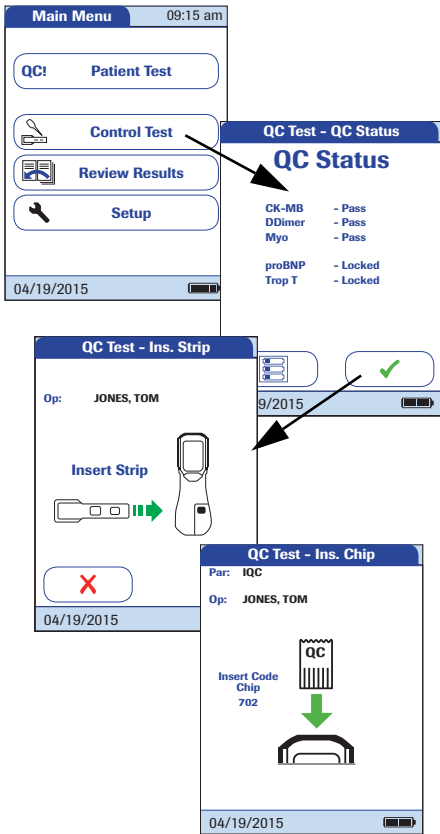

- 3 Touch **Control Test**.
- 4 Touch  in the *QC Status* screen to continue with quality control test. The test strip icon prompts you now to insert a test strip.
- 5 Remove the test strip from its IQC test strip container.
- 6 Insert the test strip as described before, see page 140.
- 7 If you are using a new lot of IQC test strips and have not inserted the code chip yet, you must do so now. If you have used the test strips already, skip this step.

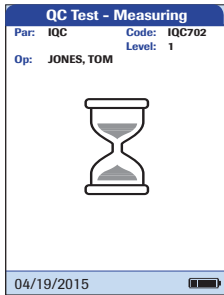

The hourglass icon is displayed and the (internal) measurement begins.

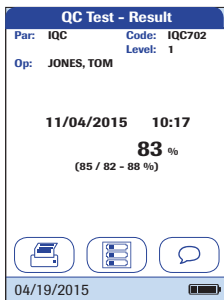

The result of this IQC quality control test is displayed as *Pass* or *Fail* (default setting) or in the format selected in the QC result format setting.

There are three options for displaying an IQC test result:

- Pass/Fail
- Value
- Value & Pass/Fail

For details refer to “QC result format” on page 102 and “Settings summary” on page 47.

This page intentionally left blank.

## 6 Review Results

The cobas h 232 meter can save 2000 patient test results as well as 500 quality control tests to memory, together with time, date, lot number and, if applicable, IDs and comments.


### Reviewing test results



- 1 Power the meter on by pressing the **ⓘ** button for approximately 1 second.

You can also power on the meter directly by connecting the power adapter or by placing the meter on the Handheld Base Unit.


Depending on the system settings the operator may have to log in and/or enter a password. Follow the procedures as described on page 121.

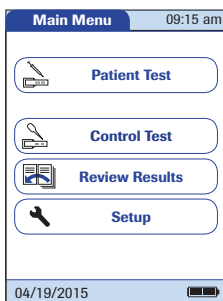

- 2 Wait until the main menu is displayed.


From the *Main Menu* you have access to all stored results, sorted by selectable criteria:


- **Patient History**
- **All Results**
- **QC Results** (Quality Control)
- **IQC Results** (Instrument Quality Control)
- **Maintenance History**

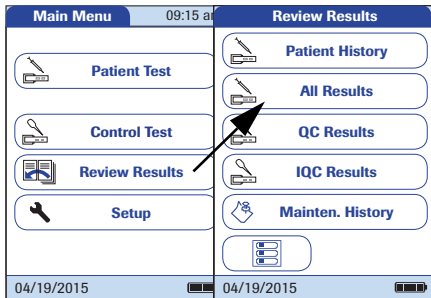

3 Touch **Review Results**.

4 Select the function you want to perform in the memory.


 The print button only appears if the printer function is activated. Otherwise it is not displayed.

 The QR code button only appears if the QR code display function is activated. Otherwise it is not displayed.




## Patient history

This memory area contains a list of all patients, sorted by *Patient ID*. From here, you can call up results for any individual patient.

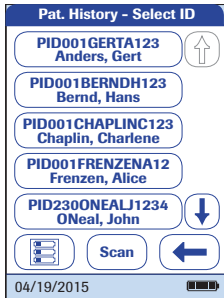


- 1 Touch or to display the entry of choice on the screen. If the arrow is just an outline , you have reached the beginning or end of the patient list.
- 2 Touch the patient name whose results you want to open.

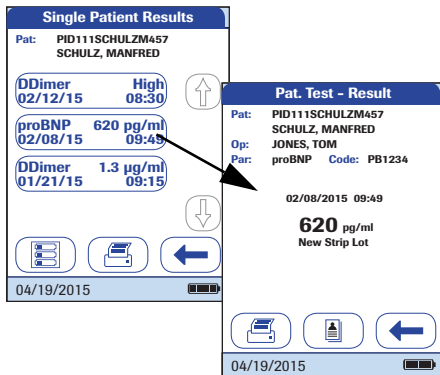




- 3 Touch or to display the test result of choice on the screen.
- 4 Touch the test result you would like to view in detail.

## All results

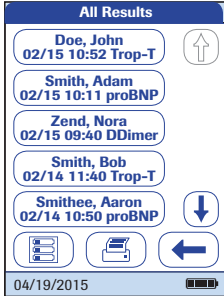This memory area contains a list of all tests, sorted chronologically. From here, you can call up results for specific times.



- 1 Touch  or  to display the entry of choice on the screen.
- 2 Touch the patient name whose result you want to open.

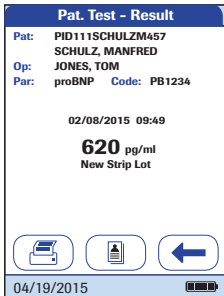




The selected test result dialog box will appear.


When you touch , a list of results for the selected patient is displayed (see page 153).

## Quality control (QC) results

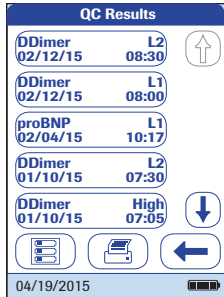This memory area contains all test parameter-specific quality control tests (QC, starting on page 139) that were run, sorted chronologically. The most recent results are at the top of the list.




- 1 Touch  or  to display the entry of choice on the screen.
- 2 Touch the entry you want to open.

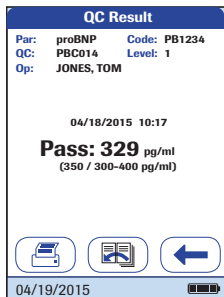




The entry is displayed.

## Instrument quality control (IQC) results

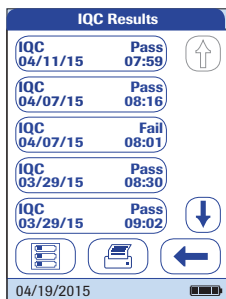This memory area contains all instrument quality control tests (IQC, starting on page 147) that were run, sorted chronologically. The most recent results are at the top of the list.




- 1 Touch  or  to display the entry of choice on the screen.
- 2 Touch the entry you want to open.

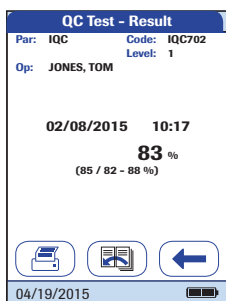



The entry is displayed.

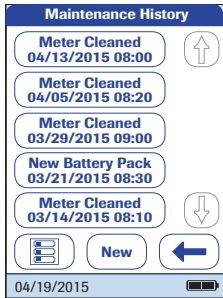


## Maintenance history

*Maintenance History* actually is a list of comments. Whenever routine maintenance or service is done for the meter, this event can be stored in form of a comment. You may use the default preset comments in the meter or enter a custom comment.



- 1 Touch  or  to display the entry of choice on the screen.

Once saved, a maintenance comment can not be opened or edited. *Maintenance History* is (like test results) a view-only list. You can, however, add new entries:

- 2 Touch **New**.
- 3 Touch  or  to display the predefined comment of choice on the screen.
- 4 Touch the predefined comment you would like to add to the *Maintenance History* or:
- 5 Touch **Custom** to enter your own text using the onscreen keypad.
- 6 Touch  to save your comment.

This page intentionally left blank.

## 7 Extended Functionalities

### Data handling

Extended data handling functionality is dependent on the capabilities of the particular Data Management System (DMS) being used and may vary.

When used in conjunction either with WLAN (if the meter is equipped with WLAN) or with the (optional) Handheld Base Unit from Roche Diagnostics, the **cobas h 232** meter can conveniently connect to a data management system (DMS). The main advantages of such a connection between meter and DMS may include:

- Transferring patient lists, operator lists, and settings from the DMS to the meter. In this way the setup for daily work as well as general meter setup can be performed (for one or more meters) quickly and conveniently.
- Transferring all stored test results with the corresponding *Patient ID*, *Operator ID*, and comments from the meter to the DMS. This transfer of data permits further evaluation or proper archiving according to your needs.

Through the DMS the system administrator (such as the POC coordinator) may specify what settings are to be applied to a set of meters within a site or unit (e.g., hospitals or wards). All meters assigned to a specific site and/or unit would consequently share the same settings. The different operator and patient lists (that match a site or unit) are then made available on their respective meters. Other settings such as *QC Lockout* and *Operator Lockout* may easily be set (once) and distributed to all meters.

The option of setting up an *Operator Lockout* is available **only** when operator lists are created on the DMS, stored in the meter, and *Operator* login is activated. For more details on *Operator* login see page 89, for details on *Operator Lockout* see page 95.

## **Computer (Setup option)**

For initial connection to a DMS, the ability to communicate within a network has to be set up as follows:

- In the *Setup* menu, the *Connection* option must be set to *Computer* (see page 68).
- The Handheld Base Unit must be configured correctly. (For details please refer to the manual of the Handheld Base Unit and the Technical Note stored on the Handheld Base Unit itself.)
- Place the meter on a Handheld Base Unit that is connected to the network. The meter will automatically be recognized by the DMS and may now be assigned to a site and/or unit managed by the DMS (if your DMS has that functionality).
- Depending on DMS functionality available, the DMS may transfer settings and lists to the meter as well as enable test results to be transferred from the meter to the DMS.



## Operator lists

After powering the meter on, an *Operator* login may be configured. If no list is available, the options described on page 89 apply.

- *Operator* is set to *Optional* or *Required*:  
The *Operator* can either log in via the onscreen keypad or a barcode scan. The use of the meter is not restricted to a predefined *Operator* group, so *Operator* IDs are entered via the onscreen keypad or read from the barcode and stored with test results for informational purposes only.
- *Operator* is set to *Scan Only*:  
The display of the meter's onscreen keypad is blocked. The *Operator* can only log in via a barcode scan as there is no onscreen keypad. The use of the meter is not restricted to a predefined *Operator* group, so *Operator* IDs are read from the barcode and stored with test results for informational purposes only.

Meters which are managed by a DMS may receive an *Operator* list.

- *Operator ID* is set to *List*, a list is available and displayed on the login screen:  
The *Operator* selects the ID from the displayed list. If so configured by the *Administrator*, the *Operator* may also be required to enter a password to log in.
- *Operator ID* is set to *Hidden List*:  
The *Operator* can login via a barcode scan or the onscreen keypad. If so configured by the *Administrator*, the *Operator* may also be required to enter a password to log in.

When working with operator lists: The use of the meter is restricted to the operators on this list.

## Patient lists

When starting a Patient Test, there will be the option of either selecting a *Patient ID* from a patient list, entering the *Patient ID* using the onscreen keypad or, if available, of scanning the barcode with the *Patient ID*. The *Patient ID*, if entered on the DMS, comes with an additional identification entry. Up to 20 characters can be used and assigned as a second confirmation (e.g., name, date of birth).

There are four possible configurations for a *Patient ID* input:

- The *Patient ID* is set to *Optional* or *Required*, and there is no list available:  
A *Patient ID* can be entered manually via the onscreen keypad or read in via a barcode scan. The Patient ID is stored with the test result.
- The *Patient ID* is set to *No*, but there is a list available:  
The list of Patient IDs is always shown when starting a test. You may now
  - select a patient from the list;
  - scan a *Patient ID* using the barcode scanner;
  - create a new patient entry by touching **New**. Instead of an ID a consecutive number will be assigned to this result.
- The *Patient ID* is set to *Optional* or *Required*, and there is a list available:  
The list of Patient IDs is always shown when starting a measurement. You may now
  - select a patient from the list;
  - scan a *Patient ID* using the barcode scanner;
  - create a new patient entry by touching **New**. You may either read the ID using the barcode scanner or enter a new ID via the onscreen keypad.
- The *Patient ID* is set to *Hidden List* (only possible with a data management system), and there is a list available:  
An empty list is shown when starting a measurement. You may now
  - scan a *Patient ID* using the barcode scanner;
  - create a new patient entry by touching **New**. You may either read the ID using the barcode scanner or enter a new ID via the onscreen keypad.

When working with *Patient* lists, you have the following options:

- A *Patient ID* does not have to be on the list to be scanned and used.
- The meter displays a warning that the *Patient ID* scanned is not on the list.
- The meter displays an error message that the ID was not found on the list of valid *Patient IDs*. You must enter a valid ID to proceed.

## Barcode scanner

Using a DMS, the barcode scanner may be configured to accept only selected barcode symbologies. Furthermore, it is possible to define a barcode mask for *Patient ID* and *Operator ID*. With masks being defined, the meter checks a barcode after scanning and rejects it, if it does not match the mask. See “Example of barcode symbologies” on page 183 for further information.

## Stored test results and comments

When performing a test, the test result will be stored along with additional information, including the *Patient ID*, *Operator ID*, the kind of test performed, and optional *Comments*. The meter comes with a default set of *Comments* that can be assigned to each test result. If you are working with a DMS, these comments can be replaced by the comments in the DMS. The new wording will then be available for selection from the *Comments* list in the meter.

Up to 3 comments can be assigned to each result.

The *Comment* function may be configured to be either optional, required for test results which are out of range, or always required. If the user is required to enter/select a comment, completely empty comments will not be accepted by the meter.

The default preset comments in the meter are:

**Patient and QC Result comments**

- Symptomatic
- Asymptomatic
- Chest Pain
- Dyspnea
- Intermediate Zone
- Doctor Notified
- RN Notified
- Under Medication
- Will Repeat Test
- No Action
- New Strip Lot
- New Strip/Kit Lot
- New Lot Number
- Lab Draw
- Cleaned Meter
- Switched QC Vial
- Repeat Control Test
- Acceptable Control
- New Control Lot
- Procedure Error

**Maintenance comments**

- Meter Cleaned
- Samp. App. Area Cl. (Sample Application Area Cleaned)
- Test Strip Guide Cl. (Test Strip Guide Cleaned)
- New Battery Pack

## 8 Maintenance and Care

### 8.1 Conditions for storage and shipping

#### Storage

- Store the system and test strips in the same environment in which they are used.
- Do not store the meter in direct sunlight or under extreme temperature conditions.
- Observe the limits for temperature and humidity when storing and using the meter (see Chapter 10).

## Shipping



---

Observe the following safety information when shipping the meter and battery pack. Failure to do so may result in injury to persons or damage to the meter or battery pack.

---

- If the meter is shipped or transported over long distances, always remove the battery pack from the meter. This eliminates the possibility of the battery pack overheating due to a short-circuit in the meter. It also prevents deep discharge and other damage to the battery pack or meter.
- Only ship undamaged battery packs. Damaged battery packs must be disposed of locally. See page 18 for the risks associated with damaged battery packs and disposal information.
- Package the battery pack for shipping so that it cannot move around in the packaging. Also observe any other applicable national regulations.
- When shipping via third parties (e.g., by air or parcel service), work with the carrier to check whether specific requirements need to be met in relation to the lithium-ion battery packs on the basis of national or international laws on hazardous goods and, where applicable, if special packaging and labeling requirements apply.

For short distances - example between a facility's sites - users may transport the battery packs (either installed in the meter or separately) by road without having to meet further requirements.

## 8.2 Cleaning and disinfecting the meter

### Difference between cleaning and disinfecting

**Cleaning** is the physical removal of organic soiling (e.g., dirt, or other foreign material) from the meter surface.

**Disinfecting** is the removal of most, but not all, disease-causing and other types of micro-organisms (blood-borne pathogens) from the meter.

### When should the meter be cleaned and disinfected?

- Clean the meter to remove visible soil and organic material for safe handling and/or prior to disinfecting.
- You can use the *Cleaning Lockout* function (see page 106) to ensure regular checks are performed.

### What to clean and disinfect?

The following parts of the meter may be cleaned/disinfected:

- The visible area of the test strip guide
- The test strip guide cover
- The meter display (touchscreen)
- The meter housing (entire meter surface - front and back)



- 
- Observe the disinfection guidelines of your institution.
  - Use gloves.
- 

### NOTICE

---

Follow the procedures below to clean/disinfect the meter. Failure to follow these procedures may cause malfunction of the meter.

---

**NOTICE**

---

**Instrument damage due to moisture**

Ensure that no liquid enters the meter. If moisture enters the meter, it may cause malfunction of the instrument.

- Make sure the meter is powered off. Unplug the power adapter.
  - Do not spray anything onto the meter housing.
  - Do not spray into the test strip guide.
  - Do not immerse the meter in liquid.
  - Use only ordinary lint-free tissues.
  - Do not use cloths that are saturated. Squeeze off excess solution or blot on a dry paper towel to remove any excess solution before wiping the surface of the meter.
- 





## Recommended cleaning/disinfecting agents

Before using a cleaning/disinfecting solution on the meter refer to the product labeling for required contact time for cleaning/disinfecting and to ensure that the active ingredients are acceptable for use. Prepare all solutions per the manufacturer's product labeling.

Acceptable active ingredients are listed below.

### Recommended cleaning agents

- mild soapy water
- 70 % (or less) solution of isopropyl alcohol in water
- 0.625 % (or less) solution of sodium hypochlorite in water (bleach)

### Recommended disinfecting agents

- 0.625 % (or less) solution of sodium hypochlorite in water (bleach)
- quaternary ammonium compounds up to 0.5 % (single compound or mixture) in isopropyl alcohol (isopropanol) up to 55 %\*

---

**NOTICE**

Do not use any other disinfectants/cleaning agents on the meter (housing or the test strip guide cover). Use of other disinfectants/cleaning agents could result in damage to the meter.

---

## Cleaning/disinfecting the exterior (meter housing)

Use only the solutions recommended above.

Basic sequence of steps for cleaning

- **First** remove any blood and other dirt using water or soapy water.
- **Then** disinfect the meter's housing.

### **Cleaning after contamination due to mispipetting**

Observe the following precautions to prevent instrument damage

- Do not move the meter to prevent liquids from creeping and penetrating the meter.
- Dab away all visible blood (also on the test strip, but not in the application area).
- Do not use the meter before the liquid/moisture has been completely removed. While it is drying, the meter must not be moved as this may soil the optical components.

For detailed cleaning instructions (“how to clean...”) for the individual components of the meter, see the following pages. The instructions below describe only the sequence of steps to be taken in case of mispipetting.

- With the meter powered off, remove the test strip guide cover.
- Remove and dispose of the test strip.
- Clean the soiled parts of the meter with a moistened cotton swab/bud or tissue.
- Disinfect the meter.

## Cleaning the test strip guide cover



- 1 If required, remove the meter from the Handheld Base Unit and place it horizontally on a table.
- 2 Remove the test strip guide cover by pulling it forward horizontally (in the direction of the arrow).

Move the cover safely away from the meter and wipe it clean using the solutions recommended above.

In case of significant soiling or contamination, you can rinse the test strip guide cover with warm water, dry it with a fresh tissue, and then disinfect it using the solutions recommended on page 169.

Always ensure that the test strip guide cover is completely dry before re-attaching it to the meter.



- 3 Clean the outside of the meter with a lightly moistened tissue. Then dry the meter with a fresh tissue.

## Cleaning the visible area of the test strip guide




- 1 Clean the easily accessible and visible **pipetting field area** of the test strip guide with moistened lint-free tissues. Then dry the test strip guide with a fresh tissue.

Adhere to the following:

- Clean **only the visible area** of the test strip guide.
  - Do not insert any objects into the concealed areas of the test strip guide as this will damage the optical components of the meter.
  - Do **not** use objects to try to scratch off any dried contaminants in the test strip guide.
- 2 Clean the **membrane** (small circle) in the visible area at the end of the test strip guide with a moistened tissue.
  - 3 Allow the inside of the test strip guide to dry **for about 10 minutes**.
  - 4 After this time, re-attach the test strip guide cover to the housing. Make sure that the cover is properly attached. You will feel it snap into place.



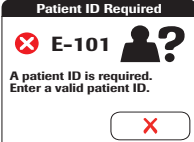

## 9 Troubleshooting

The **cobas h 232** meter continually checks its systems for unexpected and unwanted conditions. These may arise for technical reasons (defective components or consumables, environmental factors) or due to handling and procedure errors.

Depending on the circumstances, a message may appear on the display of the meter. These messages are marked with an icon, either  for a status message, or  for an error message. All messages displayed by the system are accompanied by a description of the error and a possible solution.

Take the action suggested on screen to resolve the problem. If the error disappears, you may continue using the meter as desired. If the problem persists, contact Roche (see page 197).

The two different message types are illustrated below:

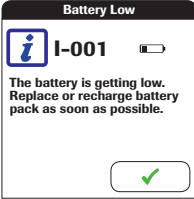Display/symptom	Description
<p><b>I-001: Battery Low</b></p> 	<p><b>Status message</b></p> <p>Touch  to proceed with the next step.</p>
<p><b>E-101: Patient ID Required</b></p> 	<p><b>Error message</b></p> <p>Touch  to exit this message. Perform the suggested step(s) to solve the problem.</p>

## Errors and unusual behavior without error messages

Some conditions may arise that have no associated error or status message.



Display/symptom	Description
<b>No message or unusual behavior</b>	
Meter display does not power on	<ul style="list-style-type: none"> <li>■ Wait 10 seconds and try powering on the unit again.</li> <li>■ Check that the meter has power.                             <ul style="list-style-type: none"> <li>▪ If you are using the external power adapter, is it connected properly to the meter? Or:</li> <li>▪ Place the meter in the Handheld Base Unit. Or:</li> <li>▪ If you are using the Handheld Battery Pack, is it properly installed in the meter?</li> </ul> </li> </ul>
Meter displays an unexpected result	Refer to the package insert for the test strips.
<b>Automatic shutdown</b>	
	The meter powers off after a configurable time without activity (e.g., pressing a key, touching the screen) to conserve energy. Reactivate the meter/screen as described in the following:
Shutdown after time specified by system administrator (default is 5 minutes, configurable by system administrator)	<ul style="list-style-type: none"> <li>■ Press the On/Off button of the meter.</li> </ul>

Display/symptom	Description
<b>Communication Problems with the Data Management System via RF</b>	
The cobas h 232 meter is unable to communicate with the Data Management System.	<ul style="list-style-type: none"><li data-bbox="396 236 1048 309">■ Check WLAN functionality on the meter (see the <i>Wireless Setup</i> screens in section “Diagnostics” starting on page 80).</li><li data-bbox="396 309 1048 373">■ Dock the RF enabled meter into a hard-wired base unit to transfer the data.</li><li data-bbox="396 373 1048 421">■ Reset the meter (see page 176).</li><li data-bbox="396 421 1048 469">■ If the error persists, contact your Roche representative.</li></ul>

---

## Meter reset

A meter reset should only be performed if all other remedies have failed.

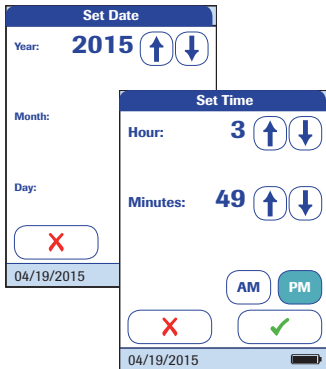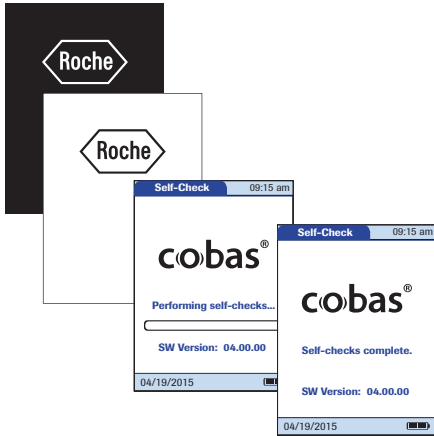

- 1 Place the meter on a level surface.
- 2 Press the **ⓘ** button for at least 12 seconds.
  - The meter powers off and on again.
  - The Roche logo is displayed. If the Roche logo does not appear within 60 seconds, place the meter in the Hand-held Base Unit for a minimum of 15 minutes to recharge the battery.
  - The meter performs a system check.
  - The screens for entering the date and the time appear.
- 3 Enter the date and time. After you have entered the correct information, confirm each screen with **✓**.
  - Unless all lockouts are disabled (see chapter 3.4), the meter will now be in QC Lockout due to the manually entered date/time.
- 4 Synchronize the meter's date/time with the date/time of your facility via the Hand-held Base Unit or, if working wirelessly, wait at least ten minutes for the next WLAN synchronization before performing any further tests.

Even if your configuration does not require it, we recommend always performing a QC test after a meter reset.



## 10 General Product Specifications

### 10.1 Operating conditions and technical data

#### Technical data

Temperature range	18-32 °C
Relative humidity	10-85% (no condensation)
Maximum altitude	4300 m
Position	Always place the meter on a level, vibration-free surface when applying the sample to the sample application area of the test strip. Once the sample has been completely absorbed by the test strip, you can move the meter.
Measuring range	Depends on test parameters
Memory	2000 Patient Test Results 500 QC Test Results 200 IQC Test Results 100 Strip Code Chips 100 QC Code Chips 4000 Patient List entries 5000 Operator List entries
Interface	Infrared interface, LED/IRED Class 1
Supported barcodes	Code 128, Code 39, Code 93, EAN 13, Interleaved 2 of 5, Codabar, QR Code, DataMatrix, PDF417, Aztec
Battery operation	Handheld battery pack
Mains connection	Power adapter: Input: 100-240 V AC ( $\pm 10\%$ ) / 50-60Hz / 350-150 mA Output: 12 V DC / 1.25A
Number of tests with fully charged battery	approx. 10 tests
Safety class	III
Automatic power-off	Programmable 1 ... 60 minutes
Dimensions	275 × 102 × 55 mm
Weight	approx. 650 g incl. battery pack and scanner

**Sample material**

Sample type	Heparinized venous whole blood
Sample size	150 µL
Interactions	Refer to test strip package insert

**Storage and transport conditions**

Temperature range	-25 °C to +70 °C
Relative humidity	10 to 85% (no condensation)

**10.2 Further information****Ordering information**

Please contact your specialist supplier.

<b>Item</b>	<b>Description</b>	<b>REF</b>
Roche CARDIAC POC Troponin T	10 tests for quantitative detection of cardiac Troponin T	07007302190
Roche CARDIAC POC Control Troponin T	Control set for use with Roche CARDIAC POC (control set for 2 x 6 quality control checks, level 1/2, and code chip)	07089643190
Roche CARDIAC T Quantitative (Troponin T)	10 tests for quantitative detection of cardiac Troponin T	04877772190
Roche CARDIAC Control Troponin T	Control set for use with Roche CARDIAC T Quantitative (control set for 2 x 6 quality control checks, level 1/2, and code chip)	04890515190
Roche CARDIAC Control Troponin T (Germany only)	Control set for use with Roche CARDIAC T Quantitative (control set for 2 x 6 quality control checks, level 2, and code chip)	05453879190
Roche CARDIAC M	20 tests for quantitative detection of myoglobin	04877799190
Roche CARDIAC Control Myoglobin	Control set for use with Roche CARDIAC M (control set for 2 x 6 quality control checks, level 1/2, and code chip)	04890469190
Roche CARDIAC D-Dimer	10 tests for quantitative detection of D-Dimer	04877802190
Roche CARDIAC Control D-Dimer	Control set for use with Roche CARDIAC D-Dimer (control set for 2 x 6 quality control checks, level 1/2, and code chip)	04890523190

<b>Item</b>	<b>Description</b>	<b>REF</b>
Roche CARDIAC proBNP+	10 tests for quantitative detection of NT-proBNP (measuring range 60 - 9,000 pg/mL for meter software version $\geq$ 01.04.01)	05533643190
Roche CARDIAC Control proBNP	Control set for use with Roche CARDIAC proBNP+ (control set for 2 x 6 quality control checks, level 1/2, and code chip)	04890493190
Roche CARDIAC CK-MB	10 tests for quantitative detection of CK-MB	04877900190
Roche CARDIAC Control CK-MB	Control set for use with Roche CARDIAC CK-MB (control set for 2 x 6 quality control checks, level 1/2, and code chip)	04890426190
Roche CARDIAC IQC	Set of two re-usable control strips (high/low) including code chip. Needed for checking the performance of the meter's optical system	04880668190
Roche CARDIAC pipettes	20 disposable syringe pipettes (150 $\mu$ L) for blood application	11622889190
Universal battery pack	Rechargeable battery pack for <b>cobas h 232</b> meter	04805640001
Test strip guide cover		04990315001
Battery compartment cover		04990307001
Power adapter		07006098001
Handheld Base Unit		04805658001
<b>cobas h 232</b> meter	Standard version without scanner	04901126190
<b>cobas h 232</b> meter with scanner	Patient/Operator ID can be scanned from a barcode	04901142190
<b>cobas h 232</b> Operator's Manual	Printed manual (English)	07469101001
<b>cobas h 232</b> Operator's Manual	Printed manual (German)	07532920001
<b>cobas h 232</b> Manual CD	Contains Operator's Manuals and Quick Reference Guides in all available languages (PDF)	07469314001
Printer <b>cobas h 232</b> (CMP-10) ROW	for most North American, Latin American and Asian countries	05404495001
Printer <b>cobas h 232</b> (CMP-10) EMEA	for most European, Middle East and African countries	05404517001
Printer <b>cobas h 232</b> (CMP-10) JAP	for Japan	05404509001
Normal thermal paper roll	5 rolls per package, 50 mm diameter	03666751001
Thermal paper roll (self-adhesive)	10 rolls per package, 50 mm diameter	05412951001

Note: Not all items are available in all countries.

### **Product limitations**

Please read the information in the package insert supplied with the test strips for detailed product data and limitations.

### **Information about software licenses**

This product incorporates software modules developed under open source licenses. The source code of this software can be requested on a standard data exchange medium from the manufacturer at the following address:

Roche Diagnostics GmbH  
Sandhofer Str. 116  
68305 Mannheim, Germany

The General Public License (GPL) licensing conditions are available (in English only for legal reasons) as a text file (file name "License\_txt.PDF") on the CD supplied with this manual. The complete license agreements are also stored as a text file (file name "license.txt") on the Handheld Base Unit. You can access this file by connecting the base unit to a PC with the USB cable. For detailed instructions on how to do this, see the Operator's Manual of the Handheld Base Unit.

Everyone is permitted to copy and distribute verbatim copies of this license document, but changing it is not allowed.

### **Repairs**

Please note that repairs and other modifications to the meter may only be performed by persons authorized by Roche.

## **11 Warranty**

The statutory guarantee provisions on rights in consumer goods sales in the country of purchase shall apply.

# A Appendix

## A.1 Working with barcodes



---

### Risk of barcode read errors

If a barcode is read incorrectly, this may lead to patient misidentification and therefore to inappropriate therapy decisions.

When creating patient or operator barcodes, always adhere to the applicable international IEC/ISO standards for the respective barcode symbology. In particular, ensure that barcode size and print quality (as defined in ISO/IEC 15416 and 15415) are adequate. Inadequate print size and/or quality may lead to erroneous decoding. In addition every user must carry out a plausibility check on all data scanned into and displayed by the instrument.

---

To reduce the probability of the barcode being misread, it is strongly recommended that you use the configuration options for patient and/or operator ID validation as applicable to your specific workflow. These options are:

- check ID against list or
- check ID for length<sup>1</sup>
- check if the scanned bar code type is allowed and activated in the DMS
- use barcodes with check digits

In combination with the above options or as a single measure, use an appropriate barcode mask if this is compatible with the structure of your barcode content.

Always make sure that the entire barcode is covered by the green light frame when scanning.

1. If no operator/patient list can be used, it is recommended that you at least set a minimum length for the respective ID, even if your facility uses IDs of varying length.

## A.2 Operator and patient ID barcode masks

Barcode mask character	Definition
A-Z, 0-9	If not preceded by the Caret (“^”), the scan data character must be the same as the mask character. This character is not saved as part of the ID. If the characters are not the same, the scan data is not a valid ID.
Dollar (“\$”)	The scan data character in this position is kept as part of the ID.
Asterisk (“*”)	The scan data character in this position is not kept as part of the ID.
Tilde (“~”)	The scan data character in this position must be a number, 0-9, and it is not kept as part of the ID. If the scan data character is not a number, the scan data is not a valid ID.
Plus (“+”)	The scan data character in this position must be an alpha character, A – Z, and it is not kept as part of the ID. If the scan data character is not an alpha character, the scan data is not a valid ID.
Caret (“^”)	This mask character denotes that the scan data character must be equal to the next character in the barcode mask after the “^”, and that the scan data character is kept as part of the ID. If the scan data character is not equal to the mask character following the “^”, the barcode reading is invalid as an ID.

The allowed maximum length of a barcode mask is

- 60 characters for 1D barcodes
- 300 characters for 2D barcodes

## A.3 Example of barcode symbologies

**EAN 13 barcodes, although widely used in retail, are less recommendable for patient/operator barcodes.** If used nonetheless:

- they demand the very highest quality standards of barcode creation and reproduction
- the start sequences 978 (ISBN) and 979 (ISMN) may not be used and will be ignored as a part of any ID code

**The use of Interleaved 2/5 barcodes is not recommended.**

The barcode samples shown here are for illustration purposes only. If printed out, they can be used to check the barcode scanner. However, they are not meant to be used as a reference for size or resolution of real patient or operator ID barcodes. When creating patient or operator barcodes always refer to the relevant standard ISO/IEC 15416 and 15415 for size and resolution requirements and to the specification listed below.

	<b>Recommended Specification</b>	<b>Remarks</b>
Print resolution	300 dpi preferred 200 dpi minimum	At 200 dpi issues with the wide-to-narrow ratio may exist.
Reflective contrast	70% or greater	Matte finish is preferred over gloss finish.
Symbol grade	Grade C or above Grade B is preferred  Symbol grades are A-F based on analysis of several quality elements.	Depending on quality grade parameters for a specific bar code, grade C may not be sufficient when motion, reflection, or poor lighting occur.
Module width (minimum)	0.16 mm (linear barcodes) 0.20 mm (2D barcodes)	



Codabar

1234567890



Code 39

1234567890



Code 93

ABC123-/+



Code 128

1234567890




EAN 13

1 112223 334448



Interleaved 2/5  
without checksum

1234567890

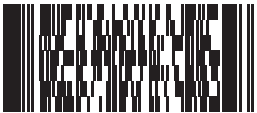

PDF417

Roche Professional Diagnostics



Aztec

Roche Professional Diagnostics



DataMatrix

Roche Professional Diagnostics



QR Code

Roche Professional Diagnostics



## A.4 Supported characters in 2D barcodes

The 2D barcode scanner is able to read characters from the following unicode ranges:

- Basic Latin (0021-007E)
- Latin-1 Supplement (00A1-00FF)
- Latin Extended-A (0100-017F)

The 2D barcode scanner does not support Asian characters.

This page intentionally left blank.

## B Appendix

### B.1 Option: Wireless network (WLAN)

#### Preliminary note

Appendix B has been developed to explain the wireless communication principles of the **cobas h 232** system<sup>1</sup> and to help your facility's information technology/management team(s) in effectively deploying the **cobas h 232** system on your wireless network.

Whether your meter has wireless capability or not depends on the system configuration that you purchased.

#### Background

The **cobas h 232** meter can only be configured through a data management system to communicate wirelessly. The data management system is also used to set-up and configure the meter to connect to one hospital specific Wireless Local Area Network (WLAN<sup>2</sup>). WLANs use electromagnetic waves in the 2.4 and/or 5.0 GHz frequency range to wirelessly transmit data<sup>3</sup>. The **cobas h 232** system adheres to IEEE Standard 802.11g (2.4 GHz range)<sup>4</sup>. The system is backwards compatible to 802.11b. During wireless communication to an Access Point (AP), the **cobas h 232** meter recognizes the existent AP WLAN protocol configuration (802.11b or 802.11g) and automatically transmits data using the appropriate communication protocol<sup>5</sup>.

1. The **cobas h 232** system is certified by the Wi-Fi Alliance.
2. WLAN is also commonly referred to as wireless LAN or Wi-Fi.
3. For the wireless functionality to work properly, the wireless module must first be configured by your system administrator.
4. While the **cobas h 232** system adheres to the 802.11g standard, it uses only channels 1-11. Channels 12-14 are not used by the system.
5. WLANs are organized in cells. A typical WLAN cell consists of Access Point(s) that are connected to the (wired) Local Area Network and one or more clients, e.g., **cobas h 232** meters along with other clients such as portable computers.

The loss of signal or access to bandwidth of one particular client may vary depending on one or more of the following situations: the type and number of other clients, the performance of the Access Point, the presence of electromagnetic disturbances, and other potential interfering factors, e.g., concrete walls.

The **cobas h 232** meter uses a burst-like communication protocol that will only consume bandwidth if there is actually data to be transferred. Compared to other applications, such as Voice over Internet Protocol (VoIP) or multi-media applications, the meter's bandwidth consumption is minimal. If the WLAN that the **cobas h 232** meter seeks to connect to is degraded, the meter design minimizes the impact on functionality.

### **Technical implementation**

Prior to connecting any wireless device to a wireless network, it is recommended that a WLAN site survey be performed. The goal of a WLAN site survey is to ensure that Access Points will provide enough coverage and performance to support any new radio frequency (RF) application or device. The survey will also detail RF signals, including all existing WLANs along with any competing RF signals and interferences (building structure related and other wireless equipment / devices).

As part of an RF implementation of the **cobas h 232** system, it is recommended that at minimum one base unit be hard wired per floor. A networked base unit provides redundancy if a wireless network malfunctions or loses service. If the **cobas h 232** meter with RF is used in an area with low signal or interferences, it is recommended to install a connected base unit for redundancy. The redundancy of the connected base unit allows immediate transmission of patient results when the meter is docked.

The current RF system consists of an antenna and a WLAN system-on-chip (SoC) along with other components. The WLAN system-on-chip is the core of the WLAN system. The RF system used in the **cobas h 232** meter specifically adheres to the following specifications:

- Its WLAN system-on-chip supports IEEE 802.11b and 802.11g. It works seamlessly together with other Wi-Fi certified transceivers. It also implements the Wi-Fi Protected Access (WPA™ - Enterprise and WPA™ - Personal), Wi-Fi Protected Access 2 (WPA2™ - Enterprise and WPA2™ - Personal), and Wired Equivalent Privacy (WEP) security mechanisms with Temporal Key Integrity Protocol (TKIP) and Advanced Encryption Standard (AES).  
In addition, the system supports the extensible authentication protocol (EAP) with EAP-TLS, EAP-TTLS/MSCHAPv2 and PEAPv0/EAP-MSCHAPv2. The **cobas h 232** meter interoperability Wi-Fi certificate can be accessed at [http://certifications.wi-fi.org/search\\_products.php](http://certifications.wi-fi.org/search_products.php).  
Further information including a glossary of terms, frequently asked questions, and other topics related to Wi-Fi technology can be found on the Wi-Fi Alliance site (<http://www.wi-fi.org/>).
- The used channels in the 2.4 GHz-band are channels 1-11. (Channels 12-14 are not used by the **cobas h 232** meter.)
- RF output power is approximately 15 dBm at a data rate of 54 MBPS.

## RF specific functionalities and effective performance claims

The **cobas h 232** system offers the option of wireless network connectivity (WLAN/Wi-Fi). If you purchase a system with this option, the WLAN function is enabled during production.

This module can only be configured by a data management system (DMS), which activates the meter's wireless communication and data transfer capabilities. Wireless connectivity can help to ensure that updates to information in the DMS are sent immediately to all networked meters.

Meters with an integrated and activated wireless option use the base unit for recharging and/or as a redundant communication option to exchange data with the DMS.

The meter also has to be docked if the hospital changes security protocols. When this change occurs, it may lock out all meters until docked and reconfigured with the new protocol.

As described above, the **cobas h 232** meter supports the 802.11g standard. This translates into the following RF specific performance claims:

- The **cobas h 232** meter is capable of transferring to a suitable DMS, via WLAN, a data set of up to 1000 result records, 100 reagent records, and 500 operator ID records in less than 15 minutes, when operated in a typical WLAN environment (correct WLAN administration, typical population of other clients present, any of the supported security models enabled).
- A **cobas h 232** meter with wireless connectivity enabled will communicate results after every test or when the meter is idle; it will automatically attempt to communicate with the DMS every 10 minutes. In line with the industry communication standard POCT1-A, the DMS must acknowledge the meter's request for connection and actively query for the result. Only upon receipt of this DMS query, the meter will send the result. Hence the effective time for transmitting results depends on infrastructure, DMS workload, etc. Once the DMS sends a query, however, the meter will respond within a few seconds.

A typical range for direct connection between the **cobas h 232** meter and the access point (air, direct view, low disturbances) is up to 15 to 20 meters (49 to 66 feet). The actual range depends on the positioning of the access point's antennas and other topological properties of the space between WLAN device and AP. Additionally, dynamic control of the transmitting power of the access point may reduce the maximum distance between WLAN device and AP within which communication can be guaranteed.

The **cobas h 232** system is designed such that it coexists with other wirelessly communicating devices. The **cobas h 232** system does NOT include any real-time or even time critical wireless functionality. It communicates exclusively single, digital data fields. It does NOT communicate continuous waveform data.

Note: A degraded Quality of Service (QoS) of the WLAN communication will not impact the functionality of the meter but may delay the communication of results to the DMS. Users should be aware that **real-time communication** of test result readings **cannot be guaranteed** by the **cobas h 232** meter.

This page intentionally left blank.




## C Supplement for Observed Test Sequence


### **Observed Test Sequence (OTS)**

The *Observed Test Sequence* (OTS) function allows an observer (supervisor) to assess and record an operator's performance (e.g., for recertification purposes). The observer monitors an operator during a test to check that the test is being performed according to the recommended procedures. He/she then evaluates the performance and passes or fails the operator. This assessment is saved together with the test result and any desired comments.

*Observed Test Sequence* options can only be configured using a DMS. The availability of electronic configuration options will thus vary according to the data management software utilized by your institution. Consult your system administrator.

## Using the OTS function

A request for an Observed Test Sequence comes from the DMS. The presence of the  icon on the *Patient Test* button indicates a pending OTS request.

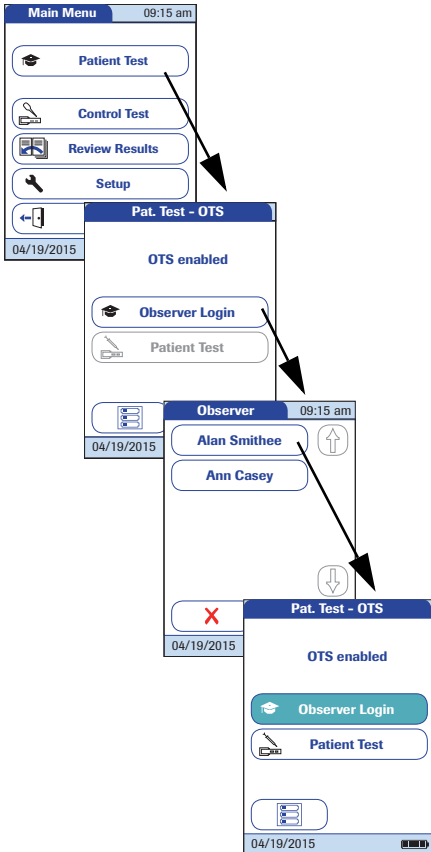

### Observer:

#### 1 Touch **Patient Test**.

In the *Pat. Test - OTS* menu, the **Patient Test** button is grayed out (disabled) until the observer has logged in.


#### 2 Touch **Observer Login**.

3 Wait until the observer list is displayed.




4 Select your observer ID by touching the corresponding button, or scan your operator ID (which is also your observer ID in this case)..

Only operators with OTS observer rights are listed in the Observer Login list.

5 Enter the (optional) password.

6 After you enter your password, touch  to log in. The *Pat. Test - OTS* menu is displayed again. The **Patient Test** button is now active.

7 Hand the meter to the operator who can now perform the patient test under supervision.

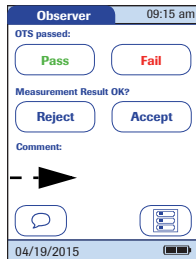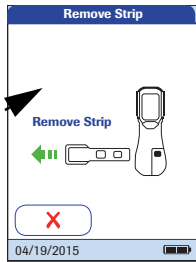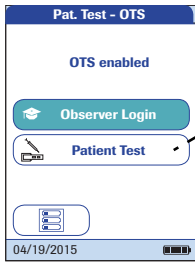

## Operator:

### 1 Touch **Patient Test**.

Perform the patient test as usual. Once the test is completed, the observer has to complete the next steps.

### 2 Hand the meter back to the observer.

## Observer:

### 3 Touch to log in again.

### 4 After you enter your password, touch to proceed with the assessment.

### 5 Assess the operator's performance by touching **Pass** or **Fail**.

### 6 Assess the test result by touching **Accept** or **Reject**.

### 7 Touch to add a comment.

### 8 Touch to return to the *Pat. Test - OTS* screen.

The OTS information is saved together with the test result.

This page intentionally left blank.

## D Contact Roche

For all questions about the **cobas h 232** system that are not answered in this manual, contact your Roche representative. If you do not already have contact details, visit our website on [www.roche.com](http://www.roche.com). Select "Roche in your country" at the top of the page and then select your country to find the appropriate local office contact information.

The **cobas h 232** system is manufactured for and distributed by:

Roche Diagnostics Australia Pty Limited  
ABN 29 003 001 205  
31 Victoria Avenue  
Castle Hill, NSW, 2154

This page intentionally left blank.

# Index

## A

Administrator .....	83
Administrator ID	
Change .....	87
Disable .....	88
Set up .....	83–86
Application area .....	29
Apply sample .....	129
Opening (meter) .....	25
Auto off .....	62

## B

Barcode .....	121, 126
Masks .....	182
Symbolologies .....	184
Barcode (test strip) .....	29
Barcode reader .....	29
Barcode scanner .....	25
Basics (setup) .....	49–63
Battery pack .....	27
Installing or replacing .....	36–40

## C

Cleaning .....	167–172
After mispipetting .....	170
Housing - outside .....	171
Test strip guide .....	172
Cleaning lockout .....	106
cobas h 232	
Contents of the pack .....	13
Introduction .....	11
Operating conditions .....	177
Overview of meter elements .....	24–26
Test principle .....	13
cobas IT 1000 .....	11
Code chip .....	29, 116–117
Opening .....	26
Comment (custom) .....	134
Comment result .....	134
Computer .....	64–68, 160
Connection .....	64–70

Contact Roche .....	197
Contents of the pack .....	13
Contrast .....	49–50
Control material .....	138
Custom range .....	103, 104

## D

Data handling .....	64–81
Data Management System (DMS) .....	73, 83, 91, 92, 159
Data transfer .....	159
Date .....	53–54
Diagnostics .....	80–81
Display format .....	57–58
Disposal .....	17, 115

## E

Error messages .....	173
----------------------	-----

## H

Handheld Base Unit .....	36
Handheld battery pack .....	25, 35

## I

Icons: see Symbols	
ID setup .....	82–93
Overview .....	82
Infections (safety instructions) .....	115
Infrared interface .....	26, 159
IQC lockout .....	101

## L

Language .....	51–52
Lockout .....	94–108
Cleaning lockout .....	106
IQC lockout .....	101
New lot lockout .....	100
Operator lockout .....	95–97
QC lockout .....	100

<b>M</b>	
Meter	
Reset	176
Meter setup	
Administrator ID	83–86
Auto off	62
Basics	49–63
Cleaning lockout	106
Computer	64–68
Contrast	49–50
Data handling	64–81
Date	53–54
Diagnostics	80–81
Display format	57–58
ID setup	82–93
IQC lockout	101
Language	51–52
Lockout	94–108
New lot lockout	100
Operator ID	89–91
Operator lockout	95–97
Optional screens	109–111
Overview	44–48
Patient ID	92–93
Printer	69–70
QC lockout	100
QC result format	102, 103
QR Code	65–67
Reset test parameters	105
Result confirmation	111
Result display mode	78–79
Result login	110
Result memory	71–74
Result unit	75–77
Sound	59–61
Start info	110
STAT test	107
Time	55–56
<b>N</b>	
New lot lockout	100
<b>O</b>	
Observed Test Sequence (OTS)	193–195
On/Off button	25
Operating conditions	177
Operating the meter	35–??
Operator ID	89–91, 120–122
Operator list	121, 161
Operator lockout	95–97
Optional screens	109–111
Ordering information	178
Overview of meter elements	24–26
<b>P</b>	
Patient ID	92–93, 124–126
Patient list	124–126, 162
Power adapter	
Connection	26, 30
Power supply	27
Contacts (Handheld Base Unit)	26
Powering on the meter	119, 139, 151
Printer	69–70
Product specifications	177
<b>Q</b>	
QC lockout	100
QC result format	102, 103
Quality control	137–149
Control material	138
IQC (instrument-specific)	147–149
Pass/Fail	149
Preparation	137
QC (test parameter-specific)	139–145



## R

Reset .....	176
Reset test parameters .....	105
Result confirmation .....	111, 133
Result deletion (automatic) .....	74
Result display mode .....	78–79
Result login .....	110, 132
Result memory .....	71–74, 151
Result unit .....	75–77
Review results .....	151–164
All results .....	154
Data transfer .....	159
IQC (instrument quality control) .....	156
Maintenance .....	157
Patient history .....	153
QC (quality control) .....	155
RF (radio frequency) .....	22, 188

## S

Safety	
Protection against infection .....	15
User qualification .....	15
Sample material .....	115, 178
Scan (button) .....	121, 126
Self-check (powering on the meter) .....	119
Software license .....	180
Sound .....	59–61
Specifications .....	177
Start info .....	110
STAT Test .....	94
Perform .....	136
STAT test	
Configure .....	107
Storage .....	165
Storage and transport conditions .....	178
Symbols	
Code chip .....	128, 141
Error messages .....	173
Hourglass .....	130, 143, 144
Meter .....	5
Packaging .....	5
Pipette .....	128, 142
Screen .....	31–33
Thermometer .....	128, 142

## T

Test .....	113–136
Performing .....	123–136
Preparing .....	115–122
Sample material .....	115
Test area .....	29
Test principle .....	13
Test steps (overview) .....	118
Test strip	
Inserting .....	127, 140
Lot .....	98
Test strip guide .....	26
Test strip guide cover .....	25
Time .....	55–56
Touchscreen .....	25, 43
Troubleshooting .....	173

## W

Wi-Fi .....	22
WLAN .....	187

This page intentionally left blank.





ROCHE CARDIAC, COBAS, COBAS H  
and IQC are trademarks of Roche.



Roche Diagnostics GmbH  
Sandhofer Strasse 116  
68305 Mannheim, Germany

[www.roche.com](http://www.roche.com)

[www.poc.roche.com](http://www.poc.roche.com)

0 7469101001 (01) 2015-XX EN