


Does Smart packaging a paper insert that is provided with the SmartCap

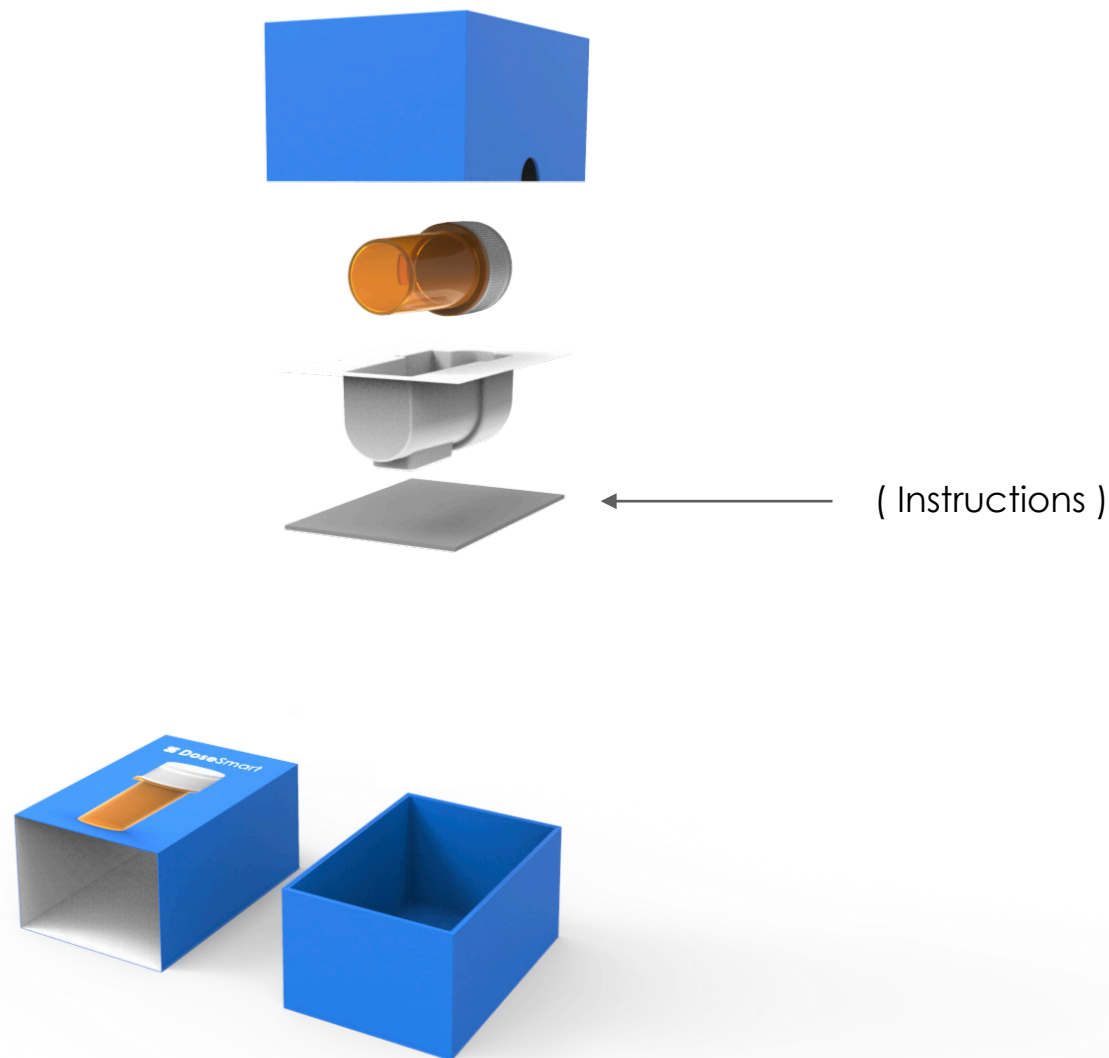

(Front: Instructions)

Step 1: Installation

Download DoseSmart from the Apple App store (iPhone) or from the GooglePlay store (Android).

Step 2: Register/Account Creation

Open the DoseSmart App and it will guide you through the registration process.

Step 3: Create a medication schedule

Press the "medicine" tab in the navigation bar or press ⊕ in the top right hand corner of the app and follow the steps.

Step 4: Add a smart pill bottle (if you have one)

Enable Bluetooth on your smart phone.
When you add a medication schedule you will see the option "use smart cap" press the toggle & follow the steps.

Step 5: Report that you took the medication

When you receive a reminder, remove the cap from your pill bottle & it will register a "take" in your DoseSmart App. If not using the smart cap, open the app & press the medication name to register a "take" under the "today" tab.

Leave the DoseSmart app running in the background of your phone in order to have data transfer from the smart cap to the phone, and for reminders to automatically disable.

Back: Instructions)

Please read the following before proceeding:

Note: This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by DoseSmart could void the user's authority to operate the equipment.

Does Smart | Exploded View | Packaging