

WFM-1 Wireless Floor Mat Transmitter User Manual

The WFM-1 Wireless Floor Mat alarm transmitter works together with a pressure sensitive floor mat to send an alarm to a WR-1 or WR-2 Alarm Receiver when pressure is exerted on mat. This is to give caregiver warning that someone may try to exist a door way.

1. Installation

- 1.1 Connect the transmitter to the power cable of the floor mat.
- 1.2 Properly put the transmitter into the pocket of the floor mat. Keep the connecting cable as far way from the transmitter as possible
- 1.3 Program the transmitter ID into the WR-2 receiver. Refer to the Operation Procedure of WR-2 Receiver.
- 1.4 Position the floor mat in front of the door exit.
- 1.5 Test completion of installation by stepping over the mat to trigger an alarm.
- 1.6 Test opening and closing of doors to make sure that there is no obstruction to freely opening of doors.

2. Normal Operation

- 2.1 Do not put objects on top of mat surface. Doing this will defeat the alarm function.
- 2.2 Unplug the power cable if the alarm function is not required for a long period of time.

3. Transmitter Specification

RF frequency: 915 Mhz ISM band transmitter
Power: 3 V lithium battery

NOTE:

THE MANUFACTURER IS NOT RESPONSIBLE FOR ANY RADIO OR TV INTERFERENCE CAUSED BY UNAUTHORIZED MODIFICATIONS TO THIS EQUIPMENT. SUCH MODIFICATIONS COULD VOID THE USER AUTHORITY TO OPERATE THE EQUIPMENT.