[www.samsung.com](http://www.samsung.com)

SAMSUNG  
ELECTRONICS

Type STA. A00151  
05/2013. Rev.1.0

# Heart Rate Monitor

## User manual

# Contents

English .....	1
Español .....	13

English

- *This manual's images and content may differ from the product and is subject to change without prior notice.*
- *This product is only a health assisting device. It cannot be used for medical diagnostic purposes.*

## Safety precautions

**Do not use the device for anything other than its intended use**

**Do not disassemble, modify, or repair the device**

Any changes or modifications to the device can void your manufacturer's warranty.

For service, take the device to a Samsung Service Centre.

**When using the device, mind the following:**

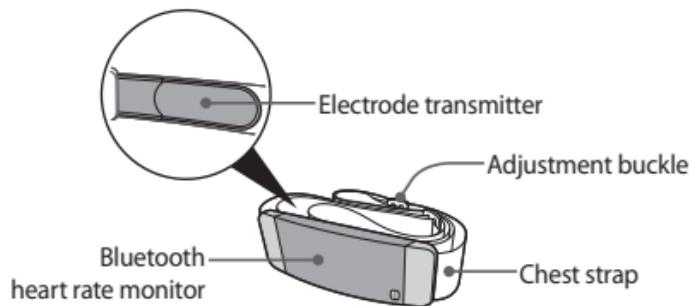
- Humidity and liquids may damage parts or electronic circuits.
- Do not store the device in hot areas. High temperature may cause damage to the device or shorten battery life.
- Do not drop the device or cause impacts to it.

1

## Overview

2

English

## Detaching the chest strap

Firmly hold the Bluetooth heart rate monitor with both hands, and then bend one side of plastic strap connector downwards. Both strap connectors can be removed simultaneously.

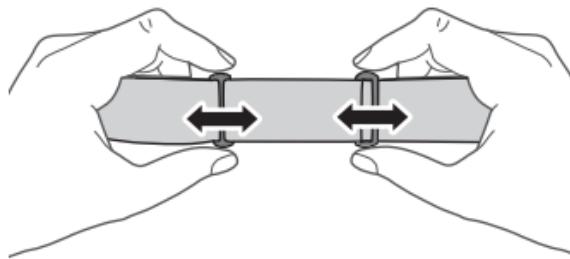
Do not pull directly on the chest strap's elastic to detach it from the Bluetooth heart rate monitor. Doing so may damage the chest strap.

3

## Adjusting the chest strap length

Hold both of the adjustment buckles on the chest strap and move them from side to side to adjust the length of the elastic.

4


English

## Wearing the Bluetooth heart rate monitor

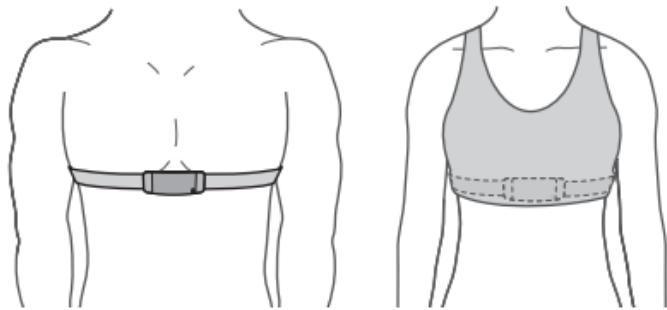To get the best results, make sure to wear the Bluetooth heart rate monitor directly against the skin. Women should wear it under a sports bra.

5

- 1 Detach one snap button from the Bluetooth heart rate monitor.
  - 2 Check the R (right) and L (left) symbol imprinted on the Bluetooth heart rate monitor, and align it correctly on the body when wearing it.
  - 3 Position the chest strap right below the upper chest muscles.
  - 4 Adjust the length of the strap to fit tightly, but comfortably, around the chest.
- 

- To ensure reliable measurement, moisten the electrode transmitter inside the chest strap with clean water or electrode gel before wearing it.
  - If the heart rate is too weak, the heart rate may not measure correctly.

6

English

## Connecting to a smartphone

- 1 Download and install the **Running pro** application from **Samsung Apps**.

To connect to a smartphone and measure heart rate, the **Running pro** application must be installed on the smartphone.

- 2 Wear the Bluetooth heart rate monitor, and then launch **Running pro** on the smartphone.
- 3 Select **Setting → Accessory setting**. The smartphone searches for the Bluetooth heart rate monitor automatically.
- 4 Select the device from the list of available devices and tap **Connect** to pair.

7

- Tap and hold a displayed name to modify the name of a device in the list.
- To set the smartphone to automatically reconnect to a previously paired device, set **Auto Connect** to **On**.
- Even if the Bluetooth heart rate monitor is paired with the smartphone, users cannot measure heart rate without launching the **Running pro** application.
- The heart rate may not be measured accurately when the Wi-Fi and Bluetooth features are turned on at the same time.

English

## **Replacing the battery**

Do not use a fingernail to open the cover. Doing so may damage the fingernail.

- 1 Firmly hold the battery cover of the Bluetooth heart rate monitor with one hand, and insert a coin to twist and open the cover.
- 2 Open the cover and remove the battery.
- 3 Insert a new battery with the positive side facing upwards.
- 4 Close the cover firmly.

- The remaining battery level can be checked on the smartphone.
- Do not open the battery cover aggressively. Doing so may damage the cover.
- The CR2032 battery for the Bluetooth heart rate monitor is sold separately.
- When using the Bluetooth heart rate monitor for 1 hour a day, the average lifespan of the battery is approximately 1 year.

English

## Storage and maintenance

- Rinse the chest strap with clean water after every use.
- Do not soak, iron, dry clean or bleach the strap.
- Store the device in a cool, dry place.
- Do not store the device in humid place, or where the device may get wet. Doing so may activate the device automatically and drain the battery.

11

## FCC

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:  
(1) This device may not cause harmful interference, and (2) this device must accept any interference received,  
including interference that may cause undesired operation.

Users are not permitted to make changes or modify the device in any way.

Changes or modifications not expressly approved by Samsung will void the user's authority to operate the  
equipment.

1

### Recycling Direct

To dispose electronics properly, follow the recycling process regulated in your area or go to Samsung website to find  
the nearest recycling location:

[www.samsung.com/recyclingdirect](http://www.samsung.com/recyclingdirect)

Or call, (877) 278 - 0799.

### Samsung Telecommunications America, LLC

1301 E. Lookout Drive

Richardson, Texas 75082

Phone: 1-800-SAMSUNG

Phone: 1-888-987-HELP (4357)

## RF Exposure Statement (2.1091)

FCC RF Radiation Exposure Statement: This equipment complies with FCC RF Radiation exposure limits set forth for an uncontrolled environment. This device and its antenna must not be co-located or operating in conjunction with any other antenna or transmitter.

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### Part 15.105 (B)

Note : This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

## IC Statement

Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

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