


SIZE:72\*97MM 正面

Automatic Wrist Bluetooth Blood Pressure Monitor DW702 Instruction Manual

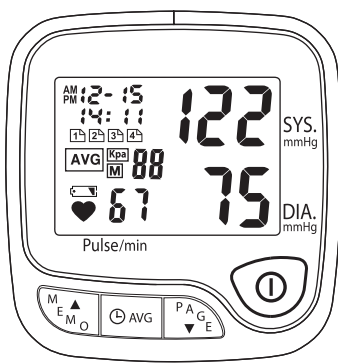

Setup Procedures Installation
Prior to first use, Please download the TrulyHealth application from Apple store and install it to your iOS device.

SAFETY PRECAUTIONS

Indications for use
Truly Automatic Wrist Bluetooth Blood Pressure Monitor DW702 intended to measure the systolic and diastolic blood pressure and pulse rate of an adult individual by using a non-invasive technique in which an inflatable cuff is wrapped around the Wrist.

- Don't not use this device for diagnosis or treatment of any health problem or disease, Measurement results are for reference only, consult a healthcare professional for interpretation of pressure measurements.

- This product is not suitable for people with arrhythmias or serious arteriosclerosis. It may have difficulty determining the proper blood pressure for pregnant women and for users with irregular heartbeat, diabetes, pre-eclampsia, poor circulation of blood, measurement on the arm on the side of a mastectomy, kidney problems or for users who have suffered from a stroke;

- When you are using the monitor to measure especially frequently used, the arm is compressed by the cuff, and then the fingertip will be caused congestion. During the congestion, please loosen the cuff and lift the hands over your head, and make your right and left hand squeeze and stretch 15 times, them will unlock the congestion;

- Don't calling around the Blood pressure monitor when it is using. Don't move and avoid electromagnetic interference and noise interference when measuring that to avoid to measure error.

- Do not use old and new batteries together.
- Align the polarities of batteries correctly.

QUICK START GUIDE

When the screen display the battery for one degree. it means that low Voltage and must change the new one to void to measure error.


Attention: Don't disassemble, repair and renovation ontology.

Installing the APP
1) Download and install the Truly Healthcare App onto your Bluetooth 4.0 supported smart devices. The free Truly Healthcare App is available on the App Store for the Apple devices and Google Play for the Android based devices.

Warning:
1. After press the "Q" button, the color of "E" on the iOS application screen is always gray. It means Monitor can connect to the application, please repeat the operation step 4-6.

When using more than one monitor with the Application, make sure that the WHO Light in the subject monitor will turn on and in Green during inflation.

OVERVIEW

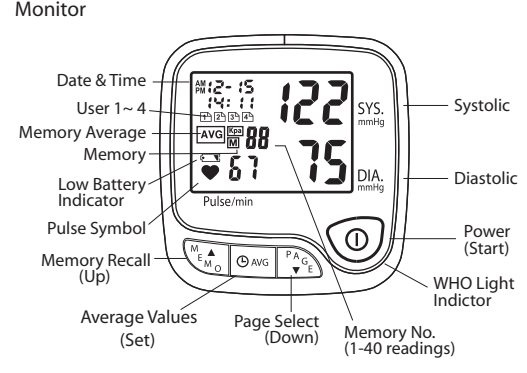

Features
- One Touch Operation
- Irregular Heartbeat Detector
- Blood Pressure Mode and Clock Mode
- 4 User / 160 Total Memory
- 4 Alarm Clock Settings
- Averaging Mode - The average value of the last 3 measurements stored in the memory will display
- Auto-off Power Saving Feature
- Operated by 2 AAA batteries

PREPARATION

2.1 Battery Installation
1. Remove the batteries cover in according with the direction of arrow.

2.2 Date & Time Setting

Note:
- Set the correct date and time before taking a measurement.
- Date and time should be reset after replacing new batteries.

2.3 Alarm Setting (This alarm is only tips for patient)

Note:
- You will not be able to set alarm clock if Clock Function is set to "OFF".
- You can set a maximum of 4 alarms.

2.4 Language Setting

When a measurement is completed, the results will read aloud: the systolic value, diastolic value and pulse. The default language is English.

SIZE:72\*97MM 反面

TAKING MEASUREMENT
Note:
- Avoid smoking, eating or exercising for 30 minutes before taking measurement.
- Do not move or talk during measurement.
- Measurement can be taken on either arm.

3.2 Applying the Wrist Cuff
1. Place your elbow on a table and palm facing upward, and wrap the arm cuff around the upper left arm.

3.4 Taking Measurement
1. Press the "Q" button. The cuff will automatically inflate and the measurement will start.

Note: An irregular heartbeat is defined as a heartbeat rhythm that has a variation of more than 25% from the average rhythm detected during measurement.

MEMORY FUNCTIONS

After measurement, the last set of measured values will be saved automatically after measurement when the unit is turned off by pressing "Q" button or by automatic power-off after 3 minutes.

4.3 Delete Past Readings
User can delete all the past readings stored in each User Page.

TROUBLESHOOTING

Table with 3 columns: Problem, Cause, Solution. Includes issues like 'No display when you press the Power /Start button', 'E\*P\*displayed', etc.

STORAGE & MAINTENANCE

- Keep away the unit from direct sunlight, extreme temperatures, humidity or moisture.
- Use a dry, soft cloth to clean the unit, or if desired, use a cloth lightly dampened with water.

Measurement Range: Pressure: 20 ~ 280 mmHg, Pulse Rate: 40 ~ 195 beats/min. Accuracy: Pressure: +/- 3 mmHg, Pulse Rate: +/- 5%.

EMC Declaration

EMC Declaration table with columns: Immunity test, EMC test level, Compliance, Electromagnetic environment - guidance.

Guidance and manufacturer's declaration - electromagnetic immunity

Table with columns: Immunity test, EMC test level, Electromagnetic environment - guidance.

Guidance and manufacturer's declaration - electromagnetic emissions

Table with columns: Emission test, Compliance, Electromagnetic environment - guidance.

Disposal

At the end of its use don't dispose the appliance, including removable parts and Accessories, together with the other urban waste, but conformally to 2002/96/EC.

Reorient or relocate the receiving antenna.

Increase the separation between the equipment and receiver. Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

SPECIFICATIONS

Model No. DW702, Measurement Method Oscillometric.

Net Weight 111g (batteries excluded)
Cuff Dimensions Wrist circumference of 135~220mm
The pressure of the running used life span of battery 1000 times

Application part (out of contact the body) connect tub, shell (contact the body) cuff

Package Content Instruction Manual, Battery Set, Warranty Card

Manufactured in accordance with International Standards
Refer to instruction manual
Keep dry
Type BF applied part
General warning (not protection against the ingress of water)

Manufacturer: Truly Instrument Ltd, 2/F, Chung Shan Kitting Centre, 1-3 Kiling Tip St., Kwai Chung, N.T., Hong Kong

Rev 01 2017/01/06

Rev 01 2017/01/06

Rev 01 2017/01/06

Rev 01 2017/01/06

Rev 01 2017/01/06