

# Propaq® LT Vital Signs Monitor

---



---

## Directions for Use

Software version 1.0X

**WelchAllyn®**

Advancing Frontline Care™

Copyright 2005 Welch Allyn. All rights are reserved. No one is permitted to reproduce or duplicate, in any form, this manual or any part thereof without permission from Welch Allyn.

Welch Allyn assumes no responsibility for any injury to anyone, or for any illegal or improper use of the product, that may result from failure to use this product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual.

Welch Allyn®, Propaq®, Acuity®, Smartcuf®, and Flexible Monitoring® are registered trademarks of Welch Allyn. FlexNet™ and ParamSet™ are trademarks of Welch Allyn.

Nellcor® and Oximax® are registered trademarks of Nellcor Puritan Bennett.

Software in this product is Copyright 2005 Welch Allyn or its vendors. All rights are reserved. The software is protected by United States of America copyright laws and international treaty provisions applicable worldwide. Under such laws, the licensee is entitled to use the copy of the software incorporated with this instrument as intended in the operation of the product in which it is embedded. The software may not be copied, decompiled, reverse-engineered, disassembled or otherwise reduced to human-perceivable form. This is not a sale of the software or any copy of the software; all right, title and ownership of the software remain with Welch Allyn or its vendors.

For information about any Welch Allyn product, call the nearest Welch Allyn representative:

USA	1 800 535 6663 + 1 315 685 4560	Australia	+ 6129 638 3000 800 074 793
Canada	1 800 561 8797	China	+ 86 216 327 9631
European Call Center	+ 353 46 906 7790	France	+ 3315 569 5849
Germany	+ 49 747 792 7186	Japan	+ 8133 219 0071
Latin America	+ 1 305 669 9003	Netherlands	+ 3115 750 5000
Singapore	+ 656 419 8100	South Africa	+ 2711 777 7555
United Kingdom	+ 44 207 365 6780	Sweden	+ 46 85 853 6551

This device complies with Part 15 of the FCC rules and with the rules of the Canadian ECES-003. Operation is subject to the following two conditions: (1) This device may not cause harmful interference and (2) this device must accept any interference received, including interference that may cause undesired operation.

**Caution!** Changes or modifications not expressly approved by Welch Allyn could void the purchaser's authority to operate the equipment.

**Reorder Part Number 810-1828-XX**

Manual Part Number 810-1702-00 Rev A, November 2005

Welch Allyn  
8500 SW Creekside Place  
Beaverton, Oregon 97008-7107

[www.monitoring.welchallyn.com](http://www.monitoring.welchallyn.com)

Printed in USA



# Contents

---

<b>1 - Introduction</b> .....	<b>1</b>
Intended Use .....	1
Symbols .....	2
Safety .....	5
General Warnings .....	5
General Cautions .....	7
Controls, Indicators, and Connectors .....	8
Features and Functions .....	10
Models .....	10
Accessories .....	11
USB Option .....	11
HIPAA Considerations .....	12
Monitor .....	12
Configuration Utility .....	12
AutoPrint Utility .....	12
<b>2 - Overview of Monitor Operation</b> .....	<b>13</b>
Turning On the Monitor .....	13
Selecting a Language .....	15
About the Charging/Communications Cradle .....	16
About the USB Data Transfer Option .....	17
Setting Up the Cradle .....	17
Removing the Monitor from the Cradle .....	18
Displaying Data .....	19
About Display Formats .....	21
Timing Out the Display and the Back Light .....	23
Locking Out the Display, Back Light, and Buttons .....	23
Large Color Display Interface .....	24
About Navigation .....	32
Using the Arrow Buttons .....	32
Using the Action Button .....	32
Using the Display Button .....	32
Using the Highlights .....	33
Menus .....	34
Using Control Menus .....	34
Using Setup Menus .....	37
About Monitor Information Screens .....	39
Using Demo Mode .....	41
Power Saving .....	45
Turning Off the Monitor .....	45
Communicating with an Acuity Central Station .....	45

About Error Detection . . . . .	45
Transporting the Monitor with the Patient . . . . .	46
<b>3 - Standalone Monitoring . . . . .</b>	<b>49</b>
Overview . . . . .	49
About the Model 802LT0N (Standalone) Monitor . . . . .	49
About the Model 802LTRN (Wireless) Monitor . . . . .	49
Preparing for a New Patient . . . . .	50
Continuing to Monitor a Patient on Power-Up . . . . .	54
Monitoring ECG and Resp . . . . .	55
Overview . . . . .	55
Safety . . . . .	55
Monitoring ECG . . . . .	57
Monitoring Respiration . . . . .	62
Monitoring SpO <sub>2</sub> . . . . .	64
Safety . . . . .	64
Procedure . . . . .	65
About SpO <sub>2</sub> Spot Check . . . . .	66
Monitoring Blood Pressure (NIBP) . . . . .	69
Safety . . . . .	69
Improving NIBP Accuracy with Smartcuf . . . . .	71
Taking Automatic NIBP Readings . . . . .	72
Taking NIBP Readings Using Turbo Mode . . . . .	72
Taking NIBP Readings Using the Digital Manometer . . . . .	73
NIBP Measurements in Power-Saving Mode . . . . .	74
NIBP Disabled When the Battery is Low . . . . .	74
To Discontinue Monitoring . . . . .	75
<b>4 - Monitoring in Communication with Acuity . . . . .</b>	<b>77</b>
About Wireless Monitoring . . . . .	77
Establishing Communication with Acuity . . . . .	78
Monitoring a Patient Outside of Network Range . . . . .	79
About Acuity Message Menus . . . . .	79
<b>5 - Alarms and Alerts . . . . .</b>	<b>85</b>
Overview . . . . .	85
Silencing an Alarm or Alert Tone . . . . .	86
Suspending the Alarm Tone . . . . .	87
Customizing Alarm Limits . . . . .	89
At the Monitor . . . . .	89
At Acuity . . . . .	89
About ParamSet . . . . .	90
Responding to an Alarm . . . . .	90
Responding to an Alert . . . . .	90
About Battery Charge Status . . . . .	91
Alert Messages and Status Messages . . . . .	92
<b>6 - Storing and Reviewing Patient Data . . . . .</b>	<b>95</b>
Overview . . . . .	95
Capturing a Data Snapshot . . . . .	96
Reviewing Data at the Monitor . . . . .	96
Reviewing Snapshots . . . . .	96

Reviewing Trends . . . . .	98
Reviewing Data at a PC . . . . .	101
Reviewing Data at Acuity . . . . .	101
<b>7 - Printing Patient Data . . . . .</b>	<b>103</b>
Overview . . . . .	103
Printing . . . . .	104
AutoPrint Options . . . . .	107
Enable/Disable AutoPrint . . . . .	107
Printing Manually . . . . .	107
Canceling a Print Request . . . . .	108
When Uploading is Complete . . . . .	108
Creating a PDF of Patient Data. . . . .	109
<b>8 - Monitor Configuration . . . . .</b>	<b>111</b>
Overview . . . . .	111
About Factory Configuration . . . . .	112
About Default Configuration . . . . .	112
About Temporary Configuration . . . . .	113
About Acuity-Defined Configuration . . . . .	113
Parameter Configuration Matrix . . . . .	113
<b>9 - PC Utility . . . . .</b>	<b>121</b>
Introduction. . . . .	121
Propaq LT Monitor Configuration Utility . . . . .	121
Propaq LT Monitor AutoPrint Utility . . . . .	122
Installation . . . . .	122
System Requirements . . . . .	122
Procedure . . . . .	122
Configuring the Monitor . . . . .	126
Making a Configuration File . . . . .	126
Printing a Configuration File . . . . .	129
Modifying a Configuration File . . . . .	130
Configuring a Monitor . . . . .	130
About the Configuration Display. . . . .	130
Error Messages . . . . .	134
About the Configuration Worksheet. . . . .	135
About AutoPrint . . . . .	136
Configuration Worksheet . . . . .	137
Configuration Worksheet Authorization . . . . .	157
<b>10 - Maintenance . . . . .</b>	<b>159</b>
Recharging the Battery . . . . .	159
Inspecting and Cleaning the Monitor and Accessories. . . . .	161
Recycling Monitor Components . . . . .	162
Within the EU. . . . .	162
Outside the EU. . . . .	162

A - Specifications .....	163
B - EMC Compliance.....	179
Limited Warranty .....	193
Index .....	195