

**Set Your Name**

When you use Yunmaihaoping App for the first time, you need input your true name so as to store your personal health data. Pls touch the textbox slightly into next step.




**Set Your age**

You must input your actual age into Yunmaihaoping because the correct physical examination data is related to your actual age. This product will keep secret to your age. Choose your age by sliding upward.



**Set Your height**

You must input your actual height too. Pls choose your actual height by sliding upward or downward.

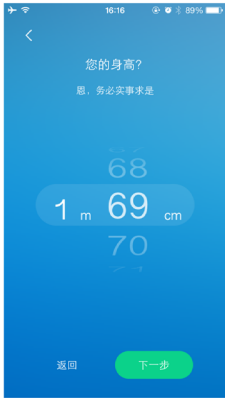


**Physical Examination Scores**

When you stand on this intelligent product, not only you can find your weight, fat and BMI data, but also comprehensive physical examination scores. If you are interested in touching the name of body,



**Bulletin**

This product can store the data when you weigh each time. You can conveniently find your weight and fat data of each week, each month, even the whole year. Pls touch the arrow key to look at the data each week.

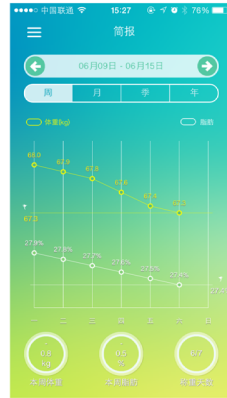

**Target**


You can set your weight target in this product by one key whatever you plan to lose or put on weight or keep it same. You can set your target weight by sliding the digital in this product.



# HELLO

Guides

Welcome you to use Yunmai haoqing.  
Pls kindly note the following Guides

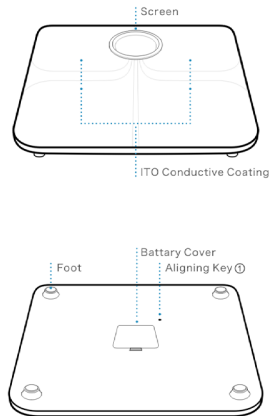


① Note: The aligning key is in the initial statement of the product ex-factory. Pls don't touch or change it at random

Assemble Carpet Mats)



When you use this product on the carpet, Pls assemble the attached carpet mats.

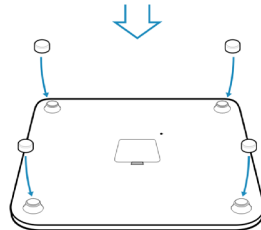

Connect to iPhone

1. Pls touch [set] icon



3. Open the bluetooth communication module.



2. Pls touch [bluetooth]



4. Pls touch [Yunmai] to enter health journey !



Connect to Android

Currently it only supports the intelligent mobile phone with Android 4.3 version or higher, and bluetooth will be opened and connected automatically.

Notice

The physical examination data of Yunmai haoqing is only to be referred for health index not for medical decisive advice. Weight Range: 3kg-150kg

Maintenance

Clean the appearance of the product with the damp cloth. Pls don't use any detergent or abstrergent for cleaning, which can damage the glass coating. The glass coating can protect the electrode to examine the fat of your body.

More Information

You can find more information in [www.iyunmai.com](http://www.iyunmai.com)



Scan the two-dimensional code to set up the application in your mobile phone to experience more powerful function of Yunmai haoqing.

After-sales

You can get permanent technique service and one-year guarantee of hardware components

Customer Service: 400-990-1092

Official wechat: [yunmai haoqing](https://www.wechat.com/qrcode/index)

Official Weibo: [@yunmai haoqing](https://www.weibo.com/qrcode/index)

## **FCC Statement**

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Changes or modifications not expressly approved by the party responsible for compliance could void your authority to operate the equipment.