

BAYER

Contour®

BLOOD GLUCOSE MONITORING SYSTEM

10/13

12:24 PM

93

mg/dL

M

NO CODING®

QUICK REFERENCE GUIDE

Contour

10/13 12:24 PM

93

mg/dL

M

Display

Memory Button

Up/Down Scroll Buttons

Test Strip Port

Sample Tip:

Blood sample pulled in here.

Gray End:

Insert this end into the test strip port.

Using Advanced Mode

See your User Guide for complete instructions.

Setting Your Meter to Advanced Mode

To set your meter to Advanced Mode, and take advantage of meal markers and other available features:

1. Press **M** to turn your meter on.

2. Press and hold **▲** or **▼** for 3 seconds.

3. L-1 will flash.

4. Press **▲** or **▼** to change to L-2.

5. Press **M** to set.

L-2

Using the Markers

See your User Guide for complete instructions.

Pre-meal

Marks a test result that was taken before a meal.

Post-meal

Marks a test result that was taken after a meal.

Logbook

Marks any result as being unique or different in some way.

Alarm

Sets an alarm to remind you to run a post-meal test.

Contour

4/23 1:10 PM

93

mg/dL

M

Basic Testing

Your lancing device may not be the one shown here. See your *CONTOUR®* User Guide and *MICROLET®* 2 or Ascensia® *MICROLET®* insert for complete instructions.

Getting Ready to Test:

Remove the endcap from the *MICROLET*2 or Ascensia *MICROLET* lancing device.

Loosen the round protective cap on a lancet by rotating it ¼ turn, but do not remove it.

Insert the lancet firmly to cock the device.

Twist off the lancet cap.

Replace the endcap.

Getting the Blood Drop:

Adjust the endcap dial to your preferred puncture depth setting.

Insert the gray end of the test strip into test strip port on the meter.

No coding is required.

The meter will turn on and a test strip with a flashing blood drop will appear on your meter display letting you know the meter is ready to test.

Press the endcap firmly against the puncture site and press the blue release button with your thumb.

Stroke your hand and finger towards the puncture site to form a drop of blood.

Testing Your Blood:

Touch the tip of the test strip to the drop of blood. The blood is pulled into the test strip.

Hold it in the blood drop until the meter beeps.

Fingertip Testing

Your result* is displayed after a 5 second countdown.

To turn your meter off, remove the test strip.

*Your meter is preset and locked to display results in mg/dL. If your result is in mmol/L, contact Bayer Diabetes Care Customer Service 1-800-348-8100.

Removing the used lancet:

In one hand, hold the lancing device with your thumb on the **grip indent**, and with your other hand, hold the **gray endcap dial** as shown. Gently snap off the endcap.

Place the round protective lancet cap on a flat surface with the Bayer logo facing down.

With the used lancet still in the lancing device, push the lancet needle completely into the middle of the exposed side of the cap.

Eject the lancet following the instructions in your lancing device insert.

Need Help?

Call toll free: 1-800-348-8100

www.bayerdiabetes.com/us

After you perform a test and see your results on the display:

TO MARK RESULTS

YOU WILL SEE

TO SET

YOU WILL SEE

WHAT IT MEANS

USING THE ALARM

Pre-Meal

Press **▲** or **▼**

(Flashing)

Press **M**

93

mg/dL

Result is marked as a pre-meal test. Press **M** again to set an alarm.**

When the alarm sounds, the meter turns on, and you will see your pre-meal test result.

Post-Meal

Press **▲** or **▼**

(Flashing)

Press **M**

124

mg/dL

Result is marked as a post-meal test.

Press any button once to silence alarm and twice to turn the meter off.

Logbook

Press **▲** or **▼**

(Flashing)

Press **M**

124

mg/dL

Result is marked in memory as unique. You should note this in your Clinilog® logbook.

Run another blood test and mark it with a **⌘** post-meal marker (see chart at left).

Error Codes and Symbols

See your User Guide for a complete list of Error Codes.

E1

Temperature out of range.

E2

There was a problem completely filling the test strip. Retest using a new test strip.

E3

The meter is sensing a used test strip. Retest using a new test strip.

E11

Abnormal result. Retest using a new test strip.

Bayer HealthCare

Bayer HealthCare LLC
Mishawaka, IN 46544 USA
www.bayerdiabetes.com/us

Bayer

Bayer, the Bayer Cross, *CONTOUR*, *MICROLET*, Clinilog, No Coding and the No Coding logo are trademarks of Bayer.

**Alarm is factory set at 2 hours. See User Guide for complete instructions on how to adjust alarm times.

50184410
Rev. 2/08