



AllTalk™, TouchTalk™, and MiniTalk™ **USER GUIDE**

Lingraphica Tech Support: 888-274-2742

TABLE OF CONTENTS

Meet the Devices	3
Functional Overview	4
Device Settings	5
The Lingraphica ACE Program	7
Icons, Page Icons & Phrase Icons	8
Using the Search Function	11
Using Type and Talk	12
Using the Whiteboard	13
Scenes	14
Therapy Activities	15

Important Device Safety and Management Information

Do not use if screen is broken or cracked, as this could cause harm or injury.

Use only the supplied cable to charge your device. Using unauthorized power adapters or cables may result in abnormal charging and potential damage to the device.

Before you clean your device, disconnect all the cables and turn off the device. Clean device with a soft cloth dampened with water. Do not use liquid or aerosol cleaners, which may contain flammable substances. Do not use harsh chemicals, cleaning solvents, or strong detergents to clean the device. Moisten a soft, lint-free cloth with either water or a display cleaner and wipe the surface of the device until it is clean. Do not allow water from the cloth enter the ports or buttons. Keep the device dry.

Please contact Lingraphica Technical Support for any repairs.

MEET THE DEVICES



The Home Page is the first screen you see when you turn on your Lingraphica communication device.

The Button Bar across the top of the screen contains the buttons that access options and functions within the device.



The Picture Gallery is the area of the screen with the blue background, where you keep your most important icons and icon pages, to access them quickly.

The Workspace will be located at the bottom of the screen, and can be opened when you are building phrases and performing other activities.

FUNCTIONAL OVERVIEW



The Home Button will take you back to the home page from wherever you are in the system.



The Back and Forward Arrows take you back to the previous page or forward to the next page, depending on your settings.



Search opens the search window, allowing you to type the name of any icon or word you would like to find.



Fast Talk opens a page that contains words and phrases you often use in daily conversation. You can edit this page by adding your own words and phrases.



Type and Talk allows you to quickly type words or phrases and have them spoken back by the device. You can save the words and phrases as icons for later use.



Camera lets you take pictures of people or objects that you can use to create custom icons and pages. Photos are stored in the Photo Album accessed by the “Photos” icon on the home page.



Whiteboard lets you write or draw on the screen with a stylus or your finger. You can use this to communicate quickly, and you can save whiteboard drawings for later use.



Menu gives you access to additional features and settings in the device.



The battery indicator shows the device battery life. When charging, you will see a lightning bolt inside the image. If your battery power is less than 25%, the battery indicator will turn red.



The WiFi indicator shows WiFi signal strength when you are connected to WiFi, ranging from weak (1 bar) to strong (4 bars). If WiFi is turned off, the indicator will have a line through it. If WiFi is on but not connected, you will see a question mark.

DEVICE SETTINGS

Within the device menu, there are several settings that allow you to make personalized changes to your Lingraphica communication device. There are also settings that will allow you to communicate directly with Lingraphica tech support. Below is a summary of the setting options. More details are available at the ACE website, ace.aphasia.com.



Choose Voice

This option changes the voice used by the device. You can also choose to use a synthetic voice for all words and phrases, and set the speed of that voice.



Navigation

You can set the navigation mode to slow, fast, or custom. The slow setting enables all the device's feedback systems: icons and phrase icons will project, and the page icons will project and speak the word or phrase. The default fast setting lets you navigate to a desired icon quickly without the added feedback systems. Custom navigation allows you to choose if each setting is on or off.



Size

You can adjust the size of icons based on visual preferences and needs.



Icon & Text Options

You can configure your icons to have all uppercase letters and choose how much text will be visible on the icon.



Phrase Building Options

Here you can make it easier to add icons to the workspace.



Themes

Choose a different color for your device background to help with low-vision contrast sensitivity.



Alignment

Align icons on the left, right, or center of the screen.



E-mail

Use this button to access your Lingraphica email.



Backup or Restore

Save a backup or restore a backup that was previously saved. Restoring a backup will overwrite the current data. You can only have one saved backup at a time.



Update

Get the latest device updates from Lingraphica over the Internet. Your device must be connected to Wireless Internet when using this setting.

DEVICE SETTINGS



Bluetooth

This setting lets you connect external accessories to your device, such as a keyboard or speakers.



WiFi

The WiFi settings button opens the WiFi settings screen and allows you to connect your device to the internet.



Keyboard

Use this button to choose your preferred type of keyboard and keyboard color.



Reset

Go to User Icons allows you to go to the page that contains all the custom icons that you have created.

Reset "page" returns the currently displayed page to the default layout.

Reset Device returns all pages to their default layout, which will cause the user to lose all customized data.

Download missing media will add any sounds or images that are missing.

Verify Lingraphica Installation confirms the presence all possible items on the device.



Video Help

Instructional videos about using your device.



Connect Device to Technical Support

Activate remote technical support and receive real-time help right through your device.



Unlock Device

Contact technical support to access other applications.



About TouchTalk™


Provides product information about your device, including version number, as well as our tech support phone number and company website.

THE LINGGRAPHICA ACE PROGRAM

The Lingraphica ACE Program: ACHIEVE COMMUNICATION EXCELLENCE

The Lingraphica ACE Program is an educational program designed to help you maximize your communication potential. As an important member of our community, we have already enrolled you in this exciting and beneficial program.

An Internet connection is necessary to access your Lingraphica email and the ACE Program.

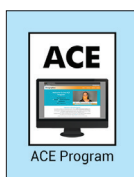

Accessing the ACE Program

To access the ACE Program, simply touch the ACE icon located on the home page of your device. This will launch the ACE website right on your communication device.



Accessing Your Email

We have automatically enrolled you in the ACE Program and have given you a free email account. To access your email:

1. Touch Menu
2. Touch Settings
3. Scroll down and touch Email


ICONS, PAGE ICONS & PHRASE ICONS



Icons are images that represent words or phrases. Icons are built into the device and can also be created by you. They can be blank, a photograph, a video or animation, a Lingraphica image, or a photo or video that you have taken. Icons will speak and can project when touched, depending on your device settings. If the icon is a video, the video will play when the icon is touched.



Page icons act as a door to other icons and pages that are in the same category. They have a dark gray shadow behind the icon. When touched, they take you to the next page. Some page icons are already in your device; you can also create your own.

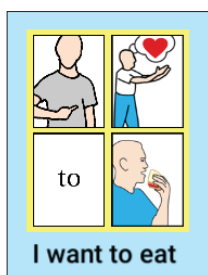

A phrase icon is a number of individual icons that are combined into one icon, which then makes a complete phrase or sentence. Phrase icons will speak each individual icon as a phrase, and may project when touched, depending on your device settings.

ICONS, PAGE ICONS & PHRASE ICONS

Making a New Icon

When creating a new icon, you can give it a name, image, and voice, and you can add or remove pages behind it. You can always go back and change these options later.




1. Touch Menu



2. Touch Make New Icon

3. This page allows you to name the new icon using the Edit Text button. Here, you can also record a voice or video, choose an image, and edit page options for your icon.

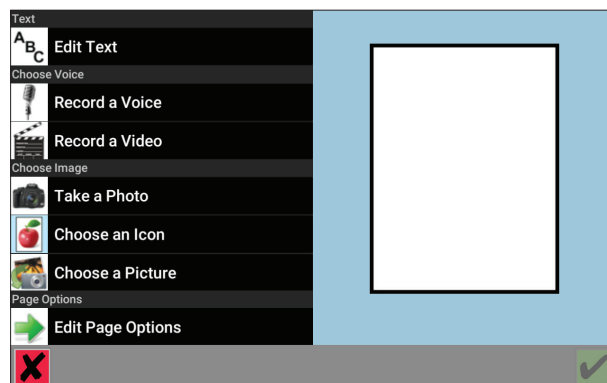

ICONS, PAGE ICONS & PHRASE ICONS


Changing Icons & Pages

When an icon has a red border around the outside, you can make changes or edits to that icon in the menu.

These edits include changing the text or name of the icon, recording a voice or video, choosing an image for the icon, and adding pages behind it to create a Page Icon.


You can also delete an icon that you no longer use. Deleted icons are only removed from their current page, but not from the device – you can always bring them back.

Just touch the icon you wish to change or delete (you'll notice the red border around the icon), and then touch the Menu button to see your options for that icon.

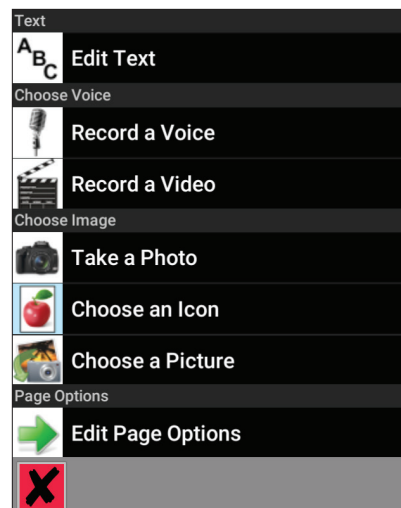

Moving & Copying Icons

You can move icons around on a page, depending on your preferences and unique needs. Simply touch and hold the icon until it begins to dance, then drag it to the new location on the screen. Once you have the icon in place, let it go and it will remain where you dropped it.

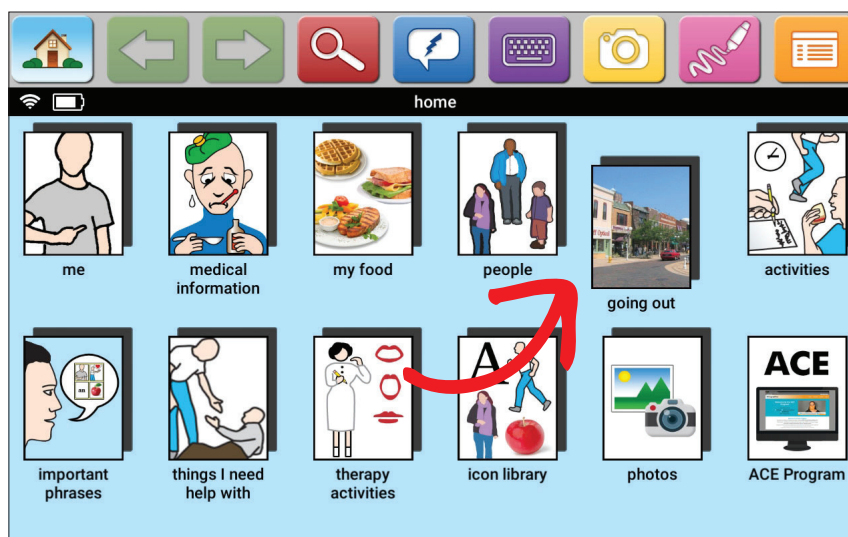

USING THE SEARCH FEATURE



You can search for any icon in the device using the Search feature.


The keyboard has a predictive text function, which will suggest words as you type.

Touching the blue plus sign adds the icon to whatever page you are on. The current page is displayed in the upper right corner of the search screen.

When you touch the icon thumbnail image, it will speak.

Touching the search result row takes you to the page on which that icon lives. Some icons can be found on more than one page. This can be helpful if you are interested in seeing related words or more words within a certain category.

In the example below, searching for the word “apple” shows results on the fruit and lunch pages, and also shows a result for “apple pie.” Touching the blue plus sign will add the apple or apple pie icon to your current page. Touching the row will take you to the fruit, lunch, or pie page. To hear the icon speak, you would simply touch the apple or pie image.

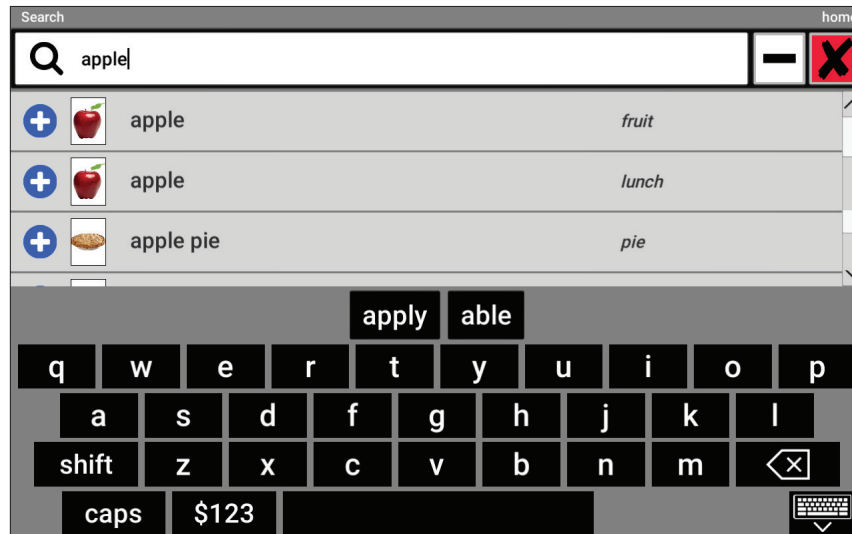


USING TYPE AND TALK



Type and Talk allows you to type a word or phrase, and then hear it spoken through your device. You can also save the phrase as an icon.


1. Touch Type and Talk.
2. Type the word or phrase you want. The keyboard will offer word suggestions as you type.
3. Touch Talk to hear the word or phrase spoken. You can also save the word or phrase as an icon.

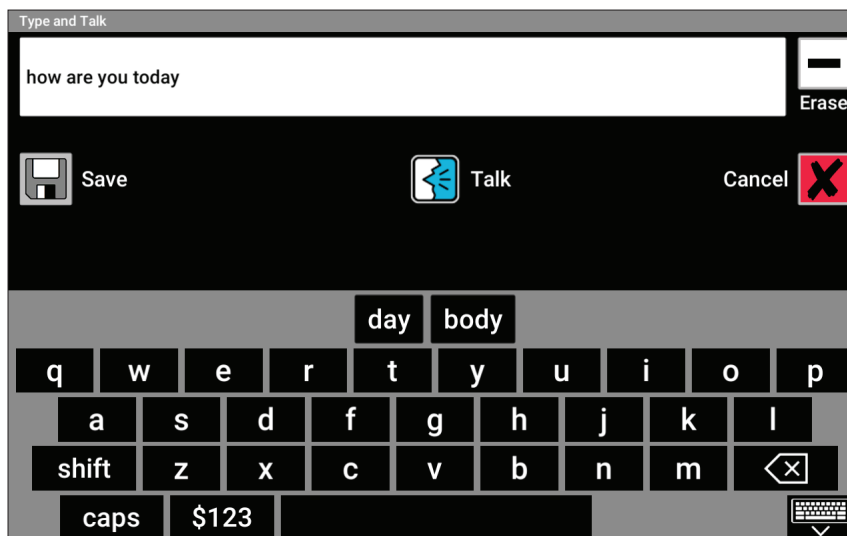

USING THE WHITEBOARD



The Whiteboard on the device will allow you to draw or write with your finger or the stylus to help aid in your communication. You can clear the Whiteboard or save your drawings to your device.

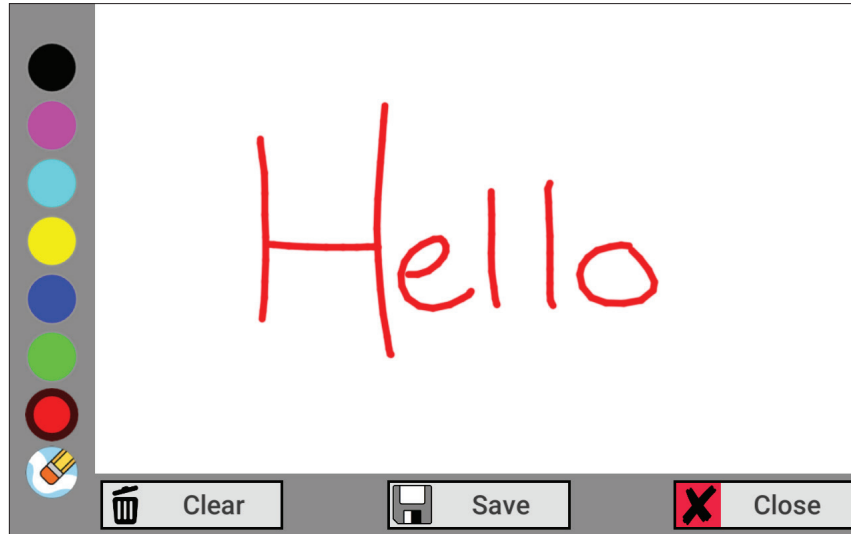


SCENES



Scenes are interactive three-dimensional contextual scenes from everyday life. You can navigate to different scenes and picture galleries by touching elements of the scene. The scene icons will project when touched. You can explore different scenes, such as a classroom and the rooms within a house.



THERAPY ACTIVITIES

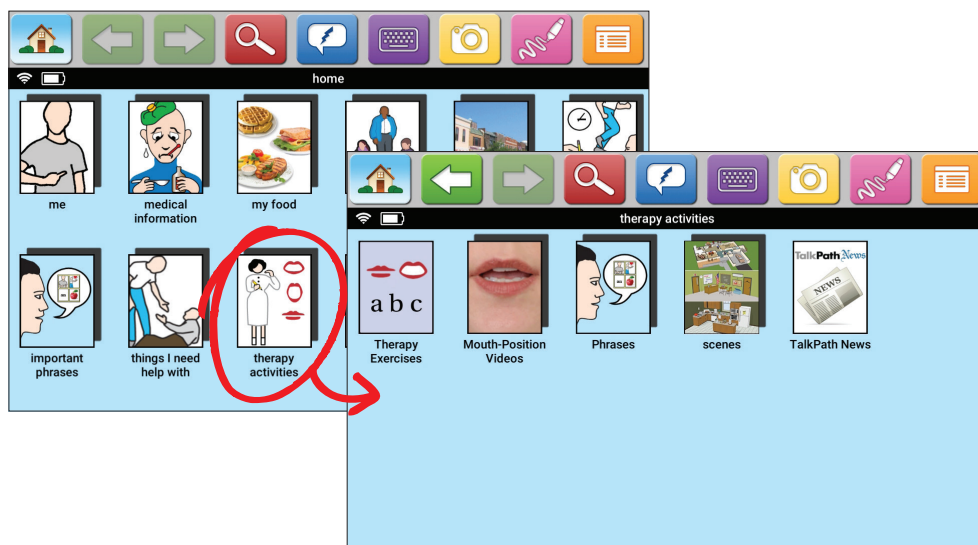

Therapy Exercises: The Therapy Exercises section contains many different types of speech-related exercises, including Completing a Phrase, Expressing Emotions, Grouping, and Speaking.

Mouth-Position Videos: The device has a series of mouth-position videos to be used for speech practice. You can practice saying sounds, words, and phrases by repeating after each video.

Phrases: The device comes with several functional communication phrases that allow you to practice conversational speech.

Scenes: Interactive three-dimensional contextual scenes from everyday life. (See page 14 for more information.)

TalkPath™ News: Our devices come with a built-in news source where you can read daily headlines, have the news read to you, and practice reading comprehension. (Requires an Internet connection.)

TalkPath™ Therapy: Our AllTalk communication device gives you access to our online speech therapy platform right from the device. Enjoy practicing your speech with more than 13,500 exercises -- including mouth position and other video exercises. (Requires an Internet connection. Only available on the AllTalk device.)



103 Carnegie Center
Suite 104
Princeton, NJ 08540
888-274-2742 (toll free)
609-275-1300 (phone)
609-275-1311 (fax)



www.aphasia.com

Lingraphica's customer support team is available, free of charge,
to answer any questions you have about your device.

To reach Lingraphica's customer support team, please call:

888-274-2742

Monday - Friday, 8:30 a.m. to 8:00 p.m. ET

You can also visit www.aphasia.com/techsupport or chat online
with a customer support representative at **aphasia.com**