


Instructions for Use: TempaChek™-LC Thermometer

Brand Name of Product	TempaChek™-LC Thermometer
Generic Name of Product	Liquid crystal thermometers
Product Code Number(s)	TEMPACHEK-LC, TEMP-LC2
Purpose of Product	To monitor the real time temperature of an instrument cleaning bath.
Range of Applications for Product	<ul style="list-style-type: none"> • Ultrasonic cleaners • Manual soaking baths • Sinks • Soak basins
Key specifications of product	<ul style="list-style-type: none"> • Liquid crystal thermometer provides real time feedback on the temperature of the bath. • Adhesive allows placement of thermometer below the water line of the bath. • 10 degree graduated scale from 60° to 150°F is easy to read. (Temp-LC2 include both Fahrenheit and Celsius scales) • Reusable. • Precision to +/-1°C.

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	1 per pack
Shelf Life	

Instructions for Using Product	
Description of Use (s)	For monitoring the temperature of a cleaning bath in real time.
Preparation for Use	Be sure surface for adherence is dry, clean and smooth.
Diagrams (drawings, pictures):	 <p style="text-align: center;"><i>TempaChek™-LC</i></p>
Steps for Use of Product	<ol style="list-style-type: none"> 1. Use one (1) TempaChek™-LC per bath. 2. Peel thermometer from release paper. 3. Apply to the inside, side wall of the tank below the intended fluid line and where the thermometer can be easily read. 4. TempaChek™-LC should be used to adjust the water temperature to the proper, optimal range for the cleaning agent used as specified by the manufacturer. 5. A color of green indicates the temperature is at the exact value. Blue is the temperature just below and tan is the temperature just above the actual temp. 6. Monitor the temperature during use of bath to insure compliance with target temperatures. Immediately report failure to achieve the target. 7. Record observations on log sheet.
Interpretation of Results	Consult the detergent manufacturer's IFU to determine the proper temperature range for the cleaning bath. Temperature indicated by the TempaCheck™-LC should be in this range.
Contraindications of Test Results	
Documentation	Record observations on log sheet.
Special Warnings and Cautions	
Disposal	<ul style="list-style-type: none"> • If adhesive residue remains on the surface, may be removed with acetone or isopropyl alcohol solvent. • May be disposed of in regular trash.

Reprocessing Instructions	
Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	

Cleaning – Manual:	
Cleaning – Automated:	
Disinfection:	
Drying:	
Maintenance, inspection, and testing:	
Reassembly Instructions:	
Packaging:	
Sterilization:	
Storage:	
Additional Information:	

Related Healthmark Products	
Other Product Support Documents	Proformance Brochure, ProFormance Price List
Reference Documents	
Customer Service contact:	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2017-12-22 Ralph J Basile