

## 1. Order



Purchase your Test Kit.

## 2. Receive Kit



Receive kit in the mail.  
(Time depends on shipping choice)

## 3. Registration



Register your Sample ID and Order Number at [BiotechAccelerated.com/register](https://BiotechAccelerated.com/register)  
(1-3 minutes)

## 4. Warning



Don't eat, drink, smoke, or chew gum for half an hour before collecting your sample.  
(30 minutes)

## 5. Collection



Collect your sample following instructions in your kit. You may watch the collection video for additional guidance.  
(2-5 minutes)

## 6. Lab Form



Fill out the Lab Requisition Form.  
(1-3 minutes)

## 7. Send to Lab



Follow instructions on Page 3 of your pamphlet to send your sample back to the lab.  
(1 day unless there are shipping delays or weekends/holidays)

## 8. Results



Receive your results in under 48 hours from your sample's arrival at the lab.  
(1-2 days)