**Please read this:**

- FOR EXTERNAL USE ONLY
- DO NOT USE THIS PRODUCT IF THERE IS ANY SIGN OF DAMAGE TO THE END MARKED (A) OPPOSITE. YOU MUST DISCARD THE PRODUCT
- Only to be used for the purposes of external, personal cleansing and wiping
- To maintain personal hygiene the easywipe should not be shared between people
- Not to be used on animals or young children
- Only use the easywipe for the intended purpose
- Any excessive force applied to the hinges could result in damage to the product
- If the hinges are damaged then you MUST DISCARD THE PRODUCT
- DO NOT use the easywipe if any damage to any part of the product is noticed

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Designed and manufactured in the UK by Buckingham Healthcare Ltd

Registered Office Address - Tower House, Parkstone Road, Poole, Dorset, BH15 2JH
www.buckinghamhealthcare.co.uk

buckingham
healthcare

Helping people to help themselves

Chris Buckingham
L1B BSC SROT

user instructions

buckingham pocket easywipe

Personal hygiene aid

Designed by Occupational Therapist Chris Buckingham

buckingham
healthcare

Made in UK

Search on YouTube for:
How to Use the Pocket Easywipe **YouTube**

**Intended Use**

The Buckingham Pocket Easywipe is a personal hygiene aid designed to assist anyone with limited mobility with personal cleansing after using the toilet. It works very well with either flushable wipes or toilet tissue.

Cleaning and Care

- You may immerse the pocket easywipe in water. Dry it thoroughly afterwards
- To clean wipe all over with a soft disposable cloth in a mild soapy water
- Dry with a soft disposable cloth
- It is recommended that the pocket easywipe is cleaned after each use
- Due to hygiene reasons, once removed from the packaging the product cannot be returned

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