

SETUP and ASSEMBLY INSTRUCTION MANUAL

ARU-01BS X-RAY STAND



TABLE OF CONTENTS:

Package Contents1

Parts List & Assembly2

Installing the X-Ray Adapter3

Using the X-Ray Stand3

Notes5

WarrantyBack Cover



P.O. Box 1548
Woodinville, WA 98072
(425) 487-3157
(800) 426-5913
www.aseptico.com
info@aseptico.com

Your new Aseptico ARU-01BS X-Ray Stand is one of the finest portable X-Ray workstands available to the professional. This stand features a telescoping frame and a rotating X-Ray mounting clamp. The lightweight, foldable stand sets up in a only a minute.

Congratulations

This system is engineered to provide many years of reliable service. Please read the instructions provided in this manual to receive the best and longest service from your Aseptico equipment.


Separate manuals may be provided to cover the operation and maintenance of the X-Ray Unit or other accessories for your unit.


PACKAGE CONTENTS:

- ARU-01BS X-Ray Stand
- 461361 X-Ray Mounting Adapter
- Soft-Padded Carrying Case
- Setup and Assembly Instruction Manual



PARTS LIST & ASSEMBLY

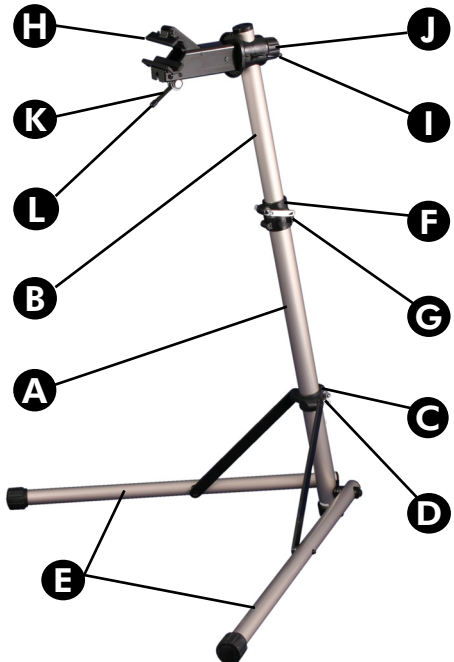


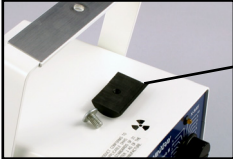


<u>Part Ref.</u>	<u>Part Description</u>
A	Bottom Tube
B	Upper Tube
C	Bottom Tube Collar
D	Bottom Locking Lever
E	Base
F	Upper Tube Collar
G	Upper Locking Lever
H	X-Ray Clamp
I	Clamp Knob
J	Rotation Locking Lever
K	X-Ray Clamp Locking Lever
L	X-Ray Clamp Adjustment Knob
M	X-Ray Adapter (<i>Aseptic</i> PN: 461361)

1. **Ref. Figures 1A Thru 1C:** Open the bottom tube collar locking lever (D) and unfold the base legs (E) until the bottom collar (C) touches the frame stop. Close the locking lever firmly. **NOTE:** When unfolding base legs, refer to setup instruction label on bottom side of legs.
Caution: When unfolding the base, keep hands and fingers clear of moving parts.
2. **Ref. Figures 2A & 2B:** Loosen the upper tube collar locking lever (G) and raise the upper sliding tube (B) to the desired height. Firmly close the locking lever.
Note: The upper locking lever should be locked flush against the tube collar to prevent slippage or stand collapse. Ref. Figure 2C.
3. **Ref. Figures 3A & 3B:** Unscrew the clamp knob (I) sufficiently to loosen X-Ray clamp assembly (H). Raise the X-Ray clamp (H) and push assembly in toward the stand,


aligning it with the square bracket. **Note:** Avoid pinching hands or fingers between the bracket and clamp assembly. Firmly tighten the clamp knob (I), to lock the X-Ray clamp assembly into position.

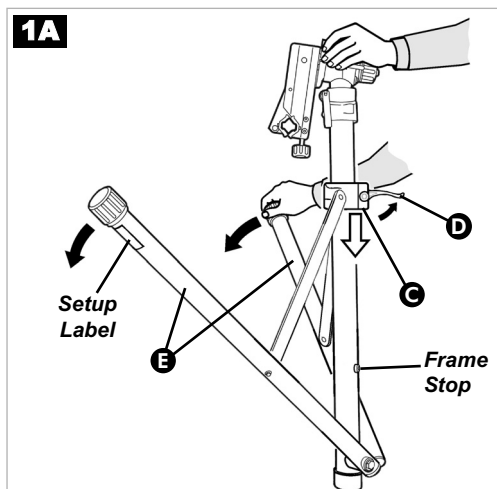

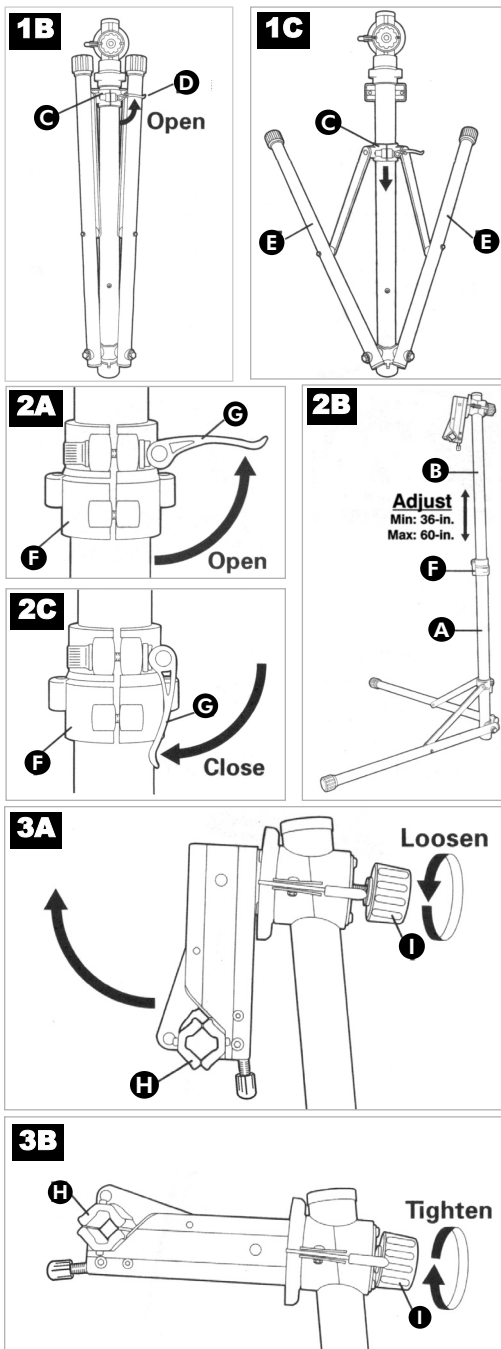

INSTALLING THE X-RAY ADAPTER

1. Mount the X-Ray Adapter onto the bottom side of the X-Ray Unit handle as shown in Figure 4A. **IMPORTANT:** Ensure that the raised edge of the Adapter is seated against the rear edge of the handle as shown in Figure 4B.

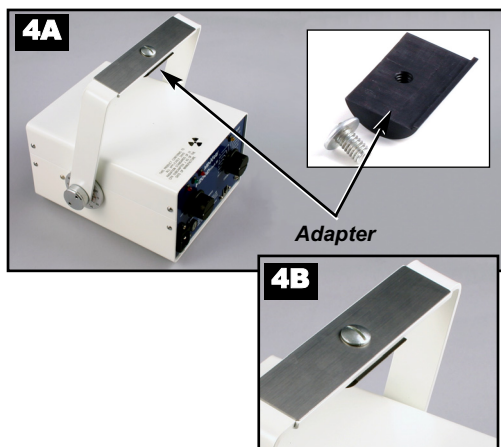

USING THE X-RAY STAND

1. Set up the ARU-01BS X-Ray Stand on a stable, horizontal surface.
2. Ref. Figure 2B: Loosen the upper tube locking lever (G) and adjust the upper sliding tube (B) to the appropriate height for the x-ray procedure. Firmly close the locking lever. (Height Adjusts from 36-in. min., to 60-in. max.)

Note: To make height adjustments easier, we recommend that adjustments be made without an X-Ray unit mounted in the stand.

3. Ref. Figure 5: The X-Ray clamp (H) can be rotated to allow precise positioning of the X-Ray unit. Unscrew the clamp knob (I) slightly to loosen clamp assembly. Press in the rotation locking lever (J) and rotate the X-Ray clamp to the desired angle. Release the locking lever and firmly re-tighten the clamp knob. **Note:** This adjustment can be made with an X-Ray Unit mounted.





USING THE X-RAY STAND - *Cont'd*

4. **Ref. Figure 6A:** Push the X-Ray clamp locking lever (K) down to open the X-Ray clamp (H).

5. **Ref. Figures 6B, 6C, and 6D:** Carefully place the X-Ray unit between the X-Ray clamp jaws. Pull the locking lever up to close and lock the clamp onto the unit.

IMPORTANT: Ensure that the X-Ray Adapter is installed on the X-Ray unit handle before clamping the unit into the stand.

6. **Ref. Figure 6B:** If there is too much, or too little, tension on the X-Ray clamp jaws to close firmly around the X-Ray unit handle, turn the clamp adjustment knob (L) clockwise to increase jaw tension or counterclockwise to loosen jaw tension.
7. To fold up the ARU-01BS X-Ray Stand, loosen the upper tube locking lever (G) and slide the upper tube (B) down into the bottom tube. Firmly close the locking lever. Unscrew the clamp knob (I) to sufficiently loosen and pull the X-Ray clamp (H) assembly away from the square bracket. Fold the assembly against the stand and retighten the clamp knob. Open the bottom tube locking lever (D), fold the base legs against the bottom tube (A), and firmly close the locking lever.

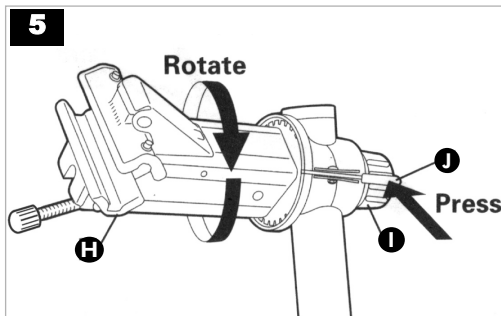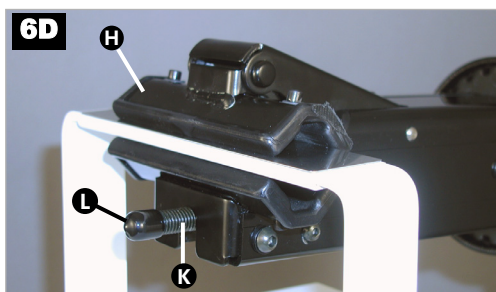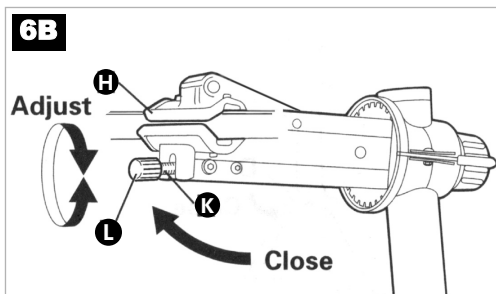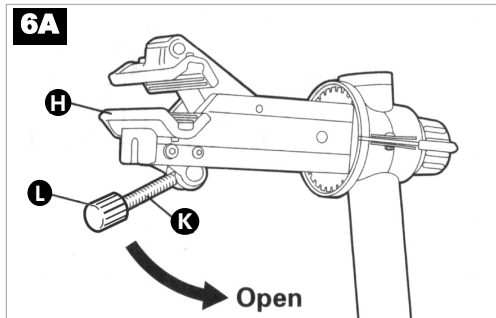

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

WARRANTY:

Aseptico warrants this product against defects in material or workmanship for a period of one year, from date of original invoice. Aseptico's sole obligation under product warranty is (at its sole option and discretion) to repair or replace any defective component or product in part or whole. Aseptico shall be the sole arbiter of such action.

In the event of alleged defect under warranty, the purchaser is to notify Aseptico's Customer Service Department promptly. Customer Service will provide instructions, usually directing that the product be returned for service. Shipment to Aseptico and the cost thereof is always the responsibility of the purchaser.

Accidental misuse, inappropriate installation, or failure to perform directed maintenance voids the warranty.

Aseptico does not assume, under this warranty, any risks or liabilities arising from the clinical use of its products, whether or not such use involves coincidental utilization of products manufactured by others.

NOTE: In the interest of serving our customers more efficiently, customers receiving service on non-warranted repairs are expected to accept charges that are less than \$250.00 without further notification.



P.O. Box 1548 Woodinville, WA 98072
8333 216th Street SE Woodinville, WA 98072
(425) 487-3157 (800) 426-5913
www.aseptico.com • info@aseptico.com