

FECAL OCCULT BLOOD TEST

McKesson Consult® FOBT Test is a fecal occult blood test designed to detect the presence of blood in stool specimens. It uses guaiac-impregnated paper, which allows for greater sensitivity for early detection of even small amounts of blood in the stool. If blood is present, a blue color quickly develops for easy-to-interpret results. The slides have two sample sites for improved detection of positive results, as well as positive and negative monitors to help ensure proper performance of the slide and developer. The test is available in both a standard or extra-sensitive configuration. The extra sensitive test offers a more vivid color development for even greater readability.

McKesson Consult FOBT Tests are available for in-office use, or can be taken home for the convenience of the patient. Instructions are included to help simplify understanding of the test procedure. USPS compliant packaging is designed to provide safe and efficient specimen transport.

FEATURES AND BENEFITS

- 30-second result time
- High sensitivity helps in early detection of blood in the stool
- Vivid color development allows for enhanced readability
- Take-home test option for patient convenience
- CLIA waived test requires no specialized training
- Not made with natural rubber latex



Satisfaction Guaranteed

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit.

Never Compromise on CareSM

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them—making it even easier for you to provide the best care for your patients.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.


The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.

© 2017 McKesson Medical-Surgical Inc.

Directions for Use:

DEVELOPING THE SAMPLE TEST

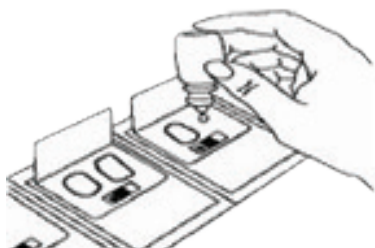

Procedure

1. Open perforated window on the back of the slide.
2. Apply two (2) drops of occult blood developer to the back side of boxes A and B containing the specimen.
3. Read results after 30 seconds and within 2 minutes.
4. Record the results; any trace of blue color, within or on the outer rim of the specimen, is positive for occult blood.

** For complete instructions, please refer to the package insert.*

DESCRIPTION

ADDITIONAL INFO

MFR#

QTY

Occult Blood Testing Patient Kit	1 slide, monitors, developers, applicators, manual	298	80ea/bx
Occult Blood Testing	100 slides, developers, applicators, manual	300	100ea/bx
Occult Blood Test Developer	Bottle, 15 mL	301	20ea/bx
Occult Blood E-S Testing Patient Kit	Extra sensitive, 1 slide, mailer envelopes, applicators, developers	302	80ea/bx
Occult Blood Testing	Extra sensitive, 100 slides, developers, applicators	303	100ea/bx
Occult Blood E-S Developer	Bottle, 15 mL	551748	1ea