UltraPro™ Tx Air
Hygiene Handpiece

EN

Description:
UltraPro™ Tx Air Hygiene Handpiece is a light-weight, ergonomic, friction grip handpiece. The motor has a working output of 4000 RPM at variable working pressures of 35–45psi.

Indications for Use:
UltraPro™ Tx handpieces are intended for prophylaxis procedures, for use by a trained professional in dentistry.


- Instructions:**
1. Prior to attaching air supply, bleed air line of moisture and contamination.
 2. Attach handpiece (motor and nosecone) to air hose.
 3. Confirm working pressure is between 35–45psi.
 4. Attach proph angle by pushing it until it is fully seated. Friction grip will hold the angle in place.
 5. To remove the prophy angle, grasp the nose cone and firmly pull the prophy angle in a straight outward direction.
 6. For a smoother swivel between the motor and nose cone, the "O" ring may be removed from the motor stem.

Sterilization and Maintenance of Motor and Nosecone:

1. Remove prophy angle from the handpiece.
2. Disconnect handpiece from hose.
3. Separate the motor section and the nosecone.

Sterilization Processing Instructions				
Initial treatment at the time of use		Sterilization is required before every use.		
Preparation before Sterilization		Wipe device with 70% IPA soaked gauze.		
Maintenance, inspection and testing		Visually inspect that the device is not bent or damaged and free of debris. If debris is present remove before sterilizing.		
Packaging		Place selected device in approved sterilization packaging (ISO11407 & ISO11607).		
Sterilization		Place sterilization package in autoclave using the following settings:		
		Cycle Type	Minimum Sterilization Exposure Time	Minimum Sterilization Exposure Temperature
		Pre-vacuum	3 mins	132°C /270°F
		Gravity	30 mins	132°C /270°F
				20 mins
				30 mins
		Do not overload the sterilizer.		
Storage		Keep device in sterilized packaging until ready for use. Keep out of sunlight and moisture.		
Additional information		If sterilization packaging is compromised, repeat sterilization process.		
		This procedure was validated by an independent and accredited laboratory.		

4. After sterilizing, place two drops of handpiece oil into the motor air tube (Fig. 1) and 1-2 drops of oil into both ends of the nosecone (Fig. 2). To properly disperse the oil, run the motor for several seconds.
5. Apply one drop of handpiece oil to the "O" ring one time per week.

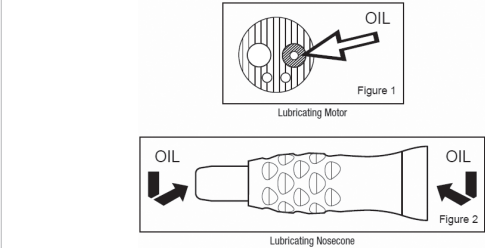- Precautions and Warnings:**
1. Carefully read and understand all instructions before using.
 2. Never place handpiece in ultrasonic cleaner.
 3. Do not use sterilization bags containing disinfectants.
 4. Always check that the compressed air is dry and free of oil.
 5. Do not exceed 45psi when operating handpiece.
 6. For best results, use prophy angles with 2.30mm – 2.35mm shanks.
 7. Always check that the angle is firmly seated in place on the handpiece before beginning the procedure.
 8. For professional use only.

Warranty:
Ultradent Products, Inc.'s ("Ultradent") UltraPro™ Tx Air hygiene handpiece warranty obligations are limited to the terms set forth below and are not transferable: Ultradent warrants this UltraPro™ Tx Air handpiece, including the motor and nosecone, against defects in materials and workmanship under normal use and proper maintenance for a period of TWO (2) YEARS from the date of purchase by the first retail purchaser ("Warranty Period"), as evidenced by invoice and shipping records. If a defect arises and a valid claim is received within the Warranty Period, Ultradent will, at its option repair or replace the defective product. The new Warranty Period for the repaired or replaced product shall be either ninety (90) days from the date of replacement or repair of the defective product or the amount of time remaining on the original Warranty Period, whichever period of time is greater. When a refund is given, the product for which the refund is provided must be returned to Ultradent. If handpiece is modified, misused or disassembled, warranty is void.

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© Copyright 2020 Ultradent Products, Inc. All Rights Reserved.